


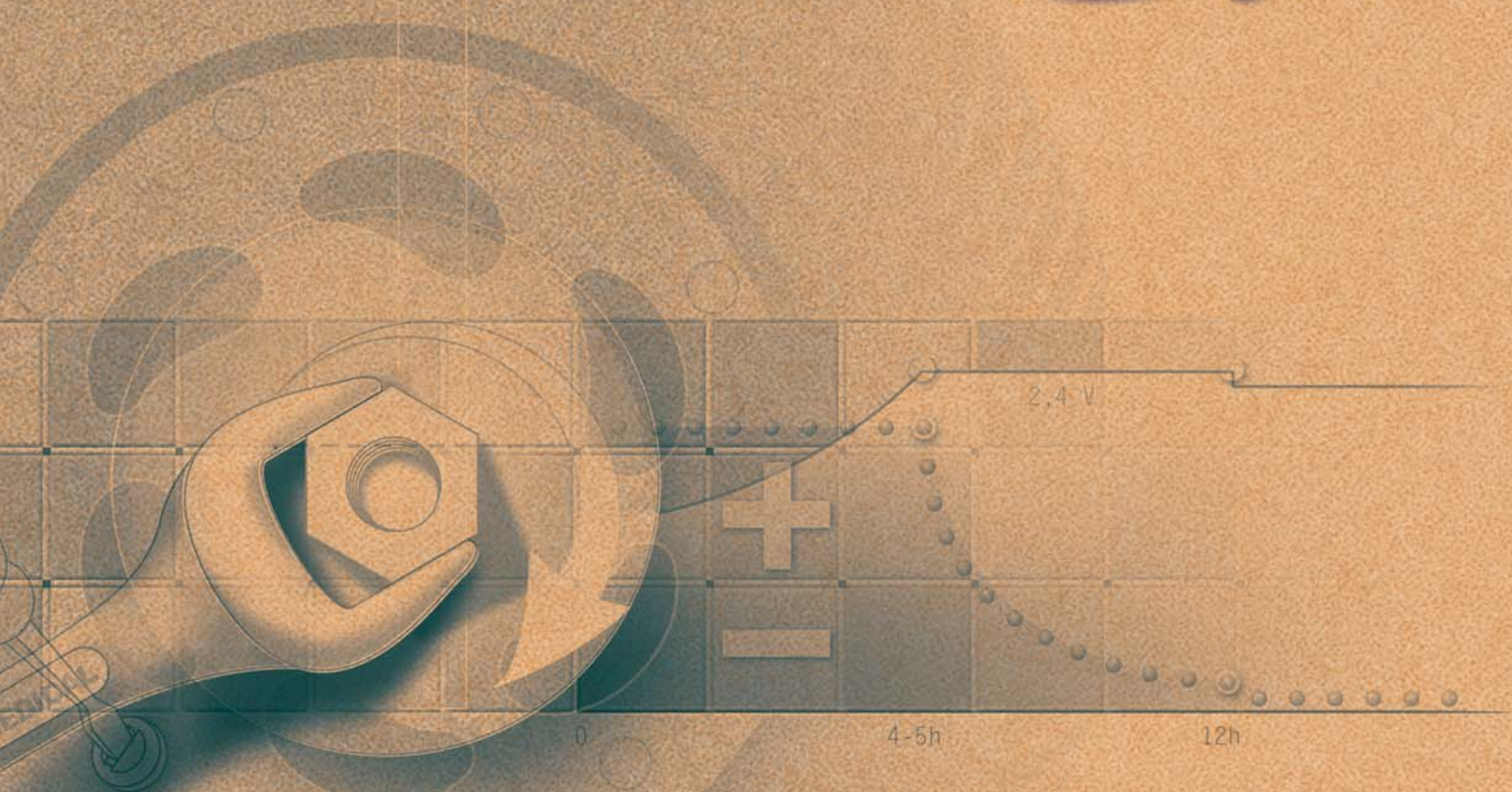

QUICKIE²

SERIAL NUMBER PREFIX Q2V & Q2H

PARTS MANUAL

(Q2N)
JANUARY 2014

QUICKIE 2 (Chairs w/ S/N Prefix Q2V & Q2H)	6
BASE FRAME	6
FRAME	6
FRONT SIDE FRAME	6
SWING-AWAY FRONT SIDE FRAME	6
HEMI SWING-AWAY FRONT SIDE FRAME	7
XL HEMI SWING-AWAY FRONT SIDE FRAME	8
REAR SIDE FRAMES	8
STRAIGHT REAR SIDE FRAME	9
ROUNDED REAR SIDE FRAME	11
HEMI STRAIGHT REAR SIDE FRAME	12
HEMI ROUNDED REAR SIDE FRAME	14
D.A.B.(Depth Adj Back) FRAME	16
CROSS TUBES	16
CROSS TUBE ASSEMBLY QUICKIE 2	17
Q2 CROSS TUBE ASSM 18-22 HD	20
AXLE & AXLE PLATES	21
AXLES	21
QUICK RELEASE AXLE MATRIX	22
THREADED AXLE UPGRADE ASSM	24
AXLE PLATES	24
STANDARD AXLE PLATE	25
AMPUTEE AXLE PLATE	27
CURVED AXLE PLATE	28
OFFSET AXLE PLATE	30
CASTERS & FORKS	31
CASTERS	31
MICRO CASTERS	32
SOFT ROLL CASTERS	33
ALUMINUM CASTERS	34
ALUMINUM SOFT ROLL CASTERS	35
8" x 2" CASTERS	36
CASTER TIRES & TUBES	37
3 SPOKE CASTERS	38
FORKS & STEMS	39
CASTER STEM & FORK ASSMS	40
MULTI-POSITION FORK	42
FROG LEG SUSPENSION FORK	44
QUICK RELEASE CASTER STEM & FORK ASSM	45
CASTER PIN LOCK	46
CASTER PLATES	47
ANTI-TIPS	48
ANTI-TIP (S/N Prefix Q2V & Q2H)	49
WHEEL LOCKS	51
PUSH TO LOCK WHEEL LOCK	52
PULL TO LOCK WHEEL LOCK	53
PUSH & PULL WITH GRADE AID ASSM	54
ERGO SCISSOR WHEEL LOCK	55
UNILATERAL WHEEL LOCK	56
WHEEL LOCK EXTENSION HANDLE	58
COMPACT WHEEL LOCK	59
WHEELS & TIRES	59
WHEELS	59
MAG	60
LITE SPOKE	62
NEW ULTRA LITE SPOKE	63
SPINERGY SPOX (18 SPOKE)	64
SPINERGY LX (12 SPOKE)	65
MOUNTAIN	66
TIRES	66
PNEUMATIC & PNEU. AIRLESS TIRE	67
FULL POLY TIRE	68
KEVLAR TIRE	69
LOW POLY TIRE	70
HIGH PRESSURE CLINCHER TIRE	71
KNOBBY TIRE	72
SCHWALBE MARATHON PLUS TIRE	73
KIK MAKO TIRE	74
URBAN ASSAULT TIRE	75
MOUNTAIN TIRE	76

HAND RIMS	76
ALUMINUM ANODIZED (SCREW MOUNT).....	77
ULTRA LITE ANODIZED (TAB MOUNT)	78
ALUMINUM ANODIZED (TAB MOUNT)	79
PLASTIC COATED (SCREW MOUNT).....	81
PLASTIC COATED (REG TAB MOUNT)	82
NATURAL FIT (SCREW MOUNT)	83
NATURAL FIT (TAB MOUNT).....	85
PROJECTION	87
PAINTED (CROSS SPOKE WHEELS ONLY)	89
CHROME (MAG WHEELS ONLY)	90
TUBES	90
TUBES AND AIRLESS INSERTS	91
SEATING & POSITIONING	92
SEAT	92
SEAT CUSHION.....	92
2" SEAT CUSHION	93
SEAT SLING (Q2H).....	95
SEAT SLING	96
SEAT SLING (Q2V).....	98
SEAT SLING	99
BACK.....	100
BACK FRAME	100
TUBE BACK PUSH HANDLE ASSM	101
TUBE BACK NON-PUSH HANDLE ASSM (OMIT PH).....	103
ANGLE ADJUSTABLE BACKREST	104
BACKREST STABILIZER BAR	106
D.A.B. BACKREST	108
D.A.B. BACKREST	108
SOLID BACK D.A.B. ONLY	112
DYNAMIC BACK	114
REMOVABLE STROLLER HANDLE.....	116
STROLLER HANDLE EXTENSION-REMOVABLE.....	118
FOLDING STABILIZER BAR.....	120
BACK UPHOLSTERY	123
STANDARD BACKREST UPHOLSTERY	123
ADJUSTABLE BACK UPHOLSTERY	124
HEAVY DUTY BACK UPHOLSTERY	126
HANGERS & FOOTRESTS	126
SWING AWAY EXTENSION MOUNT HANGERS	126
HANGERS.....	126
60 DEG HANGER SWING IN-OUT	127
70 DEG HANGER SWING IN-OUT	128
70 DEG HANGER V SWING IN-OUT	129
70 DEG HEMI HANGER SWING IN-OUT.....	130
80 DEG HANGER SWING IN-OUT	131
FOOTPLATES & EXTENSIONS	131
COMPOSITE FOOTPLATE.....	132
COMPOSITE ANGLE ADJUSTABLE FOOT PLATE	136
ADULT ANGLE-ADJ FOOTPLATE EXT MOUNT	140
KIDS ANGLE ADJ FOOTPLATE EXTENSION MOUNT	143
FOAM FOOTPLATE.....	146
LOCKING ANGLE ADJ FOOTPLATE	149
PLATFORM FLIP-UP FOOTPLATE	151
HIGH MOUNT EXTENSION TUBE	153
SWING AWAY FRONT MOUNT HANGERS	153
HANGERS.....	153
70 DEG FRONT MOUNT HANGER SWING IN-OUT	154
80 DEG FRONT MOUNT HANGER SWING IN-OUT	156
90 DEG FRONT MOUNT HANGER SWING IN-OUT	158
EXTENSIONS	159
FRONT MOUNT HANGER EXT.....	160
FOOTPLATES.....	160
ADULT ANGLE ADJ FOOTPLATE DIRECT MOUNT	161
KIDS ANGLE ADJ FOOTPLATE DIRECT MOUNT	163
90 DEG FOOTBOARD	165
LIFT-OFF FRONT MOUNT HANGERS	166
70 DEG LIFT-OFF HANGER.....	167
80 DEG LIFT-OFF HANGER.....	168
90 DEG LIFT-OFF HANGER.....	169

EXTENSIONS	169
FRONT MOUNT HANGER EXT.....	170
FOOTPLATES.....	170
KIDS ANGLE ADJ FOOTPLATE FRONT MOUNT	171
ADULT ANGLE-ADJ FOOTPLATE FRONT MOUNT.....	173
90 DEG FOOTBOARD.....	175
ELR's.....	176
EXTENSION MOUNT ELR.....	176
EXTENSION MOUNT ELR HANGERS.....	176
ELR HANGER	176
ELEVATING HANGER.....	177
ELEVATING HANGER CALF PAD 14 -15".....	179
ELEVATING HANGER CALF PAD 16-20".....	180
ELR HEMI HANGER	180
ELEVATING HEMI HANGER	181
ELEVATING HEMI HANGER KNEE PAD.....	183
ELEVATING HANGER CALF PAD 14 -15".....	184
ELEVATING HANGER CALF PAD 16-20".....	185
EXTENSION TUBES.....	185
DOUBLE D FOOTREST EXTENSION TUBE "OLD STYLE".....	186
FOOTPLATES.....	187
COMPOSITE FOOTPLATE.....	188
ADULT ANGLE-ADJUSTABLE FOOTPLATE EXT MOUNT.....	190
KIDS ANGLE ADJUSTABLE FOOTPLATE EXT MOUNT	192
ELR ALUMINUM FOOTPLATE	194
FRONT MOUNT ELR.....	194
FRONT MOUNT ELR HANGERS	194
FRONT MOUNT ELR HANGER.....	194
90 DEG ELEVATING HANGER	195
90 DEG ELEVATING CALF PAD	197
FRONT MOUNT HEMI ELR HANGER.....	197
90 DEG ELEVATING HEMI HANGER	198
ELEVATING HEMI HANGER KNEE PAD.....	200
90 DEG ELEVATING CALF PAD	201
EXTENSIONS	201
FRONT MOUNT HANGER EXT.....	202
FOOTPLATES.....	202
KIDS ANGLE ADJ FOOTPLATE FRONT MOUNT	203
ADULT ANGLE-ADJ FOOTPLATE FRONT MOUNT.....	205
ALR's.....	206
ALR HANGER	206
ARTICULATING HANGER.....	207
ARTICULATING HANGER CALF PAD	209
EXTENSION.....	209
ROUND EXTENSION TUBE	210
FOOTPLATES.....	211
COMPOSITE FOOTPLATE.....	212
ADULT ANGLE-ADJUSTABLE FOOTPLATE EXT MOUNT.....	214
KIDS ANGLE ADJUSTABLE FOOTPLATE EXT MOUNT	216
MULTI-POSITION LATCH STUD SUPPORT	218
ARMRESTS & RECEIVERS	218
PADDED SWING-AWAY ARMREST	219
PADDED SWING AWAY ARMREST RECEIVER.....	220
PADDED SWING AWAY ARMS W/ REMOVABLE STROLLER HANDLES.....	220
PADDED SWING-AWAY ARMREST.....	221
REMOVABLE STROLLER HANDLE.....	223
SINGLE POST HT ADJ ARMRESTS	224
SINGLE POST HEIGHT ADJUSTABLE ARMREST	225
SINGLE POST ARMREST RECEIVER - STD	227
FLIP BACK HT ADJ LITE ARMREST	228
FLIP BACK HEIGHT ADJUSTABLE LITE ARMREST	229
FLIP BACK ARMREST RECEIVER (LITE STYLE ARMREST)	230
FLIP-BACK HEIGHT ADJUSTABLE CLASSIC ARMREST	232
FLIP-BACK ARMREST	234
ADJUSTABLE LOCKING FLIP-UP ARMREST.....	236
LENGTH ADJUSTABLE LOCKING FLIP-UP ARMREST	237
ACCESSORIES	238
XTENDER	238
XTENDER 4X UNIVERSAL AXLE PLATE	239
XTENDER POWER ASSIST 4X PARTS.....	241


XTENDER INTERNAL REPLACEMENT PARTS.....	243
Q2/Q2HP XTENDER RETRO KIT.....	244
TRAYS	245
TUFFAK TRAY (MANUAL & PEDIATRIC CHAIRS)	246
TRAY RIBS TUFFAK.....	248
PADDED STOMACH CUTOUT.....	249
PROTRACTOR PAD	250
HINGED OVERLAY NON-REMOVABLE	251
HINGED OVERLAY REMOVABLE	252
SIDE GUARDS.....	252
SIDE GUARDS.....	253
FABRIC SIDE GUARDS.....	254
BAGS & POUCHES	254
CASE LOGIC BAGS.....	255
CELL PHONE PACK	256
BACKPACKS AND SEAT POUCHES.....	257
OXYGEN TANK HOLDER.....	258
ADJUSTABLE IV POLE ASSM	259
SPOKE GUARDS.....	260
IMPACT GUARD NEOPRENE & GEL VELCRO PADS.....	262
SOFT FEEL LEG STRAP	263
ADJUSTABLE LEG STRAP	264
TRAVEL WHEEL.....	265
CRUTCH HOLDER	266
ONE ARM DRIVE.....	267
AUTO BUCKLE POSITIONING BELTS	269
AIRCRAFT BUCKLE POSITIONING BELT.....	270
QUICKIE CADDY	271
POSITIONING BELT ATTACHMENT KIT.....	272
TIRE PUMP	273
TOOL KIT	274

NOTE: X-LONG SIDE FRAME WILL ADD 2.5 LBS TO THE TOTAL WEIGHT OF THE CHAIR


NOTE: ORDER PART 10 (TUBE CAP QTY 2) WHEN SWING-AWAY FOOTRESTS ARE NOT INSTALLED.

NOTE: NEW SIDE FRAME ASSEMBLY COMES WITH (4) SEAT SADDLES TO REPLACE OLD SEAT SADDLES.

MEASURING GUIDE

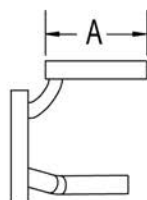

	DIM "A"
KIDS	5 1/8"
REG	7 3/8"
LONG	9 3/8"
X-LONG	11 3/8"

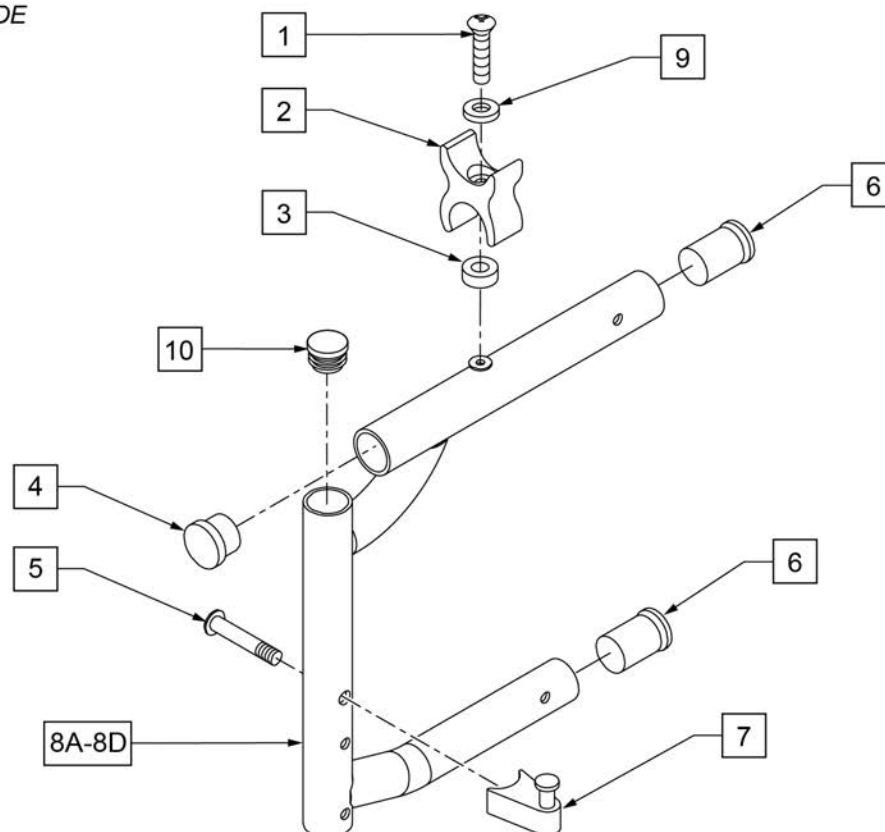


Pos.	Item Number	Description	Remarks
1	108837	BHCS 1/4 -28 x 1 PL ROHS	
2	106673	SADDLE, SEAT RAIL REVERSIBLE	
3	109367	SPACER 1/2 x 1/4 ID x 3/16 THK	
4	561002	1" INSERT PLUG .083	
5	106982	BHCS 5/16" x 1 3/4 SHLD ROHS	
5,7	107491	LATCH STUD KIT	
6	560406	FRAME BUSHING	
9	106944	WASHER FLAT THIN AN 1/4 ROHS	
1-8A,9	111211S	FRONT S/F KIDS PAINTED S/A RIGHT	RIGHT (SPECIFY COLOR) 12 -14" DEEP
1-8B,9	111213S	FRONT S/F REG PAINTED S/A RIGHT	RIGHT (SPECIFY COLOR) 13 -16" DEEP
1-8C,9	111215S	FRONT S/F LNG PAINTED S/A RIGHT	RIGHT (SPECIFY COLOR) 15 -18" DEEP
1-8D,9	111217S	FRONT S/F X-LONG PAINTED S/A RIGHT	RIGHT (SPECIFY COLOR) 17 -20" DEEP
1-8A,9	111212S	FRONT S/F KIDS PAINTED S/A LEFT	LEFT (SPECIFY COLOR) 12 -14" DEEP
1-8B,9	111214S	FRONT S/F REG PAINTED S/A LEFT	LEFT (SPECIFY COLOR) 13 -16" DEEP
1-8C,9	111216S	FRONT S/F LNG PAINTED S/A LEFT	LEFT (SPECIFY COLOR) 15 -18" DEEP
1-8D,9	111218S	FRONT S/F X-LONG PAINTED S/A LEFT	LEFT (SPECIFY COLOR) 17 -20" DEEP
10	560735	7/8" ROUND TUBE PLUG	No SA Hanger SEE NOTE

NOTE: X-LONG SIDE FRAME WILL ADD 2.5LBS TO THE TOTAL WEIGHT OF THE CHAIR.

NOTE: IF GROWING CHAIR FROM ANY SHORTER HEMI FRONT FRAME TO EXTRA LONG, THE HEMI REAR SIDE FRAME NEEDS TO BE CHANGED TO A STANDARD REAR SIDE FRAME.

NOTE: ORDER PART 14 (TUBE CAP QTY 2) WHEN SWING-AWAY FOOTRESTS ARE NOT INSTALLED.

NOTE: NEW SIDE FRAME ASSEMBLY COMES WITH (4) SEAT SADDLES TO REPLACE OLD SEAT SADDLES.

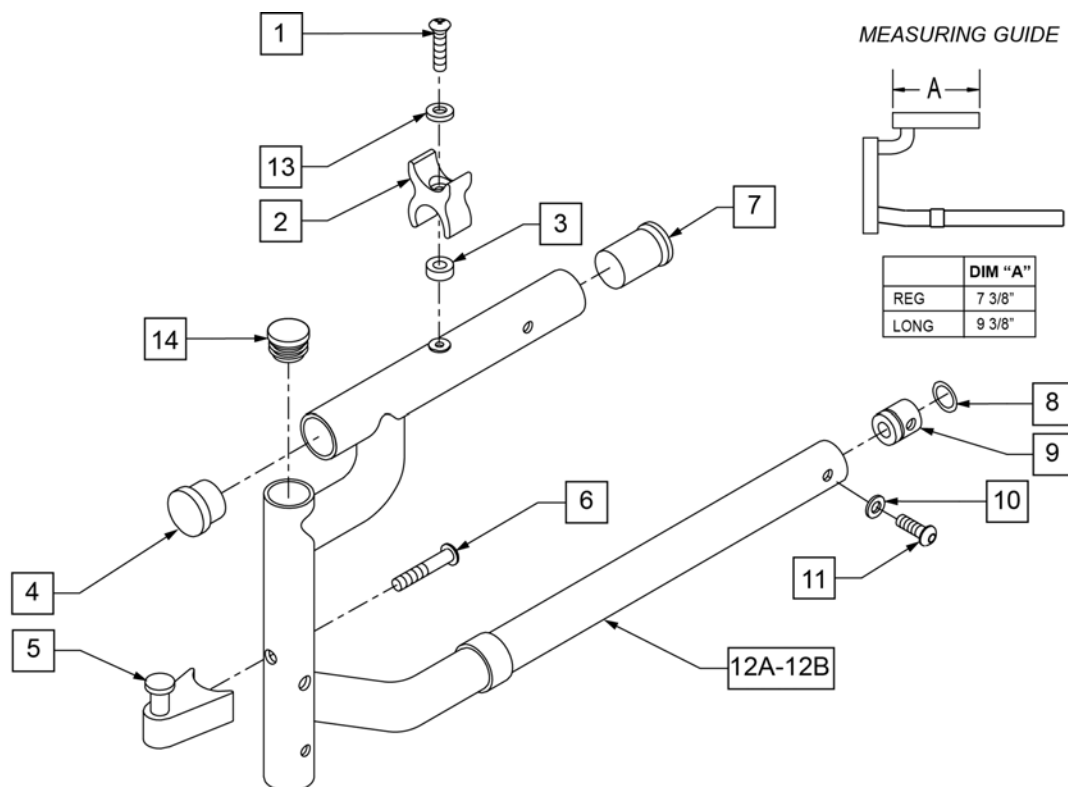


Pos.	Item Number	Description	Remarks
1	108837	BHCS 1/4 -28 x 1 PL ROHS	
2	106673	SADDLE, SEAT RAIL REVERSIBLE	
3	109367	SPACER 1/2 x 1/4 ID x 3/16 THK	
4	561002	1" INSERT PLUG .083	
6	106982	BHCS 5/16 -18 x 1 3/4 SHLD ROHS	
7	560406	FRAME BUSHING	
8	342013	PLUG O-RING RUBBER HEMI	
9	483022	FRAME INSERT HEMI	
10	262130	6mm WASHER WAVE	
11	226101	1/4 -20 x 3/4 BH C/S	
13	106944	WASHER FLAT THIN AN 1/4 ROHS	
5,6	107491	LATCH/STUD KIT	
8,9	880140	FRAME INSERT W/ O- RING HEMI	
1-12A-13	111220S	FRONT S/F STD HEMI PNTD S/A LEFT	LEFT (SPECIFY COLOR) 13 -16" DEEP
1-12A-13	111219S	FRONT S/F STD HEMI PNTD S/A RIGHT	RIGHT (SPECIFY COLOR) 13 -16" DEEP
1-12B-13	111222S	FRONT S/F LONG HEMI PNTD S/A LEFT	LEFT (SPECIFY COLOR) 15 -18" DEEP
1-12B-13	111221S	FRONT S/F LONG HEMI PNTD S/A RIGHT	RIGHT (SPECIFY COLOR) 15 -18" DEEP
14	560735	7/8" ROUND TUBE PLUG	No SA Hanger


NOTE: IF GROWING CHAIR FROM ANY SHORTER HEMI FRONT FRAME TO EXTRA LONG, THE HEMI REAR SIDE FRAME NEEDS TO BE CHANGED TO A STANDARD REAR SIDE FRAME.

NOTE: ORDER PART 10 (TUBE CAP QTY 2) WHEN SWING-AWAY FOOTRESTS ARE NOT INSTALLED.


NOTE: NEW SIDE FRAME ASSEMBLY COMES WITH (4) SEAT SADDLES TO REPLACE OLD SEAT SADDLES.

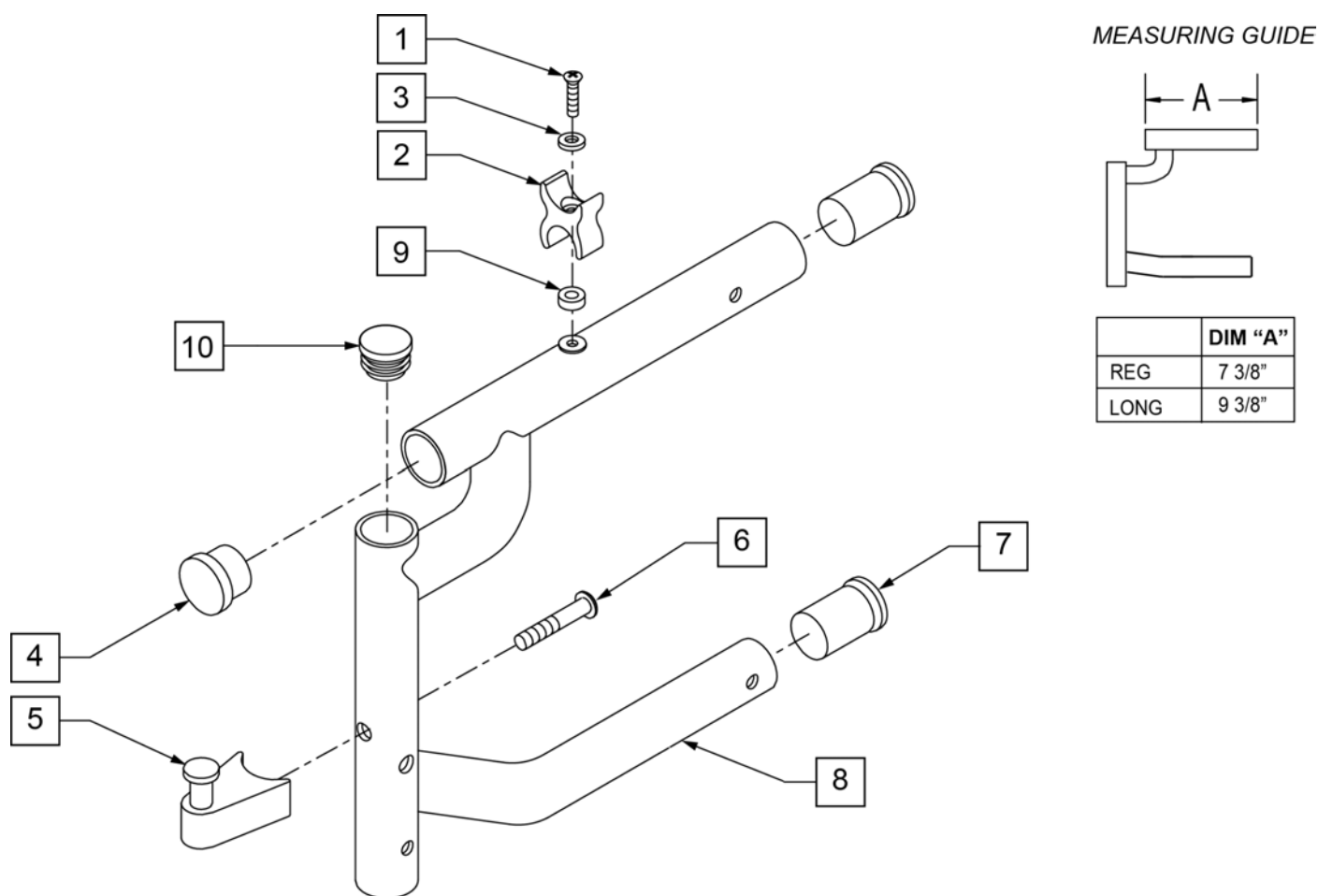

Pos.	Item Number	Description	Remarks
1	108837	BHCS 1/4 -28 x 1 PL ROHS	
2	106673	SADDLE, SEAT RAIL REVERSIBLE	
3	106944	WASHER FLAT THIN AN 1/4 ROHS	
4	561002	1" INSERT PLUG .083	
6	106982	BHCS 5/16 -18 x 1 3/4 SHLD ROHS	
7	560406	FRAME BUSHING	
9	109367	SPACER 1/2 x 1/4 ID x 3/16 THK	
5,6	107491	LATCH/STUD KIT	
1-9	111223S	FRONT S/F X-LONG HEMI PNTD RIGHT	17 -20" DEEP (SPECIFY COLOR)
1-9	111224S	FRONT S/F X-LONG HEMI PNTD LEFT	17 -20" DEEP (SPECIFY COLOR)
10	560735	7/8" ROUND TUBE PLUG	Use in place of SA Hanger

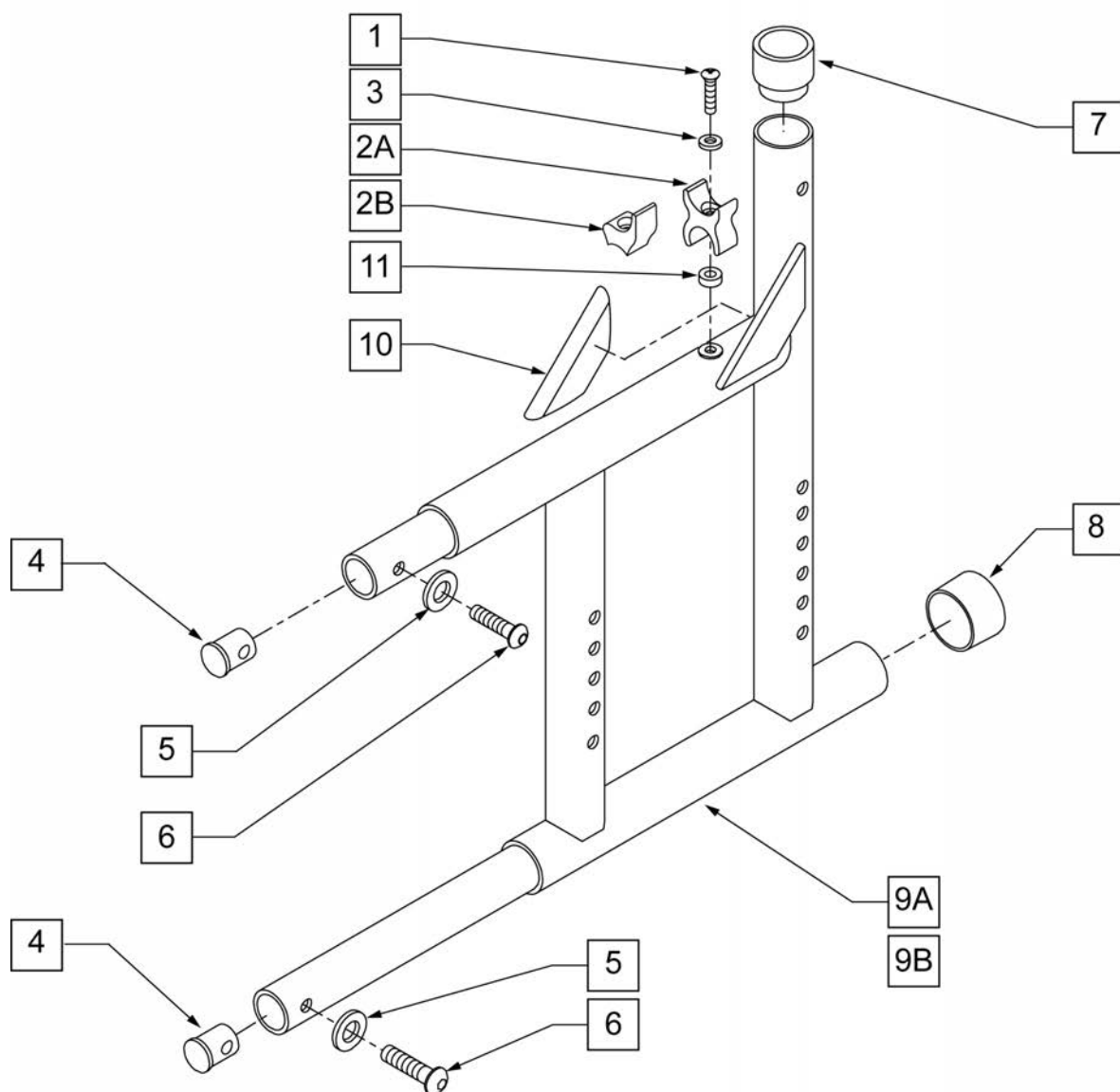


Pos.	Item Number	Description	Remarks
1	108837	BHCS 1/4 -28 x 1 PL ROHS	
2A	106673	SADDLE, SEAT RAIL REVERSIBLE	
2B	112866	SADDLE SEAT RAIL REAR HD RAMQ	HD CHAIRS ONLY
3	106944	WASHER FLAT THIN AN 1/4 ROHS	
4	483001	Q2 FRAME/FOOT RETAINING PLUG	
5	104414	WAVE WASHER 6 MM	
6	107353	BHCS 1/4-20 X 1 ROHS	
7	561083	1 1/8" TUBE END BUSHING THIN	
8	561014	1" MED CAP	
10	341300	Q2 GUSSET WEAR STRIP	
11	109367	SPACER 1/2 x 1/4ID x 3/16 THK	
1,2A-11	111227S	REAR S/F PAINTED Q2 R 1 1/8	RIGHT (SPECIFY COLOR) Fits chairs w/ seriel# prefix Q2V
1,2A-11	111228S	REAR S/F PAINTED Q2 L 1 1/8	LEFT (SPECIFY COLOR) Fits chairs w/ seriel# prefix Q2V
1,2B-11	112627S	REAR S/F PAINTED Q2 R HD	RIGHT (SPECIFY COLOR) HD ONLY Fits chairs w/ seriel# prefix Q2H & Q2
1,2B-11	112628S	REAR S/F PAINTED Q2 L HD	LEFT (SPECIFY COLOR) HD ONLY Fits chairs w/ seriel# prefix Q2H & Q2

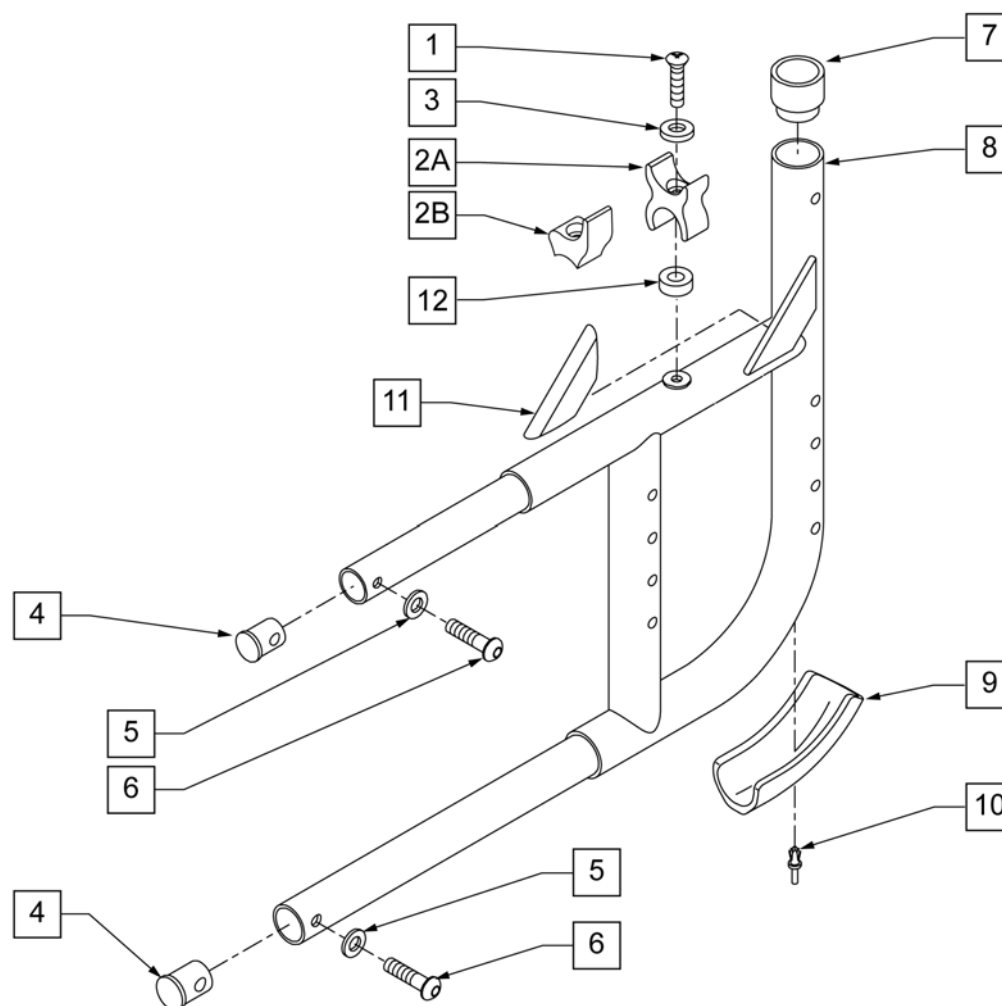


Pos.	Item Number	Description	Remarks
1	108837	BHCS 1/4 -28 x 1 PL ROHS	
2A	106673	SADDLE SEAT RAIL REVERSIBLE	
2B	112866	SADDLE SEAT RAIL REAR HD RAMQ	HD CHAIRS ONLY
3	106944	WASHER FLAT THIN AN 1/4 ROHS	
4	483001	Q2 FRAME/FOOT RETAINING PLUG	
5	104414	WAVE WASHER 6 MM	
6	107353	BHCS 1/4 -20 x 1 ROHS	
7	561083	1-1/8" TUBE END BUSHING THIN	
9	560409	REAR FRAME BUMPER Q2	
10	240210	RIVET 3/16 PLASTIC	
11	341300	Q2 GUSSET WEARSTRIP	
12	109367	SPACER 1/2 x 1/4 ID x 3/16 THK	
1,2A-12	111229S	REAR S/F PNTD Q2 CURVED	RIGHT (SPECIFY COLOR) Fits chairs w/ serial# prefix Q2V
1,2A-12	111230S	REAR S/F PAINTED Q2 CURVED	LEFT (SPECIFY COLOR) Fits chairs w/ serial# prefix Q2V
1,2B-12	112629S	REAR S/F PAINTED Q2 CURVED R HD	RIGHT (SPECIFY COLOR) HD ONLY Fits chairs w/ serial# prefix Q2H & Q2
1,2B-12	112630S	REAR S/F PAINTED Q2 CURVED L HD	LEFT (SPECIFY COLOR) HD ONLY Fits chairs w/ serial# prefix Q2H & Q2

Note: The HEMI rear side frame is used with "STANDARD AND LONG HEMI FRONT FRAME" ONLY. If Hemi X-long front side frame is used, the "STANDARD REAR SIDE FRAME" is used.

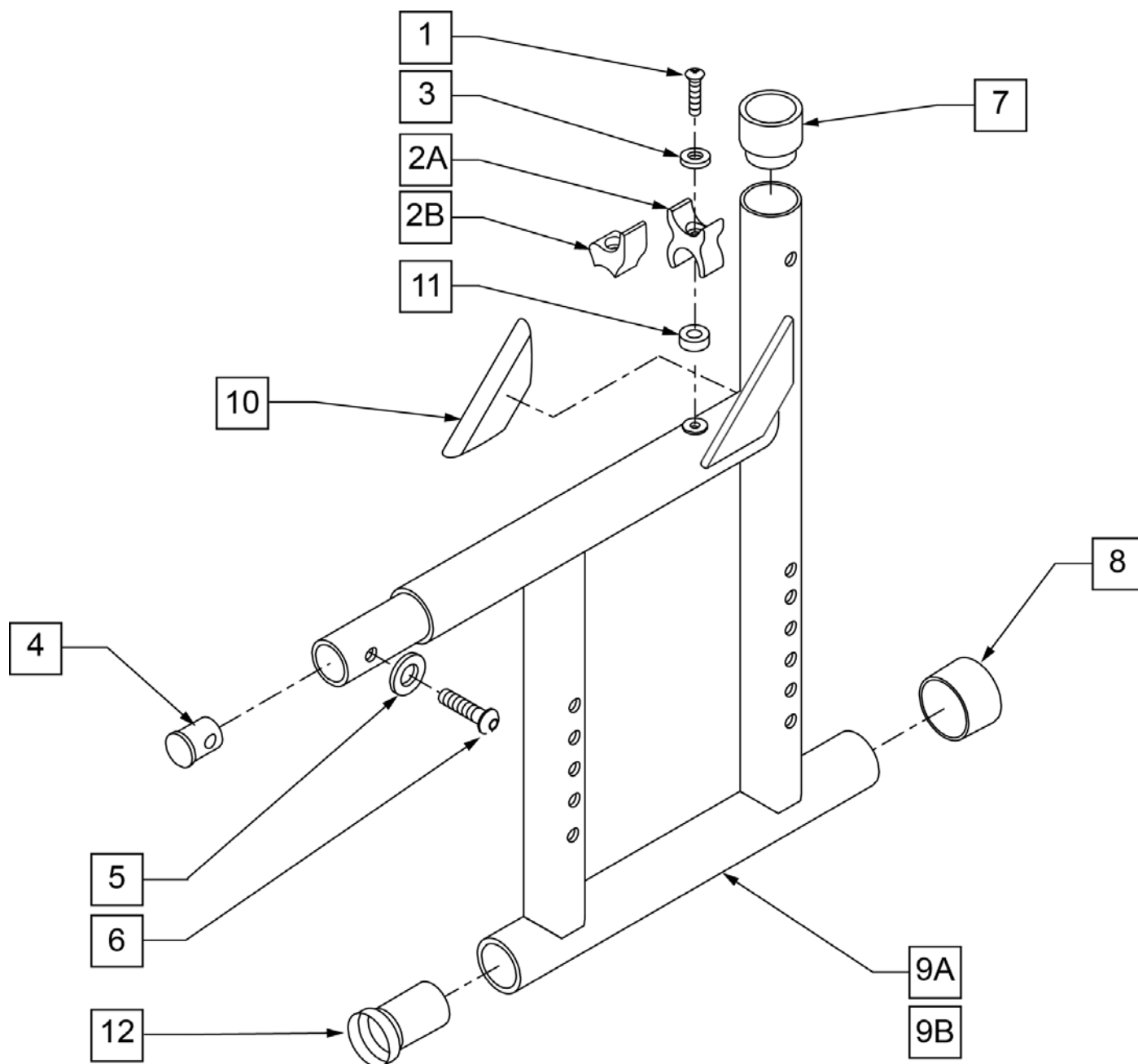


Pos.	Item Number	Description	Remarks
1	108837	BHCS 1/4 -28 x 1 PL ROHS	
2A	106673	SADDLE, SEAT RAIL REVERSIBLE	
2B	112866	SADDLE SEAT RAIL REAR HD RAMQ	HD CHAIRS ONLY
3	106944	WASHER FLAT THIN AN 1/4 ROHS	
4	483001	Q2 FRAME/FOOT RETAINING PLUG	
5	104414	WAVE WASHER 6 MM	
6	107353	BHCS 1/4-20 X 1 ROHS	
7	561083	1 1/8" TUBE END BUSHING THIN	
8	561014	1" MED CAP	
10	341300	Q2 GUSSET WEAR STRIP	
11	109367	SPACER 1/2 x 1/4ID x 3/16 THK	
1,2A-12	111225S	REAR S/F PNTD HEMI R	RIGHT (SPECIFY COLOR) Fits chairs w/ serial# prefix Q2V
1,2A-12	111226S	REAR S/F PNTD HEMI L	LEFT (SPECIFY COLOR) Fits chairs w/ serial# prefix Q2V
1,2B-12	112631S	REAR S/F PNTD Q2 HEMI R HD	RIGHT (SPECIFY COLOR) HD ONLY Fits chairs w/ serial# prefix Q2H & Q2
1,2B-12	112632S	REAR S/F PNTD Q2 HEMI L HD	LEFT (SPECIFY COLOR) HD ONLY Fits chairs w/ serial# prefix Q2H & Q2

NOTE: The HEMI rear side frame is used with "STANDARD AND LONG HEMI FRONT FRAME" ONLY. If Hemi X-long front side frame is used, the "STANDARD REAR SIDE FRAME" is used.

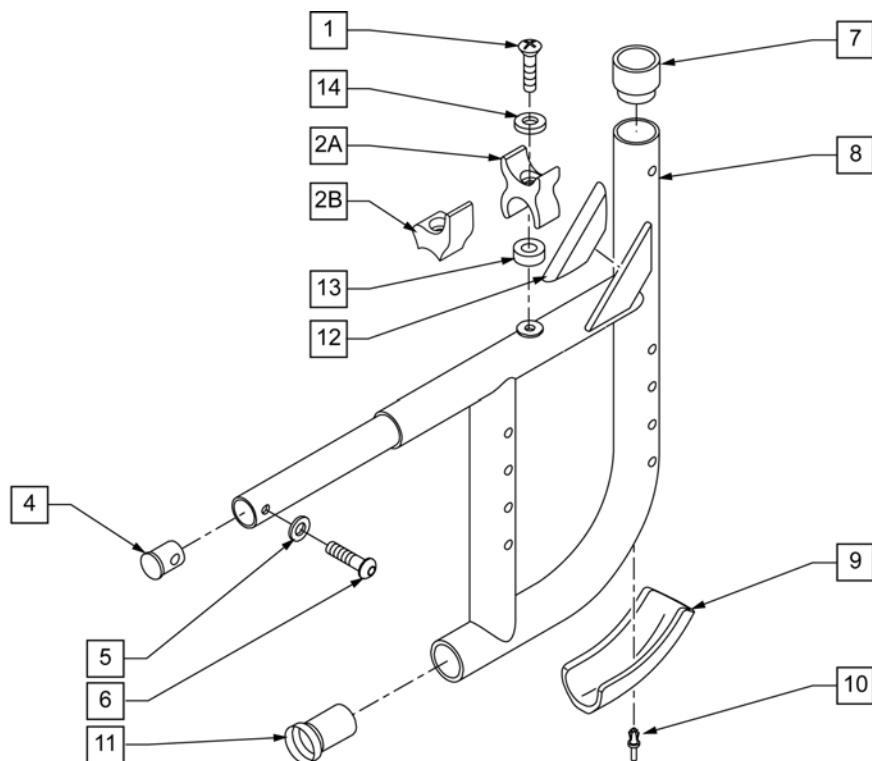


Pos.	Item Number	Description	Remarks
1	108837	BHCS 1/4 -28 x 1 PL ROHS	
2A	106673	SADDLE SEAT RAIL REVERSIBLE	
2B	112866	SADDLE SEAT RAIL REAR HD RAMQ	HD CHAIRS ONLY
4	483001	Q2 FRAME/FOOT RETAINING PLUG	
5	104414	WAVE WASHER 6 MM	
6	107353	BHCS 1/4 -20 x 1 ROHS	
7	561083	1-1/8" TUBE END BUSHING THIN	
9	560409	REAR FRAME BUMPER Q2	
10	240210	RIVET 3/16 PLASTIC	
11	560408	FRAME BUSHING 3/4 - 1 1/8"	
12	341300	Q2 GUSSET WEARSTRIP	
13	109367	SPACER 1/2 x 1/4 ID x 3/16 THK	
14	106944	WASHER FLAT THIN AN 1/4 ROHS	
1,2A-12	111231S	REAR S/F PAINTED Q2 HEMI CURVED -RH	Right (Specify Color) Fits chairs w/ SN prefix Q2V
1,2A-12	111232S	REAR S/F PAINTED Q2 HEMI CURVED -LH	LEFT (Specify Color) Fits chairs w/ serial# prefix Q2V
1,2B-12	112633S	REAR S/F PAINTED Q2 HEMI CURVED R HD	RIGHT (SPECIFY COLOR) HD ONLY Fits chairs w/ serial# prefix Q2H & Q2
1,2B-12	112634S	REAR S/F PAINTED Q2 HEMI CURVED L HD	LEFT (SPECIFY COLOR) HD ONLY Fits chairs w/ serial# prefix Q2H & Q2

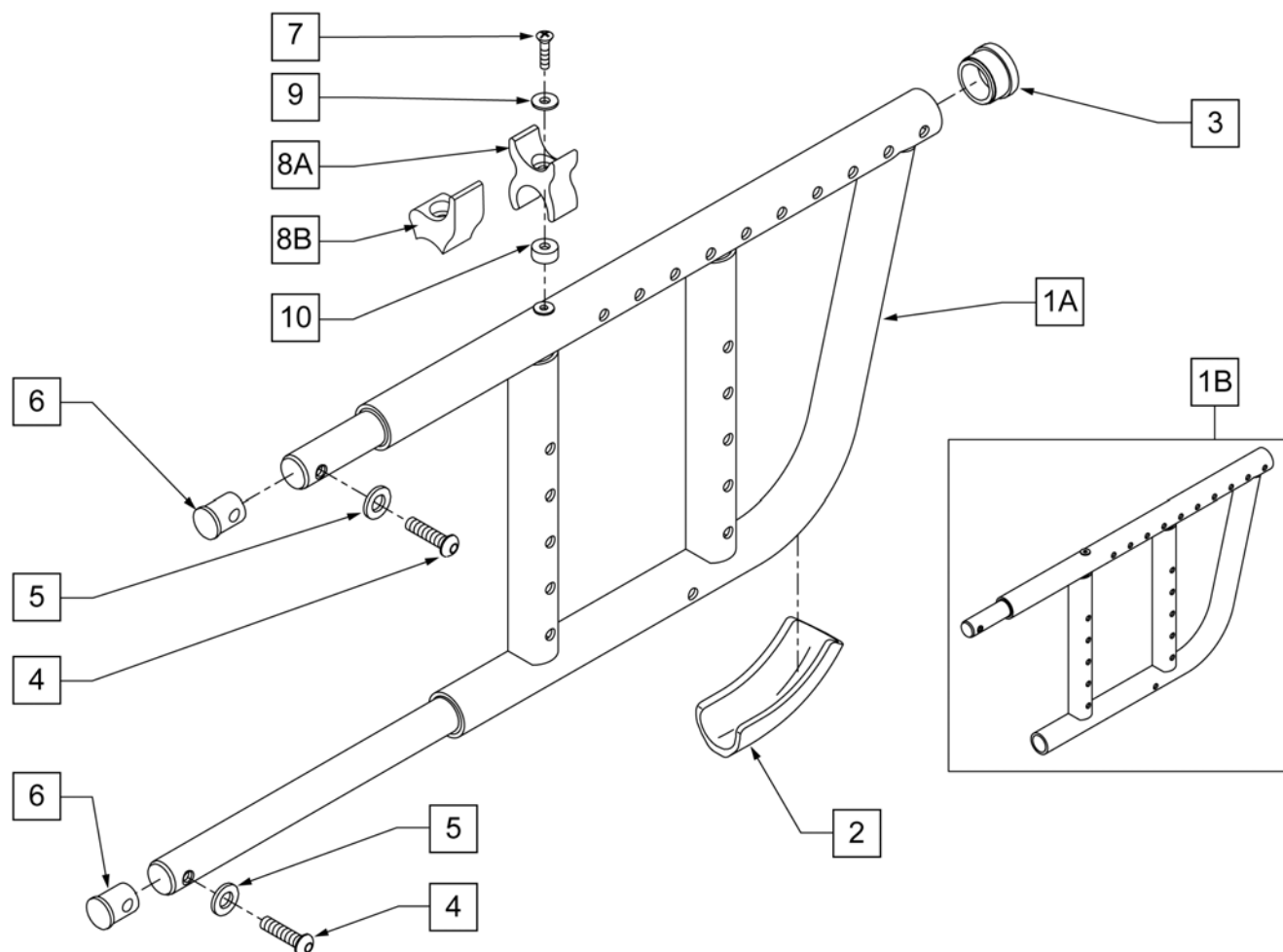

Pos.	Item Number	Description	Remarks
2	560413	REAR FRAME BUMPER GPS	
3	561002	1" INSERT PLUG .083	
4	107353	BHCS 1/4-20 x 1 ROHS	
5	104414	WAVE WASHER 6mm	
6	483001	Q2 FRAME/FOOT RETAINING PLUG	
7	108837	BHCS 1/4-28 x 1 PL ROHS	
8A	106673	SADDLE SEAT RAIL REVERSABLE	
8B	112866	SADDLE SEAT RAIL REAR HD RAMQ	HD CHAIRS ONLY
9	106944	WASHER FLAT THIN AN 1/4 ROHS	
10	109367	SPACER 1/2 x 1/4 ID x 3/16 THK	
1A,2-8A-10	111233S	REAR S/F DAB PNTD R Q2	RIGHT (SPECIFY COLOR) Fits chairs w/ serie# prefix Q2V
1A,2-8A-10	111234S	REAR S/F DAB PNTD L Q2	LEFT (SPECIFY COLOR) Fits chairs w/ serie# prefix Q2V
1B,2-8A-10	111235S	REAR S/F DAB HEMI PNTD R Q2	RIGHT (SPECIFY COLOR) Fits chairs w/ serie# prefix Q2V
1B,2-8A-10	111236S	REAR S/F DAB HEMI PNTD L Q2	LEFT (SPECIFY COLOR) Fits chairs w/ serie# prefix Q2V
1A,2-8B-10	112635S	REAR S/F DAB HEMI Q2 R HD	RIGHT (SPECIFY COLOR) HD ONLY Fits chairs w/ serie# prefix Q2H & Q2
1A,2-8B-10	112636S	REAR S/F DAB HEMI Q2 L HD	LEFT (SPECIFY COLOR) HD ONLY Fits chairs w/ serie# prefix Q2H & Q2
1B,2-8B-10	112637S	REAR S/F DAB HEMI PAINTED R HD	RIGHT (SPECIFY COLOR) HD ONLY Fits chairs w/ serie# prefix Q2H & Q2
1B,2-8B-10	112638S	REAR S/F DAB HEMI PAINTED L HD	LEFT (SPECIFY COLOR) HD ONLY Fits chairs w/ serie# prefix Q2H & Q2

NOTE: In order to properly mount a JAY DUO CUSHION; Qty (2) of the appropriate size SEAT RAIL INSERT (Item 9) must be ordered.

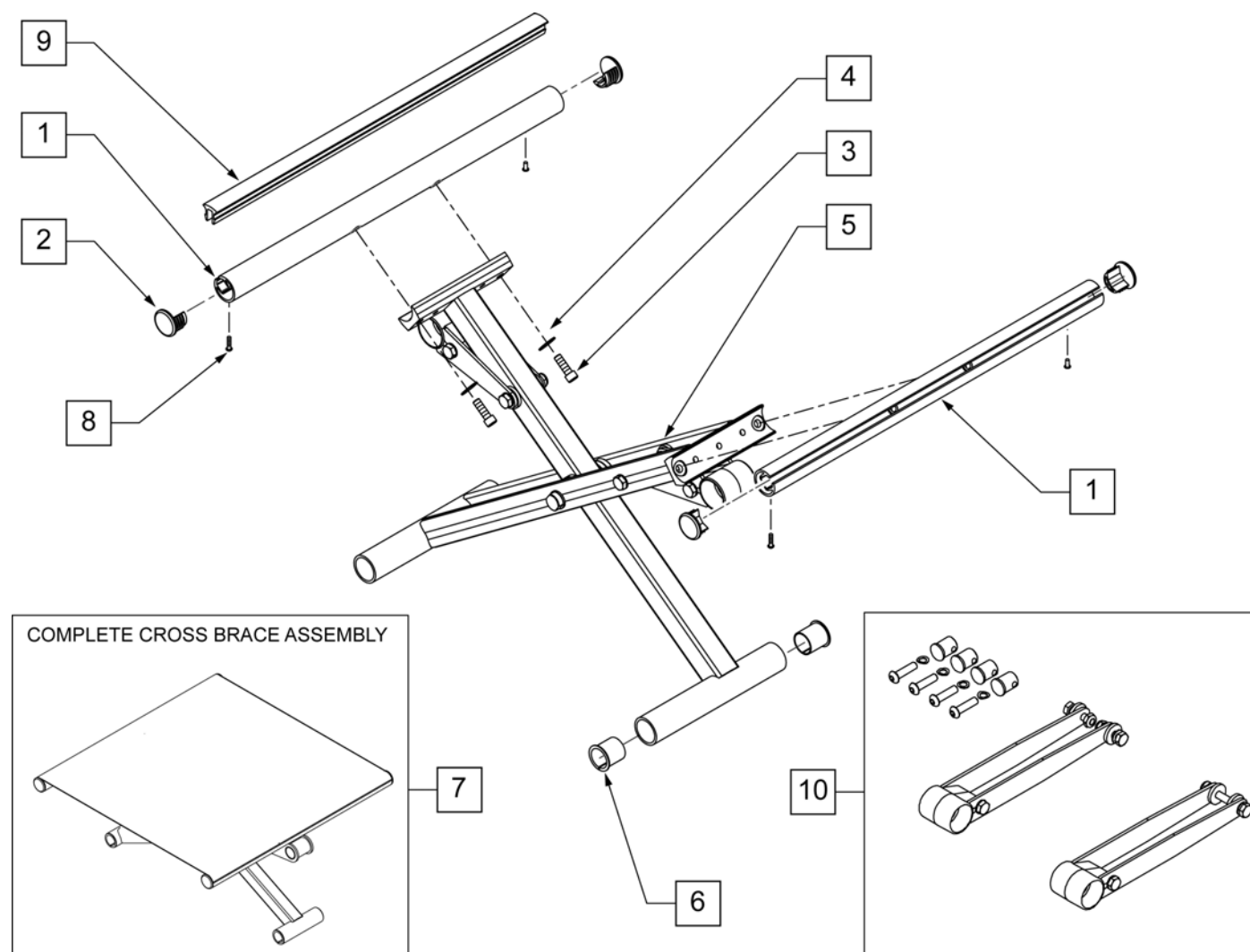

Pos.	Item Number	Description	Remarks
2,8	108117	END PLUG KIT (4)	SET OF 4
3	106974	SHCS 1/4-28 X 3/4 PL ROHS	
4	106944	WASHER FLAT THIN AN 1/4 ROHS	
6	108116	3/4" NY LINER (4)	SET OF 4
5,6	111569	XTUBE ASSM 12W	12" WIDE
5,6	111570	XTUBE ASSM 13W	13" WIDE
5,6	111571	XTUBE ASSM 14W	14" WIDE
5,6	111572	XTUBE ASSM 15W	15" WIDE
5,6	111573	XTUBE ASSM 16W	16" WIDE
5,6	111574	XTUBE ASSM 17W	17" WIDE
5,6	111575	XTUBE ASSM 18W	18" WIDE
5,6	111576	XTUBE ASSM 19W	19" WIDE
5,6	111577	XTUBE ASSM 20W	20" WIDE
1-4,8	111567	SEAT RAIL KIT 12D	12" DEEP-Each


Pos.	Item Number	Description	Remarks
1-4,8	111568	SEAT RAIL KIT 13D	13" DEEP-Each
1-4,8	107495	SEAT RAIL KIT 14D	14" DEEP-Each
1-4,8	107496	SEAT RAIL KIT 15D	15" DEEP-Each
1-4,8	107497	SEAT RAIL KIT 16D	16" DEEP-Each
1-4,8	107498	SEAT RAIL KIT 17D	17" DEEP-Each
1-4,8	107499	SEAT RAIL KIT 18D	18" DEEP-Each
1-4,8	107500	SEAT RAIL KIT 19D	19" DEEP-Each
1-4,8	107501	SEAT RAIL KIT 20D	20" DEEP-Each
7	111578	X BRACE ASSM 12 x 12	
7	111579	X BRACE ASSM 12 x 13	
7	111580	X BRACE ASSM 12 x 14	
7	111581	X BRACE ASSM 12 x 15	
7	111582	X BRACE ASSM 12 x 16	
7	111583	X BRACE ASSM 12 x 17	
7	111584	X BRACE ASSM 12 x 18	
7	111587	X BRACE ASSM 13 x 12	
7	111588	X BRACE ASSM 13 x 13	
7	111589	X BRACE ASSM 13 x 14	
7	111590	X BRACE ASSM 13 x 15	
7	111591	X BRACE ASSM 13 x 16	
7	111592	X BRACE ASSM 13 x 17	
7	111593	X BRACE ASSM 13 x 18	
7	111594	X BRACE ASSM 13 x 19	
7	111595	X BRACE ASSM 13 x 20	
7	111596	X BRACE ASSM 14 x 12	
7	111597	X BRACE ASSM 14 x 13	
7	111598	X BRACE ASSM 14 x 14	
7	111599	X BRACE ASSM 14 x 15	
7	111600	X BRACE ASSM 14 x 16	
7	111601	X BRACE ASSM 14 x 17	
7	111602	X BRACE ASSM 14 x 18	
7	111603	X BRACE ASSM 14 x 19	
7	111604	X BRACE ASSM 14 x 20	
7	111605	X BRACE ASSM 15 x 12	
7	111606	X BRACE ASSM 15 x 13	
7	111607	X BRACE ASSM 15 x 14	
7	111608	X BRACE ASSM 15 x 15	
7	111609	X BRACE ASSM 15 x 16	
7	111610	X BRACE ASSM 15 x 17	
7	111611	X BRACE ASSM 15 x 18	
7	111612	X BRACE ASSM 15 x 19	
7	111613	X BRACE ASSM 15 x 20	
7	111614	X BRACE ASSM 16 x 12	
7	111615	X BRACE ASSM 16 x 13	
7	111616	X BRACE ASSM 16 x 14	
7	111617	X BRACE ASSM 16 x 15	
7	111618	X BRACE ASSM 16 x 16	
7	111619	X BRACE ASSM 16 x 17	
7	111620	X BRACE ASSM 16 x 18	
7	111621	X BRACE ASSM 16 x 19	
7	111622	X BRACE ASSM 16 x 20	
7	111623	X BRACE ASSM 17 x 12	
7	111624	X BRACE ASSM 17 x 13	
7	111625	X BRACE ASSM 17 x 14	
7	111626	X BRACE ASSM 17 x 15	
7	111627	X BRACE ASSM 17 x 16	

Pos.	Item Number	Description	Remarks
7	111628	X BRACE ASSM 17 x 17	
7	111629	X BRACE ASSM 17 x 18	
7	111630	X BRACE ASSM 17 x 19	
7	111631	X BRACE ASSM 17 x 20	
7	111632	X BRACE ASSM 18 x 12	
7	111633	X BRACE ASSM 18 x 13	
7	111634	X BRACE ASSM 18 x 14	
7	111635	X BRACE ASSM 18 x 15	
7	111636	X BRACE ASSM 18 x 16	
7	111637	X BRACE ASSM 18 x 17	
7	111638	X BRACE ASSM 18 x 18	
7	111639	X BRACE ASSM 18 x 19	
7	111640	X BRACE ASSM 18 x 20	
7	111641	X BRACE ASSM 19 x 12	
7	111642	X BRACE ASSM 19 x 13	
7	111643	X BRACE ASSM 19 x 14	
7	111644	X BRACE ASSM 19 x 15	
7	111645	X BRACE ASSM 19 x 16	
7	111646	X BRACE ASSM 19 x 17	
7	111647	X BRACE ASSM 19 x 18	
7	111648	X BRACE ASSM 19 x 19	
7	111649	X BRACE ASSM 19 x 20	
7	111650	X BRACE ASSM 20 x 12	
7	111651	X BRACE ASSM 20 x 13	
7	111652	X BRACE ASSM 20 x 14	
7	111653	X BRACE ASSM 20 x 15	
7	111654	X BRACE ASSM 20 x 16	
7	111655	X BRACE ASSM 20 x 17	
7	111656	X BRACE ASSM 20 x 18	
7	111657	X BRACE ASSM 20 x 19	
7	111658	X BRACE ASSM 20 x 20	
8	109819	SCREW #6 x 1/4 SLF TP PHL PN HD	
9	108773	SEAT RAIL PLASTIC INSERT 18"	For 12' to 18" deep, cut to fit
9	108774	SEAT RAIL PLASTIC INSERT 22"	For 19' to 20" deep, cut to fit
10	112527	Q2 X-HINGE REPLACEMENT KIT 12"W	
10	115888	Q2 X-HINGE REPLACEMENT KIT 13"W	
10	115890	Q2 X-HINGE REPLACEMENT KIT 14"W	
10	115891	Q2 X-HINGE REPLACEMENT KIT 15"W	
10	115892	Q2 X-HINGE REPLACEMENT KIT 16"W	
10	115893	Q2 X-HINGE REPLACEMENT KIT 17"W	
10	115894	Q2 X-HINGE REPLACEMENT KIT 18"W	
10	115895	Q2 X-HINGE REPLACEMENT KIT 19"W	
10	115896	Q2 X-HINGE REPLACEMENT KIT 20"W	

NOTE: In order to properly mount a Jay Duo Cushion Qty (2) of the appropriate size seat rail insert (Item 15) must be ordered.









NOTE: Rod style Seat Sling effective 10/23/99 beginning serial number Q2-242051. If chair was equipped with Screw Mount Seat Sling then Cross Tube must be replaced.

NOTE: Item 15 is only utilized when the seat sling is not used.

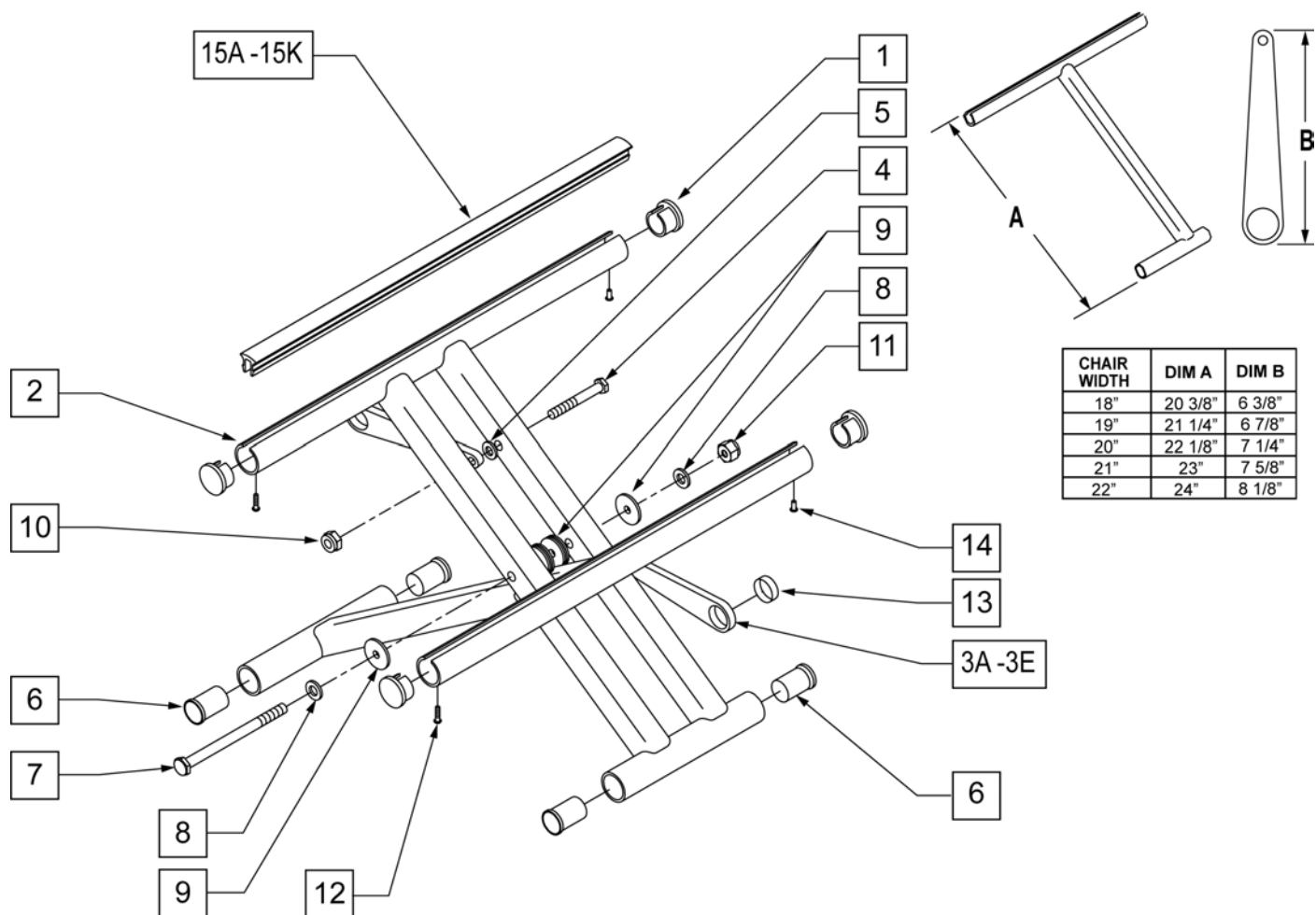


Pos.	Item Number	Description	Remarks	Pos.	Item Number	Description	Remarks
1	483552	SEAT RAIL END PLUG		10	106934	1/4-28 NYLOCK NUT	
3A	923606	X-HINGE 18"	PAIR	11	106939	5/16-24 NYLOCK NUT THIN	
3B	923607	X-HINGE 19"	PAIR	12	281131	#10 3/8" SELF TAPPING SCREW	
3C	923608	X-HINGE 20"	PAIR	13	580200	TEFLON STRIP BEARING	
3D	923609	X-HINGE 21"	PAIR	14	240111	RIVET POP AD55H BLK	
3E	923610	X-HINGE 22"	PAIR	15A	580021	SEAT RAIL INSERT STRIP 11"	
4	107015	1/4-28 x 1 3/4 G8 HEX C/S		15B	580022	SEAT RAIL INSERT STRIP 12"	
5	561028	1/4" NYLON WASHER		15C	580023	SEAT RAIL INSERT STRIP 13"	
6	560402	3/4" NY-LINER		15D	580024	SEAT RAIL INSERT STRIP 14"	
7	222263	5/16-24 x 4-3/8 HHCS GR8		15E	580025	SEAT RAIL INSERT STRIP 15"	
8	106947	5/16 AN WASHER FLAT THIN		15F	580026	SEAT RAIL INSERT STRIP 16"	
9	561016	1" WASHER 5/16" HOLE		15G	580027	SEAT RAIL INSERT STRIP 17"	

Pos.	Item Number	Description	Remarks	Pos.	Item Number	Description	Remarks
15H	580028	SEAT RAIL INSERT STRIP 18"		1-14	673615	X-TUBE ASSM 20" x14" CHNL HD	
15J	580029	SEAT RAIL INSERT STRIP 19"		1-14	673616	X-TUBE ASSM 20" x15" CHNL HD	
15K	580030	SEAT RAIL INSERT STRIP 20"		1-14	673617	X-TUBE ASSM 20" x16" CHNL HD	
1-14	673592	X-TUBE ASSM 18" x11" CHNL HD		1-14	673618	X-TUBE ASSM 20" x17" CHNL HD	
1-14	673593	X-TUBE ASSM 18" x12" CHNL HD		1-14	673619	X-TUBE ASSM 20" x18" CHNL HD	
1-14	673594	X-TUBE ASSM 18" x13" CHNL HD		1-14	673620	X-TUBE ASSM 20" x19" CHNL HD	
1-14	673595	X-TUBE ASSM 18" x14" CHNL HD		1-14	673621	X-TUBE ASSM 20" x20" CHNL HD	
1-14	673596	X-TUBE ASSM 18" x15" CHNL HD		1-14	673622	X-TUBE ASSM 21" x12" CHNL HD	
1-14	673597	X-TUBE ASSM 18" x16" CHNL HD		1-14	673623	X-TUBE ASSM 21" x13" CHNL HD	
1-14	673598	X-TUBE ASSM 18" x17" CHNL HD		1-14	673624	X-TUBE ASSM 21" x14" CHNL HD	
1-14	673599	X-TUBE ASSM 18" x18" CHNL HD		1-14	673625	X-TUBE ASSM 21" x15" CHNL HD	
1-14	673600	X-TUBE ASSM 18" x19" CHNL HD		1-14	673626	X-TUBE ASSM 21" x16" CHNL HD	
1-14	673601	X-TUBE ASSM 18" x20" CHNL HD		1-14	673627	X-TUBE ASSM 21" x17" CHNL HD	
1-14	673602	X-TUBE ASSM 19" x11" CHNL HD		1-14	673628	X-TUBE ASSM 21" x18" CHNL HD	
1-14	673603	X-TUBE ASSM 19" x12" CHNL HD		1-14	673629	X-TUBE ASSM 21" x19" CHNL HD	
1-14	673604	X-TUBE ASSM 19" x13" CHNL HD		1-14	673630	X-TUBE ASSM 21" x20" CHNL HD	
1-14	673605	X-TUBE ASSM 19" x14" CHNL HD		1-14	673631	X-TUBE ASSM 22" x12" CHNL HD	
1-14	673606	X-TUBE ASSM 19" x15" CHNL HD		1-14	673632	X-TUBE ASSM 22" x13" CHNL HD	
1-14	673607	X-TUBE ASSM 19" x16" CHNL HD		1-14	673633	X-TUBE ASSM 22" x14" CHNL HD	
1-14	673608	X-TUBE ASSM 19" x17" CHNL HD		1-14	673634	X-TUBE ASSM 22" x15" CHNL HD	
1-14	673609	X-TUBE ASSM 19" x18" CHNL HD		1-14	673635	X-TUBE ASSM 22" x16" CHNL HD	
1-14	673610	X-TUBE ASSM 19" x19" CHNL HD		1-14	673636	X-TUBE ASSM 22" x17" CHNL HD	
1-14	673611	X-TUBE ASSM 19" x20" CHNL HD		1-14	673637	X-TUBE ASSM 22" x18" CHNL HD	
1-14	673612	X-TUBE ASSM 20" x11" CHNL HD		1-14	673638	X-TUBE ASSM 22" x19" CHNL HD	
1-14	673613	X-TUBE ASSM 20" x12" CHNL HD		1-14	673639	X-TUBE ASSM 22" x 20" CHNL HD	
1-14	673614	X-TUBE ASSM 20" x13" CHNL HD					

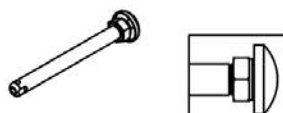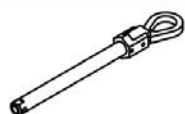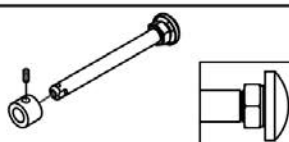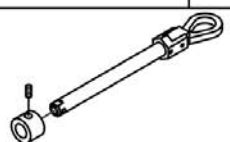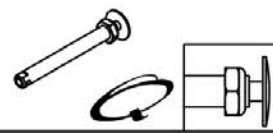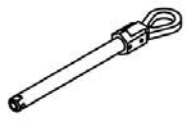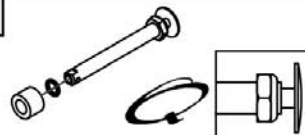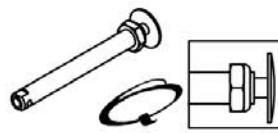
Wheel Type	Axle Type		Axle P/N	Spacer P/N
Mag Wheels	Quick Release	1A 	102063 A = 4 1/8"	N/A
	Quad Quick Release	1B 	920171 A = 4 1/8"	N/A
Mag Wheels (with Uni-lateral Wheel Locks or Travel Wheels)	Quick Release	1C 	922356 Includes Spacer A = 4 3/4"	2A  920117
	Quad Quick Release	1D 	920172 Includes Spacer A = 4 3/4"	2A  920117
Standard Spoke Wheels, Spinergy Wheels & Mountain Wheels	Quick Release	1E 	109385 A = 4 1/2"	N/A
	Quad Quick Release (N/A with Spinergy)	1F 	922170 A = 4 1/2"	N/A
Ultra Light Wheels (Not available with Quad Quick release)	Quick Release	1E 	109385 Order Spacer A = 4"	2C  103093 2B 10076 8-003
Lite Spoke	Quick Release	1G 	107268 A = 4 1/4"	N/A

Quick Release Measuring Guide

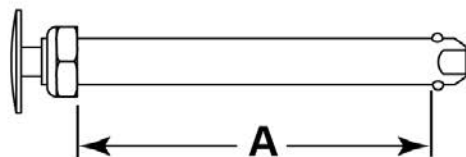


Pos.	Item Number	Description	Remarks
1A	102063	QR AXLE MAG W/M-12 NUT	
1B	920171	QR AXLE MAG W/QUAD NUT	
1C	922356	QR AXLE BIG BUTTON W/SPACER	includes spacer
1D	920172	QR AXLE MAG W/ SPCR & QUAD NUT	includes spacer
1E	109385	QR AXLE/NUT LONG BIG BUTTON	
1F	922170	QR AXLE W/QUAD NUT PUSH BUTTON	Not available with spinergy
1G	107268	QR AXLE AND NUT SHORT BIG BUTTON	
1H	100793	AXLE QR SHORT ULTRALITE	
2A	920117	MAG SPACER 6S W/SET SCREW	
2B	100768-003	SPACER AXLE 220DX12.71X12MM	
2C	103093	O-RING 1/2"IDX11/16"OD BUNA-N	

Note: To upgrade to threaded Axles from Quick Release Axles, Qty 2 is required.

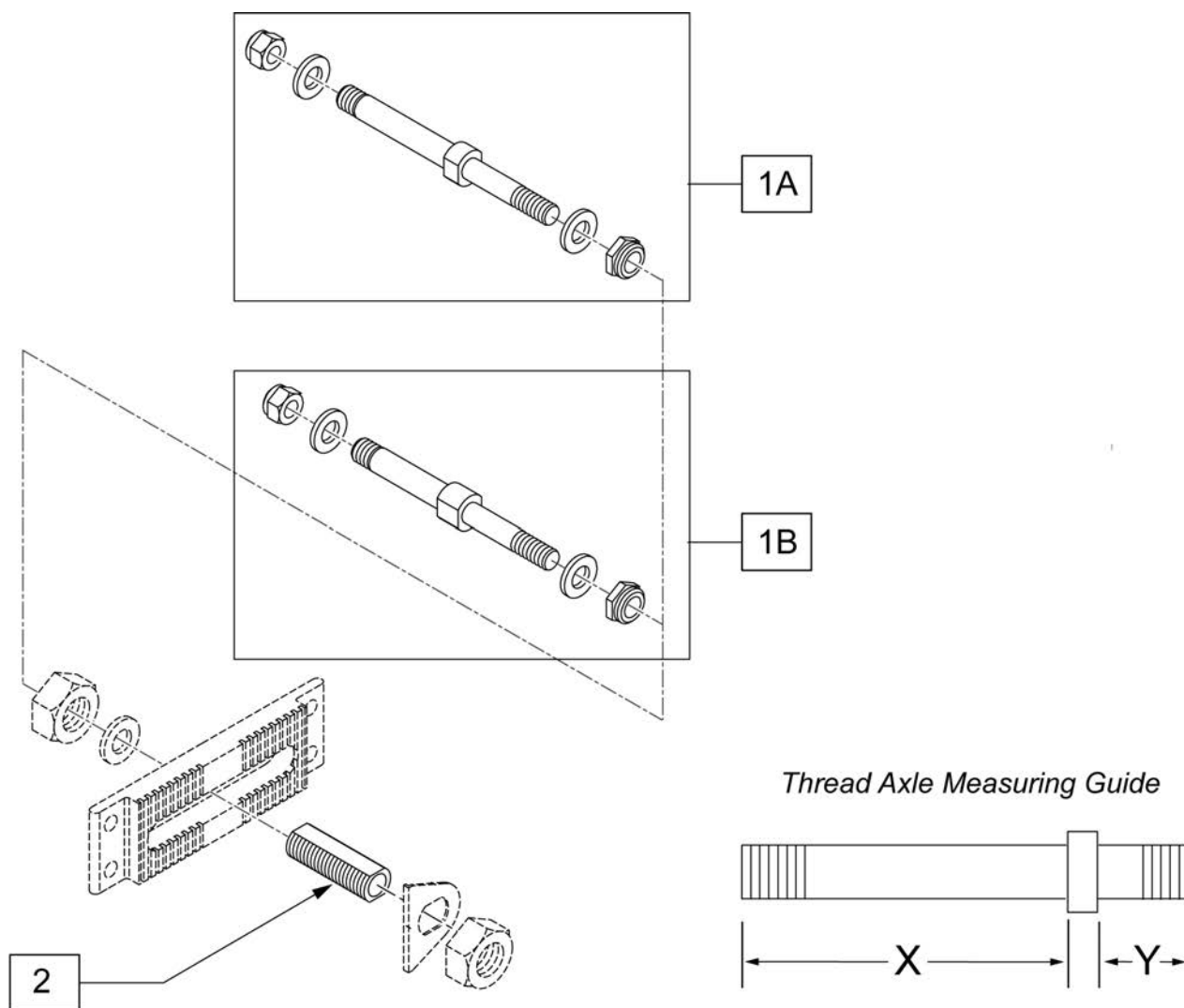


Pos.	Item Number	Description	Remarks
1A	920108	AXLE THRD MAG(ADJ AXLE PLT) LX	For Mag Wheel X= 2.5 Y=2 1/8
1B	920109	AXLE THRD SPK(ADJ AXLE PLT) LX	For Spoke Wheel X= 3 Y=2 1/8
2	920099	AXLE SLEEVE 7/16 ID W/HDWR	

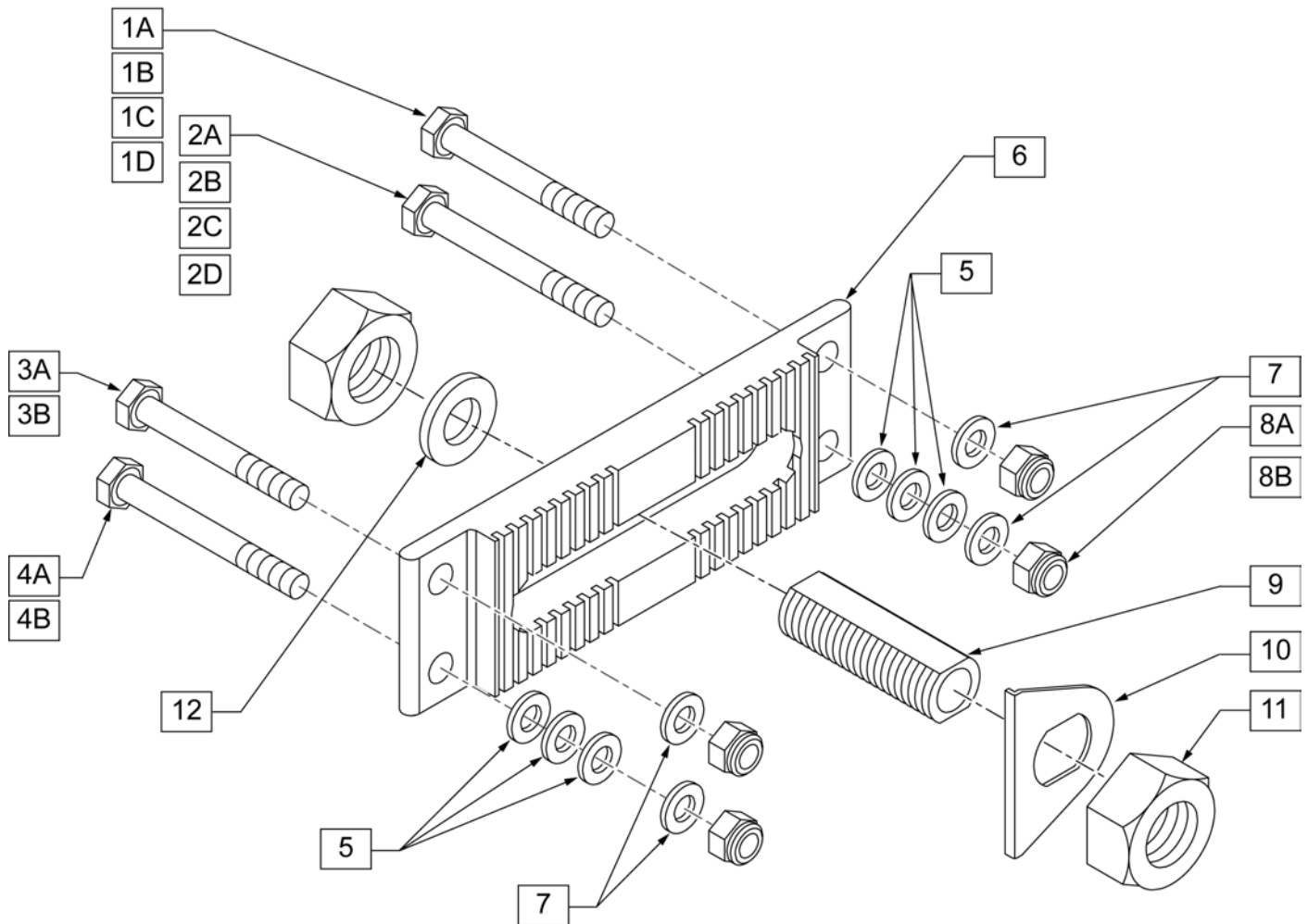


Pos.	Item Number	Description	Remarks
1A	107015	HHCS 1/4-28 X 1 3/4 G8 ROHS	(STEEL) USE W/ AXLE PLATES ONLY
1B	106973	HHCS 1/4-28 X 2 ROHS	(STEEL) USE W/ ANTI TIPS or TRANSIT
1C	107351	HHCS 1/4-28 X 2 1/4 G8 ROHS	(STEEL) USE W/ ANTI TIPS & TRANSIT &/OR ARMREST RECEIVER
1D	330099	1/4-28 x 1 3/4 HEX C/S TI	(TITANIUM) USE W/ AXLE PLATES ONLY

Pos.	Item Number	Description	Remarks
2A	106973	HHCS 1/4-28 X 2 ROHS	(STEEL) USE W/ AXLE PLATES ONLY
2B	107351	HHCS 1/4-28 X 2 1/4 G8 ROHS	(STEEL) USE W/ ANTI TIPS or TRANSIT
2C	109265	HHCS 1/4-28 X 2 1/2 G8 ROHS	(STEEL) USE W/ ANTI TIPS & TRANSIT &/OR ARMREST RECEIVER
2D	330100	1/4-28 x 2 HEX C/S Ti	(TITANIUM) USE W/ AXLE PLATES ONLY
3A	107015	HHCS 1/4-28 X 1 3/4 G8 ROHS	(STEEL)
3B	330099	1/4-28 x 1 3/4 HEX C/S TI	(TITANIUM)
4A	106973	HHCS 1/4-28 X 2 ROHS	(STEEL)
4B	330100	1/4-28 x 2 HEX C/S Ti	(TITANIUM)
5	106943	WASHER FLAT AN 1/4 ROHS	
6	502000	AXLE PLATE Q2/HP/ZP2 INDEX	INDEXED
7	106944	WASHER FLAT THIN AN 1/4 ROHS	
8A	106934	NUT NYLOCK 1/4-28 ROHS	(STEEL)
8B	330000-QU	1/4-28 NYLOCK NUT ALUMINUM	(ALUMINUM)
9	481065	AXLE SLEEVE SS	
10	505015	INDEX AXLE WASHER	
11	116569	NUT HEX 3/4-16 BLACK ZINC	
12	116570	WASHER FLAT 3/4 X 1.25 BLK ZNC	
1A,1B,1C,2A,2B,2C,3A,4A,5,7,8A	111237	AXLE PLATE HARDWARE KIT Q2	Q2 & Q2HP (HARDWARE FOR BOTH SIDES & ALL CONFIGS)
1D,2D,3B,4B,5,7,8B	920216	AXLE PLATE HARDWARE KIT HP/T2	Q2HP TITANIUM ONLY (HARDWARE FOR ONE SIDE)
1-12	111239	STD AXL PLATE KIT	PAIR
9-12	920120	AXLE SLEEVE W/ HDWR INDEX	

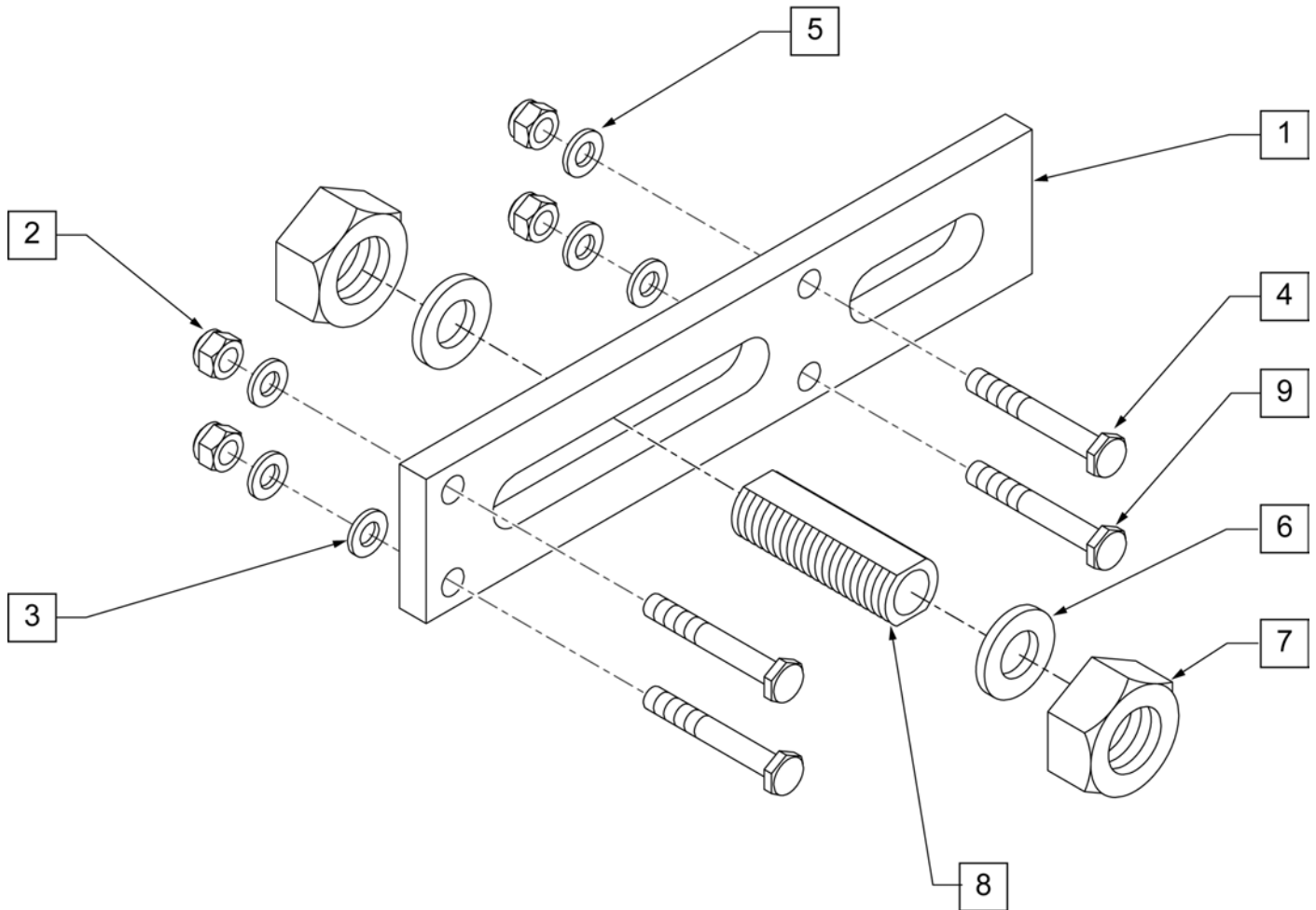


Pos.	Item Number	Description	Remarks
1	502002	AXLE PLATE AMPUTEE Q2	NON-INDEXED
2	106934	NUT NYLOCK 1/4 -28 ROHS	
3	106943	WASHER FLAT AN 1/4 ROHS	
4	107015	HHCS 1/4 -28 x 1 3/4 ROHS	
5	106944	WASHER FLAT THIN AN 1/4 ROHS	
6	116570	WASHER FLAT 3/4 X 1.25 BLK ZNC	
7	116569	NUT HEX 3/4-16 BLACK ZINC	
8	481065	AXLE SLEEVE S.S.	
9	106973	HHCS 1/4 -28 x 2 ROHS	
1-9	111238	AMPUTEE AXLE PLATE KIT	PAIR
2-5,9	111237	Q2 AXLE PLATE HARDWARE KIT	PAIR
6-9	920121	AXLE SLEEVE W/ HDWR (NON-INDEX)	

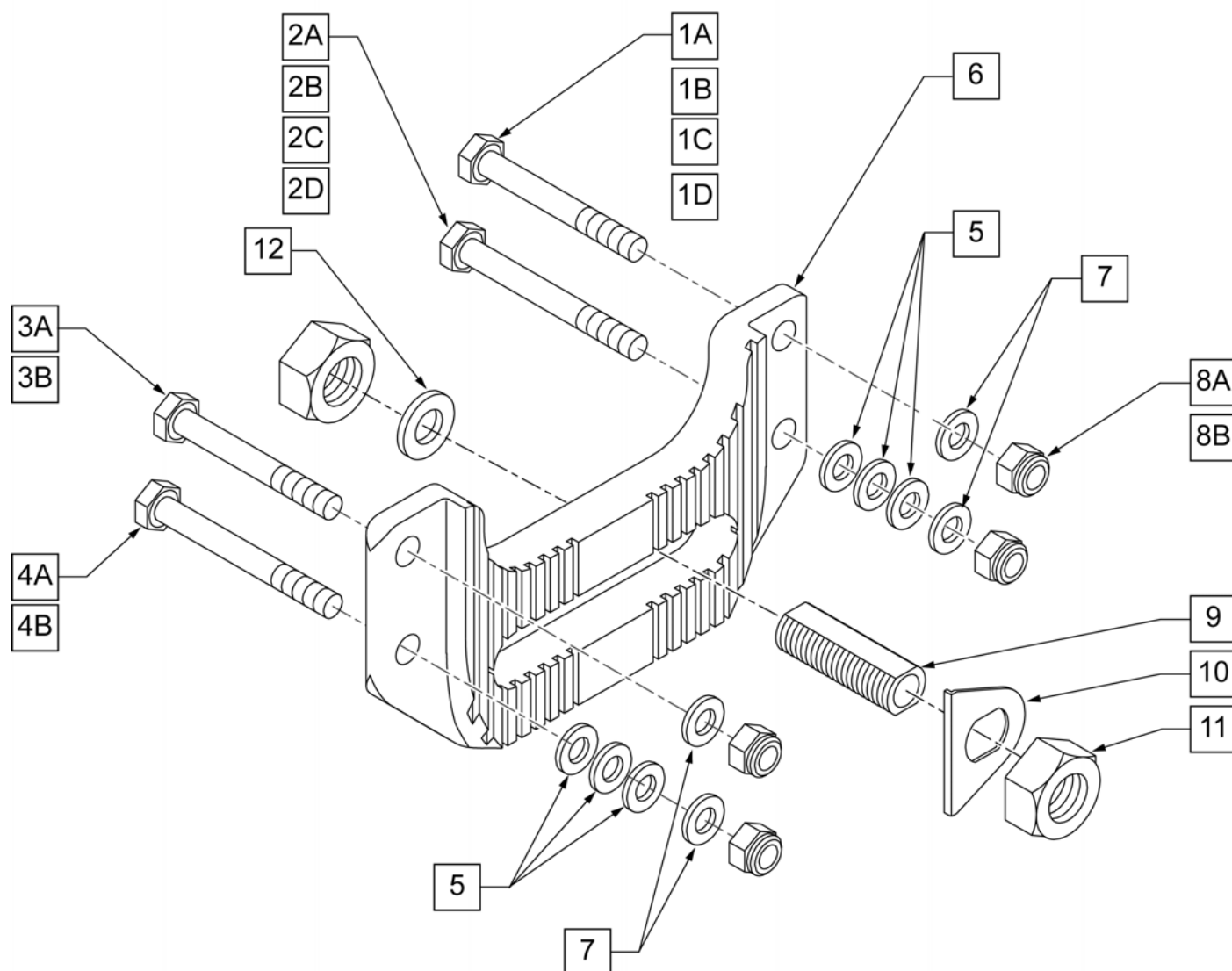


Pos.	Item Number	Description	Remarks
1A	107015	HHCS 1/4-28 X 1 3/4 G8 ROHS	(STEEL) USE W/ AXLE PLATES ONLY
1B	106973	HHCS 1/4-28 X 2 ROHS	(STEEL) USE W/ ANTI TIPS or TRANSIT
1C	107351	HHCS 1/4-28 X 2 1/4 G8 ROHS	(STEEL) USE W/ ANTI TIPS & TRANSIT &/OR ARMREST RECEIVER

Pos.	Item Number	Description	Remarks
1D	330099	1/4-28 x 1 3/4 HEX C/S TI	(TITANIUM) USE W/ AXLE PLATES ONLY
2A	106973	HHCS 1/4-28 X 2 ROHS	(STEEL) USE W/ AXLE PLATES ONLY
2B	107351	HHCS 1/4-28 X 2 1/4 G8 ROHS	(STEEL) USE W/ ANTI TIPS or TRANSIT
2C	109265	HHCS 1/4-28 X 2 1/2 G8 ROHS	(STEEL) USE W/ ANTI TIPS & TRANSIT &/OR ARMREST RECEIVER
2D	330100	1/4-28 x 2 HEX C/S Ti	(TITANIUM) USE W/ AXLE PLATES ONLY
3A	107015	HHCS 1/4-28 X 1 3/4 G8 ROHS	(STEEL)
3B	330099	1/4-28 x 1 3/4 HEX C/S TI	(TITANIUM)
4A	106973	HHCS 1/4-28 X 2 ROHS	(STEEL)
4B	330100	1/4-28 x 2 HEX C/S Ti	(TITANIUM)
5	106943	WASHER FLAT AN 1/4 ROHS	
6	502018	CURVED PLATE Q2/HP/ZP2 INDEX	INDEXED
7	106944	WASHER FLAT THIN AN 1/4 ROHS	
8A	106934	NUT NYLOCK 1/4-28 ROHS	(STEEL)
8B	330000-QU	1/4-28 NYLOCK NUT ALUMINUM	(ALUMINUM)
9	481065	AXLE SLEEVE SS	
10	505015	INDEX AXLE WASHER	
11	116569	NUT HEX 3/4-16 BLACK ZINC	
12	116570	WASHER FLAT 3/4 X 1.25 BLK ZNC	
1A,1B,1C,2A,2B,2C,3A,4A,5,7,8A	111237	AXLE PLATE HARDWARE KIT Q2	Q2 & Q2HP (HARDWARE FOR BOTH SIDES & ALL CONFIGS)
1D,2D,3B,4B,5,7,8B	920216	AXLE PLATE HARDWARE KIT HP/T2	Q2HP TITANIUM ONLY (HARDWARE FOR ONE SIDE)
1-12	111240	CURVED AXL PLATE KIT	PAIR
9-12	920120	AXLE SLEEVE W/ HDWR INDEX	

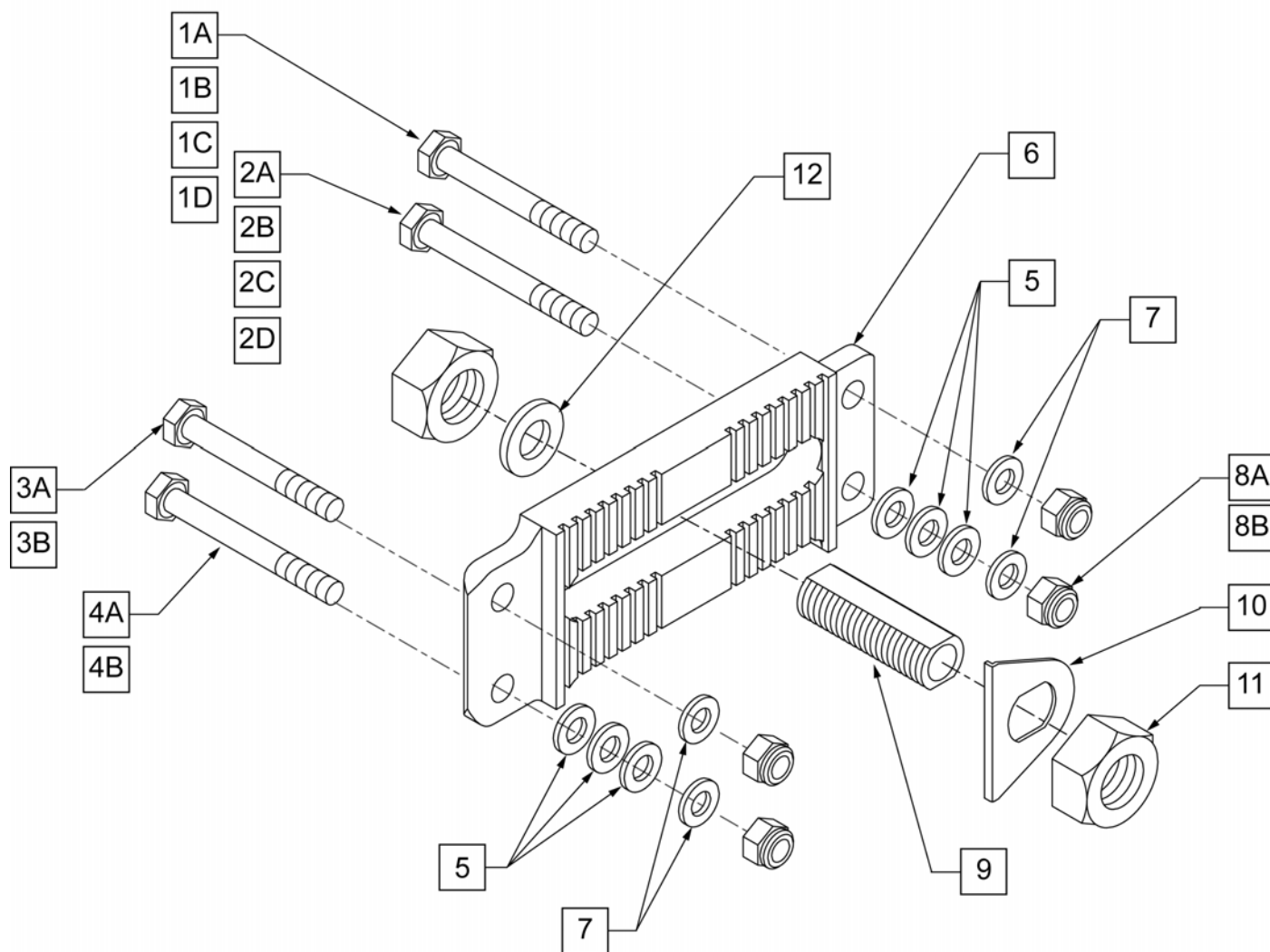


Pos.	Item Number	Description	Remarks
1A	107015	HHCS 1/4-28 X 1 3/4 G8 ROHS	(STEEL) USE W/ AXLE PLATES ONLY
1B	106973	HHCS 1/4-28 X 2 ROHS	(STEEL) USE W/ ANTI TIPS or TRANSIT
1C	107351	HHCS 1/4-28 X 2 1/4 G8 ROHS	(STEEL) USE W/ ANTI TIPS & TRANSIT &/OR ARMREST RECEIVER


Pos.	Item Number	Description	Remarks
1D	330099	1/4-28 x 1 3/4 HEX C/S TI	(TITANIUM) USE W/ AXLE PLATES ONLY
2A	106973	HHCS 1/4-28 X 2 ROHS	(STEEL) USE W/ AXLE PLATES ONLY
2B	107351	HHCS 1/4-28 X 2 1/4 G8 ROHS	(STEEL) USE W/ ANTI TIPS or TRANSIT
2C	109265	HHCS 1/4-28 X 2 1/2 G8 ROHS	(STEEL) USE W/ ANTI TIPS & TRANSIT &/OR ARMREST RECEIVER
2D	330100	1/4-28 x 2 HEX C/S Ti	(TITANIUM) USE W/ AXLE PLATES ONLY
3A	107015	HHCS 1/4-28 X 1 3/4 G8 ROHS	(STEEL)
3B	330099	1/4-28 x 1 3/4 HEX C/S TI	(TITANIUM)
4A	106973	HHCS 1/4-28 X 2 ROHS	(STEEL)
4B	330100	1/4-28 x 2 HEX C/S Ti	(TITANIUM)
5	106943	WASHER FLAT AN 1/4 ROHS	
6	550645	OFFSET AXLE PLATE	INDEXED
7	106944	WASHER FLAT THIN AN 1/4 ROHS	
8A	106934	NUT NYLOCK 1/4-28 ROHS	(STEEL)
8B	330000-QU	1/4-28 NYLOCK NUT ALUMINUM	(ALUMINUM)
9	481065	AXLE SLEEVE SS	
10	505015	INDEX AXLE WASHER	
11	116569	NUT HEX 3/4-16 BLACK ZINC	
12	116570	WASHER FLAT 3/4 X 1.25 BLK ZNC	
1A,1B,1C,2A,2B,2C,3A,4A,5,7,8A	111237	AXLE PLATE HARDWARE KIT Q2	Q2 & Q2HP (HARDWARE FOR BOTH SIDES & ALL CONFIGS)
1D,2D,3B,4B,5,7,8B	920216	AXLE PLATE HARDWARE KIT HP/T2	Q2HP TITANIUM ONLY (HARDWARE FOR ONE SIDE)
1-12	111241	OFFSET AXL PLATE KIT	PAIR
9-12	920120	AXLE SLEEVE W/ HDWR INDEX	

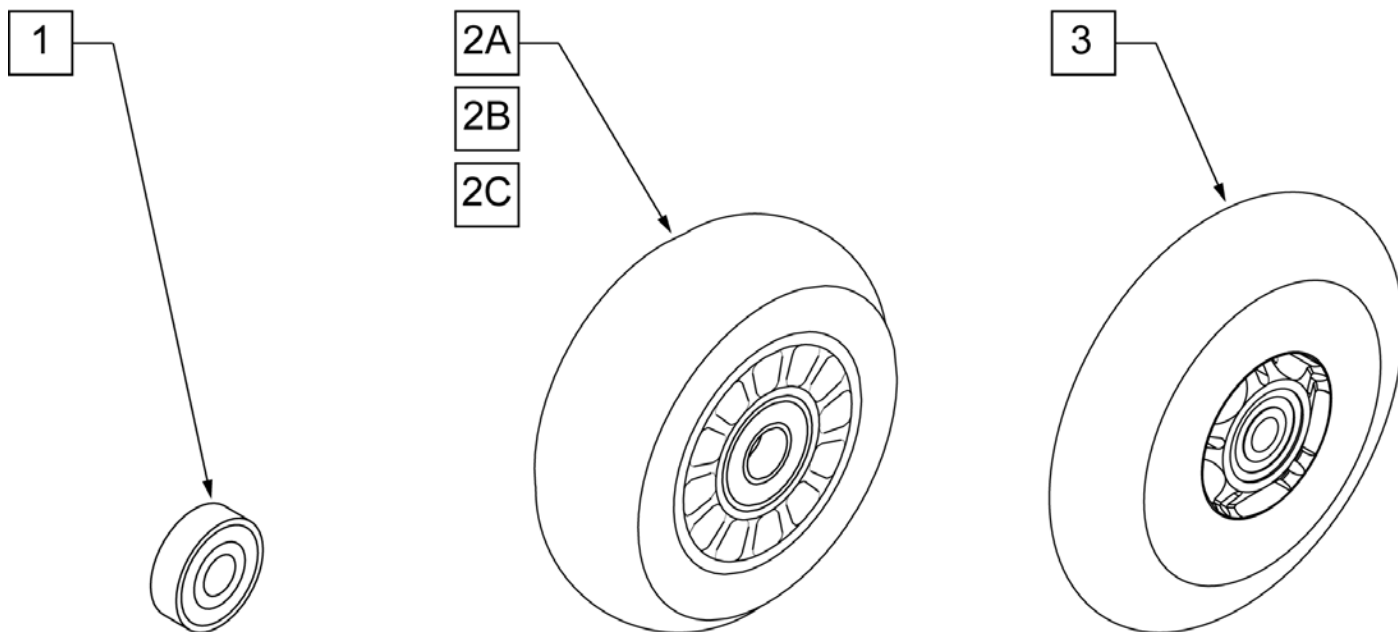

Pos.	Item Number	Description	HCPC	Remarks
1	349502	7/8"x 5/16" BEARING		
2A	400108	4" MICRO CASTER LIGHTED	K0077	4" MICRO LIGHTED
2B	920025	3" MICRO CASTER W/ BEARING -PAIR	K0077	PAIR 3" MICRO
2C	400107	3" MICRO CASTER LIGHTED	K0077	3" MICRO LIGHTED
3	052129	5" MICRO CASTER LIGHTED	K0077	5" MICRO LIGHTED

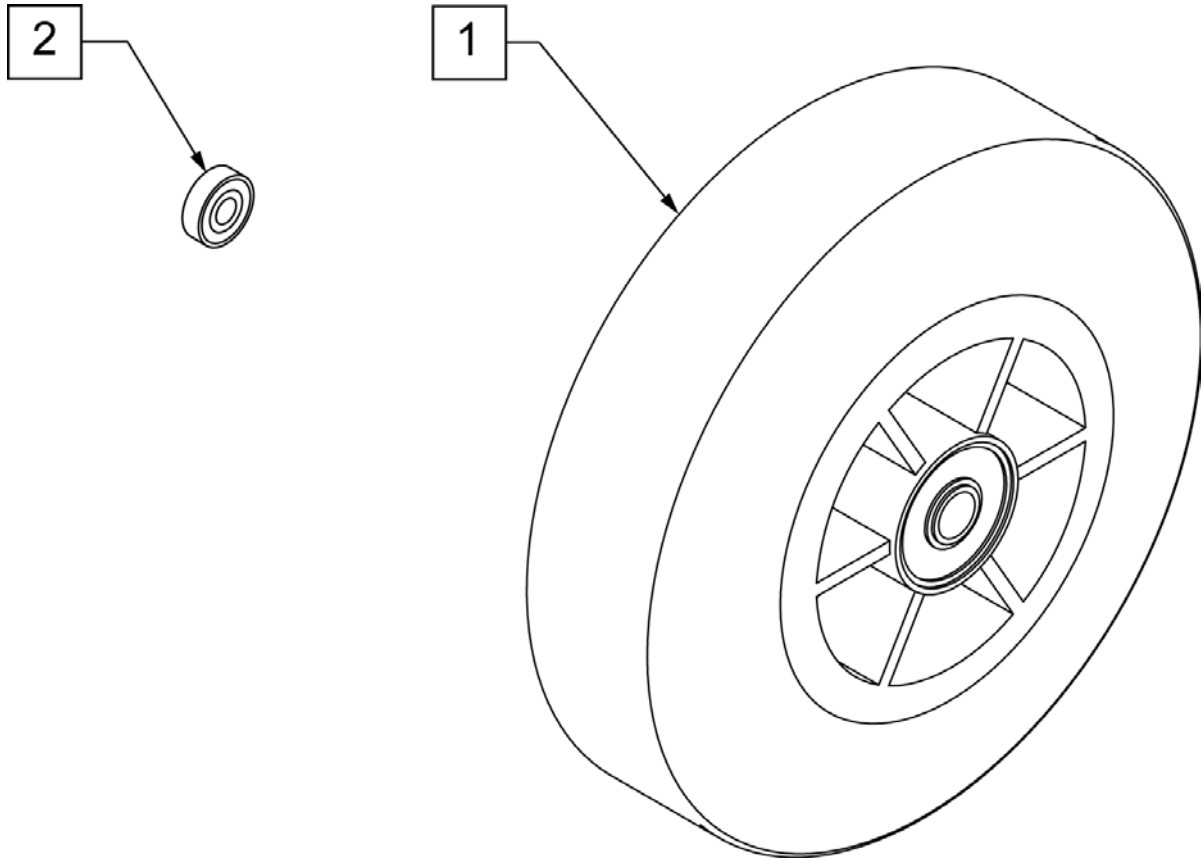


Pos.	Item Number	Description	HCPC	Remarks
1	400103	CASTER 5" SOFT ROLL W/BRNGS	K0077	5" SOFT ROLL SEMI PNEU (GREY)
2	349502	7/8"x 5/16" BEARING		

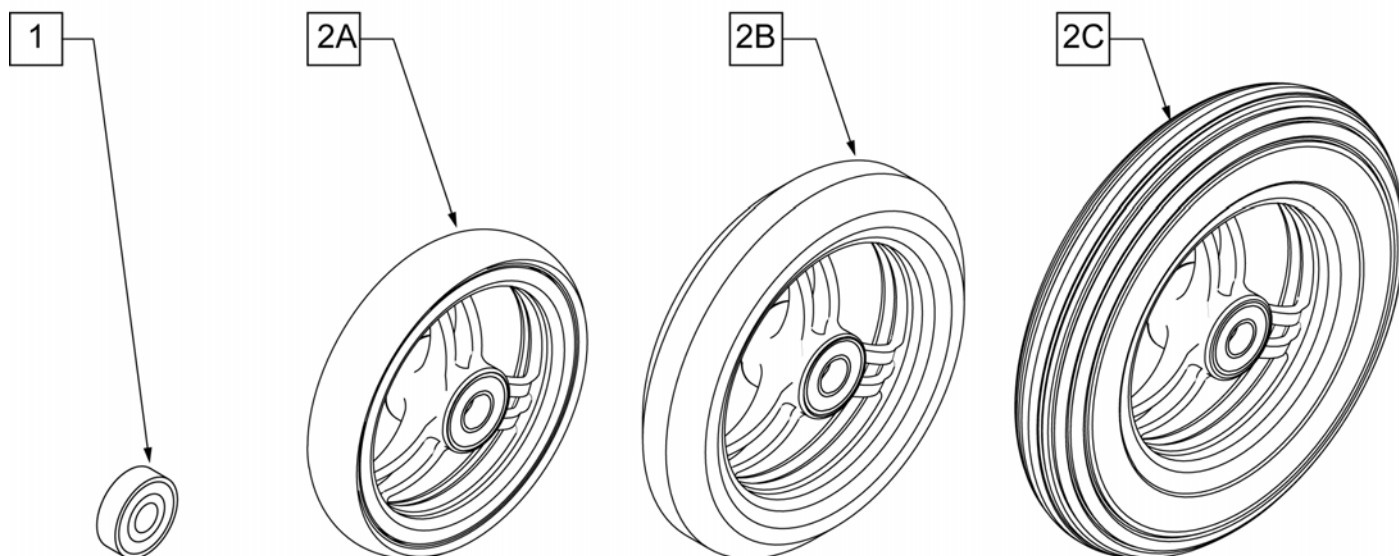

Pos.	Item Number	Description	HCPC	Remarks
1	349502	7/8"x 5/16" BEARING		
2A	101946-001	CASTR 4X1 ALUM/POLY W/BRG CLR	K0077	4" x 1" ALUMINUM POLY
2B	101946-002	CASTR 5X1 ALUM/POLY W/BRG CLR	K0077	5" x 1" ALUMINUM POLY
2C	101946-003	CASTR 6X1 ALUM/POLY W/BRG CLR	K0077	6" x 1" ALUMINUM POLY




SIDE VIEW

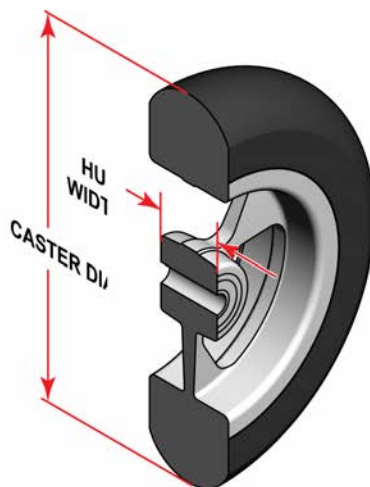


CASTER MEASURING GUIDE

TIRE DIAMETER	HUB WIDTH
3	1
4 1/8	1
5	1
6	1

Pos.	Item Number	Description	HCPC	Remarks
>>>>		3 INCH DIAMETER CASTERS WITH SILVER HUBS		
1	113394	WHEEL CASTER 3" ULTRALITE SFTRLL	K0077	3" x 1.5" Aluminum Soft Roll - Black tire - Silver hub
>>>>		4 INCH DIAMETER CASTERS WITH SILVER HUBS		
1	113395	WHEEL CASTER 4" ULTRALITE SFTRLL	K0077	4" x 1.5" Aluminum Soft Roll - Black tire - Silver hub
>>>>		5 INCH DIAMETER CASTERS WITH SILVER HUBS		
1	113396	WHEEL CASTER 5" ULTRALITE SFTRLL	K0077	5" x 1.5" Aluminum Soft Roll - Black tire - Silver hub
>>>>		6 INCH DIAMETER CASTERS WITH SILVER HUBS		
1	113397	WHEEL CASTER 6" ULTRALITE SFTRLL	K0077	6" x 1.5" Aluminum Soft Roll - Black tire - Silver hub

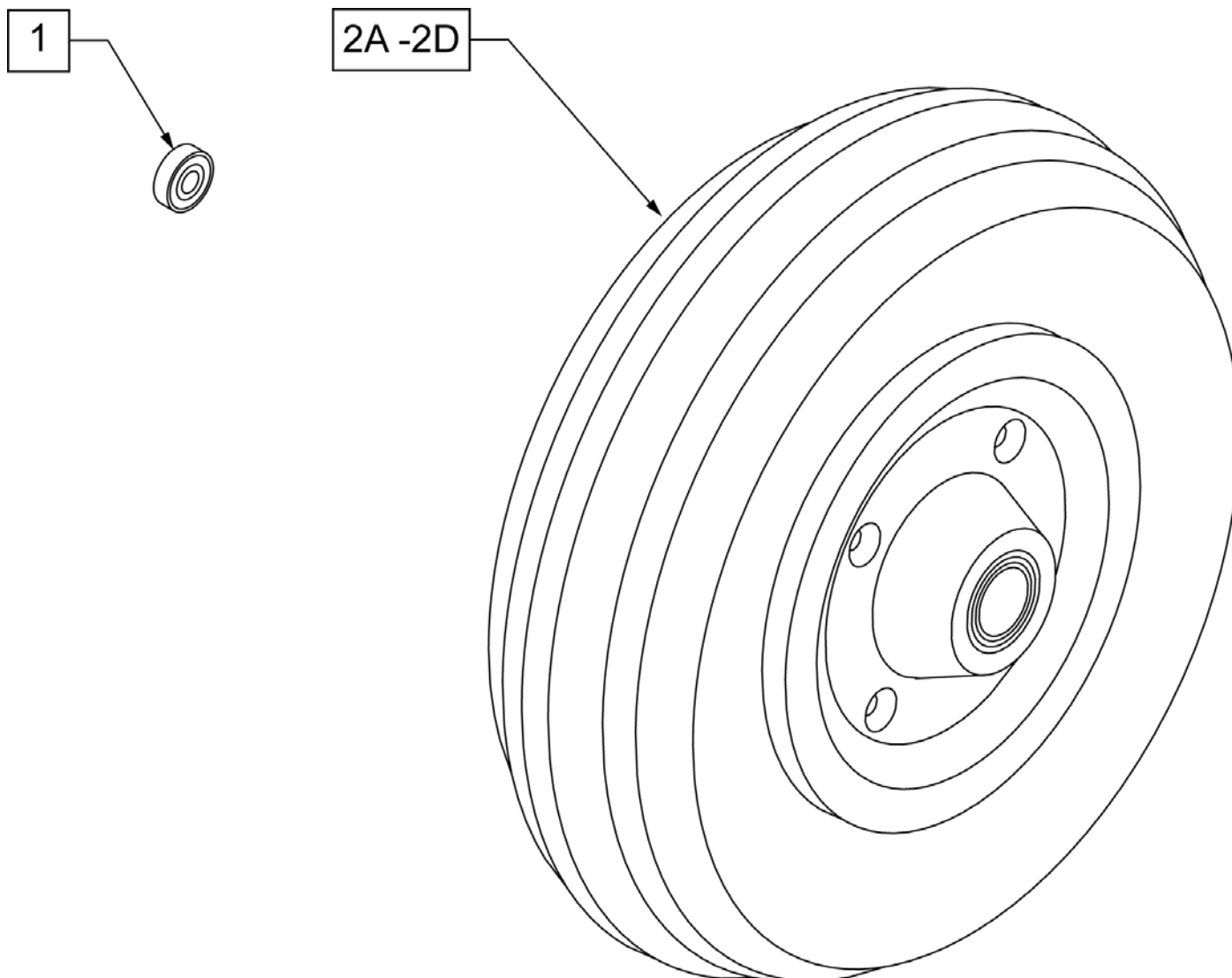

Pos.	Item Number	Description	Remarks
1	349502	7/8"x 5/16" BEARING	
2A	400700	8" X 2" CMPST W/PNEU & BEARING	8" x 2" PNEUMATIC
2B	400702	8" X 2" CMPST W/PNEU INSRT&BRG	8" x 2" PNEUMATIC W/ INSERT
2C	384700	TIRE PNEUMATIC 8" x 2"	
2D	384730	INNER TUBE CASTER 8" x 2"	

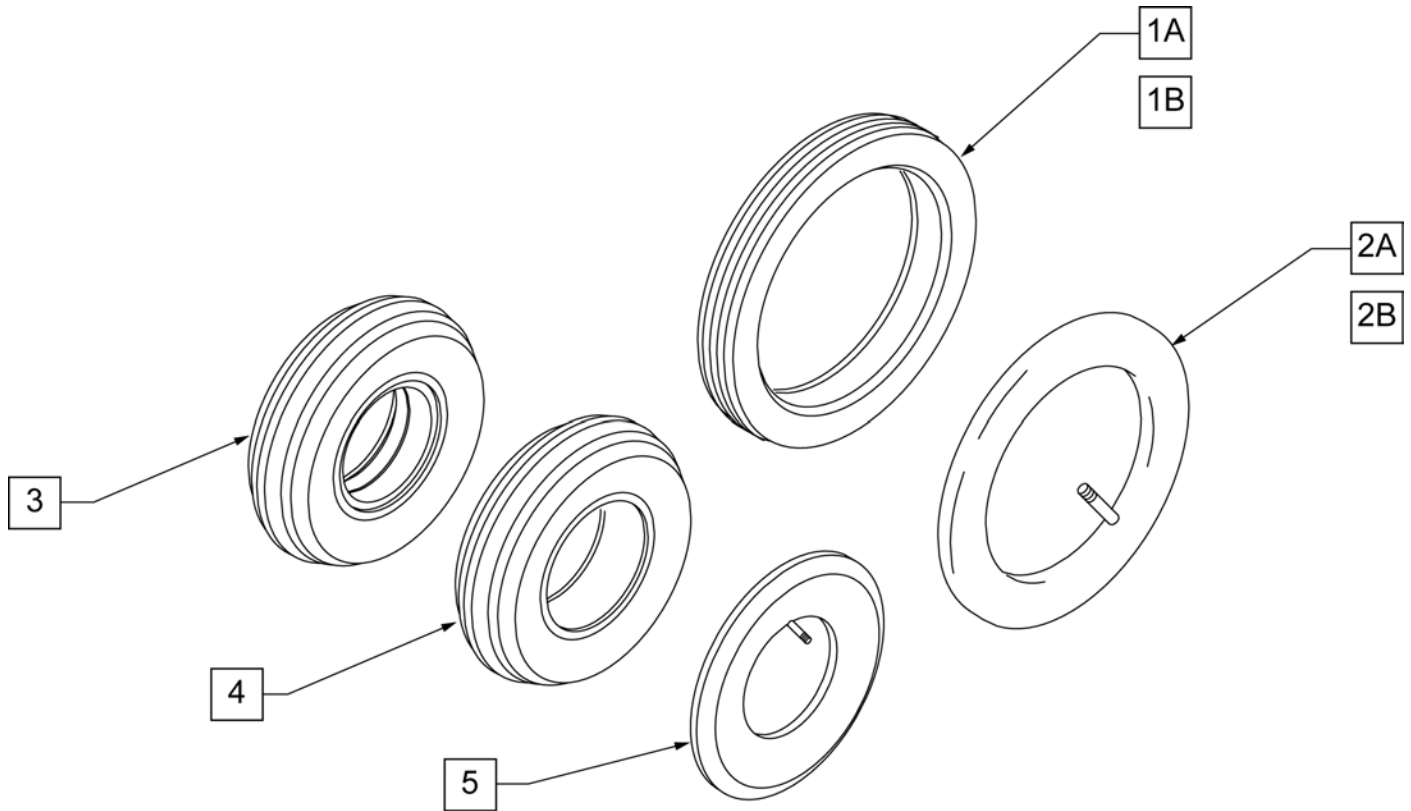


Pos.	Item Number	Description
1A	383250	8" PNEUMATIC TIRE
1B	383240	6" PNEUMATIC TIRE
2A	382250	8" TUBE
2B	382245	6" TUBE
3	384701	TIRE FOAMED INSERT 8" x 2"
4	384700	TIRE PNEUMATIC 8" x 2"
5	384730	INNER TUBE CASTER 8" x 2"
1A,3	383255	8" PNEUMATIC TIRE AND TUBE
1B		Part 1B
1A		Part 1A
2B		Part 2B
2A		Part 2A
3		Part 3
4		Part 4
5		Part 5


NOTE: 7" and 8" Casters are not available on the Q2 or the GT.

Note: If using 6" casters on Q7, Anti Flutter should be installed. See Standard Multi Position Fork page.

Note: Casters wider than 1" will not fit on Quickie Ti Titanium forks.

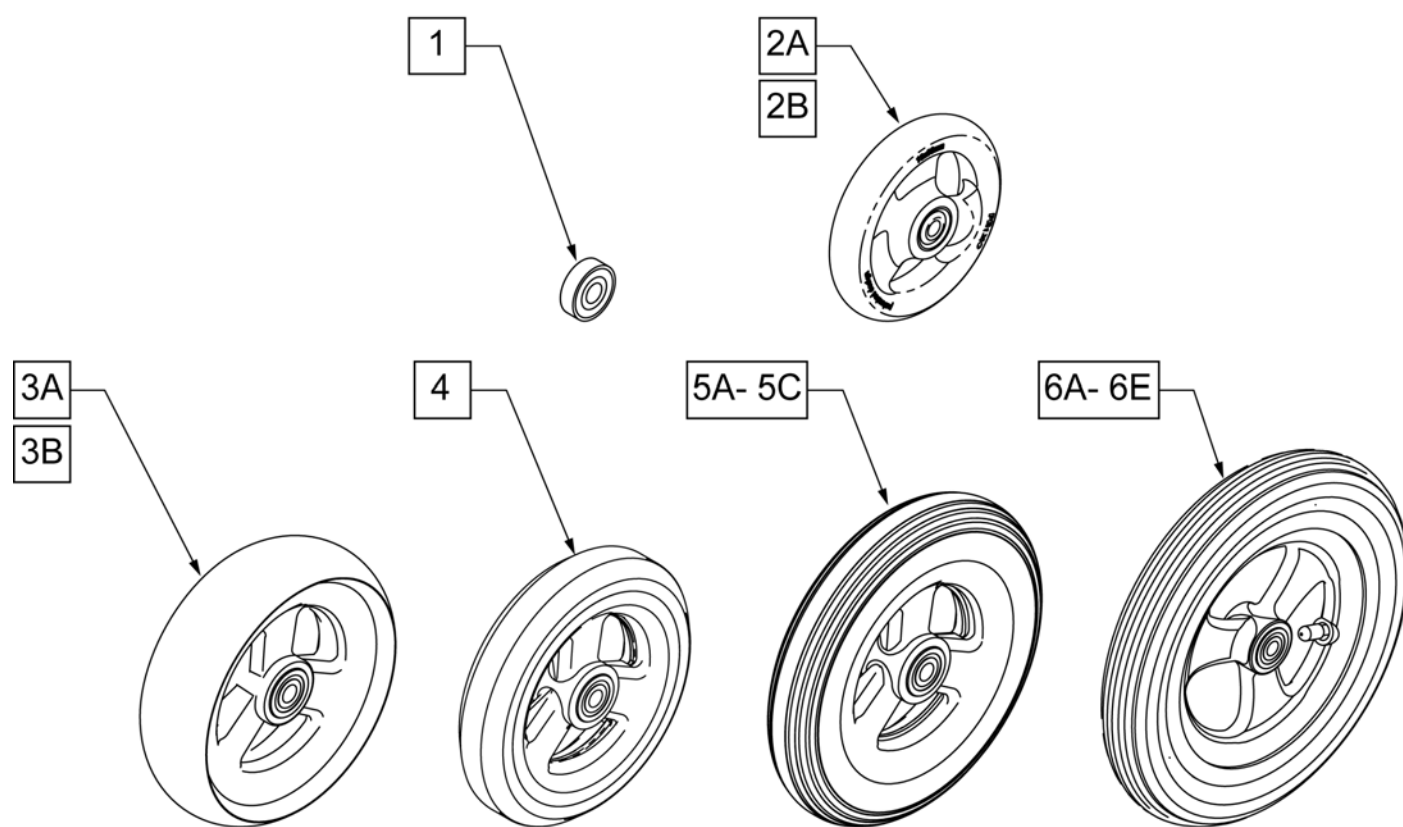

Pos.	Item Number	Description	Remarks
1	349502	7/8"x 5/16" BEARING	
2A	101957	4"x1-1/4 POLY 3 SPOKE CSTR	4" X 1.25 SEMI-PNEUMATIC
2B	101098	CASTER 4" PLASTIC/POLY W/BRG	4" LOW PROFILE POLY
3A	101259	5"x1-1/2" POLY 3 SPOKE CSTR	5" x 1.5" SEMI-PNEUMATIC
3B	101255	6"x1-1/2" POLY 3 SPOKE CSTR	6" x 1.5" SEMI-PNEUMATIC
4	101254	5" LOW POLY 3 SPOKE CASTER	5" LOW PROFILE POLY
5A	101256	6" SEMI PNEU 3 SPOKE CASTER	6" POLY
5B	101257	6" PNEUMATIC 3 SPOKE CASTER	6" PNEUMATIC
5C	383245	6" PNEUMATIC TIRE & TUBE	
6A	400212	CASTER 8" PNEU 3 SPK ASSM	8" PNEUMATIC
6B	400213	CASTER 8" POLY 3 SPK ASSM	8" POLY
6C	400214	CASTER 8" LOW POLY 3 SPK ASSM	7" x LOW PROFILE POLY
6D	383255	8" PNEUMATIC TIRE & TUBE	
6E	400215	CASTER 8" WIDE POLY 3 SPK ASSM	8" x 1.5" LOW POLY

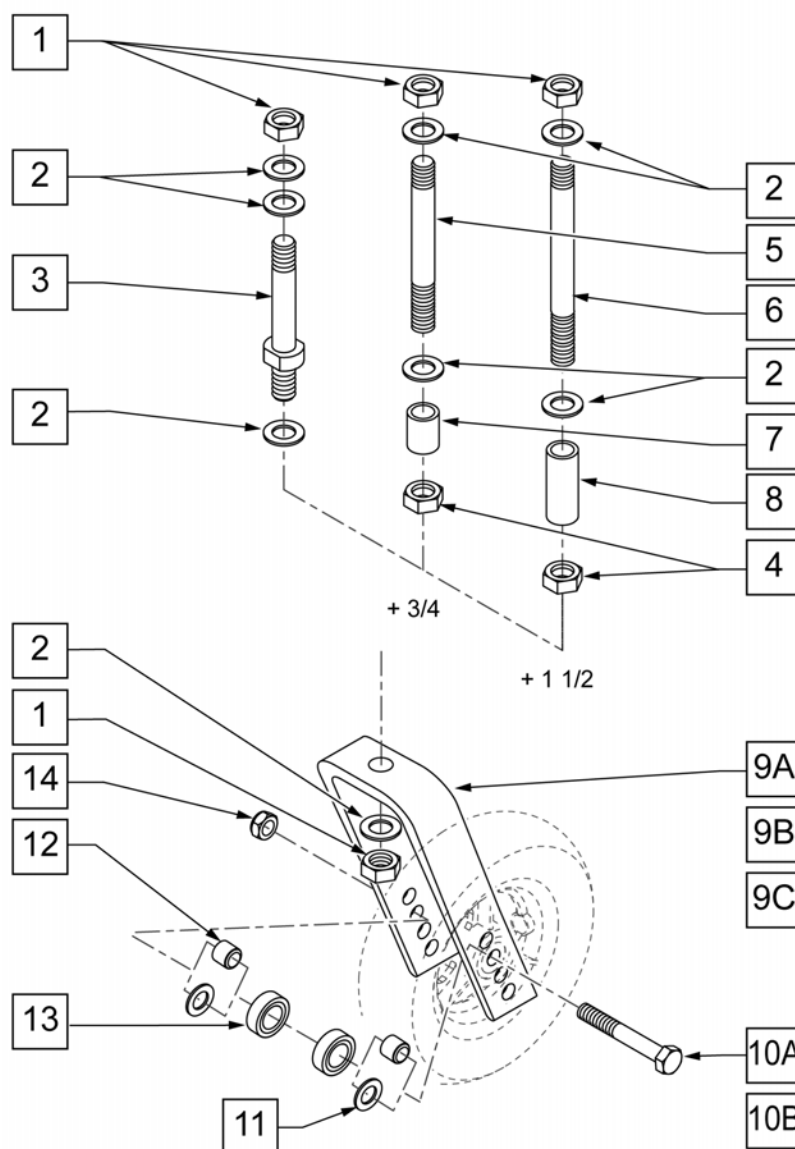


MEASURING GUIDE





	PART NUMBER	"A"
STD	920050	3 5/8"
+ 3/4	920100	4 3/8"
+1 1/2	920110	5 1/8"

NOTE: CASTER STEM DIAMETER IS 1/2".

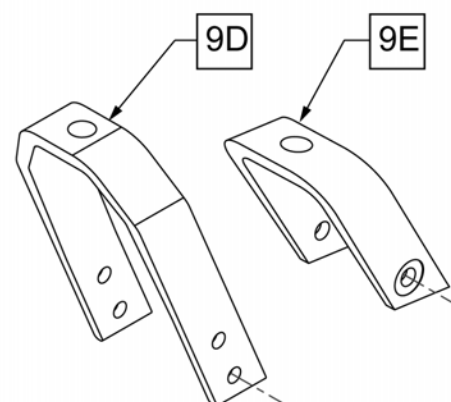


Pos.	Item Number	Description	Remarks
1	106940	NUT NYLON THIN 1/2 -20 ROHS	
2	106949	WASHER FLAT AN 1/2 ROHS	
4	203110	1/2 - 20 JAM NUT	
1-3	920050	REG CASTER STEM W/ HARDWARE	STD STEM
7	521560	+ 3/4 CASTER STEM SPACER BLK	+3/4" STEM
8	521562	+ 1 1/2 CASTER STEM SPACER BLK	+1 1/2" STEM
9A	105839-001	FORK CASTER 5 1/4"	WORKS WITH 4", 5", 6", & 8" CASTERS
9B	105839-002	FORK CASTER 6"	WORKS WITH 4", 5", 6", & 8" CASTERS
9C	105839-003	FORK CASTER 7"	WORKS WITH 4", 5", 6", & 8" CASTERS
9D	550730	8" FORK P100 ONLY	WORKS WITH 8" x 2" CASTER ONLY
9E	106810	FORK CASTER 3"	WORKS WITH 3" CASTERS ONLY
10A	106987	HHCS 5/16 -24 x 2 1/2 ROHS	
10B	106988	HHCS 5/16 -24 x 3 1/4 ROHS	Used w/ 8x2 fork only.
11	106946	WASHER FLAT AN 5/16 ROHS	Used w/ 8x2 fork only.

Pos.	Item Number	Description	Remarks
12	481113	BEARING SPACER	
13	349502	7/8 x 5/16 BEARING	
14	106939	NUT NYLOCK THIN 5/16 -24 ROHS	
1-3	920050	REG CASTER STEM W/ HARDWARE	STD STEM
1, 2, 4, 5, 7	920100	+ 3/4 CASTER STEM W/ HARDWARE BLK	+3/4" STEM
1, 2, 4, 6, 8	920110	+ 1 1/2 CASTER STEM W/ HARDWARE BLK	+1 1/2" STEM
1-3,9E,10A,12	109334	FORK CASTER 3" REG STEM	STD Stem - 3" Fork - Works w/ 3" Casters only
1-3,9A,10A,12	109335	FORK CASTER 5 1/4" REG STEM	STD Stem - 5 1/4" Fork - Works w/ 4", 5", 6", & 8" Casters
1,2,4,5,7,9A,10A,12	109336	FORK CASTER 5 1/4" + 3/4" STEM	+3/4" Stem - 5 1/4" Fork - Works w/ 4", 5", 6", & 8" Casters
1,2,4,6,8,9A,10A	109337	FORK CASTER 5 1/4" + 1 1/2" STEM	+1 1/2" Stem - 5 1/4" Fork - Works w/ 4", 5", 6", & 8" Casters
1-3,9B,10A,12	109338	FORK CASTER 6" REG STEM	STD Stem - 6" Fork - Works w/ 4", 5", 6", & 8" Casters
1,2,4,5,7,9B,10A	109339	FORK CASTER 6" + 3/4" STEM	+ 3/4" Stem - 6" Fork - Works w/ 4", 5", 6", & 8" Casters
1,2,4,6,8,9B,10A	109340	FORK CASTER 6" + 1 1/2" STEM	+1 1/2" Stem - 6" Fork - Works w/ 4", 5", 6", & 8" Casters
1-3,9C,10A,12	109341	FORK CASTER 7" REG STEM	STD Stem - 7" Fork - Works w/ 4", 5", 6", & 8" Casters
1,2,4,5,7,9C,10A,12	109342	FORK CASTER 7" + 3/4" STEM	+3/4" Stem - 7" Fork - Works w/ 4", 5", 6", & 8" Casters
1,2,4,6,8,9C,10A,12	109343	FORK CASTER 7" + 1 1/2" STEM	+1 1/2" Stem - 7" Fork - Works w/ 4", 5", 6", & 8" Casters
1-3,9D,10B,11	109344	FORK CASTER 8" REG STEM	STD Stem - 8" Fork - Works w/ 8" X 2" Casters only
1,2,4,5,7,9D,10B,11	109345	FORK CASTER 8 + 3/4" STEM	+3/4" Stem - 8" Fork - Works w/ 8" X 2" Casters only
1,2,4,6,8,9D,10B,11	109346	FORK CASTER 8 + 1 1/2" STEM	+1 1/2" Stem - 8" Fork - Works w/ 8" X 2" Casters only

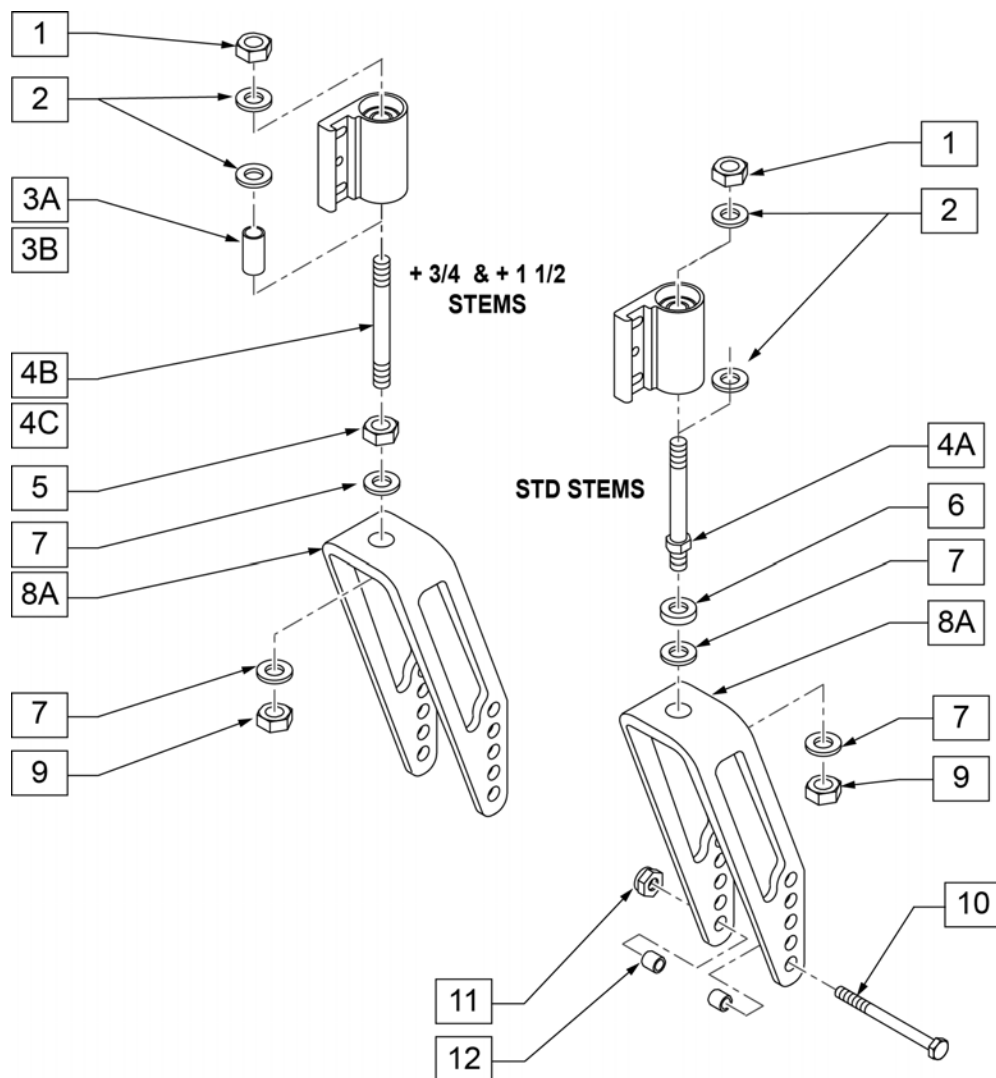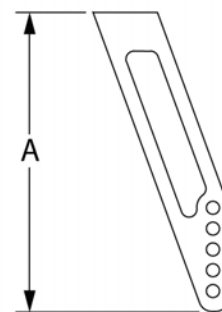**MEASURING GUIDE**



	TALL	SHORT
DIM "A"	7"	5"

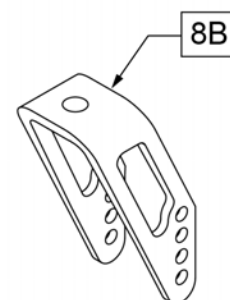


Pos.	Item Number	Description
1	106940	1/2 - 20 NYLOCK NUT
2	106949	1/2 AN WASHER FLAT
3A	521560	+ 3/4" CASTER STEM SPACER BLK
3B	521562	+ 1 1/2" CASTER STEM SPACER BLK
4A	481712	CASTER STEM SPACER P1/P5
4B	481004	CASTER STEM + 3/4"
4C	481006	CASTER STEM + 1-1/2"
5	203110	1/2 - 20 JAM NUT
6	483254	SPACER .50 ID x .88 OD x .125
7	106949	1/2 AN WASHER FLAT
8A	550644	MULTI-POSITION FORK TALL
8B	550646	MULTI-POSITION FORK SHORT
9	106940	1/2 - 20 NYLOCK NUT
10	106987	5/16 - 24 x 2 1/2 HEX C/S
11	106939	5/16 - 24 NYLOCK NUT THIN
12	481113	SPACER 5/16 x .380
4A, 6, 7, 8A, 9	780236	MULTI-POSITION FORK TALL STD STEM
4B, 5, 7, 8A, 9	780237	MULTI-POSITION FORK TALL 3/4 STEM

Pos.	Item Number	Description
4C, 5, 7, 8B, 9	780238	MULTI-POSITION FORK TALL 1 1/2 STEM
4A, 6, 7, 8B, 9	780239	MULTI-POSITION FORK SHORT STD STEM
4B, 5, 7, 8B, 9	780240	MULTI-POSITION FORK SHORT 3/4 STEM
4C, 5, 7, 8B, 9	780241	MULTI-POSITION FORK SHORT 1 1/2 STEM
1, 2, 4A, 6,7,8A-12	780330	MULTI-POSITION FORK TALL STD STEM W/ HDWR
1-3A, 4B, 5, 7,8A-12	780331	MULTI-POSITION FORK TALL 3/4 STEM W/ HDWR
1, 2, 3B, 4C, 5, 7,8A-12	780332	MULTI-POSITION FORK TALL 1 1/2 STEM W/HDWR
1, 2, 4A, 6, 7, 8B, 9-12	780333	MULTI-POSITION FORK SHORT STD STEM W/ HDWR
1-3A, 4B, 5, 7, 8B, 9-12	780334	MULTI-POSITION FORK SHORT 3/4 STEM W/HDWR
1, 2, 3B, 4C, 5, 7, 8B, 9-12	780335	MULTI-POSITION FORK SHORT 1 1/2 STEM W/HDWR


Note: Item 6 is used to aid in aligning the upper and lower fork assembly during elastomer replacement. See instruction sheet for details.

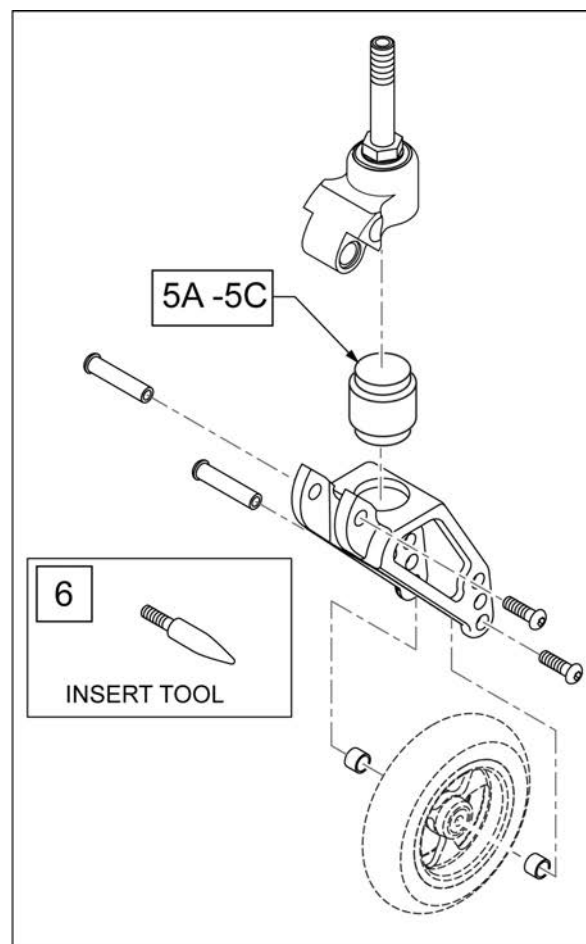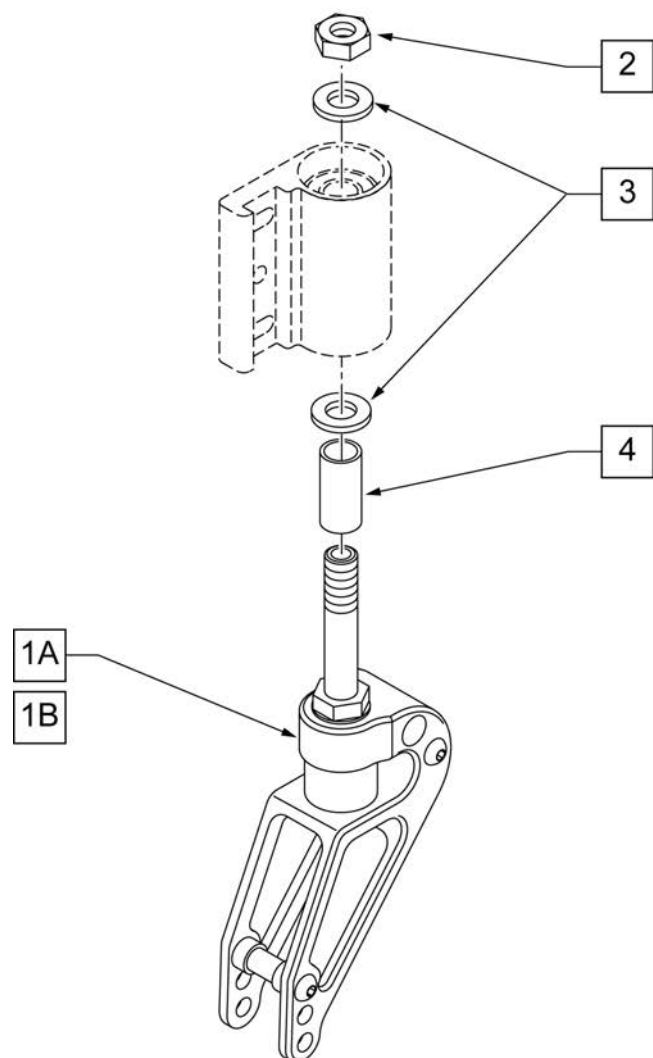

Pos.	Item Number	Description	Remarks
1A	921488	FROG LEG 6 INCH 2.5" STEM	
1B	921489	FROG LEG 6 INCH 4" STEM	
2	106940	1/2-20 NYLOCK NUT	
3	106949	1/2 AN WASHER FLAT	
4	521562	+1 1/2 CASTER STEM SPACER BLK	
5A	050156-001	ELASTOMER FROG LEGS SOFT	0 -100 Lbs
5B	050156-002	ELASTOMER FROG LEGS MEDIUM	101 -190 Lbs
5C	050156-003	ELASTOMER FROG LEGS FIRM	191 -265 Lbs
1A,2,3	921498	FROG LEG 6" W/ 2.5"STEM W/ HDWR	
1B,2-4	921499	FROG LEG 6" W/ 4"STEM W/ HDWR	
6	116351	FROG LEG FORK ASSEMBLY TOOL	

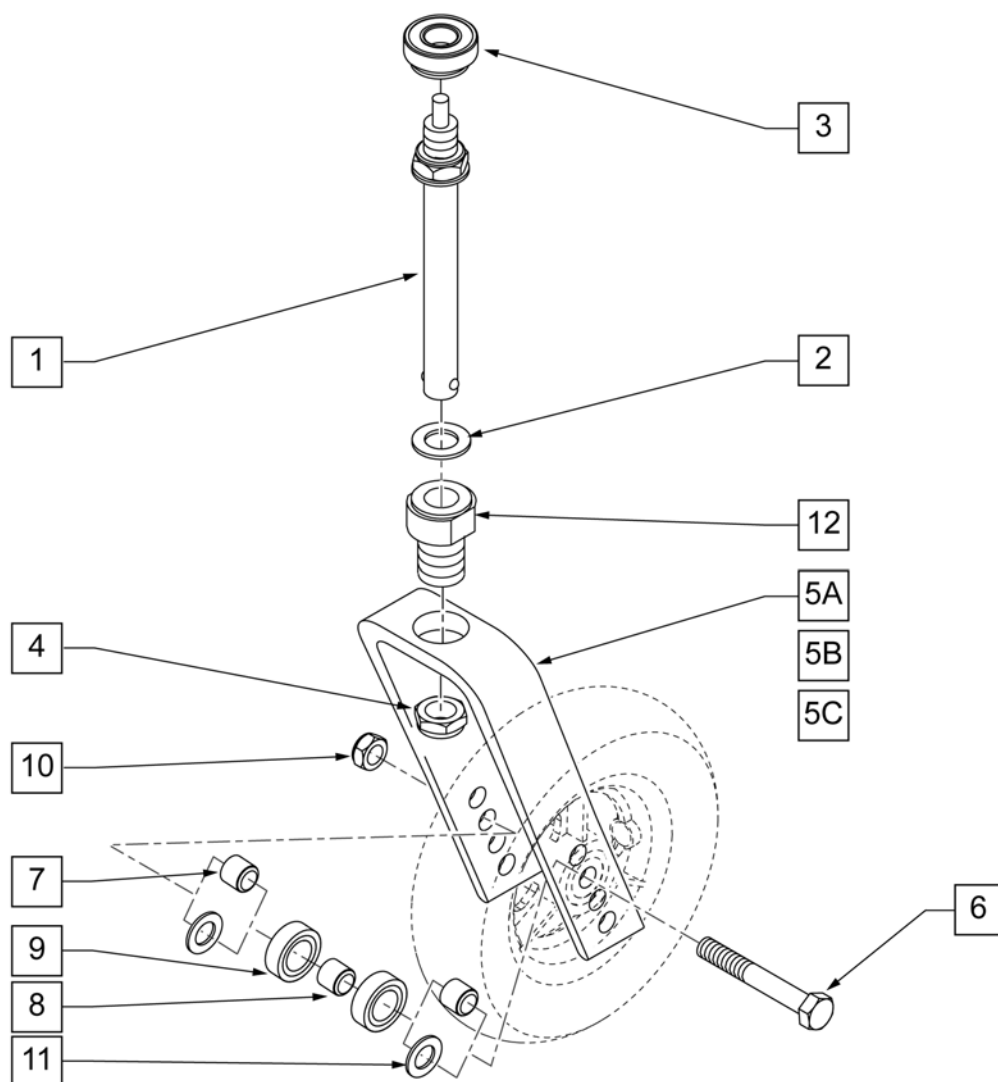


Pos.	Item Number	Description
1,2,4	920123	CASTER STEM W/ HARDWARE QR
2	106949	WASHER FLAT AN 1/2 ROHS
3	349500	1 1/8 x 1/2 BEARING
4	204000	3/4 -16 NUT
5A	110961	FORK CASTER 5 1/4" QR
5B	110964	FORK CASTER 6" QR
5C	110965	FORK CASTER 7" QR
6	106987	HHCS 5/16 -24 x 2 1/2 ROHS
7	481113	SPACER 5/16 x 0.380
8	481013	BEARING SPACER
9	349502	7/8 x 5/16 BEARING
10	106939	NUT NYLOCK THIN 5/16 -24 ROHS
11	106946	WASHER FLAT AN 5/16 ROHS
12	481045	QUICK RELEASE CASTER ADAPTER
1-4,5A,6,7,10,12	116084	FORK CASTER 5 1/4" QR ASSM
1-4,5B,6,7,10,12	116085	FORK CASTER 6" QR ASSM
1-4,5C,6,7,10,12	116086	FORK CASTER 7" QR ASSM

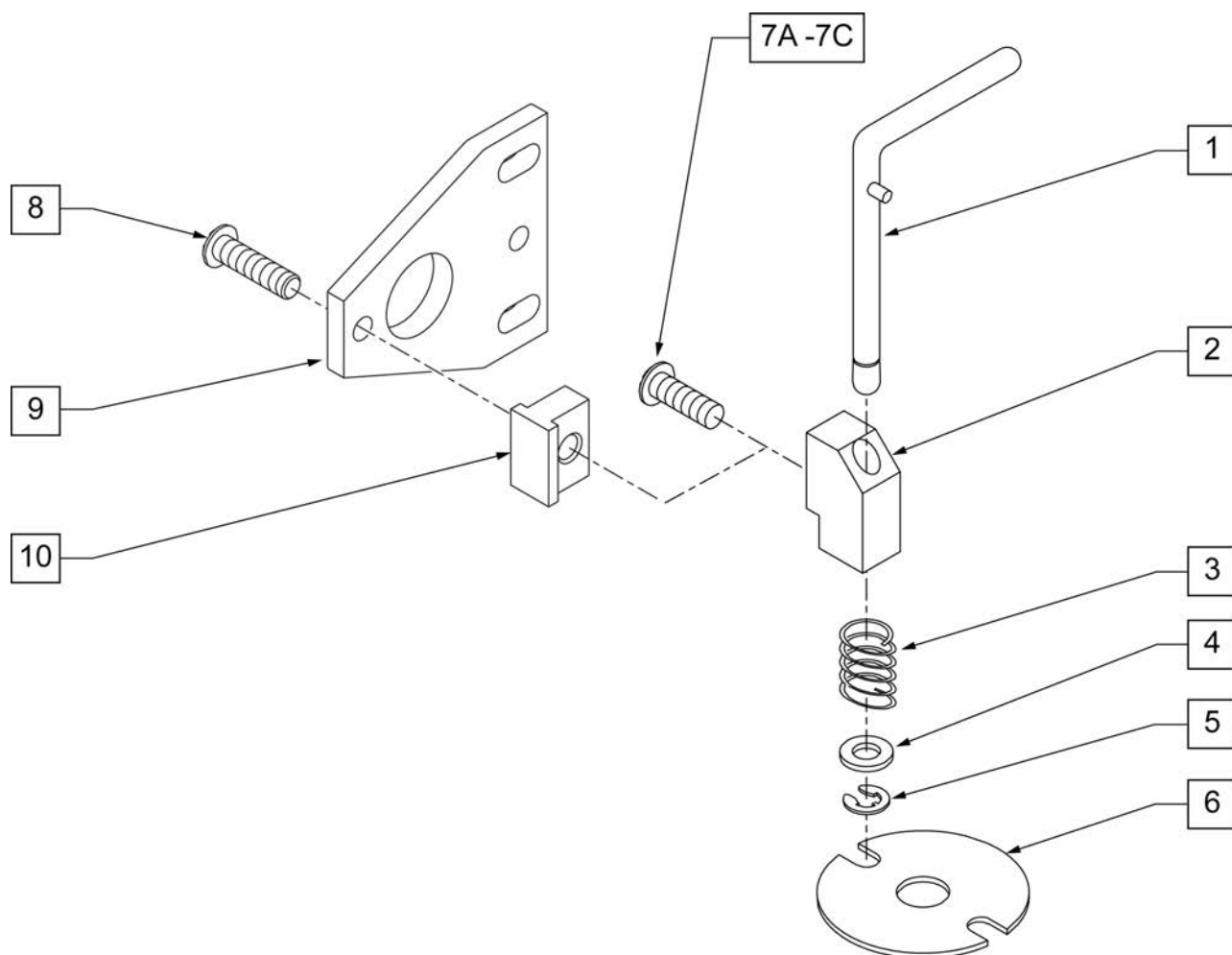

Pos.	Item Number	Description	Remarks
1	880761	PIN LOCK ARM W/ PIN S/A	RIGHT
1	880762	PIN LOCK ARM W/ PIN S/A	LEFT
2	500262	CASTER PIN LOCK BLOCK S/A BLK	
3	321141	.650L x 6 COIL SPRING	
4	106943	WASHER FLAT AN 1/4 ROHS	
5	342000	1/4" E-CLIP	
6	505012	CASTER PIN LOCK PLATE	
7A	106959	BHCS 1/4 -28 x 5/8 ROHS	USED W/ S/A HANGER FRT FRAME ONLY
7B	107015	HHCS 1/4 -28 x 1 3/4 ROHS	USED W/ TRANSIT OPTION
7C	106973	HHCS 1/4 -28 x 2 ROHS	USED FOR GP FIXED FRT FRAME ONLY
8	106959	1/4 -28 X 5/8 BH C/S	
9	505667	PIN LOCK PLATE HI-ANGLE CASTER	
10	505666	PIN LOCK BLOCK HI-ANGLE CASTER	
1-5	880750	CASTER PIN LOCK S/A	RIGHT
1-5	880751	CASTER PIN LOCK S/A	LEFT
1-6,7A	900806	CASTER PIN LOCK ASSM S/A OFF CHAIR	USED W/ S/A HANGER FRT FRAME ONLY
1-6,7B	900804	CASTER PIN LOCK KIT TRIANGULAR CASTER NON-S/A OFF CHAIR	USED W/ TRANSIT OPTION
1-6,7C	900807	CASTER PIN LOCK ASSM GP/GPV	USED FOR GP FIXED FRT FRAME ONLY

CASTER PLATES


[04/2011]

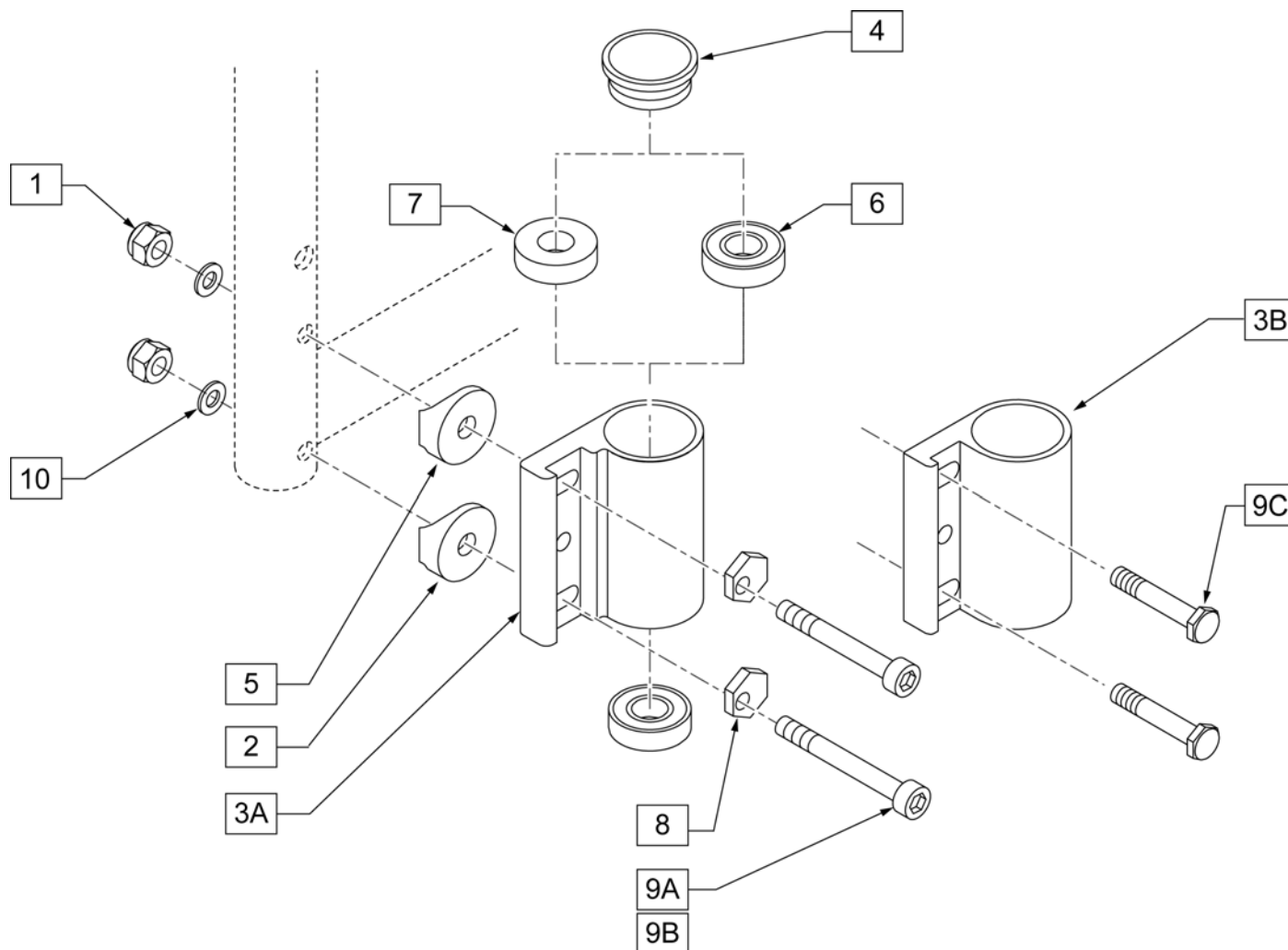

Pos.	Item Number	Description	Remarks
1	106941	NUT AUTOMATION 1/4 -28 ROHS	
2	561086	1" HALF SADDLE 1/4" HOLE	
4	561070	CASTER CAP PLASTIC	
4	920028	CASTER CAP PLASTIC	PAIR
5	561004	1" SADDLE 1/4" HOLE	
6	349500	1 1/8 x 1/2 BEARING	
7	561050	ANTI-FLUTTER BUSHING	USED W/ 8" CASTERS
8	262128	WASHER ECCENTRIC HD 1/4 HOLE	
9A	109809	SHCS 1/4 -28 x 2 ROHS	USED W/ NON-TRANSIT on Q2
9B	108791	SHCS 1/4 -28 x 2 1/4 SHLDR ROHS	USED W/ TRANSIT BRACKETS
9C	481017	1/4 -28 x 1.688 ECCENTRIC BOLT	USED W/ NON-TRANSIT on LX/LXI
10	106943	WASHER FLAT AN 1/4 ROHS	
1, 2, 4, 5, 8,9ABC,10	920474	CASTER PLATE HI ANGLE HARDWARE KIT	
3A, 6	780204	CASTER PLATE HI ANGLE DOUBLE BEARING	RIGHT-Effective on Q2 after 8/1/02;on LX/LXI TRANSIT 10/24/08.
3A, 6	780205	CASTER PLATE HI ANGLE DOUBLE BEARING	LEFT -Effective on Q2 after 8/1/02; on LX/LXI TRANSIT 10/24/08.
3A, 6, 7	780206	CASTER PLATE HI ANG R SNG BEARING	RIGHT-Effective on Q2 after 8/1/02; on LX/LXI TRANSIT 10/24/08.

Pos.	Item Number	Description	Remarks
3A, 6, 7	780207	CASTER PLATE HI ANG L SNG BEARING	LEFT - Effective on Q2 after 8/1/02; on LX/LXI TRANSIT 10/24/08.
3B,6	780210	CASTER PLATE R DBL BEARING BLK	RIGHT-Discontinued on Q2 chair 8/1/02
3B,6	780212	CASTER PLATE L DBL BEARING BLK	LEFT -Discontinued on Q2 chair 8/1/02
3B,6,7	780200	CASTER PLATE R SNGLE BEARING	RIGHT-Discontinued on Q2 chair 8/1/02
3B,6,7	780201	CASTER PLATE L SNGLE BEARING BLK	LEFT -Discontinued on Q2 chair 8/1/02

NOTE: When using Padded Swing-Away Arms, Anti-Tip Receivers are mounted on outside of frame.

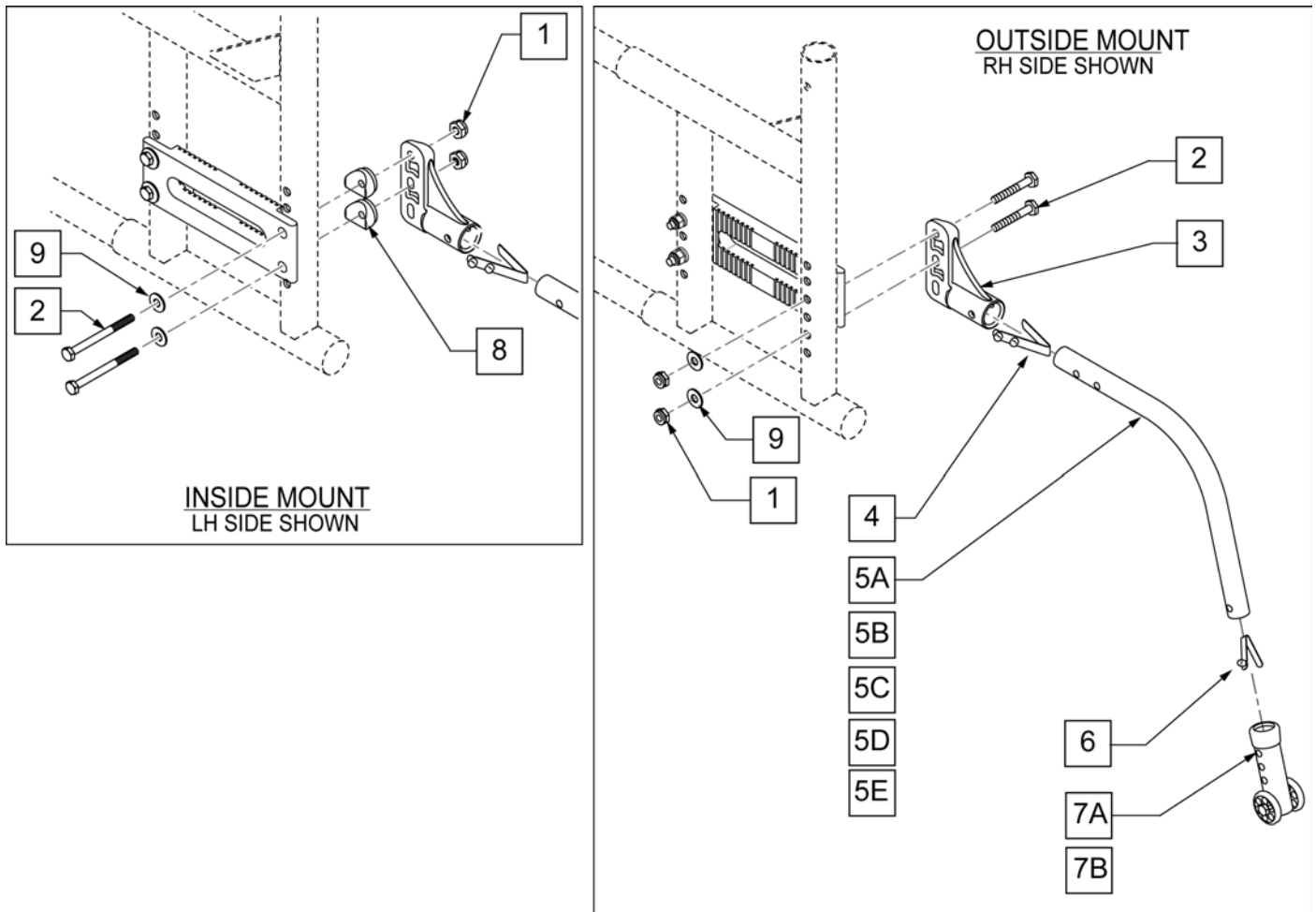


Pos.	Item Number	Description	Remarks
1	106934	1/4-28 NYLOCK NUT	
2	221181	1/4-28 x 2 1/2 G8 HEX C/S	
3	920407	ANTI-TIP RECEIVER	PAIR
4	301109	SNAP BUTTON DUAL ONE SIDE	
4,5A,6	882964	ANTI-TIP TUBE SMALL BLACK	RIGHT
4,5A,6	882965	ANTI-TIP TUBE SMALL BLACK	LEFT
4,5B,6	882896	ANTI-TIP TUBE REGULAR BLACK	RIGHT
4,5B,6	882897	ANTI-TIP TUBE REGULAR BLACK	LEFT
4,5C,6	882966	ANTI-TIP TUBE EXTENDED SMALL BLACK	RIGHT
4,5C,6	882967	ANTI-TIP TUBE EXTENDED SMALL BLACK	LEFT
4,5D,6	882974	ANTI-TIP TUBE EXTENDED REGULAR BLACK	RIGHT
4,5D,6	882975	ANTI-TIP TUBE EXTENDED REGULAR BLACK	LEFT
4,5E,6	883008	MED ANTI-TIP TUBE R W/ SNAP BUTTON	RIGHT
4,5E,6	883009	MED ANTI-TIP TUBE L W/ SNAP BUTTON	LEFT
6	301100	SNAP BUTTON 1/4" SINGLE	
7A	881512	ANTI-TIP SLEEVE W/WHEELS BLK (4H)	

Pos.	Item Number	Description	Remarks
7B	881513	ANTI-TIP SLEEVE W/WHEELS BLK (5H)	
8	561001	1 1/8" SADDLE 1/4" HOLE	N/A W/ XTENDER STRAIGHT & CURVED REAR SIDE FRAME
9	107367	1/4 WASHER ROHS	
4,5A,6,7A	882906	ANTI-TIP ASSM SMALL BLACK	RIGHT 18",20",22"(STD or 1AD AXLE PLATE)
4,5A,6,7A	882907	ANTI-TIP ASSM SMALL BLACK	LEFT 18",20",22"(STD or 1AD AXLE PLATE)
4,5B,6,7A	882900	ANTI-TIP ASSM REGULAR BLACK	RIGHT 24" (STD or 1AD AXLE PLATE)
4,5B,6,7A	882901	ANTI-TIP ASSM REGULAR BLACK	LEFT 24" (STD or 1AD AXLE PLATE)
4,5C,6,7A	882908	ANTI-TIP ASSM SMALL EXTENDED BLACK	RIGHT 18",20",22"(STD or 1AD AXLE PLATE)
4,5C,6,7A	882909	ANTI-TIP ASSM SMALL EXTENDED BLACK	LEFT 18",20",22"(STD or 1AD AXLE PLATE)
4,5D,6,7A	882910	ANTI-TIP ASSM REGULAR EXTENDED BLACK	RIGHT 24" (STD or 1AD AXLE PLATE)
4,5D,6,7A	882911	ANTI-TIP ASSM REGULAR EXTENDED BLACK	LEFT 24" (STD or 1AD AXLE PLATE)
4,5A,6,7B	882918	ANTI-TIP SML LNG SLV R ZGS BLK	RIGHT 20" & 22" (DAB - STD & CURVED)
4,5A,6,7B	882919	ANTI-TIP SML LNG SLV L ZGS BLK	LEFT 20" & 22" (DAB - STD & CURVED)
4,5B,6,7B	882912	ANTI-TIP ASSM REG LONG SLEEVE BLACK	RIGHT 24" -25" (ALL AXLE PLATES)


Pos.	Item Number	Description	Remarks
4,5B,6,7B	882913	ANTI-TIP ASSM REG LONG SLEEVE BLACK	LEFT 24" -25" (ALL AXLE PLATES)
4,5D,6,7B	882922	ANTI-TIP ASSM REG XTENDER W/ LONG SLEEVE BLACK	RIGHT & 24" -26" (DAB AMPUTEE AXLE) 26" (ALL AXLE PLATES)
4,5D,6,7B	882923	ANTI-TIP ASSM REG XTENDER W/ LONG SLEEVE BLACK	LEFT & 24" -26" (DAB AMPUTEE AXLE) 26" (ALL AXLE PLATES)
4,5E,6,7B	881128	ANTI-TIP ASSM MED RT BLK	RIGHT XTENDER ONLY
4,5E,6,7B	881129	ANTI-TIP ASSM MED LT BLK	LEFT XTENDER ONLY
4,5C,6,7B	880498	ANTI-TIP ASSM SMALL EXTND W/ LONG SLEEVE R BLK	RIGHT 20" & 22" (DAB-AMPUTEE AXLE)
4,5C,6,7B	880499	ANTI-TIP ASSM SMALL EXTND W/ LONG SLEEVE L BLK	LEFT 20" & 22" (DAB-AMPUTEE AXLE)
4,5A,6,7A	900701	ANTI-TIP ASSM SMALL	PAIR
4,5B,6,7A	900703	ANTI-TIP ASSM REGULAR	PAIR
4,5C,6,7A	900696	ANTI-TIP ASSM SMALL EXTEND	PAIR
4,5D,6,7A	900697	ANTI-TIP ASSM REGULAR EXTEND	PAIR
4,5A,6,7B	900241	ANTI TIP ASSM 5-HOLE HEMI BR	PAIR
4,5B,6,7B	900709	ANTI-TIP ASSM REG W/LNG SLEEVE	PAIR

NOTE: ATTACHMENT CLAMP mounts to 1" diameter tubing.

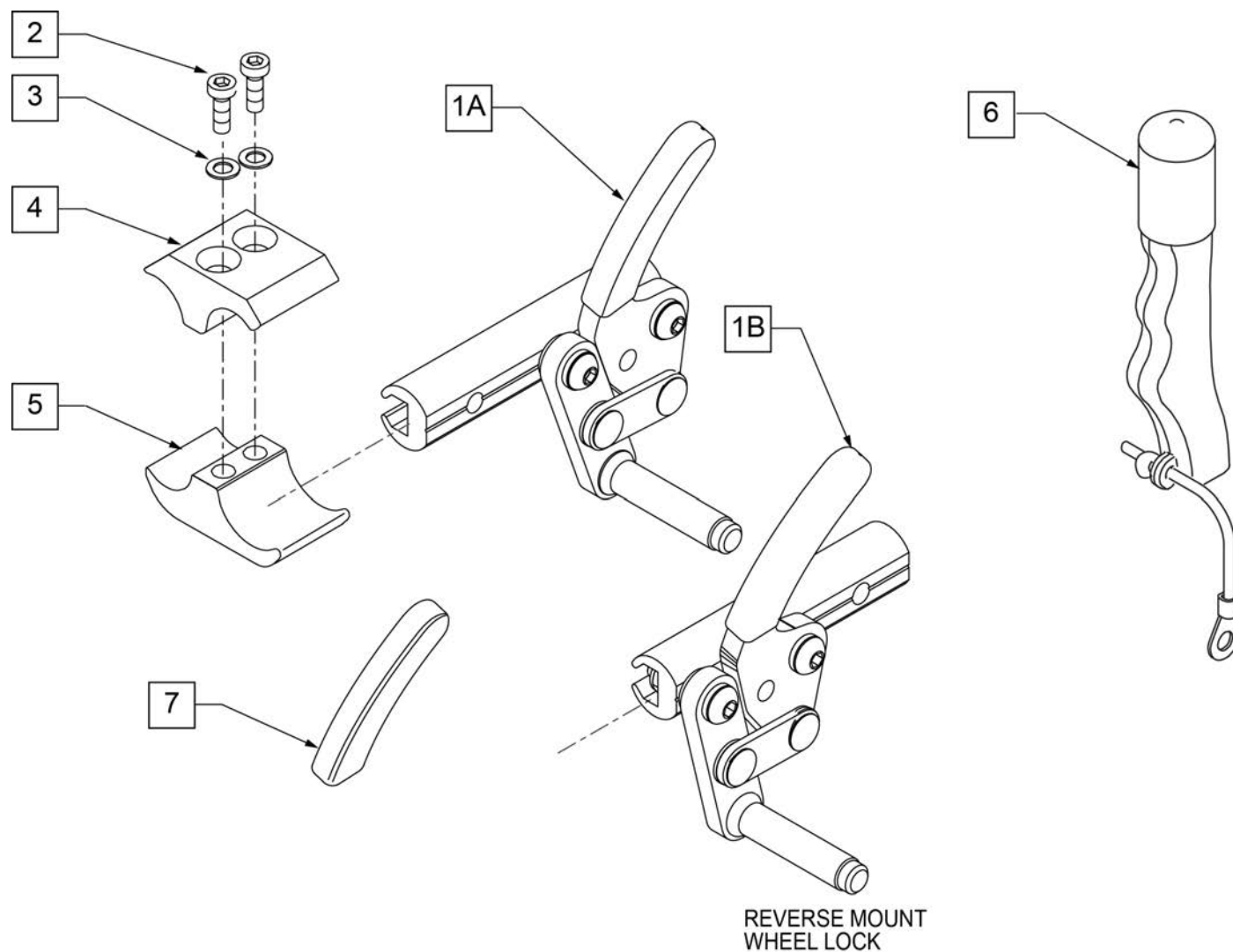


Pos.	Item Number	Description	Remarks
1A	104216-001	WHEEL LOCK ASSEMBLY PUSH RIGHT	RIGHT
1A	104216-002	WHEEL LOCK ASSEMBLY PUSH LEFT	LEFT
1B	109643	WHEEL LOCK HI PUSH REV R	RIGHT
1B	109644	WHEEL LOCK HI PUSH REV LEFT	LEFT
2	107032	SHCS 1/4-28 X 1 ROHS	
3	107367	1/4 WASHER HI COLLAR LOCK BLK	
6	118068	WHEEL LOCK EXTENSION ASSY	each
7	104141	COVER CURVED WHEEL LOCK VINYL	
1A,6	107637	WHEEL LOCK ASSM (PUSH) R W/ EXT	RIGHT
1A,6	107638	WHEEL LOCK ASSM (PUSH) L W/ EXT	LEFT
2-5	107639	WHL LCK CLMP ASM	
1B,6	112623	WHEEL LOCK ASSM (PUSH REV) R W/ EXT	RIGHT
1B,6	112624	WHEEL LOCK ASSM (PUSH REV) L W/ EXT	LEFT

NOTE: ATTACHMENT CLAMP mounts to 1" diameter tubing.

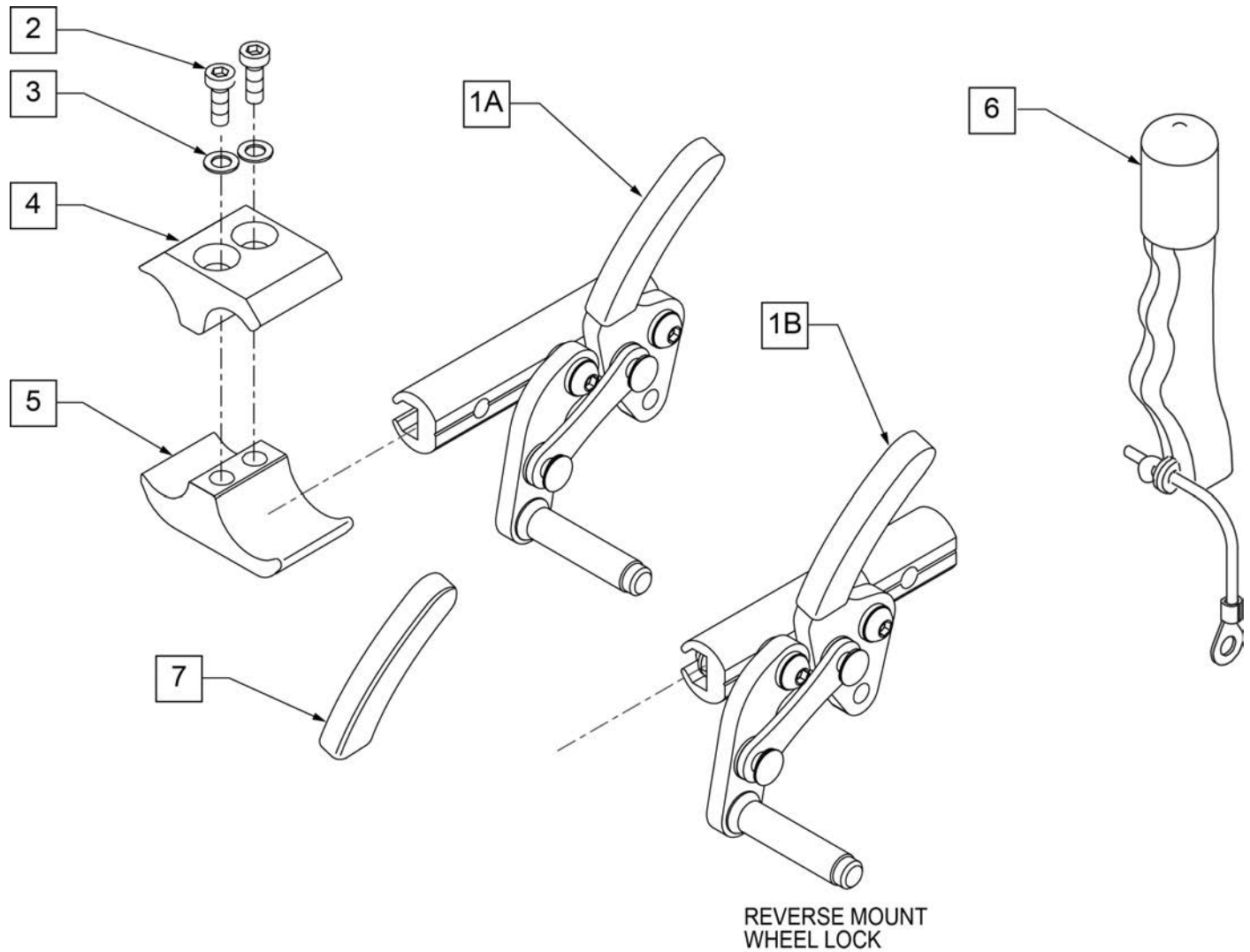


Pos.	Item Number	Description	Remarks
1A	104227-001	WHEEL LOCK ASSEMBLY (PULL) RIGHT	RIGHT
1A	104227-002	WHEEL LOCK ASSM (PULL) LEFT	LEFT
1B	109647	WHEEL LOCK HI PULL REV R	RIGHT
1B	109648	WHEEL LOCK HI PULL REV L	LEFT
2	107032	SHCS 1/4-28 X 1 ROHS	
3	107367	1/4 WASHER HI COLLAR LOCK BLK	
6	118068	WHEEL LOCK EXTENSION ASSY	each
7	104141	COVER CURVED WHEEL LOCK VINYL	
2-5	107639	WHL LCK CLMP ASM	
1A,6	107640	WHEEL LOCK ASSM (PULL) R W/ EXT	RIGHT
1A,6	107641	WHEEL LOCK ASSM (PULL) L W/ EXT	LEFT
1B,6	112625	WHEEL LOCK HI PULL REV R W/ EXT	RIGHT
1B,6	112626	WHEEL LOCK HI PULL REV L W/ EXT	LEFT

NOTE: ATTACHMENT CLAMP mounts to 1" diameter tubing.

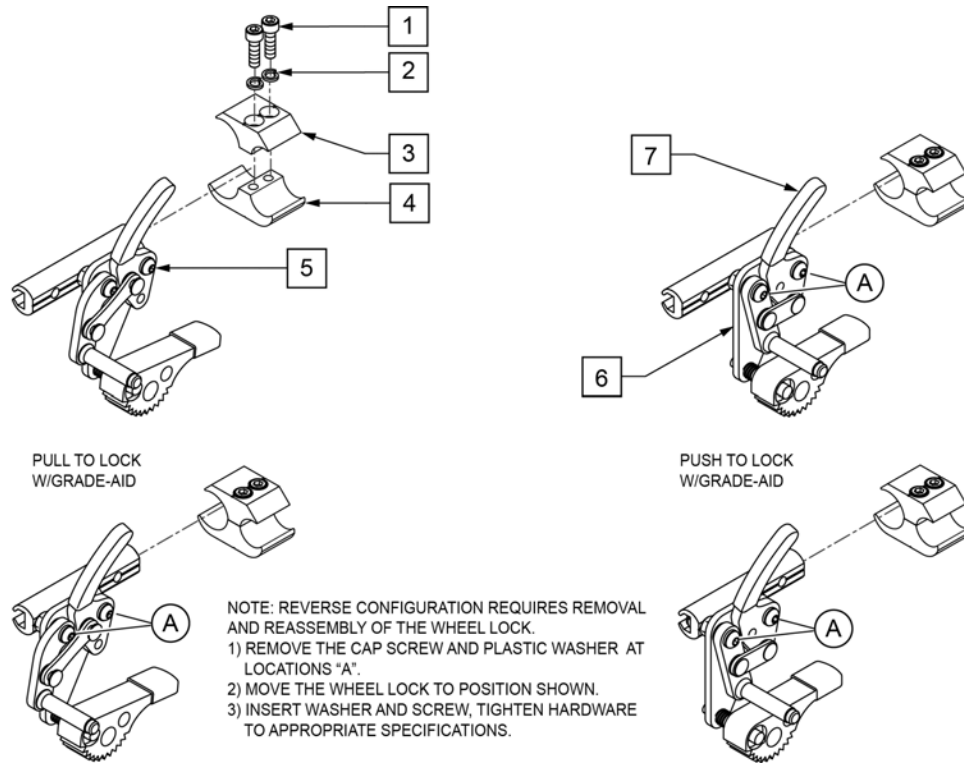

Pos.	Item Number	Description	Remarks
1	107032	SHCS 1/4-28 X 1 ROHS	
2	107367	1/4 WASHER HI COLLAR LOCK BLK	
1-4	107639	WHL LCK CLMP ASM	
5	107667	WHEEL LOCK PULL GRADE AID L	LEFT (Pull)
5	107668	WHEEL LOCK PULL GRADE AID R	RIGHT (Pull)
6	107665	WHEEL LOCK PUSH GRADE AID L	LEFT (Push)
6	107666	WHEEL LOCK PUSH GRADE AID R	RIGHT (Push)
7	104141	COVER CURVED WHEEL LOCK VIN	


NOTE: ATTACHMENT CLAMP mounts to 1" diameter tubing.

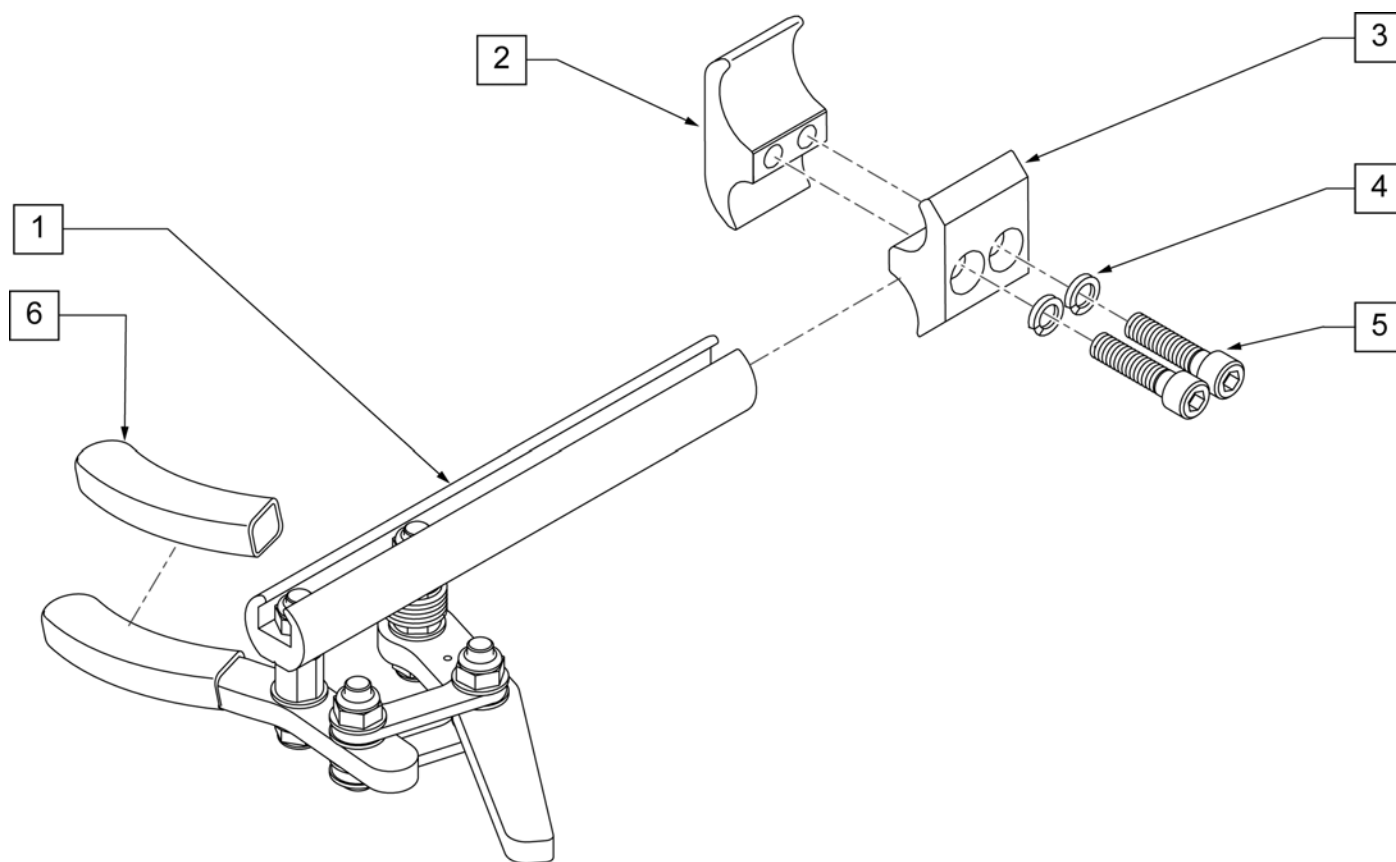

Pos.	Item Number	Description	Remarks
1	104726-001	ASSY WHL LOCK SCISSOR ERGO	RIGHT
1	104726-002	ASSY WHL LOCK SCISSOR ERGO	LEFT
4	107367	1/4 WASHER HI COLLAR LOCK BLK	
5	107032	SHCS 1/4-28 X 1 ROHS	
6	104141	COVER CURVED WHEELLOCK VIN	
2-6	107639	WHL LCK CLMP ASM	


NOTE: Refer to "WHEEL LOCK AVAILABILITY" Page for possible mount locations.

NOTE: ATTACHMENT CLAMP mounts to 1" diameter tubing.

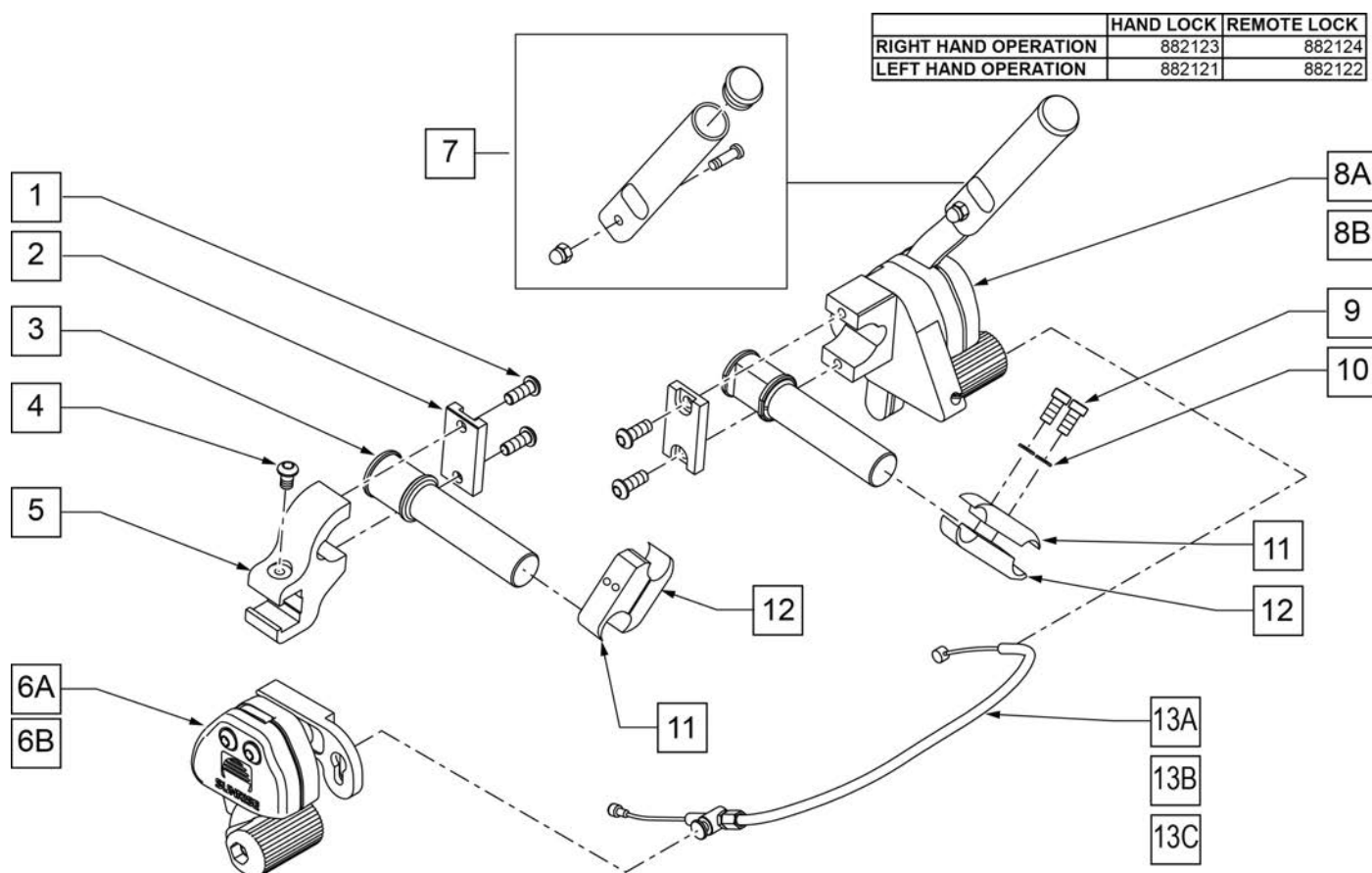

Pos.	Item Number	Description	Remarks
1	230032	M6 x 16 B/H CAP SCREW	
2	501032	CLAMP CAP	
3	501031	WHEEL LOCK EXTENSION	
4	230052	M6 1 x 10 BH C/S BLK	
5	501033	REMOTE LOCK CLAMP	
1-6A	882124	REMOTE LOCK ASSEMBLY RIGHT	Mounts on left side
1-6B	882122	REMOTE LOCK ASSEMBLY LEFT	Mounts on right side
7	883300	1-HAND WHLLCK RLCMNT HNDL KIT	4 7/8"
1,2,3,8A	882123	HAND LOCK ASSEMBLY RIGHT	Mounts on right side
1,2,3,8B	882121	HAND LOCK ASSEMBLY LEFT	Mounts on left side
9	107032	1/4-28 x 1 SOC C/S BLK	
10	262114	1/4 WASHER HI COLLAR	
13A	341760	WHEEL LOCK CABLE SHORT (11"-14")	11 -14" WIDE CHAIRS
13B	341761	WHEEL LOCK CABLE MEDIUM (15"-18")	15 -18" WIDE CHAIRS
13C	341762	WHEEL LOCK CABLE LONG (19"-22")	19 -22" WIDE CHAIRS
9-12	880422	WHL LCK EXTND CLMP ASSM BLK	
1-6A,8A-12,13A	902184	UNILTERAL 1HAND PSH LOCK SHT RH	RH SHOWN 11 -14" WIDE
1-6A,8A-12,13B	902186	UNILTERAL 1HAND PSH LOCK MED RH	RH SHOWN 15 -18" WIDE
1-6A,8A-12,13C	902188	UNILTERAL 1HAND PSH LOCK LONG RH	RH SHOWN 19 -22" WIDE


Pos.	Item Number	Description	Remarks
1-6B,8B,12,13A	902183	UNILTERAL 1HAND PSH LOCK SHT LH	LH 11 -14" WIDE
1-6B,8B,12,13B	902185	UNILTERAL 1HAND PSH LOCK MED LH	LH 15 -18" WIDE
1-6B,8B,12,13C	902187	UNILTERAL 1HAND PSH LOCK LONG LH	LH 19 -22" WIDE

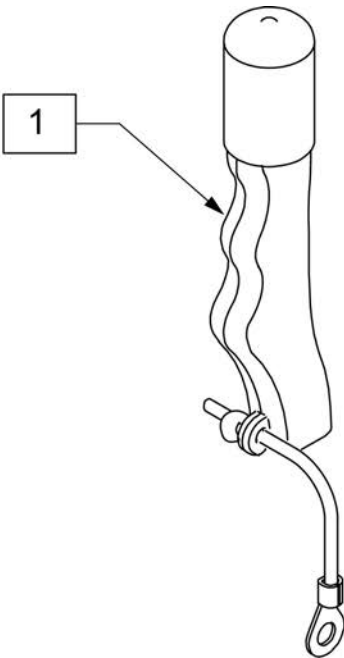

Pos.	Item Number	Description	Remarks
1	119563	WHEEL LOCK EXT ASSY (PAIR)	Pair

NOTE: Refer to "WHEEL LOCK AVAILABILITY" Page for possible mount locations.

NOTE: ATTACHMENT CLAMP mounts to 1" diameter tubing.

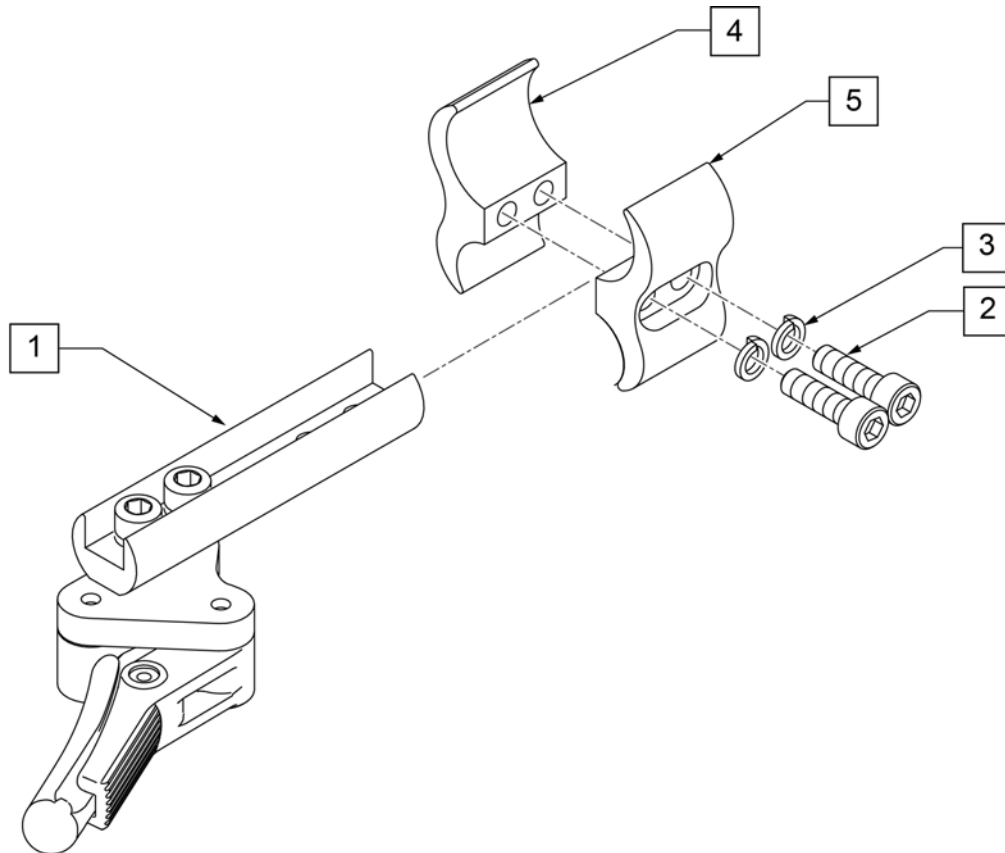

Pos.	Item Number	Description
1	066608-001	CMPCT WHLLCK STD ASSY LEFT
1	066608-002	CMPCT WHLLCK STD ASSY RIGHT
2	107032	SHCS 1/4-28 X 1 ROHS
3	107367	1/4 WASHER HI COLLAR LOCK BLK
2-5	107639	WHL LCK CLMP ASM



Pos.	Item Number	Description	Remarks
1-3	401094	20" MAG W/BRNG 6 SPOKE	20"
1-3	401093	22" MAG W/BRNG 6 SPOKE	22"(501mm)
1-3	401092	24" MAG W/BRNG 6 SPOKE	24"(540mm)
2	349500	1 1/8 X 1/2 BEARING	
3	521564	MAG BEARING SPACER	
4	101932	SPCR .194ID X .375 OD X .750	Used when previously ordered with Omit handrims or Natural Fit Handrims

(10/2010)

NOTE: THIS WHEEL UTILIZES SCREW MOUNT HANDRIMS - THE HANDRIM HAS A THREADED HOLE THAT MOUNTS THE HANDRIM TO THE WHEEL THROUGH A SPACER AND FASTNER.

NOTE: IF MAG WHEEL ASSEMBLY WAS ORDER WITH OUT HANDRIM AND A HANDRIM IS ADDED TO THE WHEEL QTY 6 OF 561037 AND 281175 MUST BE ORDERED TO PROPERLY MOUNT THE HANDRIM.

NOTE: THIS WHEEL UTILIZES SCREW MOUNT HANDRIMS - THE HANDRIM HAS A THREADED HOLE THAT MOUNTS THE HANDRIM TO THE WHEEL THROUGH A SPACER AND FASTNER.



Pos.	Item Number	Description	Remarks
1A	116178	WHEEL ASSY MBL 20"	20" (451)
1B	116179	WHEEL ASSY MBL 22"	WHL RIM/SPOKE MBL 22 (501)
1C	116180	WHEEL ASSY MBL 24"	WHL RIM/SPOKE MBL 24 (540)
1D	116181	WHEEL ASSY MBL 25"	WHL RIM/SPOKE MBL 25 (559)
1E	116182	WHEEL ASSY MBL 26"	WHL RIM/SPOKE MBL 26 (590)
2	114097	BEARING, REPLACEMENT, MBL	12.7mm i.d. - 28mm o.d. - 8mm wide
3A	118100	MBL LT 20IN SPOKE RPLMNT KIT	Kit contains 6 inner and 6 outer spokes
3B	118101	MBL LT 22IN SPOKE RPLMNT KIT	Kit contains 6 inner and 6 outer spokes
3C	118102	MBL LT 24IN SPOKE RPLMNT KIT	Kit contains 6 inner and 6 outer spokes
3D	118103	MBL LT 25IN SPOKE RPLMNT KIT	Kit contains 6 inner and 6 outer spokes
3E	118104	MBL LT 26IN SPOKE RPLMNT KIT	Kit contains 6 inner and 6 outer spokes

NOTE: THIS WHEEL UTILIZES TAB MOUNT HANDRIMS - THE HANDRIM HAS A STRAIGHT TAB THAT MOUNTS TO THE WHEEL THROUGH THE RIM.



Pos.	Item Number	Description	Remarks
1A	112857	ULTRA LIGHTWEIGHT WHEEL 20"	20"
1B	112858	ULTRA LIGHT WHEEL 22"	22"(501mm)
1C	112859	ULTRA LIGHTWEIGHT WHEEL 24"	24"(540mm)
1D	112860	ULTRA LIGHTWEIGHT WHEEL 25"	25"(559mm)
2	349500	1 1/8 X 1/2 BEARING	
3A	118105	ULTRA LT 20IN SPOKE RPLMNT KIT	Kit contains 12 spokes
3B	118106	ULTRA LT 22IN SPOKE RPLMNT KIT	Kit contains 12 spokes
3C	118107	ULTRA LT 24IN SPOKE RPLMNT KIT	Kit contains 12 spokes
3D	118108	ULTRA LT 25IN SPOKE RPLMNT KIT	Kit contains 12 spokes

NOTE: THIS WHEEL UTILIZES TAB MOUNT HANDRIMS - THE HANDRIM HAS A STRAIGHT TAB THAT MOUNTS TO THE WHEEL THROUGH THE RIM.




Pos.	Item Number	Description	Remarks
1	762438	SPINERGY STD WHEEL 540 BLK SPK	24"(540mm)
1	762441	SPINERGY STD WHL 559 BLK SPK	25"(559mm)
1	762437	SPINERGY STD WHEEL 590 BLK SPK	26"(590mm)
1	762442	SPINERGY STD WHEEL 540 BLUE	24"(540mm)
1	762444	SPINERGY STD WHEEL 559 BLU	25"(559mm)
1	762446	SPINERGY STD WHEEL 590 BLUE	26"(590mm)
1	762443	SPINERGY STD WHEEL 540 RED	24"(540mm)
1	762445	SPINERGY STD WHEEL 559 RED	25"(559mm)
1	762447	SPINERGY STD WHEEL 590 RED	26"(590mm)
1	762456	SPINERGY STD WHEEL 540 WHT	24"(540mm)
1	762463	SPINERGY STD WHL 559 WHT SPK	25"(559mm)
1	762467	SPINERGY STD WHEEL 590 WHT	26"(590mm)
1	762433	SPINERGY STD WHEEL 540 YELLOW	24"(540mm)
1	762462	SPINERGY STD WHL 559 YLW SPK	25"(559mm)
1	762431	SPINERGY STD WHEEL 590 YELLOW	26"(590mm)
2	349500	1 1/8 X 1/2 BEARING	

NOTE: THIS WHEEL UTILIZES TAB MOUNT HANDRIMS - THE HANDRIM HAS A STRAIGHT TAB THAT MOUNTS TO THE WHEEL THROUGH THE RIM.



Pos.	Item Number	Description	Remarks	Pos.	Item Number	Description	Remarks
1	101988-001	SPINRGY SPT LT EX WHL 540 BLK	24"(540mm)	1	101992-003	SPINRGY SPT LT EX WHL 590 RED	26"(590mm)
1	101989-001	SPINRGY LT EX WHEEL 540 BLK	24"(540mm)	1	101993-003	SPINRGY LT EX WHEEL 590 RED	26"(590mm)
1	101990-001	SPINRGY SPT LT EX WHL 559 BLK	25"(559mm)	1	101988-005	SPINRGY SPT LT EX WHL 540 WHT	24"(540mm)
1	101991-001	SPINRGY LT EX WHEEL 559 BLK	25"(559mm)	1	101989-005	SPINRGY LT EX WHEEL 540 WHT	24"(540mm)
1	101992-001	SPINRGY SPT LT EX WHL 590 BLK	26"(590mm)	1	101990-005	SPINRGY SPT LT EX WHL 559 WHT	25"(559mm)
1	101993-001	SPINRGY LT EX WHEEL 590 BLK	26"(590mm)	1	101991-005	SPINRGY LT EX WHEEL 559 WHT	25"(559mm)
1	101988-002	SPINRGY SPT LT EX WHL 540 BLU	24"(540mm)	1	101992-005	SPINRGY SPT LT EX WHL 590 WHT	26"(590mm)
1	101989-002	SPINRGY LT EX WHEEL 540 BLU	24"(540mm)	1	101993-005	SPINRGY LT EX WHEEL 590 WHT	26"(590mm)
1	101990-002	SPINRGY SPT LT EX WHL 559 BLU	25"(559mm)	1	101988-004	SPINRGY SPT LT EX WHL 540 YLW	24"(540mm)
1	101991-002	SPINRGY LT EX WHEEL 559 BLU	25"(559mm)	1	101989-004	SPINRGY LT EX WHEEL 540 YLW	24"(540mm)
1	101992-002	SPINRGY SPT LT EX WHL 590 BLU	26"(590mm)	1	101990-004	SPINRGY SPT LT EX WHL 559 YLW	25"(559mm)
1	101993-002	SPINRGY LT EX WHEEL 590 BLU	26"(590mm)	1	101991-004	SPINRGY LT EX WHEEL 559 YLW	25"(559mm)
1	101988-003	SPINRGY SPT LT EX WHL 540 RED	24"(540mm)	1	101992-004	SPINRGY SPT LT EX WHL 590 YLW	26"(590mm)
1	101989-003	SPINRGY LT EX WHEEL 540 RED	24"(540mm)	1	101993-004	SPINRGY LT EX WHEEL 590 YLW	26"(590mm)
1	101990-003	SPINRGY SPT LT EX WHL 559 RED	25"(559mm)	2	349500	1 1/8 X 1/2 BEARING	
1	101991-003	SPINRGY LT EX WHEEL 559 RED	25"(559mm)				

NOTE: THIS WHEEL UTILIZES SCREW MOUNT HANDRIMS - THE HANDRIM HAS A THREADED HOLE THAT MOUNTS THE HANDRIM TO THE WHEEL THROUGH A SPACER AND FASTNER.

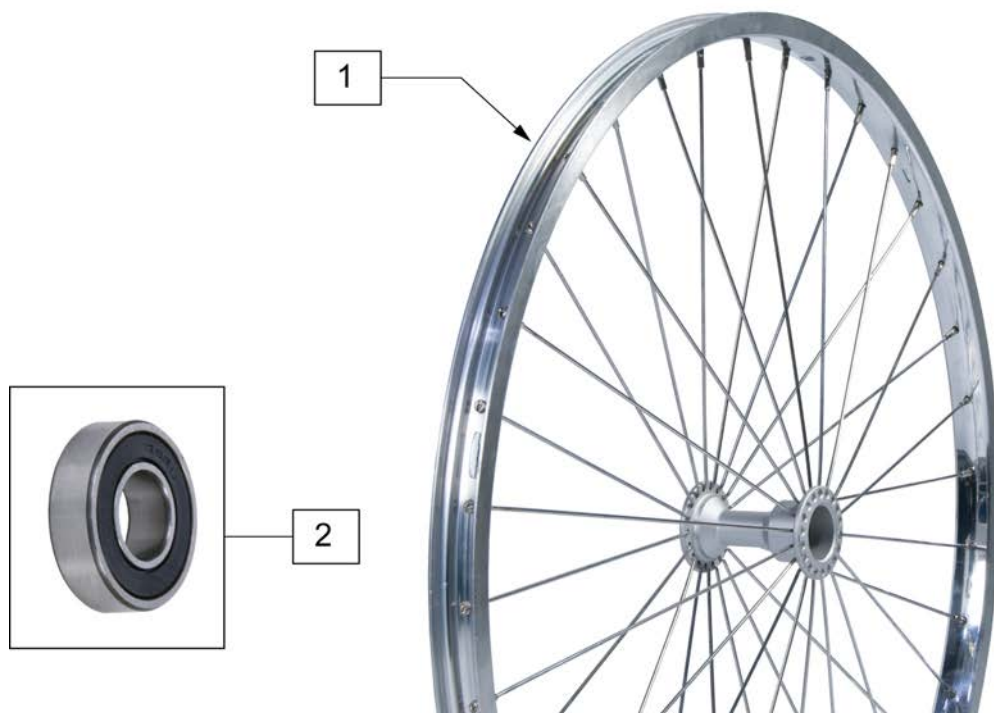

Pos.	Item Number	Description	Remarks
1	401075	24 RIM ASSM X 1.5 SUN PSH BTTN	22" (507MM)
1	401076	24" HT RIM ASSM SUN PUSH BTTN	25" (559MM)
2	349500	1 1/8 X 1/2 BEARING	




Pos.	Item Number	Description	Remarks
1	384706	TIRE PNEU 12-1/2" X 2-1/4"	12-1/2" X 2-1/4"
1	385237	16" X 1.75" PNEU TIRE	
1	385225	TIRE PNEUMATIC 18"	18" x 1 3/8
1	385250	20" PNEUMATIC TIRE	20" x 1 3/8
1	385275	22" PNEUMATIC TIRE	22" x 1 3/8 (501mm)
1	385300	24" PNEUMATIC TIRE	24" x 1 3/8 (540mm)
1	385325	26" PNEUMATIC TIRE	26" x 1 3/8 (590mm)
1	384614	24" x 2.125 PNEUMATIC TIRE	24"x2 1/8 (540mm) used w/ M6




Pos.	Item Number	Description	Remarks
1	386232	20" POLY TIRE STD	20"
1	386242	22" POLY TIRE STD	22" (501mm)
1	386252	24" POLY TIRE STD	24" (540mm)




Pos.	Item Number	Description
1	385350	24" KEVLAR TIRE

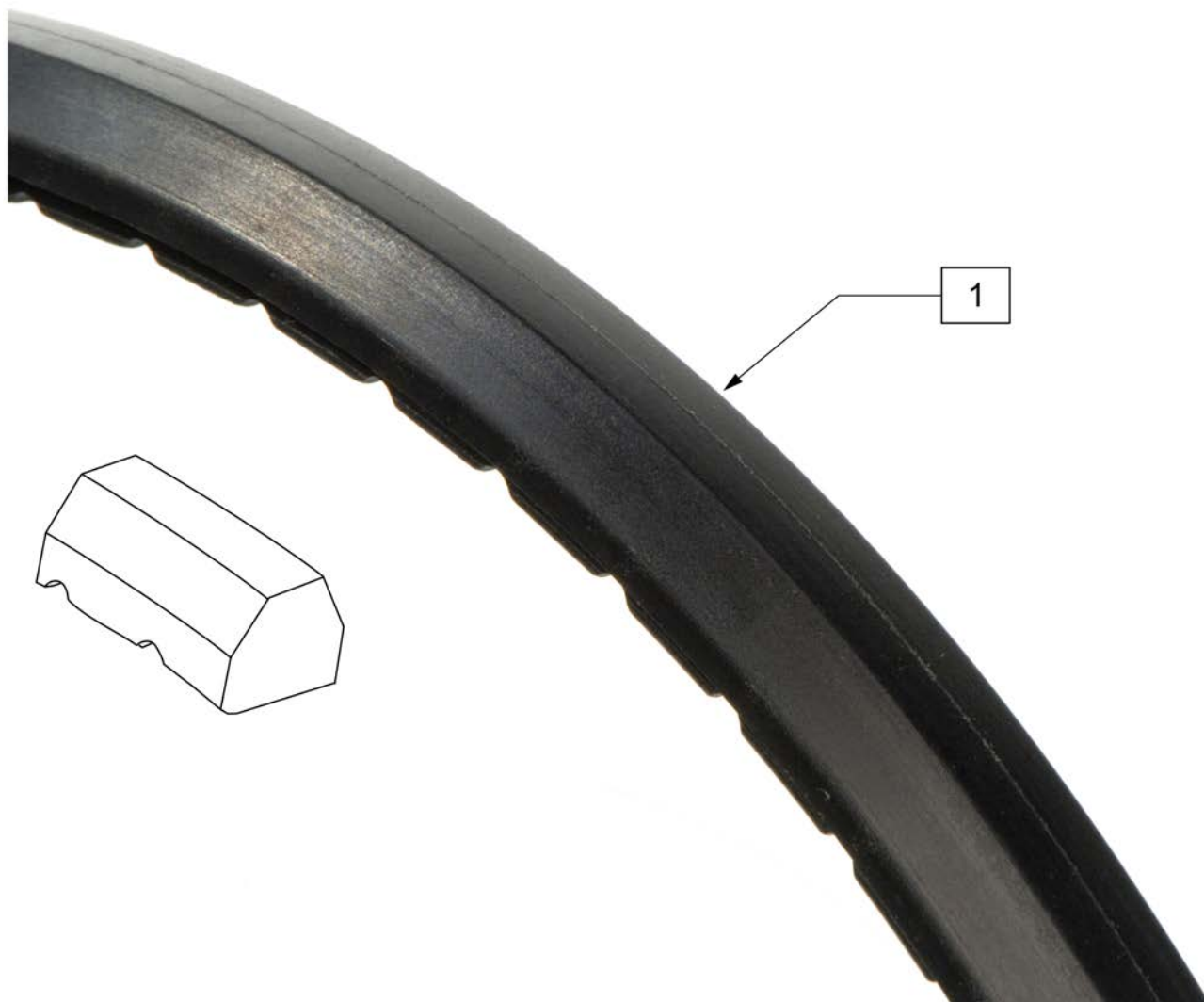

Pos.	Item Number	Description	Remarks
1	386262	20" LP POLY TIRE BR	20"
1	386263	22" LP POLY TIRE BR	22"(501mm)
1	386254	24" POLY LOW PROFILE	24"(540mm)

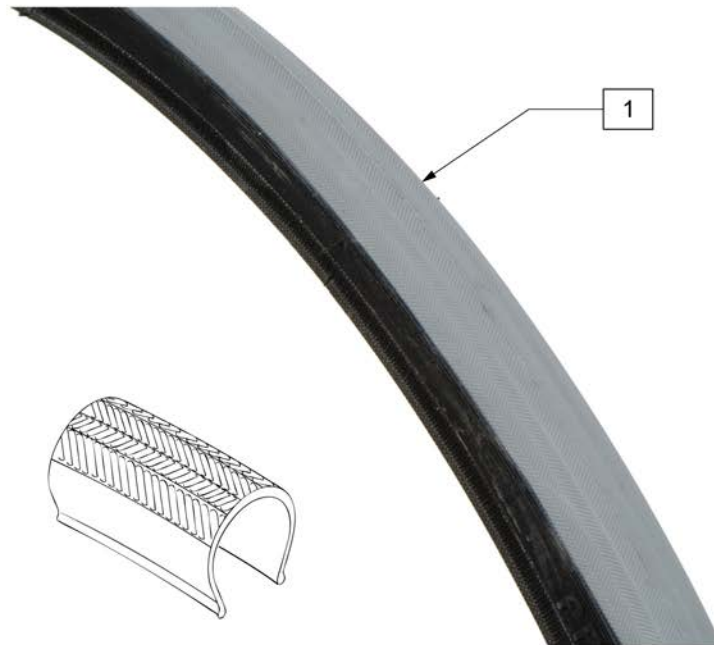


Pos.	Item Number	Description	Remarks
1	385280	22" x 1" HI PRES CLINCHER	22" (501mm)
1	389020	24" x 1" HI PRES CLNCHR TIRE	24" (540mm)
1	389012	QP 24" x 1" HT TURB BLK WR/BD 559	25" (559mm)
1	389030	26" x 1" HI PRES CLNCHR TIRE	26" (590mm)
1	105919	22" x 1" HI PRESS CLNCHR ULTRALIGHT	22" (489mm)

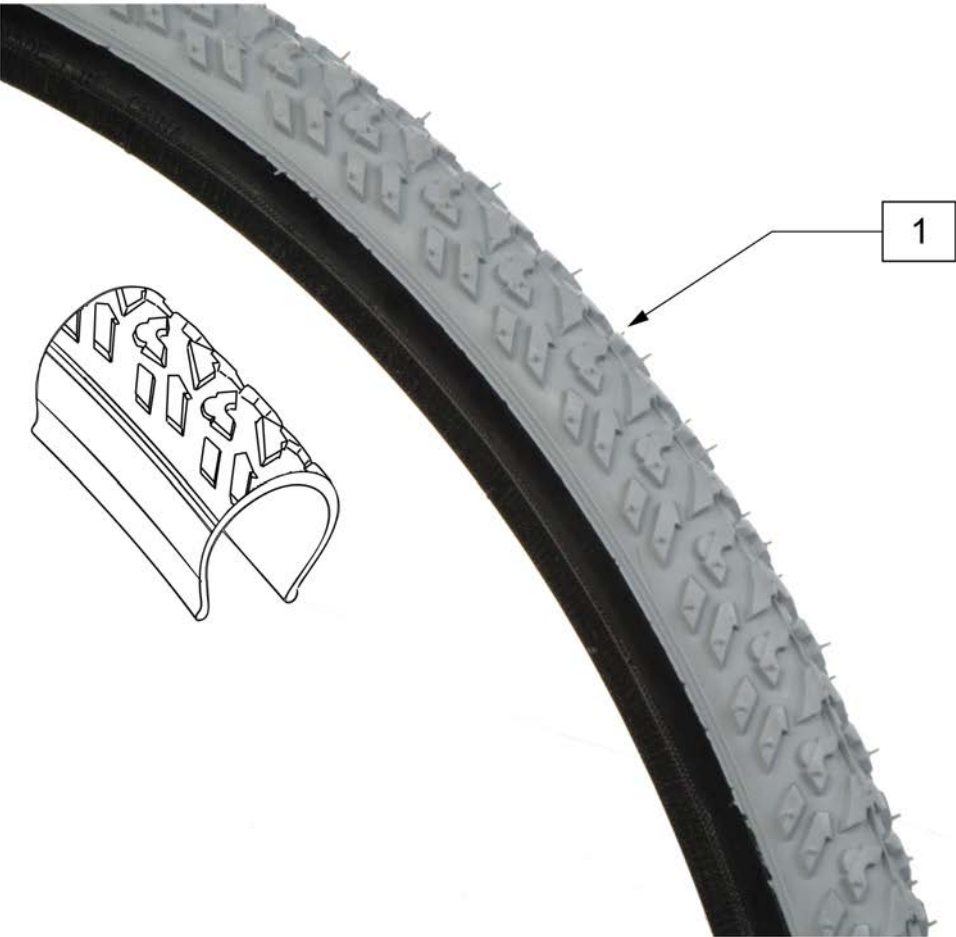

Pos.	Item Number	Description	Remarks
1	385354	24" X 1-3/8" NOBBIE TIRE	24"(540mm)



Pos.	Item Number	Description	Remarks
1	102066-004	TIRE SCHWALBE MP 489 BLK	22" (489mm)
1	102066-001	TIRE SCHWBL MPE 540 BLK	24" (540mm)
1	102066-002	TIRE SCHWBL MPE 559 BLK	25" (559mm)
1	102066-003	TIRE SCHWBL MPE 590 BLK	26" (590mm)

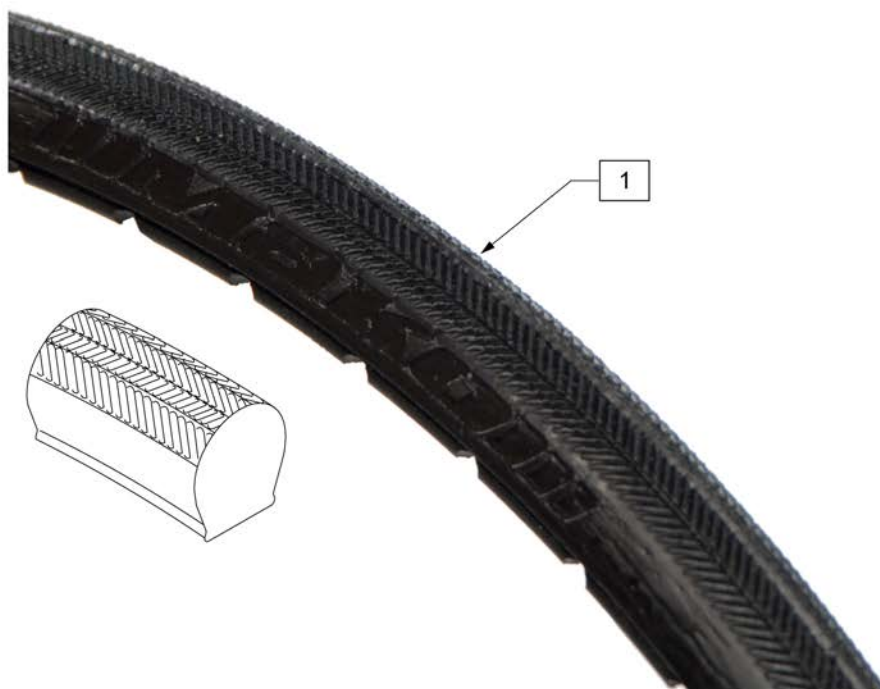

Pos.	Item Number	Description	Remarks
1	102060-001	TIRE KIK MAKO MAG SUN 540 BLK	24"(540mm) Black
1	102060-002	TIRE KIK MAKO MAG SUN 540 YLW	24"(540mm) Yellow
1	102060-003	TIRE KIK MAKO MAG SUN 540 RED	24"(540mm) Red
1	102060-004	TIRE KIK MAKO MAG SUN 540 BLU	24"(540mm) Blue
1	102060-005	TIRE KIK MAKO MAG SUN 540 DKG	24"(540mm) Dark Grey
1	102061-001	TIRE KIK MAKO SPIN 559 BLK	25"(559mm) Black
1	102061-002	TIRE KIK MAKO SPIN 559 YLW	25"(559mm) Yellow
1	102061-003	TIRE KIK MAKO SPIN 559 RED	25"(559mm) Red
1	102061-004	TIRE KIK MAKO SPIN 559 BLU	25"(559mm) Blue
1	102061-005	TIRE KIK MAKO SPIN 559 DKGRPH	25"(559mm) Dark Grey





Pos.	Item Number	Description	Remarks
1	109213	TIRE 24" URBAN ASSAULT	22"(507mm)
1	109214	TIRE 26" URBAN ASSAULT	26"(590mm)




Pos.	Item Number	Description	Remarks
1	384604	TIRE CS 24 X 2.1 SMOKER 507MM	22"(501mm)
1	384606	TIRE CS 26 X 2.1 SMOKER 559MM	25"(559mm)



Pos.	Item Number	Description	Remarks
>>>>>	>>>>>>>>>	LITE SPOKE	>>>>>
1	107172	HANDRIM 20	20" LITE SPOKE
1	107173	HANDRIM 22	22" (501) LITE SPOKE
1	107174	HANDRIM 24	24" (540) LITE SPOKE
1	107175	HANDRIM 25	25" (559) LITE SPOKE
1	107176	HANDRIM 26	26" (590) LITE SPOKE
2A	107267	HAND RIM SPACER REG THIN	LITE SPOKE
3A	108256	M5 X 0.8 X 40 PHIL BLK ROHS	LITE SPOKE
>>>>>	>>>>>>>>>	SPOKE	>>>>>
1,2B,3B	425931	20" HR 6SMT W/HDWR	20" SPOKE
1,2B,3B	425941	22" HR 6SMT W/HDWR	22" (501) SPOKE
1,2B,3B	426000	24" HR 6SMT W/ HDWR	24" (540) SPOKE
2B	561036	HANDRIM SPACER (REG)	SPOKE
3B	281170	10-32 X 1 5/8 PAN PHIL SS	SPOKE
>>>>>	>>>>>>>>>	MAG	>>>>>
1,3C	425934	20" HR 6SMT 6MAG W/HDWR	20" MAG
1,3C	425951	22" HR 6SMT 6MAG W/HDWR	22" (501) MAG
1,3C	426009	24" HR 6SMT 6MAG W/HDWR	24" (540) MAG
3C	281175	10-32 X 2 1/8 PAN PHIL SS	MAG

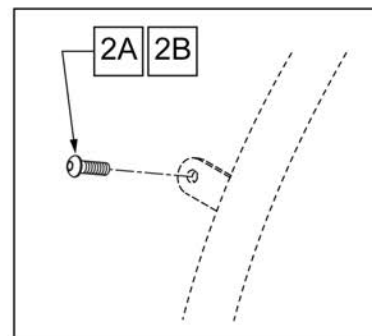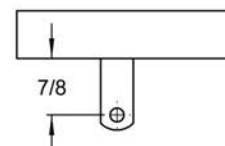
**MEASURING GUIDE**

Pos.	Item Number	Description	Remarks
>>>>>	>>>>>>>>	ULTRA LITE SPOKE	>>>>>
1	100739-001	HANDRIM 22" WHL MBL 1260	22" (489)
1	100739-002	HANDRIM 24" WHL MBL 1260	24" (540)
1	100739-003	HANDRIM 26" WHL MBL 1260	26" (590)
2A	102259-001	BHCS M5 X .8 X 12 W/PL INORG	QTY 6 REG
>>>>>	>>>>>>>>	NEW ULTRA LITE SPOKE	>>>>>
1	066516-002	LIGHTWEIGHT HANDRIM 22"	22" (489)
1	066516-003	LIGHTWEIGHT HANDRIM 24"	24" (540)
1	066516-004	LIGHTWEIGHT HANDRIM 25"	26" (559)
2B	012931	BHCS M5 X 10 W/PL BLK CL 10.9	QTY 6 REG

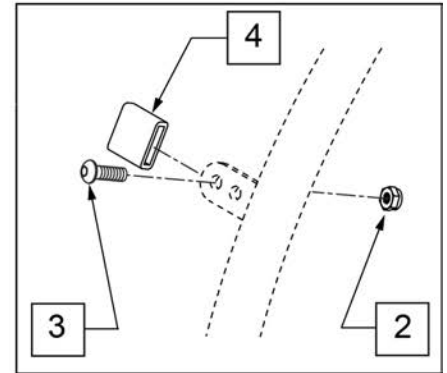


Pos.	Item Number	Description	Remarks
>>>>>	>>>>>>>>>	CROSS SPOKE & SPOKE	>>>>>
1 -4	425095	HANDRIM 18" 4T W/HDWR ANO	18" SPOKE
1	529529	QP 24" HR .065 (6) TAB ANO	24" (540) CROSS SPOKE -SHORT & REG TAB
1 -4	425030	24" HR 6T BB/HTBB W/HDWR	25" (590) CROSS SPOKE
>>>>>	>>>>>>>>>	CROSS SPOKE QP (QUICKIE PERFORMANCE)	>>>>>
1 -4	425050	26" HR 6T BB/TN W/HDWR	26" (590) CROSS SPOKE TENNIS & BBALL
1 -4	425045	24" HR SHT(6T)ANO W/X/S HDWR	25" (590) CROSS SPOKE TENNIS & BBALL-SHORT TAB
1 -4	425048	26" HR (6T)ANO W/X/S HDWR	26" (700) CROSS SPOKE TENNIS & BBALL -REG TAB
1 -4	425055	26" HR SHT(6T)ANO W/X/S HDWR	26" (700) CROSS SPOKE TENNIS & BBALL-SHORT TAB
1 -4	426015	QP 24" HR .049 W/HDWR SMT	24" (540) TENNIS
1 -4	425030	24" HR 6T BB/HTBB W/HDWR	25" (559) TENNIS & BASKETBALL
1 -4	426050	QP 26" HR .065 W/HDWR (6T)	26" (700) TENNIS & BASKETBALL
1 -4	425050	26" HR 6T BB/TN W/HDWR	26" (590) TENNIS & BASKETBALL
>>>>>	>>>>>>>>>	SPINERGY	>>>>>
1	529529	QP 24" HR .065 (6) TAB ANO	24" (540) SPINERGY
1 -4	425038	QP 24" HR .049 W/HDWR (6T)	25" (559) SPINERGY
1 -4	425050	26" HR 6T BB/TN W/HDWR	26" (590) SPINERGY
>>>>>	>>>>>>>>>	MOUNTAIN	>>>>>
1 -4	425020	22" HR 4T W/HDWR	24" (540) MOUNTAIN
1 -4	425030	24" HR 6T BB/HTBB W/HDWR	26" (590) MOUNTAIN
2	200100-QU	10-32 NYLOCK NUT	

Pos.	Item Number	Description	Remarks
3	281110	10-32 X 1/2 PAN PHIL	
4	580301	HR TAB COVER BLK	

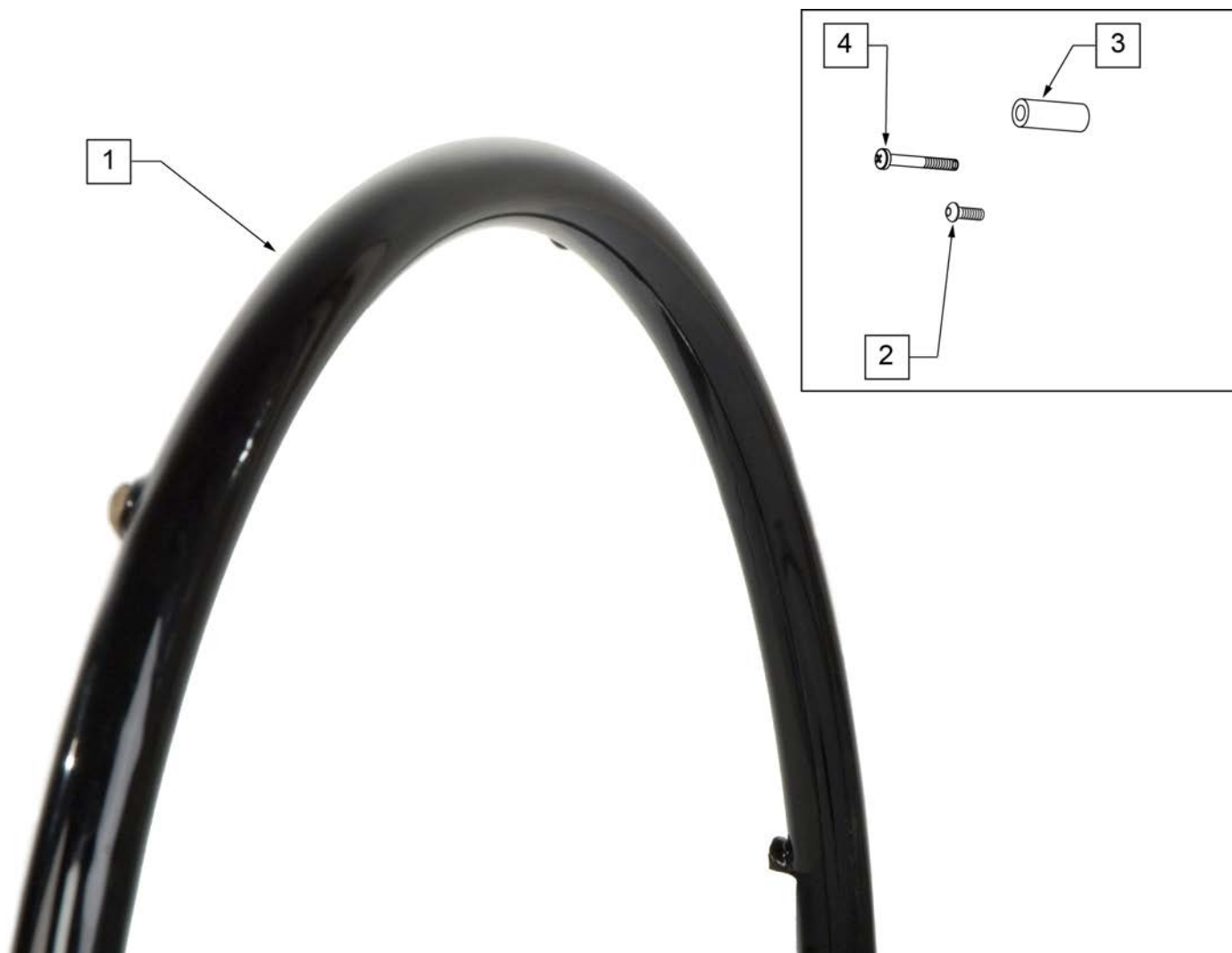


Pos.	Item Number	Description	Remarks
>>>>>	>>>>>>>>>>	SPOKE & LITE SPOKE	>>>>>
1 -4	425933	20" HR 6SMT PLST W/HDWR	20" SPOKE
1 -4	425943	22" HR 6SMT PLST W/HDWR	22" (501) SPOKE
1 -4	426004	24" HR 6SMT PLST W/HDWR	24" (540) SPOKE
>>>>>	>>>>>>>>>>	MAG	>>>>>
1 -4	425935	20" HR 6SMT 6MAG PLST W/HDWR	20" MAG
1 -4	425953	22" HR 6SMT 6MAG PLST W/HDWR	22" (501) MAG
1 -4	426013	24" HR 6SMT 6MAG PLST W/HDWR	24" (540) MAG
>>>>>	>>>>>>>>>>	M6 SPOKE	>>>>>
1 -4	425953	22" HR 6SMT 6 MAG PLST W/ HDWR	24" (540) M6 SPOKE
2	281170	10 -32 x 1 5/8 PAN PHIL SS	SPOKE
3	561036	HANDRIM SPACER REG	SPOKE
4	281175	10 -32 x 2 1/8 PAN PHIL SS	MAG

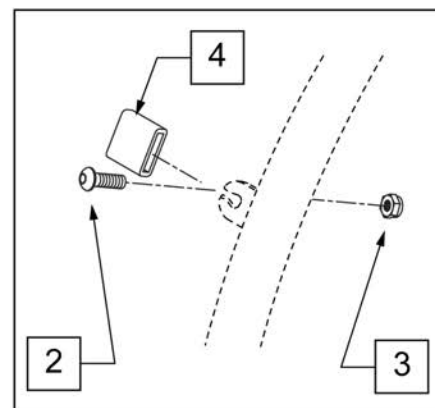

Pos.	Item Number	Description	Remarks
>>>>>	>>>>>>>>>	MOUNTAIN	>>>>>
1 -4	425022	22" HR 4T PLST W/ HDWR	24" (540)
1 -4	425032	24"HR 6T PLST BB/NTBB/TN W/HDWR	26" (590)
>>>>>	>>>>>>>>>	SPINERGY	>>>>>
1 -4	425039	24"HR 6T BLK PLST W/HDWR	24" (540)
1 -4	425032	24"HR 6T PLST BB/NTBB/TN W/HDWR	25" (559)
1 -4	425052	26" HR 6T PLST BB/TN W/HDWR	26" (590)
>>>>>	>>>>>>>>>	QP (QUICKIE PERFORMANCE)	>>>>>
1 -4	425032	24"HR 6T PLST BB/NTBB/TN W/HDWR	25" (559)
1 -4	425052	26" HR 6T PLST BB/TN W/HDWR	26" (590)
1 -4	426060	QP 26" HR W/ HDWR (6T) PLASTIC	26" (700)
2	281110	10 -32 x 1/2 PAN PHIL	
3	200100-QU	10 -32 NYLOCK NUT	
4	580301	HR TAB COVER BLK	


Note: Screw Mount Handrims are utilized with Lite Spoke, Spoke and Mag Wheels.

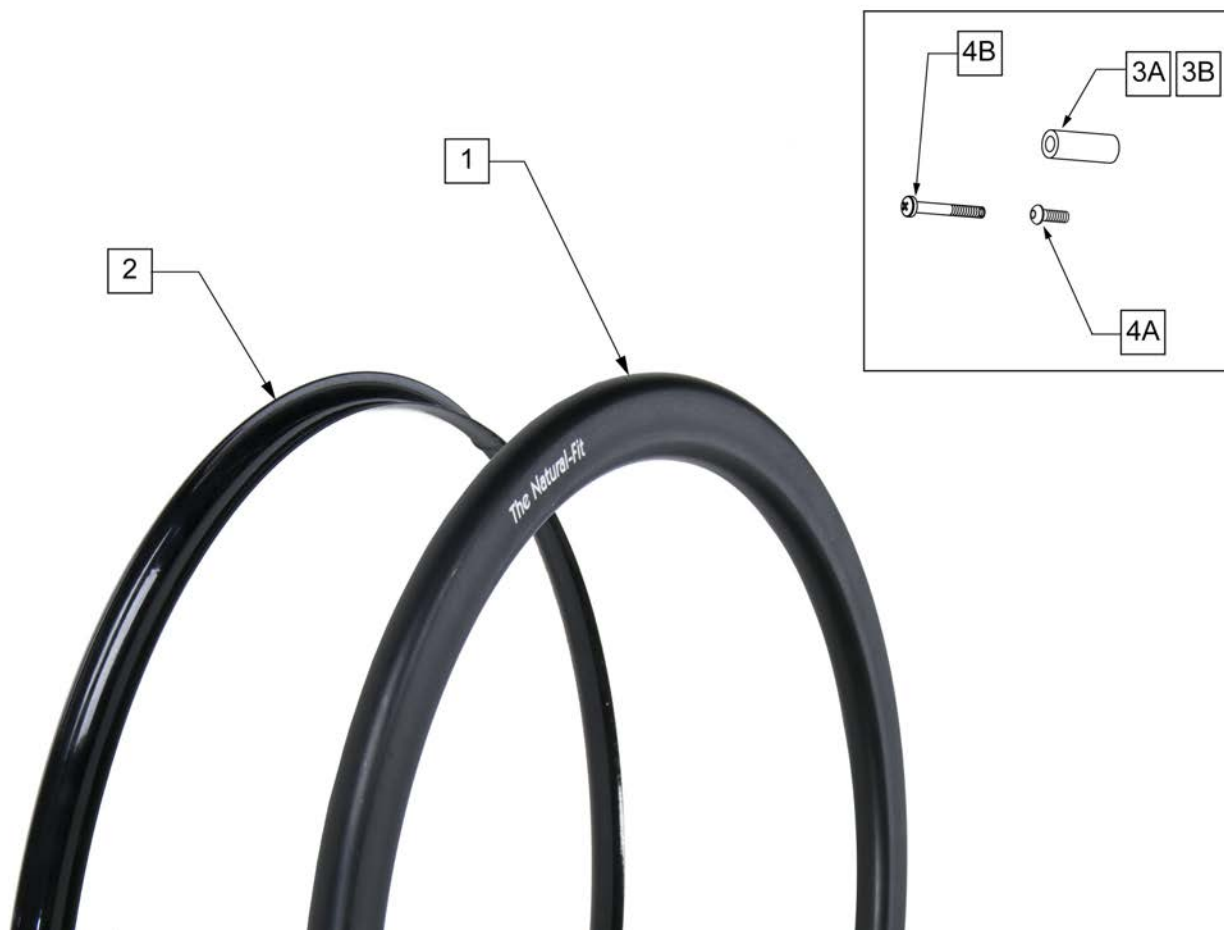

Pos.	Item Number	Description	Remarks
>>>>>	>>>>>>>>>>	STD GRIP THUMB GUARD	>>>>>
2	102168-001	GUARD THUMB NAT FIT 22 PC BLK	22" (501)- Std Grip Thumb Guard
2	102168-003	GUARD THUMB NAT FIT 24 PC BLK	24" (540)- Std Grip Thumb Guard
2	102168-005	GUARD THUMB NAT FIT 25 PC BLK	25" (559)- Std Grip Thumb Guard
2	102168-007	GUARD THUMB NAT FIT 26 PC BLK	26" (590)- Std Grip Thumb Guard
>>>>>	>>>>>>>>>>	SUPER GRIP THUMB GUARD	>>>>>
2	102168-002	GUARD THUMB NAT FIT 22 SG BLK	22" (501)- Super Grip Thumb Guard
2	102168-004	GUARD THUMB NAT FIT 24 SG BLK	24" (540)- Super Grip Thumb Guard
2	102168-006	GUARD THUMB NAT FIT 25 SG BLK	25" (559)- Super Grip Thumb Guard
2	102168-008	THUMB GUARD NAT FIT 26 SG BLK	26" (590)- Super Grip Thumb Guard
>>>>>	>>>>>>>>>>	NATURAL FIT LITE HANDRIM	>>>>>
1	108834-001	HAND RIM NAT-FIT LT RV NT 22"	22" (501)
1	108834-002	HAND RIM NAT-FIT LT RV NT 24"	24" (540)
1	108834-003	HAND RIM NAT-FIT LT RV NT 25"	25" (559)
1	108834-004	HAND RIM NAT-FIT LT RV NT 26"	26" (590)
>>>>>	>>>>>>>>>>	NATURAL FIT HANDRIM	>>>>>
1	102169-001	HAND RIM NAT FIT RV NT 22 BLK	22" (501)


Pos.	Item Number	Description	Remarks
1	102169-002	HAND RIM NAT FIT RV NT 24 BLK	24" (540)
1	102169-003	HAND RIM NATF RIV 25" BLK	25" (559)
1	102169-004	HAND RIM NATF RIV 26" BLK	26" (590)
3A	101931	SPCR .194ID X .43OD X 1.13LG	USED W/ LITE SPOKE & SPOKE
3B	101932	SPCR .194ID X .375 OD X .750	USED W/ MAG
4A	226034	10-32 X 1 3/4 BH C/S BLK	USED W/ LITE SPOKE & SPOKE
4B	281175	10-32 X 2 1/8 PAN PHIL SS	USED W/ MAG

Note: Tab Mount Hand Rims are utilized with Spinergy Wheels.

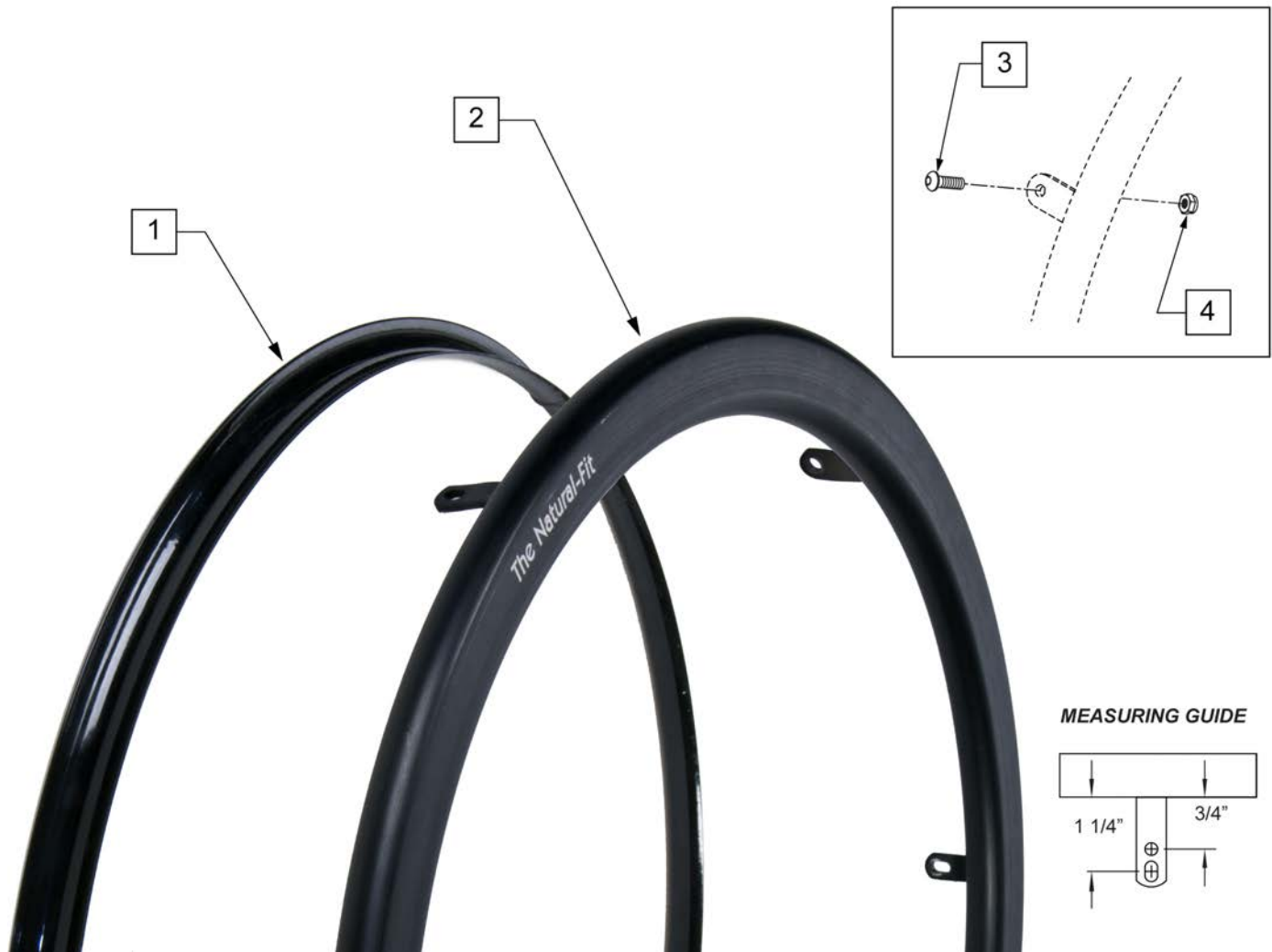


Pos.	Item Number	Description	Remarks
>>>>>	>>>>>>>>>	STD GRIP THUMB GRIP	>>>>>
1	102168-001	GUARD THUMB NAT FIT 22 PC BLK	22" (501)
1	102168-003	GUARD THUMB NAT FIT 24 PC BLK	24" (540)
1	102168-005	GUARD THUMB NAT FIT 25 PC BLK	25" (559)
1	102168-007	GUARD THUMB NAT FIT 26 PC BLK	26" (590)
>>>>>	>>>>>>>>>	SUPER GRIP THUMB GUARD	>>>>>
1	102168-002	GUARD THUMB NAT FIT 22 SG BLK	22" (501)
1	102168-004	GUARD THUMB NAT FIT 24 SG BLK	24" (540)
1	102168-006	GUARD THUMB NAT FIT 25 SG BLK	25" (559)
1	102168-008	THUMB GUARD NAT FIT 26 SG BLK	26" (590)
>>>>>	>>>>>>>>>	NATURAL FIT LITE HANDRIM	>>>>>
2	108859	HAND RIM NAT FT LT TAB 22 BLK	22" (501)
2	108860	HAND RIM NAT FT LT TAB 24 BLK	24" (540)
2	108861	HAND RIM NAT FT LT TAB 25 BLK	25" (559)


Pos.	Item Number	Description	Remarks
2	110981	HAND RIM NAT FIT LT TAB 26 BLK	26" (590)
>>>>>	>>>>>>>>>>	NATURAL FIT -W/O THUMB GRIP	>>>>>
2	102170-001	HAND RIM NAT FIT TAB 22 BLK	22" (501)
2	102170-002	HAND RIM NAT FIT TAB 24 BLK	24" (540)
2	102170-003	HAND RIM NAT FIT TAB 25 BLK	25" (559)
2	102170-004	HAND RIM NAT FIT TAB 26 BLK	26" (590)
3	281110	10-32 X 1/2 PAN PHIL	USED W/ SPINERGYS
4	200100-QU	10-32 NYLOCK NUT	USED W/ SPINERGYS

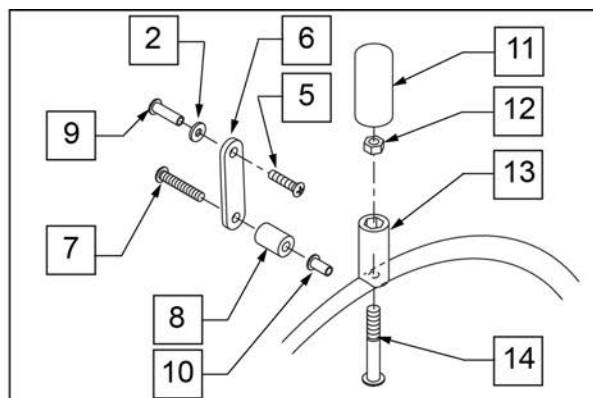

MAG VIEW

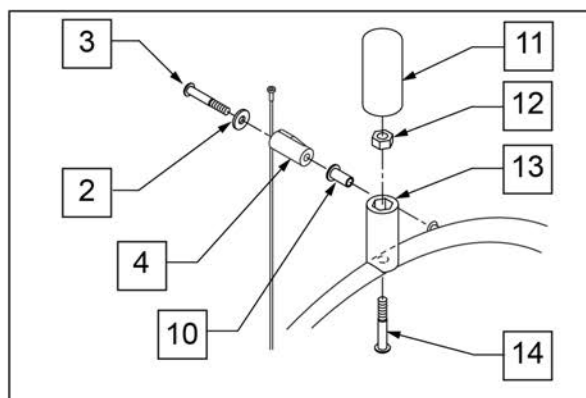

SPOKE VIEW

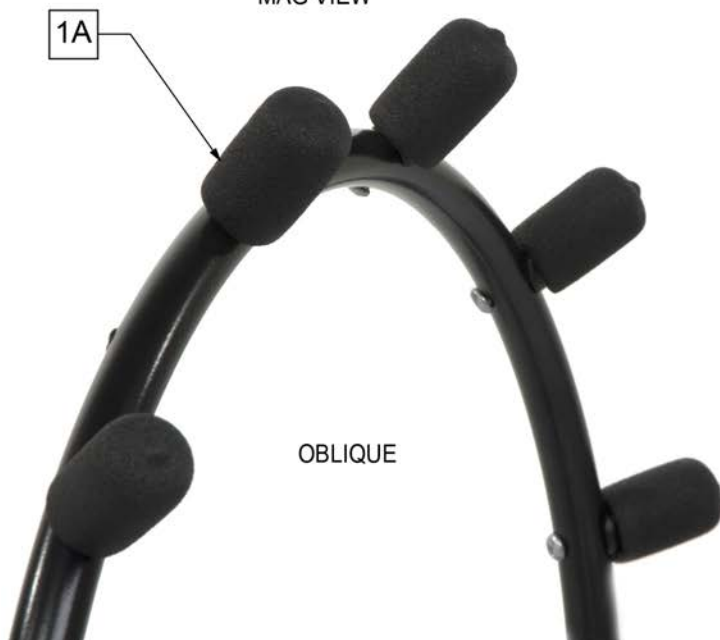


OBLIQUE




VERTICAL

Pos.	Item Number	Description	Remarks
>>>>>	>>>>>>>>>	8" OBLIQUE	>>>>>
1A -4, 10-14	427003	PROJ 24/26" HR 8OBLQ W/HDWR	24" (540), 25" (559), 26" (590) SPOKE
1A, 2, 3-14	427511	PROJ 24" HR 6MAG 8OBLQ W/HDWR	24" (540) MAG
>>>>>	>>>>>>>>>	12" OBLIQUE	>>>>>
1A -4, 10-14	427007	PROJ 24/26" HR 12OBLQ W/HDWR	24" (540), 25" (559), 26" (590) SPOKE
1A,2 5-14	427515	PROJ 24" HR 6MAG 12OBLQ W/HDWR	24" (540) MAG
>>>>>	>>>>>>>>>	8" VERTICAL	>>>>>
1B, 4, 10-14	427002	PROJ 24/26" HR 8VERT W/HDWR	24" (540), 25" (559), 26" (590) SPOKE
1B,2, 5-14	427510	PROJ 24" HR 6 MAG 8 VERT W/ HDWR	24" (540) MAG
>>>>>	>>>>>>>>>	12" VERTICAL	>>>>>
1B,4, 10-14	427006	PROJ 24/26" HR 12VERT W/HDWR	24" (540), 25" (559), 26" (590) SPOKE
1B,2, 5-14	427514	PROJ 24" HR 6MAG 12VERT W/HDWR	24" (540) MAG
2	261300	3/16 WASHER FLAT	
3	281170	10-32 X 1 5/8 PAN PHIL SS	SPOKE ONLY
4	561046	PROJECTION SPOKE MOUNT	SPOKE ONLY
5	281120	10/32 X 5/8 PAN HD PHIL	MAG ONLY
6	505004	TOGGLE 2 - HOLE	MAG ONLY
7	281147	10-32 X 1 1/4 PAN PHIL	MAG ONLY


Pos.	Item Number	Description	Remarks
8	561037	HANDRIM SPACER MAG	MAG ONLY
9	341509	10 -32 x .900 HEX RIVNUT BLK	
10	341497	10 -32 x .020 -.130 RIVNUT GOLD	
11	560600	1 1/2 x .705 x .187 FOAM GRIP	
12	106935	NUT NYLOCK THIN 1/4-28 ROHS	
13	561044	PROJECTION PEG	
14	226155	1 1/4 -28 x 2 1/4 BH C/S	

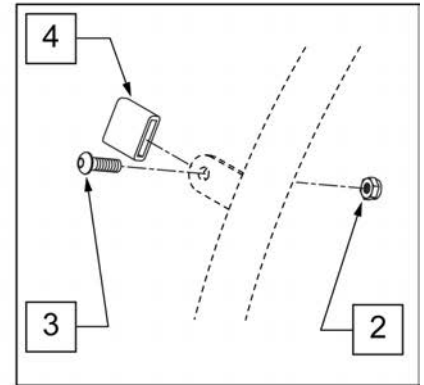


MEASURING GUIDE

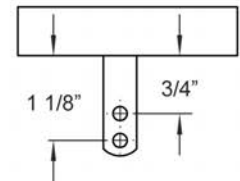

Pos.	Item Number	Description	Remarks
>>>>>	>>>>>>>>>>	PAINTED REG TAB	>>>>>
1	529528	QP 24" HR .065 (6)TAB BLK PNTD	24" (540)
1-4	425041	24" HR (6T)BLK W/X/S HDWR	25" (559)
1	529527	QP 26" HR .065 (6)TAB BLK PNTD	26" (590)
1-4	425047	26" HR (6T)BLK W/X/S HDWR	26" (700)
>>>>>	>>>>>>>>>>	PAINTED SHORT TAB	>>>>>
1	529528	QP 24" HR .065 (6)TAB BLK PNTD	24" (540)
1-4	425044	24" HR SHT(6T)BLK W/X/S HDWR	25" (559)
1	529527	QP 26" HR .065 (6)TAB BLK PNTD	26" (590)
1-4	425053	26" HR SHT(6T)BLK W/X/S HDWR	26" (700)
2	200100-QU	10-32 NYLOCK NUT	
3	281110	10-32 X 1/2 PAN PHIL	
4	580301	HR TAB COVER BLK	

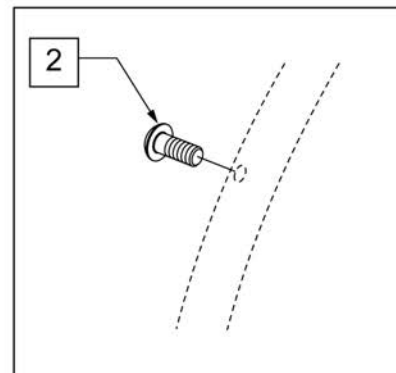


Pos.	Item Number	Description	Remarks
1,2	426026	20" SCW MT HR CHRM 6S W/HDWR	20"
1,2	426024	22" SCW MT HR CHRM 6S W/HDWR	22" (501)
1,2	426011	24 IN SCW MT HR CHRM 6S W/HDWR	24" (540)
2	281175	10-32 X 2 1/8 PAN PHIL SS	QTY 6 REG

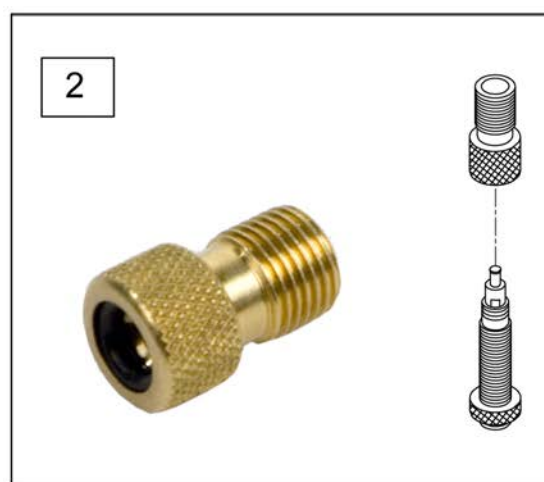

Pos.	Item Number	Description	Remarks
1A	382305	22" X 1" TUBE PRESTA VALVE	
1A	382330	24" TUBE PRESTA VALVE (540MM)	
1A	384545	QP 24" HT INNER TUBE (559MM)	
1A	382355	26" TUBE PRESTA VALVE (590MM)	
1A	384550	QP 26" HT INNER TUBE (700C)	
1B	384736	INNER TUBE 12-1/2" X 2-1/4"	
1B	384738	16" X 1.75" INNER TUBE	
1B	382255	TUBE 18" TIRE	18" X 1-3/4" TUBE
1B	382275	20" TUBE	20" 1-3/8" TUBE
1B	382300	22" TUBE	22" X 1-3/8" TUBE
1B	382336	TUBE PRIMO 24" 25-540 SV	24" X 1" TUBE
1B	382325	24 X 1.3/8IN TUBE REG	
1B	382334	TUBE INLINE 24 X 1.75	
1B	384551	26" X 1" TUBE SCHRADER	
1B	382350	26" TUBE	26" X 1-3/8" TUBE
1B	382361	TUBE 26 X 1.5/1.95/ S/V	
1B	382365	26 X 1.25/1.5 TUBE SHRADER	
1B	382375	24" x 2.125 TUBE	
1C	102972-001	12 1/2 IN AIRLESS INSERT	12-1/2" X 2-1/4"
1C	102973-001	16IN AIRLESS INSERT	16" X 1-3/4"
1C	102973-002	18IN AIRLESS INSERT	18" X 1-3/4"

Pos.	Item Number	Description	Remarks
1C	102973-003	20IN AIRLESS INSERT	20" X 1-3/8"
1C	102972-002	AIRLESS INSERT 20 X 2-1/8 P3	
1C	102973-004	22IN AIRLESS INSERT	22" X 1-3/8"
1C	102973-005	24IN AIRLESS INSERT	24" X 1-3/8"
1C	102972-003	AIRLESS INSERT 24 X 2-1/8	
1C	108259	26" AIRLESS INSERT	26" X 1-3/8"
2	381230	VALVE ADAPTER PRESTA	

NOTE: On all Quickie 2 and GP Family of Chairs an inch deeper cushion is utilized on all seat depths.

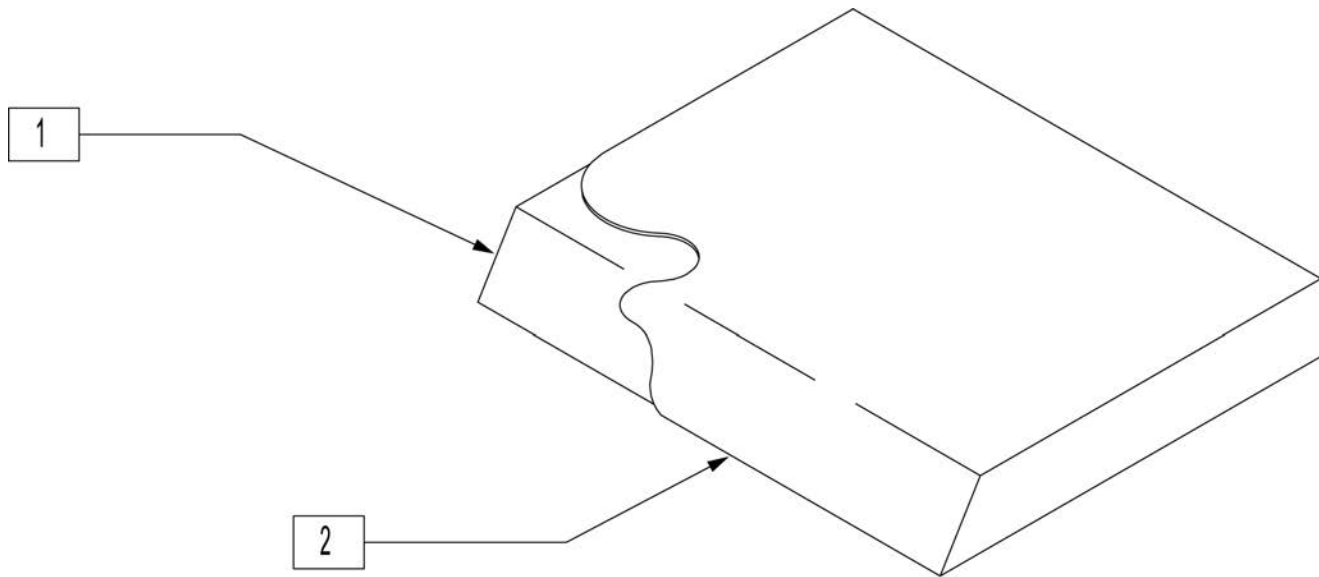

Pos.	Item Number	Description	Pos.	Item Number	Description
>>>>		10" WIDE SEAT CUSHION	1,2	700152	SEAT CUSHION 12" X 15" X 2"
1,2	700025	SEAT CUSHION 10" X 8" X 2"	1,2	700162	SEAT CUSHION 12" X 16" X 2"
1,2	700026	SEAT CUSHION 10" X 9" X 2"	1,2	700172	SEAT CUSHION 12" X 17" X 2"
1,2	700027	SEAT CUSHION 10" X 10" X 2"	1,2	700182	SEAT CUSHION 12" X 18" X 2"
1,2	700028	SEAT CUSHION 10" X 11" X 2"	1,2	700192	SEAT CUSHION 12" X 19" X 2"
1,2	700029	SEAT CUSHION 10" X 12" X 2"	>>>>		13" WIDE SEAT CUSHION
1,2	700035	SEAT CUSHION 10" X 13" X 2"	1,2	700207	SEAT CUSHION 13" X 8" X 2"
1,2	700045	SEAT CUSHION 10" X 14" X 2"	1,2	700208	SEAT CUSHION 13" X 9" X 2"
>>>>		11" WIDE SEAT CUSHION	1,2	700202	SEAT CUSHION 13" X 10" X 2"
1,2	700055	SEAT CUSHION 11" X 8" X 2"	1,2	700212	SEAT CUSHION 13" X 11" X 2"
1,2	700056	SEAT CUSHION 11" X 9" X 2"	1,2	700222	SEAT CUSHION 13" X 12" X 2"
1,2	700002	SEAT CUSHION 11" X 10" X 2"	1,2	700232	SEAT CUSHION 13" X 13" X 2"
1,2	700012	SEAT CUSHION 11" X 11" X 2"	1,2	700242	SEAT CUSHION 13" X 14" X 2"
1,2	700022	SEAT CUSHION 11" X 12" X 2"	1,2	700252	SEAT CUSHION 13" X 15" X 2"
1,2	700032	SEAT CUSHION 11" X 13" X 2"	1,2	700262	SEAT CUSHION 13" X 16" X 2"
1,2	700042	SEAT CUSHION 11" X 14" X 2"	1,2	700272	SEAT CUSHION 13" X 17" X 2"
1,2	700052	SEAT CUSHION 11" X 15" X 2"	1,2	700277	SEAT CUSHION 13" X 18" X 2"
1,2	700062	SEAT CUSHION 11" X 16" X 2"	1,2	700267	SEAT CUSHION 13" X 19" X 2"
1,2	700072	SEAT CUSHION 11" X 17" X 2"	>>>>		14" WIDE SEAT CUSHION
1,2	700082	SEAT CUSHION 11" X 18" X 2"	1,2	700287	SEAT CUSHION 14" X 8" X 2"
>>>>		12" WIDE SEAT CUSHION	1,2	700288	SEAT CUSHION 14" X 9" X 2"
1,2	700107	SEAT CUSHION 12" X 8" X 2"	1,2	700282	SEAT CUSHION 14" X 10" X 2"
1,2	700108	SEAT CUSHION 12" X 9" X 2"	1,2	700292	SEAT CUSHION 14" X 11" X 2"
1,2	700102	SEAT CUSHION 12" X 10" X 2"	1,2	700302	SEAT CUSHION 14" X 12" X 2"
1,2	700112	SEAT CUSHION 12" X 11" X 2"	1,2	700312	SEAT CUSHION 14" X 13" X 2"
1,2	700122	SEAT CUSHION 12" X 12" X 2"	1,2	700322	SEAT CUSHION 14" X 14" X 2"
1,2	700132	SEAT CUSHION 12" X 13" X 2"	1,2	700332	SEAT CUSHION 14" X 15" X 2"
1,2	700142	SEAT CUSHION 12" X 14" X 2"	1,2	700342	SEAT CUSHION 14" X 16" X 2"

Pos.	Item Number	Description	Pos.	Item Number	Description
1,2	700352	SEAT CUSHION 14" X 17" X 2"	1,2	700772	SEAT CUSHION 18" X 21" X 2"
1,2	700362	SEAT CUSHION 14" X 18" X 2"	1,2	702114	SEAT CUSHION 18" X 22" X 2"
1,2	700372	SEAT CUSHION 14" X 19" X 2"	>>>>		19" WIDE SEAT CUSHION
1,2	700382	SEAT CUSHION 14" X 20" X 2"	1,2	700782	SEAT CUSHION 19" X 12" X 2"
1,2	700392	SEAT CUSHION 14" X 21" X 2"	1,2	700792	SEAT CUSHION 19" X 13" X 2"
1,2	702110	SEAT CUSHION 14" X 22" X 2"	1,2	700802	SEAT CUSHION 19" X 14" X 2"
>>>>		15" WIDE SEAT CUSHION	1,2	700812	SEAT CUSHION 19" X 15" X 2"
1,2	700398	SEAT CUSHION 15" X 11" X 2"	1,2	700822	SEAT CUSHION 19" X 16" X 2"
1,2	700402	SEAT CUSHION 15" X 12" X 2"	1,2	700832	SEAT CUSHION 19" X 17" X 2"
1,2	700412	SEAT CUSHION 15" X 13" X 2"	1,2	700842	SEAT CUSHION 19" X 18" X 2"
1,2	700422	SEAT CUSHION 15" X 14" X 2"	1,2	700852	SEAT CUSHION 19" X 19" X 2"
1,2	700432	SEAT CUSHION 15" X 15" X 2"	1,2	700862	SEAT CUSHION 19" X 20" X 2"
1,2	700442	SEAT CUSHION 15" X 16" X 2"	1,2	700872	SEAT CUSHION 19" X 21" X 2"
1,2	700452	SEAT CUSHION 15" X 17" X 2"	1,2	702115	SEAT CUSHION 19" X 22" X 2"
1,2	700462	SEAT CUSHION 15" X 18" X 2"	>>>>		20" WIDE SEAT CUSHION
1,2	700472	SEAT CUSHION 15" X 19" X 2"	1,2	700882	SEAT CUSHION 20" X 12" X 2"
1,2	700482	SEAT CUSHION 15" X 20" X 2"	1,2	700892	SEAT CUSHION 20" X 13" X 2"
1,2	700492	SEAT CUSHION 15" X 21" X 2"	1,2	700902	SEAT CUSHION 20" X 14" X 2"
1,2	702111	SEAT CUSHION 15" X 22" X 2"	1,2	700912	SEAT CUSHION 20" X 15" X 2"
>>>>		16" WIDE SEAT CUSHION	1,2	700922	SEAT CUSHION 20" X 16" X 2"
1,2	700498	SEAT CUSHION 16" X 11" X 2"	1,2	700932	SEAT CUSHION 20" X 17" X 2"
1,2	700502	SEAT CUSHION 16" X 12" X 2"	1,2	700942	SEAT CUSHION 20" X 18" X 2"
1,2	700512	SEAT CUSHION 16" X 13" X 2"	1,2	700952	SEAT CUSHION 20" X 19" X 2"
1,2	700522	SEAT CUSHION 16" X 14" X 2"	1,2	700962	SEAT CUSHION 20" X 20" X 2"
1,2	700532	SEAT CUSHION 16" X 15" X 2"	1,2	700972	SEAT CUSHION 20" X 21" X 2"
1,2	700542	SEAT CUSHION 16" X 16" X 2"	1,2	702116	SEAT CUSHION 20" X 22" X 2"
1,2	700552	SEAT CUSHION 16" X 17" X 2"	>>>>		21" WIDE SEAT CUSHION
1,2	700562	SEAT CUSHION 16" X 18" X 2"	1,2	702080	SEAT CUSHION 21" X 13" X 2"
1,2	700572	SEAT CUSHION 16" X 19" X 2"	1,2	702081	SEAT CUSHION 21" X 14" X 2"
1,2	700582	SEAT CUSHION 16" X 20" X 2"	1,2	702082	SEAT CUSHION 21" X 15" X 2"
1,2	700592	SEAT CUSHION 16" X 21" X 2"	1,2	702083	SEAT CUSHION 21" X 16" X 2"
1,2	702112	SEAT CUSHION 16" X 22" X 2"	1,2	702084	SEAT CUSHION 21" X 17" X 2"
>>>>		17" WIDE SEAT CUSHION	1,2	702085	SEAT CUSHION 21" X 18" X 2"
1,2	700585	SEAT CUSHION 17" X 11" X 2"	1,2	702086	SEAT CUSHION 21" X 19" X 2"
1,2	700606	SEAT CUSHION 17" X 12" X 2"	1,2	702087	SEAT CUSHION 21" X 20" X 2"
1,2	700597	SEAT CUSHION 17" X 13" X 2"	1,2	702088	SEAT CUSHION 21" X 21" X 2"
1,2	700602	SEAT CUSHION 17" X 14" X 2"	1,2	702089	SEAT CUSHION 21" X 22" X 2"
1,2	700606	SEAT CUSHION 17" X 12" X 2"	>>>>		22" WIDE SEAT CUSHION
1,2	700612	SEAT CUSHION 17" X 15" X 2"	1,2	702090	SEAT CUSHION 22" X 13" X 2"
1,2	700622	SEAT CUSHION 17" X 16" X 2"	1,2	702091	SEAT CUSHION 22" X 14" X 2"
1,2	700632	SEAT CUSHION 17" X 17" X 2"	1,2	702092	SEAT CUSHION 22" X 15" X 2"
1,2	700642	SEAT CUSHION 17" X 18" X 2"	1,2	702093	SEAT CUSHION 22" X 16" X 2"
1,2	700652	SEAT CUSHION 17" X 19" X 2"	1,2	702094	SEAT CUSHION 22" X 17" X 2"
1,2	700662	SEAT CUSHION 17" X 20" X 2"	1,2	702095	SEAT CUSHION 22" X 18" X 2"
1,2	700672	SEAT CUSHION 17" X 21" X 2"	1,2	702096	SEAT CUSHION 22" X 19" X 2"
1,2	702113	SEAT CUSHION 17" X 22" X 2"	1,2	702097	SEAT CUSHION 22" X 20" X 2"
>>>>		18" WIDE SEAT CUSHION	1,2	702098	SEAT CUSHION 22" X 21" X 2"
1,2	700682	SEAT CUSHION 18" X 12" X 2"	1,2	702117	SEAT CUSHION 22" X 22" X 2"
1,2	700692	SEAT CUSHION 18" X 13" X 2"	>>>>		24" WIDE SEAT CUSHION
1,2	700702	SEAT CUSHION 18" X 14" X 2"	1,2	702099	SEAT CUSHION 24" X 14" X 2
1,2	700712	SEAT CUSHION 18" X 15" X 2"	1,2	702100	SEAT CUSHION 24" X 15" X 2
1,2	700722	SEAT CUSHION 18" X 16" X 2"	1,2	702101	SEAT CUSHION 24" X 16" X 2
1,2	700732	SEAT CUSHION 18" X 17" X 2"	1,2	702102	SEAT CUSHION 24" X 17" X 2
1,2	700742	SEAT CUSHION 18" X 18" X 2"	1,2	702103	SEAT CUSHION 24" X 18" X 2
1,2	700752	SEAT CUSHION 18" X 19" X 2"	1,2	702104	SEAT CUSHION 24" X 19" X 2
1,2	700762	SEAT CUSHION 18" X 20" X 2"	1,2	702105	SEAT CUSHION 24" X 20" X 2"

Pos.	Item Number	Description	Pos.	Item Number	Description
1,2	702106	SEAT CUSHION 24" X 21" X 2"	1,2	702226	SEAT CUSHION 28" X 19" X 2"
1,2	702118	SEAT CUSHION 24" X 22" X 2"	1,2	702227	SEAT CUSHION 28" X 20" X 2"
>>>>		26" WIDE SEAT CUSHION	1,2	702228	SEAT CUSHION 28" X 21" X 2"
1,2	702220	SEAT CUSHION 26" X 18" X 2"	1,2	702229	SEAT CUSHION 28" X 22" X 2"
1,2	702221	SEAT CUSHION 26" X 19" X 2"	>>>>		30" WIDE SEAT CUSHION
1,2	702222	SEAT CUSHION 26" X 20" X 2"	1,2	702230	SEAT CUSHION 30" X 18" X 2"
1,2	702223	SEAT CUSHION 26" X 21" X 2"	1,2	702231	SEAT CUSHION 30" X 19" X 2"
1,2	702224	SEAT CUSHION 26" X 22" X 2"	1,2	702232	SEAT CUSHION 30" X 20" X 2"
>>>>		28" WIDE SEAT CUSHION	1,2	702233	SEAT CUSHION 30" X 21" X 2"
1,2	702225	SEAT CUSHION 28" X 18" X 2"	1,2	702234	SEAT CUSHION 30" X 22" X 2"


NOTE: Effective on QUICKIE 2 chairs after S/N Q2-242050 (10/23/99).

NOTE: Effective on QUICKIE 2 HD chairs with serial number prefix Q2H

NOTE: Effective on LX/LXI chairs after S/N LX-034330 (10/24/08).

NOTE: On Q2 chairs (S/N prefix Q2 & Q2H) the Rail Rod diameter is .25" which is smaller than Q2 chairs (S/N prefix Q2V) and Q2 Lite in which the Rail Rod diameter is 8mm (.31")

MEASURING GUIDE

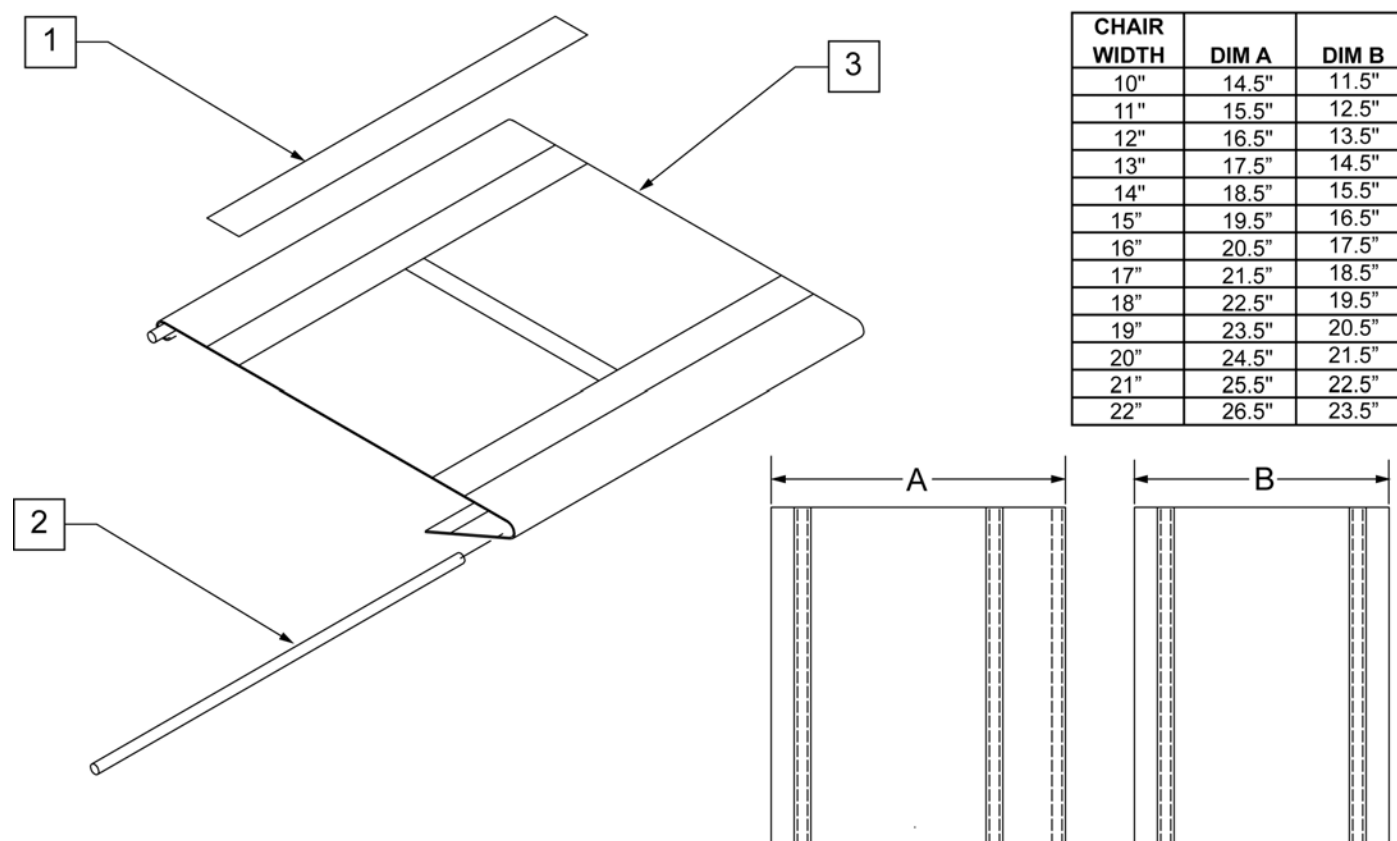


Pos.	Item Number	Description	Pos.	Item Number	Description
1	901169	SEAT SLING VELCRO® STRIP 2"	2	580230	SEAT SLING RETAINING ROD 14"
2	580226	SEAT SLING RETAINING ROD 10"	2	580231	SEAT SLING RETAINING ROD 15"
2	580227	SEAT SLING RETAINING ROD 11"	2	580232	SEAT SLING RETAINING ROD 16"
2	580228	SEAT SLING RETAINING ROD 12"	2	580233	SEAT SLING RETAINING ROD 17"
2	580229	SEAT SLING RETAINING ROD 13"	2	580234	SEAT SLING RETAINING ROD 18"

Pos.	Item Number	Description	Pos.	Item Number	Description
2	580235	SEAT SLING RETAINING ROD 19"	3	722633	SEAT SLING 15" X 17" CHNL
2	580236	SEAT SLING RETAINING ROD 20"	3	722634	SEAT SLING 15" X 18" CHNL
		10" Wide Seat Slings	3	722635	SEAT SLING 15" X 19" CHNL
3	722489	SEAT SLING 10" X 12" CHNL	3	722636	SEAT SLING 15" X 20" CHNL
3	722490	SEAT SLING 10" X 13" CHNL			16" Wide Seat Slings
3	722491	SEAT SLING 10" X 14" CHNL	3	722704	SEAT SLING 16" X 10" CHNL
3	722492	SEAT SLING 10" X 15" CHNL	3	722705	SEAT SLING 16" X 11" CHNL
3	722493	SEAT SLING 10" X 16" CHNL	3	722706	SEAT SLING 16" X 12" CHNL
		11" Wide Seat Slings	3	722707	SEAT SLING 16" X 13" CHNL
3	722580	SEAT SLING 11" X 10" CHNL	3	722640	SEAT SLING 16" X 14" CHNL
3	722581	SEAT SLING 11" X 11" CHNL	3	722641	SEAT SLING 16" X 15" CHNL
3	722582	SEAT SLING 11" X 12" CHNL	3	722642	SEAT SLING 16" X 16" CHNL
3	722583	SEAT SLING 11" X 13" CHNL	3	722643	SEAT SLING 16" X 17" CHNL
3	722584	SEAT SLING 11" X 14" CHNL	3	722644	SEAT SLING 16" X 18" CHNL
3	722585	SEAT SLING 11" X 15" CHNL	3	722645	SEAT SLING 16" X 19" CHNL
3	722586	SEAT SLING 11" X 16" CHNL	3	722646	SEAT SLING 16" X 20" CHNL
3	722587	SEAT SLING 11" X 17" CHNL			17" Wide Seat Slings
		12" Wide Seat Slings	3	722708	SEAT SLING 17" X 10" CHNL
3	722590	SEAT SLING 12" X 10" CHNL	3	722709	SEAT SLING 17" X 11" CHNL
3	722591	SEAT SLING 12" X 11" CHNL	3	722710	SEAT SLING 17" X 12" CHNL
3	722592	SEAT SLING 12" X 12" CHNL	3	722711	SEAT SLING 17" X 13" CHNL
3	722593	SEAT SLING 12" X 13" CHNL	3	722650	SEAT SLING 17" X 14" CHNL
3	722594	SEAT SLING 12" X 14" CHNL	3	722651	SEAT SLING 17" X 15" CHNL
3	722595	SEAT SLING 12" X 15" CHNL	3	722652	SEAT SLING 17" X 16" CHNL
3	722596	SEAT SLING 12" X 16" CHNL	3	722653	SEAT SLING 17" X 17" CHNL
3	722597	SEAT SLING 12" X 17" CHNL	3	722654	SEAT SLING 17" X 18" CHNL
		13" Wide Seat Slings	3	722655	SEAT SLING 17" X 19" CHNL
3	722600	SEAT SLING 13" X 10" CHNL	3	722656	SEAT SLING 17" X 20" CHNL
3	722601	SEAT SLING 13" X 11" CHNL			18" Wide Seat Slings
3	722602	SEAT SLING 13" X 12" CHNL	3	722667	SEAT SLING 18" X 11" CHNL
3	722603	SEAT SLING 13" X 13" CHNL	3	722668	SEAT SLING 18" X 12" CHNL
3	722604	SEAT SLING 13" X 14" CHNL	3	722669	SEAT SLING 18" X 13" CHNL
3	722605	SEAT SLING 13" X 15" CHNL	3	722660	SEAT SLING 18" X 14" CHNL
3	722606	SEAT SLING 13" X 16" CHNL	3	722661	SEAT SLING 18" X 15" CHNL
3	722607	SEAT SLING 13" X 17" CHNL	3	722662	SEAT SLING 18" X 16" CHNL
3	722608	SEAT SLING 13" X 18" CHNL	3	722663	SEAT SLING 18" X 17" CHNL
		14" Wide Seat Slings	3	722664	SEAT SLING 18" X 18" CHNL
3	722616	SEAT SLING 14" X 10" CHNL	3	722665	SEAT SLING 18" X 19" CHNL
3	722617	SEAT SLING 14" X 11" CHNL	3	722666	SEAT SLING 18" X 20" CHNL
3	722618	SEAT SLING 14" X 12" CHNL			19" Wide Seat Slings
3	722619	SEAT SLING 14" X 13" CHNL	3	722677	SEAT SLING 19" X 11" CHNL
3	722620	SEAT SLING 14" X 14" CHNL	3	722678	SEAT SLING 19" X 12" CHNL
3	722621	SEAT SLING 14" X 15" CHNL	3	722679	SEAT SLING 19" X 13" CHNL
3	722622	SEAT SLING 14" X 16" CHNL	3	722670	SEAT SLING 19" X 14" CHNL
3	722623	SEAT SLING 14" X 17" CHNL	3	722671	SEAT SLING 19" X 15" CHNL
3	722624	SEAT SLING 14" X 18" CHNL	3	722672	SEAT SLING 19" X 16" CHNL
3	722625	SEAT SLING 14" X 19" CHNL	3	722673	SEAT SLING 19" X 17" CHNL
3	722626	SEAT SLING 14" X 20" CHNL	3	722674	SEAT SLING 19" X 18" CHNL
		15" Wide Seat Slings	3	722675	SEAT SLING 19" X 19" CHNL
3	722700	SEAT SLING 15" X 10" CHNL	3	722676	SEAT SLING 19" X 20" CHNL
3	722701	SEAT SLING 15" X 11" CHNL			20" Wide Seat Slings
3	722702	SEAT SLING 15" X 12" CHNL	3	722687	SEAT SLING 20" X 11" CHNL
3	722703	SEAT SLING 15" X 13" CHNL	3	722688	SEAT SLING 20" X 12" CHNL
3	722630	SEAT SLING 15" X 14" CHNL	3	722689	SEAT SLING 20" X 13" CHNL
3	722631	SEAT SLING 15" X 15" CHNL	3	722680	SEAT SLING 20" X 14" CHNL
3	722632	SEAT SLING 15" X 16" CHNL	3	722681	SEAT SLING 20" X 15" CHNL

Pos.	Item Number	Description	Pos.	Item Number	Description
3	722682	SEAT SLING 20" X 16" CHNL	3	722727	SEAT SLING 21" X 19" CHNL
3	722683	SEAT SLING 20" X 17" CHNL	3	722728	SEAT SLING 21" X 20" CHNL
3	722684	SEAT SLING 20" X 18" CHNL			22" Wide Seat Slings
3	722685	SEAT SLING 20" X 19" CHNL	3	722730	SEAT SLING 22" X 12" CHNL
3	722686	SEAT SLING 20" X 20" CHNL	3	722731	SEAT SLING 22" X 13" CHNL
		21" Wide Seat Slings	3	722732	SEAT SLING 22" X 14" CHNL
3	722720	SEAT SLING 21" X 12" CHNL	3	722733	SEAT SLING 22" X 15" CHNL
3	722721	SEAT SLING 21" X 13" CHNL	3	722734	SEAT SLING 22" X 16" CHNL
3	722722	SEAT SLING 21" X 14" CHNL	3	722735	SEAT SLING 22" X 17" CHNL
3	722723	SEAT SLING 21" X 15" CHNL	3	722736	SEAT SLING 22" X 18" CHNL
3	722724	SEAT SLING 21" X 16" CHNL	3	722737	SEAT SLING 22" X 19" CHNL
3	722725	SEAT SLING 21" X 17" CHNL	3	722738	SEAT SLING 22" X 20" CHNL
3	722726	SEAT SLING 21" X 18" CHNL			


Note: When replacing the Seat Sling it is recommended to order the Seat Rail End Plug Kit.

Note: On Quickie 2 chairs with S/N prefix (Q2V) & Quickie 2 Lite chairs the diameter of the Seat Rail Rod is 8mm (.31") which is larger than the Q2 chairs with S/N prefix (Q2 & Q2H) which is .25" in diameter.

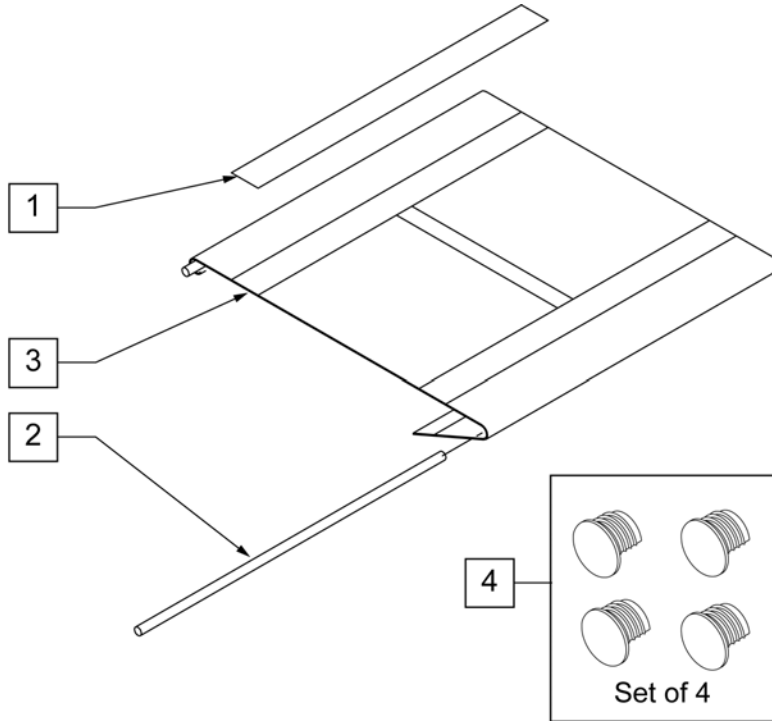

MEASURING GUIDE


CHAIR WIDTH	DIM A	DIM B
12"	16.5"	13.5"
13"	17.5"	14.5"
14"	18.5"	15.5"
15"	19.5"	16.5"
16"	20.5"	17.5"
17"	21.5"	18.5"
18"	22.5"	19.5"
19"	23.5"	20.5"
20"	24.5"	21.5"

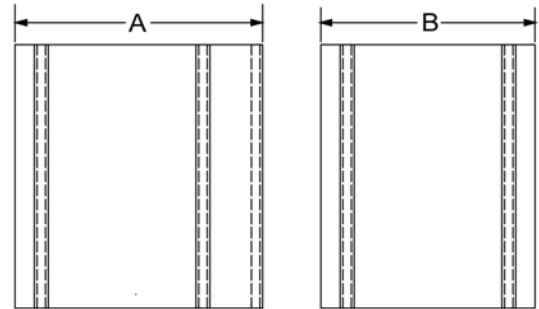

Pos.	Item Number	Description	Remarks	Pos.	Item Number	Description	Remarks
1	901169	SEAT SLING VELCRO STRIP 2"		3	722603	SEAT SLING 13" X 13" CHNL	
2	106584	SEAT RAIL ROD 12"		3	722604	SEAT SLING 13" X 14" CHNL	
2	106585	SEAT RAIL ROD 13"		3	722605	SEAT SLING 13" X 15" CHNL	
2	106586	SEAT RAIL ROD 14"		3	722606	SEAT SLING 13" X 16" CHNL	
2	106587	SEAT RAIL ROD 15"		3	722607	SEAT SLING 13" X 17" CHNL	
2	106588	SEAT RAIL ROD 16"		3	722608	SEAT SLING 13" X 18" CHNL	
2	106589	SEAT RAIL ROD 17"				14" Seat Slings	
2	106590	SEAT RAIL ROD 18"		3	722616	SEAT SLING 14" X 10" CHNL	
2	106591	SEAT RAIL ROD 19"		3	722617	SEAT SLING 14" X 11" CHNL	
2	106592	SEAT RAIL ROD 20"		3	722618	SEAT SLING 14" X 12" CHNL	
		12" Wide Seat Slings		3	722619	SEAT SLING 14" X 13" CHNL	
3	722590	SEAT SLING 12" X 10" CHNL		3	722620	SEAT SLING 14" X 14" CHNL	
3	722591	SEAT SLING 12" X 11" CHNL		3	722621	SEAT SLING 14" X 15" CHNL	
3	722592	SEAT SLING 12" X 12" CHNL		3	722622	SEAT SLING 14" X 16" CHNL	
3	722593	SEAT SLING 12" X 13" CHNL		3	722623	SEAT SLING 14" X 17" CHNL	
3	722594	SEAT SLING 12" X 14" CHNL		3	722624	SEAT SLING 14" X 18" CHNL	
3	722595	SEAT SLING 12" X 15" CHNL		3	722625	SEAT SLING 14" X 19" CHNL	
3	722596	SEAT SLING 12" X 16" CHNL		3	722626	SEAT SLING 14" X 20" CHNL	
3	722597	SEAT SLING 12" X 17" CHNL				15" Seat Slings	
		13" Wide Seat Slings		3	722630	SEAT SLING 15" X 14" CHNL	
3	722600	SEAT SLING 13" X 10" CHNL		3	722631	SEAT SLING 15" X 15" CHNL	
3	722601	SEAT SLING 13" X 11" CHNL		3	722632	SEAT SLING 15" X 16" CHNL	
3	722602	SEAT SLING 13" X 12" CHNL		3	722633	SEAT SLING 15" X 17" CHNL	


Pos.	Item Number	Description	Remarks	Pos.	Item Number	Description	Remarks
3	722634	SEAT SLING 15" X 18" CHNL		3	722662	SEAT SLING 18" X 16" CHNL	
3	722635	SEAT SLING 15" X 19" CHNL		3	722663	SEAT SLING 18" X 17" CHNL	
3	722636	SEAT SLING 15" X 20" CHNL		3	722664	SEAT SLING 18" X 18" CHNL	
		16" Seat Slings		3	722665	SEAT SLING 18" X 19" CHNL	
3	722640	SEAT SLING 16" X 14" CHNL		3	722666	SEAT SLING 18" X 20" CHNL	
3	722641	SEAT SLING 16" X 15" CHNL				19" Seat Slings	
3	722642	SEAT SLING 16" X 16" CHNL		3	722670	SEAT SLING 19" X 14" CHNL	
3	722643	SEAT SLING 16" X 17" CHNL		3	722671	SEAT SLING 19" X 15" CHNL	
3	722644	SEAT SLING 16" X 18" CHNL		3	722672	SEAT SLING 19" X 16" CHNL	
3	722645	SEAT SLING 16" X 19" CHNL		3	722673	SEAT SLING 19" X 17" CHNL	
3	722646	SEAT SLING 16" X 20" CHNL		3	722674	SEAT SLING 19" X 18" CHNL	
		17" Seat Slings		3	722675	SEAT SLING 19" X 19" CHNL	
3	722650	SEAT SLING 17" X 14" CHNL		3	722676	SEAT SLING 19" X 20" CHNL	
3	722651	SEAT SLING 17" X 15" CHNL				20" Seat Slings	
3	722652	SEAT SLING 17" X 16" CHNL		3	722680	SEAT SLING 20" X 14" CHNL	
3	722653	SEAT SLING 17" X 17" CHNL		3	722681	SEAT SLING 20" X 15" CHNL	
3	722654	SEAT SLING 17" X 18" CHNL		3	722682	SEAT SLING 20" X 16" CHNL	
3	722655	SEAT SLING 17" X 19" CHNL		3	722683	SEAT SLING 20" X 17" CHNL	
3	722656	SEAT SLING 17" X 20" CHNL		3	722684	SEAT SLING 20" X 18" CHNL	
		18" Seat Slings		3	722685	SEAT SLING 20" X 19" CHNL	
3	722660	SEAT SLING 18" X 14" CHNL		3	722686	SEAT SLING 20" X 20" CHNL	
3	722661	SEAT SLING 18" X 15" CHNL		4	108117	SEAT RAIL END PLUG KIT	Set of 4

NOTE: Stabilizer bar is sold separately and can be found on the "BACKREST STABILIZER BAR" page.

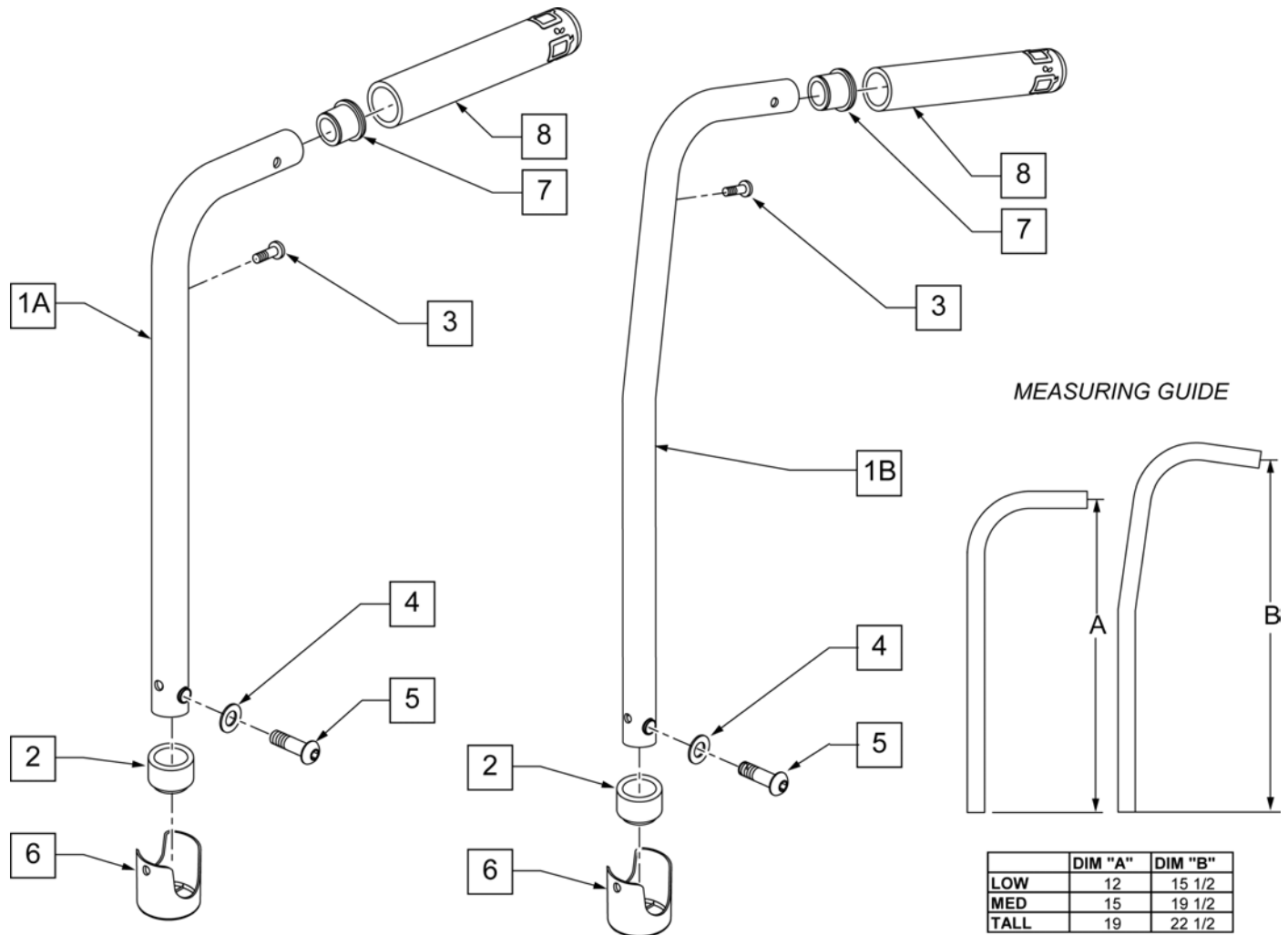

Pos.	Item Number	Description	Remarks
2	561083	1 1/8" TUBE END BUSHING	
3	107384	PAN HD PHIL 10/32 x 5/8 ROHS	
4	104414	WAVE WASHER 6MM	
5	106960	BHCS 1/4 -28 x 7/8 ROHS	
6	106919	BUSHING BACKREST 7/8 OD NF	
7	561007	7/8" INSERT PLUG .083	
8	920024	QUICKIE GRIP 7/8	
1A,2,7,8	107341	TUBE BACK PH ASSY 7/8 LOW RH	RIGHT-LOW STD PUSH HANDLE
1A,2,7,8	107342	TUBE BACK PH ASSY 7/8 LOW LH	LEFT -LOW STD PUSH HANDLE
1A,2,7,8	107343	TUBE BACK PH ASSY 7/8 MED RH	RIGHT-MED STD PUSH HANDLE
1A,2,7,8	107344	TUBE BACK PH ASSY 7/8 MED LH	LEFT -MED STD PUSH HANDLE
1A,2,7,8	107345	TUBE BACK PH ASSY 7/8 TALL RH	RIGHT-TALL STD PUSH HANDLE
1A,2,7,8	107346	TUBE BACK PH ASSY 7/8 TALL LH	LEFT -TALL STD PUSH HANDLE
1A,2	109584	TUBE BACK PH ASSY 7/8 LOW RH STABILIZER	RIGHT -LOW USED W/ STABILIZER BAR
1A,2	109585	TUBE BACK PH ASSY 7/8 LOW LH STABILIZER	LEFT -LOW USED W/ STABILIZER BAR
1A,2	109586	TUBE BACK PH ASSY 7/8 MED RH STABILIZER	RIGHT -MED USED W/ STABILIZER BAR

Pos.	Item Number	Description	Remarks
1A,2	109587	TUBE BACK PH ASSY 7/8 MED LH STABILIZER	LEFT -MED USED W/ STABILIZER BAR
1A,2	109588	TUBE BACK PH ASSY 7/8 TALL RH STABILIZER	RIGHT -TALL USED W/ STABILIZER BAR
1A,2	109589	TUBE BACK PH ASSY 7/8 TALL LH STABILIZER	LEFT -TALL USED W/ STABILIZER BAR
1B,2,7,8	107333	TUBE BACK 8° PH ASSM 7/8 MED RH	RIGHT MED 8° PUSH HANDLE
1B,2,7,8	107334	TUBE BACK 8° PH ASSM 7/8 MED LH	LEFT MED 8° PUSH HANDLE
1B,2,7,8	107335	TUBE BACK 8° PH ASSM 7/8 TALL RH	RIGHT TALL 8° PUSH HANDLE
1B,2,7,8	107336	TUBE BACK 8° PH ASSM 7/8 TALL LH	LEFT TALL 8° PUSH HANDLE
1B,2	109590	TUBE BACK 8° PH ASSM 7/8 MED STABILIZER RH	RIGHT MED 8° PUSH HANDLE USED W/ STABILIZER BAR
1B,2	109591	TUBE BACK 8° PH ASSM 7/8 MED STABILIZER LH	LEFT MED 8° PUSH HANDLE USED W/ STABILIZER BAR
1B,2	109592	TUBE BACK 8° PH ASSM 7/8 TALL STABILIZER RH	RIGHT TALL 8° PUSH HANDLE USED W/ STABILIZER BAR
1B,2	109593	TUBE BACK 8° PH ASSM 7/8 TALL STABILIZER LH	LEFT TALL 8° PUSH HANDLE USED W/ STABILIZER BAR

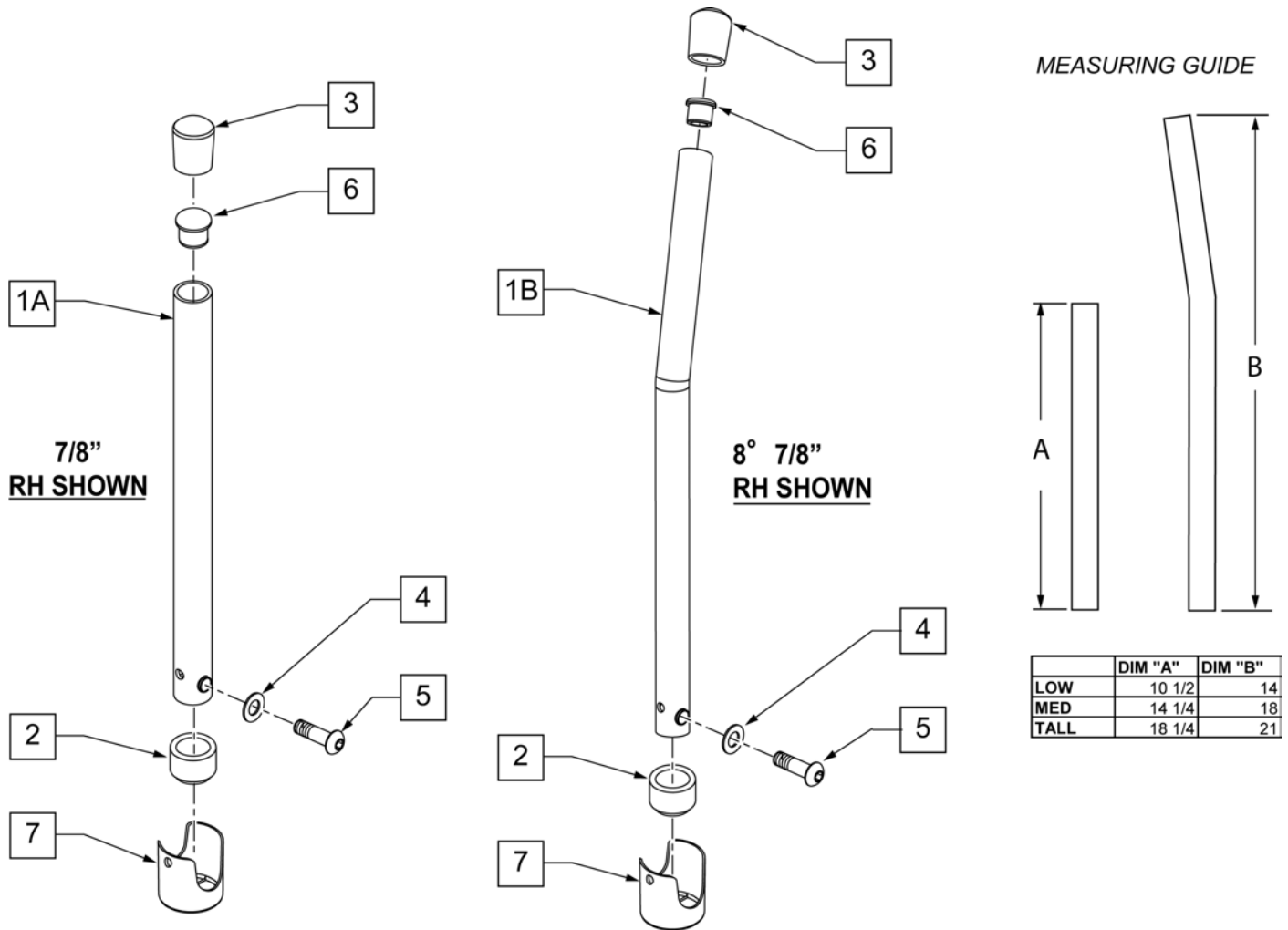


Pos.	Item Number	Description	Remarks
2	561083	1 1/8" TUBE END BUSHING	
3	561013	7/8" CAP	
4	104414	WAVE WASHER 6MM	
5	106960	BHCS 1/4 -28 x 7/8 ROHS	
6	561007	7/8" INSERT PLUG .083	
7	106919	BUSHING BACKREST 7/8 OD NF	
1A-3,6	107327	TUBE BACK ASSY 7/8 LOW RH	RIGHT LOW STD NON-PUSH HANDLE (OMIT PH)
1A-3,6	107328	TUBE BACK ASSY 7/8 LOW LH	LEFT LOW STD NON-PUSH HANDLE (OMIT PH)
1A-3,6	107329	TUBE BACK ASSY 7/8 MED RH	RIGHT MED STD NON-PUSH HANDLE (OMIT PH)
1A-3,6	107330	TUBE BACK ASSY 7/8 MED LH	LEFT MED STD NON-PUSH HANDLE (OMIT PH)
1A-3,6	107331	TUBE BACK ASSY 7/8 TALL RH	RIGHT TALL STD NON-PUSH HANDLE (OMIT PH)
1A-3,6	107332	TUBE BACK ASSY 7/8 TALL LH	LEFT TALL STD NON-PUSH HANDLE (OMIT PH)
1B-3,6	107337	TUBE BACK 8° ASSM MED RH	RIGHT MED 8° NON-PUSH HANDLE (OMIT PH)
1B-3,6	107338	TUBE BACK 8° ASSM MED LH	LEFT MED 8° NON-PUSH HANDLE (OMIT PH)
1B-3,6	107339	TUBE BACK 8° ASSM TALL RH	RIGHT TALL 8° NON-PUSH HANDLE (OMIT PH)
1B-3,6	107340	TUBE BACK 8° ASSM TALL LH	LEFT TALL 8° DEG NON-PUSH HANDLE (OMIT PH)


NOTE: Q2 Right Side Shown.

NOTE: Effective on LX/LXI 10/24/2008.

NOTE: Position #9 Adjustable Back Bracket is 1 1/8" diameter.

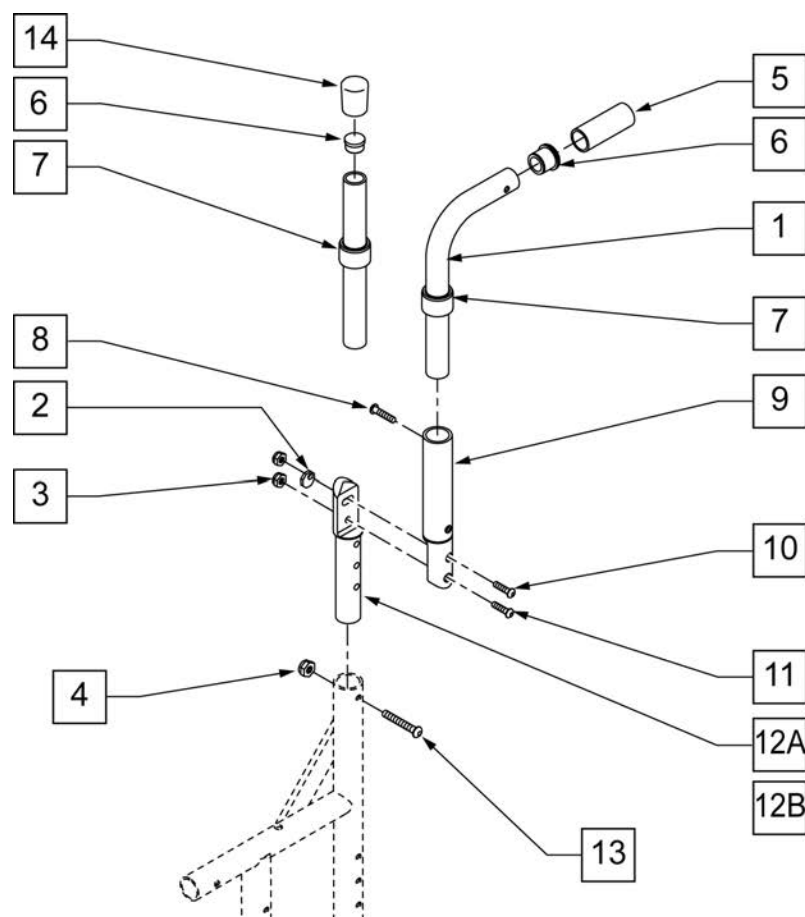

MEASURING GUIDE

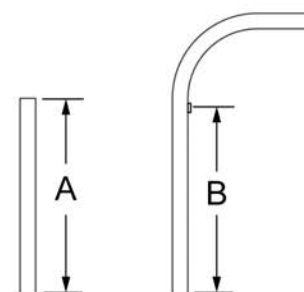


	DIM "A"	DIM "B"
LOW	6	4 3/8
MED	9	7 3/8
HIGH	11	9 3/8

Pos.	Item Number	Description	Remarks
2	341017	WASHER ECCENTRIC	
3	202300	1/4-28 NYLOCK NUT THIN BLK	
4	106934	1/4-28 NYLOCK NUT	
5	920024	QUICKIE GRIP 7/8	
7	561083	1 1/8 TUBE END BUSHING THIN	
6	561007	7/8 INSERT PLUG .083	
8	226100	1/4-28 x 3/4 BH C/S	
9	820723	ADJ BCK BRCKT W/INSERT TUBE R	Right-see assemblies
9	820724	ADJ BCK BRCKT W/INSERT TUBE L	Left-see assemblies
10	226122	1/4-28 x 1.10 BH C/S	
11	226131	1/4-28 x 1 1/4 BH C/S SHOULDER	


Pos.	Item Number	Description	Remarks
12A	109452	ADJ BACK INSERT STANDARD	-see assemblies
12B	483557	ADJUSTABLE BACK BRACKET TALL/RECEIVER	-see assemblies
13	226130	1/4-28 x 1 1/2 BH C/S	
14	561013	7/8 CAP	
1,7	820710	PSH HNDL Q2 ADJ R LOW W/ STABILIZER	RIGHT-WORKS W STABILIZER BAR
1,7	820713	PSH HNDL Q2 ADJ L LOW W/ STABILIZER	LEFT-WORKS W STABILIZER
1,7	820711	PSH HNDL Q2 ADJ R MED W/ STABILIZER	RIGHT-WORKS W STABILIZER BAR
1,7	820714	PSH HNDL Q2 ADJ L MED W/ STABILIZER	LEFT-WORKS W STABILIZER
1,7	820712	PSH HNDL Q2 ADJ R TALL W/ STABILIZER	RIGHT-WORKS W STABILIZER BAR
1,7	820715	PSH HNDL Q2 ADJ L TALL W/ STABILIZER	LEFT-WORKS W STABILIZER
1, 5-7	820730	PUSH HANDLE Q2 LOW ANGLE ADJUSTABLE 7/8	RIGHT
1, 5-7	820731	PUSH HANDLE Q2 MED ANGLE ADJUSTABLE 7/8	RIGHT
1, 5-7	820732	PUSH HANDLE Q2 TALL ANGLE ADJUSTABLE 7/8	RIGHT
1, 5-7	820733	PUSH HANDLE Q2 LOW ANGLE ADJUSTABLE 7/8	LEFT
1, 5-7	820734	PUSH HANDLE Q2 MED ANGLE ADJUSTABLE 7/8	LEFT
1, 5-7	820735	PUSH HANDLE Q2 TALL ANGLE ADJUSTABLE 7/8	LEFT
6,7, 14	820736	BACK TUBE Q2 LOW ADJUSTABLE ANGLE 7/8	RIGHT
6,7, 14	820737	BACK TUBE Q2 MED ADJUSTABLE ANGLE 7/8	RIGHT
6,7, 14	820738	BACK TUBE Q2 TALL ADJUSTABLE ANGLE 7/8	RIGHT
6,7, 14	820739	BACK TUBE Q2 LOW ADJUSTABLE ANGLE 7/8	LEFT
6,7, 14	820740	BACK TUBE Q2 MED ADJUSTABLE ANGLE 7/8	LEFT
6,7, 14	820741	BACK TUBE Q2 TALL ADJUSTABLE ANGLE 7/8	LEFT
2-3, 9-12A	820742	BACK TUBE Q2 ANGLE ADJUSTABLE	LEFT-1 1/8" diameter
2-3, 9-12A	820744	BACK TUBE Q2 ANGLE ADJUSTABLE	RIGHT-1 1/8" diameter
2-3, 9-12B	820743	BACK TUBE Q2 ANGLE ADJUSTABLE TALL	RIGHT-1 1/8" diameter
2-3, 9-12B	820745	BACK TUBE Q2 ANGLE ADJUSTABLE TALL	LEFT-1 1/8" diameter

NOTE: Items 2 & 12 are different and both are required for assembly.

***NOTE:** If replacing or adding on to DAB BACKREST, please refer to the "DAB BACKREST" page.

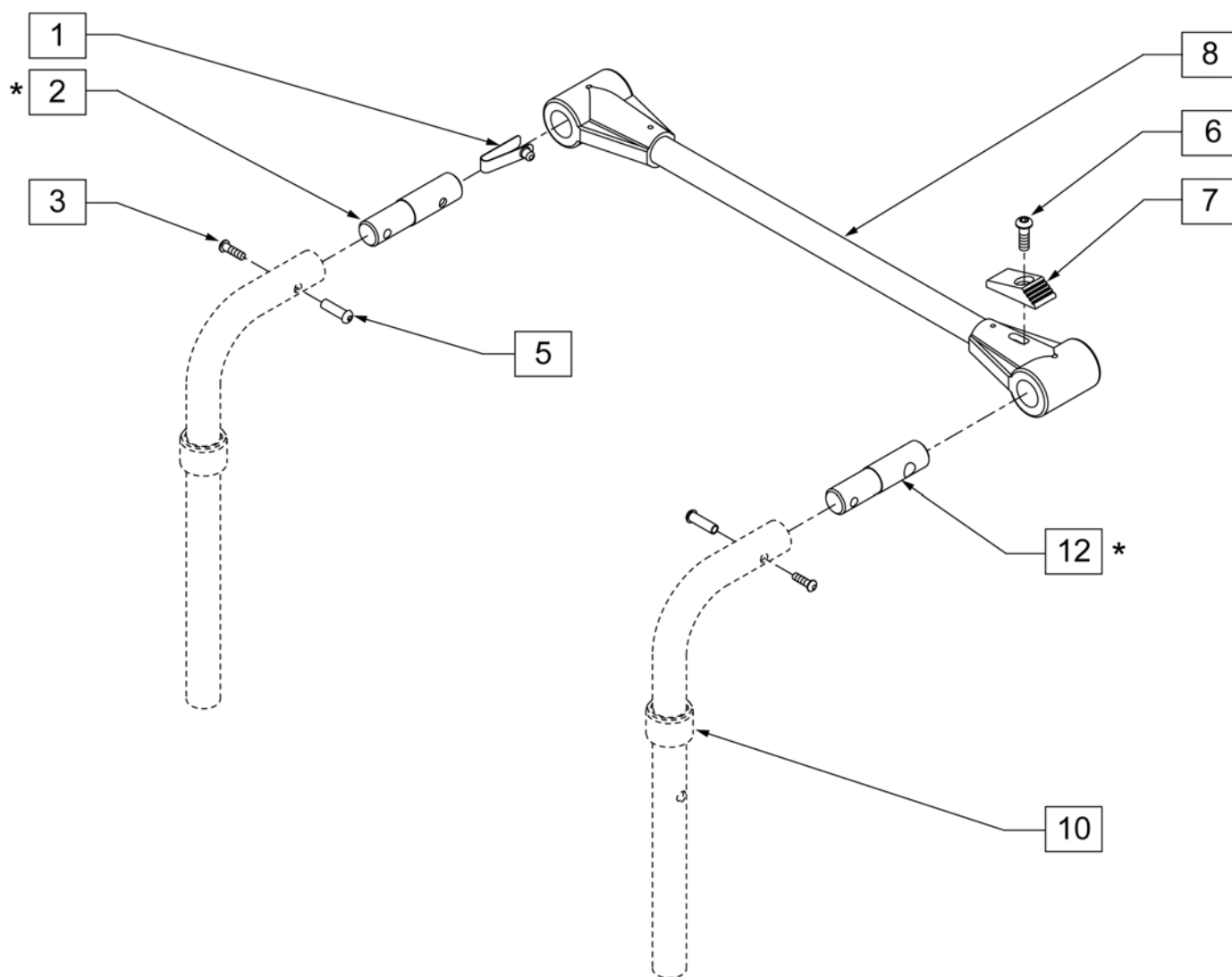

Pos.	Item Number	Description	Pos.	Item Number	Description
1	301100	1/4" SNAP BUTTON SINGLE	6-8	820695	STABILIZER CROSS BAR ASSM 14
2	483533	STABILIZER BAR PIVOT PIN Q2	6-8	820696	STABILIZER CROSS BAR ASSM 15
3	226020	10-32 x 5/8 BH C/S BLK	6-8	820697	STABILIZER CROSS BAR ASSM 16
5	221213	10-32 x 7/8 HEX SEX BOLT QTNT	6-8	820698	STABILIZER CROSS BAR ASSM 17
6	226110	1/4-28 x 1 BH C/S	6-8	820700	STABILIZER CROSS BAR ASSM 18
7	560143	STABILIZER LATCH RELEASE BR	6-8	820701	STABILIZER CROSS BAR ASSM 19
10	561083	1-1/8 TUBE END BUSHING THIN	6-8	820702	STABILIZER CROSS BAR ASSM 20
12	483534	STABILIZER BAR LOCKING PIN Q2	6-8	820703	STABILIZER CROSS BAR ASSM 21
6-8	820693	STABILIZER CROSS BAR ASSM 12	6-8	820704	STABILIZER CROSS BAR ASSM 22
6-8	820694	STABILIZER CROSS BAR ASSM 13	1-3,5,12	885015	STABILIZER PIN ASSM

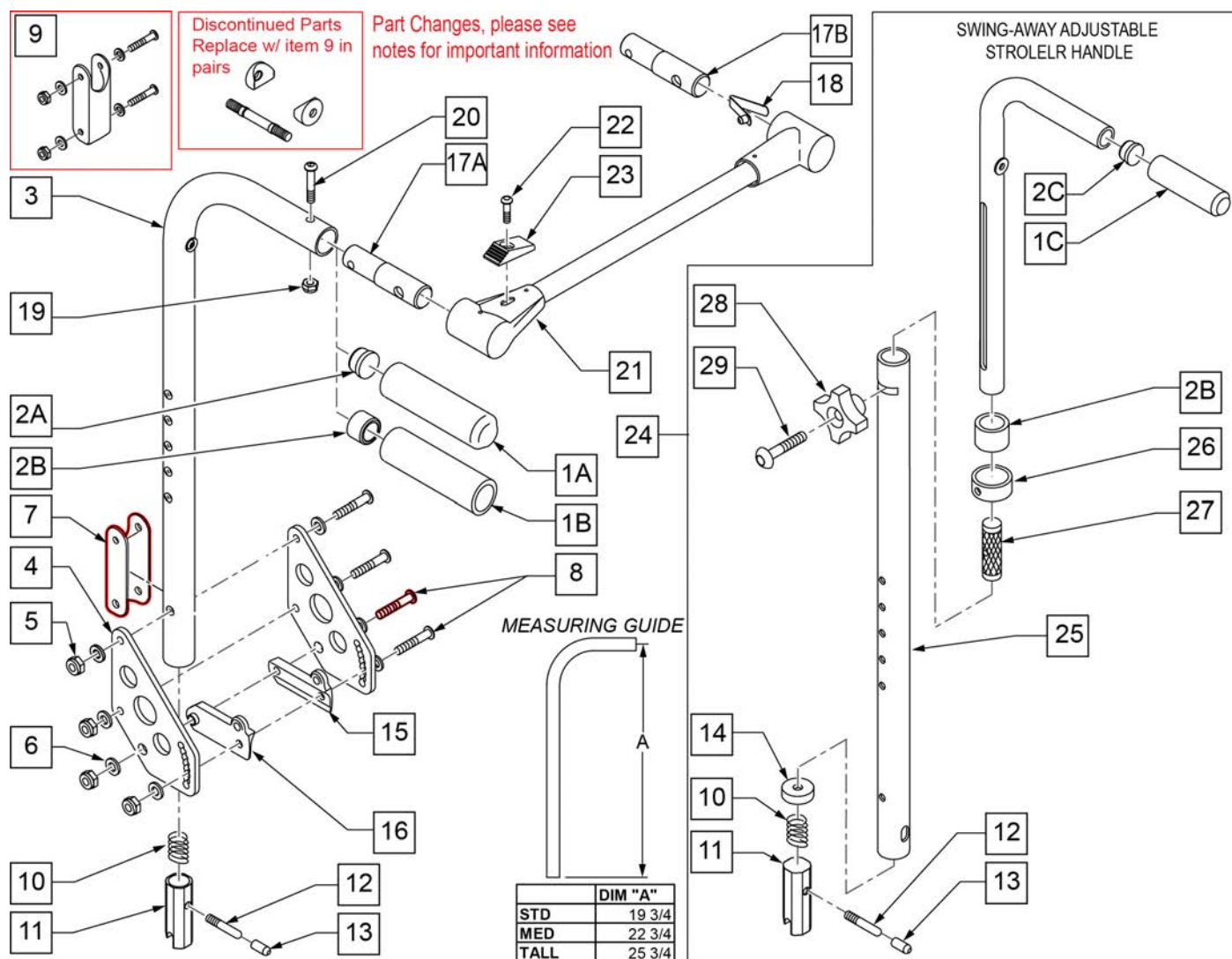


Pos.	Item Number	Description	Remarks
1A	920027	QUICKIE GRIP 1" W/ INSTR SHEET	
1B	920023	QUICKIE GRIP 1" W/ HOLE W/ INSTR	
1C	920026	QUICKIE GRIP 3/4" W/ INSTR SHEET	
2A	561002	1 INSERT PLUG .083	
2B	561082	1" TUBE END BUSHING THIN	USED WITH STROLLER HANDLE EXT
2C	561000-QU	3/4" INSERT PLUG .083	
4	506136	BACK PLATE DAB	
5	106935	NUT NYLOCK THIN 1/4 -28 ROHS	
6	106943	1/4 AN WASHER FLAT	
7	124612	BLOCK-LATCH STOP	
8	106964	1/4-28 x 2 BH C/S	
9	126099	BACK REST SADDLE W/ HDWR PAIR	Pair-Replacement Latch Stop Kit

Pos.	Item Number	Description	Remarks
9	126100	BACK REST SADDLE W/ HDWR EACH	Each- Replacement Latch Stop Kit
10	321110	S/A LATCH SPRING	
11	483019	BACK RELEASE LATCH	
12	481084	BACK RELEASE PIN Z2/QT/EX	
13	126238	PUSH ON ROUND CAP	
14	483021S	S/A SPRING RETAINER	
15	560518	SADDLE BACKREST Q2 DAB	RIGHT
16	560519	SADDLE BACKREST Q2 DAB	LEFT
17A	483536	STABILIZER BAR PIVOT PIN Q2	
17B	483535	STABILIZER BAR LOCKING PIN Q2	
18	301100	SNAP BUTTON 1/4" SINGLE	
19	202116	5/16-18 NYLOCK NUT THIN BLK	
20	222109	5/16-18 x 1 1/4 BH C/S BLK ZN	
21-23	820693	STABILIZER CROSS BAR ASSM 12	12" WIDE
21-23	820694	STABILIZER CROSS BAR ASSM 13	13" WIDE
21-23	820695	STABILIZER CROSS BAR ASSM 14	14" WIDE
21-23	820696	STABILIZER CROSS BAR ASSM 15	15" WIDE
21-23	820697	STABILIZER CROSS BAR ASSM 16	16" WIDE
21-23	820698	STABILIZER CROSS BAR ASSM 17	17" WIDE
21-23	820700	STABILIZER CROSS BAR ASSM 18	18" WIDE
21-23	820701	STABILIZER CROSS BAR ASSM 19	19" WIDE
21-23	820702	STABILIZER CROSS BAR ASSM 20	20" WIDE
21-23	820703	STABILIZER CROSS BAR ASSM 21	21" WIDE
21-23	820704	STABILIZER CROSS BAR ASSM 22	22" WIDE
24	842082	S/A ADJ STROLLER HANDLE W/ FOAM R	RIGHT
24	842083	S/A ADJ STROLLER HANDLE W/ FOAM L	LEFT
24	920306	S/A ADJ STROLLER HANDLE W/ FOAM (PR)	PAIR
25	521791	ZP S/A ADJ BACK TUBE ONLY R	RIGHT
25	521792	ZP S/A ADJ BACK TUBE ONLY L	LEFT
26	483096	S/A ADJUSTABLE STROLLER RING ZP/QT	
27	483095	S/A ADJUSTABLE STROLLER INSERT ZP/QT	
28	349018	KNOB S/A ADJUSTABLE STROLLER ZP	
29	226136	1/4-20 x 1 1/2 BH C/S BLK	
5-9	920539	BACK PLATE DAB HARDWARE KIT	
5-13	920328	BACKREST LATCH KIT	
17A,17B,18-20	116604	DAB STBLZR ATTACH KIT	
1A,2A,3	842080	STROLLER HANDLE W/ FOAM STANDARD ZPS/P5/QT	STANDARD/LOW
1A,2A,3	842079	STROLLER HANDLE W/ FOAM MED ZPS/QT	MEDIUM
1A,2A,3	842081	STROLLER HANDLE W/ FOAM TALL ZPS/QT	TALL
1A,2A,3, 10-13	920390	STROLLER HANDLE W/ LATCH STANDARD ZPS/P5/QT	STANDARD/LOW
1A,2A,3, 10-13	920389	STROLLER HANDLE W/ LATCH MED ZPS/QT	MEDIUM
1A,2A,3, 10-13	920391	STROLLER HANDLE W/ LATCH TALL ZPS/QT	TALL
1A,2A-16	881846	STROLLER HANDLE STANDARD & PLATE ASSM DAB R	STANDARD/LOW- RIGHT
1A,2A-16	105390	STROLLER HANDLE STANDARD & PLATE ASSM DAB L	STANDARD/LOW- LEFT


Pos.	Item Number	Description	Remarks
1A,2A-16	881845	STROLLER HANDLE MED & PLATE ASSM DAB R	MEDIUM-RIGHT
1A,2A-16	105389	STROLLER HANDLE MED & PLATE ASSM DAB L	MEDIUM-LEFT
1A,2A-16	881847	STROLLER HANDLE TALL & PLATE ASSM DAB R	TALL-RIGHT
1A,2A-16	105395	STROLLER HANDLE TALL & PLATE ASSM DAB L	TALL-LEFT
1B, 2B, 3	842067	EXTENSION-STROLLER HANDLE STANDARD	STANDARD/LOW -Used w/ Removable Extension.
1B, 2B, 3	842066	EXTENSION-STROLLER HANDLE MED	MEDIUM-Used w/ Removable Extension.
1B, 2B, 3	842068	EXTENSION-STROLLER HANDLE TALL	TALL-Used w/ Removable Extension.
1B, 2B, 3 ,10-13	920421	EXTENSION-STROLLER HANDLE STD W/ LATCH	STANDARD/LOW -Used w/ Removable Extension.
1B, 2B, 3 ,10-13	920423	EXTENSION-STROLLER HANDLE MED W/ LATCH	MEDIUM-Used w/ Removable Extension.
1B, 2B, 3 ,10-13	920424	EXTENSION-STROLLER HANDLE TALL W/ LATCH	TALL-Used w/ Removable Extension.
1B, 2B, 3-16	920348	EXT STROLLER HANDLE STD & PLATE ASSM	PAIR
1B, 2B, 3-16	920375	EXT STROLLER HANDLE MED & PLATE ASSM	PAIR
1B, 2B, 3-16	920349	EXT STROLLER HANDLE TALL & PLATE ASSM	PAIR
>>>>		STROLLER HANDLE ONLY ASSEMBLIES	
1A,2A,3	842080	STROLLER HANDLE W/ FOAM STANDARD ZPS/P5/QT	STANDARD/LOW
1A,2A,3	842079	STROLLER HANDLE W/ FOAM MED ZPS/QT	MEDIUM
1A,2A,3	842081	STROLLER HANDLE W/ FOAM TALL ZPS/QT	TALL
1A,2A,3, 10-13	920390	STROLLER HANDLE W/ LATCH STANDARD ZPS/P5/QT	STANDARD/LOW
1A,2A,3, 10-13	920389	STROLLER HANDLE W/ LATCH MED ZPS/QT	MEDIUM
1A,2A,3, 10-13	920391	STROLLER HANDLE W/ LATCH TALL ZPS/QT	TALL
>>>>		STROLLER HANDLES USED W/ REMOVABLE EXTENSION	
1B, 2B, 3	842067	EXTENSION-STROLLER HANDLE STANDARD	STANDARDLOW -Used w/ Removable Extension.
1B, 2B, 3	842066	EXTENSION-STROLLER HANDLE MED	MEDIUM-Used w/ Removable Extension.
1B, 2B, 3	842068	EXTENSION-STROLLER HANDLE TALL	TALL-Used w/ Removable Extension.
1B, 2B, 3 ,10-13	920421	EXTENSION-STROLLER HANDLE STANDARD W/ LATCH	STANDARD/LOW -Used w/ Removable Extension.
1B, 2B, 3 ,10-13	920423	EXTENSION-STROLLER HANDLE MED W/ LATCH	MEDIUM-Used w/ Removable Extension.

Pos.	Item Number	Description	Remarks
1B, 2B, 3 ,10-13	920424	EXTENSION-STROLLER HANDLE TALL W/ LATCH	TALL-Used w/ Removable Extension.
1B, 2B-16	920762	EXTENSION-STROLLER HANDLE STANDARD & PLATE ASSM	STANDARD/LOW -Used w/ Removable Extension.
1B, 2B-16	920761	EXTENSION-STROLLER HANDLE MED & PLATE ASSM DAB	MEDIUM-Used w/ Removable Extension.
1B, 2B-16	920763	EXTENSION-STROLLER HANDLE TALL & PLATE ASSM DAB	TALL-Used w/ Removable Extension.
>>>> STROLLER HANDLES USED W/ STABILIZER BAR			
3	842089	BACK TUBE STD DAB STBLZR	STD/LOW - Used w/ Stabilizer Bar
3	842088	BACK TUBE MED DAB STBLZR	MEDIUM - Used w/ Stabilizer Bar
3	842090	BACK TUBE TLL DAB STBLZR	TALL - Used w/ Stabilizer Bar
3-13	882676	STROLLER HANDLE W/O FOAM STD/LOW RIGHT W/ LATCH	STD/LOW-RIGHT - Used w/ Stabilizer Bar
3-13	101131	STROLLER HANDLE W/O FOAM STD/LOW LEFT W/ LATCH	STD/LOW-LEFT - Used w/ Stabilizer Bar
3-13	882677	STROLLER HANDLE W/O FOAM MED RIGHT W/ LATCH	MEDIUM-RIGHT - Used w/ Stabilizer Bar
3-13	101123	STROLLER HANDLE W/O FOAM MED LEFT W/ LATCH	MEDIUM-LEFT - Used w/ Stabilizer Bar
3-13	882678	STROLLER HANDLE W/O FOAM TALL RIGHT W/ LATCH	TALL-RIGHT - Used w/ Stabilizer Bar
3-13	101122	STROLLER HANDLE W/O FOAM TALL LEFT W/ LATCH	TALL-LEFT - Used w/ Stabilizer Bar
SWING-AWAY ADJUSTABLE STROLELR HANDLE		Part SWING-AWAY ADJUSTABLE STROLELR HANDLE	

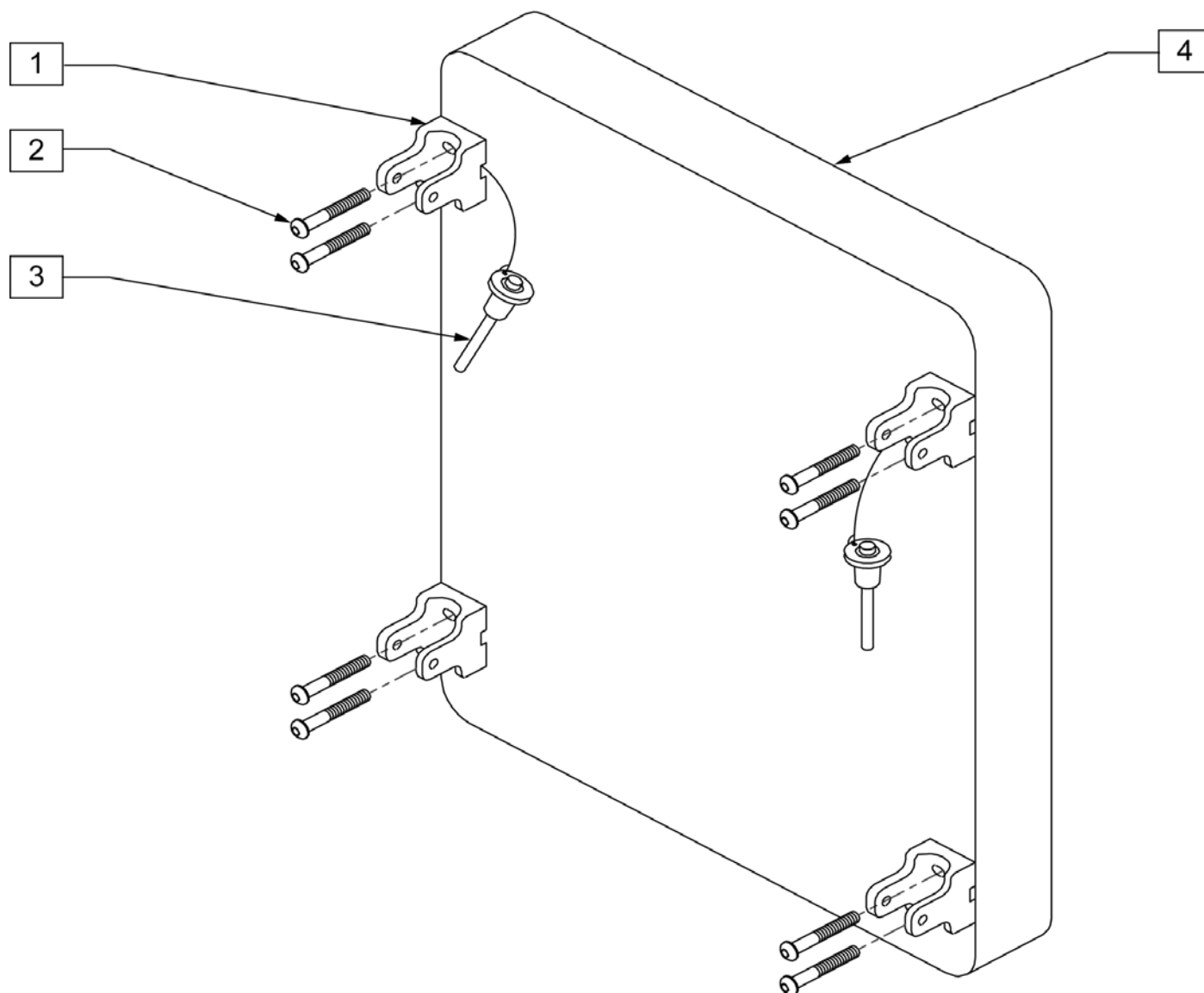


Pos.	Item Number	Description	Pos.	Item Number	Description
1	562609	BRACKET SEAT QTS/BACK Z2	1-4	681107	PLY BCK ASSM BLK 18X20 Q2 DAB
2	226101	1/4-20 x 3/4 BH C/S	1-4	681108	PLY BCK ASSM BLK 19X20 Q2 DAB
3	349020	STROLLER HANDLE RELEASE PIN	1-4	681109	PLY BCK ASSM BLK 20X20 Q2 DAB
1-3	920350	Q2 DEPTH ADJ BACK HARDWARE KIT	1-4	681321	PLY BCK ASSM BLK 11X12 Q2 DAB
1-4	681100	PLY BCK ASSM BLK 11X20 Q2 DAB	1-4	681322	PLY BCK ASSM BLK 11X14 Q2 DAB
1-4	681101	PLY BCK ASSM BLK 12X20 Q2 DAB	1-4	681323	PLY BCK ASSM BLK 11X16 Q2 DAB
1-4	681102	PLY BCK ASSM BLK 13X20 Q2 DAB	1-4	681324	PLY BCK ASSM BLK 11X18 Q2 DAB
1-4	681103	PLY BCK ASSM BLK 14X20 Q2 DAB	1-4	681325	PLY BCK ASSM BLK 12X12 Q2 DAB
1-4	681104	PLY BCK ASSM BLK 15X20 Q2 DAB	1-4	681326	PLY BCK ASSM BLK 12X14 Q2 DAB
1-4	681105	PLY BCK ASSM BLK 16X20 Q2 DAB	1-4	681327	PLY BCK ASSM BLK 12X16 Q2 DAB
1-4	681106	PLY BCK ASSM BLK 17X20 Q2 DAB	1-4	681328	PLY BCK ASSM BLK 12X18 Q2 DAB

Pos.	Item Number	Description	Pos.	Item Number	Description
1-4	681329	PLY BCK ASSM BLK 13X12 Q2 DAB	1-4	681345	PLY BCK ASSM BLK 17X12 Q2 DAB
1-4	681330	PLY BCK ASSM BLK 13X14 Q2 DAB	1-4	681346	PLY BCK ASSM BLK 17X14 Q2 DAB
1-4	681331	PLY BCK ASSM BLK 13X16 Q2 DAB	1-4	681347	PLY BCK ASSM BLK 17X16 Q2 DAB
1-4	681332	PLY BCK ASSM BLK 13X18 Q2 DAB	1-4	681348	PLY BCK ASSM BLK 17X18 Q2 DAB
1-4	681333	PLY BCK ASSM BLK 14X12 Q2 DAB	1-4	681349	PLY BCK ASSM BLK 18X12 Q2 DAB
1-4	681334	PLY BCK ASSM BLK 14X14 Q2 DAB	1-4	681350	PLY BCK ASSM BLK 18X14 Q2 DAB
1-4	681335	PLY BCK ASSM BLK 14X16 Q2 DAB	1-4	681351	PLY BCK ASSM BLK 18X16 Q2 DAB
1-4	681336	PLY BCK ASSM BLK 14X18 Q2 DAB	1-4	681352	PLY BCK ASSM BLK 18X18 Q2 DAB
1-4	681337	PLY BCK ASSM BLK 15X12 Q2 DAB	1-4	681353	PLY BCK ASSM BLK 19X12 Q2 DAB
1-4	681338	PLY BCK ASSM BLK 15X14 Q2 DAB	1-4	681354	PLY BCK ASSM BLK 19X14 Q2 DAB
1-4	681339	PLY BCK ASSM BLK 15X16 Q2 DAB	1-4	681355	PLY BCK ASSM BLK 19X16 Q2 DAB
1-4	681340	PLY BCK ASSM BLK 15X18 Q2 DAB	1-4	681356	PLY BCK ASSM BLK 19X18 Q2 DAB
1-4	681341	PLY BCK ASSM BLK 16X12 Q2 DAB	1-4	681357	PLY BCK ASSM BLK 20X12 Q2 DAB
1-4	681342	PLY BCK ASSM BLK 16X14 Q2 DAB	1-4	681358	PLY BCK ASSM BLK 20X14 Q2 DAB
1-4	681343	PLY BCK ASSM BLK 16X16 Q2 DAB	1-4	681359	PLY BCK ASSM BLK 20X16 Q2 DAB
1-4	681344	PLY BCK ASSM BLK 16X18 Q2 DAB	1-4	681360	PLY BCK ASSM BLK 20X18 Q2 DAB

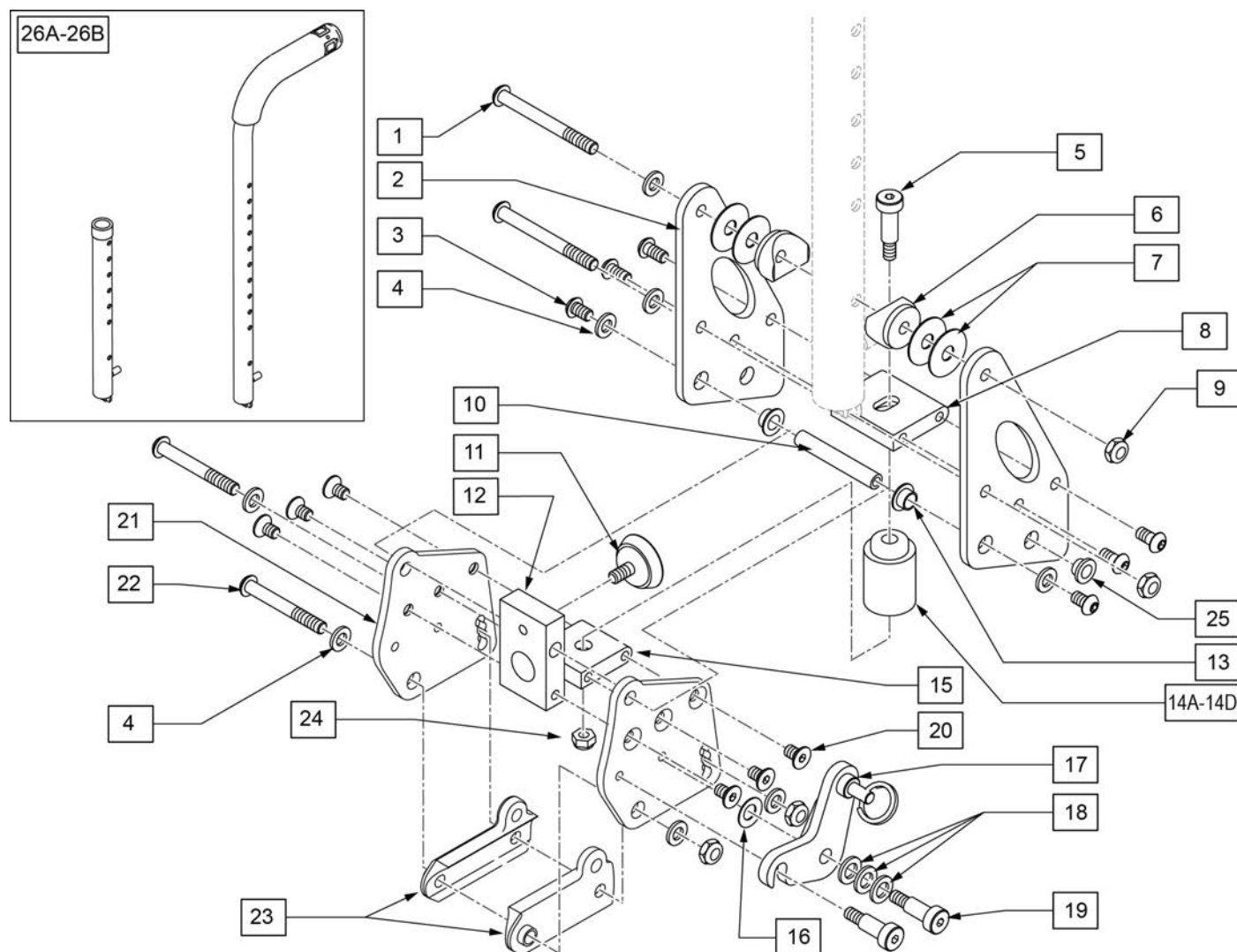


Pos.	Item Number	Description	Remarks
1	106967	BHCS 1/4-28 X 2-1/2 SHLDR ROHS	
3	103185-005	BHCS M6 X 1 X 12 FULL PL	
4	106943	WASHER FLAT AN 1/4 ROHS	
5	011649-003	SHLDR SCREW M6 X 8MM OD X 45MM	
6	561087	BACK REL SADDLE RIGID	
7	113268	WASHER NYLON 1IN OD X .312 ID	
9	106935	NUT NYLOCK THIN 1/4-28 ROHS	
11	349560	ZP RUBBER STOP	
14A	124044	SOFT RED SUSPENSION KIT	
14B	124060	MED BLUE SUSPENSION KIT	
14C	124064	FIRM GREEN SUSPENSION KIT	
14D	124082	X-FIRM BLACK SUSPENSION KIT	
16	262134	5/16 WASHER SPRING DISC	
18	106947	WASHER FLAT THIN AN 5/16 ROHS	
19	011649-001	SHLDR SCREW 8MM OD X 16MM LG	
20	013450	FHCS M6 X 1 X 10MM W/PCH	
22	106965	BHCS 1/4-28 X 2 SHLDR ROHS	
23	560518	SADDLE BACKREST Q2 DAB RIGHT	Right

Pos.	Item Number	Description	Remarks
23	560519	SADDLE BACKREST Q2 DAB LEFT	Left
24	104381	NUT HEX NYLOCK M6X1.0 INORG	
16-19,24,25	120943	DB LATCH REPLACEMENT KIT	Left & Right
1-25	126247	DYNAMIC BACKREST ASSM RH	
1-25	126248	DYNAMIC BACKREST ASSM LH	
1-25	121894	DAB DYNAMIC BACKREST ASSM RH	Right
1-25	121895	DAB DYNAMIC BACKREST ASSM LH	Left
1-26A	123944	ZP-Q2 DY BK PR ASM HGT ADJ STD	Pair
1-26A	123945	ZP-Q2 DY BK PR ASM HGT ADJ TLL	Pair
1-26B	123946	ZP-Q2 DY BK PR ASM STR EXT STD	Pair
1-26B	123948	ZP-Q2 DY BK PR ASM STR EXT MED	Pair
1-26B	123949	ZP-Q2 DY BK PR ASM STR EXT TLL	Pair
26A	123928	DYBCK ASSM ZPQ2 STD HGT ADJ RH	Right
26A	123929	DYBCK ASSM ZPQ2 STD HGT ADJ L	Left
26A	123930	DYBCK ASSM ZPQ2 TLL HGT ADJ RH	Right
26A	123931	DYBCK ASSM ZPQ2 TLL HGT ADJ LH	Left
26B	123932	DYBCK ZPQ2 STR EXT ASSM STD RH	Right
26B	123933	DYBCK ZPQ2 STR EXT ASSM STD LH	Left
26B	123935	DYBCK ZPQ2 STR EXT ASSM MED RH	Right
26B	123941	DYBCK ZPQ2 STR EXT ASSM MED LH	Left
26B	123942	DYBCK ZPQ2 STR EXT ASSM TLL RH	Right
26B	123943	DYBCK ZPQ2 STR EXT ASSM TLL LH	Left

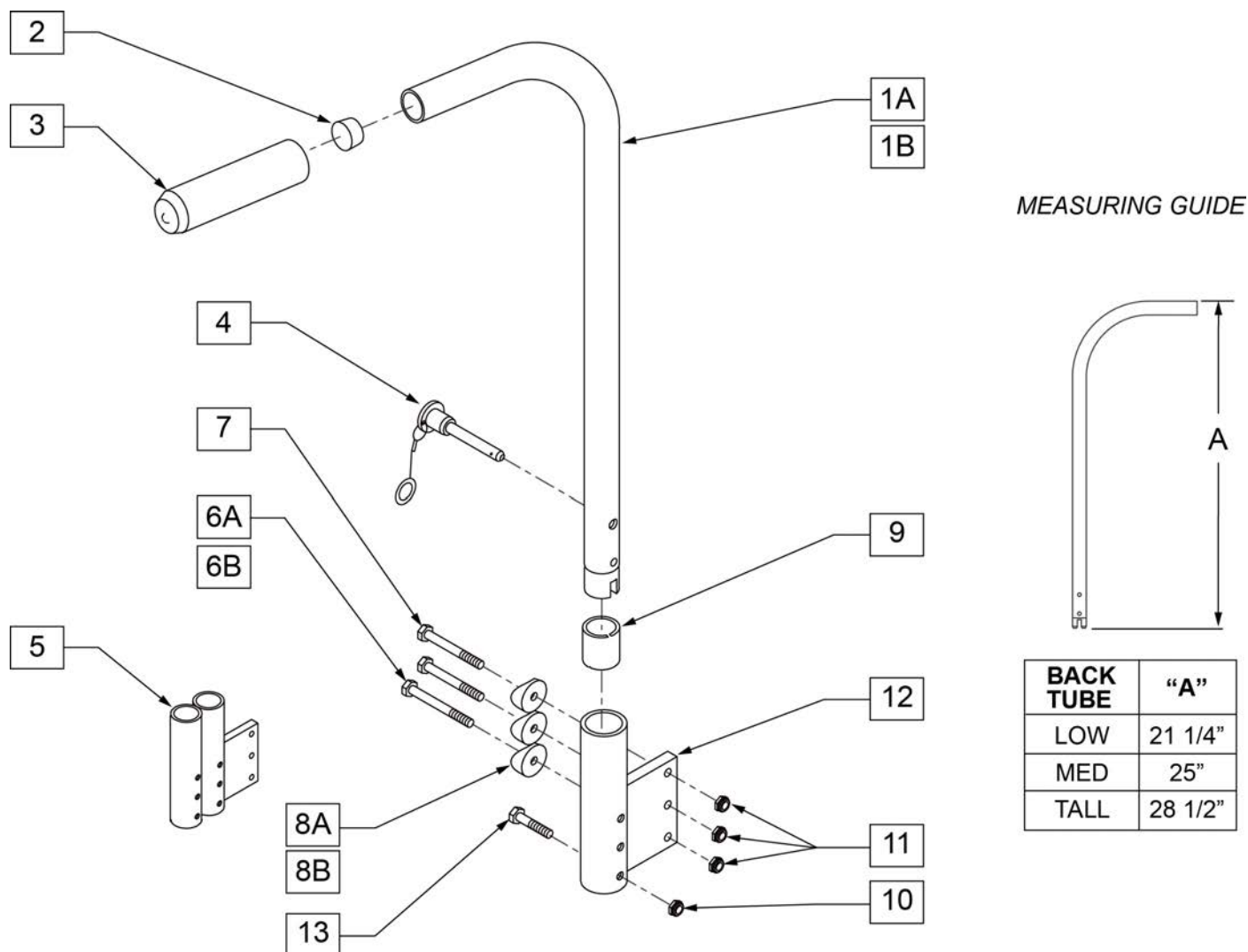

Pos.	Item Number	Description	Remarks
2	561002	1" INSERT PLUG .083	
3	920070	9 x .940 x .200 FOAM GRIP	
4	349020	STROLLER HANDLE RELEASE PIN	
5	540130	PADDED ARM RECEIVER STRAIGHT DOUBLE FOLDING FRAME	RIGHT
5	540132	PADDED ARM RECEIVER STRAIGHT DOUBLE FOLDING FRAME	LEFT
6A	106973	1/4-28 x 1 3/4 G8 HEX C/S	
6B	107351	1/4-28 x 2 G8 HEX C/S	
7	107351	1/4-28 x 2 1/4 G8 HEX C/S	
8A	561004	1" SADDLE 1/4 HOLE	
8B	561001	1 - 1/8 SADDLE 1/4 HOLE	
9	560400	1" I.D. NY-LINER	
10	106935	NUT NYLOCK THIN 1/4 -28 ROHS	
11	106934	1/4-28 NYLOCK NUT	
12	540100	PADDED ARM RECEIVER STRAIGHT FOLDING FRAME	RIGHT
12	540102	PADDED ARM RECEIVER STRAIGHT FOLDING FRAME	LEFT
13	106972	1/4-28 x 1 1/2 G8 HEX C/S	
14	341200	1/4 x 1 ROLL PIN	
15	483029	PUSH HANDLE PLUG	

Pos.	Item Number	Description	Remarks
1A-3, 14, 15	880360	STROLLER HANDLE REGULAR STRAIGHT FOLDING	
1B-3, 14, 15	880362	STROLLER HANDLE TALL STRAIGHT FOLDING	
1A-3, 14, 15	900420	STROLLER HANDLES REGULAR FOLDING	PAIR
1B-3, 14, 15	900421	STROLLER HANDLES TALL FOLDING	PAIR
4, 6-13	900500	STROLLER RECEIVER SINGLE CYL F 3/4	PAIR
4, 6B, 7, 8B, 9-13	900502	STROLLER RECEIVER SINGLE CYL FOLDING FRAME 7/8	PAIR
4-11, 13	900501	ARM/STROLLER RECEIVER DOUBLE CYL F 3/4	PAIR
4, 5, 6B, 7, 8B, 9-11, 13	900503	ARM/STROLLER RECEIVER DOUBLE CYL F 7/8	PAIR

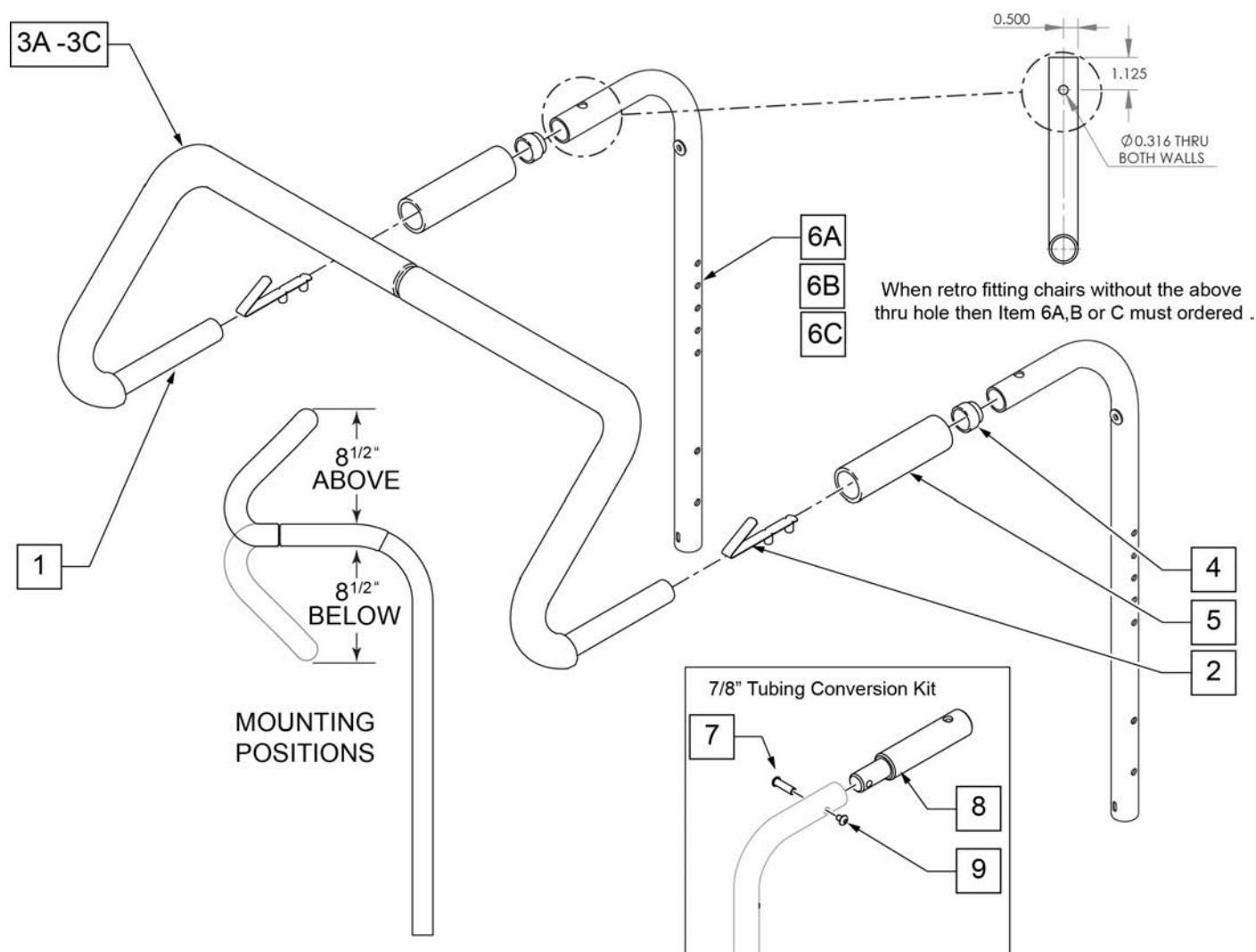


Pos.	Item Number	Description	Remarks
2	301108	5/16 SNAP BUTTON DUAL ONE SIDE	
3A	560615	STROLLER EXTENSION GRIP 10 -13"	10 -13" WIDE
3B	560616	STROLLER EXTENSION GRIP 14 -17"	14 -17" WIDE
3C	560617	STROLLER EXTENSION GRIP 18 -20"	18 -20" WIDE
4	561082	1" TUBE END BUSHING THIN	
5	920023	QUICKIE GRIP 1 W/ HOLE	
1, 2, 3A	842050	STROLLER EXTENSION ASSM 10"	
1, 2, 3A	842051	STROLLER EXTENSION ASSM 11"	
1, 2, 3A	842052	STROLLER EXTENSION ASSM 12"	
1, 2, 3A	842053	STROLLER EXTENSION ASSM 13"	
1, 2, 3B	842054	STROLLER EXTENSION ASSM 14"	
1, 2, 3B	842055	STROLLER EXTENSION ASSM 15"	
1, 2, 3B	842056	STROLLER EXTENSION ASSM 16"	
1, 2, 3B	842057	STROLLER EXTENSION ASSM 17"	
1, 2, 3C	842058	STROLLER EXTENSION ASSM 18"	
1, 2, 3C	842059	STROLLER EXTENSION ASSM 19	
1, 2, 3C	842063	STROLLER EXTENSION ASSM 20	
6A	842080	STROLLER HANDLE W/ FOAM STANDARD ZPS/P5/QT	STANDARD/LOW

Pos.	Item Number	Description	Remarks
6B	842079	STROLLER HANDLE W/ FOAM MED ZPS/QT	MEDIUM
6C	842081	STROLLER HANDLE W/ FOAM TALL ZPS/QT	TALL
7-9	125324	STRLLR HNDL CNVERSN KIT 7/8 PH	To fit to 7/8" Push Handle

NOTE: Stroller handle extension works with 1" tubing backrest only. Example: DAB on Q2 or Standard backrest on positioning chairs.


NOTE: Not compatible with Swing-Away Adjustable Stroller Handle.

STUD WEDGE SIZE FOR:	SMALL	MEDIUM	LARGE
AVERAGE INSIDE DIM	3/4"	7/8"	1"
<u>MOST</u> OUTSIDE DIM	0.575 - 0.638	0.700 - 0.785	0.825 - 0.910

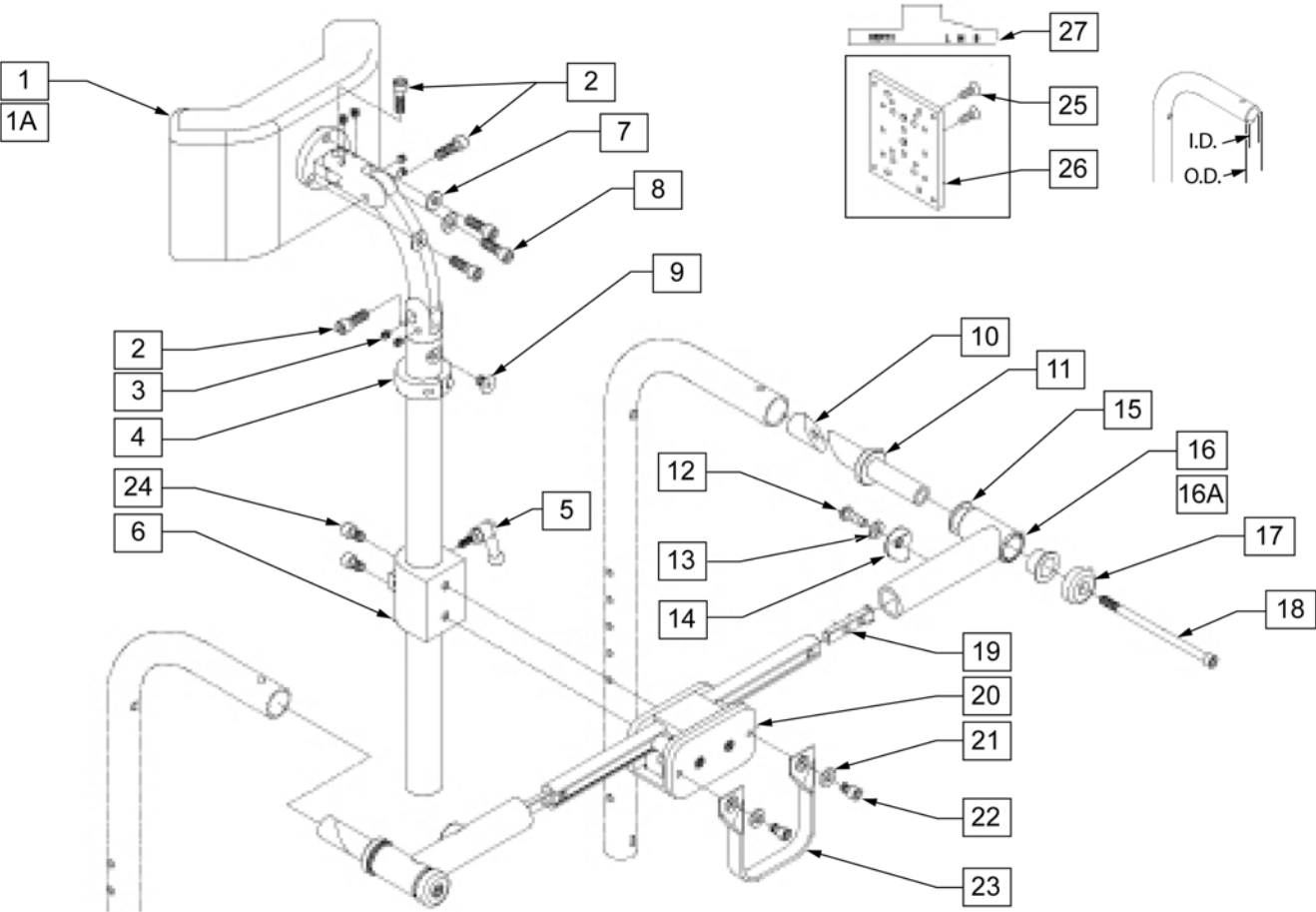

Pos.	Item Number	Description	Remarks
1A	750696	HEADREST W/ ZIPPER MEDIUM CG	8" X 4" Pad
1B	750697	HEADREST W/ ZIPPER LARGE CG	9" X 4.5" Pad
2	221115	1/4 - 20 X 3/4 SOC HD C/S BLK	
3	341045	#10 - 24 CONE POINT SET SCREW	
4	505933	HEADREST COLLAR	
5	341046	HEADREST CLAMPING HANDLE SCREW	
6	881003	CLAMP AND TUBE	
7	261204	#10 AN WASHER .063	
8	221010	10 - 32 X .75 SHCS BLK W/ PL	
9	226072	1/4 - 20 X 1/2 FH C/S BLK	
10A	520855	NUT WEDGE SMALL	
10B	520816	NUT WEDGE MEDIUM	
10C	520856	NUT WEDGE LARGE	
11A	520857	STUD WEDGE SMALL	
11B	520815	STUD WEDGE MEDIUM	
11C	520858	STUD WEDGE LARGE	
12	226064	1/4 - 28 X 3/4 BH C/S BLK W/PATCH	
13	106943	WASHER FLAT AN 1/4 ROHS	
14	561004	1 SADDLE 1/4 HOLE	
15	349595	BEARING IGUS 3/8 X 15/32 X 1/2	
16A	542158	HEADREST PVT TB WELDMOUNT 14 - 17	
16B	542159	HEADREST PVT TB WELDMOUNT 17 - 20	
17	483342	STUD WEDGE CAP	
18	226351	1/4 - 20 X 4 - 1/2 SOC HD BLK ZINC	
19	500078	INSERT D2 EXT TUBE	
21	106943	WASHER FLAT AN 1/4 ROHS	
22	226095	1/4 - 20 X 4 - 3/8 SOC HD BLK ZINC	
23	750552	ZGS FOLDING STRAP	
24	226095	1/4 - 20 X 4 - 3/8 SOC HD BLK ZINC	
25	226076	1/4 - 28 X 1/2 FH C/S W/ PATCH	
26	XM2025	MNTNG PLATE/PEDIAT.HEAD REST	
27	342046	MEASURING TOOL FOLD HEADREST	
2-9	883350	HEADREST MOUNTING BRACKET ASSM EURO	
20-24	888920	CENTER MOUNT FOLD HEADREST ASSM	
21-23	750104	FOLD STABALIZER BAR STRAP ASSM	
10, 11, 17, 18	881897	STUD WEDGE ASSM SMALL	
10, 11, 17, 18	881898	STUD WEDGE ASSM MEDIUM	
10, 11, 17, 18	881899	STUD WEDGE ASSM LARGE	
10-15, 16A, 17-20	820750	STABALIZER BAR ASSM 14 - 17 SMALL	
10-15, 16A, 17-20	820751	STABALIZER BAR ASSM 14 - 17 MEDIUM	
10-15, 16A, 17-20	820752	STABALIZER BAR ASSM 14 - 17 LARGE	
10-15, 16B, 17-20	820753	STABALIZER BAR ASSM 17 - 20 SMALL	
10-15, 16B, 17-20	820754	STABALIZER BAR ASSM 17 - 20 MEDIUM	
10-15, 16B, 17-20	820755	STABALIZER BAR ASSM 17 - 20 LARGE	
1-15, 16A, 17-20	820760	STABALIZER W/ MEDIUM HEADREST 14 - 17 SMALL	
1-15, 16A, 17-20	820761	STABALIZER W/ MEDIUM HEADREST 14 - 17 MEDIUM	
1-15, 16A, 17-20	820762	STABALIZER W/ MEDIUM HEADREST 14 - 17 LARGE	
1-15, 16B, 17-20	820763	STABALIZER W/ MEDIUM HEADREST 17 - 20 SMALL	
1-15, 16B, 17-20	820764	STABALIZER W/ MEDIUM HEADREST 17 - 20 MEDIUM	
1-15, 16B, 17-20	820765	STABALIZER W/ MEDIUM HEADREST 17 - 20 LARGE	
1A, 2-15, 16A, 17-20	820766	STABALIZER W/ LARGE HEADREST 14 - 17 SMALL	
1A, 2-15, 16A, 17-20	820767	STABALIZER W/ LARGE HEADREST 14 - 17 MEDIUM	
1A, 2-15, 16A, 17-20	820768	STABALIZER W/ LARGE HEADREST 14 - 17 LARGE	
1A, 2-15, 16B, 17-20	820769	STABALIZER W/ LARGE HEADREST 17 - 20 SMALL	
1A, 2-15, 16B, 17-20	820770	STABALIZER W/ LARGE HEADREST 17 - 20 MEDIUM	
1A, 2-15, 16B, 17-20	820771	STABALIZER W/ LARGE HEADREST 17 - 20 LARGE	


	BACK HEIGHT "A"
SHORT/ LOW	11-14"
MED	12-16"
TALL	14-18"

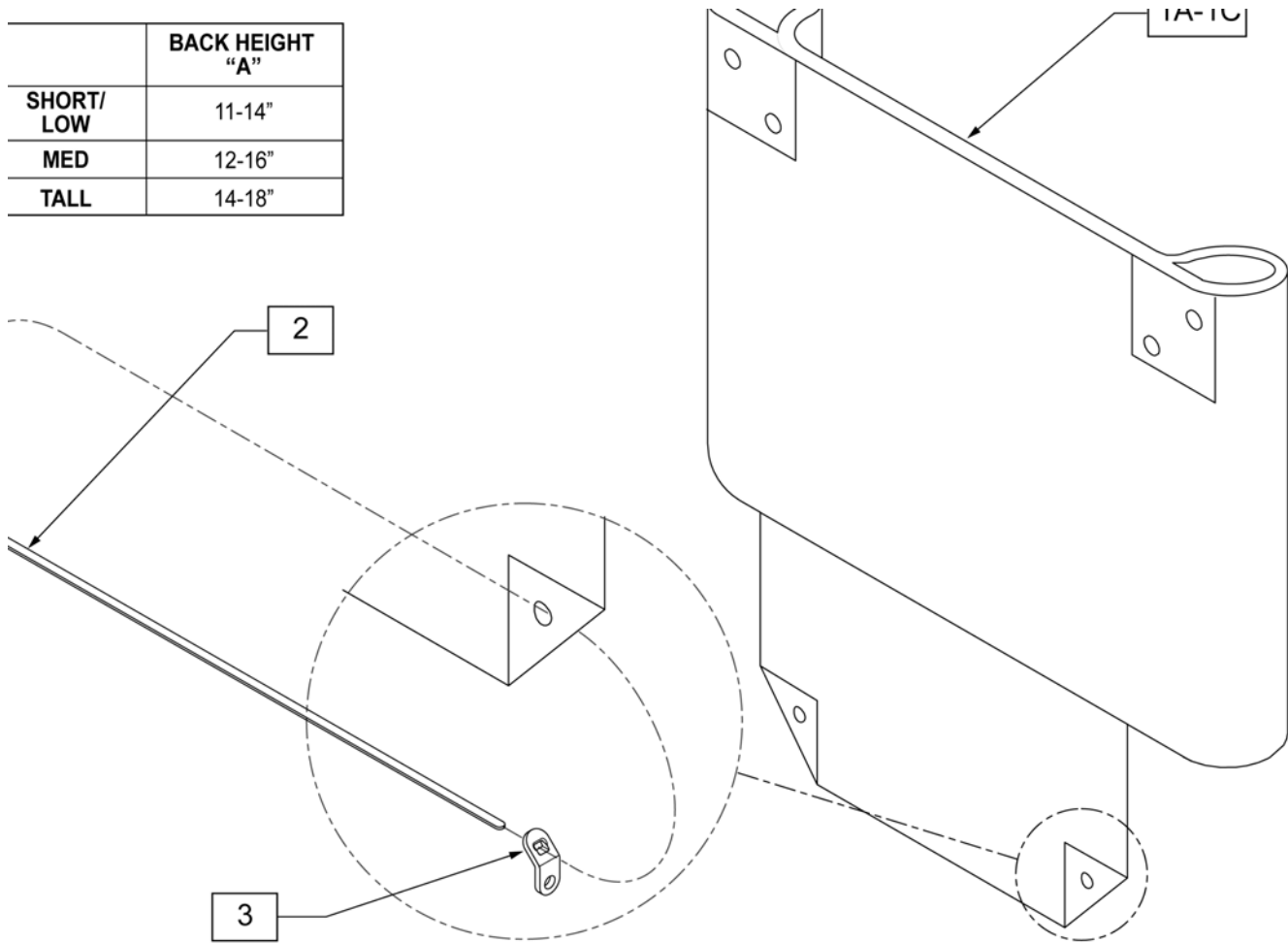

Pos.	Item Number	Description	Pos.	Item Number	Description
1A	684010	BACK UPHOLSTERY 11-12 Q2 LOW	1A	684060	BACK UPHOLSTERY 17 Q2 LOW
1B	684012	BACK UPHOLSTERY 11-12 Q2 MED Rx 14	1B	684062	BACK UPHOLSTERY 17 Q2 MED Rx 14
1C	684014	BACK UPHOLSTERY 11-12 Q2 TALL Rx 17/19	1C	684064	BACK UPHOLSTERY 17 Q2 TALL Rx 17/19
1A	684020	BACK UPHOLSTERY 13 Q2 LOW	1A	684070	BACK UPHOLSTERY 18 Q2 LOW
1B	684022	BACK UPHOLSTERY 13 Q2 MED Rx 14	1B	684072	BACK UPHOLSTERY 18 Q2 MED Rx 14
1C	684024	BACK UPHOLSTERY 13 Q2 TALL Rx 17/19	1C	684074	BACK UPHOLSTERY 18 Q2 TALL Rx 17/19
1A	684030	BACK UPHOLSTERY 14 Q2 LOW	1A	684080	BACK UPHOLSTERY 19 Q2 LOW
1B	684032	BACK UPHOLSTERY 14 Q2 MED Rx 14	1B	684082	BACK UPHOLSTERY 19 Q2 MED Rx 14
1C	684034	BACK UPHOLSTERY 14 Q2 TALL Rx 17/19	1C	684084	BACK UPHOLSTERY 19 Q2 TALL Rx 17/19
1A	684040	BACK UPHOLSTERY 15 Q2 LOW	1A	684090	BACK UPHOLSTERY 20 Q2 LOW
1B	684042	BACK UPHOLSTERY 15 Q2 MED Rx 14	1B	684092	BACK UPHOLSTERY 20 Q2 MED Rx 14
1C	684044	BACK UPHOLSTERY 15 Q2 TALL Rx 17/19	1C	684094	BACK UPHOLSTERY 20 Q2 TALL Rx 17/19
1A	684050	BACK UPHOLSTERY 16 Q2 LOW	2	560410	ZIP TIE BLK
1B	684052	BACK UPHOLSTERY 16 Q2 MED Rx 14	3	560227	2 HOLE TOGGLE BENT PLST
1C	684054	BACK UPHOLSTERY 16 Q2 TALL Rx 17/19			

NOTE: Items 1 and 2 must be ordered for complete back upholstery.

NOTE: Effective on LX/LXI as of 10/24/2008.

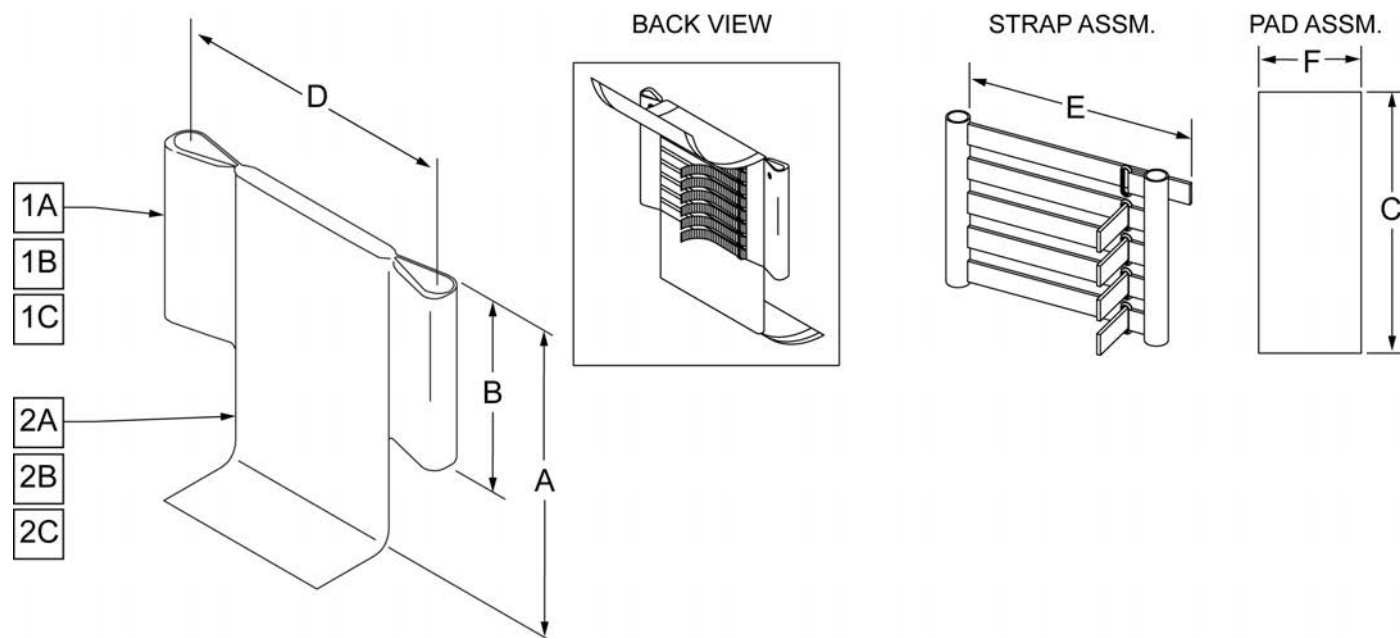

ADJUSTABLE BACK UPHOLSTERY MEASURING GUIDE

HEIGHT TABLE		STRAP ASSM.		PAD ASSM.
	BACK HEIGHT "A"	NO. OF STRAPS	HEIGHT "B"	LENGTH "C"
SHORT/LOW	11-14"	5	6 1/2"	25"
MED	12-16"	6	8 1/2"	29"
TALL	14-18"	8	10 1/2"	33"
X-TALL	16-20"	9	12 1/2"	37"

HEIGHT TABLE		WIDTH "D"					
		SMALL 10-12"	MED 12-14"	LARGE 14-16"	X-LARGE 16-18"	X-LARGE 18-20"	X-LARGE 20-22"
STRAP ASSM	STRAP LENGTH "E"	9 1/2"	13 1/2"	17 1/2"	17 1/2"	21 1/2"	21 1/2"
PAD ASSM	WIDTH "F"	8"	10"	12"	14"	16"	18"



Pos.	Item Number	Description	Remarks
1A	680230	GROWING ADJ BACK UPHOLSTERY 10-12 SHORT ZGS	11" wide-Low
1B	680232	GROWING ADJ BACK UPHOLSTERY 10-12 TALL ZGS	11" wide-Med
1C	680220	GROWING ADJ BACK UPHOLSTERY 10-12 X-TALL ZGS	11" wide-Tall
1A	680233	GROWING ADJ BACK UPHOLSTERY 12-14 SHORT ZGS	12-13" wide-Low
1B	680235	GROWING ADJ BACK UPHOLSTERY 12-14 TALL ZGS	12-13" wide-Med
1C	680221	GROWING ADJ BACK UPHOLSTERY 12-14 X-TALL ZGS	12-13" wide-Tall
1A	680236	GROWING ADJ BACK UPHOLSTERY 14-16 SHORT ZGS	14-15" wide-Low
1B	680238	GROWING ADJ BACK UPHOLSTERY 14-16 TALL ZGS	14-15" wide-Med
1C	680222	GROWING ADJ BACK UPHOLSTERY 14-16 X-TALL ZGS	14-15" wide-Tall
1A	680239	GROWING ADJ BACK UPHOLSTERY 16-18 SHORT ZGS	16-17" wide-Low
1B	680241	GROWING ADJ BACK UPHOLSTERY 16-18 TALL ZGS	16-17" wide-Med
1C	680242	GROWING ADJ BACK UPHOLSTERY 16-18 X-TALL ZGS	16-17" wide-Tall
1A	680243	GROWING ADJ BACK UPHOLSTERY 18-22 SHORT ZGS	18 -22" wide-Low
1B	680245	GROWING ADJ BACK UPHOLSTERY 18-22 TALL ZGS	18 -22" wide-Med
1C	680246	GROWING ADJ BACK UPHOLSTERY 18-22 X-TALL ZGS	18 -22" wide-Tall

Pos.	Item Number	Description	Remarks
2A	684480	ADJ BACK STRAP COVER 10-12 SHORT ZGS	11" wide-Low
2B	684482	ADJ BACK STRAP COVER 10-12 TALL ZGS	11" wide-Med
2C	684497	ADJ BACK STRAP COVER 10-12 X-TALL ZGS	11" wide-Tall
2A	684483	ADJ BACK STRAP COVER 12-14 SHORT ZGS	12-13" wide-Low
2B	684485	ADJ BACK STRAP COVER 12-14 TALL ZGS	12-13" wide-Med
2C	684498	ADJ BACK STRAP COVER 12-14 X-TALL ZGS	12-13" wide-Tall
2A	684486	ADJ BACK STRAP COVER 14-16 SHORT ZGS	14-15" wide-Low
2B	684488	ADJ BACK STRAP COVER 14-16 TALL ZGS	14-15" wide-Med
2C	684499	ADJ BACK STRAP COVER 14-16 X-TALL ZGS	14-15" wide-Tall
2A	684489	ADJ BACK STRAP COVER 16-18 SHORT ZGS	16-17" wide-Low
2B	684491	ADJ BACK STRAP COVER 16-18 TALL ZGS	16-17" wide-Med
2C	684492	ADJ BACK STRAP COVER 16-18 X-TALL ZGS	16-17" wide-Tall
2A	684493	ADJ BACK STRAP COVER 18-20 SHORT ZGS	18 -20" wide-Low
2B	684495	ADJ BACK STRAP COVER 18-20 TALL ZGS	18 -20" wide-Med
2C	684496	ADJ BACK STRAP COVER 18-20 X-TALL ZGS	18 -20" wide-Tall
2A	684510	ADJ BACK STRAP COVER 20 -22" SHORT ZGS	20 -22" wide-Low
2B	684512	ADJ BACK STRAP COVER 20 -22" TALL ZGS	20 -22" wide-Med
2C	684513	ADJ BACK STRAP COVER 20 -22" X-TALL ZGS	20 -22" wide-Tall


Note: Upholstery utilizes a layer of dacron underneath the cover to increase strength.

MEASURING GUIDE


	"A"	BACK HEIGHT
SHORT/ LOW	7 1/2"	11-14"
MEDIUM	10 1/2"	12-16"
HIGH/TALL	13 1/2"	14-18"

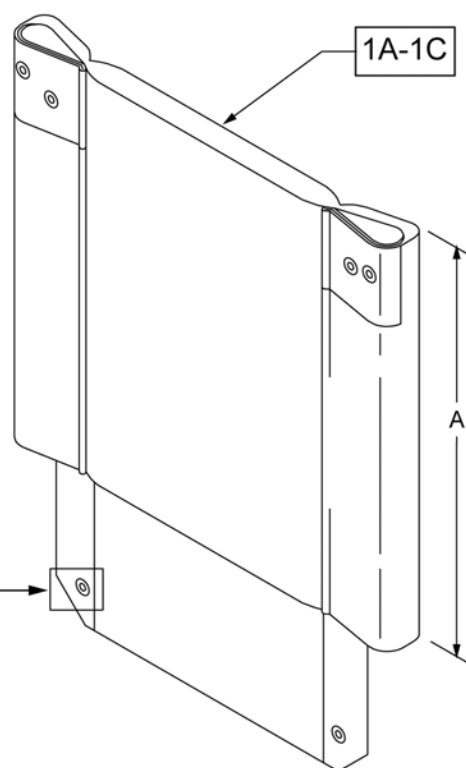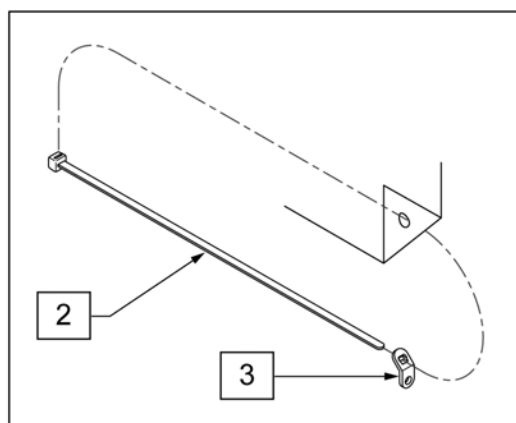


Pos.	Item Number	Description
1A	680370	BACK UPH 18" LOW Q2 H/D
1B	680371	BACK UPH 18" MED Q2 H/D
1C	680372	BACK UPH 18" TALL Q2 H/D
1A	680373	BACK UPH 19" LOW Q2 H/D
1B	680374	BACK UPH 19" MED Q2 H/D
1C	680375	BACK UPH 19" TALL Q2 H/D
1A	680376	BACK UPH 20" LOW Q2 H/D
1B	680377	BACK UPH 20" MED Q2 H/D
1C	680378	BACK UPH 20" TALL Q2 H/D
1A	680379	BACK UPH 21" LOW Q2 H/D
1B	680380	BACK UPH 21" MED Q2 H/D
1C	680381	BACK UPH 21" TALL Q2 H/D
1A	680382	BACK UPH 22" LOW Q2 H/D
1B	680383	BACK UPH 22" MED Q2 H/D
1C	680384	BACK UPH 22" TALL Q2 H/D
2	560410	ZIP TIE BLK
3	560227	2 HOLE TOGGLE BENT PLST

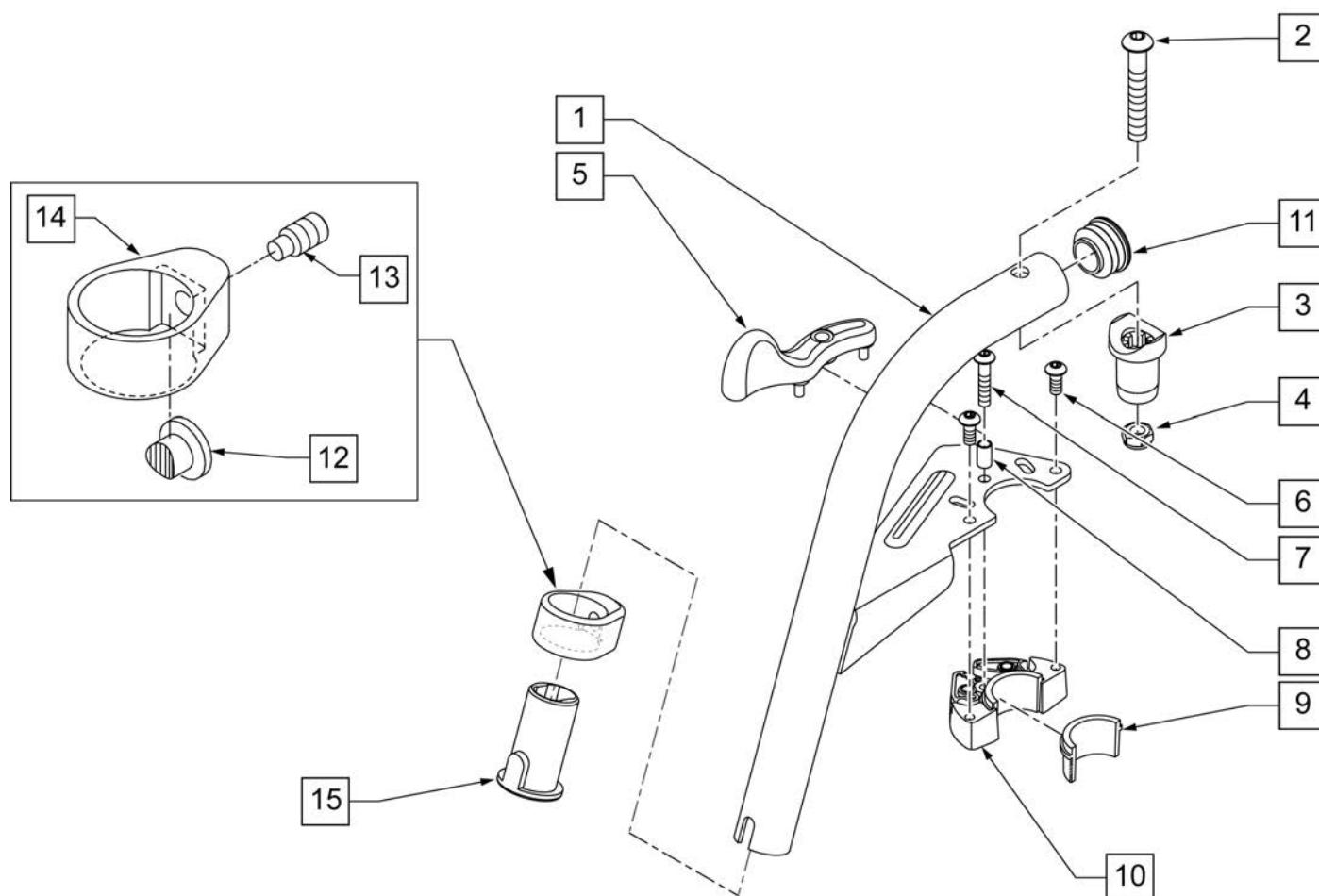


Pos.	Item Number	Description	Remarks
2	106983	5/16-18 X 2 BH C/S SHT THREAD	
3,4	561197	PIVOT SADDLE PLASTIC	
5	052296	SWING I/O LATCH LEVER RIGHT	Right
5	052297	SWING I/O LATCH LEVER LEFT	Left
6	102259-001	BHCS M5 X .8 X 12 W/PL INORG	
7	230494	M5-30 BTTN HD SOC C/S BLK	
8	052054	BUSHING PIVOT HANGER SWING I/O	
11	561062	1" INSERT PLUG .065 RIBS	
15	104160	BUSHING FOOTREST	
1-15	111243S	HANGER 60 Deg S/A ASSM R	SPECIFY COLOR,RIGHT
1-15	111242S	HANGER 60 Deg S/A ASSM L	SPECIFY COLOR,LEFT
12-15	107485	FOOTREST CLAMP ASSM	
6,9,10	120976	ASSY LATCH BLOCK ANTI LIFT	

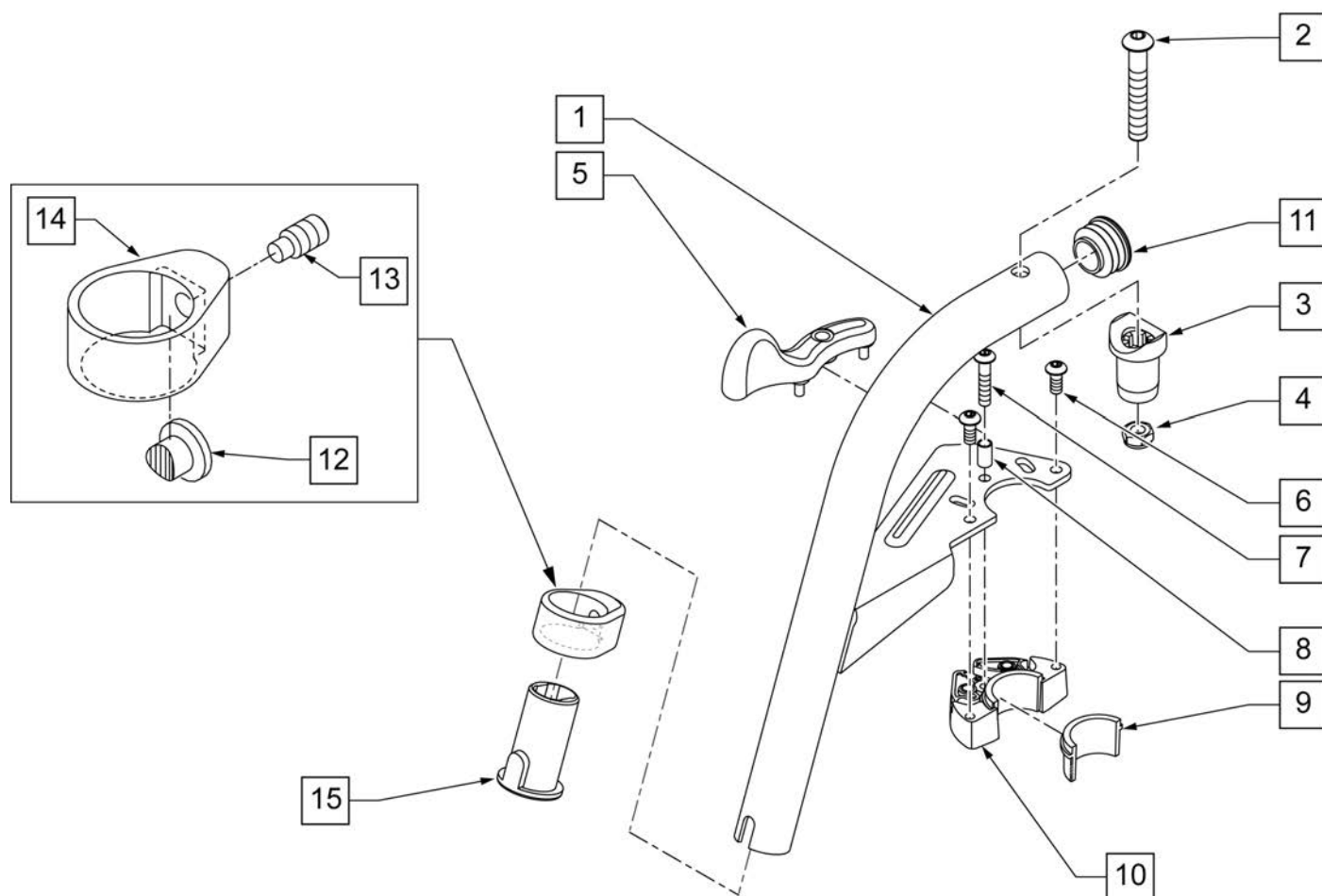


Pos.	Item Number	Description	Remarks
2	106983	5/16-18 X 2 BH C/S SHT THREAD	
3,4	561197	PIVOT SADDLE PLASTIC	
5	052296	SWING I/O LATCH LEVER RIGHT	Right
5	052297	SWING I/O LATCH LEVER LEFT	Left
6	102259-001	BHCS M5 X .8 X 12 W/PL INORG	
7	230494	M5-30 BTTN HD SOC C/S BLK	
8	052054	BUSHING PIVOT HANGER SWING I/O	
11	561062	1" INSERT PLUG .065 RIBS	
15	104160	BUSHING FOOTREST	
1-15	107671S	70 DEG SWING IN/OUT HANGER ASSEMBLY LEFT	SPECIFY COLOR,LEFT
1-15	107670S	70 DEG SWING IN/OUT HANGER ASSEMBLY RIGHT	SPECIFY COLOR,RIGHT
12-15	107485	FOOTREST CLAMP ASSM	
6,9,10	120976	ASSY LATCH BLOCK ANTI LIFT	

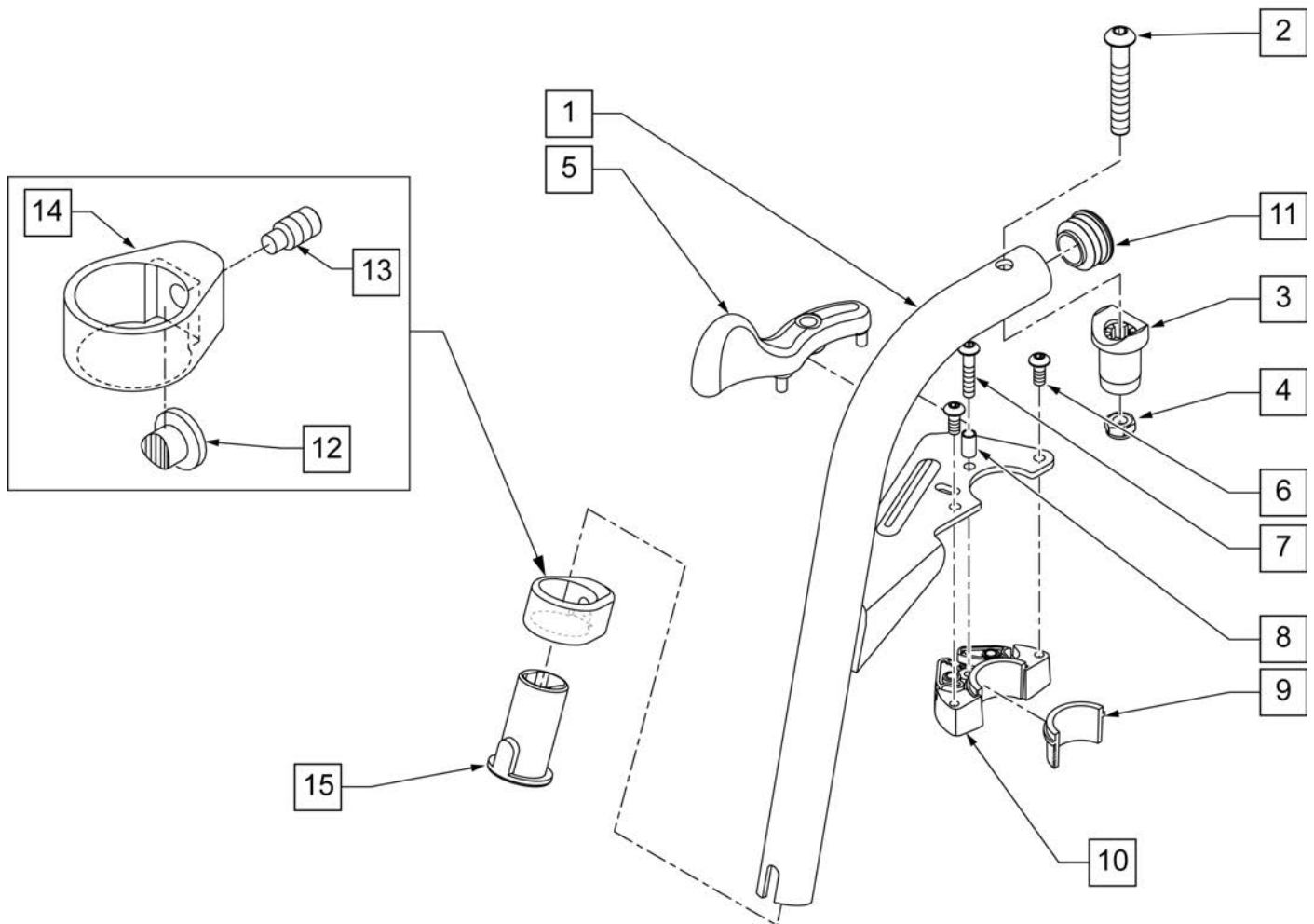

Pos.	Item Number	Description	Remarks
2	106983	5/16-18 X 2 BH C/S SHT THREAD	
3	561197	PIVOT SADDLE PLASTIC	Includes nylock nut
5	052296	SWING I/O LATCH LEVER RIGHT	Right
5	052297	SWING I/O LATCH LEVER LEFT	Left
7	230494	M5-30 BTTN HD SOC C/S BLK	
8	052054	BUSHING PIVOT HANGER SWING I/O	
11	561062	1" INSERT PLUG .065 RIBS	
15	104160	BUSHING FOOTREST	
1-15	107675S	ASSEMBLY LEFT	SPECIFY COLOR,LEFT
1-15	107674S	ASSEMBLY RIGHT	SPECIFY COLOR,RIGHT
12-15	107485	FOOTREST CLAMP ASSM	
6,9,10	120976	ASSY LATCH BLOCK ANTI LIFT	

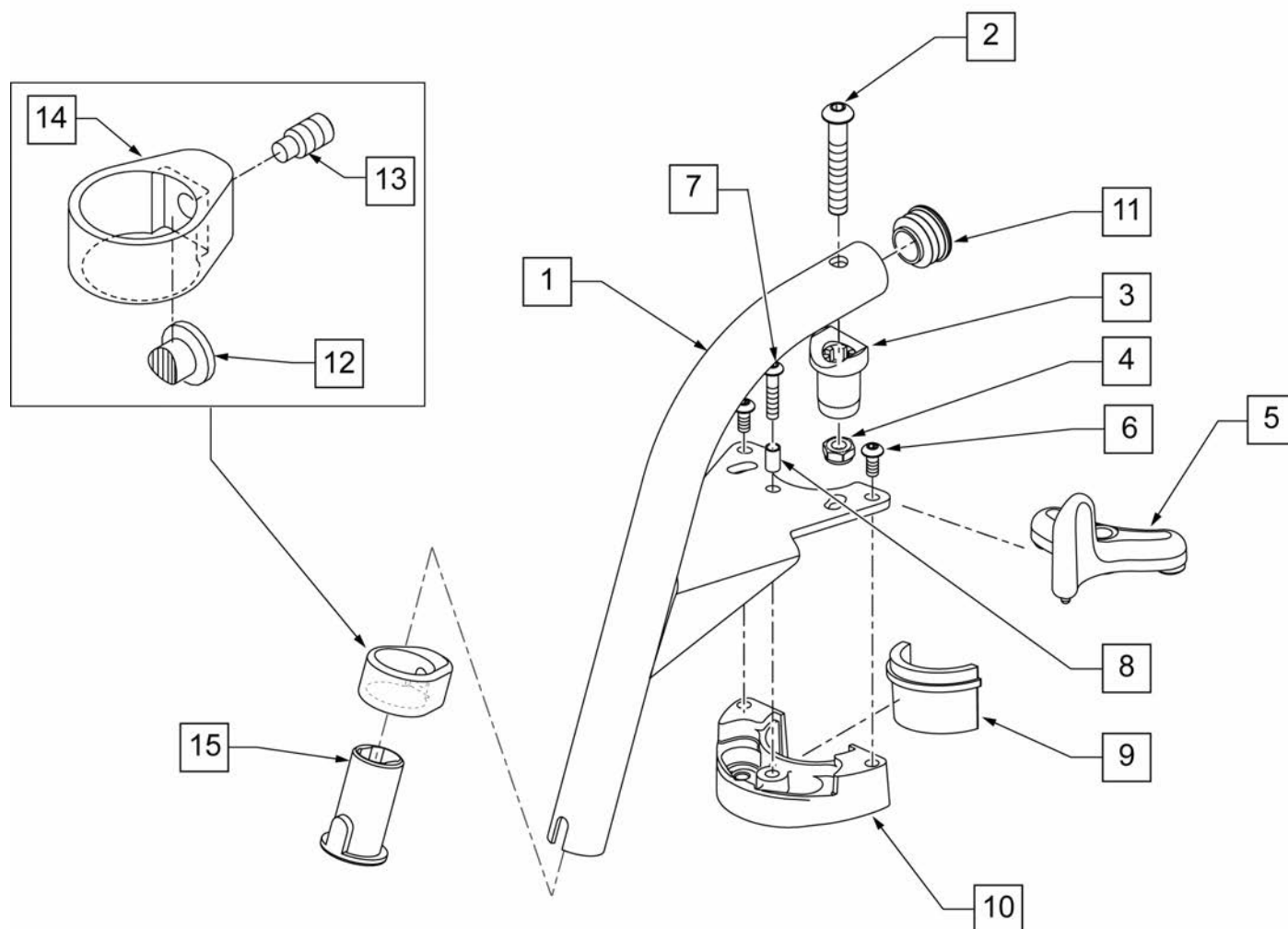

Pos.	Item Number	Description	Remarks
2	106983	5/16-18 X 2 BH C/S SHT THREAD	
3,4	561197	PIVOT SADDLE PLASTIC	
5	052296	SWING I/O LATCH LEVER RIGHT	RIGHT
5	052297	SWING I/O LATCH LEVER LEFT	LEFT
5	052234	SWING I/O LATCH LEVER HEMI	
7	230494	M5-30 BTTN HD SOC C/S BLK	
8	052054	BUSHING PIVOT HANGER SWING I/O	
11	561062	1" INSERT PLUG .065 RIBS	
15	104160	BUSHING FOOTREST	
1-15	111244S	HANGER 70 Deg HEMI WELDMENT R	SPECIFY COLOR-RIGHT
1-15	111245S	HANGER 70 Deg HEMI WELDMENT L	SPECIFY COLOR-LEFT
12-15	107485	FOOTREST CLAMP ASSM	
6,9,10	120976	ASSY LATCH BLOCK ANTI LIFT	

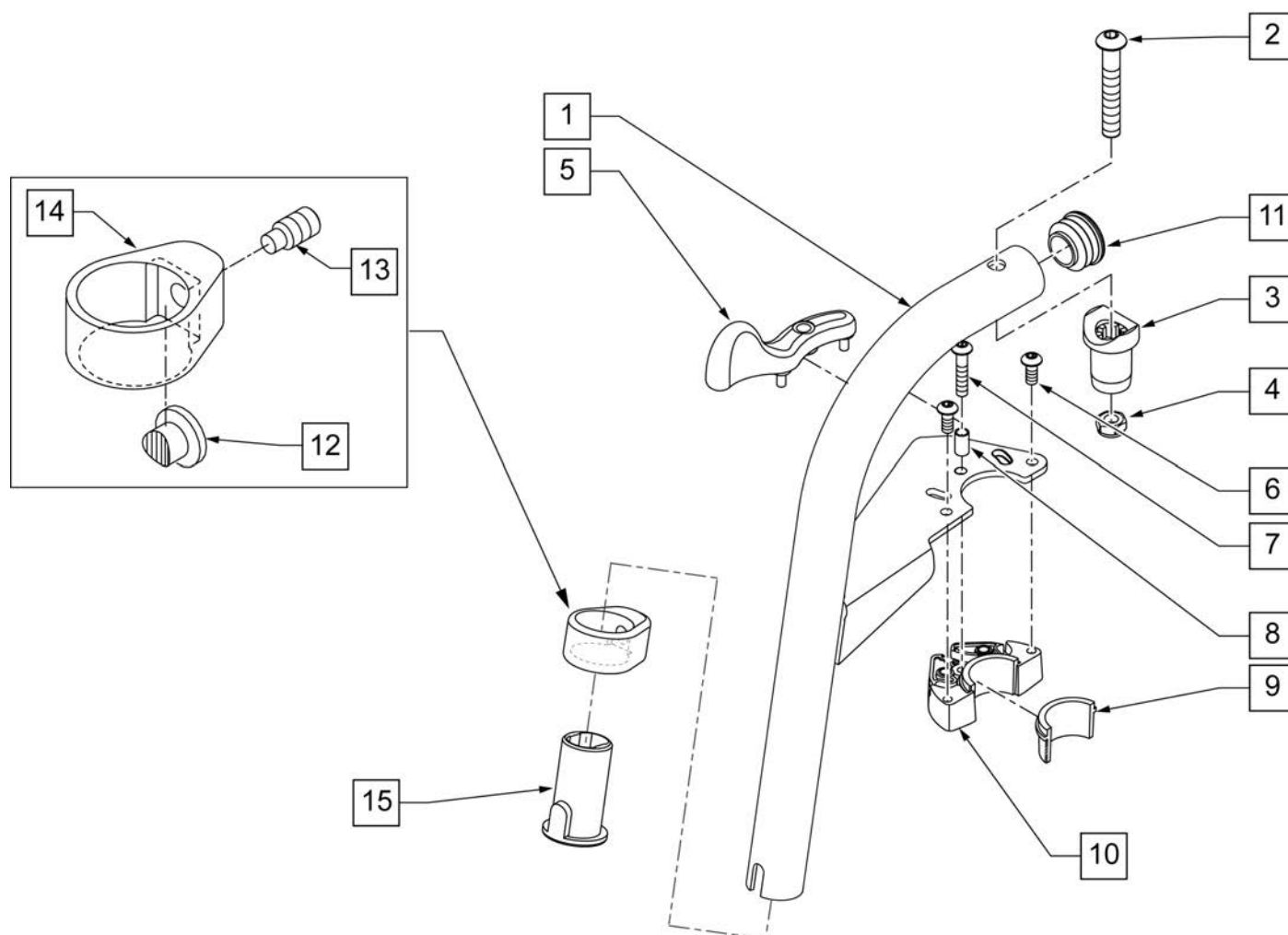

Pos.	Item Number	Description	Remarks
2	106983	5/16-18 X 2 BH C/S SHT THREAD	
3,4	561197	PIVOT SADDLE PLASTIC	
5	052296	SWING I/O LATCH LEVER RIGHT	Right
5	052297	SWING I/O LATCH LEVER LEFT	Left
6	102259-001	BHCS M5 X .8 X 12 W/PL INORG	
7	230494	M5-30 BTTN HD SOC C/S BLK	
8	052054	BUSHING PIVOT HANGER SWING I/O	
11	561062	1" INSERT PLUG .065 RIBS	
15	104160	BUSHING FOOTREST	
1-15	107673S	ASSEMBLY LEFT	SPECIFY COLOR,LEFT
1-15	107672S	ASSEMBLY RIGHT	SPECIFY COLOR,RIGHT
12-15	107485	FOOTREST CLAMP ASSM	
6,9,10	120976	ASSY LATCH BLOCK ANTI LIFT	

NOTE: FOOTPLATE ASSEMBLIES ARE SET TO FOOTREST WIDTH. PLEASE SEE "FOOTPLATE FORMULA" TABLE FOR SEAT WIDTH CONVERSION.

INSET FORMULA
0" Inset (Seat Width) = Footrest Width
1" Inset (Seat Width) -2 = Footrest Width
"V" Hanger (Seat Width) -4 = Footrest Width

Composite Footplates w/ Ext & HL for Swing I/O Hangers
(Order 1 each)

Width	60°, 70° & 70° Hemi Style Hangers (Bent Extension)			80° Style hanger ELR (Straight Extension)			70° V Style Hangers	
	Short	Medium	Long	Short	Medium	Long	Medium	Long
13" Right	107746	107760	107774	107704	107718	107732	107788	107802
13" Left	107753	107767	107781	107711	107725	107739	107795	107809
14" Right	107747	107761	107775	107705	107719	107733	107789	107803
14" Left	107754	107768	107782	107712	107726	107740	107796	107810
15" Right	107748	107762	107776	107706	107720	107734	107790	107804
15" Left	107755	107769	107783	107713	107727	107741	107797	107811
16" Right	107749	107763	107777	107707	107721	107735	107791	107805
16" Left	107756	107770	107784	107714	107728	107742	107798	107812
17" Right	107750	107764	107778	107708	107722	107736	107792	107806
17" Left	107757	107771	107785	107715	107729	107743	107799	107813
18" Right	107751	107765	107779	107709	107723	107737	107793	107807
18" Left	107758	107772	107786	107716	107730	107744	107800	107814
19"-20" Rt	107752	107766	107780	107710	107724	107738	V STYLE IS NOT AVAILABLE IN THESE SEAT WIDTHS	
19"-20" Lt	107759	107773	107787	107717	107731	107745		
21"-22" Rt	111528	111530	111532	111522	111524	111526		
21"-22" Lt	111529	111531	111533	111523	111525	111527		

FOOTPLATE MEASURING GUIDE

PART NO	DESCRIPTION	DIM -A-	DIM -B-
063216-006	FOOTPLATE COMPOSITE 157 R	6 1/8"	6"
063216-012	FOOTPLATE COMPOSITE 157 L	6 1/8"	6"
063216-015	FOOTPLATE COMPOSITE 170 R	6 5/8"	6"
063216-018	FOOTPLATE COMPOSITE 170 L	6 5/8"	6"
063216-005	FOOTPLATE COMPOSITE 182 R	7 1/8"	6"
063216-011	FOOTPLATE COMPOSITE 182 L	7 1/8"	6"
063216-014	FOOTPLATE COMPOSITE 195 R	7 5/8"	6"
063216-017	FOOTPLATE COMPOSITE 195 L	7 5/8"	6"
063216-004	FOOTPLATE COMPOSITE 207 R	8 1/8"	6"
063216-010	FOOTPLATE COMPOSITE 207 L	8 1/8"	6"
063216-013	FOOTPLATE COMPOSITE 220 R	8 5/8"	6 1/4"
063216-016	FOOTPLATE COMPOSITE 220 L	8 5/8"	6 1/4"
063216-003	FOOTPLATE COMPOSITE 232 R	9 1/8"	6 1/4"
063216-009	FOOTPLATE COMPOSITE 232 L	9 1/8"	6 1/4"
063216-002	FOOTPLATE COMPOSITE 257 R	10 1/8"	6 1/4"
063216-008	FOOTPLATE COMPOSITE 257 L	10 1/8"	6 1/4"
063216-001	FOOTPLATE COMPOSITE 282 R	11 1/8"	6 1/4"
063216-007	FOOTPLATE COMPOSITE 282 L	11 1/8"	6 1/4"

Pos.	Item Number	Description	Remarks
1A-5	120660	SHT STRT EXT COMP HDWR	80 deg hangers + ELR only
1B-5	120661	MED STRT EXT COMP HDWR	80 deg hangers + ELR only
1C-5	120662	LNG STRT EXT COMP HDWR	80 deg hangers + ELR only
1D-5	120663	SHT BNT EXT COMP HDWR	60,70,70 Hemi Hangers
1E-5	120664	MED BNT EXT COMP HDWR	60,70,70 Hemi Hangers
1F-5	120665	LNG BNT EXT COMP HDWR	60,70,70 Hemi Hangers
1G-5	120666	SHT V EXT R COMP HDWR	70 deg V hangers only-LEFT
1G-5	120667	SHT V EXT L COMP HDWR	70 deg V hangers only-RIGHT
1H-5	120668	LNG V EXT R COMP HDWR	70 deg V hangers only-LEFT
1H-5	120669	LNG V EXT L COMP HDWR	70 deg V hangers only-RIGHT
2A	063216-006	COMPOSITE FOOTPLATE RIGHT	RIGHT 13" Footrest Width
2A	063216-012	COMPOSITE FOOTPLATE LEFT	LEFT 13" Footrest Width
2B	063216-015	FOOTPLATE COMPOSITE RIGHT	RIGHT 14" Footrest Width

Pos.	Item Number	Description	Remarks
2B	063216-018	FOOTPLATE COMPOSITE LEFT	LEFT 14" Footrest Width
2C	063216-005	COMPOSITE FOOTPLATE RIGHT	RIGHT 15" Footrest Width
2C	063216-011	COMPOSITE FOOTPLATE LEFT	LEFT 15" Footrest Width
2D	063216-014	FOOTPLATE COMPOSITE RIGHT	RIGHT 16" Footrest Width
2D	063216-017	FOOTPLATE COMPOSITE LEFT	LEFT 16" Footrest Width
2E	063216-004	COMPOSITE FOOTPLATE RIGHT	RIGHT 17" Footrest Width
2E	063216-010	COMPOSITE FOOTPLATE LEFT	LEFT 17" Footrest Width
2F	063216-013	FOOTPLATE COMPOSITE RIGHT	RIGHT 18" Footrest Width
2F	063216-016	FOOTPLATE COMPOSITE LEFT	LEFT 18" Footrest Width
2G	063216-003	COMPOSITE FOOTPLATE RIGHT	RIGHT 19" & 20" Footrest Width
2G	063216-009	COMPOSITE FOOTPLATE LEFT	LEFT 19" & 20" Footrest Width
2H	063216-002	FOOTPLATE COMPOSITE RIGHT	RIGHT 21" & 22" Footrest Width
2H	063216-008	FOOTPLATE COMPOSITE LEFT	LEFT 21" & 22" Footrest Width
3	104377	3/4" SADDLE THIN	
4	103175	SHCS M6 X 1 X 40 INORG	
5	104381	NUT HEX NYLOCK M6X1.0	Inserted in bottom of footplate.
6	104328	HEEL LOOP PEG	
7A	104380-001	ADJUSTABLE FOOT REST STRAP	14" wide
7B	104380-002	ADJUSTABLE FOOT REST STRAP	15-16" wide
7C	104380-003	ADJUSTABLE FOOT REST STRAP	17-18" wide
7D	104380-004	ADJUSTABLE FOOT REST STRAP	19-22" wide
8	102268-001	HHCS M6 X 1 X 25	
12	104160	BUSHING FOOTREST	
9-12	107485	FOOTREST CLAMP ASSM	
>>>>		60°,70°,70° HEMI HANGERS - SHORT EXT - RIGHT	
1D,2A,3-6,7A,8	107746	COMP BNT SHT EXT ASM 13" W R	Right- 70D- 13" W- SHORT
1D,2B,3-6,7A,8	107747	COMP BNT SHT EXT ASM 14" W R	Right- 70D- 14" W- SHORT
1D,2C,3-6,7B,8	107748	COMP BNT SHT EXT ASM 15" W R	Right- 70D- 15" W- SHORT
1D,2D,3-6,7B,8	107749	COMP BNT SHT EXT ASM 16" W R	Right- 70D- 16" W- SHORT
1D,2E,3-6,7C,8	107750	COMP BNT SHT EXT ASM 17" W R	Right- 70D- 17" W- SHORT
1D,2F,3-6,7C,8	107751	COMP BNT SHT EXT ASM 18" W R	Right- 70D- 18" W- SHORT
1D,2G,3-6,7D,8	107752	COMP BNT SHT EXT ASM 19 & 20"W R	Right- 70D- 19 & 20" W- SHORT
1D,2H,3-6,7D,8	111528	COMP BNT SHT EXT ASM 21 & 22"W R	Right- 70D- 21 & 22" W- SHORT
>>>>		60°,70°,70° HEMI HANGERS - SHORT EXT - LEFT	
1D,2A,3-6,7A,8	107753	COMP BNT SHT EXT ASM 13" W L	Left- 70D- 13 W- SHORT
1D,2B,3-6,7A,8	107754	COMP BNT SHT EXT ASM 14" W L	Left- 70D- 14 W- SHORT
1D,2C,3-6,7B,8	107755	COMP BNT SHT EXT ASM 15" W L	Left- 70D- 15 W- SHORT
1D,2D,3-6,7B,8	107756	COMP BNT SHT EXT ASM 16" W L	Left- 70D- 16 W- SHORT
1D,2E,3-6,7C,8	107757	COMP BNT SHT EXT ASM 17" W L	Left- 70D- 17 W- SHORT
1D,2F,3-6,7C,8	107758	COMP BNT SHT EXT ASM 18" W L	Left- 70D- 18 W- SHORT
1D,2G,3-6,7D,8	107759	COMP BNT SHT EXT ASM 19 & 20"W L	Left- 70D- 19 & 20" W- SHORT
1D,2H,3-6,7D,8	111529	COMP BNT SHT EXT ASM 21 & 22"W L	Left- 70D- 21 & 22" W- SHORT
>>>>		60°,70°,70° HEMI HANGERS - MED EXT - RIGHT	
1E,2A,3-6,7A,8	107760	COMP BNT MED EXT ASM 13" W R	Right- 70D- 13" W- MED
1E,2B,3-6,7A,8	107761	COMP BNT MED EXT ASM 14" W R	Right- 70D- 14" W- MED
1E,2C,3-6,7B,8	107762	COMP BNT MED EXT ASM 15" W R	Right- 70D- 15" W- MED
1E,2D,3-6,7B,8	107763	COMP BNT MED EXT ASM 16" W R	Right- 70D- 16" W- MED
1E,2E,3-6,7C,8	107764	COMP BNT MED EXT ASM 17" W R	Right- 70D- 17" W- MED
1E,2F,3-6,7C,8	107765	COMP BNT MED EXT ASM 18" W R	Right- 70D- 18" W- MED
1E,2G,3-6,7D,8	107766	COMP BNT MED EXT ASM 19 & 20"W R	Right- 70D- 19 & 20" W- MED
1E,2H,3-6,7D,8	111530	COMP BNT MED EXT ASM 21 & 22"W R	Right- 70D- 21 & 22" W- MED
>>>>		60°,70°,70° HEMI HANGERS - MED EXT - LEFT	
1E,2A,3-6,7A,8	107767	COMP BNT MED EXT ASM 13" W L	Left- 70D- 13 W- MED
1E,2B,3-6,7A,8	107768	COMP BNT MED EXT ASM 14" W L	Left- 70D- 14 W- MED
1E,2C,3-6,7B,8	107769	COMP BNT MED EXT ASM 15" W L	Left- 70D- 15 W- MED
1E,2D,3-6,7B,8	107770	COMP BNT MED EXT ASM 16" W L	Left- 70D- 16 W- MED

Pos.	Item Number	Description	Remarks
1E,2E,3-6,7C,8	107771	COMP BNT MED EXT ASM 17" W L	Left- 70D- 17 W- MED
1E,2F,3-6,7C,8	107772	COMP BNT MED EXT ASM 18" W L	Left- 70D- 18 W- MED
1E,2G,3-6,7D,8	107773	COMP BNT MED EXT ASM 19 & 20"W L	Left- 70D- 19 & 20" W- MED
1E,2H,3-6,7D,8	111531	COMP BNT MED EXT ASM 21 & 22"W L	Left- 70D- 21 & 22" W- MED
>>>>		60°,70°,70° HEMI HANGERS - LONG EXT - RIGHT	
1F,2A,3-6,7A,8	107774	COMP BNT LONG EXT ASM 13" W R	Right- 70D- 13" W- LONG
1F,2B,3-6,7A,8	107775	COMP BNT LONG EXT ASM 14" W R	Right- 70D- 14" W- LONG
1F,2C,3-6,7B,8	107776	COMP BNT LONG EXT ASM 15" W R	Right- 70D- 15" W- LONG
1F,2D,3-6,7B,8	107777	COMP BNT LONG EXT ASM 16" W R	Right- 70D- 16" W- LONG
1F,2E,3-6,7C,8	107778	COMP BNT LONG EXT ASM 17" W R	Right- 70D- 17" W- LONG
1F,2F,3-6,7C,8	107779	COMP BNT LONG EXT ASM 18" W R	Right- 70D- 18" W- LONG
1F,2G,3-6,7D,8	107780	COMP BNT LONG EXT ASM 19 & 20"W R	Right- 70D- 19 & 20" W- LONG
1F,2H,3-6,7D,8	111532	COMP BNT LONG EXT ASM 21 & 22"W R	Right- 70D- 21 & 22" W- LONG
>>>>		60°,70°,70° HEMI HANGERS - LONG EXT - LEFT	
1F,2A,3-6,7A,8	107781	COMP BNT LONG EXT ASM 13" W L	Left- 70D- 13 W- LONG
1F,2B,3-6,7A,8	107782	COMP BNT LONG EXT ASM 14" W L	Left- 70D- 14 W- LONG
1F,2C,3-6,7B,8	107783	COMP BNT LONG EXT ASM 15" W L	Left- 70D- 15 W- LONG
1F,2D,3-6,7B,8	107784	COMP BNT LONG EXT ASM 16" W L	Left- 70D- 16 W- LONG
1F,2E,3-6,7C,8	107785	COMP BNT LONG EXT ASM 17" W L	Left- 70D- 17 W- LONG
1F,2F,3-6,7C,8	107786	COMP BNT LONG EXT ASM 18" W L	Left- 70D- 18 W- LONG
1F,2G,3-6,7D,8	107787	COMP BNT LONG EXT ASM 19 & 20"W L	Left- 70D- 19 & 20" W- LONG
1F,2H,3-6,7D,8	111533	COMP BNT LONG EXT ASM 21 & 22"W L	Left- 70D- 21 & 22" W- LONG
>>>>		70 DEGREE "V" HANGERS - MED EXT - RIGHT	
1G,2A,3-8	107788	COMP V MED EXT ASM 13W R	Right- 70D V hangers only 13 W- MED
1G,2B,3-8	107789	COMP V MED EXT ASM 14W R	Right- 70D V hangers only 14 W- MED
1G,2C,3-8	107790	COMP V MED EXT ASM 15W R	Right- 70D V hangers only 15 W- MED
1G,2D,3-8	107791	COMP V MED EXT ASM 16W R	Right- 70D V hangers only 16 W- MED
1G,2E,3-8	107792	COMP V MED EXT ASM 17W R	Right- 70D V hangers only 17 W- MED
1G,2F,3-8	107793	COMP V MED EXT ASM 18W R	Right- 70D V hangers only 18 W- MED
>>>>		70 DEGREE "V" HANGERS - MED EXT - LEFT	
1G,2A,3-8	107795	COMP V MED EXT ASM 13W L	Left- 70D V hangers only 13 W- MED
1G,2B,3-8	107796	COMP V MED EXT ASM 14W L	Left- 70D V hangers only 14 W- MED
1G,2C,3-8	107797	COMP V MED EXT ASM 15W L	Left- 70D V hangers only 15 W- MED
1G,2D,3-8	107798	COMP V MED EXT ASM 16W L	Left- 70D V hangers only 16 W- MED
1G,2E,3-8	107799	COMP V MED EXT ASM 17W L	Left- 70D V hangers only 17 W- MED
1G,2F,3-8	107800	COMP V MED EXT ASM 18W L	Left- 70D V hangers only 18 W- MED
>>>>		70 DEGREE "V" HANGERS - LONG EXT - RIGHT	
1H,2A,3-8	107802	COMP V LNG EXT ASM 13W R	Right- 70D V hangers only 13 W- LONG
1H,2B,3-8	107803	COMP V LNG EXT ASM 14W R	Right- 70D V hangers only 14 W- LONG
1H,2C,3-8	107804	COMP V LNG EXT ASM 15W R	Right- 70D V hangers only 15 W- LONG
1H,2D,3-8	107805	COMP V LNG EXT ASM 16W R	Right- 70D V hangers only 16 W- LONG
1H,2E,3-8	107806	COMP V LNG EXT ASM 17W R	Right- 70D V hangers only 17 W- LONG
1H,2F,3-8	107807	COMP V LNG EXT ASM 18W R	Right- 70D V hangers only 18 W- LONG
>>>>		70 DEGREE "V" HANGERS - LONG EXT - LEFT	
1H,2A,3-8	107809	COMP V LNG EXT ASM 13W L	Left- 70D V hangers only 13 W- LONG
1H,2B,3-8	107810	COMP V LNG EXT ASM 14W L	Left- 70D V hangers only 14 W- LONG
1H,2C,3-8	107811	COMP V LNG EXT ASM 15W L	Left- 70D V hangers only 15 W- LONG
1H,2D,3-8	107812	COMP V LNG EXT ASM 16W L	Left- 70D V hangers only 16 W- LONG
1H,2E,3-8	107813	COMP V LNG EXT ASM 17W L	Left- 70D V hangers only 17 W- LONG
1H,2F,3-8	107814	COMP V LNG EXT ASM 18W L	Left- 70D V hangers only 18 W- LONG
>>>>		80 DEGREE HANGERS - SHORT EXT - RIGHT	
1A,2A,3-6,7A,8	107704	COMP STRT SHT EXT ASM 13"W R	Right- 80D- 13" W- SHORT
1A,2B,3-6,7A,8	107705	COMP STRT SHT EXT ASM 14"W R	Right- 80D- 14" W- SHORT
1A,2C,3-6,7B,8	107706	COMP STRT SHT EXT ASM 15"W R	Right- 80D- 15" W- SHORT
1A,2D,3-6,7B,8	107707	COMP STRT SHT EXT ASM 16"W R	Right- 80D- 16" W- SHORT
1A,2E,3-6,7C,8	107708	COMP STRT SHT EXT ASM 17"W R	Right- 80D- 17" W- SHORT

Pos.	Item Number	Description	Remarks
1A,2F,3-6,7C,8	107709	COMP STRT SHT EXT ASM 18"W R	Right- 80D- 18" W- SHORT
1A,2G,3-6,7D,8	107710	COMP STRT SHT EXT ASM 19 & 20"W R	Right- 80D- 19 & 20" W- SHORT
1A,2H,3-6,7D,8	111522	COMP STRT SHT EXT ASM 21 & 22"W R	Right- 80D- 21 & 22" W- SHORT
>>>>		80 DEGREE HANGERS - SHORT EXT - LEFT	
1A,2A,3-6,7A,8	107711	COMP STRT SHT EXT ASM 13"W L	LEFT- 80D- 13" W- SHORT
1A,2B,3-6,7A,8	107712	COMP STRT SHT EXT ASM 14"W L	LEFT- 80D- 14" W- SHORT
1A,2C,3-6,7B,8	107713	COMP STRT SHT EXT ASM 15"W L	LEFT- 80D- 15" W- SHORT
1A,2D,3-6,7B,8	107714	COMP STRT SHT EXT ASM 16"W L	LEFT- 80D- 16" W- SHORT
1A,2E,3-6,7C,8	107715	COMP STRT SHT EXT ASM 17"W L	LEFT- 80D- 17" W- SHORT
1A,2F,3-6,7C,8	107716	COMP STRT SHT EXT ASM 18"W L	LEFT- 80D- 18" W- SHORT
1A,2G,3-6,7D,8	107717	COMP STRT SHT EXT ASM 19 & 20"W L	LEFT- 80D- 19 & 20" W- SHORT
1A,2H,3-6,7D,8	111523	COMP STRT SHT EXT ASM 21 & 22"W L	LEFT- 80D- 21 & 22" W- SHORT
>>>>		80 DEGREE HANGERS - MED EXT - RIGHT	
1B,2A,3-6,7A,8	107718	COMP STRT MED EXT ASM 13"W R	Right- 80D- 13" W- MED
1B,2B,3-6,7A,8	107719	COMP STRT MED EXT ASM 14"W R	Right- 80D- 14" W- MED
1B,2C,3-6,7B,8	107720	COMP STRT MED EXT ASM 15"W R	Right- 80D- 15" W- MED
1B,2D,3-6,7B,8	107721	COMP STRT MED EXT ASM 16"W R	Right- 80D- 16" W- MED
1B,2E,3-6,7C,8	107722	COMP STRT MED EXT ASM 17"W R	Right- 80D- 17" W- MED
1B,2F,3-6,7C,8	107723	COMP STRT MED EXT ASM 18"W R	Right- 80D- 18" W- MED
1B,2G,3-6,7D,8	107724	COMP STRT MED EXT ASM 19 & 20"W R	Right- 80D- 19 & 20" W- MED
1B,2H,3-6,7D,8	111524	COMP STRT MED EXT ASM 21 & 22"W R	Right- 80D- 21 & 22" W- MED
>>>>		80 DEGREE HANGERS - MED EXT - LEFT	
1B,2A,3-6,7A,8	107725	COMP STRT MED EXT ASM 13"W L	LEFT- 80D- 13" W- MED
1B,2B,3-6,7A,8	107726	COMP STRT MED EXT ASM 14"W L	LEFT- 80D- 14" W- MED
1B,2C,3-6,7B,8	107727	COMP STRT MED EXT ASM 15"W L	LEFT- 80D- 15" W- MED
1B,2D,3-6,7B,8	107728	COMP STRT MED EXT ASM 16"W L	LEFT- 80D- 16" W- MED
1B,2E,3-6,7C,8	107729	COMP STRT MED EXT ASM 17"W L	LEFT- 80D- 17" W- MED
1B,2F,3-6,7C,8	107730	COMP STRT MED EXT ASM 18"W L	LEFT- 80D- 18" W- MED
1B,2G,3-6,7D,8	107731	COMP STRT MED EXT ASM 19 & 20"W L	LEFT- 80D- 19 & 20" W- MED
1B,2H,3-6,7D,8	111525	COMP STRT MED EXT ASM 21 & 22"W L	LEFT- 80D- 21 & 22" W- MED
>>>>		80 DEGREE HANGERS - LONG EXT - RIGHT	
1C,2A,3-6,7A,8	107732	COMP STRT LONG EXT ASM 13"W R	Right- 80D- 13" W- LONG
1C,2B,3-6,7A,8	107733	COMP STRT LONG EXT ASM 14"W R	Right- 80D- 14" W- LONG
1C,2C,3-6,7B,8	107734	COMP STRT LONG EXT ASM 15"W R	Right- 80D- 15" W- LONG
1C,2D,3-6,7B,8	107735	COMP STRT LONG EXT ASM 16"W R	Right- 80D- 16" W- LONG
1C,2E,3-6,7C,8	107736	COMP STRT LONG EXT ASM 17"W R	Right- 80D- 17" W- LONG
1C,2F,3-6,7C,8	107737	COMP STRT LONG EXT ASM 18"W R	Right- 80D- 18" W- LONG
1C,2G,3-6,7D,8	107738	COMP STRT LONG EXT ASM 19 & 20"W R	Right- 80D- 19 & 20" W- LONG
1C,2H,3-6,7D,8	111526	COMP STRT LONG EXT ASM 21 & 22"W R	Right- 80D- 21 & 22" W- LONG
>>>>		80 DEGREE HANGERS - LONG EXT - LEFT	
1C,2A,3-6,7A,8	107739	COMP STRT LONG EXT ASM 13"W L	LEFT- 80D- 13" W- LONG
1C,2B,3-6,7A,8	107740	COMP STRT LONG EXT ASM 14"W L	LEFT- 80D- 14" W- LONG
1C,2C,3-6,7B,8	107741	COMP STRT LONG EXT ASM 15"W L	LEFT- 80D- 15" W- LONG
1C,2D,3-6,7B,8	107742	COMP STRT LONG EXT ASM 16"W L	LEFT- 80D- 16" W- LONG
1C,2E,3-6,7C,8	107743	COMP STRT LONG EXT ASM 17"W L	LEFT- 80D- 17" W- LONG
1C,2F,3-6,7C,8	107744	COMP STRT LONG EXT ASM 18"W L	LEFT- 80D- 18" W- LONG
1C,2G,3-6,7D,8	107745	COMP STRT LONG EXT ASM 19 & 20"W L	LEFT- 80D- 19 & 20" W- LONG
1C,2H,3-6,7D,8	111527	COMP STRT LONG EXT ASM 21 & 22"W L	LEFT- 80D- 21 & 22" W- LONG

NOTE: FOOTPLATE ASSEMBLIES ARE SET TO FOOTREST WIDTH. PLEASE SEE "FOOTPLATE FORMULA" TABLE FOR SEAT WIDTH CONVERSION.

INSET FORMULA


0" Inset (Seat Width) = Footrest Width
 1" Inset (Seat Width) - 2 = Footrest Width
 "V" Hanger (Seat Width) - 4 = Footrest Width

Composite Angle Adjustable "CAA" Footplate w/ Ext & HL for Swing I/O Hangers


(Order 1 each)

Ext. Tube Width	60°, 70° & 70° Hemi Style Hangers (Bent Extension)			80° Style hanger (Straight Extension)			70° V Style Hangers	
	Short	Medium	Long	Short	Medium	Long	Medium	Long
15" Right	107858	107872	107886	107816	107830	107844	107900	107914
15" Left	107865	107879	107893	107823	107837	107851	107907	107921
16" Right	107859	107873	107887	107817	107831	107845	107901	107915
16" Left	107866	107880	107894	107824	107838	107852	107908	107922
17" Right	107860	107874	107888	107818	107832	107846	107902	107916
17" Left	107867	107881	107895	107825	107839	107853	107909	107923
18" Right	107861	107875	107889	107819	107833	107847	107903	107917
18" Left	107868	107882	107896	107826	107840	107854	107910	107924
19" Right	107862	107876	107890	107820	107834	107848	V STYLE IS NOT	
19" Left	107869	107883	107897	107827	107841	107855		
20" Right	107863	107877	107891	107821	107835	107849		
20" Left	107870	107884	107898	107828	107842	107856		
21"-22" Rt	107864	107878	107892	107822	107836	107850		
21"-22" Lt	107871	107885	107899	107829	107843	107857		

FOOTPLATE MEASURING GUIDE

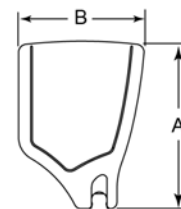

PART NO	DESCRIPTION	DIM -A-	DIM -B-
063216-006	FOOTPLATE COMPOSITE 157 R	6 1/8"	6"
063216-012	FOOTPLATE COMPOSITE 157 L	6 1/8"	6"
063216-015	FOOTPLATE COMPOSITE 170 R	6 5/8"	6"
063216-018	FOOTPLATE COMPOSITE 170 L	6 5/8"	6"
063216-005	FOOTPLATE COMPOSITE 182 R	7 1/8"	6"
063216-011	FOOTPLATE COMPOSITE 182 L	7 1/8"	6"
063216-014	FOOTPLATE COMPOSITE 195 R	7 5/8"	6"
063216-017	FOOTPLATE COMPOSITE 195 L	7 5/8"	6"
063216-004	FOOTPLATE COMPOSITE 207 R	8 1/8"	6"
063216-010	FOOTPLATE COMPOSITE 207 L	8 1/8"	6"
063216-013	FOOTPLATE COMPOSITE 220 R	8 5/8"	6 1/4"
063216-016	FOOTPLATE COMPOSITE 220 L	8 5/8"	6 1/4"
063216-003	FOOTPLATE COMPOSITE 232 R	9 1/8"	6 1/4"
063216-009	FOOTPLATE COMPOSITE 232 L	9 1/8"	6 1/4"
063216-002	FOOTPLATE COMPOSITE 257 R	10 1/8"	6 1/4"
063216-008	FOOTPLATE COMPOSITE 257 L	10 1/8"	6 1/4"
063216-001	FOOTPLATE COMPOSITE 282 R	11 1/8"	6 1/4"
063216-007	FOOTPLATE COMPOSITE 282 L	11 1/8"	6 1/4"

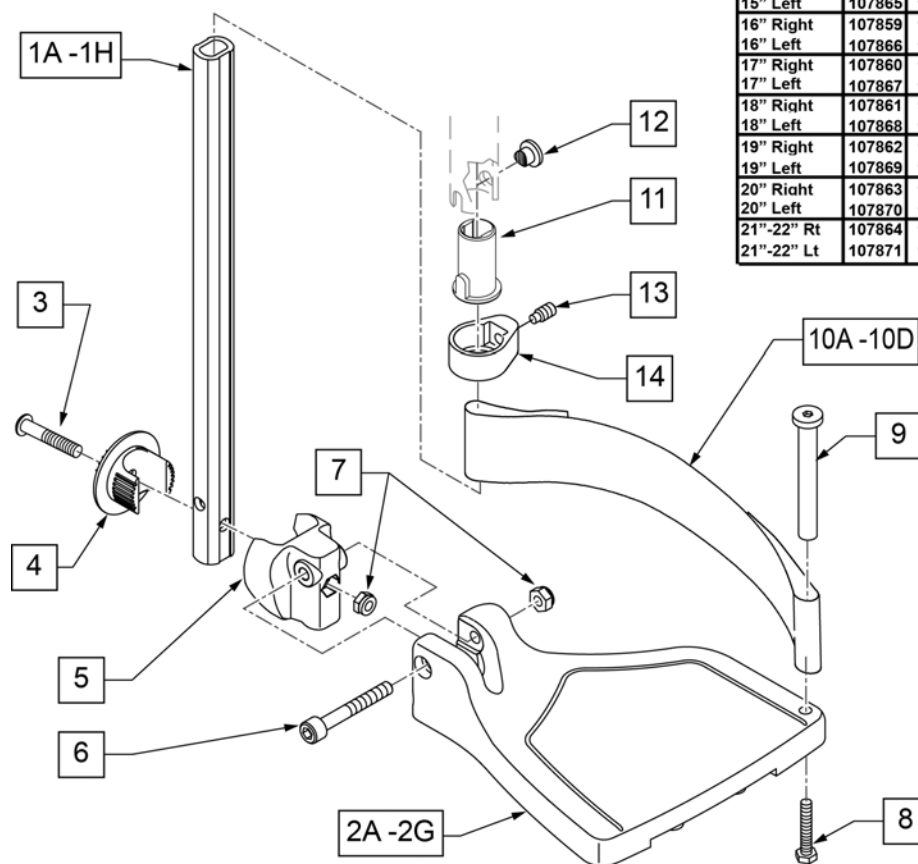

Pos.	Item Number	Description	Remarks
1A,6,7	120660	SHT STRT EXT COMP HDWR	80 deg hangers only
1B,6,7	120661	MED STRT EXT COMP HDWR	80 deg hangers only
1C,6,7	120662	LNG STRT EXT COMP HDWR	80 deg hangers only
1D,6,7	120663	SHT BNT EXT COMP HDWR	60,70,70 Hemi Hangers
1E,6,7	120664	MED BNT EXT COMP HDWR	60,70,70 Hemi Hangers
1F,6,7	120665	LNG BNT EXT COMP HDWR	60,70,70 Hemi Hangers
1G,6,7	120666	SHT V EXT R COMP HDWR	70 deg V hangers only-LEFT
1G,6,7	120667	SHT V EXT L COMP HDWR	70 deg V hangers only-RIGHT
1H,6,7	120668	LNG V EXT R COMP HDWR	70 deg V hangers only-LEFT
1H,6,7	120669	LNG V EXT L COMP HDWR	70 deg V hangers only-RIGHT
2A	063216-006	COMPOSITE FOOTPLATE RIGHT	RIGHT 15" Footrest Width
2A	063216-012	COMPOSITE FOOTPLATE LEFT	LEFT 15" Footrest Width
2B	063216-015	COMPOSITE FOOTPLATE RIGHT	RIGHT 16" Footrest Width

Pos.	Item Number	Description	Remarks
2B	063216-018	COMPOSITE FOOTPLATE LEFT	LEFT 16" Footrest Width
2C	063216-005	COMPOSITE FOOTPLATE RIGHT	RIGHT 17" Footrest Width
2C	063216-011	COMPOSITE FOOTPLATE LEFT	LEFT 17" Footrest Width
2D	063216-014	COMPOSITE FOOTPLATE RIGHT	RIGHT 18" Footrest Width
2D	063216-017	COMPOSITE FOOTPLATE LEFT	LEFT 18" Footrest Width
2E	063216-004	COMPOSITE FOOTPLATE RIGHT	RIGHT 19" Footrest Width
2E	063216-010	COMPOSITE FOOTPLATE LEFT	LEFT 19" Footrest Width
2F	063216-013	COMPOSITE FOOTPLATE RIGHT	RIGHT 20" Footrest Width
2F	063216-016	COMPOSITE FOOTPLATE LEFT	LEFT 20" Footrest Width
2G	063216-003	COMPOSITE FOOTPLATE RIGHT	RIGHT 21" & 22" Footrest Width
2G	063216-009	COMPOSITE FOOTPLATE LEFT	LEFT 21" & 22" Footrest Width
3	108233	BHCS M6 X 1 X 40	
4	054242	ANGLE COVER	
5	054241	ANGLE SEGMENT	
6	103175	SHCS M6 X 1 X 40 INORG	
7	104381	NUT HEX NYLOCK M6X1.0	
8	102268-001	HHCS M6X1X25	Inserted in bottom of footplate.
9	104328	HEEL LOOP PEG	
10A	104380-001	ADJUSTABLE FOOT REST STRAP	15" Footrest Width
10B	104380-002	ADJUSTABLE FOOT REST STRAP	16" Footrest Width
10C	104380-003	ADJUSTABLE FOOT REST STRAP	17-18" Footrest Width
10D	104380-004	ADJUSTABLE FOOT REST STRAP	19-22" Footrest Width
11	104160	BUSHING FOOTREST	
11-14	107485	FOOTREST CLAMP ASSM	
>>>>		60°,70°,70° HEMI HANGERS - SHORT EXT - RIGHT	
1D,2A,3-10A	107858	CAA BNT SHT EXT ASM 15W R	Right- 60° & 60° & 70° 15 W- SHORT
1D,2B,3-10B	107859	CAA BNT SHT EXT ASM 16W R	Right- 60° & 70° 16 W- SHORT
1D,2C,3-10B	107860	CAA BNT SHT EXT ASM 17W R	Right- 60° & 70° 17 W- SHORT
1D,2D,3-10C	107861	CAA BNT SHT EXT ASM 18W R	Right- 60° & 70° 18 W- SHORT
1D,2E,3-10C	107862	CAA BNT SHT EXT ASM 19W R	Right- 60° & 70° 19 W- SHORT
1D,2F,3-10D	107863	CAA BNT SHT EXT ASM 20W R	Right- 60° & 70° 20 W- SHORT
1D,2G,3-10D	107864	CAA BNT SHT EXT ASM 21&22W R	Right- 60° & 70° 21&22 W- SHORT
>>>>		60°,70°,70° HEMI HANGERS - SHORT EXT - LEFT	
1D,2A,3-10A	107865	CAA BNT SHT EXT ASM 15W L	Left- 60° & 70° 15 W- SHORT
1D,2B,3-10B	107866	CAA BNT SHT EXT ASM 16W L	Left- 60° & 70° 16 W- SHORT
1D,2C,3-10B	107867	CAA BNT SHT EXT ASM 17W L	Left- 60° & 70° 17 W- SHORT
1D,2D,3-10C	107868	CAA BNT SHT EXT ASM 18W L	Left- 60° & 70° 18 W- SHORT
1D,2E,3-10C	107869	CAA BNT SHT EXT ASM 19W L	Left- 60° & 70° 19 W- SHORT
1D,2F,3-10D	107870	CAA BNT SHT EXT ASM 20W L	Left- 60° & 70° 20 W- SHORT
1D,2G,3-10D	107871	CAA BNT SHT EXT ASM 21&22W L	Left- 60° & 70° 21&22 W- SHORT
>>>>		60°,70°,70° HEMI HANGERS - MED EXT - RIGHT	
1E,2A,3-10A	107872	CAA BNT MED EXT ASM 15W R	Right- 60° & 70° 15 W- MED
1E,2B,3-10B	107873	CAA BNT MED EXT ASM 16W R	Right- 60° & 70° 16 W- MED
1E,2C,3-10B	107874	CAA BNT MED EXT ASM 17W R	Right- 60° & 70° 17 W- MED
1E,2D,3-10C	107875	CAA BNT MED EXT ASM 18W R	Right- 60° & 70° 18 W- MED
1E,2E,3-10C	107876	CAA BNT MED EXT ASM 19W R	Right- 60° & 70° 19 W- MED
1E,2F,3-10D	107877	CAA BNT MED EXT ASM 20W R	Right- 60° & 70° 20 W- MED
1E,2G,3-10D	107878	CAA BNT MED EXT ASM 21&22W R	Right- 60° & 70° 21&22 W- MED
>>>>		60°,70°,70° HEMI HANGERS - MED EXT - LEFT	
1E,2A,3-10A	107879	CAA BNT MED EXT ASM 15W L	Left- 60° & 70° 15 W- MED
1E,2B,3-10B	107880	CAA BNT MED EXT ASM 16W L	Left- 60° & 70° 16 W- MED
1E,2C,3-10B	107881	CAA BNT MED EXT ASM 17W L	Left- 60° & 70° 17 W- MED
1E,2D,3-10C	107882	CAA BNT MED EXT ASM 18W L	Left- 60° & 70° 18 W- MED
1E,2E,3-10C	107883	CAA BNT MED EXT ASM 19W L	Left- 60° & 70° 19 W- MED
1E,2F,3-10D	107884	CAA BNT MED EXT ASM 20W L	Left- 60° & 70° 20 W- MED
1E,2G,3-10D	107885	CAA BNT MED EXT ASM 21&22W L	Left- 60° & 70° 21&22 W- MED

Pos.	Item Number	Description	Remarks
>>>>		60°,70°,70° HEMI HANGERS - LONG EXT - RIGHT	
1F,2A,3-10A	107886	CAA BNT LNG EXT ASM 15W R	Right- 60° & 70° 15 W- LONG
1F,2B,3-10B	107887	CAA BNT LNG EXT ASM 16W R	Right- 60° & 70° 16 W- LONG
1F,2C,3-10B	107888	CAA BNT LNG EXT ASM 17W R	Right- 60° & 70° 17 W- LONG
1F,2D,3-10C	107889	CAA BNT LNG EXT ASM 18W R	Right- 60° & 70° 18 W- LONG
1F,2E,3-10C	107890	CAA BNT LNG EXT ASM 19W R	Right- 60° & 70° 19 W- LONG
1F,2F,3-10D	107891	CAA BNT LNG EXT ASM 20W R	Right- 60° & 70° 20 W- LONG
1F,2G,3-10D	107892	CAA BNT LNG EXT ASM 21&22W R	Right- 60° & 70° 21&22 W- LONG
>>>>		60°,70°,70° HEMI HANGERS - LONG EXT - LEFT	
1F,2A,3-10A	107893	CAA BNT LNG EXT ASM 15W L	Left- 60° & 70° 15 W- LONG
1F,2B,3-10B	107894	CAA BNT LNG EXT ASM 16W L	Left- 60° & 70° 16 W- LONG
1F,2C,3-10B	107895	CAA BNT LNG EXT ASM 17W L	Left- 60° & 70° 17 W- LONG
1F,2D,3-10C	107896	CAA BNT LNG EXT ASM 18W L	Left- 60° & 70° 18 W- LONG
1F,2E,3-10C	107897	CAA BNT LNG EXT ASM 19W L	Left- 60° & 70° 19 W- LONG
1F,2F,3-10D	107898	CAA BNT LNG EXT ASM 20W L	Left- 60° & 70° 20 W- LONG
1F,2G,3-10D	107899	CAA BNT LNG EXT ASM 21&22W L	Left- 60° & 70° 21&22 W- LONG
>>>>		70° "V" HANGERS - MED EXT - RIGHT	
1G,2A,3-10A	107900	CAA V MED EXT ASM 15W R	Right- 70° V hangers only 15 W- MED
1G,2B,3-10B	107901	CAA V MED EXT ASM 16W R	Right- 70° V hangers only 16 W- MED
1G,2C,3-10B	107902	CAA V MED EXT ASM 17W R	Right- 70° V hangers only 17 W- MED
1G,2D,3-10C	107903	CAA V MED EXT ASM 18W R	Right- 70° V hangers only 18 W- MED
>>>>		70° "V" HANGERS - MED EXT - LEFT	
1G,2A,3-10A	107907	CAA V MED EXT ASM 15W L	Left- 70° V hangers only 15 W- MED
1G,2B,3-10B	107908	CAA V MED EXT ASM 16W L	Left- 70° V hangers only 16 W- MED
1G,2C,3-10B	107909	CAA V MED EXT ASM 17W L	Left- 70° V hangers only 17 W- MED
1G,2D,3-10C	107910	CAA V MED EXT ASM 18W L	Left- 70° V hangers only 18 W- MED
>>>>		70° "V" HANGERS - LONG EXT - RIGHT	
1H,2A,3-10A	107914	CAA V LNG EXT ASM 15W R	Right- 70° V hangers only 15 W- LONG
1H,2B,3-10B	107915	CAA V LNG EXT ASM 16W R	Right- 70° V hangers only 16 W- LONG
1H,2C,3-10B	107916	CAA V LNG EXT ASM 17W R	Right- 70° V hangers only 17 W- LONG
1H,2D,3-10C	107917	CAA V LNG EXT ASM 18W R	Right- 70° V hangers only 18 W- LONG
>>>>		70° "V" HANGERS - LONG EXT - LEFT	
1H,2A,3-10A	107921	CAA V LNG EXT ASM 15W L	Left- 70° V hangers only 15 W- LONG
1H,2B,3-10B	107922	CAA V LNG EXT ASM 16W L	Left- 70° V hangers only 16 W- LONG
1H,2C,3-10B	107923	CAA V LNG EXT ASM 17W L	Left- 70° V hangers only 17 W- LONG
1H,2D,3-10C	107924	CAA V LNG EXT ASM 18W L	Left- 70° V hangers only 18 W- LONG
>>>>		80° HANGERS - SHORT EXT - RIGHT	
1A,2A,3-10A	107816	CAA STRT SHT EXT ASM 15W R	Right- 80° 15 W- SHORT
1A,2B,3-10B	107817	CAA STRT SHT EXT ASM 16W R	Right- 80° 16 W- SHORT
1A,2C,3-10B	107818	CAA STRT SHT EXT ASM 17W R	Right- 80° 17 W- SHORT
1A,2D,3-10C	107819	CAA STRT SHT EXT ASM 18W R	Right- 80° 18 W- SHORT
1A,2E,3-10C	107820	CAA STRT SHT EXT ASM 19W R	Right- 80° 19 W- SHORT
1A,2F,3-10D	107821	CAA STRT SHT EXT ASM 20W R	Right- 80° 20 W- SHORT
1A,2G,3-10D	107822	CAA STRT SHT EXT ASM 21&22W R	Right- 80° 21&22 W- SHORT
>>>>		80° HANGERS - SHORT EXT - LEFT	
1A,2A,3-10A	107823	CAA STRT SHT EXT ASM 15W L	Left- 80° 15 W- SHORT
1A,2B,3-10B	107824	CAA STRT SHT EXT ASM 16W L	Left- 80° 16 W- SHORT
1A,2C,3-10B	107825	CAA STRT SHT EXT ASM 17W L	Left- 80° 17 W- SHORT
1A,2D,3-10C	107826	CAA STRT SHT EXT ASM 18W L	Left- 80° 18 W- SHORT
1A,2E,3-10C	107827	CAA STRT SHT EXT ASM 19W L	Left- 80° 19 W- SHORT
1A,2F,3-10D	107828	CAA STRT SHT EXT ASM 20W L	Left- 80° 20 W- SHORT
1A,2G,3-10D	107829	CAA STRT SHT EXT ASM 21&22W L	Left- 80° 21&22 W- SHORT
>>>>		80° HANGERS - MED EXT - RIGHT	
1B,2A,3-10A	107830	CAA STRT MED EXT ASM 15W R	Right- 80° 15 W- MED
1B,2B,3-10B	107831	CAA STRT MED EXT ASM 16W R	Right- 80° 16 W- MED
1B,2C,3-10B	107832	CAA STRT MED EXT ASM 17W R	Right- 80° 17 W- MED


Pos.	Item Number	Description	Remarks
1B,2D,3-10C	107833	CAA STRT MED EXT ASM 18W R	Right- 80° 18 W- MED
1B,2E,3-10C	107834	CAA STRT MED EXT ASM 19W R	Right- 80° 19 W- MED
1B,2F,3-10D	107835	CAA STRT MED EXT ASM 20W R	Right- 80° 20 W- MED
1B,2G,3-10D	107836	CAA STRT MED EXT ASM 21&22W R	Right- 80° 21&22 W- MED
>>>>		80° HANGERS - MED EXT - LEFT	
1B,2A,3-10A	107837	CAA STRT MED EXT ASM 15W L	Left- 80° 15 W- MED
1B,2B,3-10B	107838	CAA STRT MED EXT ASM 16W L	Left- 80° 16 W- MED
1B,2C,3-10B	107839	CAA STRT MED EXT ASM 17W L	Left- 80° 17 W- MED
1B,2D,3-10C	107840	CAA STRT MED EXT ASM 18W L	Left- 80° 18 W- MED
1B,2E,3-10C	107841	CAA STRT MED EXT ASM 19W L	Left- 80° 19 W- MED
1B,2F,3-10D	107842	CAA STRT MED EXT ASM 20W L	Left- 80° 20 W- MED
1B,2G,3-10D	107843	CAA STRT MED EXT ASM 21&22W L	Left- 80° 21&22 W- MED
>>>>		80° HANGERS - LONG EXT - RIGHT	
1C,2A,3-10A	107844	CAA STRT LNG EXT ASM 15W R	Right- 80° 15 W- LONG
1C,2B,3-10B	107845	CAA STRT LNG EXT ASM 16W R	Right- 80° 16 W- LONG
1C,2C,3-10B	107846	CAA STRT LNG EXT ASM 17W R	Right- 80° 17 W- LONG
1C,2D,3-10C	107847	CAA STRT LNG EXT ASM 18W R	Right- 80° 18 W- LONG
1C,2E,3-10C	107848	CAA STRT LNG EXT ASM 19W R	Right- 80° 19 W- LONG
1C,2F,3-10D	107849	CAA STRT LNG EXT ASM 20W R	Right- 80° 20 W- LONG
1C,2G,3-10D	107850	CAA STRT LNG EXT ASM 21&22W R	Right- 80° 21&22 W- LONG
>>>>		80° HANGERS - LONG EXT - LEFT	
1C,2A,3-10A	107851	CAA STRT LNG EXT ASM 15W L	Left- 80° 15 W- LONG
1C,2B,3-10B	107852	CAA STRT LNG EXT ASM 16W L	Left- 80° 16 W- LONG
1C,2C,3-10B	107853	CAA STRT LNG EXT ASM 17W L	Left- 80° 17 W- LONG
1C,2D,3-10C	107854	CAA STRT LNG EXT ASM 18W L	Left- 80° 18 W- LONG
1C,2E,3-10C	107855	CAA STRT LNG EXT ASM 19W L	Left- 80° 19 W- LONG
1C,2F,3-10D	107856	CAA STRT LNG EXT ASM 20W L	Left- 80° 20 W- LONG
1C,2G,3-10D	107857	CAA STRT LNG EXT ASM 21&22W L	Left- 80° 21&22 W- LONG

NOTE: FOOTPLATE ASSEMBLIES ARE SET TO FOOTREST WIDTH. PLEASE SEE "FOOTPLATE FORMULA" TABLE FOR SEAT WIDTH CONVERSION.


Adult Angle Adjustable "AAA" Footplates w/ Ext & HL for Swing I/O Hangers								
(Order 1 each)								
Ext. Tube	60°, 70° & 70° Hemi Style Hangers (Bent Extension)			80° Style hanger (Straight Extension)			70° V Style Hangers	
Width	Short	Med	Long	Short	Med	Long	Med	Long
10"-11" Rt	NOT AVAILABLE IN THIS WIDTH			NOT AVAILABLE IN THIS WIDTH			111546	111550
10"-11" Lt	NOT AVAILABLE IN THIS WIDTH			NOT AVAILABLE IN THIS WIDTH			111547	111551
12-13" Rt	111540	111542	111544	111534	111536	111538	111548	111552
12-13" Lt	111541	111543	111545	111535	111537	111539	111549	111553
14"-15" Rt	107973	107987	108001	107928	107942	107956	108015	108029
14"-15" Lt	107980	107994	108008	107935	107949	107963	108022	108036
16"-17" Rt	107975	107989	108003	107930	107944	107958	108017	108031
16"-17" Lt	107982	107996	108010	107937	107951	107965	108024	108038
18"-22" Rt	107977	107991	108005	107932	107946	107960	108019	108033
18"-22" Lt	107984	107998	108012	107939	107953	107968	108026	108040

INSET FORMULA
 0" Inset (Seat Width) = Footrest Width
 1" Inset (Seat Width) - 2 = Footrest Width
 "V" Hanger (Seat Width) - 4 = Footrest Width

MEASURING GUIDE

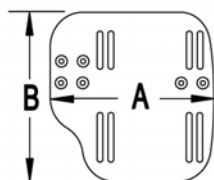

CHAIR WIDTH	"A"	"B"
10" - 11"	3 1/2"	7 1/4"
12" - 13"	4 1/2"	7 3/4"
14" - 15"	5 1/2"	7 3/4"
16" - 17"	6 1/2"	7 3/4"
18" - 20"	7 1/2"	7 3/4"

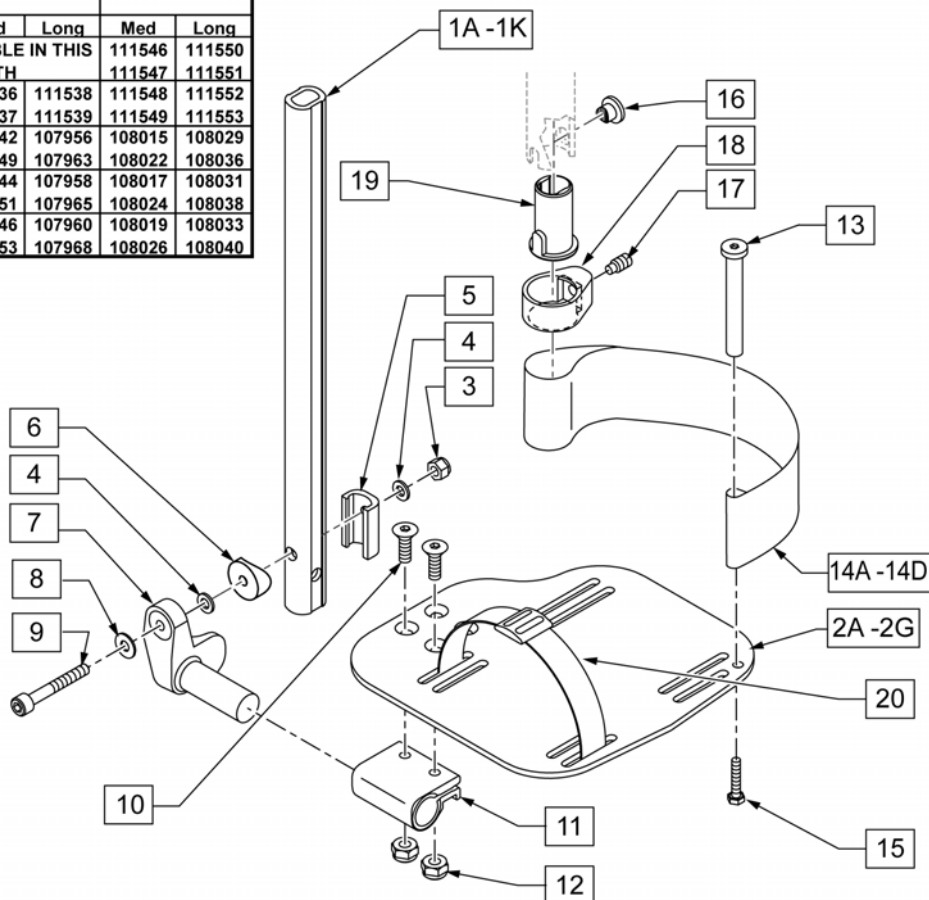

Pos.	Item Number	Description	Remarks
1A,3-5,8,9	120670	SHT STRT EXT AAA HDWR	80 deg Hangers Only
1B,3-5,8,9	120671	MED STRT EXT AAA HDWR	80 deg Hangers Only
1C,3-5,8,9	120672	LNG STRT EXT AAA HDWR	80 deg Hangers Only
1D,3-5,8,9	120673	SHT BNT EXT AAA HDWR	60,70,70 Hemi Hangers
1E,3-5,8,9	120674	MED BNT EXT AAA HDWR	60,70,70 Hemi Hangers
1F,3-5,8,9	120675	LNG BNT EXT AAA HDWR	60,70,70 Hemi Hangers
1G,3-5,8,9	120676	SHT V EXT R AAA HDWR	Left - 70 deg V Hangers Only
1H,3-5,8,9	120677	SHT V EXT L AAA HDWR	Right - 70 deg V Hangers Only
1I,3-5,8,9	120678	LNG V EXT R AAA HDWR	Left - 70 deg V Hangers Only
1J,3-5,8,9	120679	LNG V EXT L AAA HDWR	Right - 70 deg V Hangers Only
2A	500656	ADULT ANG ADJ F/PLT 10/11" R	RIGHT 10-11" Footrest Width
2A	500657	ADULT ANG ADJ F/PLT 10/11" L	LEFT 10-11" Footrest Width
2B	500658	ADULT ANG ADJ F/PLT 12/13" R	RIGHT 12-13" Footrest Width


Pos.	Item Number	Description	Remarks
2B	500659	ADULT ANG ADJ F/PLT 12/13" L	LEFT 12-13" Footrest Width
2C	500660	ADLT ANGL ADJ F/PLT 14/15	RIGHT 14-15" Footrest Width
2C	500661	ADLT ANGL ADJ F/PLT 14/15	LEFT 14-15" Footrest Width
2D	500662	ADLT ANGL ADJ F/PLT 16/17	RIGHT 16-17" Footrest Width
2D	500663	ADLT ANGL ADJ F/PLT 16/17	LEFT 16-17" Footrest Width
2E	500664	ADLT ANGL ADJ F/PLT 18-20	RIGHT 18-22" Footrest Width
2E	500665	ADLT ANGL ADJ F/PLT 18-20	LEFT 18-22" Footrest Width
3	104381	NUT HEX NYLOCK M6X1.0	
4	106944	WASHER FLAT THIN AN 1/4 ROHS	
5	107199	HANGER EXTENSION SADDLE	
6	104377	3/4" SADDLE THIN	
7	541002	FTRST MOUNTING BRKT-3/8 RAD R	RIGHT
7	541003	FTRST MOUNTING BRKT-3/8 RAD L	LEFT
8	106945	WASHER FLAT SAE 1/4 ROHS	
9	104399	SHCS M6X1X45 12 MM PART THD	
10	106978	FHCS 1/4-28 X 7/8 ROHS	
11	550350	3/4" CLMP ADJ FT PLT	
12	106935	NUT NYLOCK THIN 1/4-28 ROHS	
13	104328	POST HEEL LOOP	
14A	104380-001	ADJUSTABLE FOOT REST STRAP	12 - 14" WIDE
14B	104380-002	ADJUSTABLE FOOT REST STRAP	15 - 16" WIDE
14C	104380-003	ADJUSTABLE FOOT REST STRAP	17 - 18" WIDE
14D	104380-004	ADJUSTABLE FOOT REST STRAP	19 - 20" WIDE
15	102268-001	HHCS M6 X 1 X 25	
3-6,8,9	120659	EXT MNT HARDWARE 1 SIDE	
16-19	107485	FOOTREST CLAMP ASSM	
20	740260	TOE-ANKLE STRAP ADJ ADULT FOOTPLATE	
>>>>		80° HANGERS - SHORT EXT - RIGHT	
1A,2B,3-14A,15	111534	AAA STRT SHT EXT ASM 12-13" W R	Right- 80D- 12-13" W- SHORT
1A,2C,3-14B,15	107928	AAA STRT SHT EXT ASM 14-15" W R	Right- 80D- 14-15" W- SHORT
1A,2D,3-14C,15	107930	AAA STRT SHT EXT ASM 16-17" W R	Right- 80D- 16-17" W- SHORT
1A,2E,3-14D,15	107932	AAA STRT SHT EXT ASM 18-22" W R	Right- 80D- 18-22" W- SHORT
>>>>		80° HANGERS - SHORT EXT - LEFT	
1A,2B,3-14A,15	111535	AAA STRT SHT EXT ASM 12-13" W L	Left- 80D- 12-13" W- SHORT
1A,2C,3-14B,15	107935	AAA STRT SHT EXT ASM 14" W L	Left- 80D- 14-15" W- SHORT
1A,2D,3-14C,15	107937	AAA STRT SHT EXT ASM 16" W L	Left- 80D- 16-17" W- SHORT
1A,2E,3-14D,15	107939	AAA STRT SHT EXT ASM 18" W L	Left- 80D- 18-22" W- SHORT
>>>>		80° HANGERS - MED EXT - RIGHT	
1B,2B,3-14A,15	111536	AAA STRT MED EXT ASM 12-13" W R	Right- 80D- 12-13" W- MED
1B,2C,3-14B,15	107942	AAA STRT MED EXT ASM 14-15" W R	Right- 80D- 14-15" W- MED
1B,2D,3-14C,15	107944	AAA STRT MED EXT ASM 16-17" W R	Right- 80D- 16-17" W- MED
1B,2E,3-14D,15	107946	AAA STRT MED EXT ASM 18-22" W R	Right- 80D- 18-22" W- MED
>>>>		80° HANGERS - MED EXT - LEFT	
1B,2B,3-14A,15	111537	AAA STRT MED EXT ASM 12-13" W L	Left- 80D- 12-13" W- MED
1B,2C,3-14B,15	107949	AAA STRT MED EXT ASM 14" W L	Left- 80D- 14-15" W- MED
1B,2D,3-14C,15	107951	AAA STRT MED EXT ASM 16" W L	Left- 80D- 16-17" W- MED
1B,2E,3-14D,15	107953	AAA STRT MED EXT ASM 18" W L	Left- 80D- 18-22" W- MED
>>>>		80° HANGERS - LONG EXT - RIGHT	
1C,2B,3-14A,15	111538	AAA STRT LNG EXT ASM 12-13" W R	Right- 80D- 12-13" W- LONG
1C,2C,3-14B,15	107956	AAA STRT LNG EXT ASM 14 -15" W R	Right- 80D- 14-15" W- LONG
1C,2D,3-14C,15	107958	AAA STRT LNG EXT ASM 16 -17" W R	Right- 80D- 16-17" W- LONG
1C,2E,3-14D,15	107960	AAA STRT LNG EXT ASM 18 -22" W R	Right- 80D- 18-22" W- LONG
>>>>		80° HANGERS - LONG EXT - LEFT	
1C,2B,3-14A,15	111539	AAA STRT LNG EXT ASM 12-13" W L	Left- 80D- 12-13" W- LONG
1C,2C,3-14B,15	107963	AAA STRT LNG EXT ASM 14W L	Left- 80D- 14-15" W- LONG
1C,2D,3-14C,15	107965	AAA STRT LNG EXT ASM 16W L	Left- 80D- 16-17" W- LONG

Pos.	Item Number	Description	Remarks
1C,2E,3-14D,15	107968	AAA STRT LNG EXT ASM 18W L	Left- 80D- 18-22" W- LONG
>>>>		70° HANGERS - SHORT EXT - RIGHT	
1D,2B,3-14A,15	111540	AAA BNT SHT EXT ASM 12-13" W R	Right- 60D,70D,70D HEMI - 12-13" W- SHORT
1D,2C,3-14B,15	107973	AAA BNT SHT EXT ASM 14 -15" W R	Right- 60D,70D,70D HEMI - 14-15" W- SHORT
1D,2D,3-14C,15	107975	AAA BNT SHT EXT ASM 16 -17" W R	Right- 60D,70D,70D HEMI - 16-17" W- SHORT
1D,2E,3-14D,15	107977	AAA BNT SHT EXT ASM 18 -22" W R	Right- 60D,70D,70D HEMI - 18-22" W- SHORT
>>>>		70° HANGERS - SHORT EXT - LEFT	
1D,2B,3-14A,15	111541	AAA BNT SHT EXT ASM 12-13" W L	Left- 60D,70D,70D HEMI - 12-13" W- SHORT
1D,2C,3-14B,15	107980	AAA BNT SHT EXT ASM 14 -15" W L	Left- 60D,70D,70D HEMI - 14-15" W- SHORT
1D,2D,3-14C,15	107982	AAA BNT SHT EXT ASM 16 -17" W L	Left- 60D,70D,70D HEMI - 16-17" W- SHORT
1D,2E,3-14D,15	107984	AAA BNT SHT EXT ASM 18 -22" W L	Left- 60D,70D,70D HEMI - 18-22" W- SHORT
>>>>		70° HANGERS - MED EXT - RIGHT	
1E,2B,3-14A,15	111542	AAA BNT MED EXT ASM 12-13" W R	Right- 60D,70D,70D HEMI - 12-13" W- MED
1E,2C,3-14B,15	107987	AAA BNT MED EXT ASM 14 -15" W R	Right- 60D,70D,70D HEMI - 14-15" W- MED
1E,2D,3-14C,15	107989	AAA BNT MED EXT ASM 16 -17" W R	Right- 60D,70D,70D HEMI - 16-17" W- MED
1E,2E,3-14D,15	107991	AAA BNT MED EXT ASM 18 -22" W R	Right- 60D,70D,70D HEMI - 18-22" W- MED
>>>>		70° HANGERS - MED EXT - LEFT	
1E,2B,3-14A,15	111543	AAA BNT MED EXT ASM 12-13" W L	Left- 60D,70D,70D HEMI - 12-13" W- MED
1E,2C,3-14B,15	107994	AAA BNT MED EXT ASM 14 -15" W L	Left- 60D,70D,70D HEMI - 14-15" W- MED
1E,2D,3-14C,15	107996	AAA BNT MED EXT ASM 16 -17" W L	Left- 60D,70D,70D HEMI - 16-17" W- MED
1E,2E,3-14D,15	107998	AAA BNT MED EXT ASM 18 -22" W L	Left- 60D,70D,70D HEMI - 18-22" W- MED
>>>>		70° HANGERS - LONG EXT - RIGHT	
1F,2B,3-14A,15	111544	AAA BNT LONG EXT ASM 12-13" W R	Right- 60D,70D,70D HEMI - 12-13" W- LONG
1F,2C,3-14B,15	108001	AAA BNT LONG EXT ASM 14 -15" W R	Right- 60D,70D,70D HEMI - 14-15" W- LONG
1F,2D,3-14C,15	108003	AAA BNT LONG EXT ASM 16 -17" W R	Right- 60D,70D,70D HEMI - 16-17" W- LONG
1F,2E,3-14D,15	108005	AAA BNT LONG EXT ASM 18 -22" W R	Right- 60D,70D,70D HEMI - 18-22" W- LONG
>>>>		70° HANGERS - LONG EXT - LEFT	
1F,2B,3-14A,15	111545	AAA BNT LONG EXT ASM 12-13" W L	Left- 60D,70D,70D HEMI - 12-13" W- LONG
1F,2C,3-14B,15	108008	AAA BNT LONG EXT ASM 14 -15" W L	Left- 60D,70D,70D HEMI - 14-15" W- LONG
1F,2D,3-14C,15	108010	AAA BNT LONG EXT ASM 16 -17" W L	Left- 60D,70D,70D HEMI - 16-17" W- LONG
1F,2E,3-14D,15	108012	AAA BNT LONG EXT ASM 18 -22" W L	Left- 60D,70D,70D HEMI - 18-22" W- LONG
>>>>		70° "V" HANGERS - MED EXT - RIGHT	
1G,2A,3-15	111546	AAA V MED EXT ASM 10-11" W R	Right- 70D V hangers only 10-11" W- MED
1G,2B,3-15	111548	AAA V MED EXT ASM 12-13" W R	Right- 70D V hangers only 12-13" W- MED
1G,2C,3-15	108015	AAA V MED EXT ASM 14 -15" W R	Right- 70D V hangers only 14-15" W- MED
1G,2D,3-15	108017	AAA V MED EXT ASM 16 -17" W R	Right- 70D V hangers only 16-18" W- MED
1G,2E,3-15	108019	AAA V MED EXT ASM 18 -18" W R	Right- 70D V hangers only 18-22" W- MED
>>>>		70° "V" HANGERS - MED EXT - LEFT	
1G,2A,3-15	111547	AAA V MED EXT ASM 10-11" W L	Left- 70D V hangers only 10-11" W- MED
1G,2B,3-15	111549	AAA V MED EXT ASM 12-13" W L	Left- 70D V hangers only 12-13" W- MED
1G,2C,3-15	108022	AAA V MED EXT ASM 14 -15" W L	Left- 70D V hangers only 14-15" W- MED
1G,2D,3-15	108024	AAA V MED EXT ASM 16 -17" W L	Left- 70D V hangers only 16-18" W- MED
1G,2E,3-15	108026	AAA V MED EXT ASM 18 -22" W L	Left- 70D V hangers only 18-22" W- MED
>>>>		70° "V" HANGERS - LONG EXT - RIGHT	
1H,2A,3-15	111550	AAA V LONG EXT ASM 10-11" W R	Right- 70D V hangers only 10-11" W- LONG
1H,2B,3-15	111552	AAA V LONG EXT ASM 12-13" W R	Right- 70D V hangers only 12-13" W- LONG
1H,2C,3-15	108029	AAA V LONG EXT ASM 14 -15" W R	Right- 70D V hangers only 14-15" W- LONG
1H,2D,3-15	108031	AAA V LONG EXT ASM 16 -17" W R	Right- 70D V hangers only 16-18" W- LONG
1H,2E,3-15	108033	AAA V LONG EXT ASM 18 -22" W R	Right- 70D V hangers only 18-22" W- LONG
>>>>		70° "V" HANGERS - LONG EXT - LEFT	
1H,2A,3-15	111551	AAA V LONG EXT ASM 10-11" W L	Left- 70D V hangers only 10-11" W- LONG
1H,2B,3-15	111553	AAA V LONG EXT ASM 12-13" W L	Left- 70D V hangers only 12-13" W- LONG
1H,2C,3-15	108036	AAA V LONG EXT ASM 14 -15" W L	Left- 70D V hangers only 14-15" W- LONG
1H,2D,3-15	108038	AAA V LONG EXT ASM 16 -17" W L	Left- 70D V hangers only 16-18" W- LONG
1H,2E,3-15	108040	AAA V LONG EXT ASM 18 -22" W L	Left- 70D V hangers only 18-22" W- LONG


NOTE: FOOTPLATE ASSEMBLIES ARE SET TO FOOTREST WIDTH. PLEASE SEE "FOOTPLATE FORMULA" TABLE FOR SEAT WIDTH CONVERSION.

FOOTPLATE FORMULA

0" Inset (Seat Width) = Footrest Width
 1" Inset (Seat Width) -2 = Footrest Width
 V Hanger (Seat Width) -4 = Footrest Width

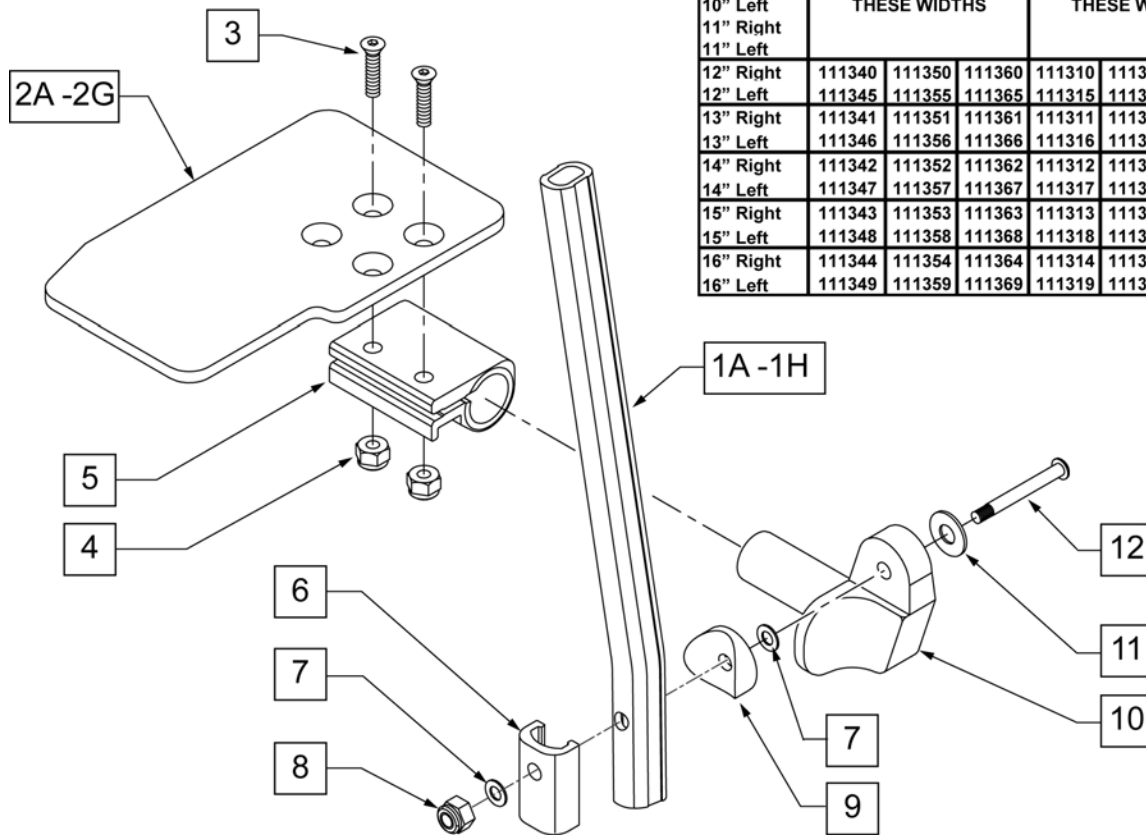

MEASURING GUIDE

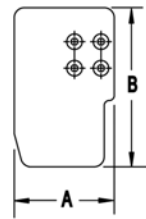

CHAIR WIDTH	"A"	"B"
10" W	3.55"	6.00"
11" W	4.05"	6.00"
12" W	4.55"	6.00"
13" W	5.05"	6.00"
14" W	5.55"	6.00"
15" W	6.05"	6.00"
16" W	6.55"	6.00"


Kids Angle Adjustable "KAA" Footplates w/ Ext. for Swing I/O									
Hangers—leg strap not included									
(Order 1 each)									
Ext. Tube	60°, 70° & 70° Hemi Style Hangers (Bent Extension)			80° Style hanger (Straight Extension)			70° V Style Hangers		
	Short	Med	Long	Short	Med	Long	Med	Long	
Width	NOT AVAILABLE IN THESE WIDTHS			NOT AVAILABLE IN THESE WIDTHS					
10" Right							111370	111384	
10" Left							111377	111391	
11" Right							111371	111385	
11" Left							111378	111392	
12" Right	111340	111350	111360	111310	111320	111330	111372	111386	
12" Left	111345	111355	111365	111315	111325	111335	111379	111393	
13" Right	111341	111351	111361	111311	111321	111331	111373	111387	
13" Left	111346	111356	111366	111316	111326	111336	111380	111394	
14" Right	111342	111352	111362	111312	111322	111332	111374	111388	
14" Left	111347	111357	111367	111317	111327	111337	111381	111395	
15" Right	111343	111353	111363	111313	111323	111333	111375	111389	
15" Left	111348	111358	111368	111318	111328	111338	111382	111396	
16" Right	111344	111354	111364	111314	111324	111334	111376	111390	
16" Left	111349	111359	111369	111319	111329	111339	111383	111397	

Pos.	Item Number	Description	Remarks
1A	107119	TUBE FOOTREST EXT STRAIGHT SHORT	80 deg hangers only
1B	107120	TUBE FOOTREST EXT STRAIGHT MED	80 deg hangers only
1C	107121	TUBE FOOTREST EXT STRAIGHT LONG	80 deg hangers only
1D	107126	TUBE FOOTREST EXT SHORT	60,70,70 Hemi Hangers
1E	107127	TUBE FOOTREST EXT MED	60,70,70 Hemi Hangers
1F	107128	TUBE FOOTREST EXT LONG	60,70,70 Hemi Hangers
1G	106907	EXT TUBE V ONLY D2 MED LEFT	70 deg V hangers only-LEFT
1G	106908	EXT TUBE V ONLY D2 MED RIGHT	70 deg V hangers only-RIGHT
1H	106909	EXT TUBE V ONLY D2 X-LONG LEFT	70 deg V hangers only-LEFT
1H	106910	EXT TUBE V ONLY D2 X-LONG RIGHT	70 deg V hangers only-RIGHT
2A	500635	KIDS ANGL ADJ F/PLT 10" R	RIGHT -10" Footrest Width
2A	500636	KIDS ANGL ADJ F/PLT 10" L	LEFT -10" Footrest Width
2B	500637	KIDS ANGL ADJ F/PLT 11" R	RIGHT -11" Footrest Width

Pos.	Item Number	Description	Remarks
2B	500638	KIDS ANGL ADJ F/PLT 11" L	LEFT -11" Footrest Width
2C	500639	KIDS ANGL ADJ F/PLT 12" R	RIGHT -12" Footrest Width
2C	500640	KIDS ANGL ADJ F/PLT 12" L	LEFT -12" Footrest Width
2D	500641	KIDS ANGL ADJ F/PLT 13" R	RIGHT -13" Footrest Width
2D	500642	KIDS ANGL ADJ F/PLT 13" L	LEFT -13" Footrest Width
2E	500643	KIDS ANGL ADJ F/PLT 14" R	RIGHT -14" Footrest Width
2E	500644	KIDS ANGL ADJ F/PLT 14" L	LEFT -14" Footrest Width
2F	500645	KIDS ANGL ADJ F/PLT 15" R	RIGHT -15" Footrest Width
2F	500646	KIDS ANGL ADJ F/PLT 15" L	LEFT -15" Footrest Width
2G	500647	KIDS ANGL ADJ F/PLT 16" R	RIGHT -16" Footrest Width
2G	500648	KIDS ANGL ADJ F/PLT 16" L	LEFT -16" Footrest Width
3	106978	FHCS 1/4 -28 x 7/8 ROHS	
4	106935	NUT NYLOCK THIN 1/4-28 ROHS	
5	550350	3/4" CLAMP ADJ FT PLT	
6	107199	HANGER EXTENSION SADDLE	
7	106944	WASHER FLAT THIN AN 1/4 ROHS	
8	104381	NUT HEX NYLOCK M6 x 1.0	
9	104377	3/4" SADDLE THIN	
10	541002	FOOTREST MOUNTING BRKT 3/8 RAD R	RIGHT
10	541003	FOOTREST MOUNTING BRKT 3/8 RAD L	LEFT
11	106945	WASHER FLAT SAE 1/4 ROHS	
12	104399	SHCS M6 x 1 x 45 12mm PART THD	
1A,2C,3-12	111310	KAA STRT SHT EXT ASM 12W R	RIGHT- 80D- 12 W- SHORT
1A,2D,3-12	111311	KAA STRT SHT EXT ASM 13W R	RIGHT- 80D- 13 W- SHORT
1A,2E,3-12	111312	KAA STRT SHT EXT ASM 14W R	RIGHT- 80D- 14 W- SHORT
1A,2F,3-12	111313	KAA STRT SHT EXT ASM 15W R	RIGHT- 80D- 15 W- SHORT
1A,2G,3-12	111314	KAA STRT SHT EXT ASM 16W R	RIGHT- 80D- 16 W- SHORT
1A,2C,3-12	111315	KAA STRT SHT EXT ASM 12W L	LEFT- 80D- 12 W- SHORT
1A,2D,3-12	111316	KAA STRT SHT EXT ASM 13W L	LEFT- 80D- 13 W- SHORT
1A,2E,3-12	111317	KAA STRT SHT EXT ASM 14W L	LEFT- 80D- 14 W- SHORT
1A,2F,3-12	111318	KAA STRT SHT EXT ASM 15W L	LEFT- 80D- 15 W- SHORT
1A,2G,3-12	111319	KAA STRT SHT EXT ASM 16W L	LEFT- 80D- 16 W- SHORT
1B,2C,3-12	111320	KAA STRT MED EXT ASM 12W R	RIGHT- 80D- 12 W- MED
1B,2D,3-12	111321	KAA STRT MED EXT ASM 13W R	RIGHT- 80D- 13 W- MED
1B,2E,3-12	111322	KAA STRT MED EXT ASM 14W R	RIGHT- 80D- 14 W- MED
1B,2F,3-12	111323	KAA STRT MED EXT ASM 15W R	RIGHT- 80D- 15 W- MED
1B,2G,3-12	111324	KAA STRT MED EXT ASM 16W R	RIGHT- 80D- 16 W- MED
1B,2C,3-12	111325	KAA STRT MED EXT ASM 12W L	LEFT- 80D- 12 W- MED
1B,2D,3-12	111326	KAA STRT MED EXT ASM 13W L	LEFT- 80D- 13 W- MED
1B,2E,3-12	111327	KAA STRT MED EXT ASM 14W L	LEFT- 80D- 14 W- MED
1B,2F,3-12	111328	KAA STRT MED EXT ASM 15W L	LEFT- 80D- 15 W- MED
1B,2G,3-12	111329	KAA STRT MED EXT ASM 16W L	LEFT- 80D- 16 W- MED
1C,2C,3-12	111330	KAA STRT LNG EXT ASM 12W R	RIGHT- 80D- 12 W- LONG
1C,2D,3-12	111331	KAA STRT LNG EXT ASM 13W R	RIGHT- 80D- 13 W- LONG
1C,2E,3-12	111332	KAA STRT LNG EXT ASM 14W R	RIGHT- 80D- 14 W- LONG
1C,2F,3-12	111333	KAA STRT LNG EXT ASM 15W R	RIGHT- 80D- 15 W- LONG
1C,2G,3-12	111334	KAA STRT LNG EXT ASM 16W R	RIGHT- 80D- 16 W- LONG
1C,2C,3-12	111335	KAA STRT LNG EXT ASM 12W L	LEFT- 80D- 12 W- LONG
1C,2D,3-12	111336	KAA STRT LNG EXT ASM 13W L	LEFT- 80D- 13 W- LONG
1C,2E,3-12	111337	KAA STRT LNG EXT ASM 14W L	LEFT- 80D- 14 W- LONG
1C,2F,3-12	111338	KAA STRT LNG EXT ASM 15W L	LEFT- 80D- 15 W- LONG
1C,2G,3-12	111339	KAA STRT LNG EXT ASM 16W L	LEFT- 80D- 16 W- LONG
1D,2C,3-12	111340	KAA BNT SHT EXT ASM 12W R	RIGHT- 60 & 70D- 12 W- SHORT
1D,2D,3-12	111341	KAA BNT SHT EXT ASM 13W R	RIGHT- 60 & 70D- 13 W- SHORT
1D,2E,3-12	111342	KAA BNT SHT EXT ASM 14W R	RIGHT- 60 & 70D- 14 W- SHORT
1D,2F,3-12	111343	KAA BNT SHT EXT ASM 15W R	RIGHT- 60 & 70D- 15 W- SHORT

Pos.	Item Number	Description	Remarks
1D,2G,3-12	111344	KAA BNT SHT EXT ASM 16W R	RIGHT- 60 & 70D- 16 W- SHORT
1D,2C,3-12	111345	KAA BNT SHT EXT ASM 12W L	LEFT- 70D- 12 W- SHORT
1D,2D,3-12	111346	KAA BNT SHT EXT ASM 13W L	LEFT- 70D- 13 W- SHORT
1D,2E,3-12	111347	KAA BNT SHT EXT ASM 14W L	LEFT- 70D- 14 W- SHORT
1D,2F,3-12	111348	KAA BNT SHT EXT ASM 15W L	LEFT- 70D- 15 W- SHORT
1D,2G,3-12	111349	KAA BNT SHT EXT ASM 16W L	LEFT- 70D- 16 W- SHORT
1E,2C,3-12	111350	KAA BNT MED EXT ASM 12W R	RIGHT- 70D- 12 W- MED
1E,2D,3-12	111351	KAA BNT MED EXT ASM 13W R	RIGHT- 70D- 13 W- MED
1E,2E,3-12	111352	KAA BNT MED EXT ASM 14W R	RIGHT- 60 & 70D- 14 W- MED
1E,2F,3-12	111353	KAA BNT MED EXT ASM 15W R	RIGHT- 60 & 70D- 15 W- MED
1E,2G,3-12	111354	KAA BNT MED EXT ASM 16W R	RIGHT- 60 & 70D- 16 W- MED
1E,2C,3-12	111355	KAA BNT MED EXT ASM 12W L	LEFT- 60 & 70D- 12 W- MED
1E,2D,3-12	111356	KAA BNT MED EXT ASM 13W L	LEFT- 60 & 70D- 13 W- MED
1E,2E,3-12	111357	KAA BNT MED EXT ASM 14W L	LEFT- 60 & 70D- 14 W- MED
1E,2F,3-12	111358	KAA BNT MED EXT ASM 15W L	LEFT- 60 & 70D- 15 W- MED
1E,2G,3-12	111359	KAA BNT MED EXT ASM 16W L	LEFT- 60 & 70D- 16 W- MED
1F,2C,3-12	111360	KAA BNT LNG EXT ASM 12W R	RIGHT- 60 & 70D- 12 W- LONG
1F,2D,3-12	111361	KAA BNT LNG EXT ASM 13W R	RIGHT- 60 & 70D- 13 W- LONG
1F,2E,3-12	111362	KAA BNT LNG EXT ASM 14W R	RIGHT- 60 & 70D- 14 W- LONG
1F,2F,3-12	111363	KAA BNT LNG EXT ASM 15W R	RIGHT- 60 & 70D- 15 W- LONG
1F,2G,3-12	111364	KAA BNT LNG EXT ASM 16W R	RIGHT- 60 & 70D- 16 W- LONG
1F,2C,3-12	111365	KAA BNT LNG EXT ASM 12W L	LEFT- 60 & 70D- 12 W- LONG
1F,2D,3-12	111366	KAA BNT LNG EXT ASM 13W L	LEFT- 60 & 70D- 13 W- LONG
1F,2E,3-12	111367	KAA BNT LNG EXT ASM 14W L	LEFT- 60 & 70D- 14 W- LONG
1F,2F,3-12	111368	KAA BNT LNG EXT ASM 15W L	LEFT- 60 & 70D- 15 W- LONG
1F,2G,3-12	111369	KAA BNT LNG EXT ASM 16W L	LEFT- 60 & 70D- 16 W- LONG
1G,2A,3-12	111370	KAA V MED EXT ASM 10W R	RIGHT- 70D V hangers only 10 W- MED
1G,2B,3-12	111371	KAA V MED EXT ASM 11W R	RIGHT- 70D V hangers only 11 W- MED
1G,2C,3-12	111372	KAA V MED EXT ASM 12W R	RIGHT- 70D V hangers only 12 W- MED
1G,2D,3-12	111373	KAA V MED EXT ASM 13W R	RIGHT- 70D V hangers only 13 W- MED
1G,2E,3-12	111374	KAA V MED EXT ASM 14W R	RIGHT- 70D V hangers only 14 W- MED
1G,2F,3-12	111375	KAA V MED EXT ASM 15W R	RIGHT- 70D V hangers only 15 W- MED
1G,2G,3-12	111376	KAA V MED EXT ASM 16W R	RIGHT- 70D V hangers only 16 W- MED
1G,2A,3-12	111377	KAA V MED EXT ASM 10W L	LEFT- 70D V hangers only 10 W- MED
1G,2B,3-12	111378	KAA V MED EXT ASM 11W L	LEFT- 70D V hangers only 11 W- MED
1G,2C,3-12	111379	KAA V MED EXT ASM 12W L	LEFT- 70D V hangers only 12 W- MED
1G,2D,3-12	111380	KAA V MED EXT ASM 13W L	LEFT- 70D V hangers only 13 W- MED
1G,2E,3-12	111381	KAA V MED EXT ASM 14W L	LEFT- 70D V hangers only 14 W- MED
1G,2F,3-12	111382	KAA V MED EXT ASM 15W L	LEFT- 70D V hangers only 15 W- MED
1G,2G,3-12	111383	KAA V MED EXT ASM 16W L	LEFT- 70D V hangers only 16 W- MED
1H,2A,3-12	111384	KAA V LNG EXT ASM 10W R	RIGHT- 70D V hangers only 10 W- LONG
1H,2B,3-12	111385	KAA V LNG EXT ASM 11W R	RIGHT- 70D V hangers only 11 W- LONG
1H,2C,3-12	111386	KAA V LNG EXT ASM 12W R	RIGHT- 70D V hangers only 12 W- LONG
1H,2D,3-12	111387	KAA V LNG EXT ASM 13W R	RIGHT- 70D V hangers only 13 W- LONG
1H,2E,3-12	111388	KAA V LNG EXT ASM 14W R	RIGHT- 70D V hangers only 14 W- LONG
1H,2F,3-12	111389	KAA V LNG EXT ASM 15W R	RIGHT- 70D V hangers only 15 W- LONG
1H,2G,3-12	111390	KAA V LNG EXT ASM 16W R	RIGHT- 70D V hangers only 16 W- LONG
1H,2A,3-12	111391	KAA V LNG EXT ASM 10W L	LEFT- 70D V hangers only 10 W- LONG
1H,2B,3-12	111392	KAA V LNG EXT ASM 11W L	LEFT- 70D V hangers only 11 W- LONG
1H,2C,3-12	111393	KAA V LNG EXT ASM 12W L	LEFT- 70D V hangers only 12 W- LONG
1H,2D,3-12	111394	KAA V LNG EXT ASM 13W L	LEFT- 70D V hangers only 13 W- LONG
1H,2E,3-12	111395	KAA V LNG EXT ASM 14W L	LEFT- 70D V hangers only 14 W- LONG
1H,2F,3-12	111396	KAA V LNG EXT ASM 15W L	LEFT- 70D V hangers only 15 W- LONG
1H,2G,3-12	111397	KAA V LNG EXT ASM 16W L	LEFT- 70D V hangers only 16 W- LONG


NOTE: FOOTPLATE ASSEMBLIES ARE SET TO FOOTREST WIDTH. PLEASE SEE "FOOTPLATE FORMULA" TABLE FOR SEAT WIDTH CONVERSION.

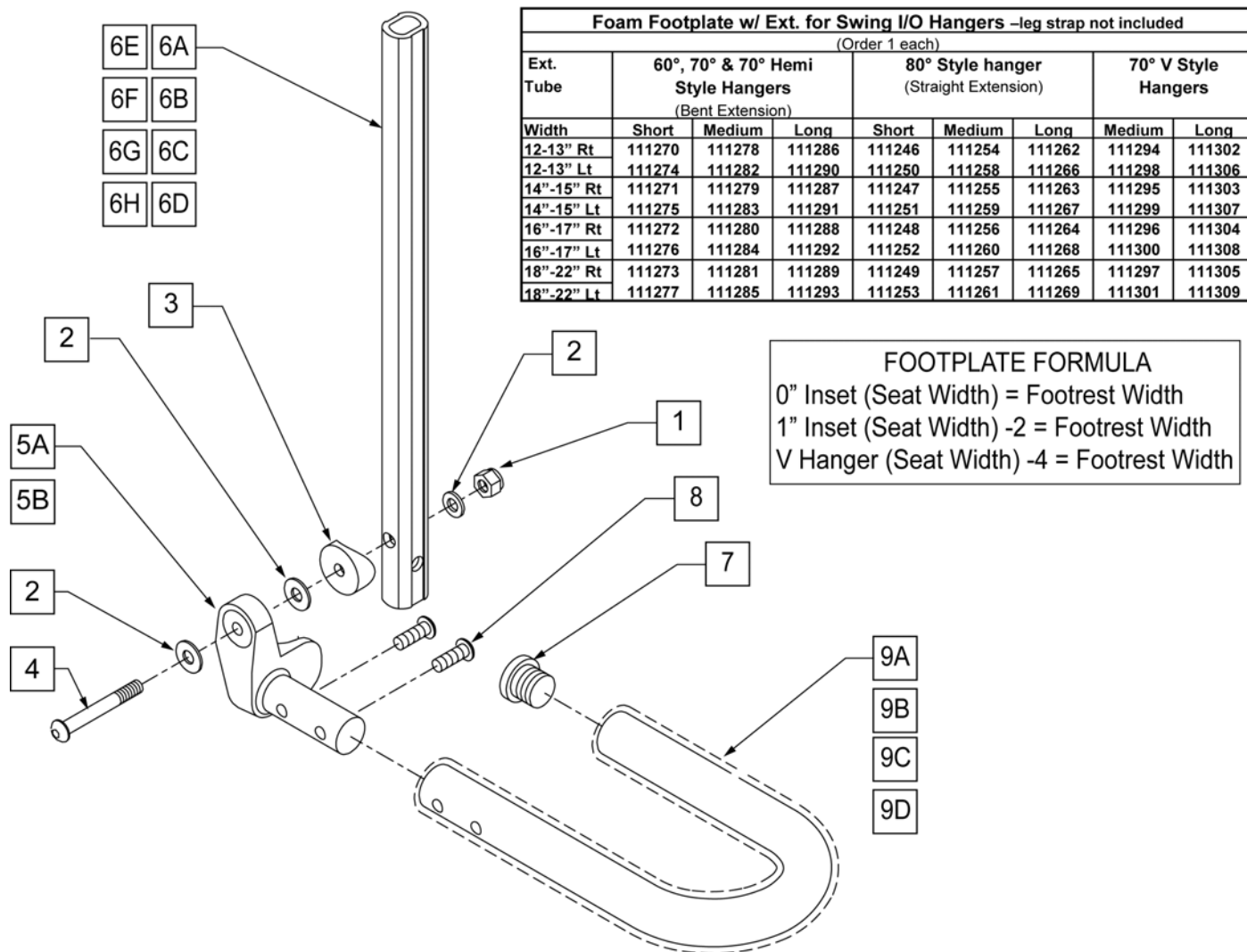

Pos.	Item Number	Description	Remarks
1	104381	NUT HEX NYLOCK M6X1.0	
2	107383	WASHER FLAT M6 ROHS	
3	561055	3/4" SADDLE 1" OD	
4	104399	SHCS M6X1X45 12 MM PART THD	
5	541002	FTRST MOUNTING BRKT-3/8 RAD R	RIGHT
5	541003	FTRST MOUNTING BRKT-3/8 RAD L	LEFT
6A	107119	SHORT STRAIGHT	80 deg hangers only
6B	107120	MED STRAIGHT	80 deg hangers only
6C	107121	LONG STRAIGHT	80 deg hangers only
6D	107126	SHORT BENT	60,70,70 Hemi Hangers
6E	107127	MED BENT	60,70,70 Hemi Hangers
6F	107128	LONG BENT	60,70,70 Hemi Hangers
6G	106907	EXT TUBE V ONLY D2 MED L	70 deg V hangers only-LEFT
6G	106908	EXT TUBE V ONLY D2 MED R	70 deg V hangers only-RIGHT


Pos.	Item Number	Description	Remarks
6H	106909	EXT TUBE V ONLY D2 X-LONG L	70 deg V hangers only-LEFT
6H	106910	EXT TUBE V ONLY D2 X-LONG R	70 deg V hangers only-RIGHT
7	560735	7/8" ROUND TUBE PLUG	
8	226168	1/4-28 X .875 BH C/S	
5,7-9A	820422	FT REST LOOP 12 RIGHT	RIGHT 12-13" Footrest Width
5,7-9A	820423	FT REST LOOP 12 LEFT	LEFT 12-13" Footrest Width
5,7-9B	820424	FT REST LOOP 14 RIGHT	RIGHT 14-15" Footrest Width
5,7-9B	820425	FT REST LOOP 14 LEFT	LEFT 14-15" Footrest Width
5,7-9C	820426	FT REST LOOP 16 RIGHT	RIGHT 16-17" Footrest Width
5,7-9C	820427	FT REST LOOP 16 LEFT	LEFT 16-17" Footrest Width
5,7-9D	820428	FT REST LOOP 18 RIGHT	RIGHT 18-19" Footrest Width
5,7-9D	820429	FT REST LOOP 18 LEFT	LEFT 16-19" Footrest Width
1-5,6A,7-8,9A	111246	FOAM STRT SHT EXT ASM 12W R	RIGHT- 80D- 12"-13" W- SHORT
1-5,6A,7-8,9B	111247	FOAM STRT SHT EXT ASM 14W R	RIGHT- 80D- 14"-15" W- SHORT
1-5,6A,7-8,9C	111248	FOAM STRT SHT EXT ASM 16W R	RIGHT- 80D- 16"-17" W- SHORT
1-5,6A,7-8,9D	111249	FOAM STRT SHT EXT ASM 18W R	RIGHT- 80D- 18"-22" W- SHORT
1-5,6A,7-8,9A	111250	FOAM STRT SHT EXT ASM 12W L	LEFT- 80D- 12"-13" W- SHORT
1-5,6A,7-8,9B	111251	FOAM STRT SHT EXT ASM 14W L	LEFT- 80D- 14"-15" W- SHORT
1-5,6A,7-8,9C	111252	FOAM STRT SHT EXT ASM 16W L	LEFT- 80D- 16"-17" W- SHORT
1-5,6A,7-8,9D	111253	FOAM STRT SHT EXT ASM 18W L	LEFT- 80D- 18"-22" W- SHORT
1-5,6B,7-8,9A	111254	FOAM STRT MED EXT ASM 12W R	RIGHT- 80D- 12"-13" W- MED
1-5,6B,7-8,9B	111255	FOAM STRT MED EXT ASM 14W R	RIGHT- 80D- 14"-15" W- MED
1-5,6B,7-8,9C	111256	FOAM STRT MED EXT ASM 16W R	RIGHT- 80D- 16"-17" W- MED
1-5,6B,7-8,9D	111257	FOAM STRT MED EXT ASM 18W R	RIGHT- 80D- 18"-22" W- MED
1-5,6B,7-8,9A	111258	FOAM STRT MED EXT ASM 12W L	LEFT- 80D- 12"-13" W- MED
1-5,6B,7-8,9B	111259	FOAM STRT MED EXT ASM 14W L	LEFT- 80D- 14"-15" W- MED
1-5,6B,7-8,9C	111260	FOAM STRT MED EXT ASM 16W L	LEFT- 80D- 16"-17" W- MED
1-5,6B,7-8,9D	111261	FOAM STRT MED EXT ASM 18W L	LEFT- 80D- 18"-22" W- MED
1-5,6C,7-8,9A	111262	FOAM STRT LNG EXT ASM 12W R	RIGHT- 80D- 12"-13" W- LONG
1-5,6C,7-8,9B	111263	FOAM STRT LNG EXT ASM 14W R	RIGHT- 80D- 14"-15" W- LONG
1-5,6C,7-8,9C	111264	FOAM STRT LNG EXT ASM 16W R	RIGHT- 80D- 16"-17" W- LONG
1-5,6C,7-8,9D	111265	FOAM STRT LNG EXT ASM 18W R	RIGHT- 80D- 18"-22" W- LONG
1-5,6C,7-8,9A	111266	FOAM STRT LNG EXT ASM 12W L	LEFT- 80D- 12"-13" W- LONG
1-5,6C,7-8,9B	111267	FOAM STRT LNG EXT ASM 14W L	LEFT- 80D- 14"-15" W- LONG
1-5,6C,7-8,9C	111268	FOAM STRT LNG EXT ASM 16W L	LEFT- 80D- 16"-17" W- LONG
1-5,6C,7-8,9D	111269	FOAM STRT LNG EXT ASM 18W L	LEFT- 80D- 18"-22" W- LONG
1-5,6D,7-8,9A	111270	FOAM BNT SHT EXT ASM 12W R	RIGHT- 70D- 12"-13" W- SHORT
1-5,6D,7-8,9B	111271	FOAM BNT SHT EXT ASM 14W R	RIGHT- 70D- 14"-15" W- SHORT
1-5,6D,7-8,9C	111272	FOAM BNT SHT EXT ASM 16W R	RIGHT- 70D- 16"-17" W- SHORT
1-5,6D,7-8,9D	111273	FOAM BNT SHT EXT ASM 18W R	RIGHT- 70D- 18"-22" W- SHORT
1-5,6D,7-8,9A	111274	FOAM BNT SHT EXT ASM 12W L	LEFT- 70D- 12"-13" W- SHORT
1-5,6D,7-8,9B	111275	FOAM BNT SHT EXT ASM 14W L	LEFT- 70D- 14"-15" W- SHORT
1-5,6D,7-8,9C	111276	FOAM BNT SHT EXT ASM 16W L	LEFT- 70D- 16"-17" W- SHORT
1-5,6D,7-8,9D	111277	FOAM BNT SHT EXT ASM 18W L	LEFT- 70D- 18"-22" W- SHORT
1-5,6E,7-8,9A	111278	FOAM BNT MED EXT ASM 12W R	RIGHT- 70D- 12"-13" W- MED
1-5,6E,7-8,9B	111279	FOAM BNT MED EXT ASM 14W R	RIGHT- 70D- 14"-15" W- MED
1-5,6E,7-8,9C	111280	FOAM BNT MED EXT ASM 16W R	RIGHT- 70D- 16"-17" W- MED
1-5,6E,7-8,9D	111281	FOAM BNT MED EXT ASM 18W R	RIGHT- 70D- 18"-22" W- MED
1-5,6E,7-8,9A	111282	FOAM BNT MED EXT ASM 12W L	LEFT- 70D- 12"-13" W- MED
1-5,6E,7-8,9B	111283	FOAM BNT MED EXT ASM 14W L	LEFT- 70D- 14"-15" W- MED
1-5,6E,7-8,9C	111284	FOAM BNT MED EXT ASM 16W L	LEFT- 70D- 16"-17" W- MED
1-5,6E,7-8,9D	111285	FOAM BNT MED EXT ASM 18W L	LEFT- 70D- 18"-22" W- MED
1-5,6F,7-8,9A	111286	FOAM BNT LNG EXT ASM 12W R	RIGHT- 70D- 12"-13" W- LONG
1-5,6F,7-8,9B	111287	FOAM BNT LNG EXT ASM 14W R	RIGHT- 70D- 14"-15" W- LONG
1-5,6F,7-8,9C	111288	FOAM BNT LNG EXT ASM 16W R	RIGHT- 70D- 16"-17" W- LONG
1-5,6F,7-8,9D	111289	FOAM BNT LNG EXT ASM 18W R	RIGHT- 70D- 18"-22" W- LONG

Pos.	Item Number	Description	Remarks
1-5,6F,7-8,9A	111290	FOAM BNT LNG EXT ASM 12W L	LEFT- 70D- 12"-13" W- LONG
1-5,6F,7-8,9B	111291	FOAM BNT LNG EXT ASM 14W L	LEFT- 70D- 14"-15" W- LONG
1-5,6F,7-8,9C	111292	FOAM BNT LNG EXT ASM 16W L	LEFT- 70D- 16"-17" W- LONG
1-5,6F,7-8,9D	111293	FOAM BNT LNG EXT ASM 18W L	LEFT- 70D- 18"-22" W- LONG
1-5,6G,7-8,9A	111294	FOAM V MED EXT ASM 12W R	RIGHT- 70D V hangers only 12"-13" W- MED
1-5,6G,7-8,9B	111295	FOAM V MED EXT ASM 14W R	RIGHT- 70D V hangers only 14"-15" W- MED
1-5,6G,7-8,9C	111296	FOAM V MED EXT ASM 16W R	RIGHT- 70D V hangers only 16"-17" W- MED
1-5,6G,7-8,9D	111297	FOAM V MED EXT ASM 18W R	RIGHT- 70D V hangers only 18"-22" W- MED
1-5,6G,7-8,9A	111298	FOAM V MED EXT ASM 12W L	LEFT- 70D V hangers only 12"-13" W- MED
1-5,6G,7-8,9B	111299	FOAM V MED EXT ASM 14W L	LEFT- 70D V hangers only 14"-15" W- MED
1-5,6G,7-8,9C	111300	FOAM V MED EXT ASM 16W L	LEFT- 70D V hangers only 16"-17" W- MED
1-5,6G,7-8,9D	111301	FOAM V MED EXT ASM 18W L	LEFT- 70D V hangers only 18"-22" W- MED
1-5,6H,7-8,9A	111302	FOAM V LNG EXT ASM 12W R	RIGHT- 70D V hangers only 12"-13" W- LONG
1-5,6H,7-8,9B	111303	FOAM V LNG EXT ASM 14W R	RIGHT- 70D V hangers only 14"-15" W- LONG
1-5,6H,7-8,9C	111304	FOAM V LNG EXT ASM 16W R	RIGHT- 70D V hangers only 16"-17" W- LONG
1-5,6H,7-8,9D	111305	FOAM V LNG EXT ASM 18W R	RIGHT- 70D V hangers only 18"-22" W- LONG
1-5,6H,7-8,9A	111306	FOAM V LNG EXT ASM 12W L	LEFT- 70D V hangers only 12"-13" W- LONG
1-5,6H,7-8,9B	111307	FOAM V LNG EXT ASM 14W L	LEFT- 70D V hangers only 14"-15" W- LONG
1-5,6H,7-8,9C	111308	FOAM V LNG EXT ASM 16W L	LEFT- 70D V hangers only 16"-17" W- LONG
1-5,6H,7-8,9D	111309	FOAM V LNG EXT ASM 18W L	LEFT- 70D V hangers only 18"-22" W- LONG

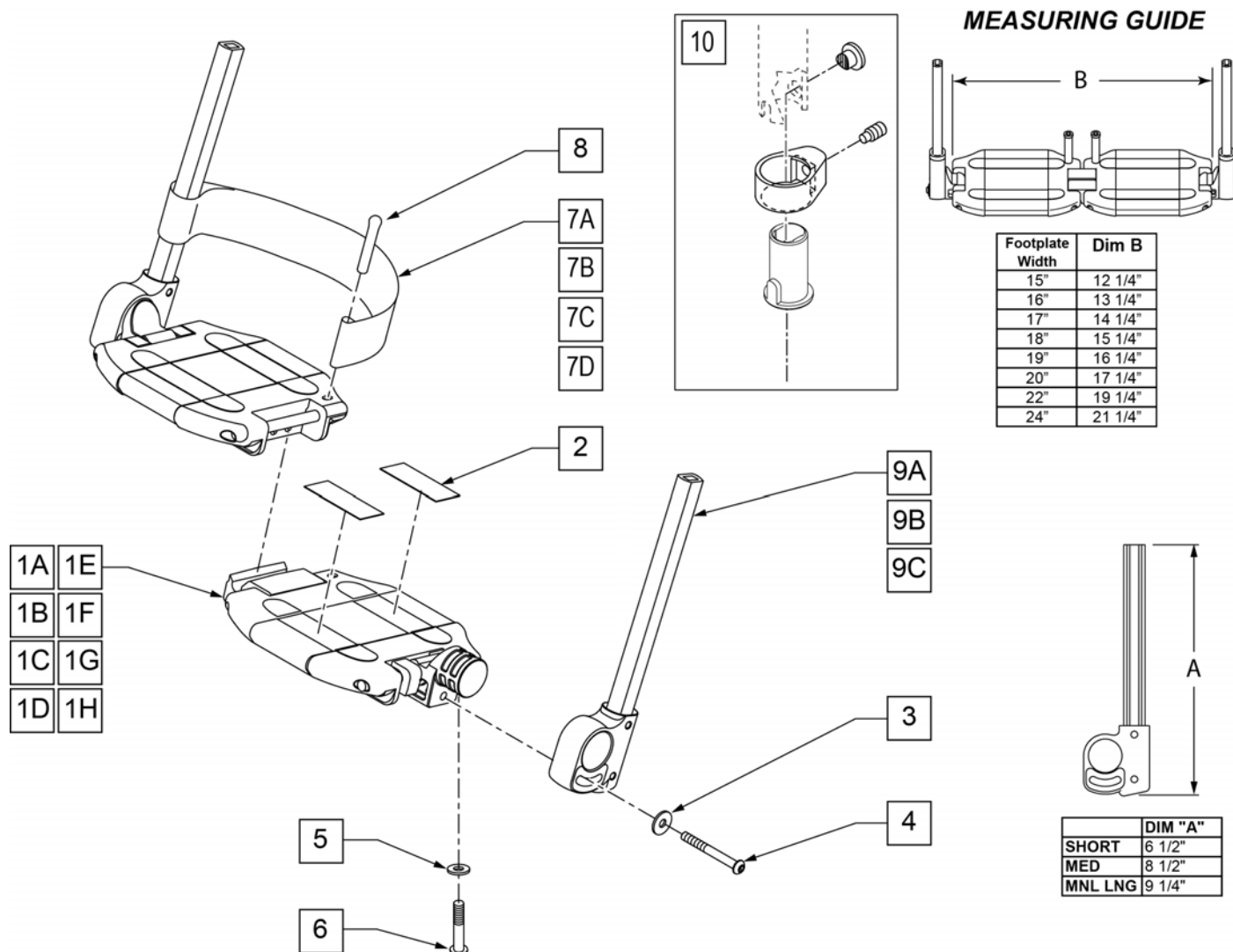


Pos.	Item Number	Description	Remarks
2	360010	FOOT BOARD SKID TAPE	
3	250070	M6 WASHER	
4	210000	M6 x 55 8.8 BUTTON HD CAP SCRW	
5	106944	1/4 AN WASHER FLAT THIN	
6	106958	1/4-20 x 1 HEX C/S	
7A	741151	HEEL LOOP 14-15 CMPST EA 13.24 44	15" Footrest Width
7B	741156	HEEL LOOP 18-20 CMPST HEMI	16 -17" Footrest Width
7C	741153	HEEL LOOP 18-20 CMPST/14-15 ANG ADJ	18 -20" Footrest Width
7D	741253	HEEL LOOP ADLT ADJ F/PLT 18-20	21 & 22" Footrest Width
8	561040	HEEL LOOP PEG	
9A	109780	EXT TUBE PLTFRM SMALL ASSM	SMALL EXT
9B	109781	EXT TUBE PLTFRM MEDIUM ASSM	MEDIUM EXT
9C	109782	EXT TUBE PLTFRM LONG ASSM	LONG EXT
10	107485	FOOTREST CLAMP ASSM	
1A,2-7A,8	820620	LOCKING FTPLT 15" W LEFT	Left -15" Footrest Width
1A,2-7A,8	820621	LOCKING FTPLT 15" W RIGHT	Right -15" Footrest Width
1B,2-7B,8	820622	LOCKING FTPLT 16" W LEFT	Left -16" Footrest Width
1B,2-7B,8	820623	LOCKING FTPLT 16" W RIGHT	Right -16" Footrest Width

Pos.	Item Number	Description	Remarks
1C,2-7B,8	820624	LOCKING FTPLT 17" W LEFT	Left -17" Footrest Width
1C,2-7B,8	820625	LOCKING FTPLT 17" W RIGHT	Right -17" Footrest Width
1D,2-7C,8	820626	LOCKING FTPLT 18" W LEFT	Left -18" Footrest Width
1D,2-7C,8	820627	LOCKING FTPLT 18" W RIGHT	Right -18" Footrest Width
1E,2-7C,8	820628	LOCKING FTPLT 19" W LEFT	Left -19" Footrest Width
1E,2-7C,8	820629	LOCKING FTPLT 19" W RIGHT	Right -19" Footrest Width
1F,2-7C,8	820630	LOCKING FTPLT 20" W LEFT	Left -20" Footrest Width
1F,2-7C,8	820631	LOCKING FTPLT 20" W RIGHT	Right -20" Footrest Width
1H,2-7D,8	820632	LOCKING FTPLT 22" W LEFT	Left -22" Footrest Width
1H,2-7D,8	820633	LOCKING FTPLT 22" W RIGHT	Right -22" Footrest Width
		FOOTPLATE W/ SMALL EXT	
1A,2-7A,8,9A	109783	15" W LOCKING F/PLT SMALL EXT HL	Small Ext 15" Footrest Width
1B,2-7B,8,9A	109787	16" W LOCKING F/PLT SMALL EXT HL	Small Ext 16" Footrest Width
1C,2-7B,8,9A	109790	17" W LOCKING F/PLT SMALL EXT HL	Small Ext 17" Footrest Width
1D,2-7C,8,9A	109793	18" W LOCKING F/PLT SMALL EXT HL	Small Ext 18" Footrest Width
1E,2-7C,8,9A	109796	19" W LOCKING F/PLT SMALL EXT HL	Small Ext 19" Footrest Width
1F,2-7C,8,9A	109799	20" W LOCKING F/PLT SMALL EXT HL	Small Ext 20" Footrest Width
1G,2-7D,8,9A	109802	21" W LOCKING F/PLT SMALL EXT HL	Small Ext 21" Footrest Width
1H,2-7D,8,9A	109805	22" W LOCKING F/PLT SMALL EXT HL	Small Ext 22" Footrest Width
		FOOTPLATE W/ MED EXT	
1A,2-7A,8,9B	109784	15" W LOCKING F/PLT MED EXT HL	Med Ext 15" Footrest Width
1B,2-7B,8,9B	109788	16" W LOCKING F/PLT MED EXT HL	Med Ext 16" Footrest Width
1C,2-7B,8,9B	109791	17" W LOCKING F/PLT MED EXT HL	Med Ext 17" Footrest Width
1D,2-7C,8,9B	109794	18" W LOCKING F/PLT MED EXT HL	Med Ext 18" Footrest Width
1E,2-7C,8,9B	109797	19" W LOCKING F/PLT MED EXT HL	Med Ext 19" Footrest Width
1F,2-7C,8,9B	109800	20" W LOCKING F/PLT MED EXT HL	Med Ext 20" Footrest Width
1G,2-7D,8,9B	109803	21" W LOCKING F/PLT MED EXT HL	Med Ext 21" Footrest Width
1H,2-7D,8,9B	109806	22" W LOCKING F/PLT MED EXT HL	Med Ext 22" Footrest Width
		FOOTPLATE W/ LONG EXT	
1A,2-7A,8,9C	109786	15" W LOCKING F/PLT LONG EXT HL	Long Ext 15" Footrest Width
1B,2-7B,8,9C	109789	16" W LOCKING F/PLT LONG EXT HL	Long Ext 16" Footrest Width
1C,2-7B,8,9C	109792	17" W LOCKING F/PLT LONG EXT HL	Long Ext 17" Footrest Width
1D,2-7C,8,9C	109795	18" W LOCKING F/PLT LONG EXT HL	Long Ext 18" Footrest Width
1E,2-7C,8,9C	109798	19" W LOCKING F/PLT LONG EXT HL	Long Ext 19" Footrest Width
1F,2-7C,8,9C	109801	20" W LOCKING F/PLT LONG EXT HL	Long Ext 20" Footrest Width
1G,2-7D,8,9C	109804	21" W LOCKING F/PLT LONG EXT HL	Long Ext 21" Footrest Width
1H,2-7D,8,9C	109807	22" W LOCKING F/PLT LONG EXT HL	Long Ext 22" Footrest Width

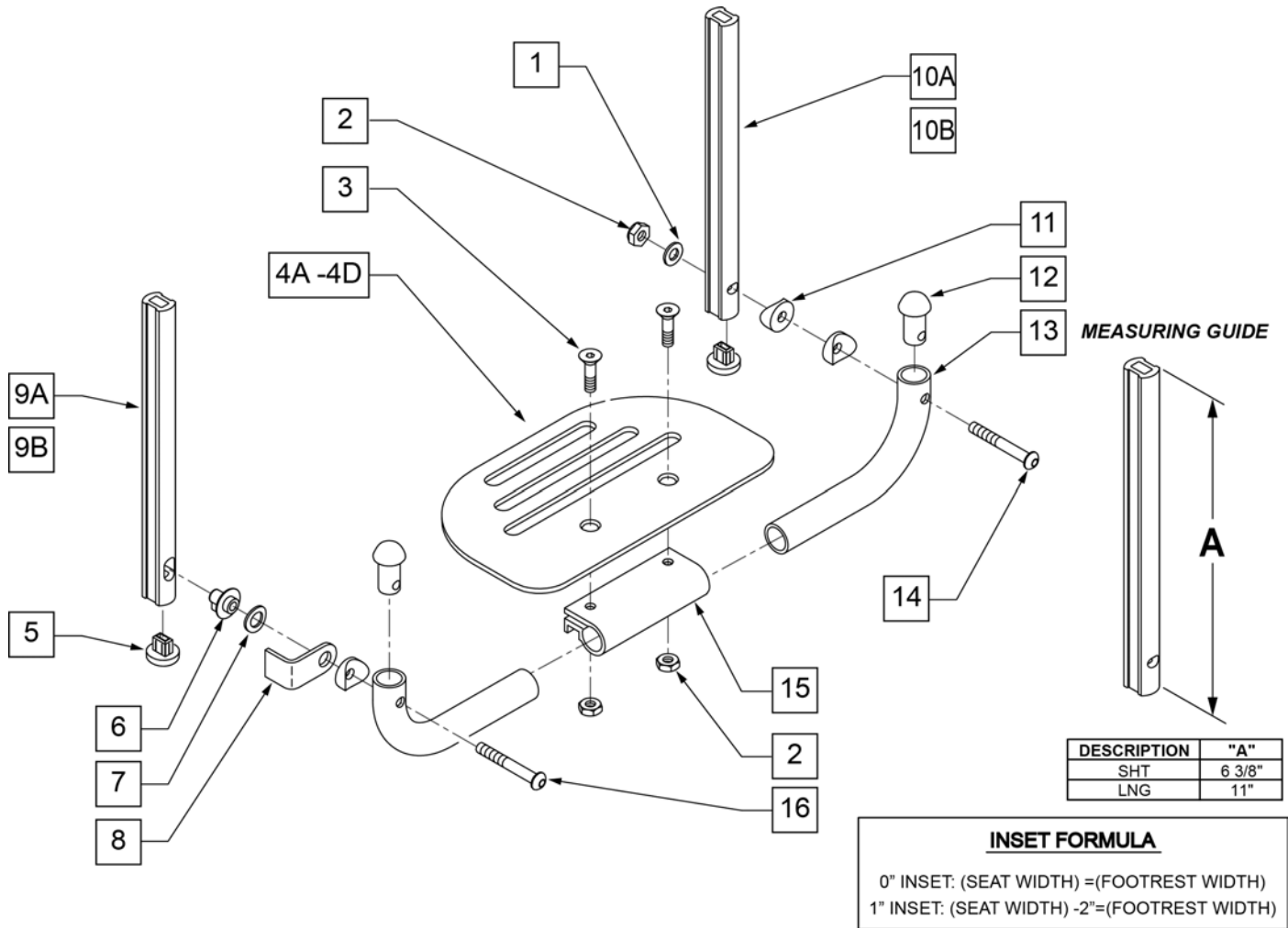


Pos.	Item Number	Description	Remarks
1	106944	WASHER FLAT THIN AN 1/4 ROHS	
2	106941	NUT AUTOMATION 1/4-28 ROHS	
3	106978	FHCS 1/4-28 X 7/8 ROHS	
4A	500651	PLATFORM F/PLT 8 x 5 1/2"	12 -14" Footrest Width
4B	500652	PLATFORM F/PLT 10" x 5 1/2"	15 -17" Footrest Width
4C	500653	PLATFORM F/PLT 12" x 5 1/2"	18 -20" Footrest Width
4D	500654	PLATFORM F/PLT 14" x 5 1/2"	21 -22" Footrest Width
5	113221	PLUG DOUBLE D EXT TUBE	
6	540410	F/UP FOOTREST KEY STUD D2	
7	107385	WASHER CURVED 3/8 ROHS	
8	505078	FOOTREST RETAINING BRACKET	
9A	106875	EXT TBE F/UP KEY SHT	Short -Latch Side
9B	106876	EXT TBE F/UP KEY LNG	Long -Latch Side
10A	106873	EXT TBE F/UP SLT SHT	Short -Pivot Side
10B	106874	EXT TBE F/UP SLT LNG	Long -Pivot Side
11	561003	3/4" SADDLE THIN	
12	561034	SHORT DOME PLUG	
14	106964	BHCS 1/4-28 X 2 ROHS	
15	550351	PLATFORM FOOTPLATE CLAMP	


Pos.	Item Number	Description	Remarks
16	107027	BHCS 1/4-28 X 1-1/4 SHLDR ROHS	
17A	740130	LEG STRAP 10"-14" BLK	12-14" Footrest Width
17B	740131	LEG STRAP 15"-17" BLK	15-17" Footrest Width
17C	740132	LEG STRAP 18"-20" BLK	18-20" Footrest Width
1-4A,6-8,11-16	900306	PLTFM F/P PLT BLK 12" D2 W/ INSTR	12" Footrest Width
1-4A,6-8,11-16	900307	PLTFM F/P PLT BLK 13" D2 W/ INSTR	13" Footrest Width
1-4A,6-8,11-16	900308	PLTFM F/P PLT BLK 14" D2 W /INSTR	14" Footrest Width
1-4B,6-8,11-16	900309	PLTFM F/P PLT BLK 15" D2 W /INSTR	15" Footrest Width
1-4B,6-8,11-16	900310	PLTFM F/P PLT BLK 16" D2 W /INSTR	16" Footrest Width
1-4B,6-8,11-16	900311	PLTFM F/P PLT BLK 17" D2 W /INSTR	17" Footrest Width
1-4C,6-8,11-16	900312	PLTFM F/P PLT BLK 18" D2 W /INSTR	18" Footrest Width
1-4C,6-8,11-16	900313	PLTFM F/P PLT BLK 19" D2 W /INSTR	19" Footrest Width
1-4C,6-8,11-16	900314	PLTFM F/P PLT BLK 20" D2 W/ INSTR	20" Footrest Width
1-4D,6-8,11-16	900315	PLTFM F/P PLT BLK 21" D2 W/ INSTR	21" Footrest Width
1-4D,6-8,11-16	900316	PLTFM F/P PLT BLK 22" D2 W/ INSTR	22" Footrest Width
		SHORT EXT ASSEMBLIES	
1-3,4A,5-8,9A,10B,11-16	108043	PLTFRM FLIP FTRST SHT 12W	12" wide (Short)
1-3,4A,5-8,9A,10B,11-16	108044	PLTFRM FLIP FTRST SHT 13W	13" wide (Short)
1-3,4A,5-8,9A,10B,11-16	108045	PLTFRM FLIP FTRST SHT 14W	14" wide (Short)
1-3,4B,5-8,9A,10B,11-16	108046	PLTFRM FLIP FTRST SHT 15W	15" wide (Short)
1-3,4B,5-8,9A,10B,11-16	108047	PLTFRM FLIP FTRST SHT 16W	16" wide (Short)
1-3,4B,5-8,9A,10B,11-16	108048	PLTFRM FLIP FTRST SHT 17W	17" wide (Short)
1-3,4C,5-8,9A,10B,11-16	108049	PLTFRM FLIP FTRST SHT 18W	18" wide (Short)
1-3,4C,5-8,9A,10B,11-16	108051	PLTFRM FLIP FTRST SHT 19W	19" wide (Short)
1-3,4C,5-8,9A,10B,11-16	108052	PLTFRM FLIP FTRST SHT 20W	20" wide (Short)
1-3,4D,5-8,9A,10B,11-16	111554	PLTFRM FLIP FTRST SHT 21W	21" wide (Short)
1-3,4D,5-8,9A,10B,11-16	111555	PLTFRM FLIP FTRST SHT 22W	22" wide (Short)
		LONG EXT ASSEMBLIES	
1-3,4A,5-8,9B,10A,11-16	108053	PLTFRM FLIP FTRST LONG 12W	12" wide (Long)
1-3,4A,5-8,9B,10A,11-16	108054	PLTFRM FLIP FTRST LONG 13W	13" wide (Long)
1-3,4A,5-8,9B,10A,11-16	108055	PLTFRM FLIP FTRST LONG 14W	14" wide (Long)
1-3,4B,5-8,9B,10A,11-16	108056	PLTFRM FLIP FTRST LONG 15W	15" wide (Long)
1-3,4B,5-8,9B,10A,11-16	108057	PLTFRM FLIP FTRST LONG 16W	16" wide (Long)
1-3,4B,5-8,9B,10A,11-16	108058	PLTFRM FLIP FTRST LONG 17W	17" wide (Long)
1-3,4C,5-8,9B,10A,11-16	108059	PLTFRM FLIP FTRST LONG 18W	18" wide (Long)
1-3,4C,5-8,9B,10A,11-16	108060	PLTFRM FLIP FTRST LONG 19W	19" wide (Long)
1-3,4C,5-8,9B,10A,11-16	108061	PLTFRM FLIP FTRST LONG 20W	20" wide (Long)
1-3,4D,5-8,9B,10A,11-16	111556	PLTFRM FLIP FTRST LONG 21W	21" wide (Long)
1-3,4D,5-8,9B,10A,11-16	111557	PLTFRM FLIP FTRST LONG 22W	22" wide (Long)

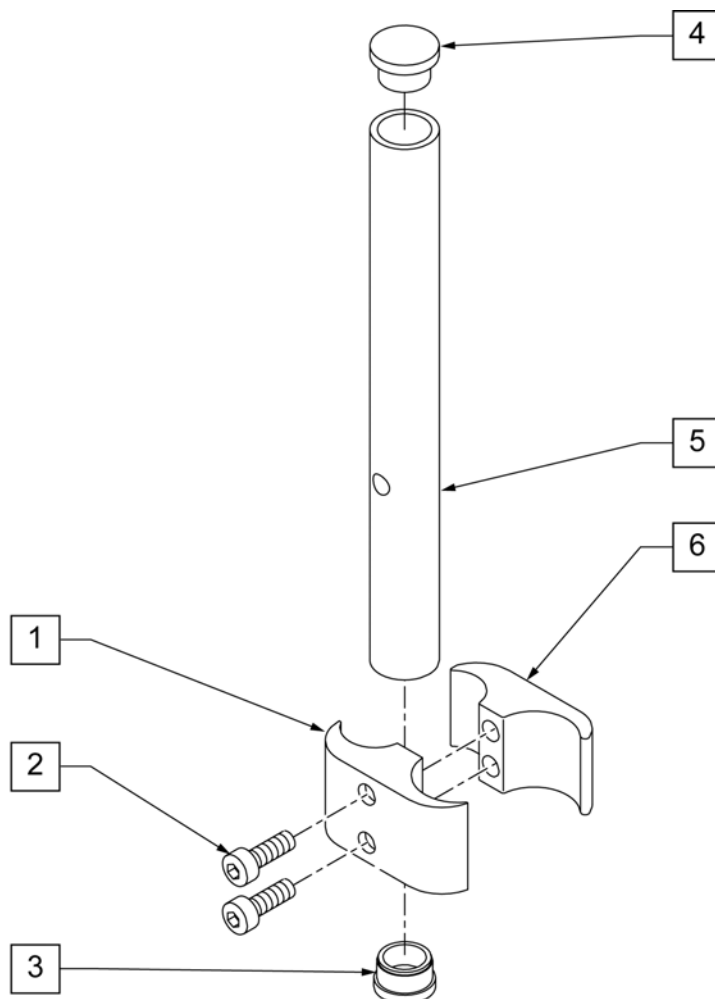

Pos.	Item Number	Description	Remarks
1	550657	CLAMP 1 1/4" RECESS FULL BLK	
2	107032	SHCS 1/4-28 X 1 ROHS	
3	561000-QU	3/4" INSERT PLUG .083	
4	482016	HIGH-MOUNT FL FT TUBE PLUG	
5	524100	H/M FL FT EXT TUBE BLK	
6	550658	CLAMP 1 1/4" THREADED BLK	
1,2,6	880538	HI-MOUNT FR CLAMP ASSEMBLY BLK	
1-6	921550S	H/M FL FT CLAMPS AND TUBES (PR)	Pair

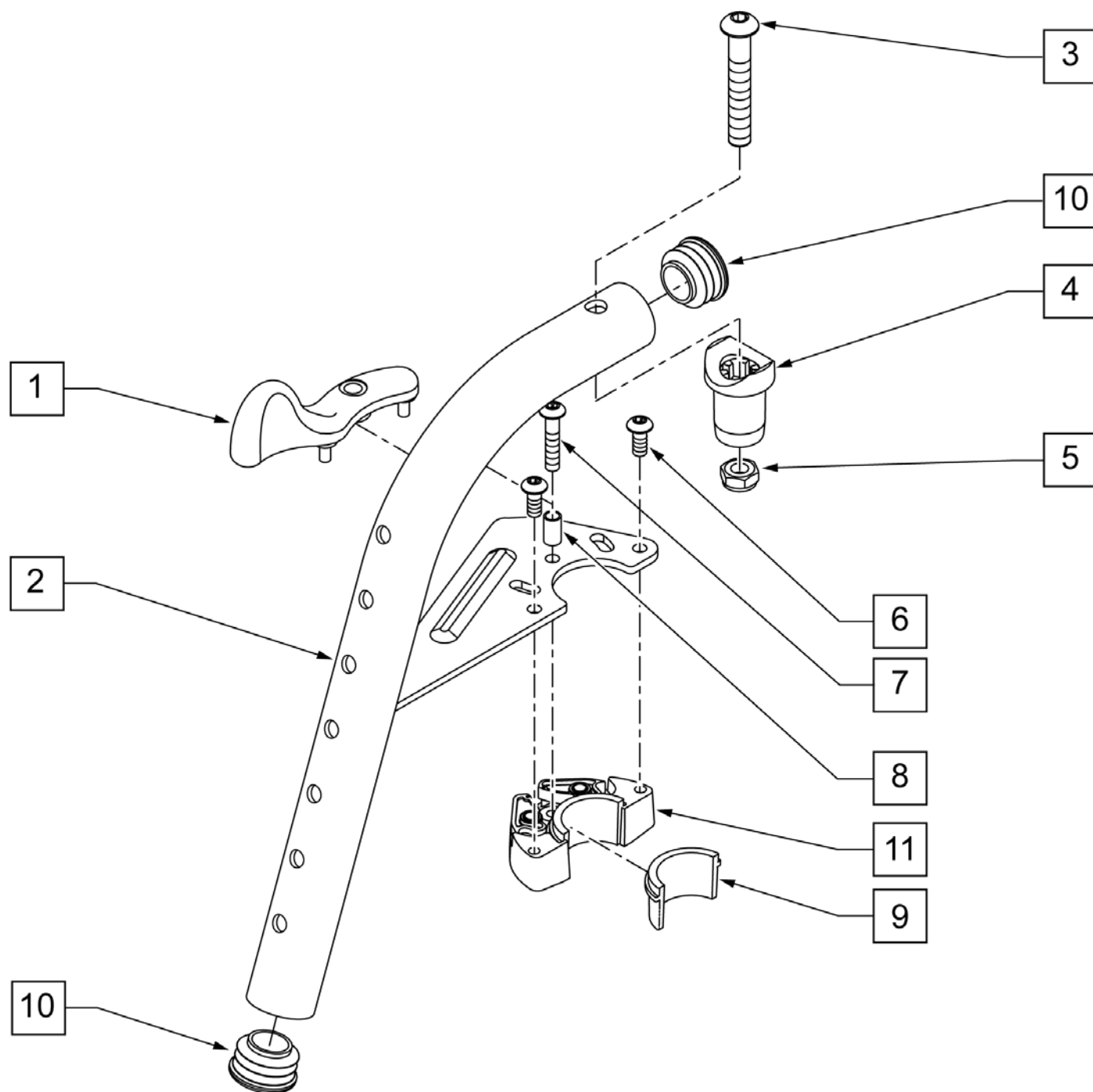


Pos.	Item Number	Description	Remarks
1	052296	SWING I/O LATCH LEVER R W/ PINS	RIGHT
1	052297	SWING I/O LATCH LEVER L W/ PINS	LEFT
3	106983	BHCS 5/16 -18 x 2 SHLDR ROHS	
4	561197	PLASTIC PIVOT SADDLE	Includes nylock nut
5	202117	5/16-18 NYLOCK NUT THICK BLK	
6	102259-001	BHCS M5 X .8 X 12 BLK W/ PL	
7	230494	M5-30 BTTN HD SOC C/S BLK	
8	052054	BUSHING PIVOT HANGER SWING I/O	
10	561062	1" INSERT PLUG .065 RIBS	
6,9,11	120976	ASSY LATCH BLOCK ANTI LIFT	
1-11	101153S	70D FM SWING I/O HNDR ASSY R	RIGHT
1-11	101154S	70D FM SWING I/O HNDR ASSY L	LEFT

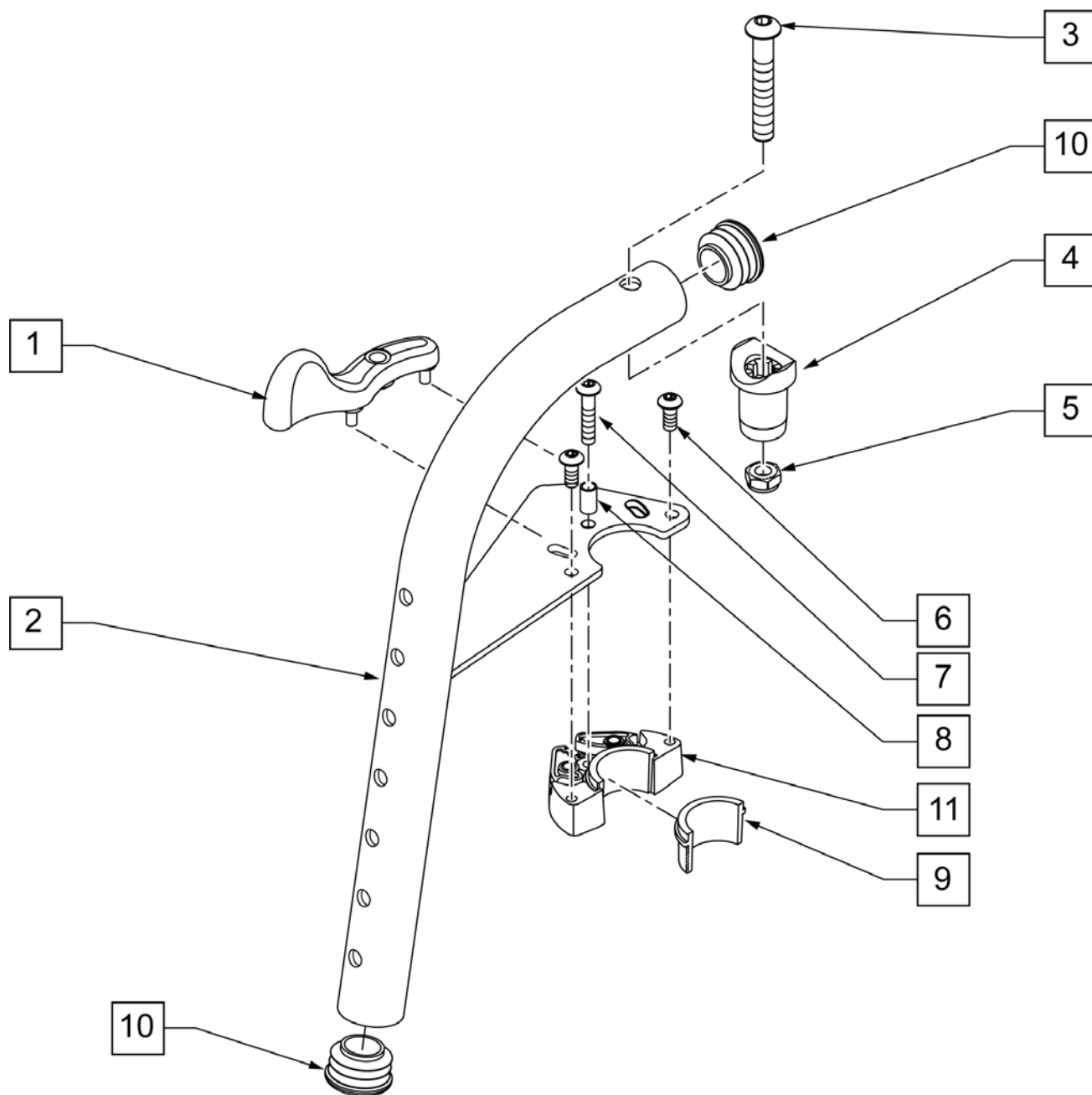


Pos.	Item Number	Description	Remarks
1	052296	SWING I/O LATCH LEVER R w/ PINS	RIGHT
1	052297	SWING I/O LATCH LEVER L w/ PINS	LEFT
3	106983	BHCS 5/16 -18 x 2 SHLDR ROHS	
4	561197	PLASTIC PIVOT SADDLE	Includes nylock nut
5	202117	5/16-18 NYLOCK NUT THICK BLK	
6	102259-001	BHCS M5 X .8 X 12 BLK W/PL	
7	230494	M5-30 BTTN HD SOC C/S BLK	
8	052054	BUSHING PIVOT HANGER SWING I/O	
10	561062	1" INSERT PLUG .065 RIBS	
6,9,11	120976	ASSY LATCH BLOCK ANTI LIFT	
1A,2A,3-11	101155S	80D FM SWNG I/O HNGR ASSY R	RIGHT
1B,2B,3-11	101156S	80D FM SWNG I/O HNGR ASSY L	LEFT


NOTE: Effective on LX/LXI as of 10/24/2008.

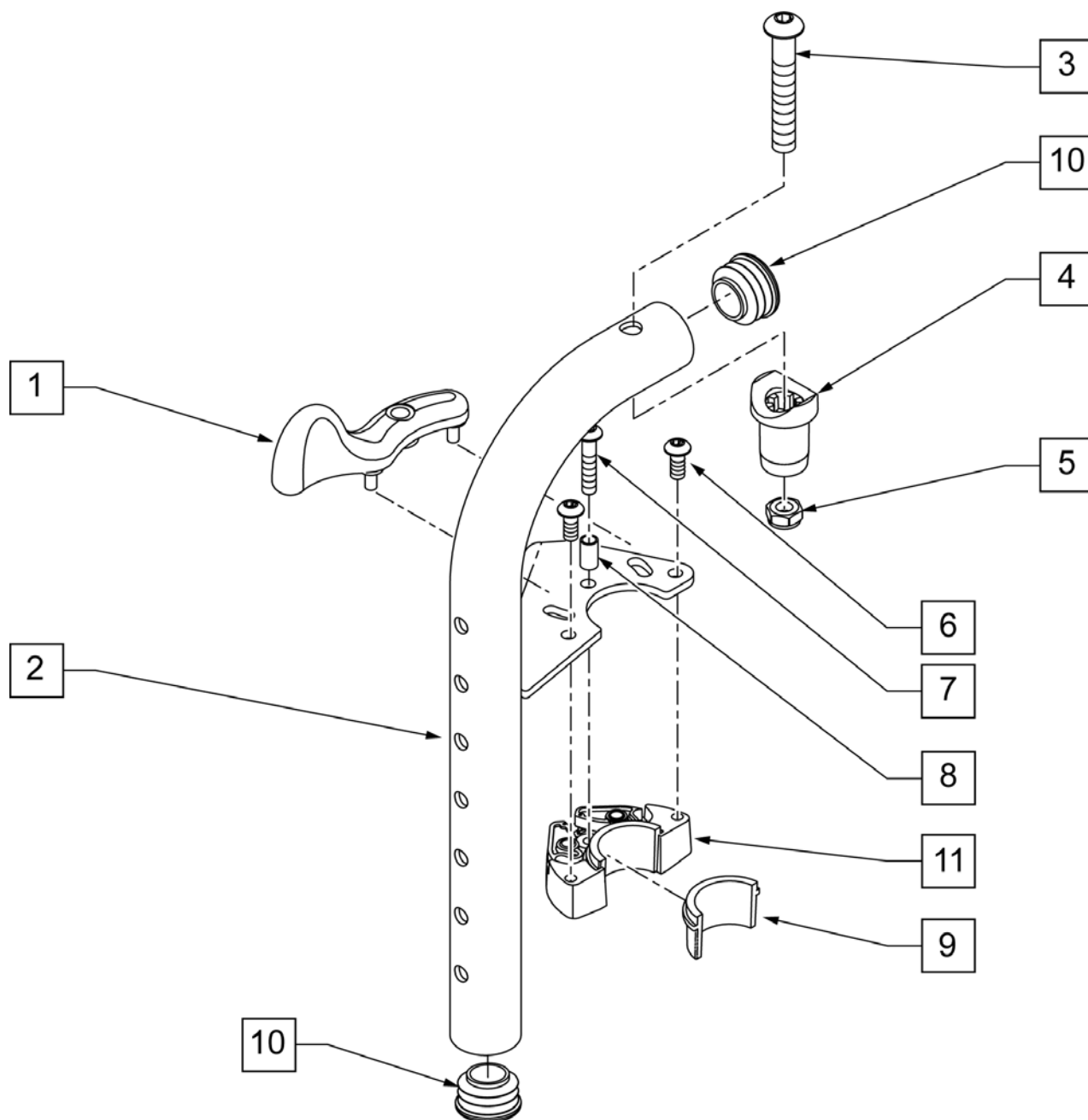

Pos.	Item Number	Description	Remarks
1	052296	SWING I/O LATCH LEVER R w/ PINS	RIGHT
1	052297	SWING I/O LATCH LEVER L w/ PINS	LEFT
3	106983	BHCS 5/16 -18 x 2 SHLDR ROHS	
4	561197	PLASTIC PIVOT SADDLE	
5	202117	5/16-18 NYLOCK NUT THICK BLK	
6	102259-001	BHCS M5 x .8 x 12 W /PL INORG	
7	230494	M5-30 BTTN HD SOC C/S BLK	
8	052054	BUSHING PIVOT HANGER SWING I/O	
10	561062	1" INSERT PLUG .065 RIBS	
6,9,11	120976	ASSY LATCH BLOCK ANTI LIFT	
1A,2A,3-11	101157S	90D FM SWNG I/O HNGR ASSY R	RIGHT
1B,2B,3-11	101158S	90D FM SWNG I/O HNGR ASSY L	LEFT

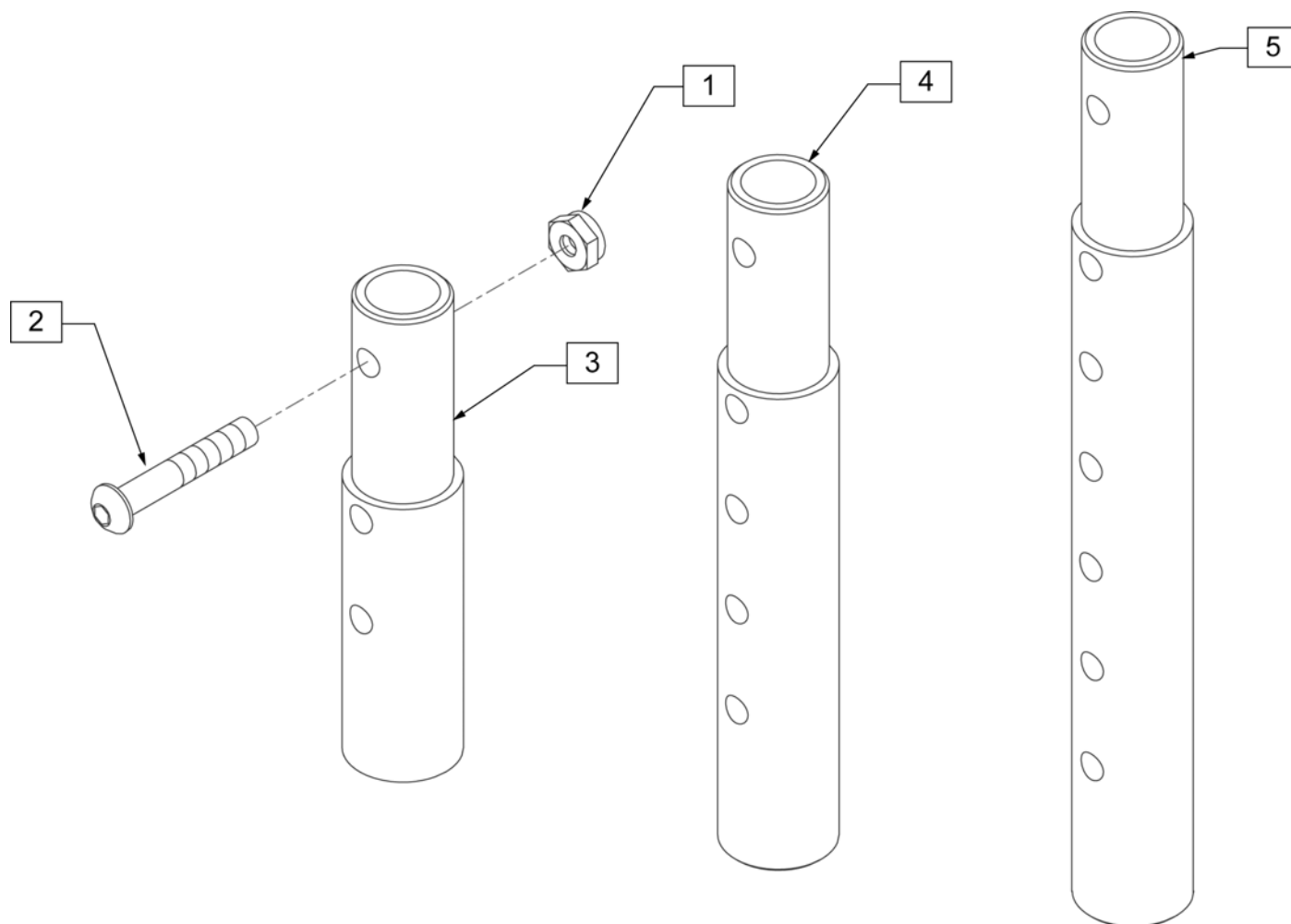


Pos.	Item Number	Description	Remarks
1	106935	NUT NYLOCK THIN 1/4 -28 ROHS	
2	106961	BHCS 1/4 -28 x 1 1/4 ROHS	
1-3	111505	2" FOOTREST EXTENSION TUBE W/ HDWR ASSM	2" EXT (PAIR
1, 2, 4	111506	4" FOOTREST EXTENSION TUBE W/ HDWR ASSM	4" EXT (PAIR)
1, 2, 5	111507	6" FOOTREST EXTENSION TUBE W/ HDWR ASSM	6" EXT (PAIR)

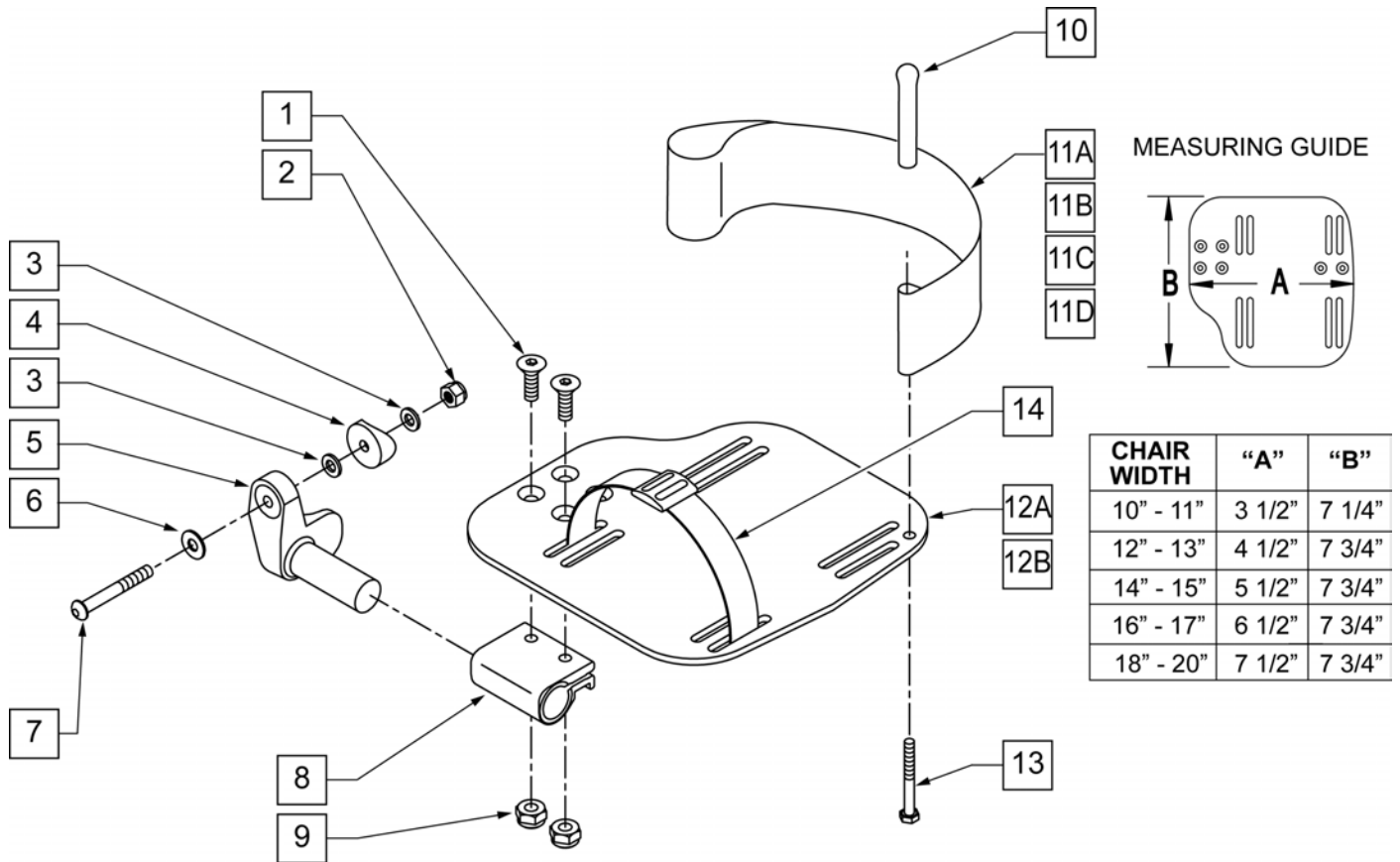


Pos.	Item Number	Description	Remarks
1	106978	FHCS 1/4 -28 x 7/8 ROHS	
2	106934	NUT NYLOCK 1/4 -28 ROHS	
3	106944	WASHER FLAT THIN AN 1/4 ROHS	
4	561004	1 SADDLE 1/4 HOLE	
5	541000	FTRST MOUNTING BRKT-1 RAD R	RIGHT
5	541001	FTRST MOUNTING BRKT-1 RAD L	LEFT
6	106945	WASHER FLAT SAE 1/4 ROHS	
7	106964	BHCS 1/4 -28 x 2 ROHS	
8	550350	3/4 CLAMP ADJ FOOTPLATE	
9	106935	NUT NYLOCK THIN 1/4 -28 ROHS	
10	104328	POST HEEL LOOP	
11A	104380-001	ADJ FOOTREST STRAP 14" WIDE	USED ON 12-14" WIDE
11B	104380-002	ADJ FOOTREST STRAP 15 -16" WIDE	USED ON 15 -16" WIDE
11C	104380-003	ADJ FOOTREST STRAP 17 -18" WIDE	USED ON 17 -18" WIDE
11D	104380-004	ADJ FOOTREST STRAP 19 -20" WIDE	USED ON 19 -22" WIDE
12A	500658	ADLT ANGL ADJ F/PLT 12/13" R	RIGHT -12-13" WIDE
12A	500660	ADLT ANGL ADJ F/PLT 14/15" R	RIGHT -14-15" WIDE
12A	500662	ADLT ANGL ADJ F/PLT 16/17" R	RIGHT -16-17" WIDE
12A	500664	ADLT ANGL ADJ F/PLT 18-20" R	RIGHT -18-22" WIDE
12B	500659	ADLT ANGL ADJ F/PLT 12/13" L	LEFT -12-13" WIDE
12B	500661	ADLT ANGL ADJ F/PLT 14/15" L	LEFT -14-15" WIDE
12B	500663	ADLT ANGL ADJ F/PLT 16/17" L	LEFT -16-17" WIDE
12B	500665	ADLT ANGL ADJ F/PLT 18-20" L	LEFT -18-22" WIDE
13	102268-001	HHCS M6 x 1 x 25	
14	740260	TOE-ANKLE STRAP ADJ ADULT FOOTPLATE	

Pos.	Item Number	Description	Remarks
1-10,11A,12A	111514	AAA FTPLT 90D 12/13" R	RIGHT 12-13" Footrest Width
1-10,11A,12B	111515	AAA FTPLT 90D 12/13" L	LEFT 12-13" Footrest Width
1-10,11B,12A	111516	AAA FTPLT 90D 14/15" R	RIGHT 14-15" Footrest Width
1-10,11B,12B	111517	AAA FTPLT 90D 14/15" L	LEFT 14-15" Footrest Width
1-10,11C,12A	111518	AAA FTPLT 90D 16/17" R	RIGHT 16-17" Footrest Width
1-10,11C,12B	111519	AAA FTPLT 90D 16/17" L	LEFT 16-17" Footrest Width
1-10,11D,12A	111520	AAA FTPLT 90D 18/20" R	RIGHT 18-20" Footrest Width
1-10,11D,12B	111521	AAA FTPLT 90D 18/20" L	LEFT 18-20" Footrest Width

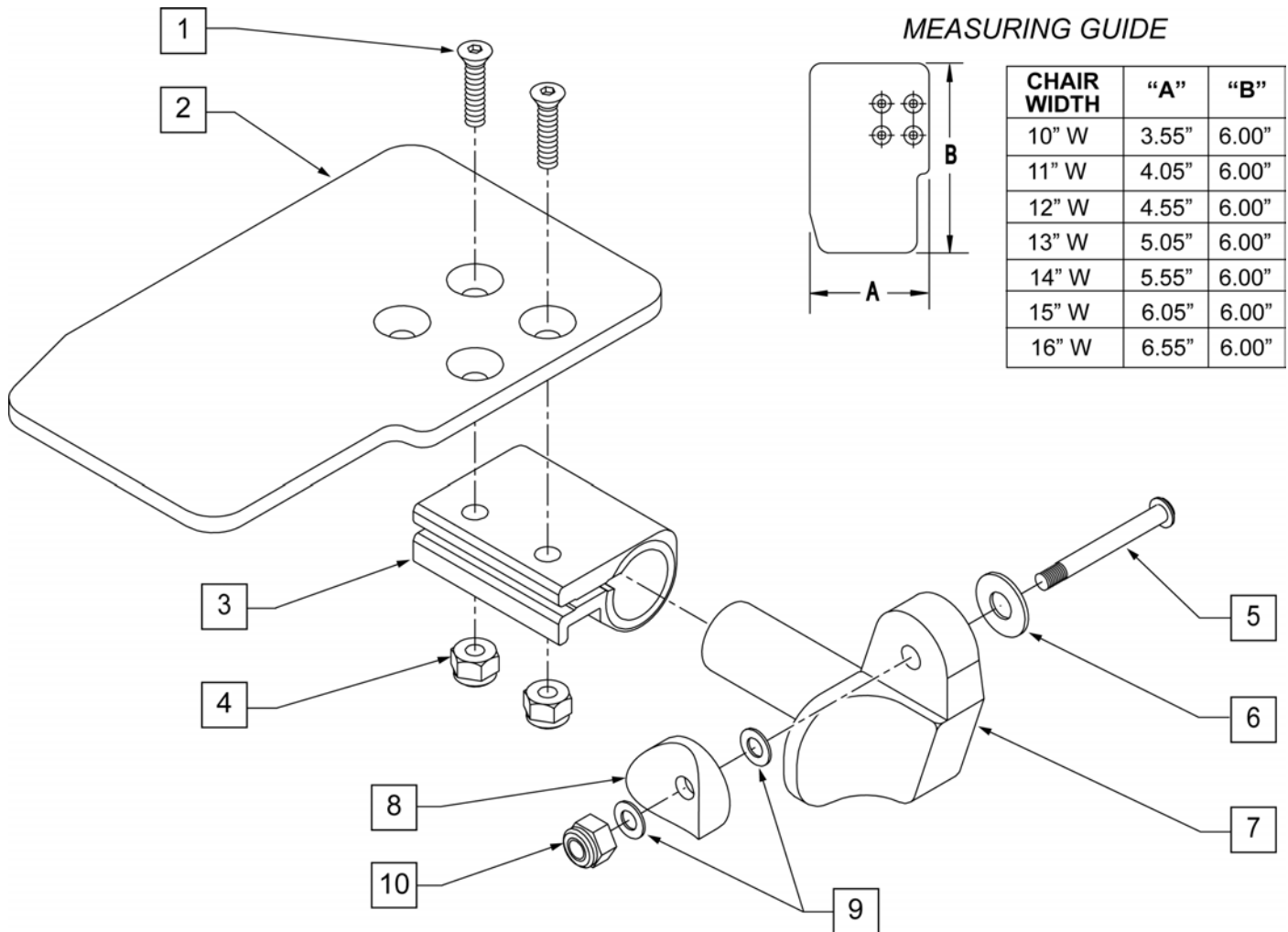

Pos.	Item Number	Description	Remarks
1	106978	FHCS 1/4 -28 x 7/8 ROHS	
2	500635	KIDS ANGLE ADJ FOOTPLATE 10" R	
2	500636	KIDS ANGLE ADJ FOOTPLATE 10" L	
2	500637	KIDS ANGLE ADJ FOOTPLATE 11" R	
2	500638	KIDS ANGLE ADJ FOOTPLATE 11" L	
2	500639	KIDS ANGLE ADJ FOOTPLATE 12" R	
2	500640	KIDS ANGLE ADJ FOOTPLATE 12" L	
2	500641	KIDS ANGLE ADJ FOOTPLATE 13" R	
2	500642	KIDS ANGLE ADJ FOOTPLATE 13" L	
2	500643	KIDS ANGLE ADJ FOOTPLATE 14" R	
2	500644	KIDS ANGLE ADJ FOOTPLATE 14" L	
2	500645	KIDS ANGLE ADJ FOOTPLATE 15" R	
2	500646	KIDS ANGLE ADJ FOOTPLATE 15" L	
2	500647	KIDS ANGLE ADJ FOOTPLATE 16" R	
2	500648	KIDS ANGLE ADJ FOOTPLATE 16" L	
3	550350	3/4" CLAMP ADJ FOOTPLATE	
4	106935	NUT NYLOCK THIN 1/4 -28 ROHS	
5	106964	BHCS 1/4 -28 x 2 ROHS	
6	106945	WASHER FLAT SAE 1/4 ROHS	
7	541000	FOOTREST MOUNTING BRKT 1" RAD R	RIGHT


Pos.	Item Number	Description	Remarks
7	541001	FOOTREST MOUNTING BRKT 1" RAD R	LEFT
8	561004	1" SADDLE 1/4" HOLE	
9	106944	WASHER FLAT THIN AN 1/4 ROHS	
10	106934	NUT NYLOCK 1/4 -28 ROHS	
1-10	111491	KIDS ANGLE ADJ FOOTPLATE 10" R	RIGHT -10" Footrest Width
1-10	111492	KIDS ANGLE ADJ FOOTPLATE 10" L	LEFT -10" Footrest Width
1-10	111493	KIDS ANGLE ADJ FOOTPLATE 11" R	RIGHT -11" Footrest Width
1-10	111494	KIDS ANGLE ADJ FOOTPLATE 11" L	LEFT -11" Footrest Width
1-10	111495	KIDS ANGLE ADJ FOOTPLATE 12" R	RIGHT -12" Footrest Width
1-10	111496	KIDS ANGLE ADJ FOOTPLATE 12" L	LEFT -12" Footrest Width
1-10	111497	KIDS ANGLE ADJ FOOTPLATE 13" R	RIGHT -13" Footrest Width
1-10	111498	KIDS ANGLE ADJ FOOTPLATE 13" L	LEFT -13" Footrest Width
1-10	111499	KIDS ANGLE ADJ FOOTPLATE 14" R	RIGHT -14" Footrest Width
1-10	111500	KIDS ANGLE ADJ FOOTPLATE 14" L	LEFT -14" Footrest Width
1-10	111501	KIDS ANGLE ADJ FOOTPLATE 15" R	RIGHT -15" Footrest Width
1-10	111502	KIDS ANGLE ADJ FOOTPLATE 15" L	LEFT -15" Footrest Width
1-10	111503	KIDS ANGLE ADJ FOOTPLATE 16" R	RIGHT -16" Footrest Width
1-10	111504	KIDS ANGLE ADJ FOOTPLATE 16" L	LEFT -16" Footrest Width

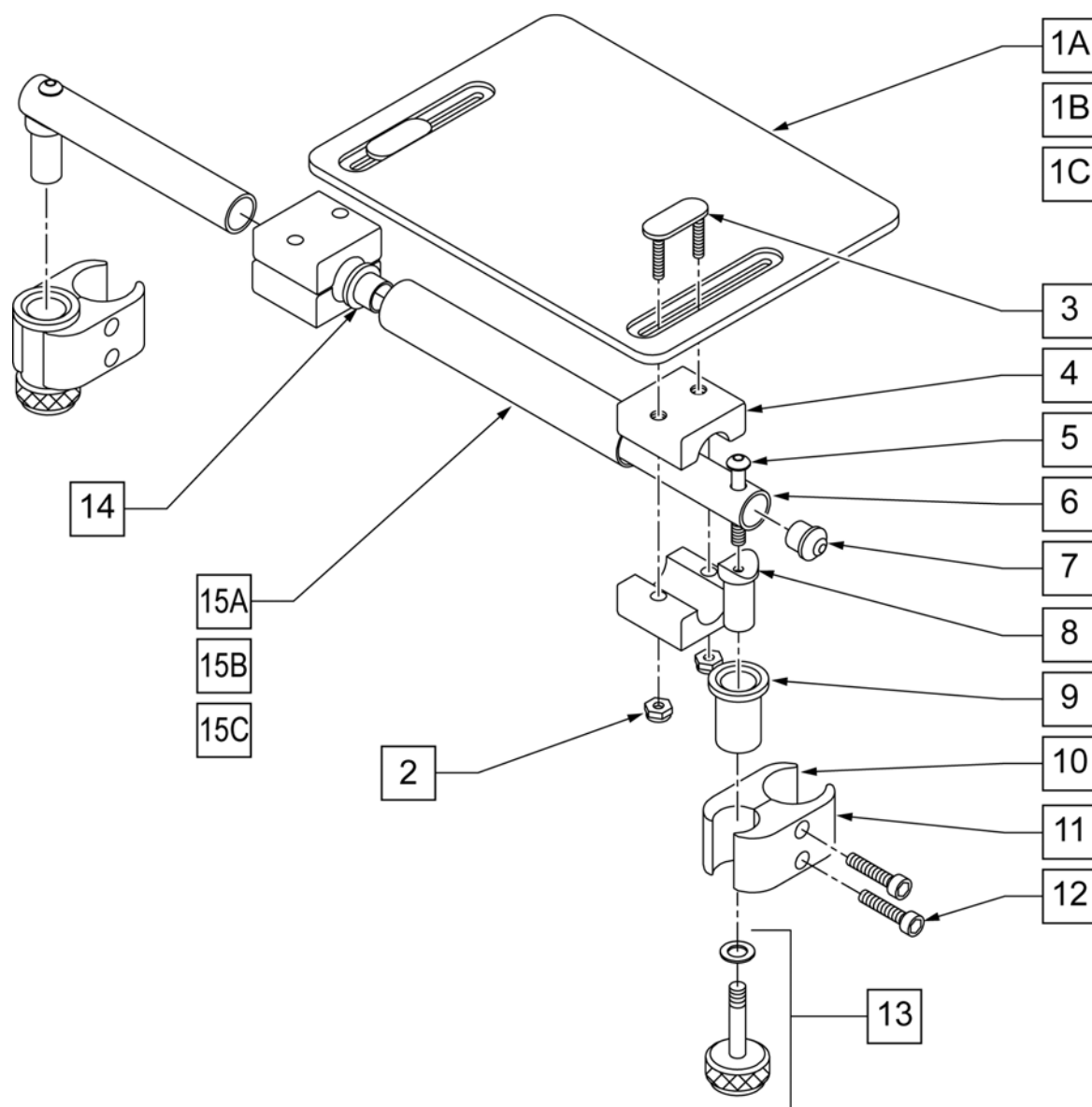

Pos.	Item Number	Description	Remarks
1A	562610	SLD FOOTPLATE ONLY 8" x 8" (10"-12" W)	10 -12" WIDE
1B	562612	SLD FOOTPLATE ONLY 10" x 8" (13"-14" W)	13 -14" WIDE
1C	500709	SLD FT PLT ALUM ONLY 12" x 8" (15"-16" W)	15 -16" WIDE
2	106933	NUT NYLOCK THIN 1/4-20 ROHS	
3	330120	SLD FT/ANGLE ADJ DOUBLE T BOLT	
4	550444	SOLID FOOT/ANG ADJ CLAMP BLK	
5	107028	BHCS 1/4-28 X 1-1/2 ROHS	
7	561034	SHORT DOME PLUG	
8	483236	SOLID FOOT INSERT SADDLE BLK	
10	550658	CLAMP 1 1/4" THREADED BLK	
11	550657	CLAMP 1 1/4" RECESS FULL BLK	
12	107032	SHCS 1/4-28 X 1 ROHS	
9,13	880559	SOLID FOOT LOCK KNOB ASSEMBLY	


Pos.	Item Number	Description	Remarks
14	560402	3/4" ID NY-LINER	
15A	521621	SLD FOOT SUPPORT TUBE BLK 10"-12"	10 -12" WIDE
15B	521622	SLD FOOT SUPPORT TUBE BLK 13"-14"	13 -14" WIDE
15C	521624	SLD FOOT SUPPORT TUBE BLK 15"-16"	15 -16" WIDE
5-8	880183	SLD FT PLT PIVOT TUBE SUB-ASSM	
1A,2-14, 15A	921801	SLD FT PLT ASSEMBLY 8" x 8" (10"-12" W)	10 -12" WIDE
1B,2-14, 15B	921803	SOLID FOOTPLATE ASSEMBLY 10" x 8" (13"-14" W)	13 -14" WIDE
1C,2-14, 15C	921805	SOLID FOOTPLATE ASSEMBLY 12" x 8" (15"-16" W)	15 -16" WIDE

NOTE: * ORDER ITEM 16 WHEN NOT USING FOOTREST EXTENSION.

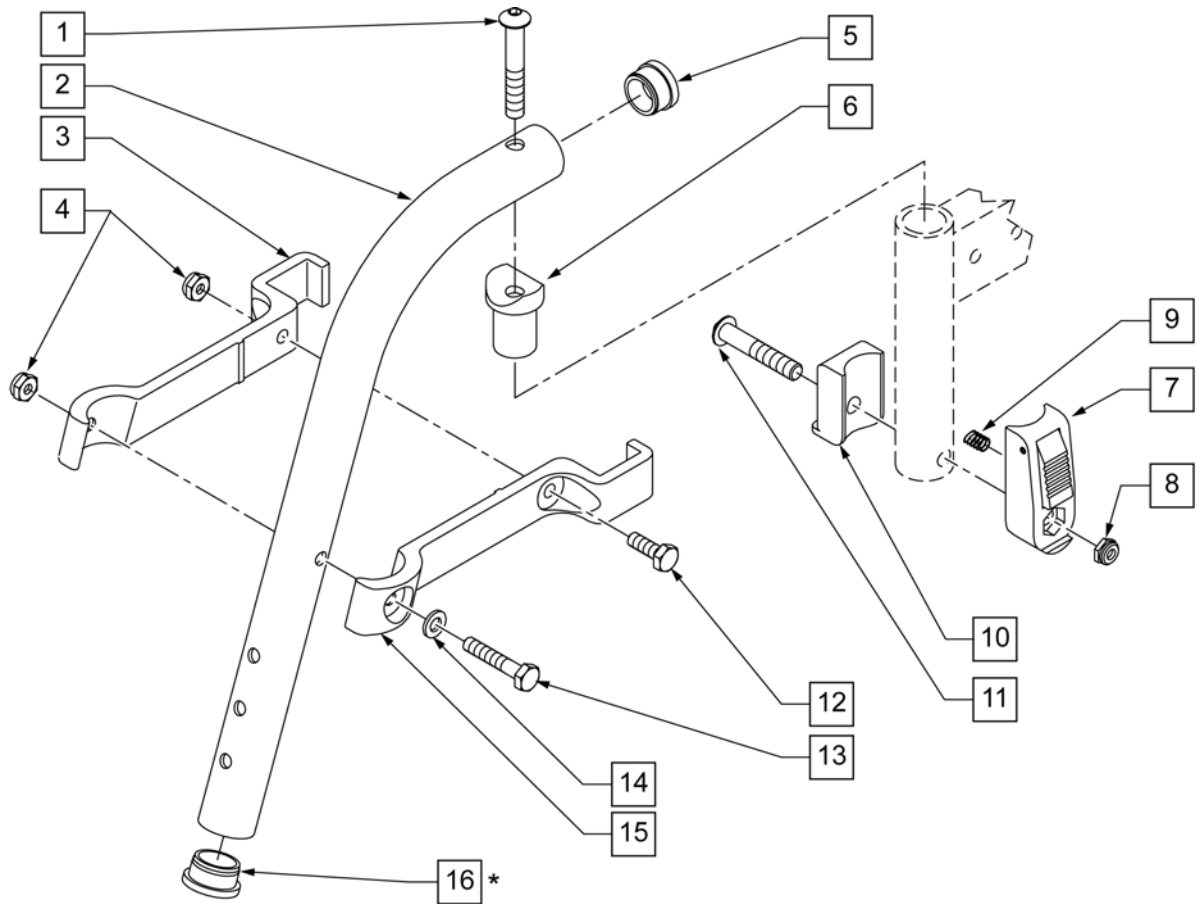


Pos.	Item Number	Description	Remarks
1	106983	BHCS 5/16 -18 x 2 SHLDR ROHS	
3	550152	T45 HANGER BRACKET 70° R	
4	106935	NUT NYLOCK THIN 1/4 -28 ROHS	
5	561002	1" INSERT PLUG .083	
6	561197	PLASTIC PIVOT SADDLE	
7	880053	FOOTREST BLOCK T45	
8	202117	5/16-18 NYLOCK NUT THICK BLK	
9	321141	.650 x 6 COIL SPRING	
10	560684	T45 FOOTREST BLOCK RECESSED	
11	107029	BHCS 5/16 -18 x 1 5/8 SHLD ROHS	
12	107014	HHCS 1/4 -28 x 3/4 ROHS	
13	106971	HHCS 1/4 -28 x 1 3/8 ROHS	
14	106943	WASHER FLAT AN 1/4 ROHS	
15	550153	T45 HANGER BRACKET 70° L	
16	561002	1" INSERT PLUG .083	ORDER WHEN NOT USING FOOTREST EXTENSION
7-11	920720	FOOTREST BLOCK ASSEMBLY W/BOLT HD	
1-6,12-15	641210S	HANGER 70D PNTD R HD	
1-6,12-15	641211S	HANGER 70D PNTD L HD	

NOTE: * ORDER ITEM 16 WHEN NOT USING FOOTREST EXTENSION.

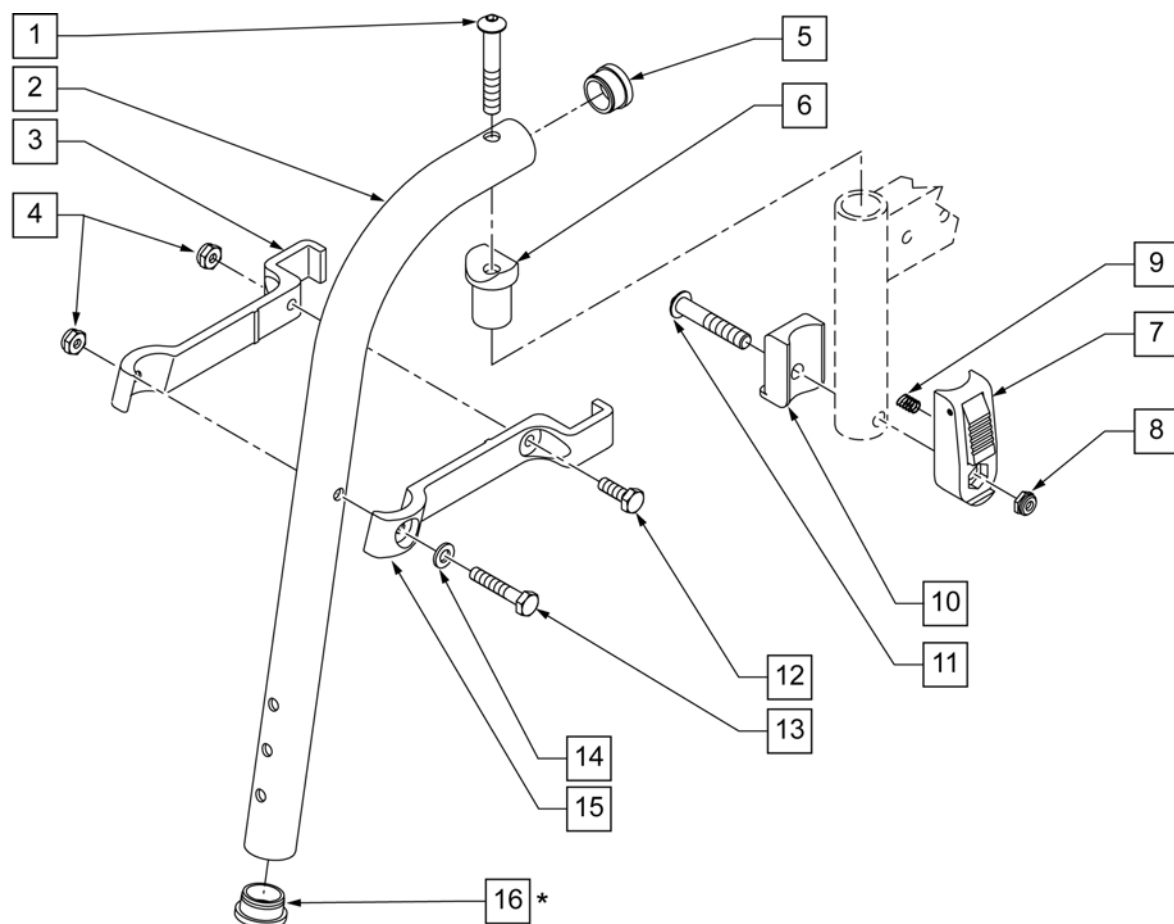


Pos.	Item Number	Description	Remarks
1	106983	BHCS 5/16 -18 x 2 SHLDR ROHS	
3	550154	T45 HANGER BRACKET 80° R	
4	106935	NUT NYLOCK THIN 1/4 -28 ROHS	
5	561002	1" INSERT PLUG .083	
6	561197	PLASTIC PIVOT SADDLE	
7	880053	FOOTREST BLOCK T45	
8	202117	5/16-18 NYLOCK NUT THICK BLK	
9	321141	.650L x 6 COIL SPRING	
10	560684	T45 FOOTREST BLOCK RECESSED	
11	107029	BHCS 5/16 -18 x 1 5/8 SHLD ROHS	
12	107014	HHCS 1/4 -28 x 3/4 ROHS	
13	106971	HHCS 1/4 -28 x 1 3/8 ROHS	
14	106943	WASHER FLAT AN 1/4 ROHS	
15	550155	T45 HANGER BRACKET 80° L	
16	561002	1" INSERT PLUG .083	ORDER WHEN NOT USING FOOTREST EXTENSION
7-11	920720	FOOTREST BLOCK ASSEMBLY W/BOLT HD	
1-6,12-15	641200S	HANGER 80D PNTD R HD	
1-6,12-15	641201S	HANGER 80D PNTD L HD	


NOTE: THIS HANGER HAS BEEN TESTED AND APPROVED FOR OCCUPANTS THAT WEIGH LESS THAN 150 POUNDS.

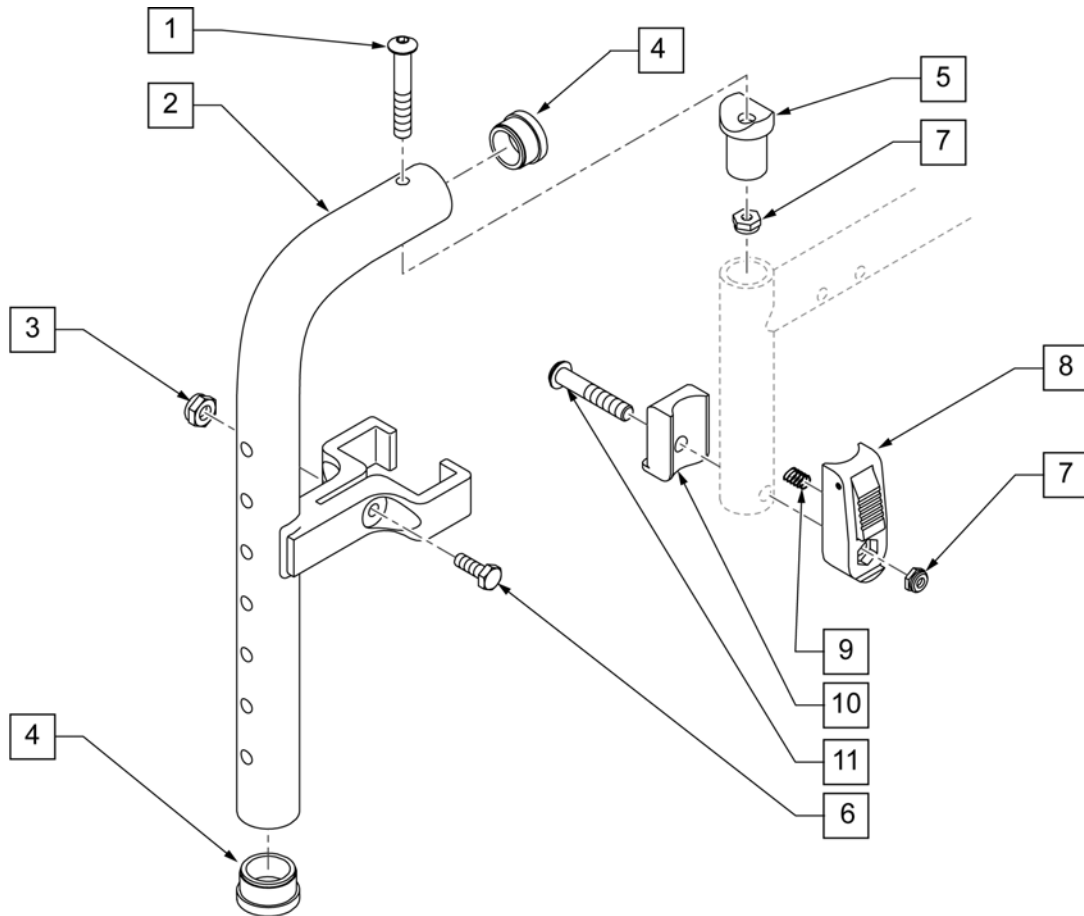

Pos.	Item Number	Description	Remarks
1	106983	BHCS 5/16 -18 x 2 SHLDR ROHS	
3	106935	NUT NYLOCK THIN 1/4 -28 ROHS	
4	561002	1" INSERT PLUG .083	
5	561197	PLASTIC PIVOT SADDLE	Includes nylock nut
6	107013	HHCS 1/4 -28 x 5/8 ROHS	
7	880053	FOOTREST BLOCK T45	
8	202117	5/16-18 NYLOCK NUT THICK BLK	
9	321141	.650L x 6 COIL SPRING	
10	560684	T45 FOOTREST BLOCK RECESSED	
11	107029	BHCS 5/16 -18 x 1 5/8 SHLD ROHS	
7-11	920720	FOOTREST BLOCK ASSM W/BOLT HD	
1-7	885005S	90 DEG F/D H/D L/O HNGR ASSM PNTD	

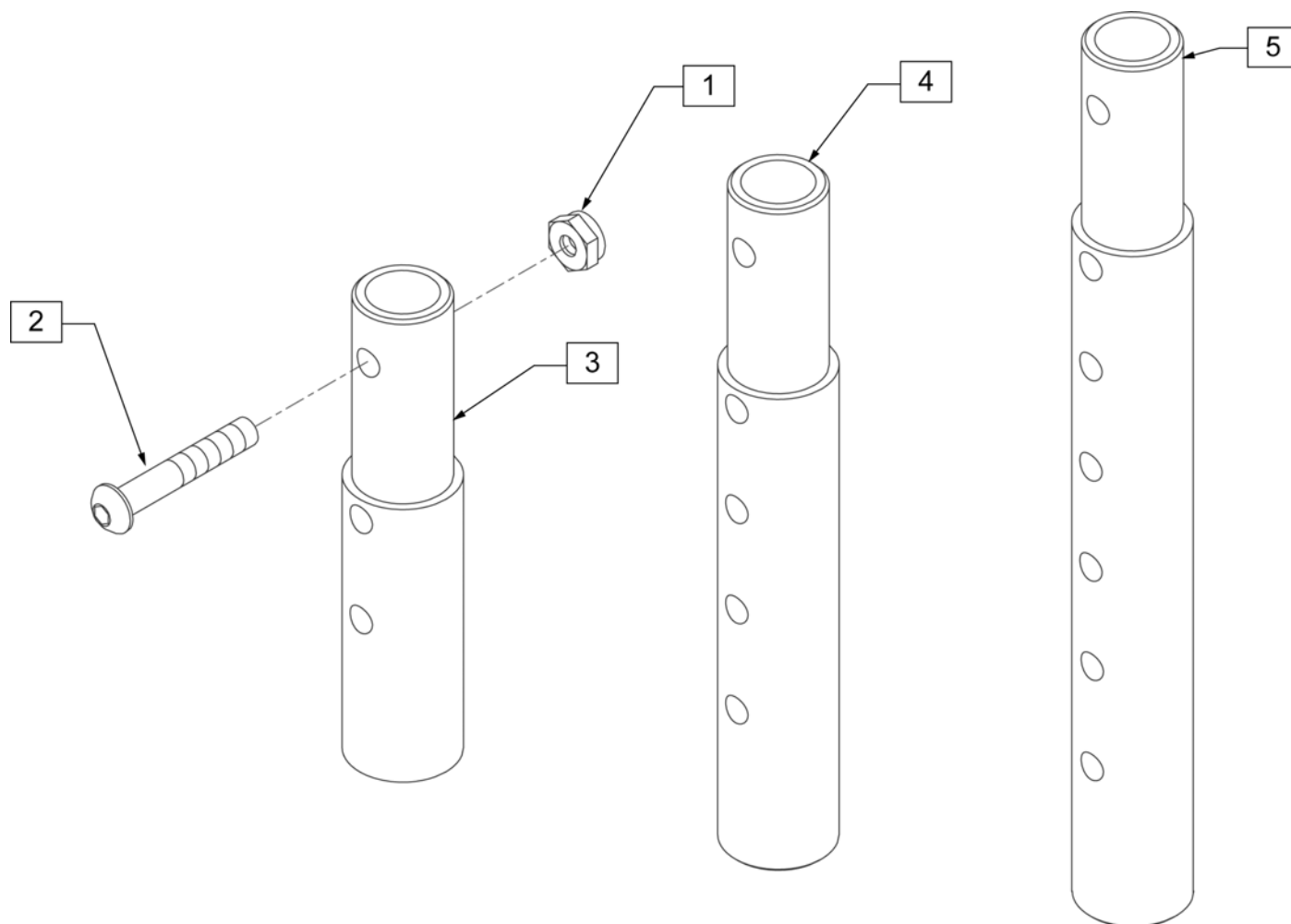


Pos.	Item Number	Description	Remarks
1	106935	NUT NYLOCK THIN 1/4 -28 ROHS	
2	106961	BHCS 1/4 -28 x 1 1/4 ROHS	
1-3	111505	2" FOOTREST EXTENSION TUBE W/ HDWR ASSM	2" EXT (PAIR
1, 2, 4	111506	4" FOOTREST EXTENSION TUBE W/ HDWR ASSM	4" EXT (PAIR)
1, 2, 5	111507	6" FOOTREST EXTENSION TUBE W/ HDWR ASSM	6" EXT (PAIR)

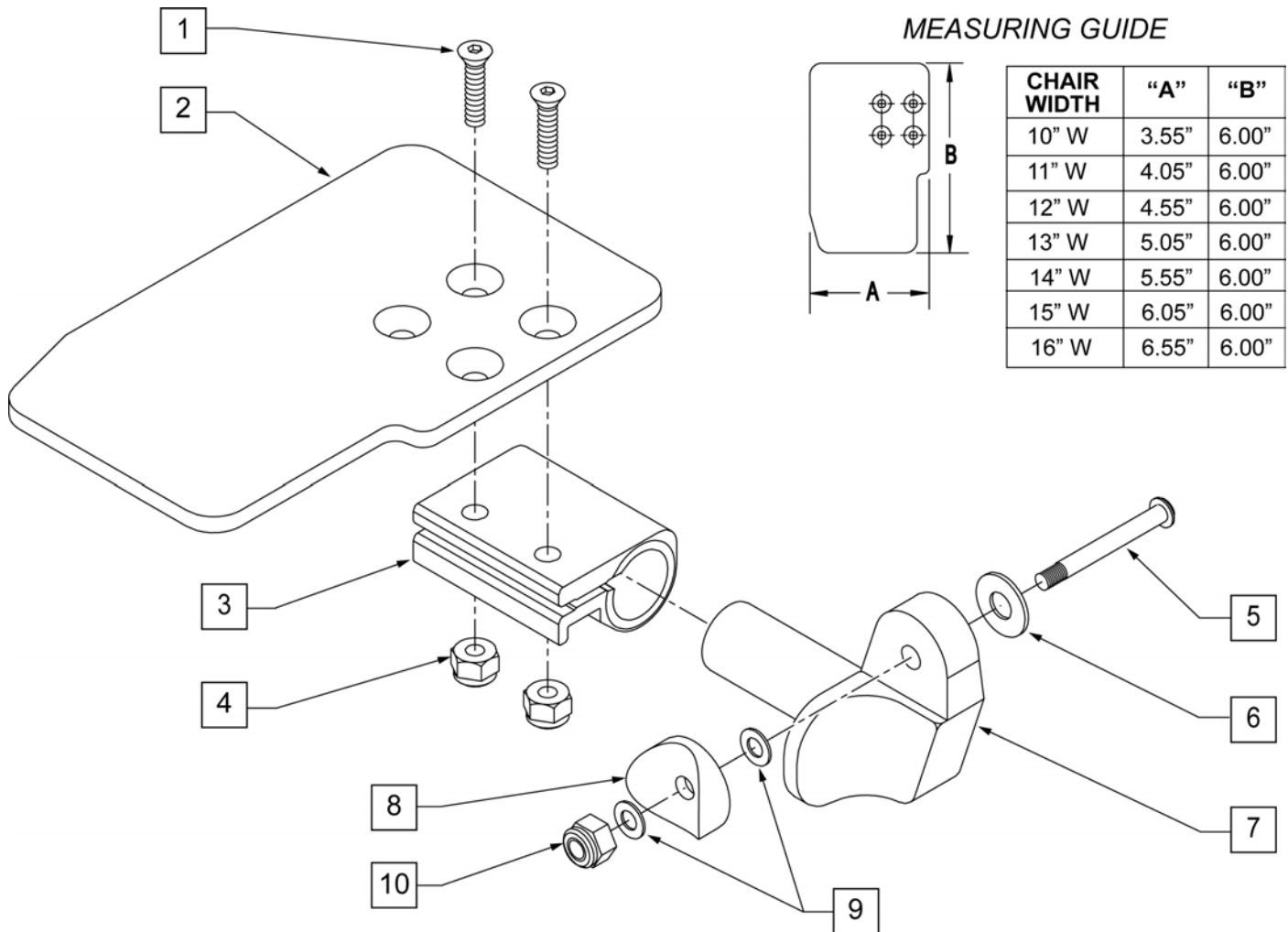


Pos.	Item Number	Description	Remarks	Pos.	Item Number	Description	Remarks
1	106978	FHCS 1/4-28 X 7/8 ROHS		7	541001	FTRST MOUNTING BRKT 1 RAD	LEFT
2	500635	KIDS ANGL ADJ F/PLT 10	RIGHT	8	561004	1 SADDLE 1/4 HOLE	
2	500636	KIDS ANGL ADJ F/PLT 10	LEFT	9	106943	WASHER FLAT THIN AN 1/4	
2	500637	KIDS ANGL ADJ F/PLT 11	RIGHT	10	106941	NUT AUTOMATION 1/4-28 ROHS	
2	500638	KIDS ANGL ADJ F/PLT 11	LEFT	1-10	828821	KIDS ANG ADJ F/PLT ASSM 10	RIGHT
2	500639	KIDS ANGL ADJ F/PLT 12	RIGHT	1-10	828822	KIDS ANG ADJ F/PLT ASSM 10	LEFT
2	500640	KIDS ANGL ADJ F/PLT 12	LEFT	1-10	828823	KIDS ANG ADJ F/PLT ASSM 11	RIGHT
2	500641	KIDS ANGL ADJ F/PLT 13	RIGHT	1-10	828824	KIDS ANG ADJ F/PLT ASSM 11	LEFT
2	500642	KIDS ANGL ADJ F/PLT 13	LEFT	1-10	828825	KIDS ANG ADJ F/PLT ASSM 12	RIGHT
2	500643	KIDS ANGL ADJ F/PLT 14	RIGHT	1-10	828826	KIDS ANG ADJ F/PLT ASSM 12	LEFT
2	500644	KIDS ANGL ADJ F/PLT 14	LEFT	1-10	828827	KIDS ANG ADJ F/PLT ASSM 13	RIGHT
2	500645	KIDS ANGL ADJ F/PLT 15	RIGHT	1-10	828828	KIDS ANG ADJ F/PLT ASSM 13	LEFT
2	500646	KIDS ANGL ADJ F/PLT 15	LEFT	1-10	828829	KIDS ANG ADJ F/PLT ASSM 14	RIGHT
2	500647	KIDS ANGL ADJ F/PLT 16	RIGHT	1-10	828830	KIDS ANG ADJ F/PLT ASSM 14	LEFT
2	500648	KIDS ANGL ADJ F/PLT 16	LEFT	1-10	828831	KIDS ANG ADJ F/PLT ASSM 15	RIGHT
3	550350	3/4 CLAMP ADJ FOOTPLATE		1-10	828832	KIDS ANG ADJ F/PLT ASSM 15	LEFT
4	106935	NUT NYLOCK THIN 1/4-28 ROHS		1-10	828833	KIDS ANG ADJ F/PLT ASSM 16	RIGHT
5	226147	1/4-28 X 1.85 BH C/S BLK ZN		1-10	828834	KIDS ANG ADJ F/PLT ASSM 16	LEFT
6	106945	WASHER FLAT SAE 1/4 ROHS		1-10	921226	KIDS ANG ADJ F/PLT ASSM 10	PAIR
7	541000	FTRST MOUNTING BRKT 1 RAD	RIGHT	1-10	921227	KIDS ANG ADJ F/PLT ASSM 11	PAIR

Pos.	Item Number	Description	Remarks	Pos.	Item Number	Description	Remarks
1-10	921228	KIDS ANG ADJ F/PLT ASSM 12	PAIR	1-10	921231	KIDS ANG ADJ F/PLT ASSM 15	PAIR
1-10	921229	KIDS ANG ADJ F/PLT ASSM 13	PAIR	1-10	921232	KIDS ANG ADJ F/PLT ASSM 16	PAIR
1-10	921230	KIDS ANG ADJ F/PLT ASSM 14	PAIR				

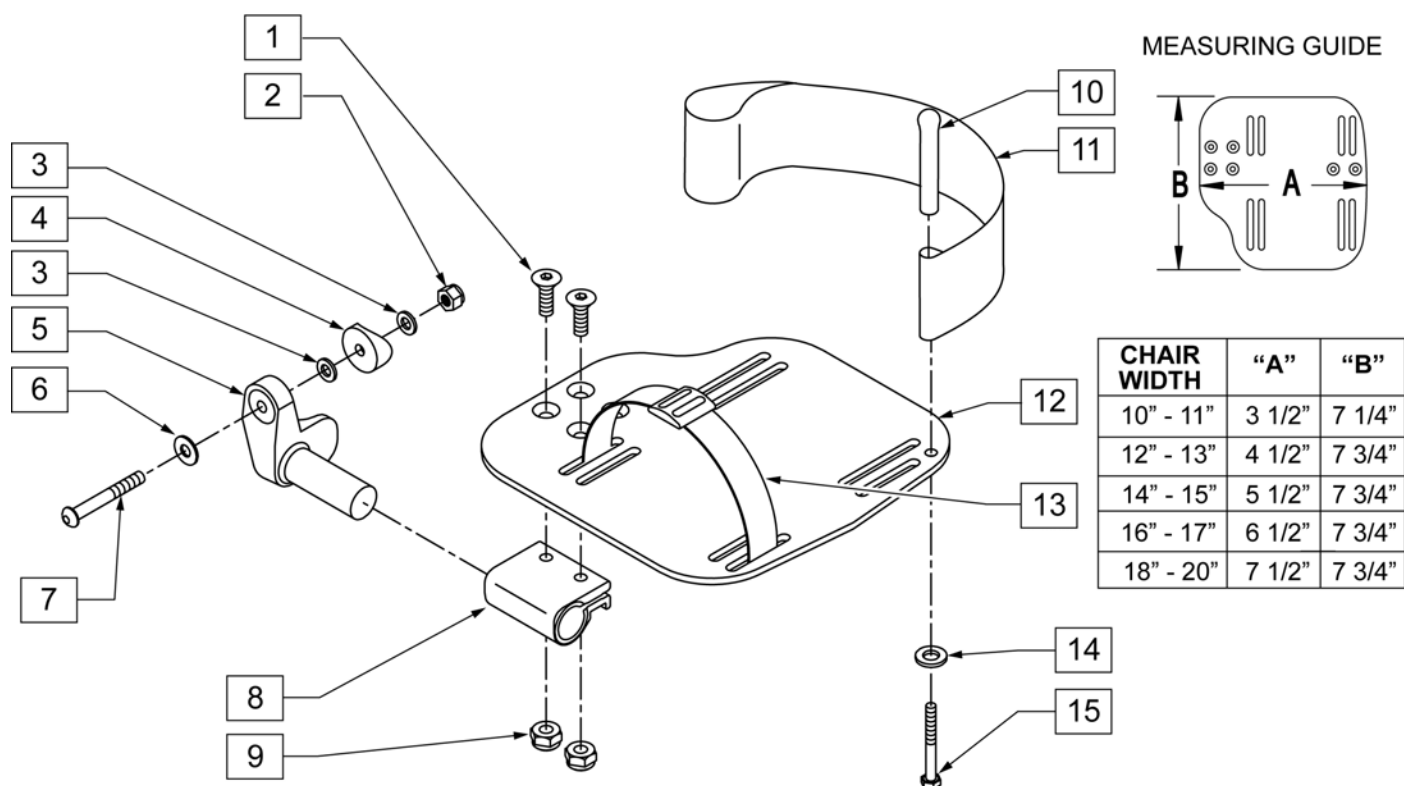


Pos.	Item Number	Description	Remarks
1	106978	FHCS 1/4-28 X 7/8 ROHS	
2	106941	NUT AUTOMATION 1/4-28 ROHS	
3	106943	WASHER FLAT AN 1/4 ROHS	
4	561004	1" SADDLE 1/4" HOLE	
5	541000	FOOTREST MOUNTING BRKT-1" RAD	RIGHT
5	541001	FOOTREST MOUNTING BRKT-1" RAD	LEFT
6	106945	WASHER FLAT SAE 1/4 ROHS	
7	106964	BHCS 1/4-28 X 2	
8	550350	3/4" CLAMP ADJ FOOTPLATE	
9	106935	NUT NYLOCK THIN 1/4-28 ROHS	
10	561040	HEEL LOOP PEG	
11	741153	HEEL LOOP 18-20 CMPST/14-15 ANG ADJ	USED W/ 14-15"/16-17" ADULT ANG ADJ FTPLTS
11	741253	HEEL LOOP ADLT ADJ F/PLT 18-20	USED W/ 18 -20" ADULT ANG ADJ FTPLTS
12	500656	ADULT ANGL ADJ F/PLT 10/11	RIGHT
12	500657	ADULT ANGL ADJ F/PLT 10/11	LEFT
12	500658	ADULT ANGL ADJ F/PLT 12/13	RIGHT
12	500659	ADULT ANGL ADJ F/PLT 12/13	LEFT
12	500660	ADULT ANGL ADJ F/PLT 14/15	RIGHT
12	500661	ADULT ANGL ADJ F/PLT 14/15	LEFT
12	500662	ADULT ANGL ADJ F/PLT 16/17	RIGHT
12	500663	ADULT ANGL ADJ F/PLT 16/17	LEFT
12	500664	ADULT ANGL ADJ F/PLT 18-20	RIGHT
12	500665	ADULT ANGL ADJ F/PLT 18-20	LEFT
13	740260	TOE-ANKLE STRAP ADJ ADULT FOOTPLATE	
14	106943	WASHER FLAT AN 1/4 ROHS	

Pos.	Item Number	Description	Remarks
15	106958	HHCS 1/4-20 X 1 ROHS	
1-9,12	820544	FP ADULT ADJ 90D 10/11 R	RIGHT
1-9,12	820554	FP ADULT ADJ 90D 10/11 L	LEFT
1-9,12	820546	FP ADULT ADJ 90D 12/13 R	RIGHT
1-9,12	820556	FP ADULT ADJ 90D 12/13 L	LEFT
1-9,12	820472	FP ADULT ADJ 90D 14/15 R	RIGHT
1-9,12	820473	FP ADULT ADJ 90D 14/15 L	LEFT
1-9,12	820474	FP ADULT ADJ 90D 16/17 R	RIGHT
1-9,12	820475	FP ADULT ADJ 90D 16/17 L	LEFT
1-9,12	820476	FP ADULT ADJ 90D 18-20 R	RIGHT
1-9,12	820477	FP ADULT ADJ 90D 18-20 L	LEFT
1-9,12	921474	ADULT ANGL ADJ 90D 10/11 OMIT H/L	PAIR
1-9,12	921476	ADULT ANGL ADJ 90D 12/13 OMIT H/L	PAIR
1-9,12	921330	ADULT ANGL ADJ 90D 14/15 OMIT H/L	PAIR
1-9,12	921331	ADULT ANGL ADJ 90D 16/17 OMIT H/L	PAIR
1-9,12	921332	ADULT ANGL ADJ 90D 18-20 OMIT H/L	PAIR
1-12,14,15	820540	F/PLT ADULT ADJ 90D 10/11 R W/ HL	RIGHT
1-12,14,15	820550	F/PLT ADULT ADJ 90D 10/11 L W/ HL	LEFT
1-12,14,15	820542	F/PLT ADULT ADJ 90D 12/13 R W/ HL	RIGHT
1-12,14,15	820552	F/PLT ADULT ADJ 90D 12/13 L W/ HL	LEFT
1-12,14,15	820478	F/PLT ADULT ADJ 90D 14/15 R W/ HL	RIGHT
1-12,14,15	820479	F/PLT ADULT ADJ 90D 14/15 L W/ HL	LEFT
1-12,14,15	820480	F/PLT ADULT ADJ 90D 16/17 R W/ HL	RIGHT
1-12,14,15	820481	F/PLT ADULT ADJ 90D 16/17 L W/ HL	LEFT
1-12,14,15	820482	F/PLT ADULT ADJ 90D 18-20 R W/ HL	RIGHT
1-12,14,15	820483	F/PLT ADULT ADJ 90D 18-20 L W/ HL	LEFT
1-12,14,15	921470	F/PLT ADULT ADJ 90D 10/11 W/ HL	PAIR
1-12,14,15	921472	F/PLT ADULT ADJ 90D 12/13 W/ HL	PAIR
1-12,14,15	921333	F/PLT ADULT ADJ 90D 14/15 W/ HL	PAIR
1-12,14,15	921334	F/PLT ADULT ADJ 90D 16/17 W/ HL	PAIR
1-12,14,15	921335	F/PLT ADULT ADJ 90D 18-20 W/ HL	PAIR

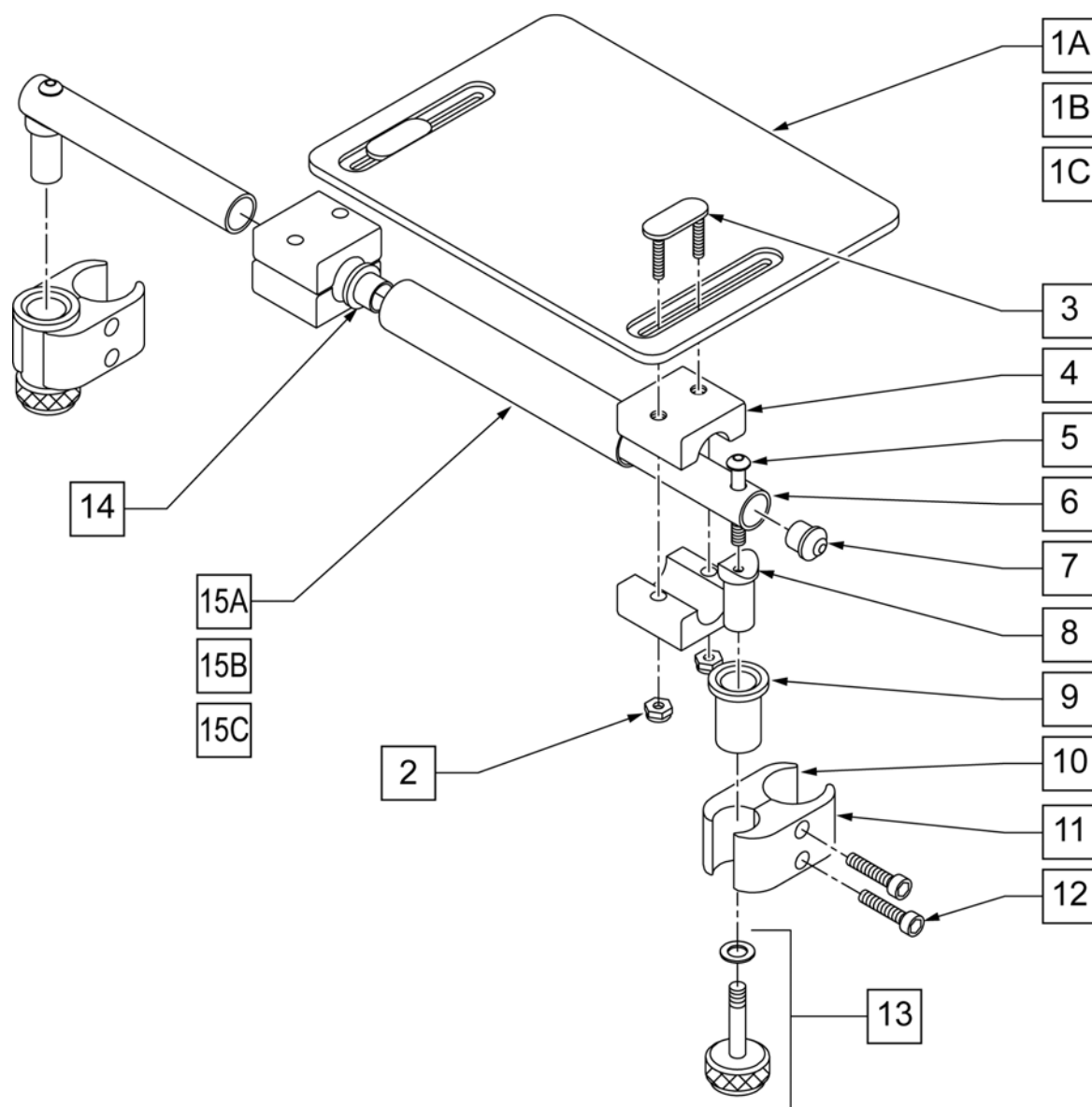


Pos.	Item Number	Description	Remarks
1A	562610	SLD FOOTPLATE ONLY 8" x 8" (10"-12" W)	10 -12" WIDE
1B	562612	SLD FOOTPLATE ONLY 10" x 8" (13"-14" W)	13 -14" WIDE
1C	500709	SLD FT PLT ALUM ONLY 12" x 8" (15"-16" W)	15 -16" WIDE
2	106933	NUT NYLOCK THIN 1/4-20 ROHS	
3	330120	SLD FT/ANGLE ADJ DOUBLE T BOLT	
4	550444	SOLID FOOT/ANG ADJ CLAMP BLK	
5	107028	BHCS 1/4-28 X 1-1/2 ROHS	
7	561034	SHORT DOME PLUG	
8	483236	SOLID FOOT INSERT SADDLE BLK	
10	550658	CLAMP 1 1/4" THREADED BLK	
11	550657	CLAMP 1 1/4" RECESS FULL BLK	
12	107032	SHCS 1/4-28 X 1 ROHS	
9,13	880559	SOLID FOOT LOCK KNOB ASSEMBLY	

Pos.	Item Number	Description	Remarks
14	560402	3/4" ID NY-LINER	
15A	521621	SLD FOOT SUPPORT TUBE BLK 10"-12"	10 -12" WIDE
15B	521622	SLD FOOT SUPPORT TUBE BLK 13"-14"	13 -14" WIDE
15C	521624	SLD FOOT SUPPORT TUBE BLK 15"-16"	15 -16" WIDE
5-8	880183	SLD FT PLT PIVOT TUBE SUB-ASSM	
1A,2-14, 15A	921801	SLD FT PLT ASSEMBLY 8" x 8" (10"-12" W)	10 -12" WIDE
1B,2-14, 15B	921803	SOLID FOOTPLATE ASSEMBLY 10" x 8" (13"-14" W)	13 -14" WIDE
1C,2-14, 15C	921805	SOLID FOOTPLATE ASSEMBLY 12" x 8" (15"-16" W)	15 -16" WIDE

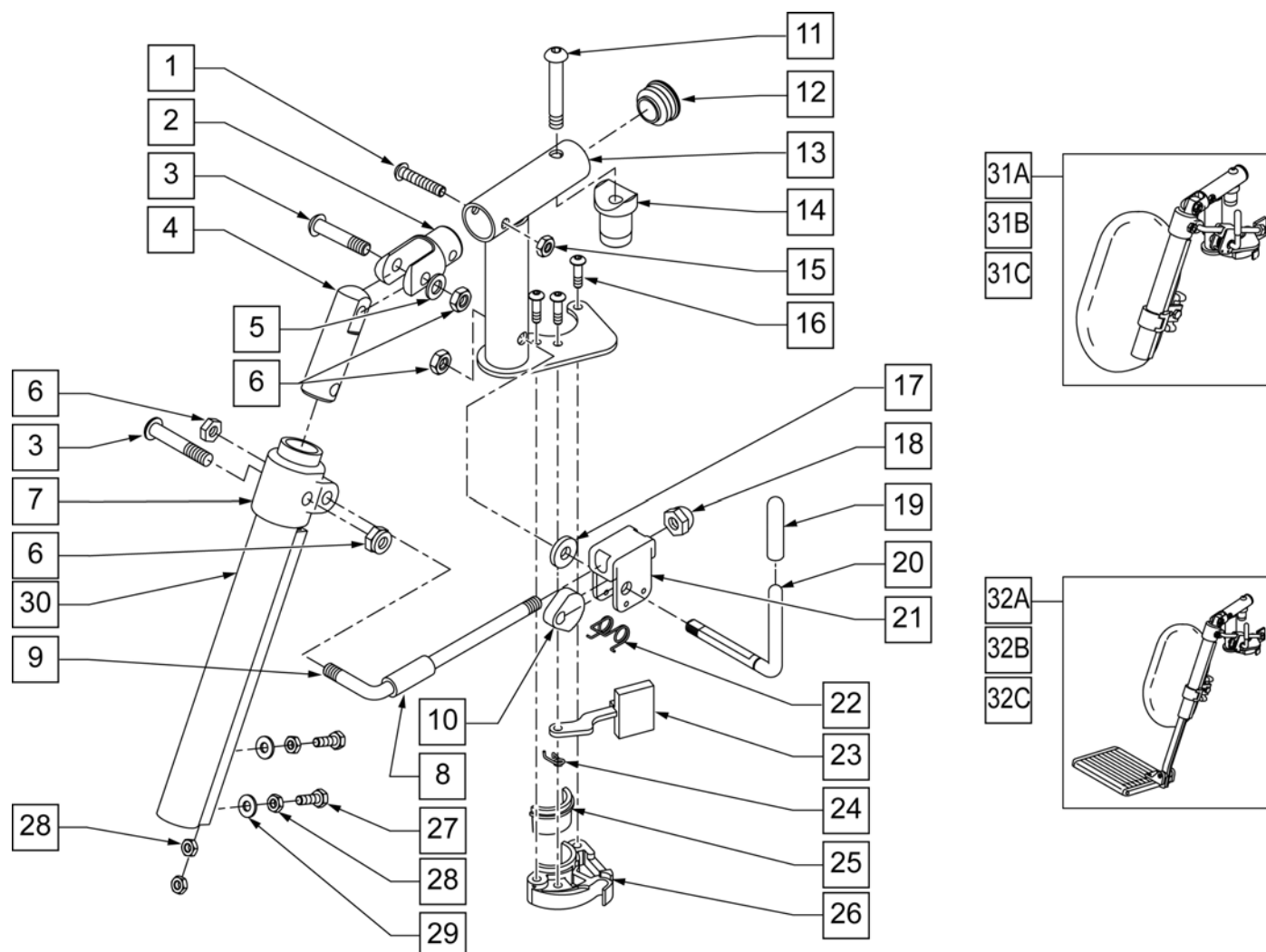


Pos.	Item Number	Description	Remarks
1	106961	1/4-28 X 1 1/4 BH C/S	
2	505281	ELR CLEVIS	
3	107037	BHCS 5/16-24 X 1-1/2 SHLD ROHS	
4	540252	ELR PIVOT PRS	
5	106946	WASHER FLAT AN 5/16 ROHS	
6	106939	NUT NYLOCK THIN 5/16-24 ROHS	
7	560420	ELR UPPER PIVOT PLST	
8	560784	ELR ADJUSTMENT STOP	
9	481117	ELR LOCK ROD	
10	481119	ELR CAM LOCKING POWDERED METAL	
11	106982	BHCS 5/16 -18 x 1 3/4 SHLD ROHS	
12	561062	1" INSERT PLUG .065 RIBS	
14	122729	ELR ALUM PIVOT SADDLE	
15	106935	NUT NYLOCK THIN 1/4-28 ROHS	
16	107035	BHCS 10-24 X .725 PL ROHS	
17	561059	3/4 OD x .318 ID WASHER	
18	202109	5/16-24 ACORN NUT HEX	
19	561049	ELR RELEASE ROD COVER	
19,20	882410S	ELR ACTUATION ARM ASSEMBLY	


Pos.	Item Number	Description	Remarks
21	880990	ELR LOCK HOUSING W/PIN	
22	321106	ELR TORSION SPRING	
23	561079	S/A LATCH LEVER	
24	321136	S/A SPRING L RED	LEFT
24	321137	S/A SPRING R CLEAR	RIGHT
27	107014	1/4-28 x 3/4 HEX C/S	
28	107249	NUT HEX THIN 1/4-28 ROHS	
29	106944	WASHER FLAT THIN AN 1/4 ROHS	
30	550722S	ELR ADJUSTMENT TUBE	
6,8-10,17-22	881077S	ELR LOCK ROD/ARM ASSEMBLY	
25,26	881500	LATCH BLOCK ASSM	RIGHT
25,26	881501	LATCH BLOCK ASSM	LEFT
1-6,11-16,20-24	882413	ELR PIVOT BLOCK ASSM	RIGHT
1-6,11-16,20-24	882414	ELR PIVOT BLOCK ASSM	LEFT
1-26	881422	ELR PIVOT ASSM RIGHT	RIGHT
1-26	881423	ELR PIVOT ASSM LEFT	LEFT
1-30	641452S	ELR HNGR W/O PAD PNTD MNL	RIGHT
1-30	641453S	ELR HNGR W/O PAD PNTD MNL	LEFT
31A	923082S	ELR HNGR ASSM 14-15 W /PAD (PR)	PAIR
31B	923041S	ELR HNGR ASSM W/PAD 16-17"	PAIR
31C	923061S	ELR HNGR ASSM W/PAD 18-20"	PAIR
32A	923088S	ELR HNGR REG EXT 14 15" F/PLT	Pair-includes short extensions, 11.5" - 15.5"
32B	923084S	ELR HNGR REG EXT 16 17" F/PLT	Pair-includes short extensions, 11.5" - 15.5"
32C	923085S	ELR HNGR REG EXT 18-20" F/PLT	Pair-includes short extensions, 11.5" - 15.5"
16,23-26	881521	S/A LATCH KIT CASTING RIGHT	RIGHT
16,23-26	881522	S/A LATCH KIT CASTING LEFT	LEFT

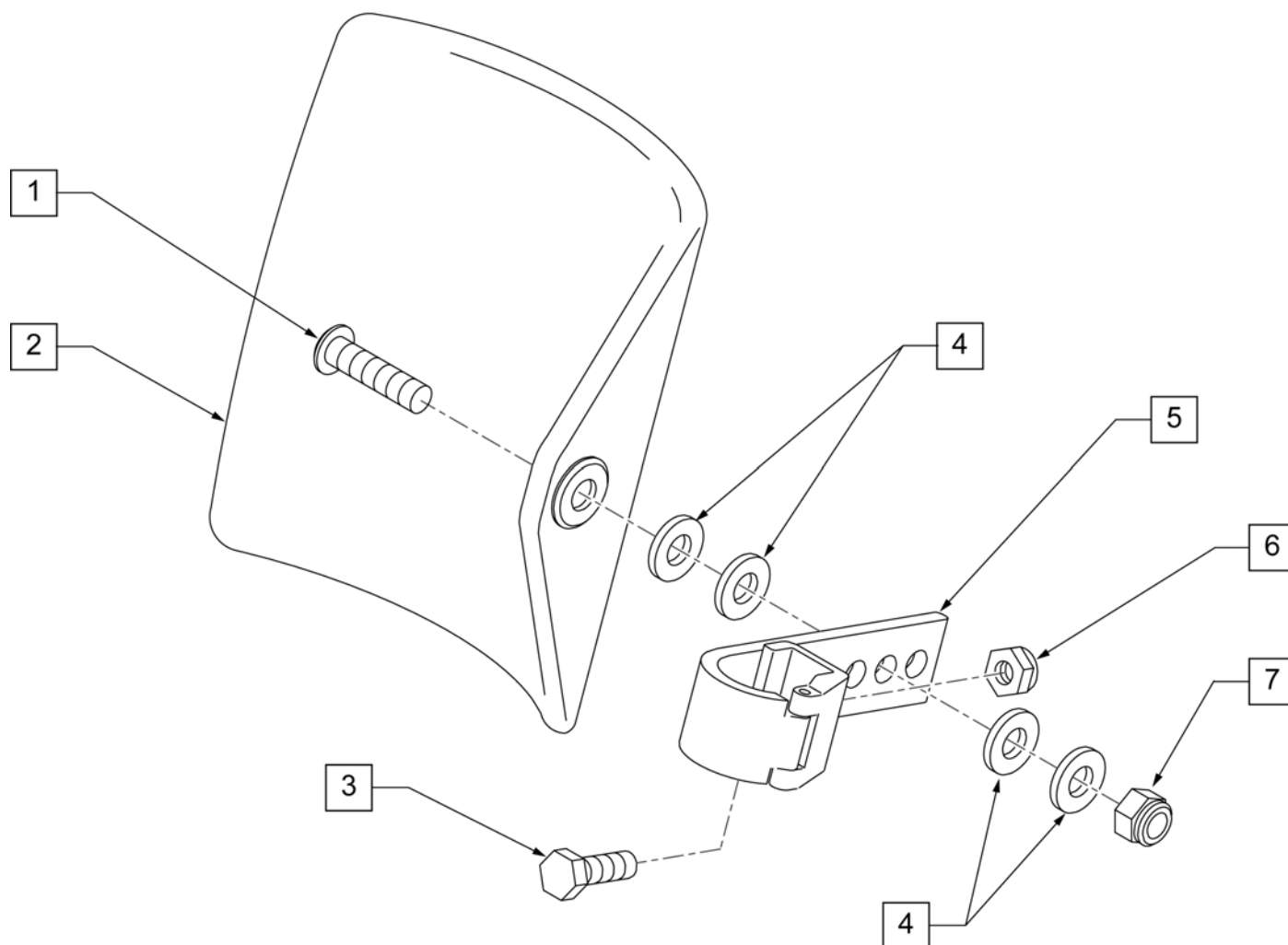

Pos.	Item Number	Description
1	106980	BHCS 5/16-18 X 1-1/4 ROHS
2	755014	CALF PAD TROUGH STYLE 14"/16"
3	107013	HHCS 1/4-28 X 5/8 ROHS
4	561016	1" WASHER 5/16" HOLE
5	881413	CALF PAD BRACKET ASSEMBLY
6	106934	NUT NYLOCK 1/4-28 ROHS
7	202117	5/16-18 NYLOCK NUT THICK BLK
1-7	881414	CALF PAD J-HOOK 14"-15" ASSEMBLY

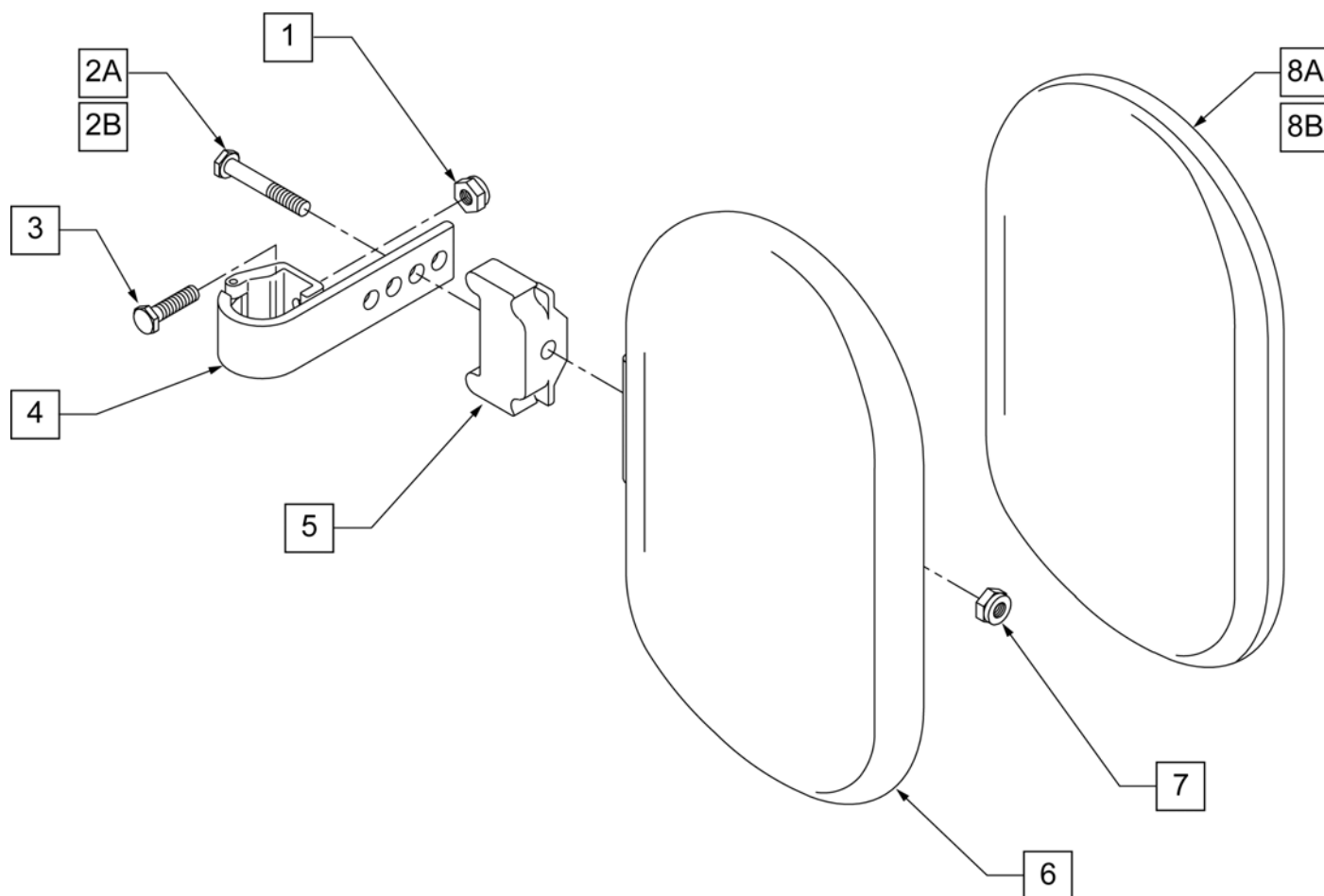


Pos.	Item Number	Description	Remarks
1	106934	NUT NYLOCK 1/4-28 ROHS	
2A	107018	HHCS 5/16-18 X 2-1/2 ROHS	16"-17" wide only
2B	107019	HHCS 5/16-18 X 3-1/2 ROHS	18"-20" wide only
3	107013	HHCS 1/4-28 X 5/8 ROHS	
4	881413	CALF PAD BRACKET ASSEMBLY	
5	560199	ELR CALF HINGE SPACER	18"-20" wide only
6	560108	CALF PAD MOLDED 16"-20"	
7	106936	NUT NYLOCK 5/16-18 ROHS	
8A	751045	CALF PAD COVER	
8B	751046	CALF PAD COVER (BREEZY)	
1,2A,3,4,6,7,8A	881416	CALF MOLDED 16"-17" ASSEMBLY	
1,2B,3-7,8A	881412	CALF PAD MOLDED 18"-20" ASSEMBLY	

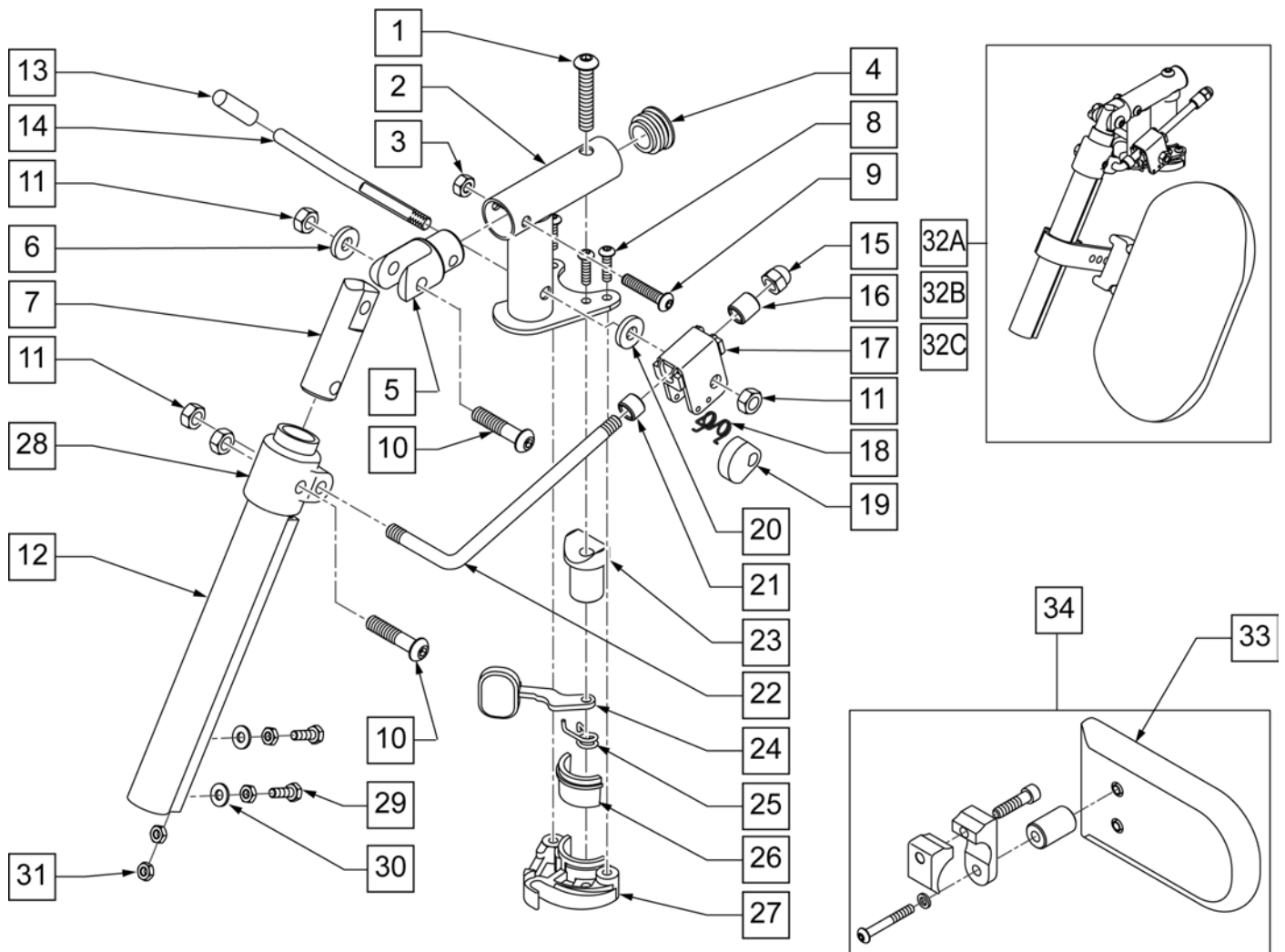


Pos.	Item Number	Description	Remarks
1	226217	5/16-18 x 2" BH C/S BLK	
3	106935	NUT NYLOCK THIN 1/4-28 ROHS	
4	561062	1" INSERT PLUG .065 RIBS	
5	505281	ELR CLEVIS	
6	263100	5/16 AN WASHER FLAT	
7	540252	ELR PIVOT PRS	
8	226057	10/24 x .725 BTTN HD SOC BZ	
9	106961	BHCS 1/4-28 X 1-1/4 ROHS	
10	226138	5/16-24 x 1 1/2 BH C/S	
11	106939	5/16-24 NYLOCK NUT THIN	
12	550722S	ELR KIDS TUBE (SPECIFY COLOR)	SPECIFY COLOR
15	202109	5/16-24 ACORN NUT HEX	
16	560815	HEMI ELR ADJ STOP BACK LG	
17	880990	ELR LOCK HOUSING W/PIN	
18	321106	ELR TORSION SPRING	
19	481119	ELR CAM LOCKING POWDERED METAL	
20	561059	3/4 OD x .318 ID WASHER	


Pos.	Item Number	Description	Remarks
21	500806	HEMI ELR ADJ STOP FRONT SMALL	
22	481117	ELR LOCK ROD	
23	561197	PLASTIC PIVOT SADDLE	
24	561079	S/A LATCH LEVER	
25	321136	S/A SPRING L RED	LEFT
25	321137	S/A SPRING R CLEAR	RIGHT
28	560420	ELR UPPER PIVOT PLST	
29	107014	HHCS 1/4-28 X 3/4 ROHS	
30	106944	WASHER FLAT THIN AN 1/4 ROHS	
31	201112	1/4 -28 NUT THIN	
32A	922324S	HEMI ELR HANGER ASSM 14 -15" W/ PAD	
32B	922325S	HEMI ELR HANGER ASSM 16 -17" W/ PAD	
32C	922326S	HEMI ELR HANGER ASSM 18 -20" W/ PAD	
33	750226	KNEE PAD FLAT 3H X 5L W/COVER	
34	922328	ELR HEMI KNEE PAD ASSM	
26,27	881500	LATCH BLOCK ASSM R	RIGHT
26,27	881501	LATCH BLOCK ASSM L	LEFT
13,14	882411	ELR 90D ACTUATION ARM ASSM	
1-11,23-27	921826	ELR PIVOT BLOCK ASSM R	RIGHT
1-11,23-27	921827	ELR PIVOT BLOCK ASSM L	LEFT
1-11,13-28	921888	HEMI ELR PIVOT ASSEMBLY RIGHT	RIGHT
1-11,13-28	921889	HEMI ELR PIVOT ASSEMBLY LEFT	LEFT
11,13-19, 21,22	922323S	HEMI ELR LOCK ROD/ARM ASSM	
1-31	921828S	HEMI ELR HANGER W/O PAD R MNL	RIGHT
1-31	921829S	HEMI ELR HANGER W/O PAD L MNL	LEFT

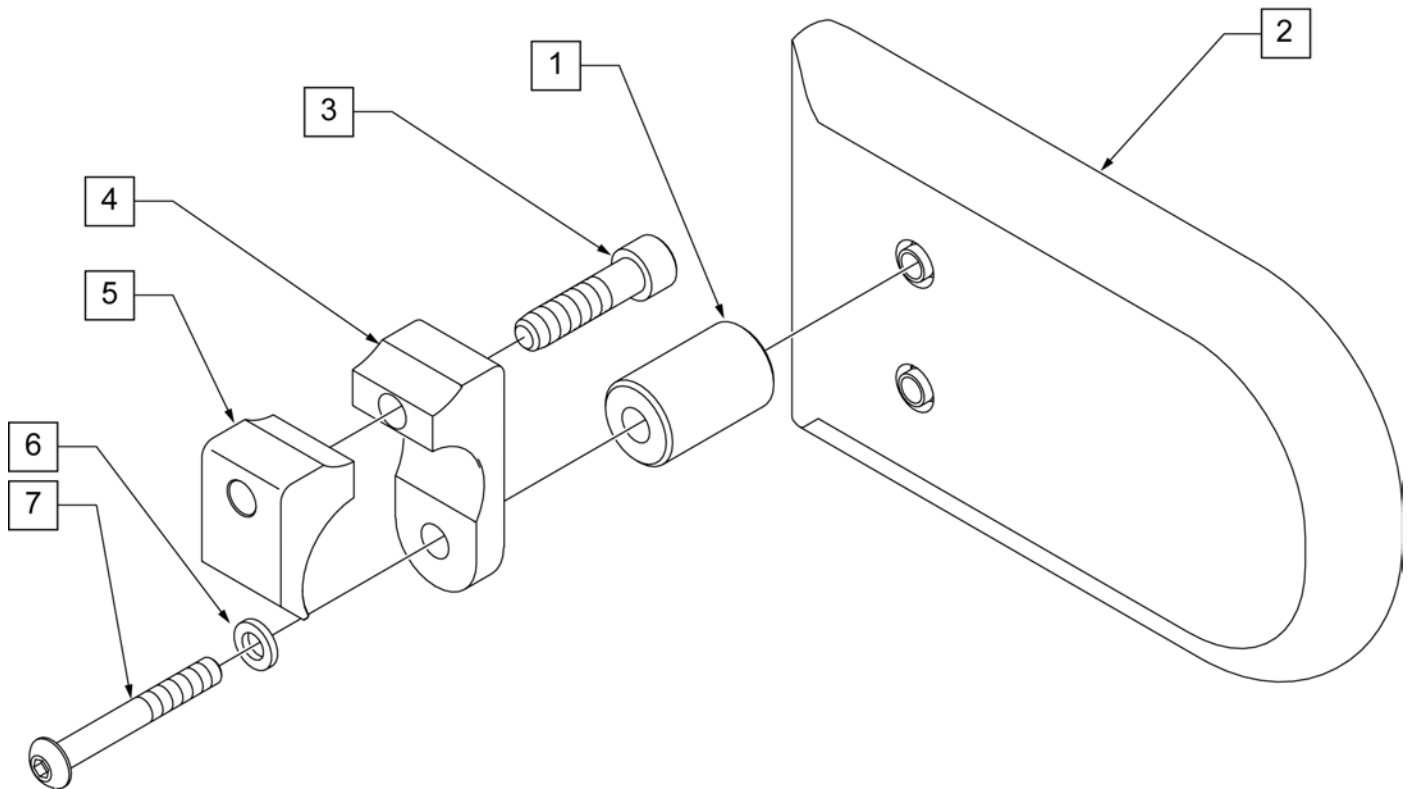

Pos.	Item Number	Description
2	750226	KNEE PAD FLAT 3H X 5L W/COVER
1-7	922328	ELR HEMI KNEE PAD ASSM

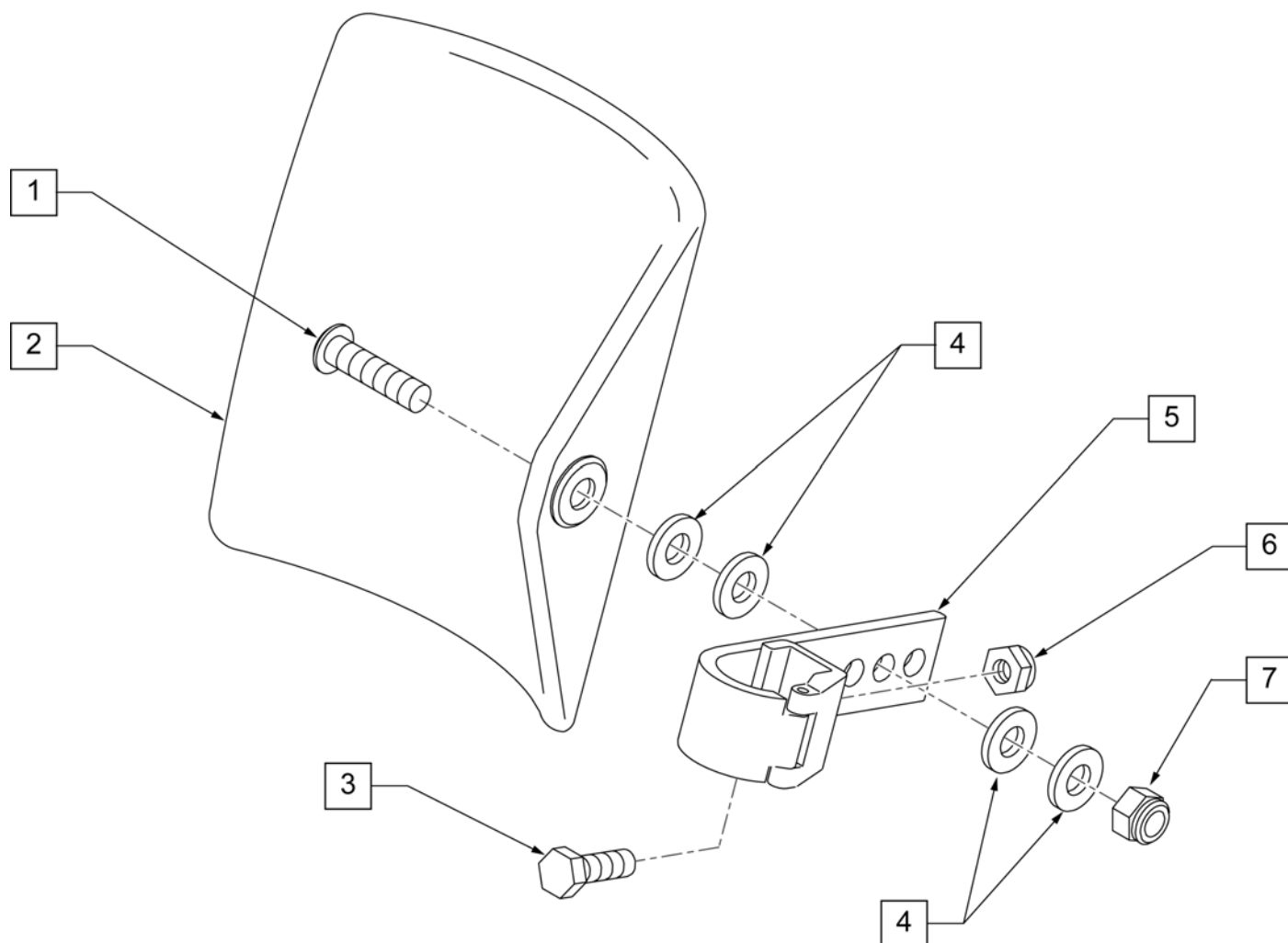


Pos.	Item Number	Description
1	106980	BHCS 5/16-18 X 1-1/4 ROHS
2	755014	CALF PAD TROUGH STYLE 14"/16"
3	107013	HHCS 1/4-28 X 5/8 ROHS
4	561016	1" WASHER 5/16" HOLE
5	881413	CALF PAD BRACKET ASSEMBLY
6	106934	NUT NYLOCK 1/4-28 ROHS
7	202117	5/16-18 NYLOCK NUT THICK BLK
1-7	881414	CALF PAD J-HOOK 14"-15" ASSEMBLY

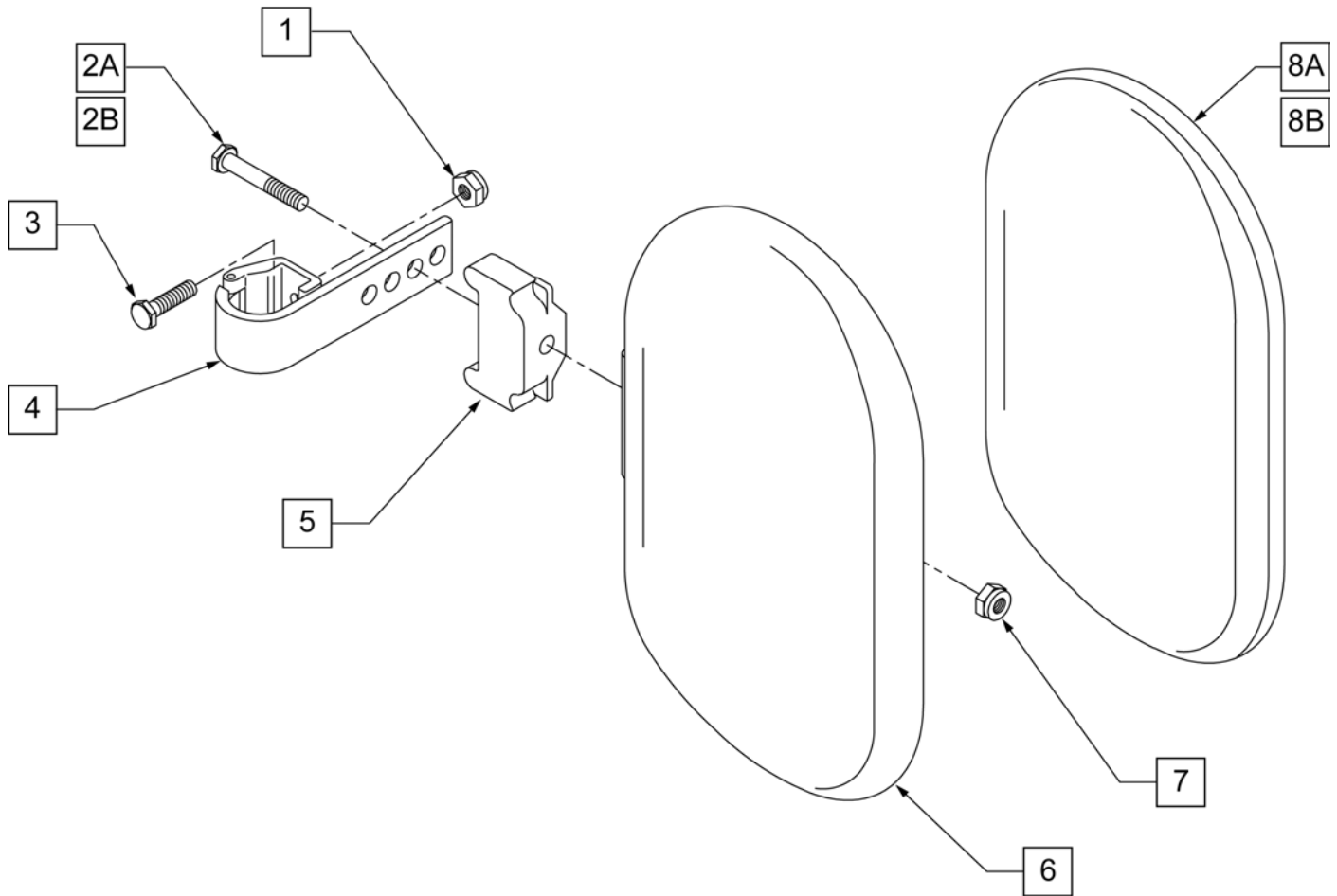

Pos.	Item Number	Description	Remarks
1	106934	NUT NYLOCK 1/4-28 ROHS	
2A	107018	HHCS 5/16-18 X 2-1/2 ROHS	16"-17" wide only
2B	107019	HHCS 5/16-18 X 3-1/2 ROHS	18"-20" wide only
3	107013	HHCS 1/4-28 X 5/8 ROHS	
4	881413	CALF PAD BRACKET ASSEMBLY	
5	560199	ELR CALF HINGE SPACER	18"-20" wide only
6	560108	CALF PAD MOLDED 16"-20"	
7	106936	NUT NYLOCK 5/16-18 ROHS	
8A	751045	CALF PAD COVER	
8B	751046	CALF PAD COVER (BREEZY)	
1,2A,3,4,6,7,8A	881416	CALF MOLDED 16"-17" ASSEMBLY	
1,2B,3-7,8A	881412	CALF PAD MOLDED 18"-20" ASSEMBLY	


NOTE: IF REPLACING AN EXTENSION TUBE ON AN MANUAL ALR PLEASE REFER TO THE ROUND EXT TUBE PARTS MANUAL PAGE.

NOTE: WHEN ORDERING REPLACEMENT EXTENSION TUBES PLEASE VERIFY IF YOU HAVE THE OLD STYLE DOUBLE D EXTENSION TUBE OR THE NEW STYLE DOUBLE D EXTENSION TUBE. THIS CAN BE DONE BY REVIEWING THE HANGER THE EXTENSION TUBE IS USED ON. IF THE HANGER UTILIZES A SINGLE SET SCREW AT THE BOTTOM OF THE HANGER TUBE TO ATTACH THE EXTENSION TUBE TO THE HANGER, THEN THIS UTILIZES THE NEW STYLE DOUBLE D EXT TUBE ALL OTHER HANGERS REQUIRING A DOUBLE D EXTENSION TUBE UTILIZE THE OLD STYLE.

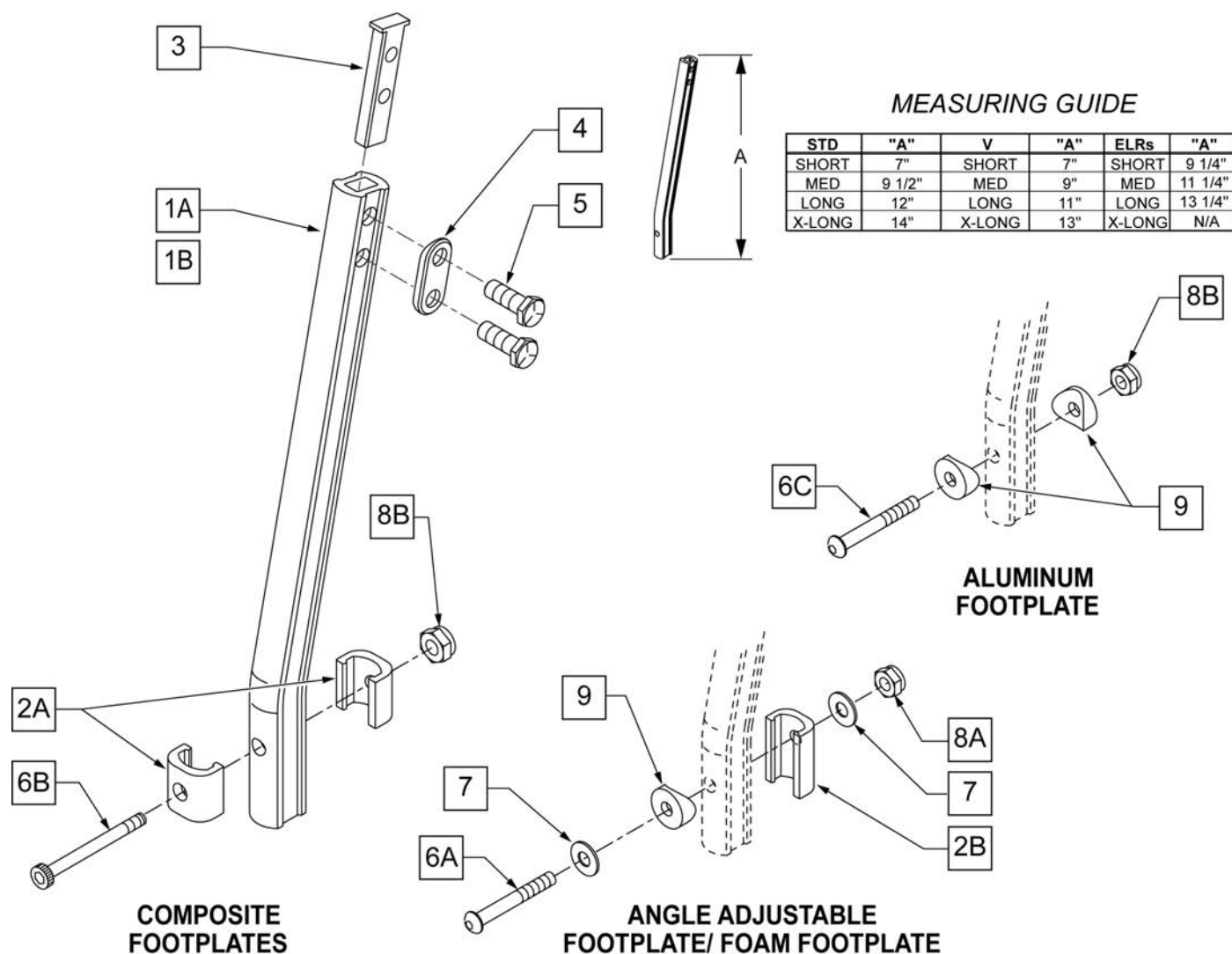

Pos.	Item Number	Description	Remarks
1A	550819	EXT TUBE S/A ONLY SHT	Short Ext-Swing Away Hanger
1A	550820	EXT TUBE S/A ONLY MED	Med Ext-Swing Away Hanger
1A	550821	EXT TUBE S/A ONLY LONG	Long Ext-Swing Away Hanger
1A	550822	EXT TUBE S/A ONLY X-LONG	X-Long Ext-Swing Away Hanger
1B	550834	EXT TUBE V IN ONLY D2 SHT R	Right Short Ext-Swing Away Hanger
1B	550835	EXT TUBE V IN ONLY D2 SHT L	Left Short Ext-Swing Away Hanger

Pos.	Item Number	Description	Remarks
1B	550836	EXT TUBE V IN ONLY D2 MED R	Right Med Ext-Swing Away Hanger
1B	550837	EXT TUBE V IN ONLY D2 MED L	Left Med Ext-Swing Away Hanger
1B	550838	EXT TUBE V IN ONLY D2 LONG R	Right Long Ext-Swing Away Hanger
1B	550839	EXT TUBE V IN ONLY D2 LONG L	Left Long Ext-Swing Away Hanger
1B	550840	EXT TUBE V IN ONLY D2 X-LONG R	Right X-Long Ext-Swing Away Hanger
1B	550841	EXT TUBE V IN ONLY D2 X-LONG L	Left X-Long Ext-Swing Away Hanger
2A	483718	HANGER EXTENSION SADDLE	USE W/ COMPOSITE FTPLT
2B	483720	HANGER EXTENSION SADDLE LONG	USE W/ ANGLE ADJ FTPLT
1C	550764	S/A HANGER EXT D2 SHORT BLACK	Short Ext ELR Hangers
1C	550765	S/A HANGER EXT D2 MED BLACK	Medium Ext ELR Hangers
1C	550766	S/A HANGER EXT D2 LONG BLACK	Long Ext ELR Hangers
3	500078	INSERT D2 EXT TUBE	
4	483717	HANGER SADDLE	
5	221102	1/4-28 x 3/4 HEX HEAD C/S BLK W/PTCH	
6A	106962	1/4-28 x 1 3/4 BH C/S BLK ZN	USE W/ ANGLE ADJ FTPLT
6B	226165	1/4 -28 x 2 1/2 SOC C/S	USE W/ COMPOSITE FTPLT
6C	226147	1/4 -28 X 1.85 BH C/S	
7	106945	1/4" SAE WASHER	
8A	106941	1/4 -28 AUTOMATION NUT	USE W/ ANGLE ADJ FTPLT
8B	106935	1/4 -28 NYLOCK NUT THIN	USE W/ COMPOSITE FTPLT
9	561055	3/4 SADDLE 1" OD	USE W/ ANGLE ADJ FTPLT
1A,2A-5	921759	EXT TUBE S/A SHT ASSM PCKT	USE W/ COMPOSITE FTPLT
1A,2A-5	921760	EXT TUBE S/A MED ASSM PCKT	USE W/ COMPOSITE FTPLT
1A,2A-5	921761	EXT TUBE S/A LONG ASSM PCKT	USE W/ COMPOSITE FTPLT
1A,2A-5	921762	EXT TUBE S/A X-LONG ASSM PCKT	USE W/ COMPOSITE FTPLT
1B,2A-5	921751	EXT TUBE "V" D2 SHT R ASSM PCKT	RIGHT - USE W/ COMPOSITE FTPLT V HANGER
1B,2A-5	921752	EXT TUBE "V" D2 SHT L ASSM PCKT	LEFT - USE W/ COMPOSITE FTPLT V HANGER
1B,2A-5	921753	EXT TUBE "V" D2 MED R ASSM PCKT	RIGHT - USE W/ COMPOSITE FTPLT V HANGER
1B,2A-5	921754	EXT TUBE "V" D2 MED L ASSM PCKT	LEFT - USE W/ COMPOSITE FTPLT V HANGER
1B,2A-5	921755	EXT TUBE "V" D2 LONG R ASSM PCKT	RIGHT - USE W/ COMPOSITE FTPLT V HANGER
1B,2A-5	921756	EXT TUBE "V" D2 LONG L ASSM PCKT	LEFT - USE W/ COMPOSITE FTPLT V HANGER
1B,2A-5	921757	EXT TUBE "V" D2 X-LONG R ASSM PCKT	RIGHT - USE W/ COMPOSITE FTPLT V HANGER
1B,2A-5	921758	EXT TUBE "V" D2 X-LONG L ASSM PCKT	LEFT - USE W/ COMPOSITE FTPLT V HANGER
1A,2A-7,9	921772	EXT TUBE S/A SHT ANG ADJ PCKT	USE W/ ANGLE ADJ FTPLT
1A,2A-7,9	921773	EXT TUBE S/A MED ANG ADJ PCKT	USE W/ ANGLE ADJ FTPLT
1A,2A-7,9	921774	EXT TUBE S/A LONG ANG ADJ PCKT	USE W/ ANGLE ADJ FTPLT
1A,2A-7,9	921775	EXT TUBE S/A X-LONG ANG ADJ PCKT	USE W/ ANGLE ADJ FTPLT
1B,2A-7,9	921764	EXT TUBE "V" D2 SHT R ANG ADJ PCKT	RIGHT - USE W/ ANGLE ADJ FTPLT V HANGER
1B,2A-7,9	921765	EXT TUBE "V" D2 SHT L ANG ADJ PCKT	LEFT - USE W/ ANGLE ADJ FTPLT V HANGER
1B,2A-7,9	921766	EXT TUBE "V" D2 MED R ANG ADJ PCKT	RIGHT - USE W/ ANGLE ADJ FTPLT V HANGER
1B,2A-7,9	921767	EXT TUBE "V" D2 MED L ANG ADJ PCKT	LEFT - USE W/ ANGLE ADJ FTPLT V HANGER
1B,2A-7,9	921768	EXT TUBE "V" D2 LONG R ANG ADJ PCKT	RIGHT - USE W/ ANGLE ADJ FTPLT V HANGER
1B,2A-7,9	921769	EXT TUBE "V" D2 LONG L ANG ADJ PCKT	LEFT - USE W/ ANGLE ADJ FTPLT V HANGER
1B,2A-7,9	921770	EXT TUBE "V" D2 X-LONG R ANG AD PCKT	RIGHT - USE W/ ANGLE ADJ FTPLT V HANGER
1B,2A-7,9	921771	EXT TUBE "V" D2 X-LONG L ANG AD PCKT	LEFT - USE W/ ANGLE ADJ FTPLT V HANGER

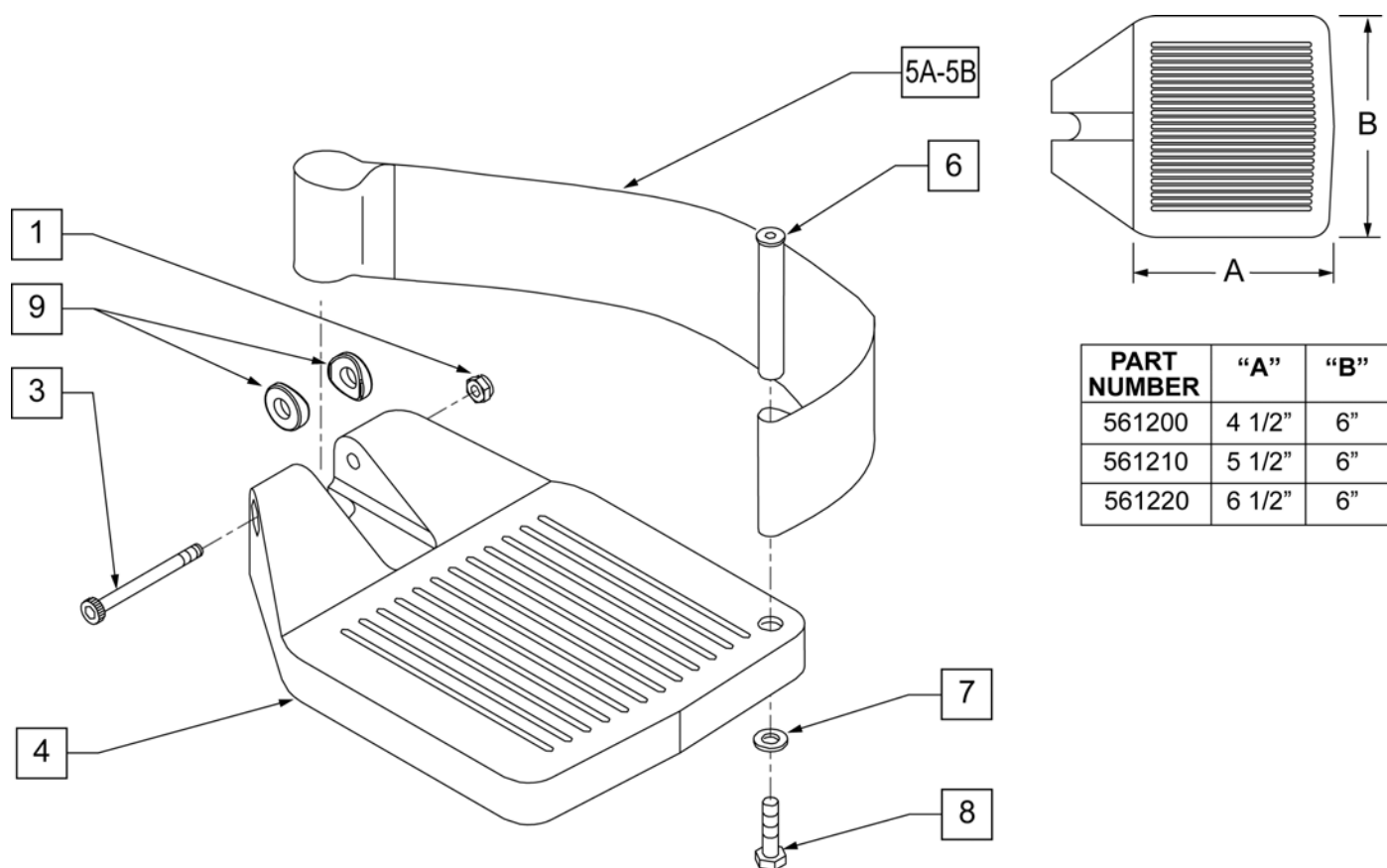

Pos.	Item Number	Description	Remarks
1	106935	1/4 - 28 NYLOCK NUT	
3	226165	1/4 - 28 x 2 1/2 SOC C/S	
4	561200	COMPOSITE FOOTPLATE 14 -15 (16 -18 V)	14 -15 OR 16 -18V
4	561210	COMPOSITE FOOTPLATE 16 -17 (19 -20 V)	16 -17 OR 19 -20V
4	561220	COMPOSITE FOOTPLATE 18 -20	18 -20
5A	741151	HEEL LOOP 14-15 COMPOSITE (14 -15 or 16 -18V)	14 -15 OR 16 -18V
5A	741156	HEEL LOOP 16 -17 or 19 -20V	16 -17 OR 19 -20V
5A	741153	HEEL LOOP 18 -20 COMPOSITE/14-15 ANG ADJ	18 -20
5A	902270	HEEL LOOP COMPOSITE FT 14-15	PAIR 14 -15 OR 16 -18V
5A	902271	HEEL LOOP COMPOSITE FT 16-17	PAIR 16 -17 OR 19 -20V
5A	902272	HEEL LOOP COMPOSITE FT 18-20	PAIR 18 -20
5B	741154	HEEL LOOP 14-15 COMPOSITE HEMI	14 -15 HEMI
5B	741155	HEEL LOOP 16-17 COMPOSITE HEMI	16 -17 HEMI
5B	741153	HEEL LOOP 18-20 COMPOSITE HEMI	18 -10 HEMI
5B	902260	HEEL LOOPS HEMI COMPOSITE 14-15	PAIR 14 -15 HEMI
5B	902261	HEEL LOOPS HEMI COMPOSITE 16-17	PAIR 16 -17 HEMI
5B	902262	HEEL LOOPS HEMI COMPOSITE 18-20	PAIR 18 -20 HEMI
6	561040	HEEL LOOP PEG	
7	106944	1/4 AN WASHER FLAT THIN	
8	106958	HHCS 1/4-20 X 1 ROHS	
9	561003	3/4" SADDLE THIN	USED W/ ROUND EXT TUBES ONLY
1,3,4	921180	COMPOSITE FOOTPLATE ASSM 14-15	RIGHT (14 -15 OR 16 -18V) W/O HEEL LOOP
1,3,4	921181	COMPOSITE FOOTPLATE ASSM 14-15	LEFT (14 -15 OR 16 -18V) W/O HEEL LOOP
1,3,4	921182	COMPOSITE FOOTPLATE ASSM 16-17	RIGHT (16 -17 OR 19 -20V) W/O HEEL LOOP
1,3,4	921183	COMPOSITE FOOTPLATE ASSM 16-17	LEFT (16 -17 OR 19 -20V) W/O HEEL LOOP

Pos.	Item Number	Description	Remarks
1,3,4	921184	COMPOSITE FOOTPLATE ASSM 18-20	RIGHT (18 -20) W/O HEEL LOOP
1,3,4	921185	COMPOSITE FOOTPLATE ASSM 18-20	LEFT (18 -20) W/O HEEL LOOP
1,3,4	921151	COMPOSITE FOOTPLATE ASSM 14-15/16-18V	PAIR (14 -15 OR 16 -18V) W/O HEEL LOOP
1,3,4	921152	COMPOSITE FOOTPLATE ASSM 16-17/19-20V	PAIR (16 -17 OR 19 -20V) W/O HEEL LOOP
1,3,4	921153	COMPOSITE FOOTPLATE ASSM 18-20	PAIR (18 -20) W/O HEEL LOOP
1,3,4A-8	921190	COMPOSITE FOOTPLATE ASSM HL LP 14-15	RIGHT (14 -15 OR 16 -18V) W/ HEEL LOOP
1,3,4A-8	921191	COMPOSITE FOOTPLATE ASSM HL LP 14-15	LEFT (14 -15 OR 16 -18V) W/ HEEL LOOP
1,3,4A-8	921192	COMPOSITE FOOTPLATE ASSM HL LP 16-17	RIGHT (16 -17 OR 19 -20V) W/ HEEL LOOP
1,3,4A-8	921193	COMPOSITE FOOTPLATE ASSM HL LP 16-17	LEFT (16 -17 OR 19 -20V) W/ HEEL LOOP
1,3,4A-8	921194	COMPOSITE FOOTPLATE ASSM HL LP 18-20	RIGHT (18 -20) W/ HEEL LOOP
1,3,4A-8	921195	COMPOSITE FOOTPLATE ASSM HL LP 18-20	LEFT (18 -20) W/ HEEL LOOP
1,3,4A-8	921161	COMPOSITE FOOTPLATE ASSM HL 14-15/16-18V	PAIR (14 -15 OR 16 -18V) W/ HEEL LOOP
1,3,4A-8	921162	COMPOSITE FOOTPLATE ASSM HL 16-17/19-20V	PAIR (16 -17 OR 19 -20V) W/ HEEL LOOP
1,3,4A-8	921163	COMPOSITE FOOTPLATE ASSM HL 18-20	PAIR (18 -20) W/ HEEL LOOP
1,3,4B-8	921140	COMPOSITE FOOTPLATE ASSM HEMI HL LP 14-15	RIGHT (14 -15) W/ HEEL LOOP
1,3,4B-8	921141	COMPOSITE FOOTPLATE ASSM HEMI HL LP 14-15	LEFT (14 -15) W/ HEEL LOOP
1,3,4B-8	921142	COMPOSITE FOOTPLATE ASSM HEMI HL LP 16-17	RIGHT (16 -17) W/ HEEL LOOP
1,3,4B-8	921143	COMPOSITE FOOTPLATE ASSM HEMI HL LP 16-17	LEFT (16 -17) W/ HEEL LOOP
1,3,4B-8	921144	COMPOSITE FOOTPLATE ASSM HEMI HL LP 18-20	RIGHT (18 -20) W/ HEEL LOOP
1,3,4B-8	921145	COMPOSITE FOOTPLATE ASSM HEMI HL LP 18-20	LEFT (18 -20) W/ HEEL LOOP
1,3,4B-8	921167	COMPOSITE FOOTPLATE ASSM HEMI HL LPS 14-15	PAIR (14 -15) W/ HEEL LOOP
1,3,4B-8	921168	COMPOSITE FOOTPLATE ASSM HEMI HL LPS 16-17	PAIR (16 -17) W/ HEEL LOOP
1,3,4B-8	921169	COMPOSITE FOOTPLATE ASSM HEMI HL LPS 18-20	PAIR (18 -20) W/ HEEL LOOP


NOTE: FOR HIGH-MOUNT ADULT ANGLE-ADJUSTABLE FOOTPLATE, SEE "HIGH-MOUNT EXTENSION TUBE" PAGE.

NOTE: ORDER ITEM 4B FOR CHAIRS PRIOR TO 8/13/01.

NOTE: ONLY 1 WASHER PRIOR TO 8/13/01.


NOTE: ORDER PART 226167 PRIOR TO 8/13/01.

NOTE: See attached REFERENCE GUIDE.

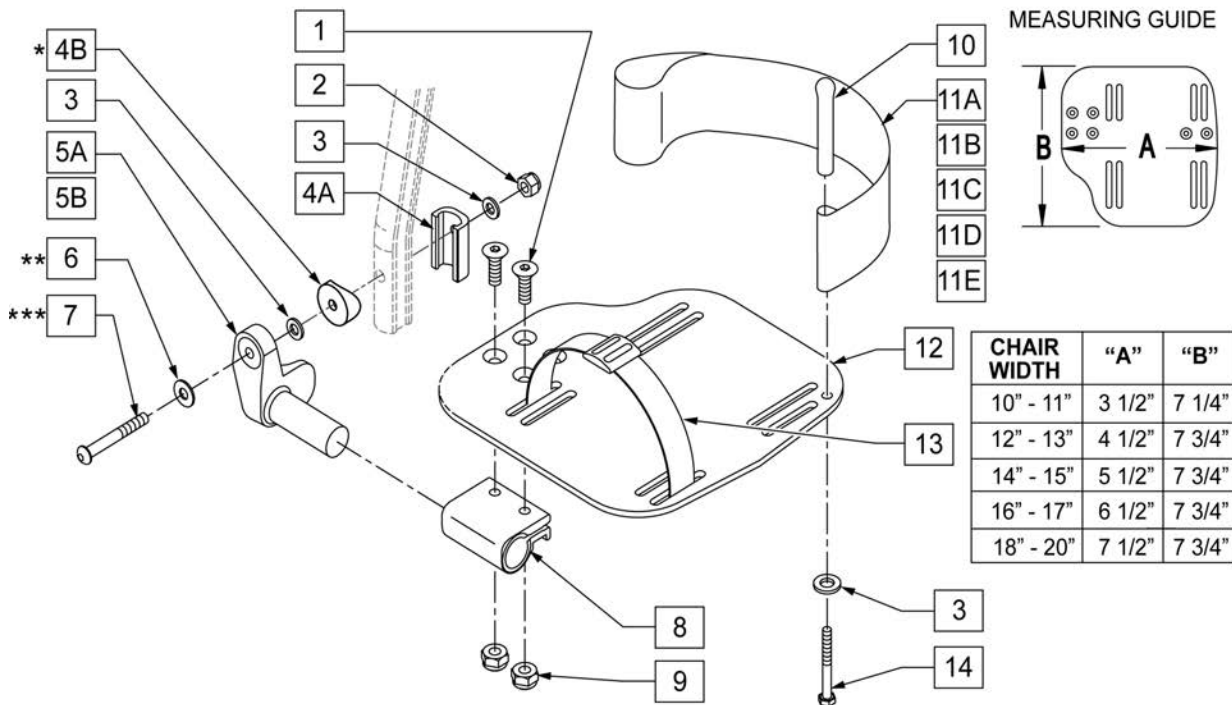

Pos.	Item Number	Description	Remarks
1	106978	1/4-28 x 7/8 FH C/S	
2	106941	1/4-28 AUTOMATION NUT	


Pos.	Item Number	Description	Remarks
3	106943	WASHER FLAT AN 1/4 ROHS	
4A	483720	HANGER EXTENSION SADDLE LONG	USE W/ DOUBLE D EXT TUBE
4B	561055	3/4" SADDLE 1" OD	USE W/ ROUND EXT TUBE
5	541002	FTRST MOUNTING BRKT-3/8 RAD R	RIGHT
5	541003	FTRST MOUNTING BRKT-3/8 RAD L	LEFT
6	106945	WASHER FLAT SAE 1/4 ROHS	
7A	106962	BHCS 1/4-28 X 1-3/4 ROHS	USE W/ DOUBLE D EXT TUBE
7B	226167	1/4-28 x 1.625 BH C/S BLK	USE W/ ROUND EXT TUBE
8	550350	3/4" CLMP ADJ FT PLT	
9	106935	NUT NYLOCK THIN 1/4-28 ROHS	
10	561040	HEEL LOOP PEG	
11A	741154	HEEL LOOP 14-15 CMPST HEMI	USED W/ 10-11" ADULT ANG ADJ FTPLTS
11B	741155	HEEL LOOP 16-17 CMPST HEMI	USED W/ 12 -13" ADULT ANG ADJ FTPLTS
11C	741153	H/L 18-20 CMPST/14-15 ANG ADJ	USED W/ 14-15"/16-17" ADULT ANG ADJ FTPLTS
11D	741253	HEEL LOOP ADLT ADJ F/PLT 18-20	USED W/ 18 -20" ADULT ANG ADJ FTPLTS
12	500656	ADLT ANGL ADJ F/PLT 10/11" R	RIGHT
12	500657	ADLT ANGL ADJ F/PLT 10/11" L	LEFT
12	500658	ADLT ANGL ADJ F/PLT 12/13" R	RIGHT
12	500659	ADLT ANGL ADJ F/PLT 12/13" L	LEFT
12	500660	ADLT ANGL ADJ F/PLT 14/15	RIGHT
12	500661	ADLT ANGL ADJ F/PLT 14/15	LEFT
12	500662	ADLT ANGL ADJ F/PLT 16/17	RIGHT
12	500663	ADLT ANGL ADJ F/PLT 16/17	LEFT
12	500664	ADLT ANGL ADJ F/PLT 18-20	RIGHT
12	500665	ADLT ANGL ADJ F/PLT 18-20	LEFT
13	740260	TOE/ANKLE STRAP ADJ ADLT F/PLT	
14	106958	1/4-20 x 1 HEX C/S	
1-3,5,6,8,9,12	820526	AD ANG ADJ F/PLT R 12"/13" OMIT HL	RIGHT
1-3,5,6,8,9,12	820536	AD ANG ADJ F/PLT L 12"/13" OMT HL	LEFT
1-3,5,6,8,9,12	820460	FP ADLT ADJ 14/15 OMIT HL	RIGHT
1-3,5,6,8,9,12	820462	FP ADLT ADJ 14/15 OMIT HL	LEFT
1-3,5,6,8,9,12	820464	FP ADLT ADJ 16/17 OMIT HL	RIGHT
1-3,5,6,8,9,12	820466	FP ADLT ADJ 16/17 OMIT HL	LEFT
1-3,5,6,8,9,12	820468	FP ADLT ADJ 18-20 OMIT HL	RIGHT
1-3,5,6,8,9,12	820470	FP ADLT ADJ 18-20 OMIT HL	LEFT
1-3,5,6,8,9,12	921466	ADLT ANGL ADJ F/PLT 12"/13" OMIT HL	PAIR
1-3,5,6,8,9,12	921320	FP ADLT ADJ 14/15 OMIT HL	PAIR
1-3,5,6,8,9,12	921322	FP ADLT ADJ 16/17 OMIT HL	PAIR
1-3,5,6,8,9,12	921324	FP ADLT ADJ 18-20 OMIT HL	PAIR
1-3,5,6,8,9,12	820522	AD ANGL ADJ F/PLT R 12/13" H/L	RIGHT
1-3,5,6,8,9,12	820532	AD ANGL ADJ F/PLT L 12/13" H/L	LEFT
1-3,5,6,8-10,11C, 12,14	820461	FP ADLT ADJ 14/15 HL	RIGHT
1-3,5,6,8-10,11C, 12,14	820463	FP ADLT ADJ 14/15 HL	LEFT
1-3,5,6,8-10,11C, 12,14	820465	FP ADLT ADJ 16/17 HL	RIGHT
1-3,5,6,8-10,11C, 12,14	820467	FP ADLT ADJ 16/17 HL	LEFT
1-3,5,6,8-10,11D, 12,14	820469	FP ADLT ADJ 18-20 HL	RIGHT
1-3,5,6,8-10,11D, 12,14	820471	FP ADLT ADJ 18-20 HL	LEFT
1-3,5,6,8-10,11B, 12,14	921462	ADLT ANGL ADJ F/PLATE 12"/13" HL	PAIR
1-3,5,6,8-10,11C, 12,14	921321	FP ADLT ADJ 14/15 HL	PAIR
1-3,5,6,8-10,11C, 12,14	921323	FP ADLT ADJ 16/17 HL	PAIR
1-3,5,6,8-10,11D, 12,14	921325	FP ADLT ADJ 18-20 HL	PAIR
1-3,6,7A,9-10,14	101533	ADLT ANG ADJ FP D2 HDWR KIT	FOR DOUBLE D EXT TUBES
1-3B,4B,6,7B,9,10,14	101534	ADLT ANG ADJ FP RND HDWR KIT	FOR ROUND EXT TUBES (Q2 HP ONLY)

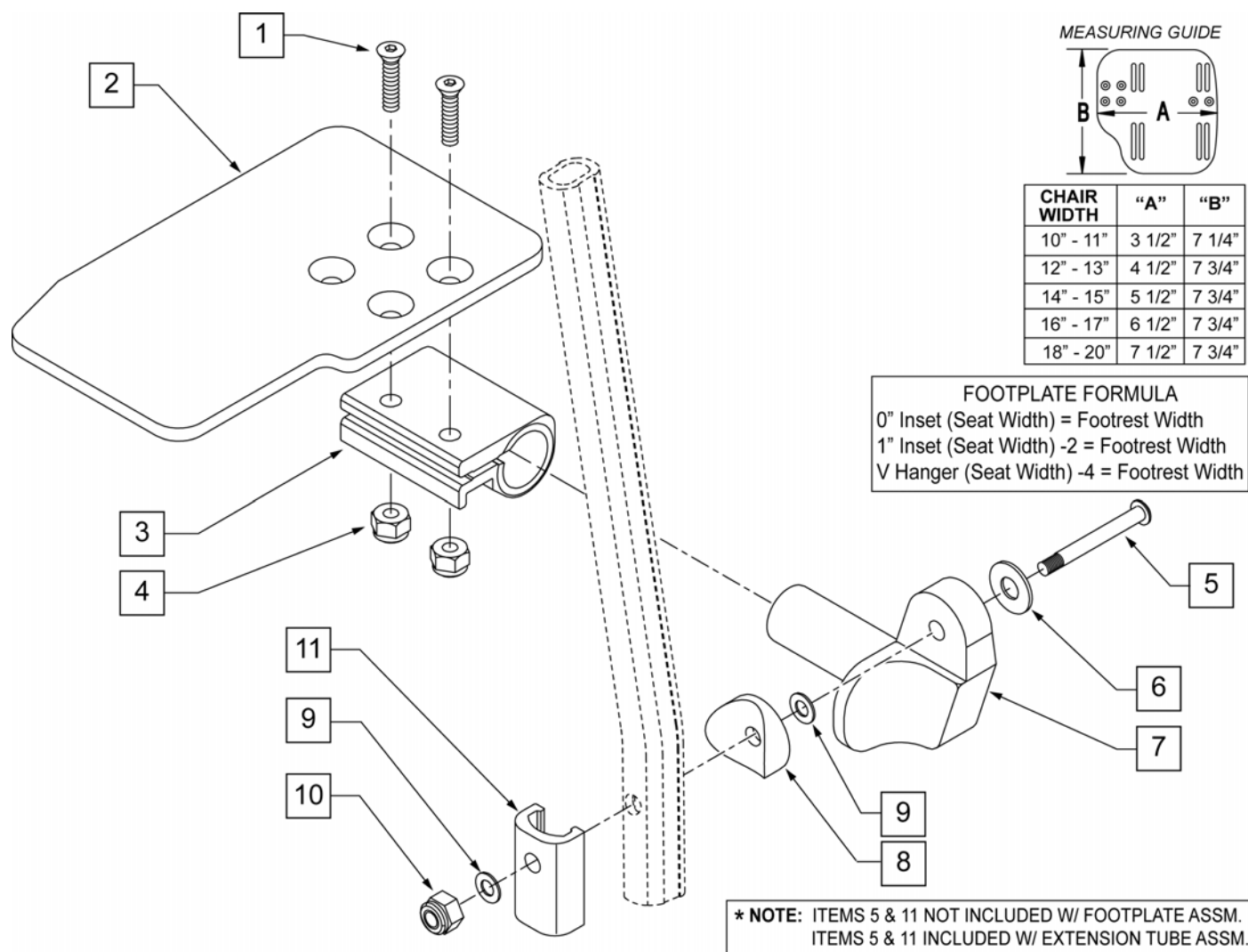

Pos.	Item Number	Description	Remarks
1	106978	FHCS 1/4 -28 x 7/8 ROHS	
2	500635	KIDS ANGL ADJ F/PLT 10" R	Right
2	500636	KIDS ANGL ADJ F/PLT 10" L	Left
2	500637	KIDS ANGL ADJ F/PLT 11" R	Right
2	500638	KIDS ANGL ADJ F/PLT 11" L	Left
2	500639	KIDS ANGL ADJ F/PLT 12" R	Right
2	500640	KIDS ANGL ADJ F/PLT 12" L	Left
2	500641	KIDS ANGL ADJ F/PLT 13" R	Right
2	500642	KIDS ANGL ADJ F/PLT 13" L	Left
2	500643	KIDS ANGL ADJ F/PLT 14" R	Right
2	500644	KIDS ANGL ADJ F/PLT 14" L	Left
2	500645	KIDS ANGL ADJ F/PLT 15" R	Right
2	500646	KIDS ANGL ADJ F/PLT 15" L	Left
2	500647	KIDS ANGL ADJ F/PLT 16" R	Right
2	500648	KIDS ANGL ADJ F/PLT 16" L	Left
3	550350	3/4" CLAMP ADJ FOOTPLATE	
4	106935	NUT NYLOCK THIN 1/4 -28 ROHS	
5	106962	1/4-28 x 1 3/4 BH C/S BLK ZN (D2 EXTENSION TUBES)	


Pos.	Item Number	Description	Remarks
5	226167	1/4-28 x 1.625 BH C/S BLK (ROUND EXTENSION TUBES)	
6	106945	1/4 SAE WASHER	
7	541002	FTRST MOUNTING BRKT-3/8 RAD R	
7	541003	FTRST MOUNTING BRKT-3/8 RAD L	
8	561055	3/4" SADDLE 1" OD	
9	106944	WASHER FLAT THIN AN 1/4 ROHS	
10	106941	NUT AUTOMATION 1/4 -28 ROHS	
1-7,9,10	828850	KIDS ANG ADJ F/PLT EXT-MNT 10"R	Right
1-7,9,10	828851	KIDS ANG ADJ F/PLT EXT-MNT 10"L	Left
1-7,9,10	828852	KIDS ANG ADJ F/PLT EXT-MNT 11"R	Right
1-7,9,10	828853	KIDS ANG ADJ F/PLT EXT-MNT 11"L	Left
1-7,9,10	828854	KIDS ANG ADJ F/PLT EXT-MNT 12"R	Right
1-7,9,10	828855	KIDS ANG ADJ F/PLT EXT-MNT 12"L	Left
1-7,9,10	828856	KIDS ANG ADJ F/PLT EXT-MNT 13"R	Right
1-7,9,10	828857	KIDS ANG ADJ F/PLT EXT-MNT 13"L	Left
1-7,9,10	828858	KIDS ANG ADJ F/PLT EXT-MNT 14"R	Right
1-7,9,10	828859	KIDS ANG ADJ F/PLT EXT-MNT 14"L	Left
1-7,9,10	828860	KIDS ANG ADJ F/PLT EXT-MNT 15"R	Right
1-7,9,10	828861	KIDS ANG ADJ F/PLT EXT-MNT 15"L	Left
1-7,9,10	828862	KIDS ANG ADJ F/PLT EXT-MNT 16"R	Right
1-7,9,10	828863	KIDS ANG ADJ F/PLT EXT-MNT 16"L	Left
1-7,9,10	921280	KIDS ANG ADJ F/PLT EXT-MNT 10"	Pair
1-7,9,10	921281	KIDS ANG ADJ F/PLT EXT-MNT 11"	Pair
1-7,9,10	921282	KIDS ANG ADJ F/PLT EXT-MNT 12"	Pair
1-7,9,10	921283	KIDS ANG ADJ F/PLT EXT-MNT 13"	Pair
1-7,9,10	921284	KIDS ANG ADJ F/PLT EXT-MNT 14"	Pair
1-7,9,10	921285	KIDS ANG ADJ F/PLT EXT-MNT 15"	Pair
1-7,9,10	921286	KIDS ANG ADJ F/PLT EXT-MNT 16"	Pair

NOTE: Left Side Shown

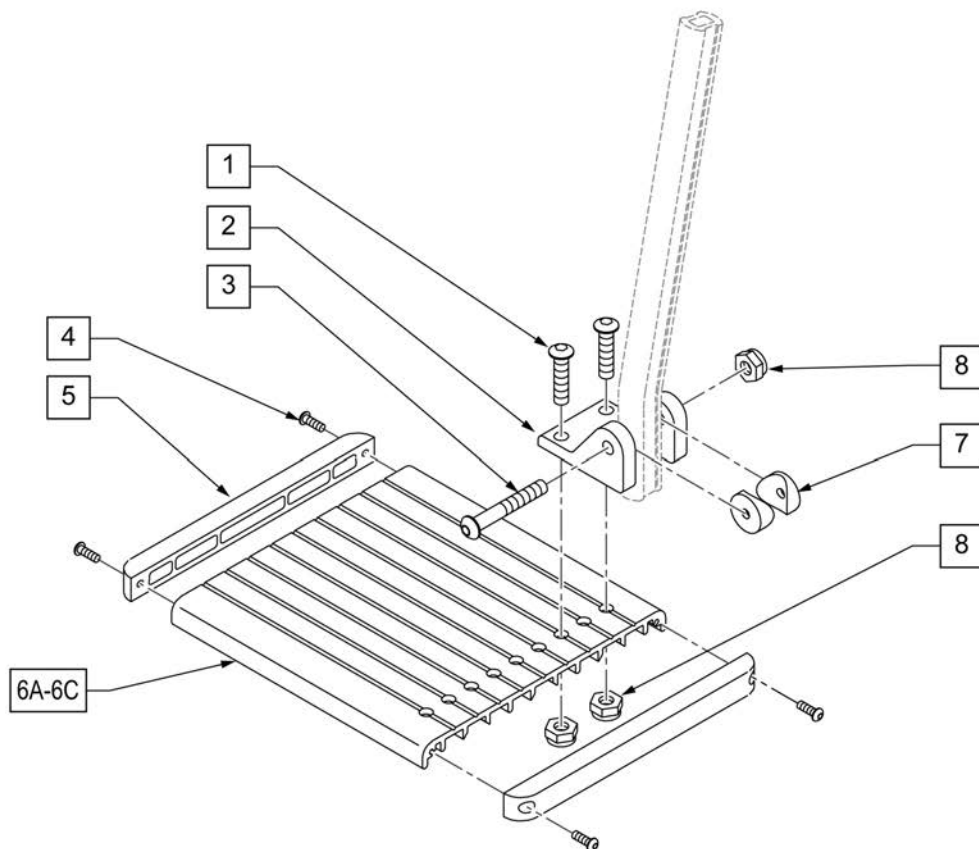


Pos.	Item Number	Description	Remarks
1	106959	1/4-28 x 5/8 BH C/S	
2	550739	FOOTPLATE MOUNTING BRACKET	
3	226147	1/4-28 x 1.85 BH C/S	
4	281043	#4 x 5/8 PAN PHIL S/TAP	
5	560101	END CAPS FOOTREST	
7	561003	3/4" SADDLE THIN	
8	106935	1/4-28 NYLOCK NUT THIN	
4,5,6A	882337	ELR FOOTPLATE ASSEMBLY 14 -15"	
4,5,6B	882338	ELR FOOTPLATE ASSEMBLY 16 -17"	
4,5,6C	882339	ELR FOOTPLATE ASSEMBLY 18 -20"	
1-5,6A-8	921400S	ELR FOOTPLATE KIT 14 -15"	Does not include Extension
1-5,6B-8	921401S	ELR FOOTPLATE KIT 16 -17"	Does not include Extension
1-5,6C-8	921402S	ELR FOOTPLATE KIT 18 -20"	Does not include Extension

NOTE: Order Item 28 (561002) when NOT using Footrest Extension.

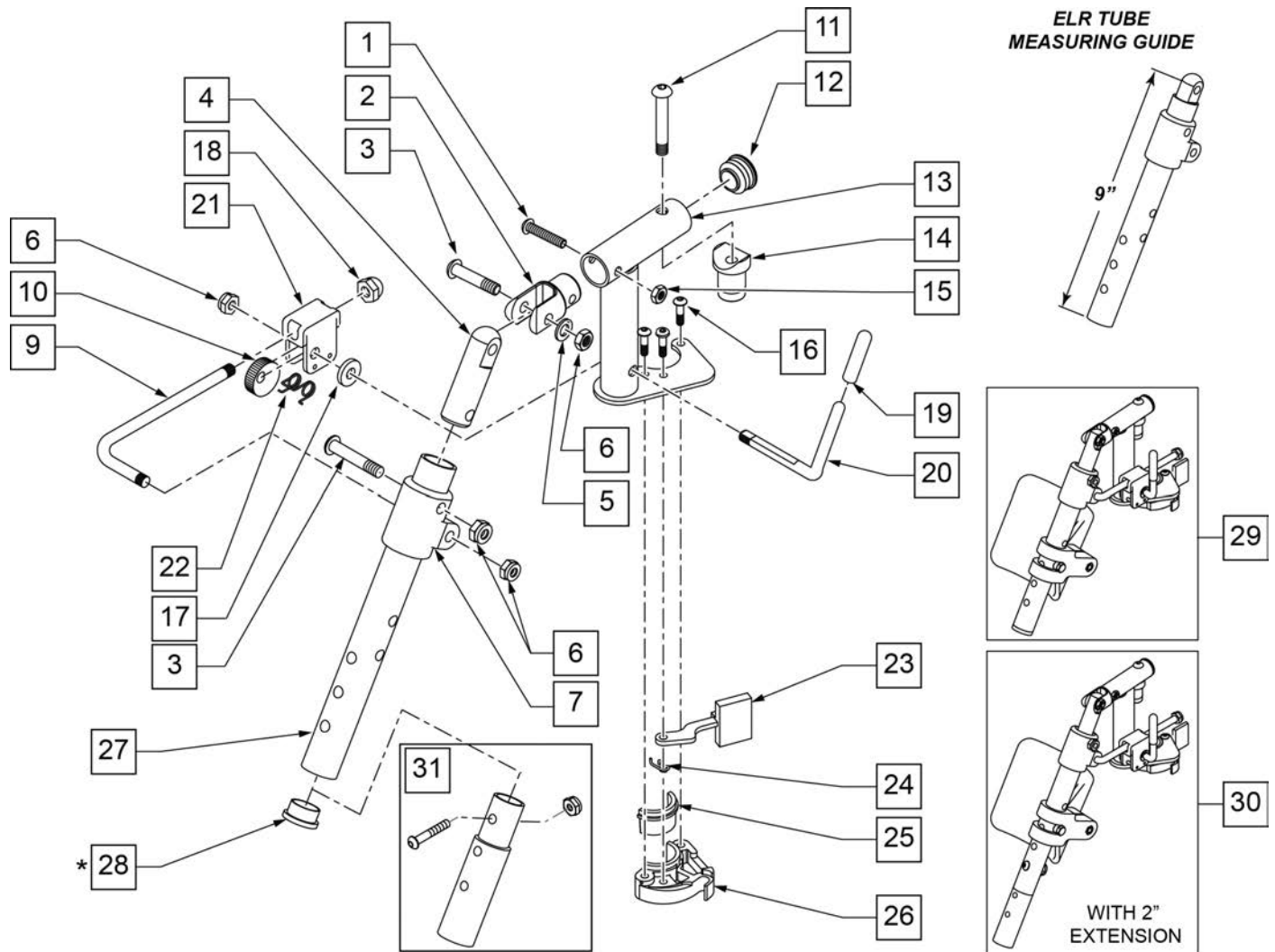


Pos.	Item Number	Description	Remarks
1	106961	BHCS 1/4-28 X 1-1/4 ROHS	
2	505281	ELR CLEVIS	
3	226138	5/16-24 x 1 1/2 BH C/S	
4	540252	ELR PIVOT PRS	
5	263100	5/16 AN WASHER FLAT	
6	106939	5/16-24 NYLOCK NUT THIN	
7	560430	ELR ROD UPPER PIVOT	
9	481505	ELR LOCK ROD 90D	
10	481119	ELR CAM LOCKING POWDERED METAL	
11	226217	5/16-18 x 2" BH C/S BLK	
12	561062	1" INSERT PLUG .065 RIBS	
14	561197	PLASTIC PIVOT SADDLE	
15	106935	NUT NYLOCK THIN 1/4-28 ROHS	
16	226057	10/24 x .725 BTTN HD SOC BZ	
17	561059	3/4 OD x .318 ID WASHER	

Pos.	Item Number	Description	Remarks
18	202109	5/16-24 ACORN NUT HEX	
19	561049	ELR RELEASE ROD COVER	
19,20	882411	ELR 90D ACTUATION ARM ASSM	
21	880990	ELR LOCK HOUSING W/PIN	
22	321106	ELR TORSION SPRING	
23	561079	S/A LATCH LEVER	
24	321136	S/A SPRING L RED	LEFT
24	321137	S/A SPRING R CLEAR	RIGHT
25,26	881500	LATCH BLOCK ASSM R	RIGHT
25,26	881501	LATCH BLOCK ASSM L	LEFT
27	505097S	ELR KIDS TUBE	
28	561002	1" INSERT PLUG .083	
29	923067S	ELR HNGR ASSM 90° W/ PAD	
30,31	923068S	ELR HNGR ASSM 90° W/ PAD 2" EXT	
31	111505	2" FOOTREST EXTENSION TUBE W/ HDWR ASSM	
1-6,11-16,23-26	882413	ELR PIVOT BLOCK ASSM R	RIGHT
1-6,11-16,23-26	882414	ELR PIVOT BLOCK ASSM L	LEFT
1-26	641458S	ELR HNGR 90° W/O PAD PNTD R	RIGHT
1-26	641459S	ELR HNGR 90° W/O PAD PNTD L	LEFT
1-26	641468S	HANGER ELR 90° W/O PAD R 2" EXT	RIGHT
1-26	641469S	HANGER ELR 90° W/O PAD L 2" EXT	LEFT

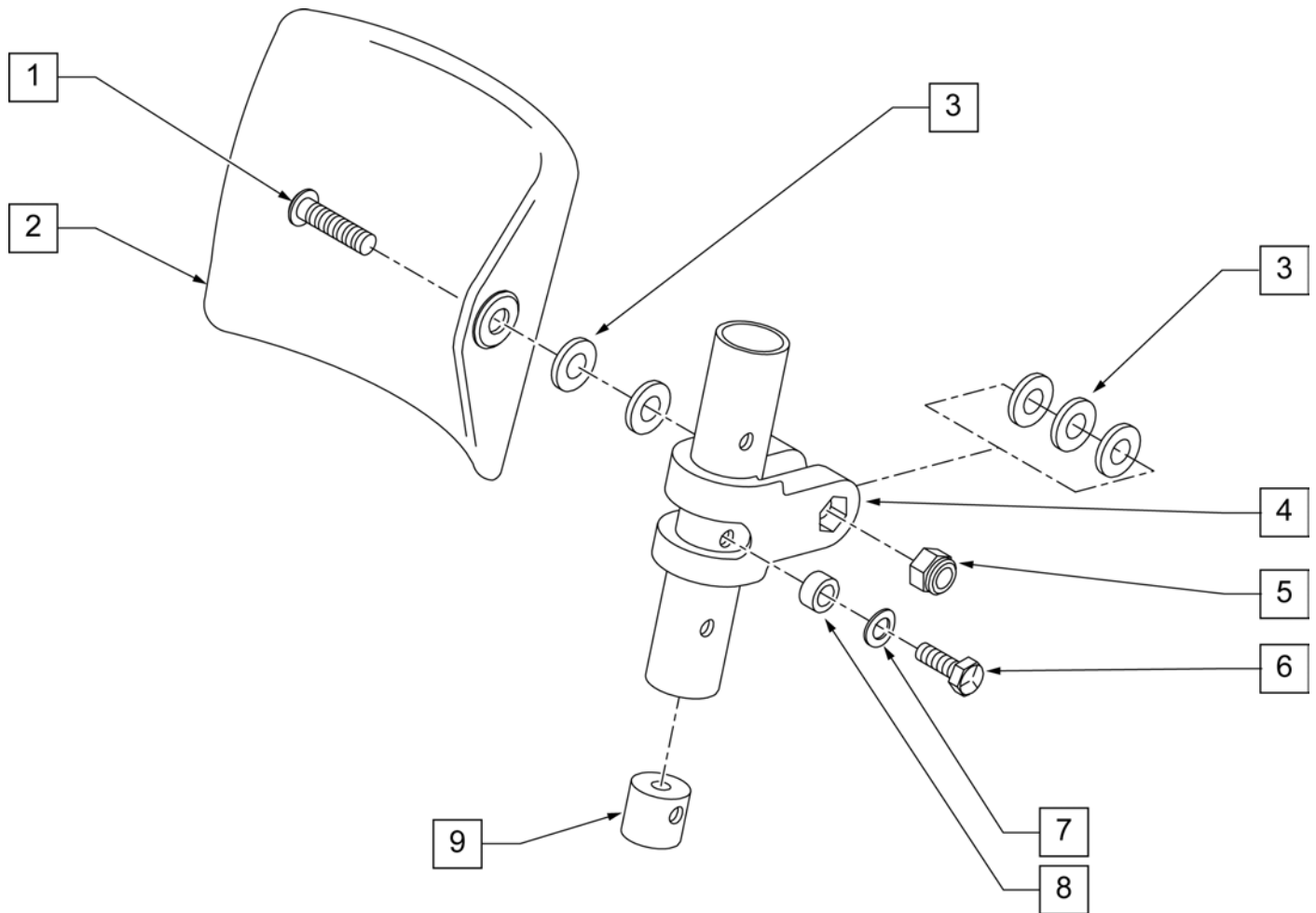


Pos.	Item Number	Description
1	226196	5/16-18 x 2 BH C/S ZN
2	755013	CALF PAD THROUGH STYLE KIDS
3	561016	1" WASHER 5/16" HOLE
4	561097	ELR ELEVATING TUBE CLAMP
5	106937	5/16 -18 NYLOCK NUT THIN
6	107017	1/4-20 x 1 1/4 G8 HEX C/S
7	262130	6mm WASHER WAVE
8	483027	1/4 x 1/4 ALUMINUM SPACER
9	500151	KIDS ALR RETAINING PLUG
1-9	880533	ELR TROUGH PAD ASSM KIDS

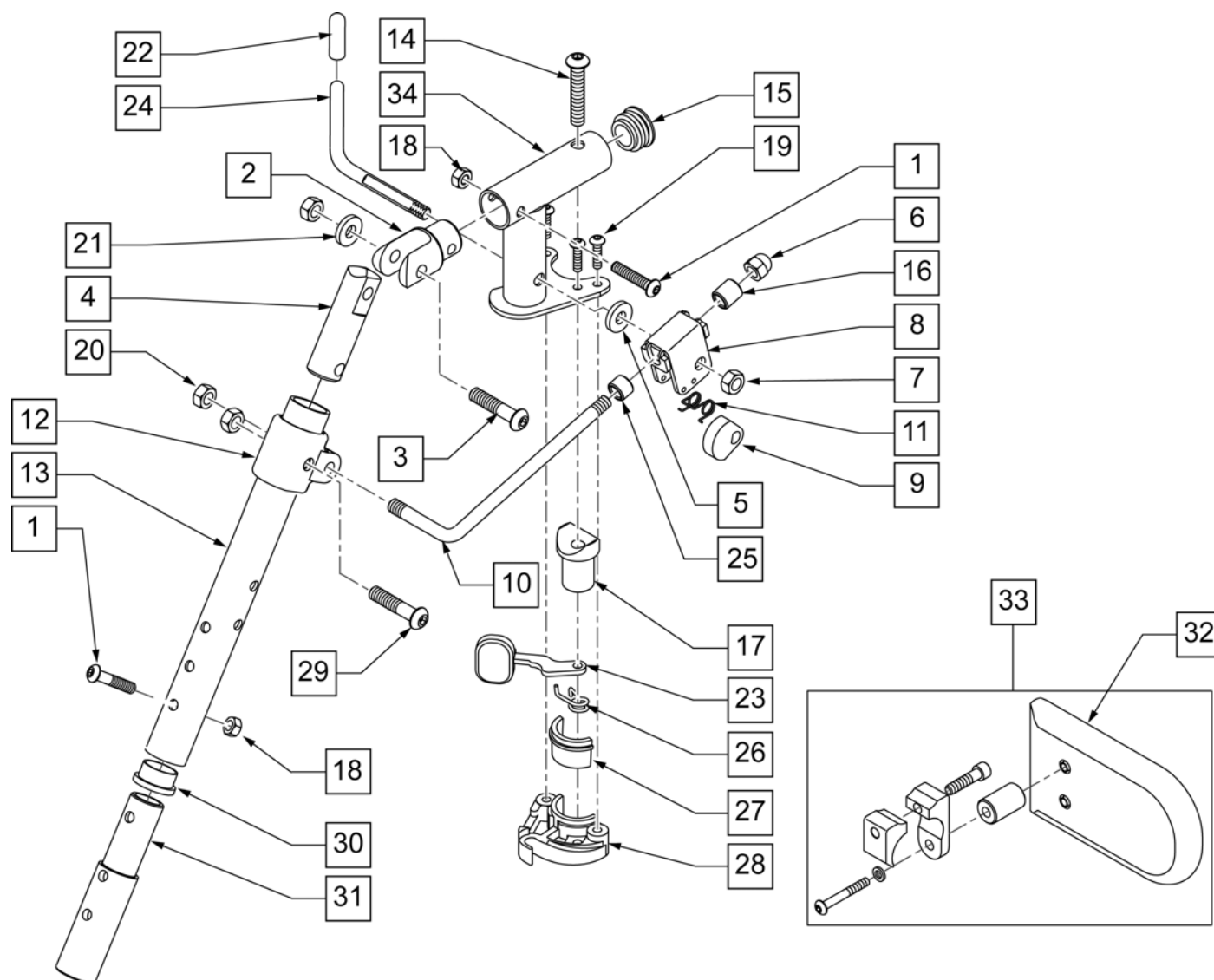


Pos.	Item Number	Description	Remarks
1	106961	1/4-28 x 1 1/4 BH C/S	
2	505281	ELR CLEVIS	
3	226138	5/16-24 x 1 1/2 BH C/S	
4	540252	ELR PIVOT PRS	
5	561059	3/4 OD x .318 ID WASHER	
6	202109	5/16-24 ACORN NUT HEX	
7	106939	5/16-24 NYLOCK NUT THIN	
8	880990	ELR LOCK HOUSING W/PIN	
9	481119	ELR CAM LOCKING POWDERED METAL	
10	481117	ELR LOCK ROD	
11	321106	ELR TORSION SPRING	
12	560420	ELR UPPER PIVOT	
13	505097S	ELR KIDS TUBE	
14	226217	5/16-18 x 2" BH C/S BLK	
15	561062	1" INSERT PLUG .065 RIBS	
16	560815	HEMI ELR ADJ STOP BACK LG	
17	561197	PLASTIC PIVOT SADDLE	


Pos.	Item Number	Description	Remarks
18	106935	NUT NYLOCK THIN 1/4-28 ROHS	
19	226057	10/24 x .725 BTTN HD SOC BZ	
20	106939	5/16-24 NYLOCK NUT THIN	
21	263100	5/16 AN WASHER FLAT	
22	561049	ELR RELEASE ROD COVER	
23	561079	S/A LATCH LEVER	
23	321136	S/A SPRING L RED	
24	481123	ELR ACTUATION ARM 90 DEG KDS	
25	500806	HEMI ELR ADJ STOP FRONT SM	
26	321137	S/A SPRING R CLEAR	
30	561002	1" INSERT PLUG .083	
31	500150	2" FOOTREST EXT TUBE	
29	107037	BHCS 5/16-24 X 1-1/4 SHLD ROHS	
1,18,31	111505	2" FTREST EXT W/ HDWR	
27,28	881500	LATCH BLOCK ASSM R	Right
27,28	881501	LATCH BLOCK ASSM L	Left
1-28,32,33	882881	HEMI ELR SUB ASSM R	Right
1-28,32,33	882882	HEMI ELR SUB ASSM L	Left
32	750226	KNEE PAD FLAT 3H X 5L W/COVER	
33	922328	ELR HEMI KNEE PAD ASSM	
1-30,32,33	119295S	HEMI FM ELR ASSM PNTD R	Right
1-30,32,33	119296S	HEMI FM ELR ASSM PNTD L	Left

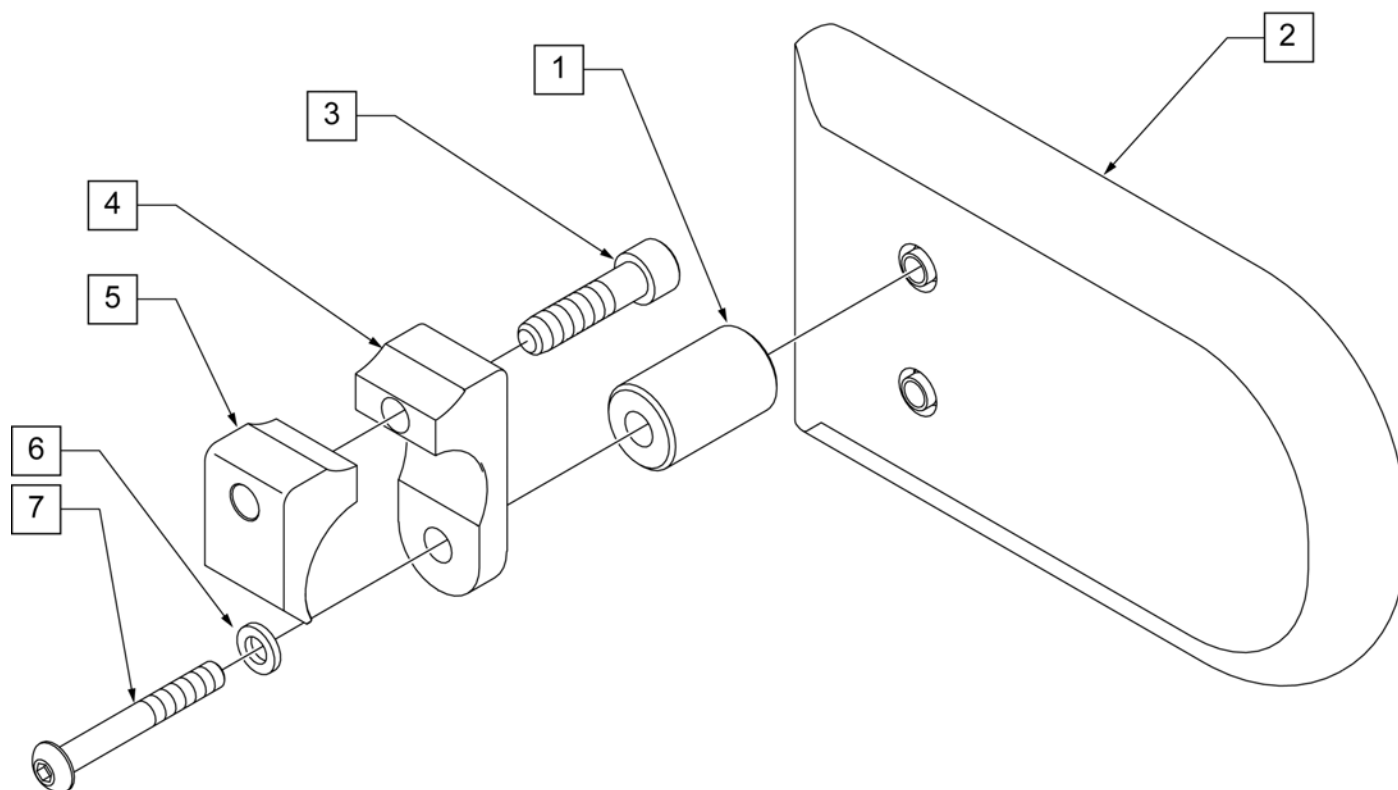


Pos.	Item Number	Description
2	750226	KNEE PAD FLAT 3H X 5L W/COVER
1-7	922328	ELR HEMI KNEE PAD ASSM

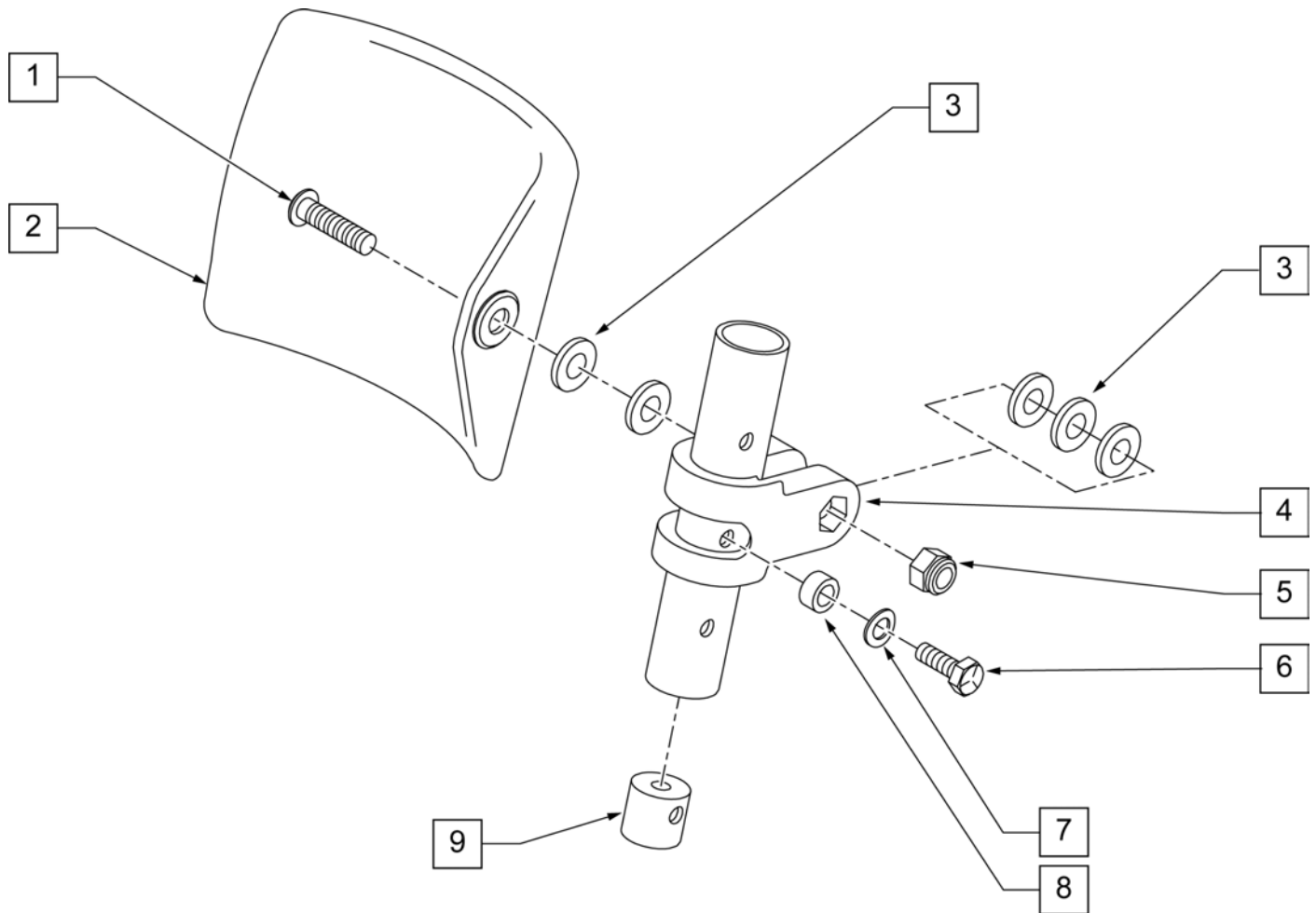

Pos.	Item Number	Description
1	226196	5/16-18 x 2 BH C/S ZN
2	755013	CALF PAD THROUGH STYLE KIDS
3	561016	1" WASHER 5/16" HOLE
4	561097	ELR ELEVATING TUBE CLAMP
5	106937	5/16 -18 NYLOCK NUT THIN
6	107017	1/4-20 x 1 1/4 G8 HEX C/S
7	262130	6mm WASHER WAVE
8	483027	1/4 x 1/4 ALUMINUM SPACER
9	500151	KIDS ALR RETAINING PLUG
1-9	880533	ELR TROUGH PAD ASSM KIDS

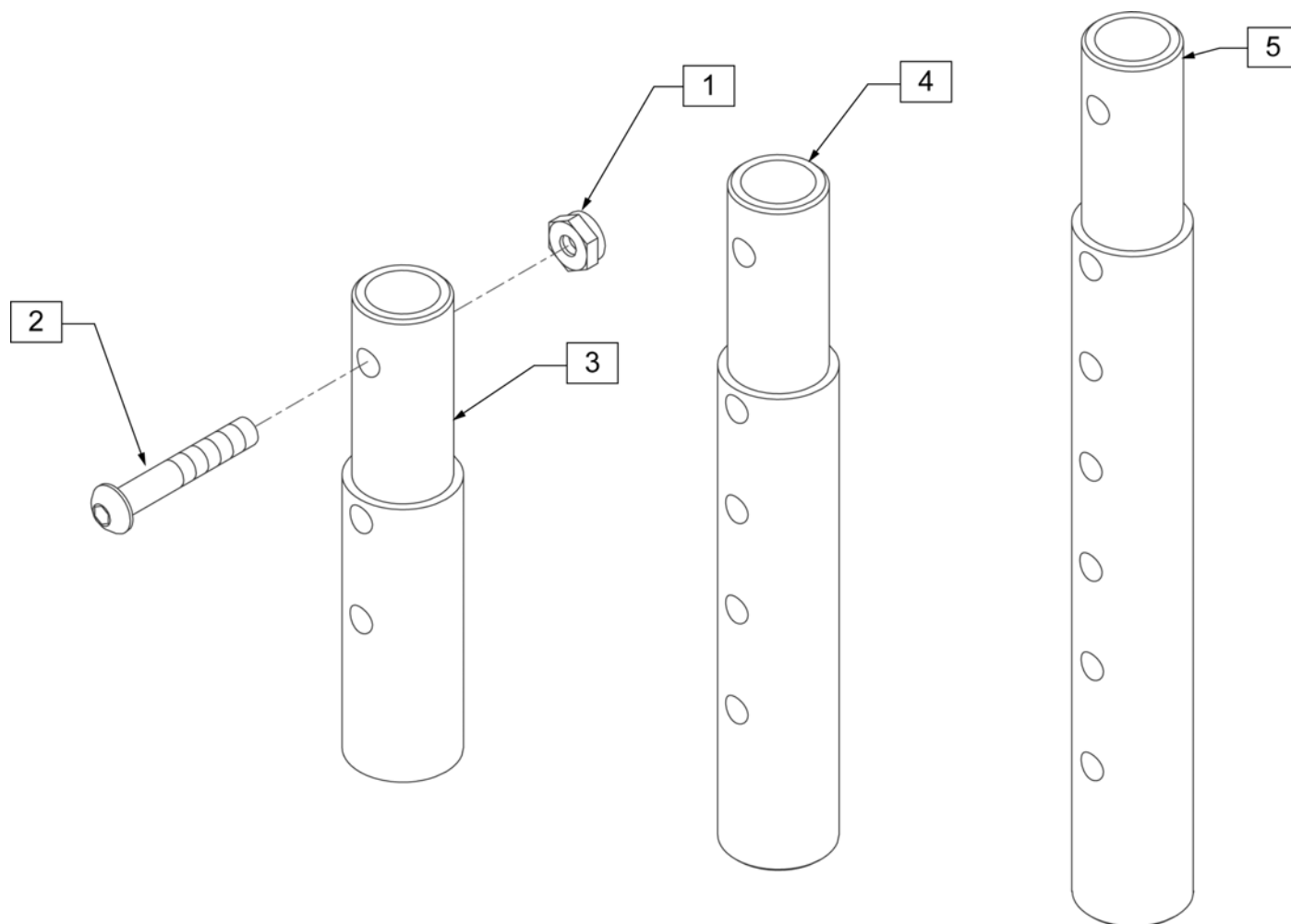


Pos.	Item Number	Description	Remarks
1	106935	NUT NYLOCK THIN 1/4 -28 ROHS	
2	106961	BHCS 1/4 -28 x 1 1/4 ROHS	
1-3	111505	2" FOOTREST EXTENSION TUBE W/ HDWR ASSM	2" EXT (PAIR
1, 2, 4	111506	4" FOOTREST EXTENSION TUBE W/ HDWR ASSM	4" EXT (PAIR)
1, 2, 5	111507	6" FOOTREST EXTENSION TUBE W/ HDWR ASSM	6" EXT (PAIR)

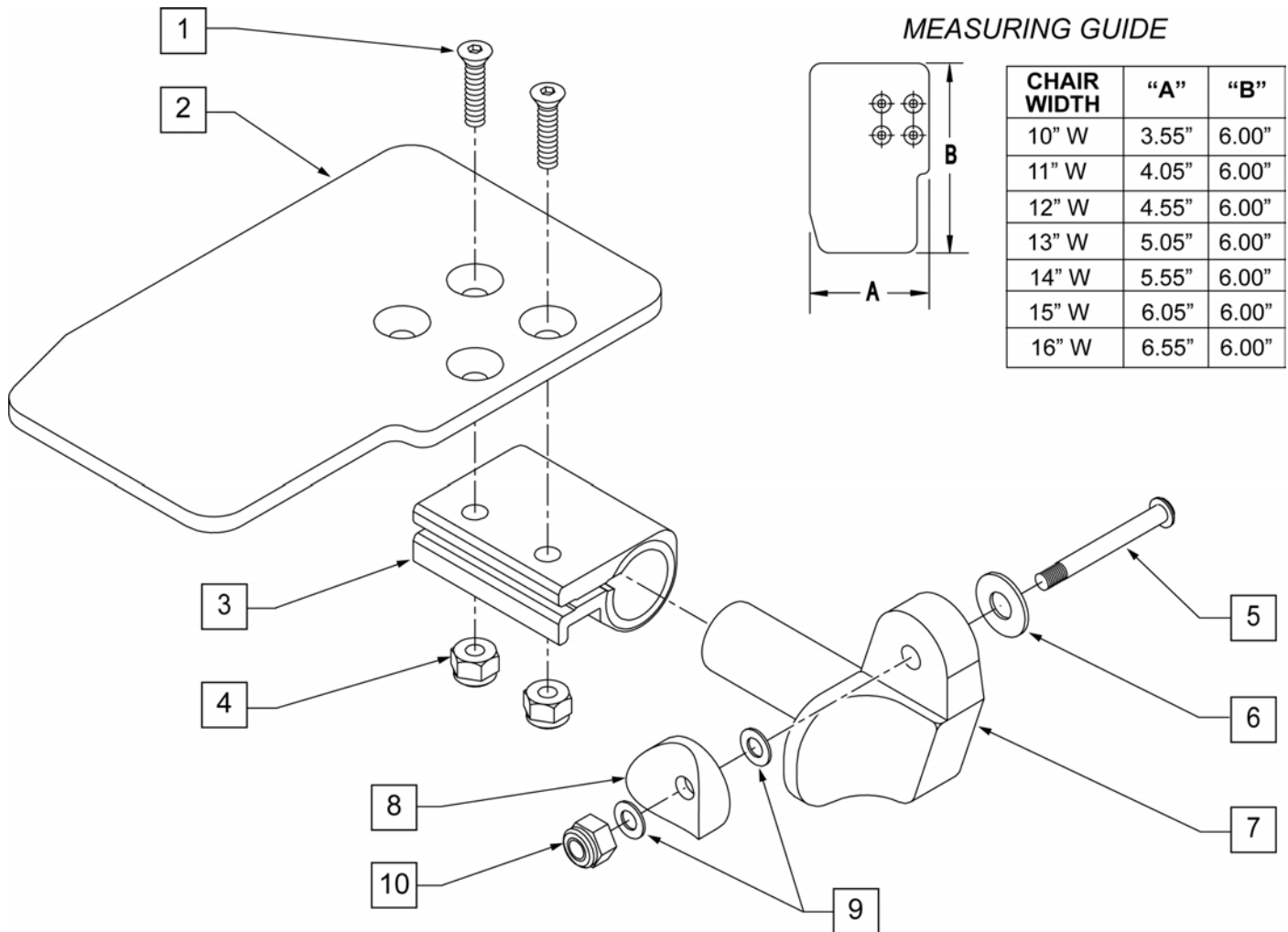

Pos.	Item Number	Description	Remarks	Pos.	Item Number	Description	Remarks
1	106978	FHCS 1/4-28 X 7/8 ROHS		7	541001	FTRST MOUNTING BRKT 1 RAD	LEFT
2	500635	KIDS ANGL ADJ F/PLT 10	RIGHT	8	561004	1 SADDLE 1/4 HOLE	
2	500636	KIDS ANGL ADJ F/PLT 10	LEFT	9	106943	WASHER FLAT THIN AN 1/4	
2	500637	KIDS ANGL ADJ F/PLT 11	RIGHT	10	106941	NUT AUTOMATION 1/4-28 ROHS	
2	500638	KIDS ANGL ADJ F/PLT 11	LEFT	1-10	828821	KIDS ANG ADJ F/PLT ASSM 10	RIGHT
2	500639	KIDS ANGL ADJ F/PLT 12	RIGHT	1-10	828822	KIDS ANG ADJ F/PLT ASSM 10	LEFT
2	500640	KIDS ANGL ADJ F/PLT 12	LEFT	1-10	828823	KIDS ANG ADJ F/PLT ASSM 11	RIGHT
2	500641	KIDS ANGL ADJ F/PLT 13	RIGHT	1-10	828824	KIDS ANG ADJ F/PLT ASSM 11	LEFT
2	500642	KIDS ANGL ADJ F/PLT 13	LEFT	1-10	828825	KIDS ANG ADJ F/PLT ASSM 12	RIGHT
2	500643	KIDS ANGL ADJ F/PLT 14	RIGHT	1-10	828826	KIDS ANG ADJ F/PLT ASSM 12	LEFT
2	500644	KIDS ANGL ADJ F/PLT 14	LEFT	1-10	828827	KIDS ANG ADJ F/PLT ASSM 13	RIGHT
2	500645	KIDS ANGL ADJ F/PLT 15	RIGHT	1-10	828828	KIDS ANG ADJ F/PLT ASSM 13	LEFT
2	500646	KIDS ANGL ADJ F/PLT 15	LEFT	1-10	828829	KIDS ANG ADJ F/PLT ASSM 14	RIGHT
2	500647	KIDS ANGL ADJ F/PLT 16	RIGHT	1-10	828830	KIDS ANG ADJ F/PLT ASSM 14	LEFT
2	500648	KIDS ANGL ADJ F/PLT 16	LEFT	1-10	828831	KIDS ANG ADJ F/PLT ASSM 15	RIGHT
3	550350	3/4 CLAMP ADJ FOOTPLATE		1-10	828832	KIDS ANG ADJ F/PLT ASSM 15	LEFT
4	106935	NUT NYLOCK THIN 1/4-28 ROHS		1-10	828833	KIDS ANG ADJ F/PLT ASSM 16	RIGHT
5	226147	1/4-28 X 1.85 BH C/S BLK ZN		1-10	828834	KIDS ANG ADJ F/PLT ASSM 16	LEFT
6	106945	WASHER FLAT SAE 1/4 ROHS		1-10	921226	KIDS ANG ADJ F/PLT ASSM 10	PAIR
7	541000	FTRST MOUNTING BRKT 1 RAD	RIGHT	1-10	921227	KIDS ANG ADJ F/PLT ASSM 11	PAIR


Pos.	Item Number	Description	Remarks	Pos.	Item Number	Description	Remarks
1-10	921228	KIDS ANG ADJ F/PLT ASSM 12	PAIR	1-10	921231	KIDS ANG ADJ F/PLT ASSM 15	PAIR
1-10	921229	KIDS ANG ADJ F/PLT ASSM 13	PAIR	1-10	921232	KIDS ANG ADJ F/PLT ASSM 16	PAIR
1-10	921230	KIDS ANG ADJ F/PLT ASSM 14	PAIR				

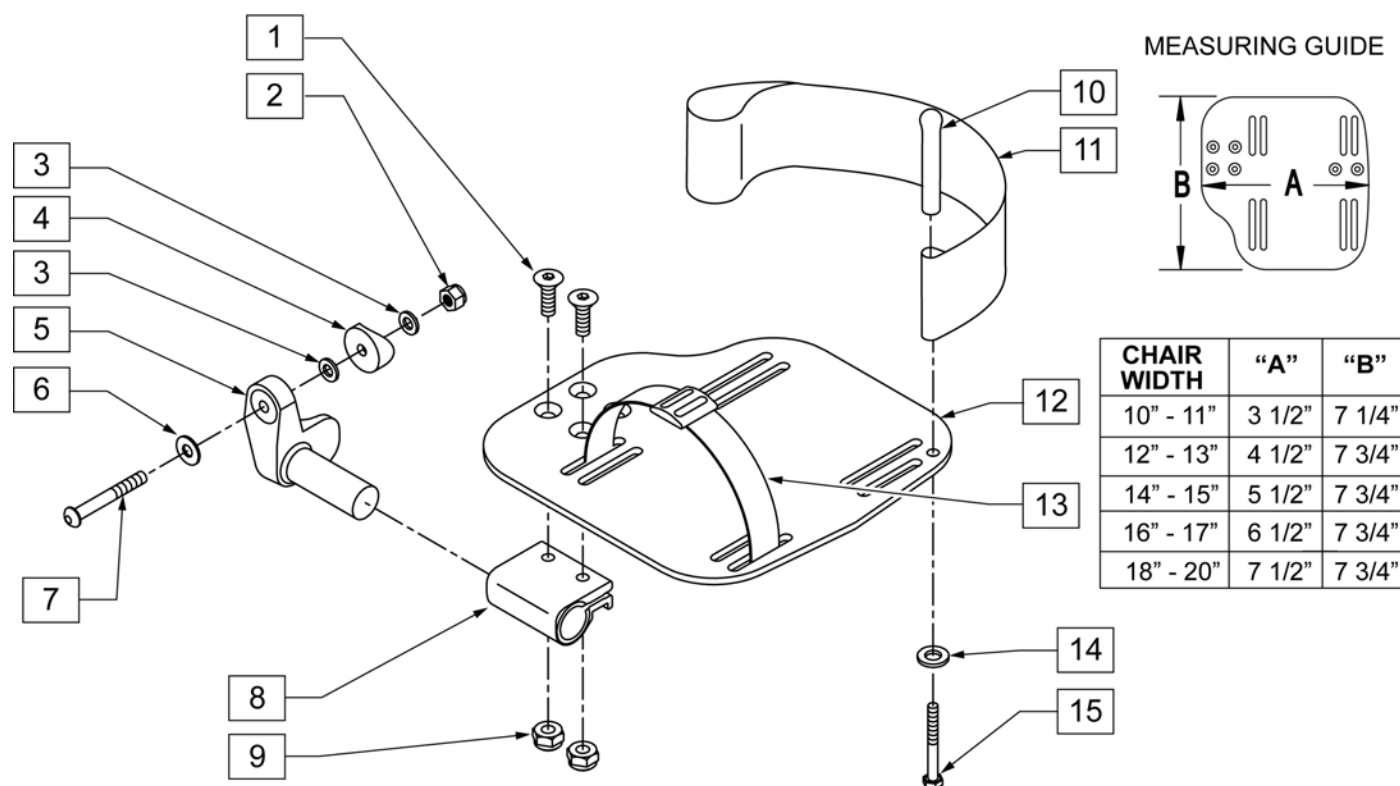

Pos.	Item Number	Description	Remarks
1	106978	FHCS 1/4-28 X 7/8 ROHS	
2	106941	NUT AUTOMATION 1/4-28 ROHS	
3	106943	WASHER FLAT AN 1/4 ROHS	
4	561004	1" SADDLE 1/4" HOLE	
5	541000	FOOTREST MOUNTING BRKT-1" RAD	RIGHT
5	541001	FOOTREST MOUNTING BRKT-1" RAD	LEFT
6	106945	WASHER FLAT SAE 1/4 ROHS	
7	106964	BHCS 1/4-28 X 2	
8	550350	3/4" CLAMP ADJ FOOTPLATE	
9	106935	NUT NYLOCK THIN 1/4-28 ROHS	
10	561040	HEEL LOOP PEG	
11	741153	HEEL LOOP 18-20 CMPST/14-15 ANG ADJ	USED W/ 14-15"/16-17" ADULT ANG ADJ FTPLTS
11	741253	HEEL LOOP ADLT ADJ F/PLT 18-20	USED W/ 18 -20" ADULT ANG ADJ FTPLTS
12	500656	ADULT ANGL ADJ F/PLT 10/11	RIGHT
12	500657	ADULT ANGL ADJ F/PLT 10/11	LEFT
12	500658	ADULT ANGL ADJ F/PLT 12/13	RIGHT
12	500659	ADULT ANGL ADJ F/PLT 12/13	LEFT
12	500660	ADULT ANGL ADJ F/PLT 14/15	RIGHT
12	500661	ADULT ANGL ADJ F/PLT 14/15	LEFT
12	500662	ADULT ANGL ADJ F/PLT 16/17	RIGHT
12	500663	ADULT ANGL ADJ F/PLT 16/17	LEFT
12	500664	ADULT ANGL ADJ F/PLT 18-20	RIGHT
12	500665	ADULT ANGL ADJ F/PLT 18-20	LEFT
13	740260	TOE-ANKLE STRAP ADJ ADULT FOOTPLATE	
14	106943	WASHER FLAT AN 1/4 ROHS	


Pos.	Item Number	Description	Remarks
15	106958	HHCS 1/4-20 X 1 ROHS	
1-9,12	820544	FP ADULT ADJ 90D 10/11 R	RIGHT
1-9,12	820554	FP ADULT ADJ 90D 10/11 L	LEFT
1-9,12	820546	FP ADULT ADJ 90D 12/13 R	RIGHT
1-9,12	820556	FP ADULT ADJ 90D 12/13 L	LEFT
1-9,12	820472	FP ADULT ADJ 90D 14/15 R	RIGHT
1-9,12	820473	FP ADULT ADJ 90D 14/15 L	LEFT
1-9,12	820474	FP ADULT ADJ 90D 16/17 R	RIGHT
1-9,12	820475	FP ADULT ADJ 90D 16/17 L	LEFT
1-9,12	820476	FP ADULT ADJ 90D 18-20 R	RIGHT
1-9,12	820477	FP ADULT ADJ 90D 18-20 L	LEFT
1-9,12	921474	ADULT ANGL ADJ 90D 10/11 OMIT H/L	PAIR
1-9,12	921476	ADULT ANGL ADJ 90D 12/13 OMIT H/L	PAIR
1-9,12	921330	ADULT ANGL ADJ 90D 14/15 OMIT H/L	PAIR
1-9,12	921331	ADULT ANGL ADJ 90D 16/17 OMIT H/L	PAIR
1-9,12	921332	ADULT ANGL ADJ 90D 18-20 OMIT H/L	PAIR
1-12,14,15	820540	F/PLT ADULT ADJ 90D 10/11 R W/ HL	RIGHT
1-12,14,15	820550	F/PLT ADULT ADJ 90D 10/11 L W/ HL	LEFT
1-12,14,15	820542	F/PLT ADULT ADJ 90D 12/13 R W/ HL	RIGHT
1-12,14,15	820552	F/PLT ADULT ADJ 90D 12/13 L W/ HL	LEFT
1-12,14,15	820478	F/PLT ADULT ADJ 90D 14/15 R W/ HL	RIGHT
1-12,14,15	820479	F/PLT ADULT ADJ 90D 14/15 L W/ HL	LEFT
1-12,14,15	820480	F/PLT ADULT ADJ 90D 16/17 R W/ HL	RIGHT
1-12,14,15	820481	F/PLT ADULT ADJ 90D 16/17 L W/ HL	LEFT
1-12,14,15	820482	F/PLT ADULT ADJ 90D 18-20 R W/ HL	RIGHT
1-12,14,15	820483	F/PLT ADULT ADJ 90D 18-20 L W/ HL	LEFT
1-12,14,15	921470	F/PLT ADULT ADJ 90D 10/11 W/ HL	PAIR
1-12,14,15	921472	F/PLT ADULT ADJ 90D 12/13 W/ HL	PAIR
1-12,14,15	921333	F/PLT ADULT ADJ 90D 14/15 W/ HL	PAIR
1-12,14,15	921334	F/PLT ADULT ADJ 90D 16/17 W/ HL	PAIR
1-12,14,15	921335	F/PLT ADULT ADJ 90D 18-20 W/ HL	PAIR

NOTE: WHEN USING AN ARTICULATING LEGREST ASSEMBLY, THE PULL-TO-LOCK WHEEL LOCK MUST BE USED.

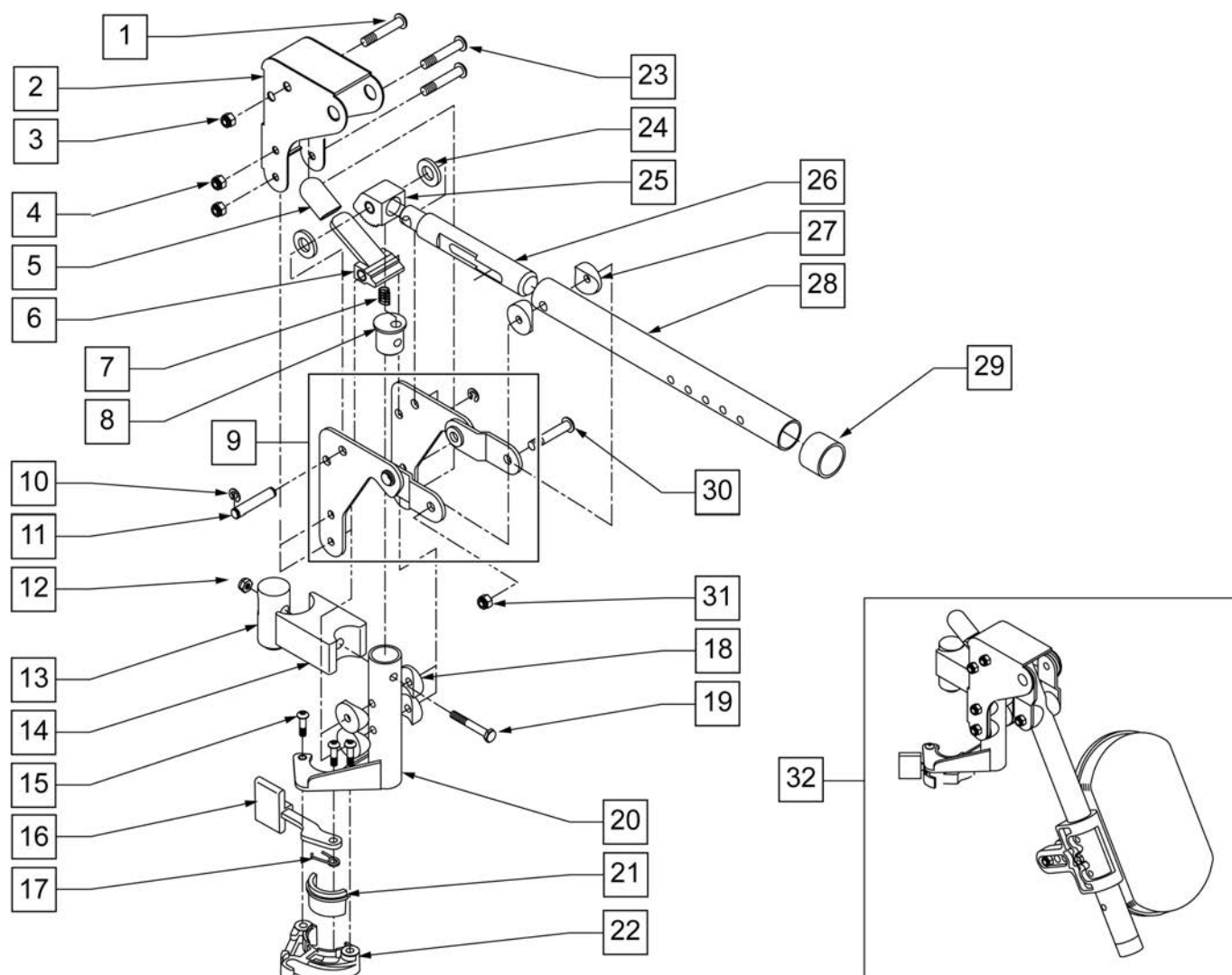

Pos.	Item Number	Description	Remarks
1	106983	5/16-18 x 2 BH C/S SHT THREAD	
2	562720	ALR COVER STD	
3	106937	5/16-18 NYLOCK NUT THIN	
4	106935	1/4-28 NYLOCK NUT THIN	
5	560501	ALR RELEASE LEVER COVER	
6	505902	ALR RELEASE LEVER	
7	321141	650L x 6 COIL SPRING	
8	483060	ALR SPRING PLUG	
9	880526	ALR BRACKET W/LINK R	RIGHT
9	880527	ALR BRACKET W/LINK L	LEFT
10	342000	1/4" E-CLIP	
11	481052	ALR RETAINING STUD	
12	106934	1/4-28 NYLOCK NUT	
13	122991	ALR PIVOT SADDLE WITH LEDGE	


Pos.	Item Number	Description	Remarks
14	122992	ALR PIVOT SADDLE BLOCK W/NOTCH	
15	226057	10/24 x .725 BTTN HD SOC BZ	
16	561079	S/A LATCH LEVER	
17	321136	S/A SPRING L RED	
17	321137	S/A SPRING R CLEAR	
18	561004	1" SADDLE 1/4" HOLE	
19	107016	1/4-28 x 3 G8 HEX C/S	
20	882466	ALR S/A TUBE ASSM L	LEFT
20	882467	ALR S/A TUBE ASSM R	RIGHT
23	106964	1/4-28 x 2 BH C/S	
24	561016	1" WASHER 5/16" HOLE	
25	505900	ALR RATCHET BLOCK	
26	483062	ALR RATCHET ROD	
27	561006	1" SADDLE 5/16" HOLE	
28	505026S	ALR TUBE ONLY	
29	561082	1" TUBE END BUSHING THIN	
30	226195	5/16-18 x 1.82 BH C/S	
31	106937	NUT NYLOCK THIN 5/16-18 ROHS	
21,22	881500	LATCH BLOCK ASSM R	RIGHT
21,22	881501	LATCH BLOCK ASSM L	LEFT
1-26	880092	HANGER ALR SUB ASSM R	RIGHT
1-26	880093	HANGER ALR SUB ASSM L	LEFT
1-31	641464S	HANGER ALR W/O PAD PNTD R	RIGHT
1-31	641465S	HANGER ALR W/O PAD PNTD L	LEFT
32	923051S	ALR HANGER ASSM W/PAD ALUM LB	PAIR
15-17,21,22	881521	S/A LATCH KIT CASTING RIGHT	
15-17,21,22	881522	S/A LATCH KIT CASTING LEFT	

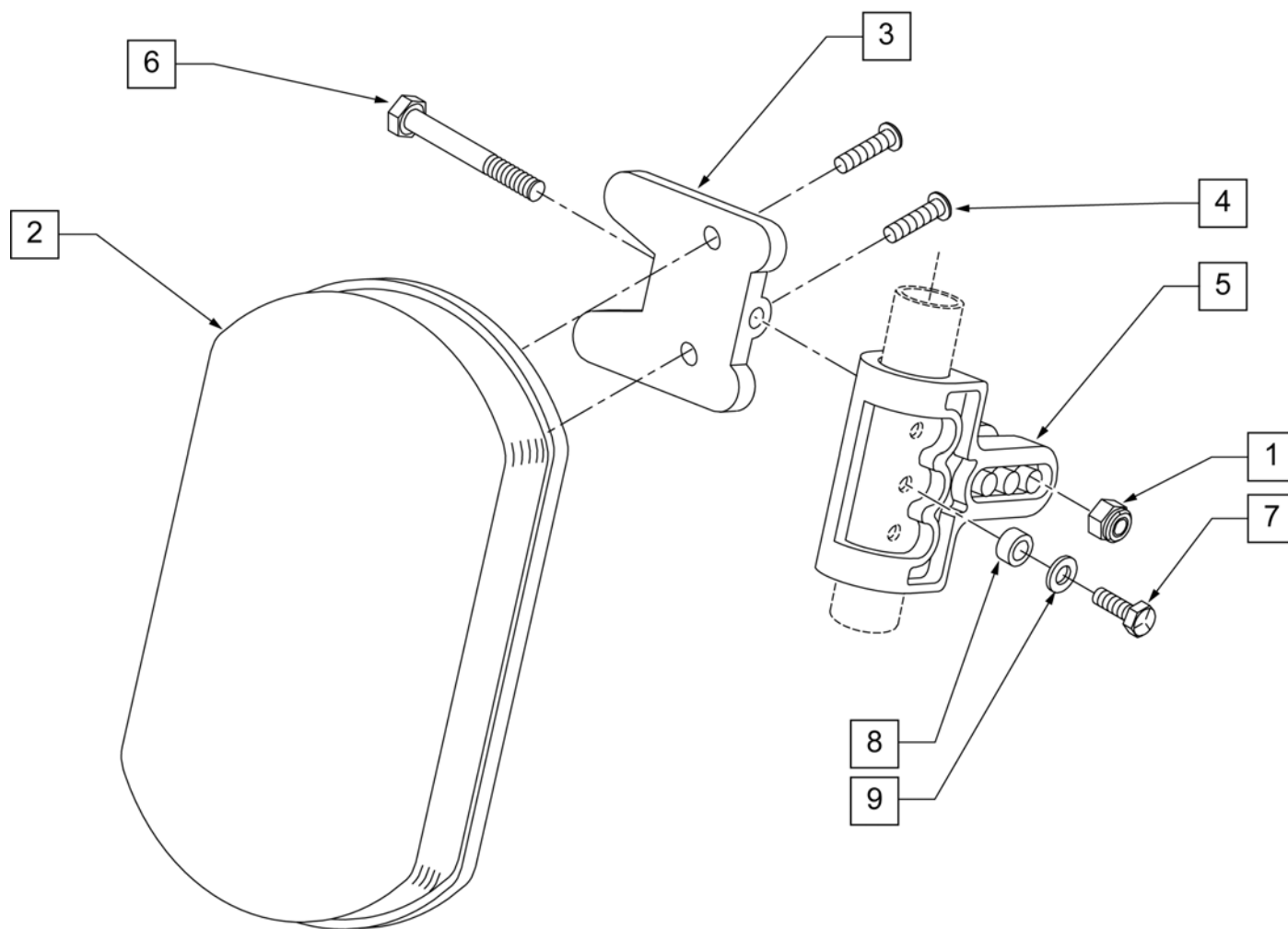


Pos.	Item Number	Description
1	106936	5/16-18 NYLOCK NUT THICK
2	755010	ALR/ELR PAD
3	561096	ALR/ELR CALF PAD BRACKET
4	281182	1/4-20 x 5/8 PAN PHIL
5	561099	CALF CLAMP ALR
6	107019	HHCS 5/16-18 X 3-1/2 ROHS
7	107017	1/4-20 x 1 1/4 G8 HEX C/S
8	483027	1/4 x 1/4 ALUMINUM SPACER BLK
9	104414	WAVE WASHER 6 MM
1-6	881399	ALR CALF PAD W/BACKET
1,3-9	881398	ALR CALF CLAMP ASSM W/O PAD
1-9	881400	ALR CALF PAD ASSEMBLY (EA)

NOTE: (Discontinued on Quickie 2 for 60° and 70° Hangers 10/23/99- Parts only).

NOTE: Used on Articulating Hanger Assemblies.

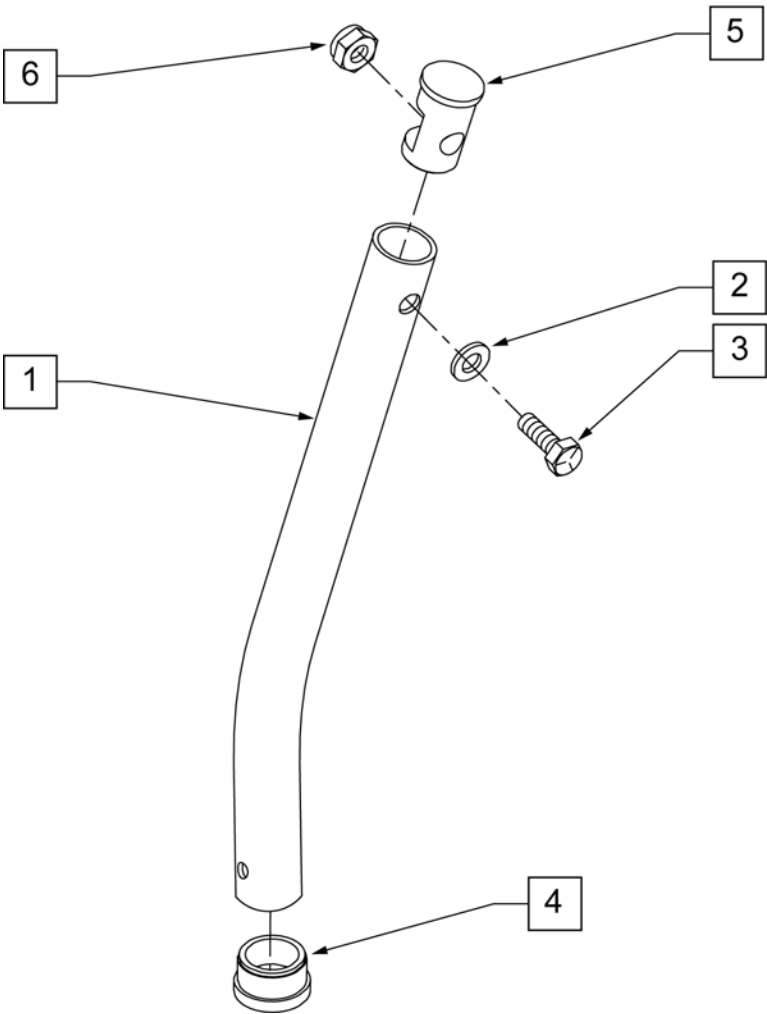


MEASURING GUIDE

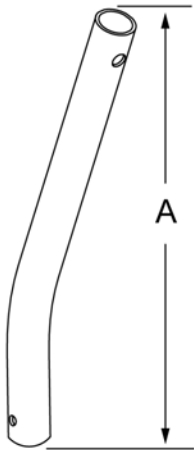

	"A"
SHORT	6 5/8"
MED	9"
LONG	11 5/8"
X-LONG	13 1/2"

Pos.	Item Number	Description
1	521010	FOOTREST EXT TUBE ONLY SHT BLK
1	521011	FOOTREST EXT TUBE ONLY MED BLK
1	521012	FOOTREST EXT TUBE ONLY LNG BLK
1	521013	FOOTREST EXT TUBE ONLY X-LNG BLK
2	104414	WAVE WASHER 6 MM
3	106958	HHCS 1/4-20 X 1 ROHS
4	561000-QU	3/4" INSERT PLUG .083
5	483553	FOOTREST INSERT PLUG Q2
6	106932	NUT NYLOCK 1/4-20 ROHS
1-6	921221	FOOTREST EXT TUBE SHT ASSM BLK (EA)
1-6	921222	FOOTREST EXT TUBE MED ASSM BLK (EA)
1-6	921223	FOOTREST EXT TUBE LNG ASSM BLK (EA)
1-6	921224	FOOTREST EXT TUBE X-LNG ASSM BLK (EA)

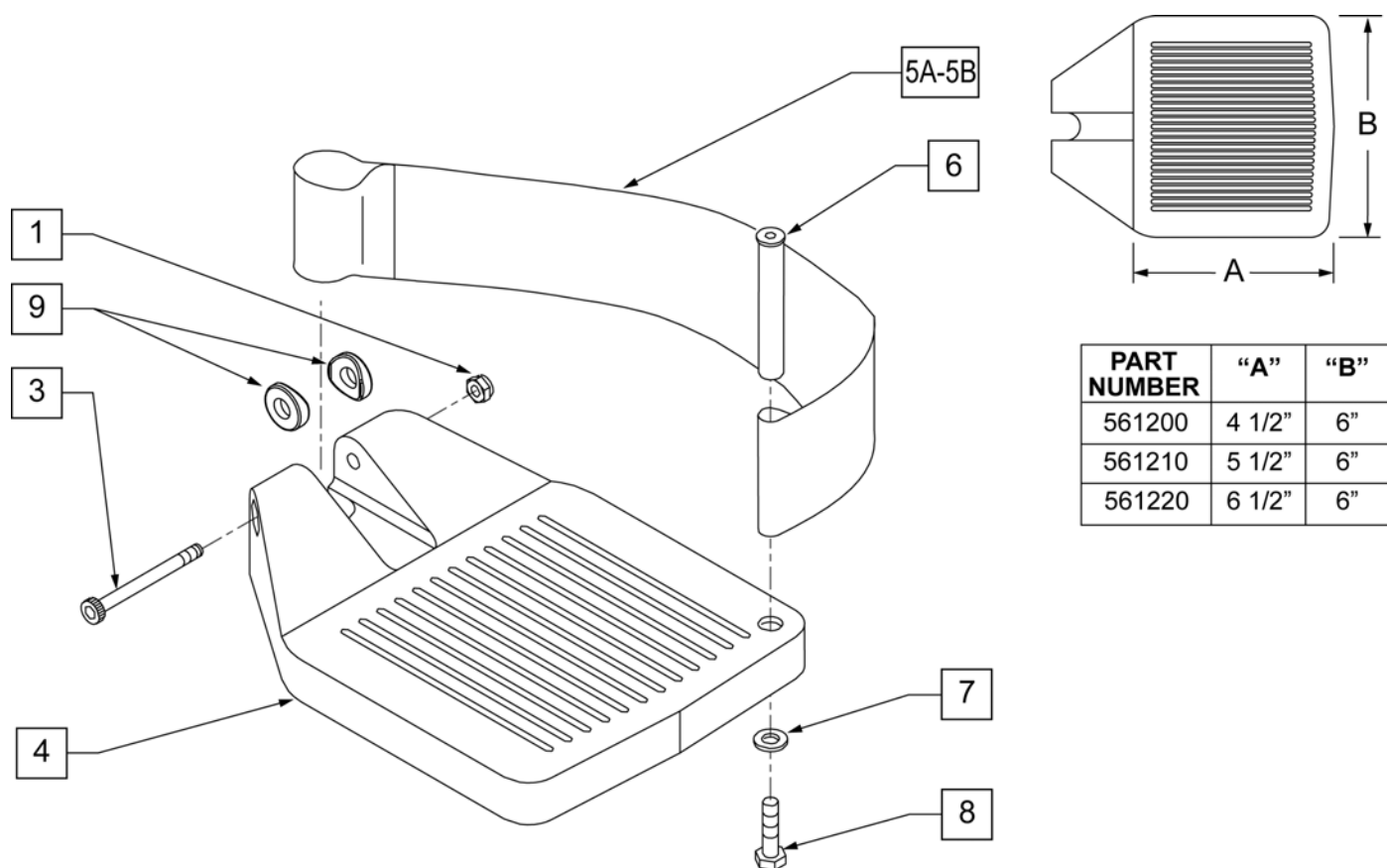

Pos.	Item Number	Description	Remarks
1	106935	1/4 - 28 NYLOCK NUT	
3	226165	1/4 - 28 x 2 1/2 SOC C/S	
4	561200	COMPOSITE FOOTPLATE 14 -15 (16 -18 V)	14 -15 OR 16 -18V
4	561210	COMPOSITE FOOTPLATE 16 -17 (19 -20 V)	16 -17 OR 19 -20V
4	561220	COMPOSITE FOOTPLATE 18 -20	18 -20
5A	741151	HEEL LOOP 14-15 COMPOSITE (14 -15 or 16 -18V)	14 -15 OR 16 -18V
5A	741156	HEEL LOOP 16 -17 or 19 -20V	16 -17 OR 19 -20V
5A	741153	HEEL LOOP 18 -20 COMPOSITE/14-15 ANG ADJ	18 -20
5A	902270	HEEL LOOP COMPOSITE FT 14-15	PAIR 14 -15 OR 16 -18V
5A	902271	HEEL LOOP COMPOSITE FT 16-17	PAIR 16 -17 OR 19 -20V
5A	902272	HEEL LOOP COMPOSITE FT 18-20	PAIR 18 -20
5B	741154	HEEL LOOP 14-15 COMPOSITE HEMI	14 -15 HEMI
5B	741155	HEEL LOOP 16-17 COMPOSITE HEMI	16 -17 HEMI
5B	741153	HEEL LOOP 18-20 COMPOSITE HEMI	18 -10 HEMI
5B	902260	HEEL LOOPS HEMI COMPOSITE 14-15	PAIR 14 -15 HEMI
5B	902261	HEEL LOOPS HEMI COMPOSITE 16-17	PAIR 16 -17 HEMI
5B	902262	HEEL LOOPS HEMI COMPOSITE 18-20	PAIR 18 -20 HEMI
6	561040	HEEL LOOP PEG	
7	106944	1/4 AN WASHER FLAT THIN	
8	106958	HHCS 1/4-20 X 1 ROHS	
9	561003	3/4" SADDLE THIN	USED W/ ROUND EXT TUBES ONLY
1,3,4	921180	COMPOSITE FOOTPLATE ASSM 14-15	RIGHT (14 -15 OR 16 -18V) W/O HEEL LOOP
1,3,4	921181	COMPOSITE FOOTPLATE ASSM 14-15	LEFT (14 -15 OR 16 -18V) W/O HEEL LOOP
1,3,4	921182	COMPOSITE FOOTPLATE ASSM 16-17	RIGHT (16 -17 OR 19 -20V) W/O HEEL LOOP
1,3,4	921183	COMPOSITE FOOTPLATE ASSM 16-17	LEFT (16 -17 OR 19 -20V) W/O HEEL LOOP

Pos.	Item Number	Description	Remarks
1,3,4	921184	COMPOSITE FOOTPLATE ASSM 18-20	RIGHT (18 -20) W/O HEEL LOOP
1,3,4	921185	COMPOSITE FOOTPLATE ASSM 18-20	LEFT (18 -20) W/O HEEL LOOP
1,3,4	921151	COMPOSITE FOOTPLATE ASSM 14-15/16-18V	PAIR (14 -15 OR 16 -18V) W/O HEEL LOOP
1,3,4	921152	COMPOSITE FOOTPLATE ASSM 16-17/19-20V	PAIR (16 -17 OR 19 -20V) W/O HEEL LOOP
1,3,4	921153	COMPOSITE FOOTPLATE ASSM 18-20	PAIR (18 -20) W/O HEEL LOOP
1,3,4A-8	921190	COMPOSITE FOOTPLATE ASSM HL LP 14-15	RIGHT (14 -15 OR 16 -18V) W/ HEEL LOOP
1,3,4A-8	921191	COMPOSITE FOOTPLATE ASSM HL LP 14-15	LEFT (14 -15 OR 16 -18V) W/ HEEL LOOP
1,3,4A-8	921192	COMPOSITE FOOTPLATE ASSM HL LP 16-17	RIGHT (16 -17 OR 19 -20V) W/ HEEL LOOP
1,3,4A-8	921193	COMPOSITE FOOTPLATE ASSM HL LP 16-17	LEFT (16 -17 OR 19 -20V) W/ HEEL LOOP
1,3,4A-8	921194	COMPOSITE FOOTPLATE ASSM HL LP 18-20	RIGHT (18 -20) W/ HEEL LOOP
1,3,4A-8	921195	COMPOSITE FOOTPLATE ASSM HL LP 18-20	LEFT (18 -20) W/ HEEL LOOP
1,3,4A-8	921161	COMPOSITE FOOTPLATE ASSM HL 14-15/16-18V	PAIR (14 -15 OR 16 -18V) W/ HEEL LOOP
1,3,4A-8	921162	COMPOSITE FOOTPLATE ASSM HL 16-17/19-20V	PAIR (16 -17 OR 19 -20V) W/ HEEL LOOP
1,3,4A-8	921163	COMPOSITE FOOTPLATE ASSM HL 18-20	PAIR (18 -20) W/ HEEL LOOP
1,3,4B-8	921140	COMPOSITE FOOTPLATE ASSM HEMI HL LP 14-15	RIGHT (14 -15) W/ HEEL LOOP
1,3,4B-8	921141	COMPOSITE FOOTPLATE ASSM HEMI HL LP 14-15	LEFT (14 -15) W/ HEEL LOOP
1,3,4B-8	921142	COMPOSITE FOOTPLATE ASSM HEMI HL LP 16-17	RIGHT (16 -17) W/ HEEL LOOP
1,3,4B-8	921143	COMPOSITE FOOTPLATE ASSM HEMI HL LP 16-17	LEFT (16 -17) W/ HEEL LOOP
1,3,4B-8	921144	COMPOSITE FOOTPLATE ASSM HEMI HL LP 18-20	RIGHT (18 -20) W/ HEEL LOOP
1,3,4B-8	921145	COMPOSITE FOOTPLATE ASSM HEMI HL LP 18-20	LEFT (18 -20) W/ HEEL LOOP
1,3,4B-8	921167	COMPOSITE FOOTPLATE ASSM HEMI HL LPS 14-15	PAIR (14 -15) W/ HEEL LOOP
1,3,4B-8	921168	COMPOSITE FOOTPLATE ASSM HEMI HL LPS 16-17	PAIR (16 -17) W/ HEEL LOOP
1,3,4B-8	921169	COMPOSITE FOOTPLATE ASSM HEMI HL LPS 18-20	PAIR (18 -20) W/ HEEL LOOP


NOTE: FOR HIGH-MOUNT ADULT ANGLE-ADJUSTABLE FOOTPLATE, SEE "HIGH-MOUNT EXTENSION TUBE" PAGE.

NOTE: ORDER ITEM 4B FOR CHAIRS PRIOR TO 8/13/01.

NOTE: ONLY 1 WASHER PRIOR TO 8/13/01.


NOTE: ORDER PART 226167 PRIOR TO 8/13/01.

NOTE: See attached REFERENCE GUIDE.

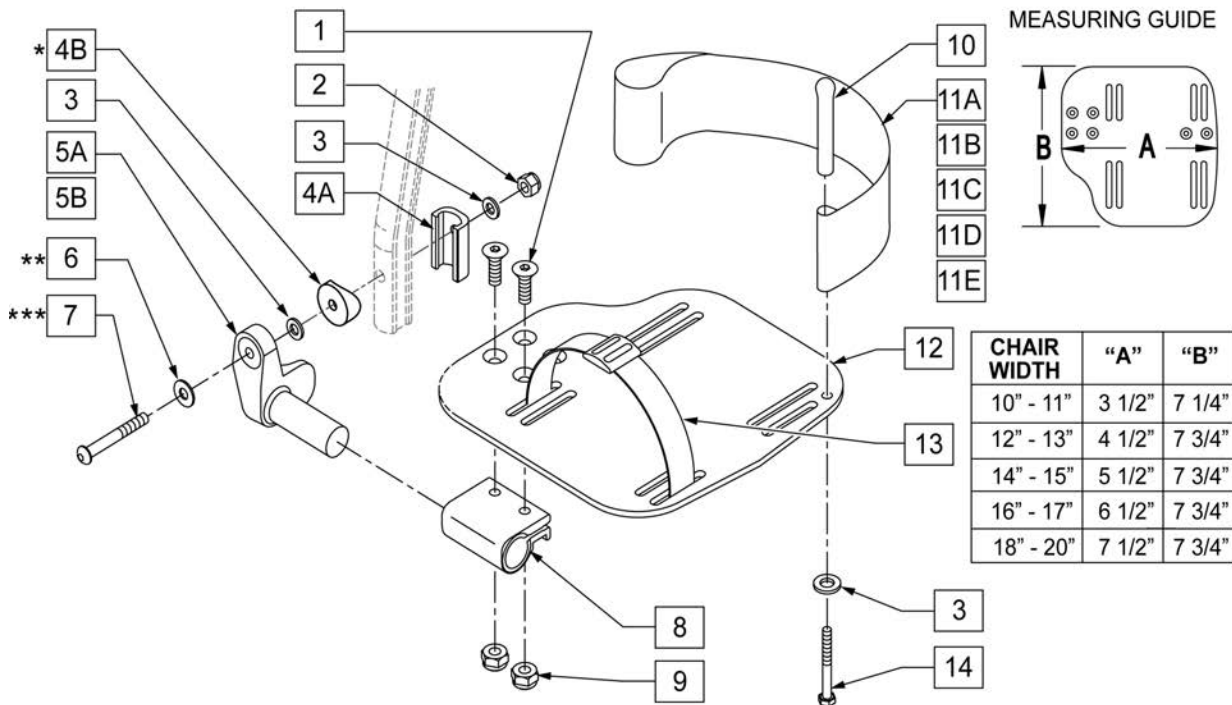



Pos.	Item Number	Description	Remarks
1	106978	1/4-28 x 7/8 FH C/S	
2	106941	1/4-28 AUTOMATION NUT	

Pos.	Item Number	Description	Remarks
3	106943	WASHER FLAT AN 1/4 ROHS	
4A	483720	HANGER EXTENSION SADDLE LONG	USE W/ DOUBLE D EXT TUBE
4B	561055	3/4" SADDLE 1" OD	USE W/ ROUND EXT TUBE
5	541002	FTRST MOUNTING BRKT-3/8 RAD R	RIGHT
5	541003	FTRST MOUNTING BRKT-3/8 RAD L	LEFT
6	106945	WASHER FLAT SAE 1/4 ROHS	
7A	106962	BHCS 1/4-28 X 1-3/4 ROHS	USE W/ DOUBLE D EXT TUBE
7B	226167	1/4-28 x 1.625 BH C/S BLK	USE W/ ROUND EXT TUBE
8	550350	3/4" CLMP ADJ FT PLT	
9	106935	NUT NYLOCK THIN 1/4-28 ROHS	
10	561040	HEEL LOOP PEG	
11A	741154	HEEL LOOP 14-15 CMPST HEMI	USED W/ 10-11" ADULT ANG ADJ FTPLTS
11B	741155	HEEL LOOP 16-17 CMPST HEMI	USED W/ 12 -13" ADULT ANG ADJ FTPLTS
11C	741153	H/L 18-20 CMPST/14-15 ANG ADJ	USED W/ 14-15"/16-17" ADULT ANG ADJ FTPLTS
11D	741253	HEEL LOOP ADLT ADJ F/PLT 18-20	USED W/ 18 -20" ADULT ANG ADJ FTPLTS
12	500656	ADLT ANGL ADJ F/PLT 10/11" R	RIGHT
12	500657	ADLT ANGL ADJ F/PLT 10/11" L	LEFT
12	500658	ADLT ANGL ADJ F/PLT 12/13" R	RIGHT
12	500659	ADLT ANGL ADJ F/PLT 12/13" L	LEFT
12	500660	ADLT ANGL ADJ F/PLT 14/15	RIGHT
12	500661	ADLT ANGL ADJ F/PLT 14/15	LEFT
12	500662	ADLT ANGL ADJ F/PLT 16/17	RIGHT
12	500663	ADLT ANGL ADJ F/PLT 16/17	LEFT
12	500664	ADLT ANGL ADJ F/PLT 18-20	RIGHT
12	500665	ADLT ANGL ADJ F/PLT 18-20	LEFT
13	740260	TOE/ANKLE STRAP ADJ ADLT F/PLT	
14	106958	1/4-20 x 1 HEX C/S	
1-3,5,6,8,9,12	820526	AD ANG ADJ F/PLT R 12"/13" OMIT HL	RIGHT
1-3,5,6,8,9,12	820536	AD ANG ADJ F/PLT L 12"/13" OMT HL	LEFT
1-3,5,6,8,9,12	820460	FP ADLT ADJ 14/15 OMIT HL	RIGHT
1-3,5,6,8,9,12	820462	FP ADLT ADJ 14/15 OMIT HL	LEFT
1-3,5,6,8,9,12	820464	FP ADLT ADJ 16/17 OMIT HL	RIGHT
1-3,5,6,8,9,12	820466	FP ADLT ADJ 16/17 OMIT HL	LEFT
1-3,5,6,8,9,12	820468	FP ADLT ADJ 18-20 OMIT HL	RIGHT
1-3,5,6,8,9,12	820470	FP ADLT ADJ 18-20 OMIT HL	LEFT
1-3,5,6,8,9,12	921466	ADLT ANGL ADJ F/PLT 12"/13" OMIT HL	PAIR
1-3,5,6,8,9,12	921320	FP ADLT ADJ 14/15 OMIT HL	PAIR
1-3,5,6,8,9,12	921322	FP ADLT ADJ 16/17 OMIT HL	PAIR
1-3,5,6,8,9,12	921324	FP ADLT ADJ 18-20 OMIT HL	PAIR
1-3,5,6,8,9,12	820522	AD ANGL ADJ F/PLT R 12/13" H/L	RIGHT
1-3,5,6,8,9,12	820532	AD ANGL ADJ F/PLT L 12/13" H/L	LEFT
1-3,5,6,8-10,11C, 12,14	820461	FP ADLT ADJ 14/15 HL	RIGHT
1-3,5,6,8-10,11C, 12,14	820463	FP ADLT ADJ 14/15 HL	LEFT
1-3,5,6,8-10,11C, 12,14	820465	FP ADLT ADJ 16/17 HL	RIGHT
1-3,5,6,8-10,11C, 12,14	820467	FP ADLT ADJ 16/17 HL	LEFT
1-3,5,6,8-10,11D, 12,14	820469	FP ADLT ADJ 18-20 HL	RIGHT
1-3,5,6,8-10,11D, 12,14	820471	FP ADLT ADJ 18-20 HL	LEFT
1-3,5,6,8-10,11B, 12,14	921462	ADLT ANGL ADJ F/PLATE 12"/13" HL	PAIR
1-3,5,6,8-10,11C, 12,14	921321	FP ADLT ADJ 14/15 HL	PAIR
1-3,5,6,8-10,11C, 12,14	921323	FP ADLT ADJ 16/17 HL	PAIR
1-3,5,6,8-10,11D, 12,14	921325	FP ADLT ADJ 18-20 HL	PAIR
1-3,6,7A,9-10,14	101533	ADLT ANG ADJ FP D2 HDWR KIT	FOR DOUBLE D EXT TUBES
1-3B,4B,6,7B,9,10,14	101534	ADLT ANG ADJ FP RND HDWR KIT	FOR ROUND EXT TUBES (Q2 HP ONLY)

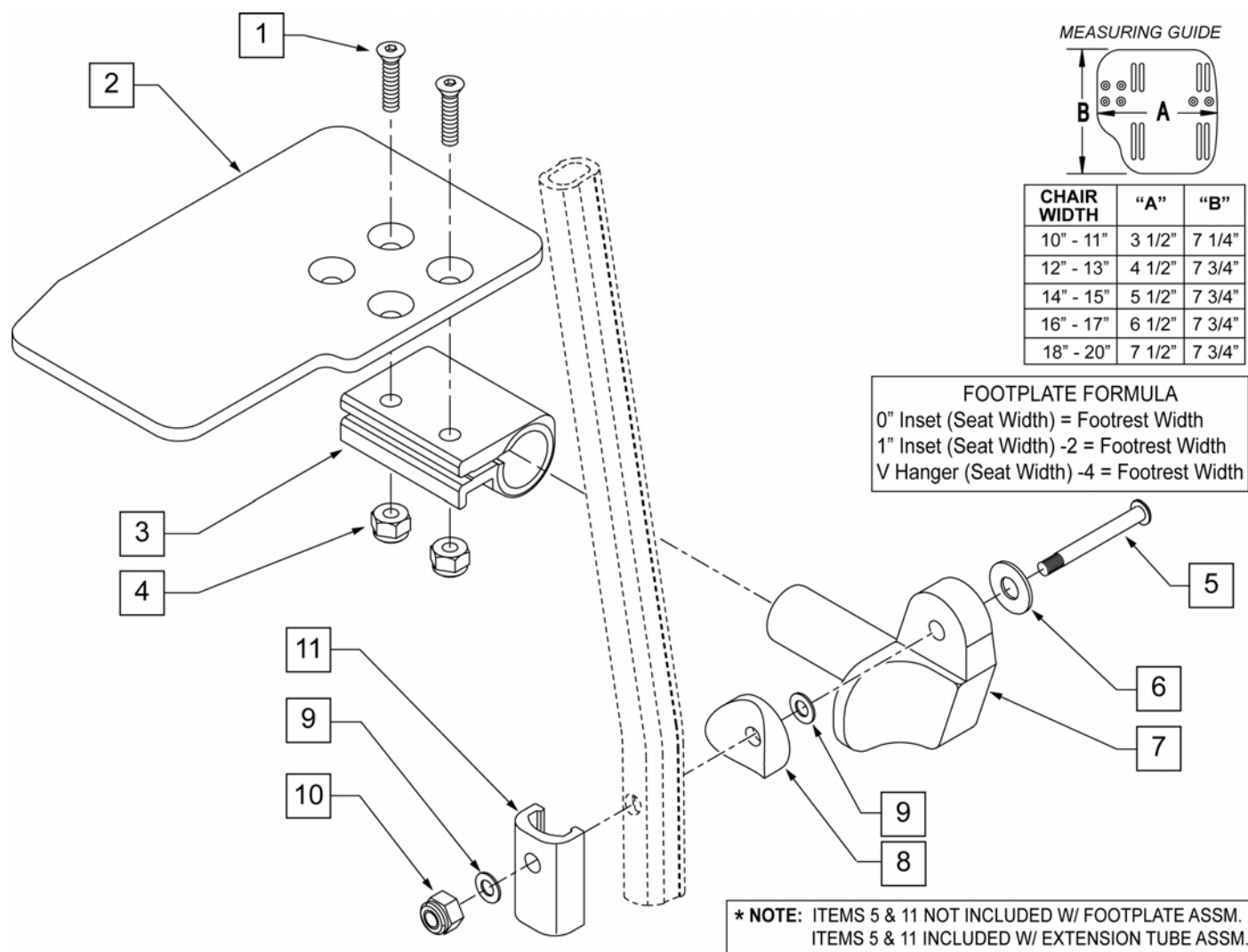

Pos.	Item Number	Description	Remarks
1	106978	FHCS 1/4 -28 x 7/8 ROHS	
2	500635	KIDS ANGL ADJ F/PLT 10" R	Right
2	500636	KIDS ANGL ADJ F/PLT 10" L	Left
2	500637	KIDS ANGL ADJ F/PLT 11" R	Right
2	500638	KIDS ANGL ADJ F/PLT 11" L	Left
2	500639	KIDS ANGL ADJ F/PLT 12" R	Right
2	500640	KIDS ANGL ADJ F/PLT 12" L	Left
2	500641	KIDS ANGL ADJ F/PLT 13" R	Right
2	500642	KIDS ANGL ADJ F/PLT 13" L	Left
2	500643	KIDS ANGL ADJ F/PLT 14" R	Right
2	500644	KIDS ANGL ADJ F/PLT 14" L	Left
2	500645	KIDS ANGL ADJ F/PLT 15" R	Right
2	500646	KIDS ANGL ADJ F/PLT 15" L	Left
2	500647	KIDS ANGL ADJ F/PLT 16" R	Right
2	500648	KIDS ANGL ADJ F/PLT 16" L	Left
3	550350	3/4" CLAMP ADJ FOOTPLATE	
4	106935	NUT NYLOCK THIN 1/4 -28 ROHS	
5	106962	1/4-28 x 1 3/4 BH C/S BLK ZN (D2 EXTENSION TUBES)	

Pos.	Item Number	Description	Remarks
5	226167	1/4-28 x 1.625 BH C/S BLK (ROUND EXTENSION TUBES)	
6	106945	1/4 SAE WASHER	
7	541002	FTRST MOUNTING BRKT-3/8 RAD R	
7	541003	FTRST MOUNTING BRKT-3/8 RAD L	
8	561055	3/4" SADDLE 1" OD	
9	106944	WASHER FLAT THIN AN 1/4 ROHS	
10	106941	NUT AUTOMATION 1/4 -28 ROHS	
1-7,9,10	828850	KIDS ANG ADJ F/PLT EXT-MNT 10"R	Right
1-7,9,10	828851	KIDS ANG ADJ F/PLT EXT-MNT 10"L	Left
1-7,9,10	828852	KIDS ANG ADJ F/PLT EXT-MNT 11"R	Right
1-7,9,10	828853	KIDS ANG ADJ F/PLT EXT-MNT 11"L	Left
1-7,9,10	828854	KIDS ANG ADJ F/PLT EXT-MNT 12"R	Right
1-7,9,10	828855	KIDS ANG ADJ F/PLT EXT-MNT 12"L	Left
1-7,9,10	828856	KIDS ANG ADJ F/PLT EXT-MNT 13"R	Right
1-7,9,10	828857	KIDS ANG ADJ F/PLT EXT-MNT 13"L	Left
1-7,9,10	828858	KIDS ANG ADJ F/PLT EXT-MNT 14"R	Right
1-7,9,10	828859	KIDS ANG ADJ F/PLT EXT-MNT 14"L	Left
1-7,9,10	828860	KIDS ANG ADJ F/PLT EXT-MNT 15"R	Right
1-7,9,10	828861	KIDS ANG ADJ F/PLT EXT-MNT 15"L	Left
1-7,9,10	828862	KIDS ANG ADJ F/PLT EXT-MNT 16"R	Right
1-7,9,10	828863	KIDS ANG ADJ F/PLT EXT-MNT 16"L	Left
1-7,9,10	921280	KIDS ANG ADJ F/PLT EXT-MNT 10"	Pair
1-7,9,10	921281	KIDS ANG ADJ F/PLT EXT-MNT 11"	Pair
1-7,9,10	921282	KIDS ANG ADJ F/PLT EXT-MNT 12"	Pair
1-7,9,10	921283	KIDS ANG ADJ F/PLT EXT-MNT 13"	Pair
1-7,9,10	921284	KIDS ANG ADJ F/PLT EXT-MNT 14"	Pair
1-7,9,10	921285	KIDS ANG ADJ F/PLT EXT-MNT 15"	Pair
1-7,9,10	921286	KIDS ANG ADJ F/PLT EXT-MNT 16"	Pair

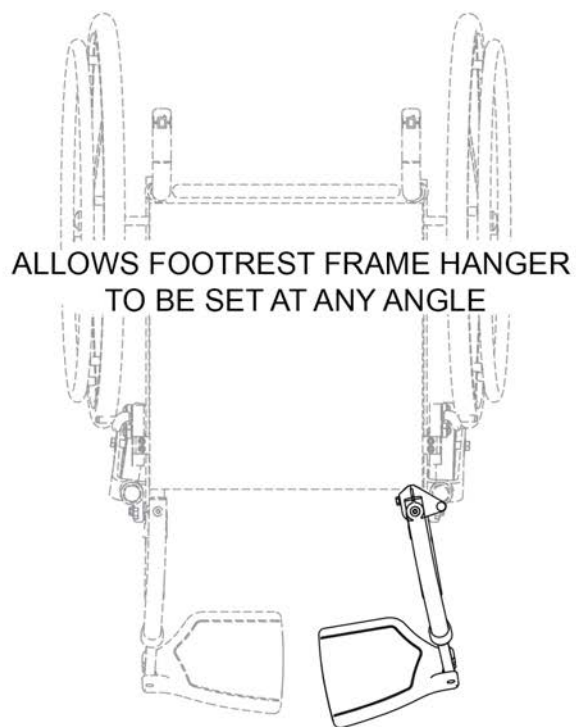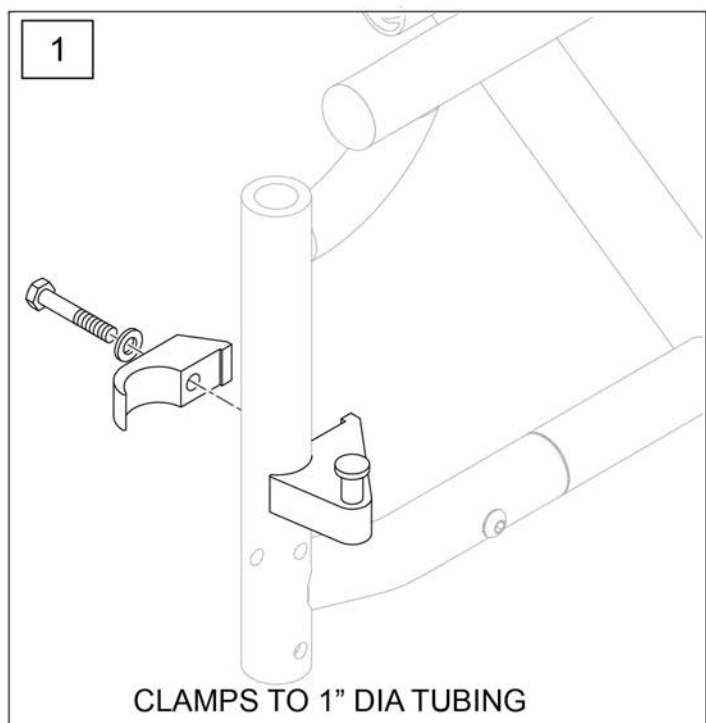

Pos.	Item Number	Description	Remarks
1	112807	LATCH STUD SUPPORT R MULTIPOS	Single
1	112808	LATCH STUD SUPPORT L MULTIPOS	Single
1	112809	LATCH STUD SUPPORT PR MULTIPOS	Pair


NOTE: Effective on GTi chairs after S/N T4-505800 & T4A-505900.

NOTE: Effective on GT chairs after S/N R3-012700 & R3A-012762.

NOTE: Armrest heights that can be achieved are; 7, 8, & 9 with the RECEIVER IN THE DOWN position, and 10, 11, & 12 with RECEIVER IN THE UP position.


NOTE: GT,GTi & Q2 Lite chair uses Low Armrest only.

Part Changes, please see notes/remarks for important information

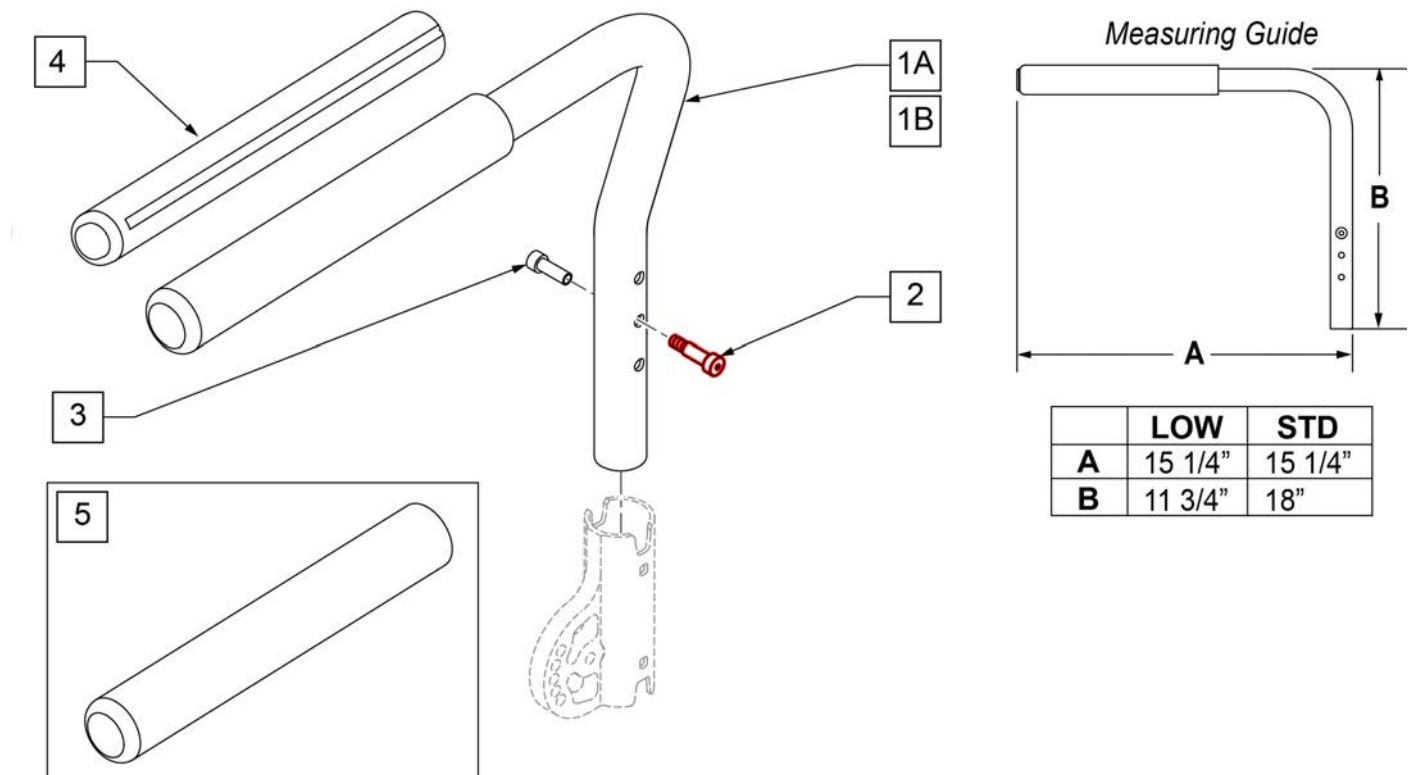


Pos.	Item Number	Description	Remarks
1A-3	111153	SWING-AWAY ARMREST LOW W/ BOLT R	RIGHT-LOW
1A-3	111154	SWING-AWAY ARMREST LOW W/ BOLT L	LEFT -LOW
1B-3	111155	SWING-AWAY ARMREST STD W/ BOLT R	RIGHT-STD
1B-3	111156	SWING-AWAY ARMREST STD W/ BOLT L	LEFT -STD
2,3	108118	SEXBOL TSA ARMREST KIT	
4	108315	GEL PAD S/ ARM	PAIR
5	560640	9X.93X.155 FOAM GRIP	Single replacement grip

NOTE: Item 6 can be flipped for additional adjustment, when flipped Left becomes Right and Right becomes Left.

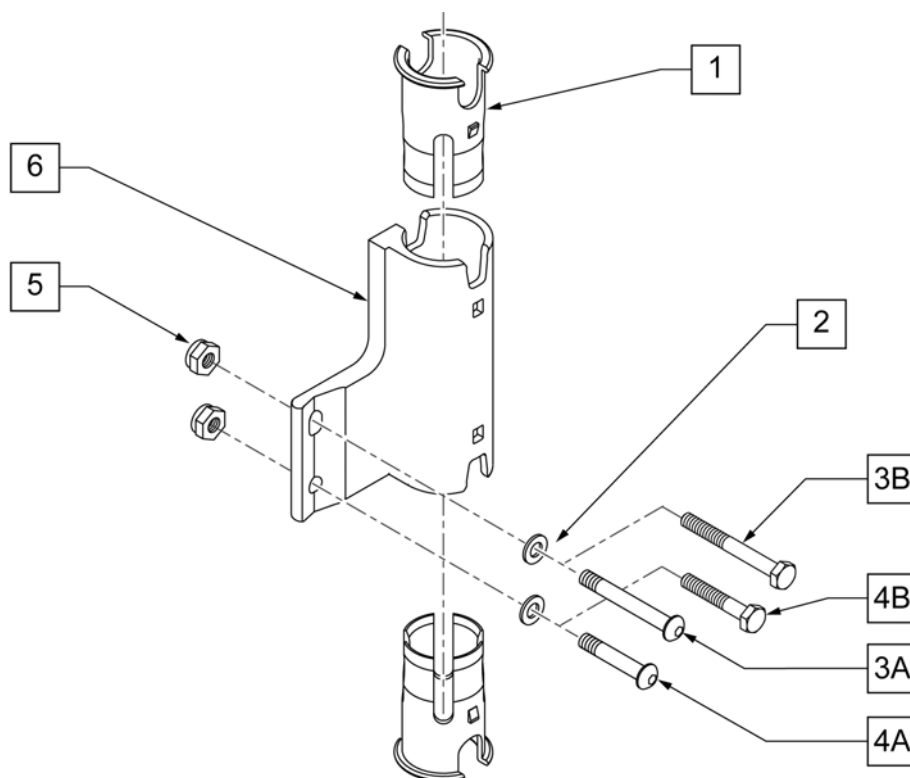


Pos.	Item Number	Description	Remarks
1	106505	SLEEVE SWING AWAY ARMREST	
2	106943	WASHER FLAT AN 1/4 ROHS	
3A	106964	BHCS 1/4-28 X 2 ROHS	Q2 LITE ONLY
3B	107351	HHCS 1/4-28 X 2 1/4 ROHS	Q2 ONLY
4A	106962	BHCS 1/4-28 X 1-3/4 ROHS	Q2 LITE ONLY
4B	109265	HHCS 1/4-28 X 2-1/2 ROHS	Q2 ONLY
5	106935	NUT NYLOCK THIN 1/4-28 ROHS	
1,6	106720	RECEIVER, SA ASSY, RH	RIGHT - SHOWN
1,6	106721	RECEIVER, SA ASSY, LH	LEFT

NOTE: IF CHAIR IS EQUIPPED WITH REMOVABLE STROLLER HANDLES AND PADDED SWING AWAY ARMREST THEN A DOUBLE BARRELL RECEIVER MUST BE USED. PLEASE REFER TO THE REMOVABLE STROLLER HANDLE PARTS PAGE FOR REPLACEMENT PART NUMBERS.

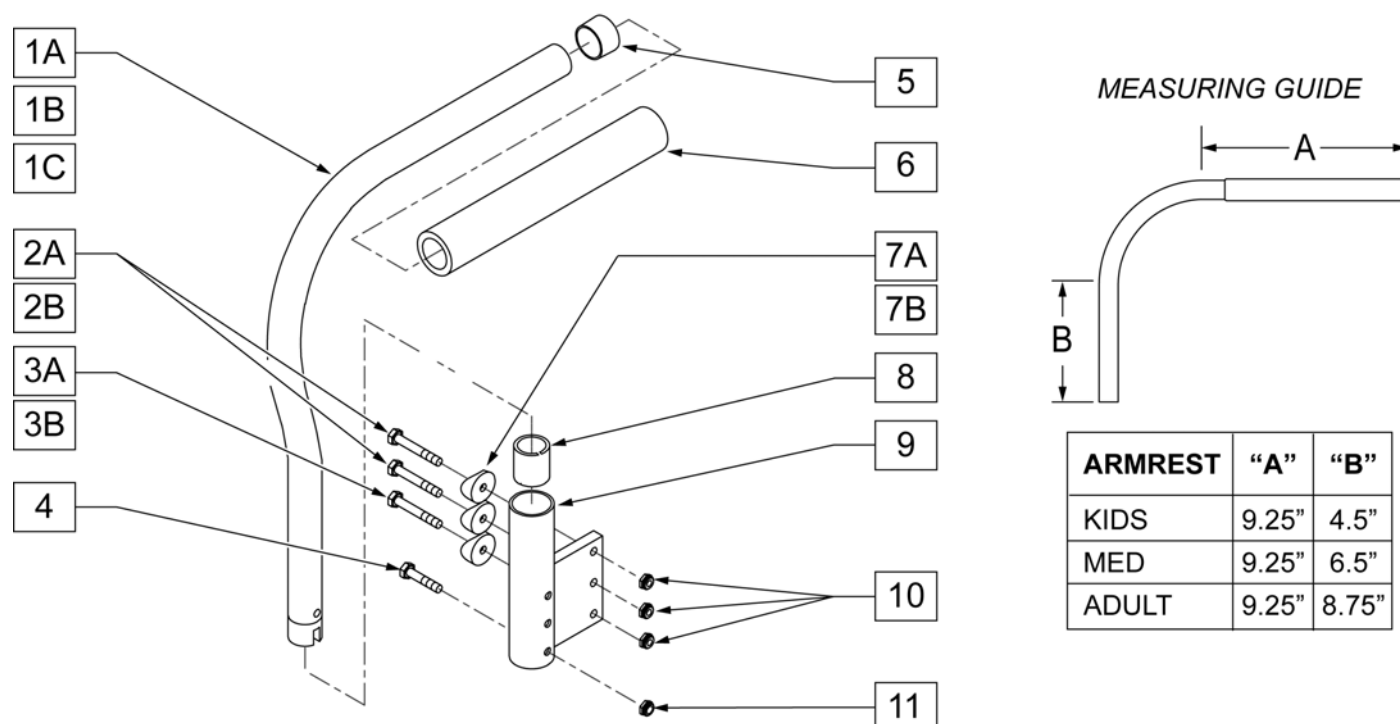


Pos.	Item Number	Description	Remarks
2A	107351	1/4-28 x 2 1/4 G8 HEX C/S	For chairs built AFTER 4/30/1990.
2B	106973	1/4-28 x 2 G8 HEX C/S	For chairs built BEFORE 4/30/1990.
3A	107015	1/4-28 x 1 3/4 G8 HEX C/S	For chairs built BEFORE 4/30/1990.
3B	106973	1/4-28 x 2 G8 HEX C/S	For chairs built AFTER 4/30/1990.
4	106972	1/4-28 x 1 1/2 G8 HEX C/S	
5	561002	1" INSERT PLUG .083	
6	920070	9 x .940 x .200 FOAM GRIP	
7A	561004	1" SADDLE 1/4" HOLE	For chairs built BEFORE 4/30/1990.
7B	561001	1 1/8" SADDLE 1/4" HOLE	For chairs built AFTER 4/30/1990.
8	560400	1" ID NY-LINER	
9	540100	PADDED ARM RECEIVER STRT F	RIGHT - SEE NOTE
9	540102	PADDED ARM RECEIVER STRT F	LEFT - SEE NOTE
10	106934	1/4-28 NYLOCK NUT	
11	106935	NUT NYLOCK THIN 1/4-28 ROHS	
2A,3B,7A,10	920154	PADDED ARM REC HDWR KIT 1 1/8 F	
1A, 5, 6	880320	PADDED ARM KIDS	RIGHT-KIDS
1A, 5, 6	880322	PADDED ARM KIDS	LEFT-KIDS
1B, 5, 6	880324	PADDED ARM RX KIDS MEDIUM	RIGHT-MED
1B, 5, 6	880326	PADDED ARM RX KIDS MEDIUM	LEFT-MED
1C, 5, 6	880328	PADDED ARM ADULT	RIGHT-ADULT

Pos.	Item Number	Description	Remarks
1C, 5, 6	880330	PADDED ARM ADULT	LEFT-ADULT
1A, 5, 6	900400	PADDED ARMS Q2 KIDZ F	PAIR-KIDS
1B, 5, 6	900401	PADDED ARMS Rx KIDZ/MED F	PAIR-MED
1C, 5, 6	900402	PADDED ARMS REG F	PAIR-ADULT
2B, 3A, 4, 7A,8-11	900510	ARM REC SINGLE CYL F 3/4"	PAIR, For chairs built BEFORE 4/30/1990.
2A, 3B, 4, 7B,8-11	900512	ARM REC SINGLE CYL F 7/8"	PAIR, For chairs built AFTER 4/30/1990

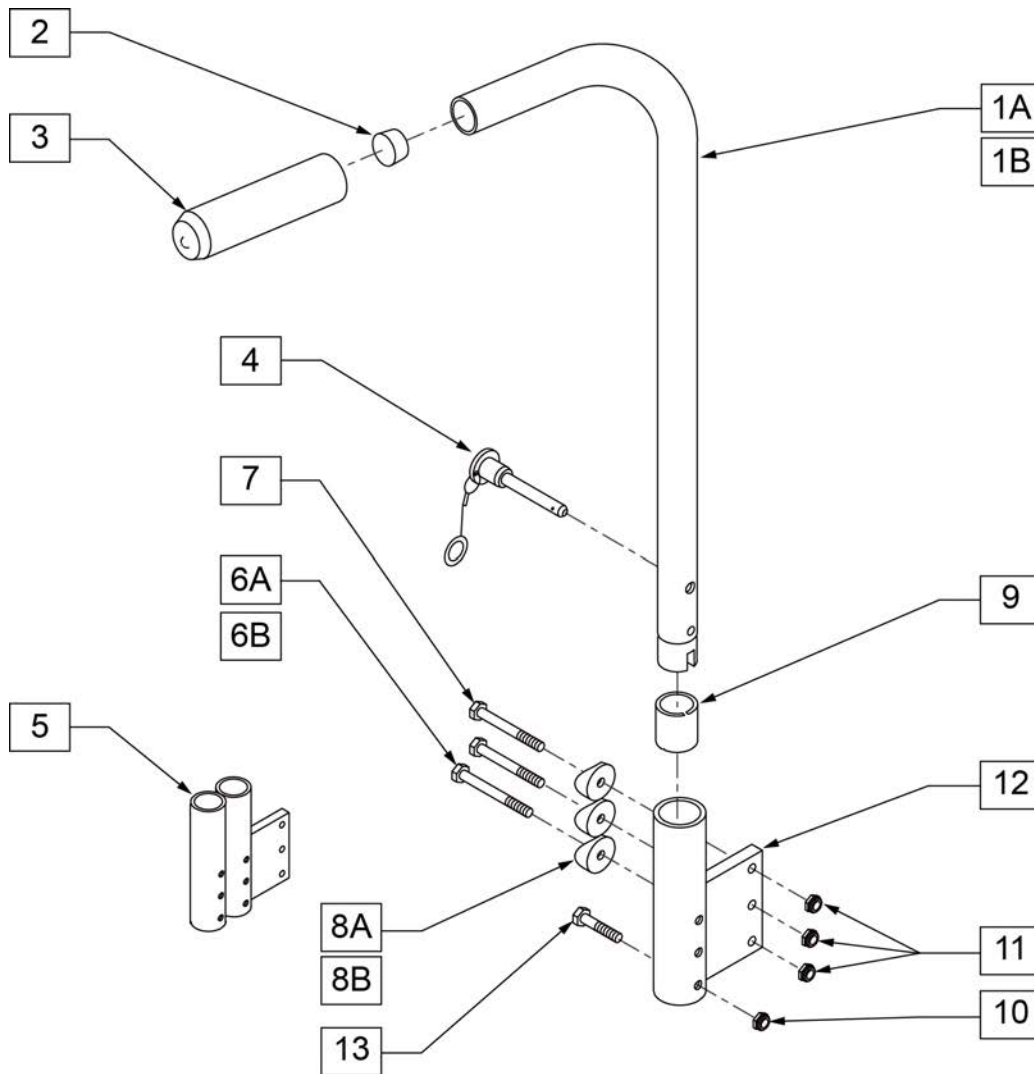


MEASURING GUIDE

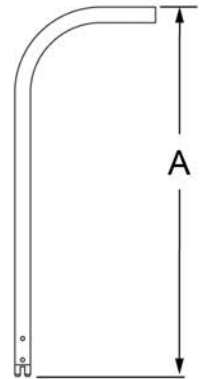

BACK TUBE	"A"
LOW	21 1/4"
MED	25"
TALL	28 1/2"

Pos.	Item Number	Description	Remarks
2	561002	1" INSERT PLUG .083	
3	920070	9 x .940 x .200 FOAM GRIP	
4	349020	STROLLER HANDLE RELEASE PIN	
5	540130	PADDED ARM RECEIVER STRAIGHT DOUBLE FOLDING FRAME	RIGHT
5	540132	PADDED ARM RECEIVER STRAIGHT DOUBLE FOLDING FRAME	LEFT
6A	106973	1/4-28 x 1 3/4 G8 HEX C/S	
6B	107351	1/4-28 x 2 G8 HEX C/S	
7	107351	1/4-28 x 2 1/4 G8 HEX C/S	
8A	561004	1" SADDLE 1/4 HOLE	
8B	561001	1 - 1/8 SADDLE 1/4 HOLE	
9	560400	1" I.D. NY-LINER	
10	106935	NUT NYLOCK THIN 1/4 -28 ROHS	
11	106934	1/4-28 NYLOCK NUT	
12	540100	PADDED ARM RECEIVER STRAIGHT FOLDING FRAME	RIGHT
12	540102	PADDED ARM RECEIVER STRAIGHT FOLDING FRAME	LEFT
13	106972	1/4-28 x 1 1/2 G8 HEX C/S	
14	341200	1/4 x 1 ROLL PIN	
15	483029	PUSH HANDLE PLUG	

Pos.	Item Number	Description	Remarks
1A-3, 14, 15	880360	STROLLER HANDLE REGULAR STRAIGHT FOLDING	
1B-3, 14, 15	880362	STROLLER HANDLE TALL STRAIGHT FOLDING	
1A-3, 14, 15	900420	STROLLER HANDLES REGULAR FOLDING	PAIR
1B-3, 14, 15	900421	STROLLER HANDLES TALL FOLDING	PAIR
4, 6-13	900500	STROLLER RECEIVER SINGLE CYL F 3/4	PAIR
4, 6B, 7, 8B, 9-13	900502	STROLLER RECEIVER SINGLE CYL FOLDING FRAME 7/8	PAIR
4-11, 13	900501	ARM/STROLLER RECEIVER DOUBLE CYL F 3/4	PAIR
4, 5, 6B, 7, 8B, 9-11, 13	900503	ARM/STROLLER RECEIVER DOUBLE CYL F 7/8	PAIR

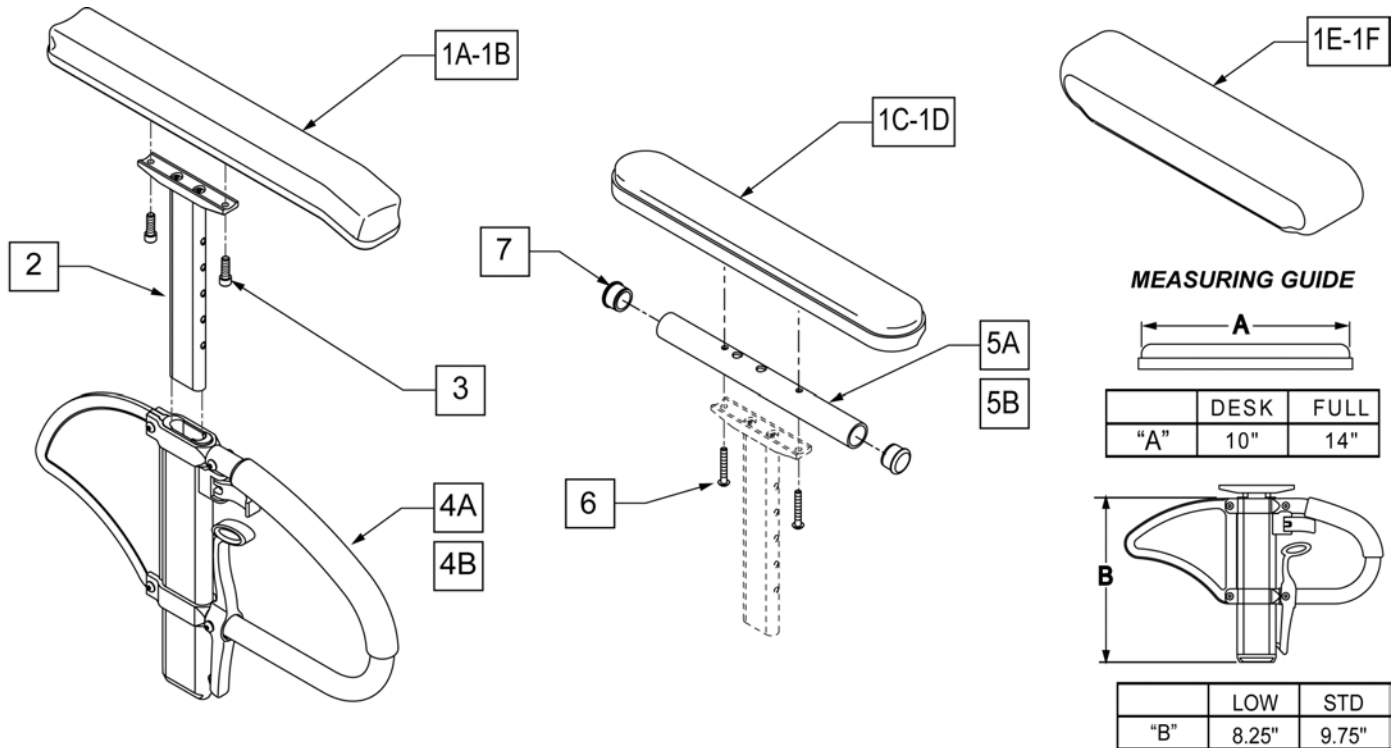

Pos.	Item Number	Description	Remarks
1A	104568-001	SPHA PAD DESK	CONTOUR DESK
1B	104568-002	SPHA PAD FULL	CONTOUR FULL
1C	561024	ARMREST PAD 10" DESK	CLASSIC DESK
1D	561031	ARMREST PAD 14" FULL	CLASSIC FULL
1E	561280	ARMREST PAD 10" LEFT W/F	Leftt-Waterfall Desk
1E	561281	ARMREST PAD 10" RIGHT W/F	Rightt-Waterfall Desk
1F	561284	ARMREST PAD 14" LEFT W/F	Leftt-Waterfall Full
1F	561285	ARMREST PAD 14" RIGHT W/F	Rightt-Waterfall Full
2	116312	SNGL POST UPPR ASSM LOW	LOW/SMALL
2	116313	SNGL POST UPPR ASSM STD	STANDARD
3	106956	SHCS 10-32 X 3/4 ROHS	
2,4A	104509-001	SPHA ARMREST ASSY STD	STANDARD
2,4B	104509-002	SPHA ARMREST ASSY LOW	LOW/SMALL
5A	521812	HT ADJ PD TUBE 10IN DESK POWE	Desk
5B	521813	HT ADJ PD TUBE 14IN FULL POWE	Full
6	107024	BHCS 10-32 X 1 1/2 ROHS	
7	560735	7/8" ROUND TUBE PLUG	
1C-4A-5A,6,7	108599	SPHA STD CLASIC DESK PAD R	Right
1C-4A-5A,6,7	108603	SPHA STD CLASIC DESK PAD L	Left
1D-4A-5B-7	108601	SPHA STD CLASIC FULL PAD R	Right
1D-4A-5B-7	108605	SPHA STD CLASIC FULL PAD L	Left
1A-4A	108595	SPHA STD CNTUR DESK PAD	
1B-4A	108597	SPHA STD CNTUR FULL PAD	
1C-4B-5A,6,7	108600	SPHA LOW CLASIC DESK PAD R	Right
1C-4B-5A,6,7	108604	SPHA LOW CLASIC DESK PAD L	Left
1D-4B-5B-7	108602	SPHA LOW CLASIC FULL PAD R	Right
1D-4B-5B-7	108606	SPHA LOW CLASIC FULL PAD L	Left
1A-4B	108596	SPHA LOW CNTUR DESK PAD	

Pos.	Item Number	Description	Remarks
1B-4B	108598	SPHA LOW CNTUR FULL PAD	

NOTE: USED ON QUICKIE 2 CHAIRS W/ S/N PREFIX Q2V.


NOTE: USED ON QUICKIE 2 LITE CHAIRS.

NOTE: USED ON ZIPPIE TS CHAIRS W/ S/N PREFIX Z1A & Z1SE.


NOTE: USED ON ZIPPIE GS CHAIRS W/ S/N PREFIX ZGS & ZGSE.

NOTE: USED ON ZIPPIE 2 CHAIRS W/ S/N PREFIX Z2A.


NOTE: CLAMP MOUNTS TO 1" TUBING.

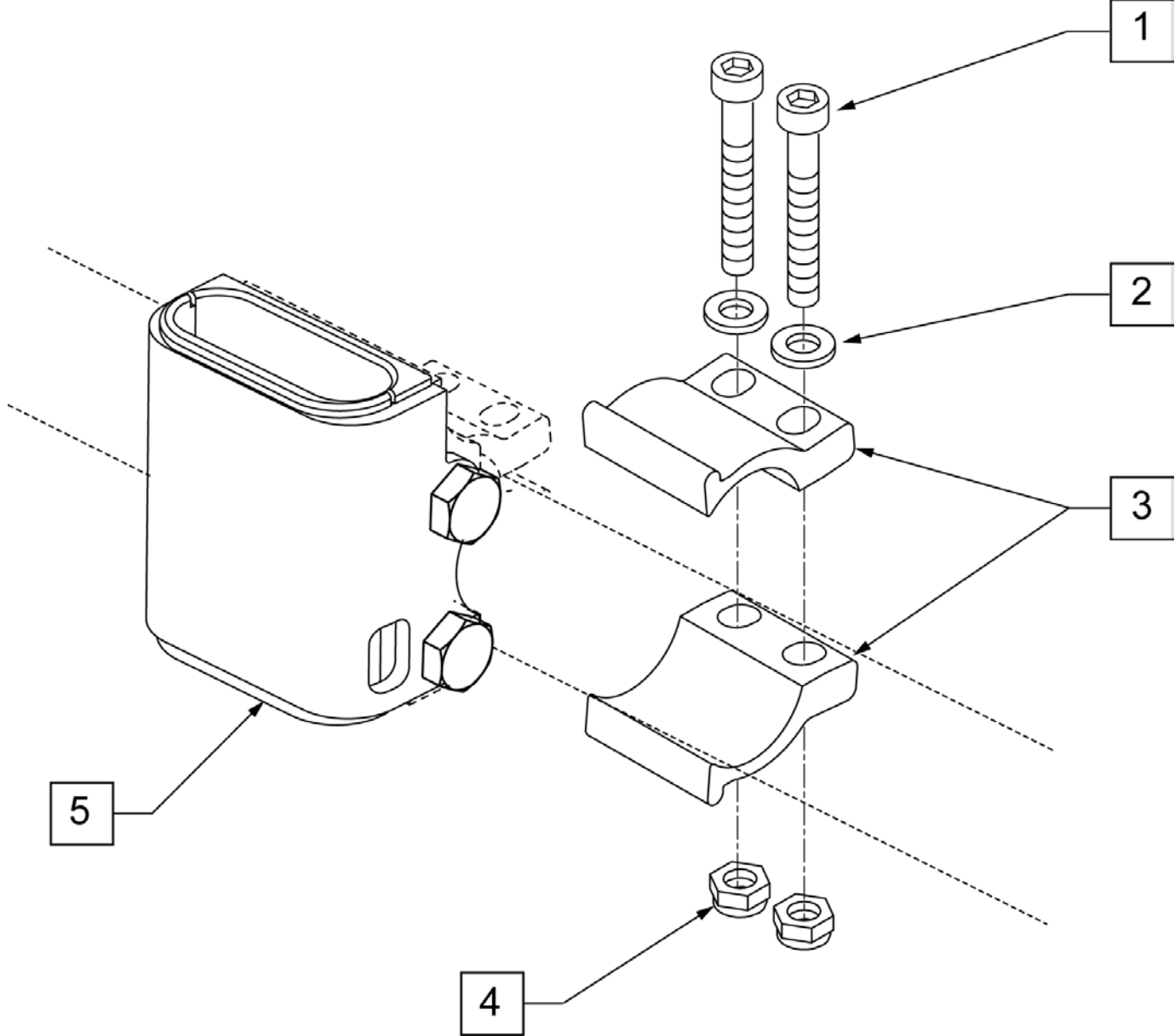


Pos.	Item Number	Description
1	106975	SHCS 1/4 -28 x 1 1/4 ROHS
2	106943	WASHER FLAT AN 1/4 ROHS
3	107123	CLAMP, RECEIVER, 1" OD
4	106935	NUT NYLOCK THIN 1/4 -28 ROHS
5	107124	RECEIVER, SPHA 1" OD
1-5	108101	SPHA RECEIVER W/ CLAMP KIT

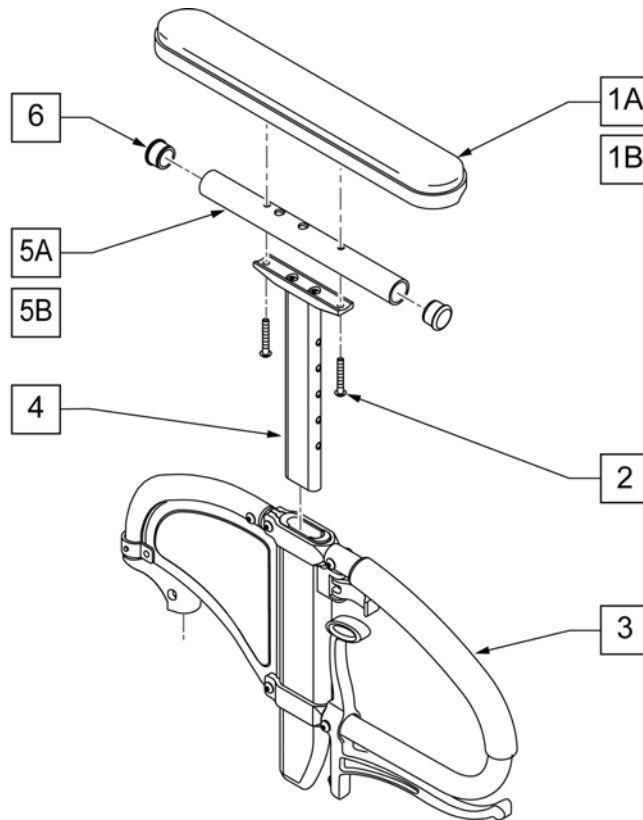

MEASURING GUIDE

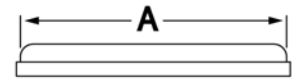


	DESK	FULL
A	10"	14"

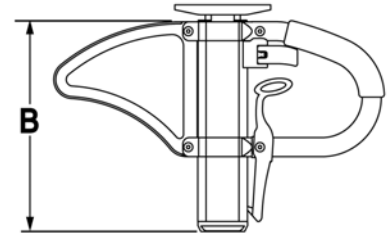


	STD
B	9.75"

Pos.	Item Number	Description	Remarks
1A	561024	ARMREST PAD 10" DESK	CLASSIC DESK
1B	561031	ARMREST PAD 14" FULL	CLASSIC FULL
2	107024	10-32 X 1 1/2 BH C/S BLK	
3,4	109405	SPHA ARMREST ASSM RIGHT	RIGHT
3,4	109400	SPHA ARMREST ASSM LEFT	LEFT
4	116313	SPHA ARM PAD MOUNT	
5A	521812	HT ADJ PD TUBE 10IN DESK POWE	
5B	521813	HT ADJ PD TUBE 14IN FULL POWE	
6	560735	7/8" ROUND TUBE PLUG	
1A,2,4,5A,6	115221	SPHA UPPER CLASSIC DESK PAD	Classic-Desk
1B,2,4,5B,6	115222	SPHA UPPER CLASSIC FULL PAD	Classic-Full
1A-5A,6	117834	SPHA FLP BCK LITE ARM DESK R	RIGHT
1A-5A,6	117835	SPHA FLP BCK LITE ARM DESK L	LEFT
1B-5B,6	117836	SPHA FLP BCK LITE ARM FULL R	RIGHT
1B-5B,6	117837	SPHA FLP BCK LITE ARM FULL L	LEFT

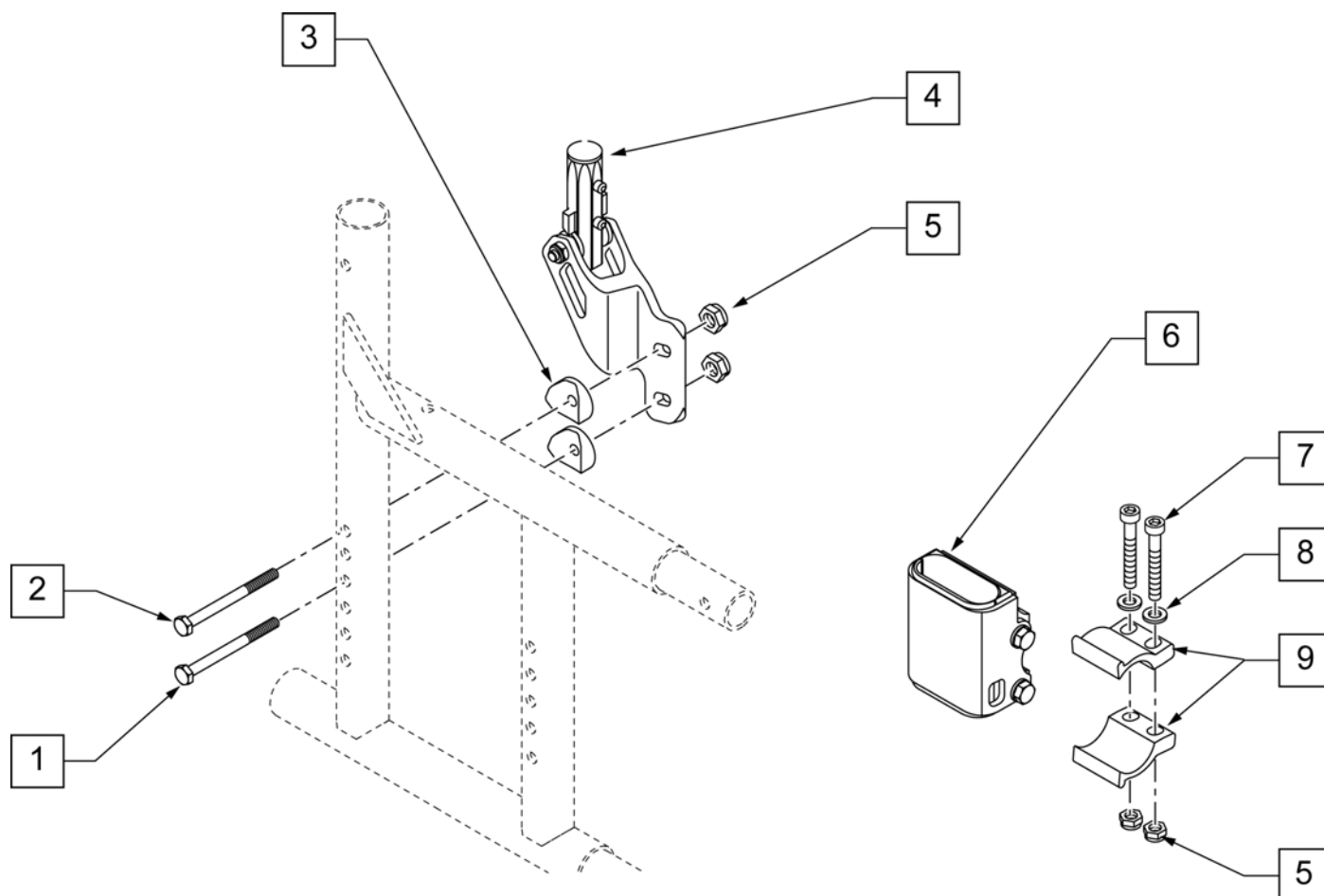


Pos.	Item Number	Description	Remarks
1	107351	HHCS 1/4 -28 x 2 1/4 ROHS	
2	106973	HHCS 1/4 -28 x 2 ROHS	
3	561001	1 -1/8" SADDLE 1/4" HOLE	
4	109438	SPHA FB RECEIVER ASSM HI R	RIGHT
4	109437	SPHA FB RECEIVER ASSM HI L	LEFT
5	106935	NUT NYLOCK THIN 1/4 -28 ROHS	
6	107124	SPHA RECEIVER KIT	
7	106975	SHCS 1/4 -28 x 1 1/4 ROHS	
8	106943	WASHER FLAT AN 1/4 ROHS	
9	107123	CLAMP, RECEIVER, 1" OD	
5-9	108101	SPHA RECIEVER W/ CLAMP KIT	
1-5	120655	REAR RECIVER ASSM - RIGHT	RIGHT
1-5	120656	REAR RECEIVER ASSM - LEFT	LEFT

NOTE: USED ON QUICKIE 2 CHAIRS W/ S/N PREFIX Q2V.

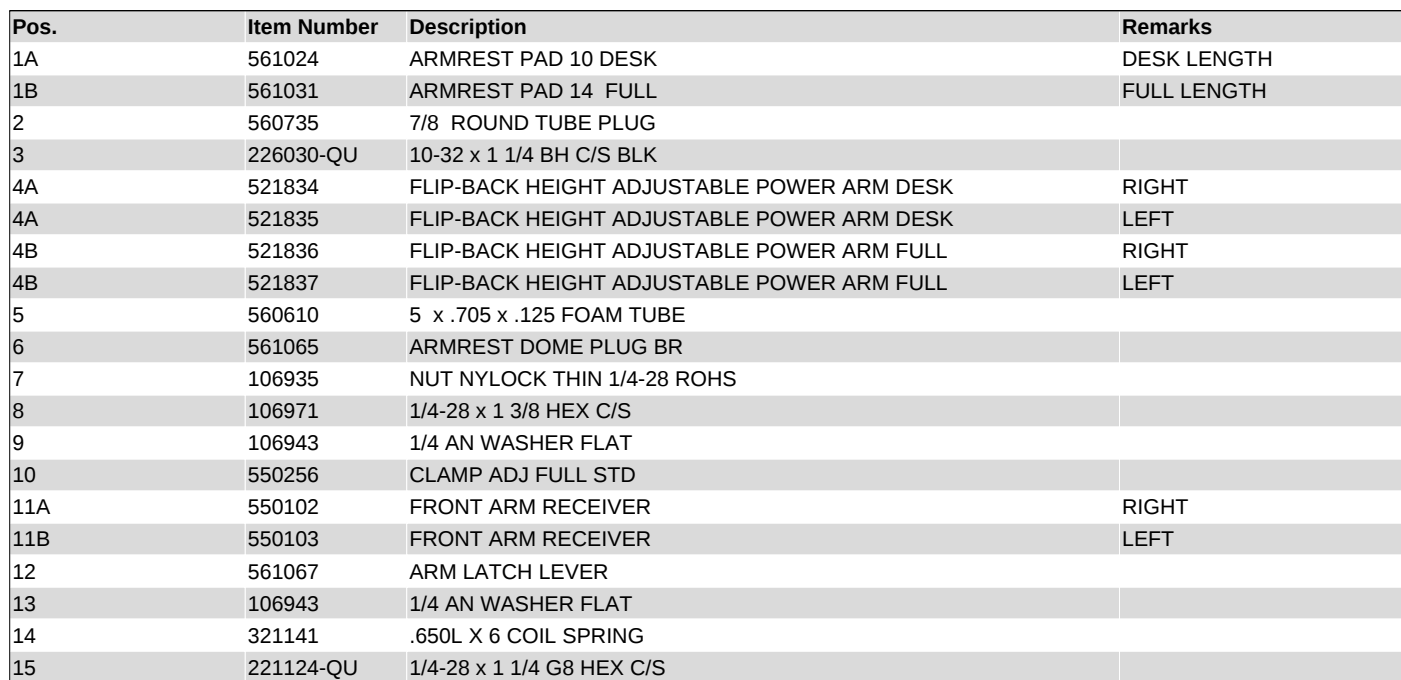


Pos.	Item Number	Description	Remarks
26	561080	ARM RECEIVER BR	
27	106972	1/4-28 X 1 1/2 G8 HEX C/S	
28	107015	1/4-28 x 1 3/4 G8 HEX C/S	
29	106973	1/4-28 x 2 G8 HEX C/S	
30	106934	1/4-28 NYLOCK NUT	
31	505352	FLIP-BACK ARM PIVOT BRACKET	
8-11A	880106	FLIP-BACK FRONT RECEIVER W/CLAMP	RIGHT
8-11B	880107	FLIP-BACK FRONT RECEIVER W/CLAMP	LEFT
23-25,27,30	880261	RECEIVER POST W/BUTTON	
1A-4A	880390	FLIP-BACK HEIGHT ADJUSTABLE UPPER ARM W/DESK	RIGHT - DESK LENGTH
1A-4A	880391	FLIP-BACK HEIGHT ADJUSTABLE UPPER ARM W/DESK	LEFT - DESK LENGTH
1B-4B	880392	FLIP-BACK HEIGHT ADJUSTABLE UPPER ARM ASSM FULL	RIGHT - FULL LENGTH
1B-4B	880393	FLIP-BACK HEIGHT ADJUSTABLE UPPER ARM ASSM FULL	LEFT - FULL LENGTH
5-7,12-21,22A	880288	FLIP-BACK HEIGHT ADJUSTABLE LOWER ARM ASSM	RIGHT
5-7,12-21,22B	880289	FLIP-BACK HEIGHT ADJUSTABLE LOWER ARM ASSM	LEFT
1A-4A-7,12-21,22A	880296	FLIP-BACK HEIGHT ADJUSTABLE ARM ASSM DESK	RIGHT - DESK LENGTH
1A-4A-7,12-21,22B	880297	FLIP-BACK HEIGHT ADJUSTABLE ARM ASSM DESK	LEFT - DESK LENGTH
1B-4B-7,12-21, 22A	880396	FLIP-BACK HEIGHT ADJUSTABLE ARM ASSM FULL	RIGHT - FULL LENGTH
1B-4B-7,12-21, 22B	880397	FLIP-BACK HEIGHT ADJUSTABLE ARM ASSM FULL	LEFT - FULL LENGTH
1A-4A-31	923220	FLIP-BACK HEIGHT ADJUSTABLE DESK ARM RETRO	PAIR - DESK LENGTH
1B-4B-31	923221	FLIP-BACK HEIGHT ADJUSTABLE FULL ARM RETRO	PAIR - FULL LENGTH
26, 27, 30	880009	RETRO KIT BR FLIP-BACK HEIGHT ADJUSTABLE ARM	
23-26,28-31	880109	FLIP-BACK HEIGHT ADJUSTABLE RR REC ASSM W/PST	

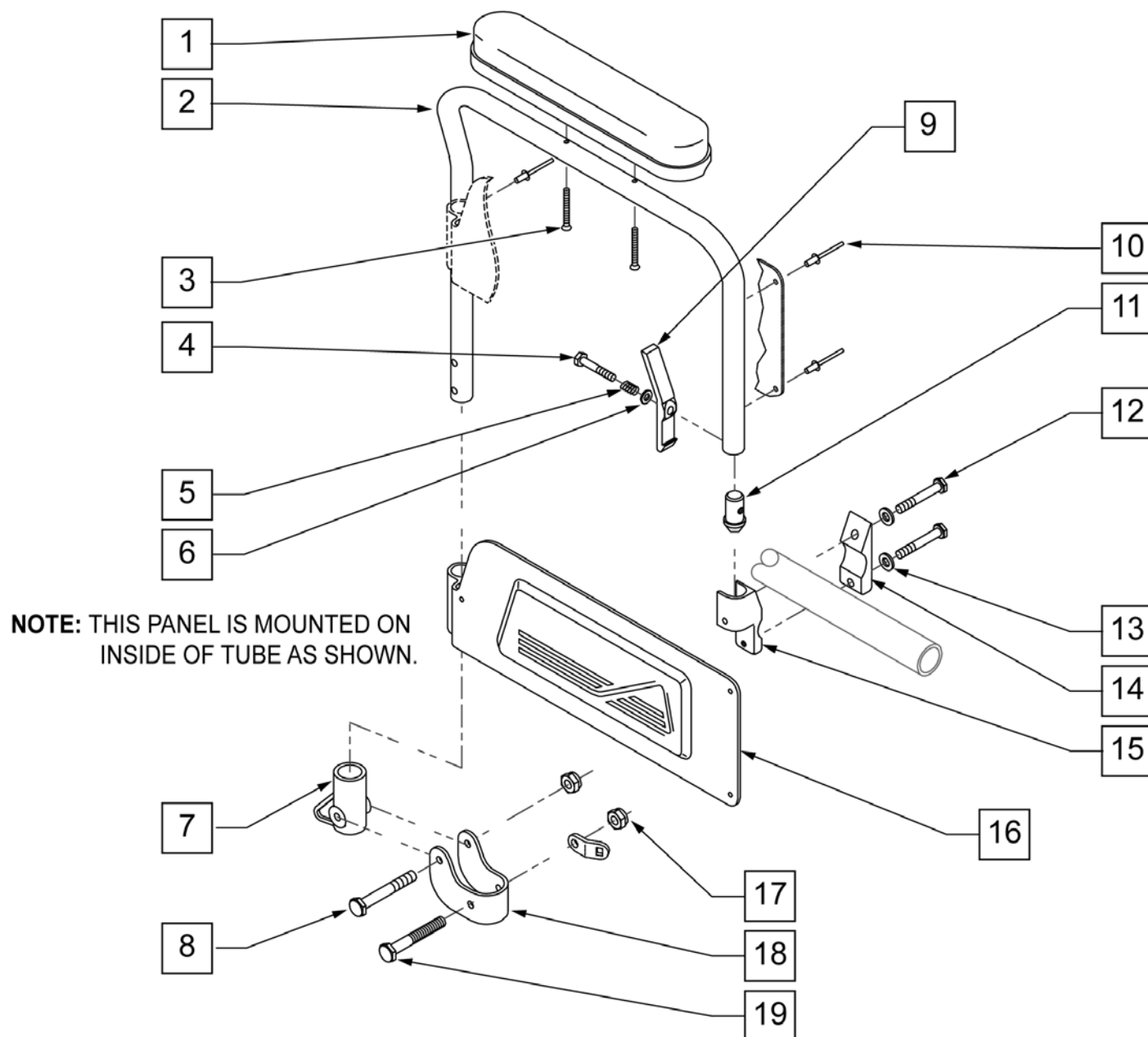


Pos.	Item Number	Description	Remarks
1	561024	ARMREST PAD 10" DESK	
3	226030-QU	10-32 x 1 1/4 BH C/S BLK	
4	221124-QU	1/4-28 x 1 1/4 G8 HEX C/S	
5	321141	.650L x 6 COIL SPRING	
6	106944	1/4 AN WASHER FLAT THIN	
7	561080	ARM RECEIVER BRACKET	
8	107015	1/4-28 x 1 3/4 G8 HEX C/S	
9	561067	ARM LATCH LEVER	
11	561075	ARM DOME PLUG BZ	
12	106971	1/4-28 x 1 3/8 HEX C/S	

Pos.	Item Number	Description	Remarks
13	106943	1/4 AN WASHER FLAT	
14	550256	CLAMP ADJ FL STANDARD	
15	550102	FRONT ARM RECEIVER	RIGHT
15	550103	FRONT ARM RECEIVER	LEFT
17	106934	1/4-28 NYLOCK NUT	
18	505352	FLIP-BACK ARM PIVOT BRACKET	
19	106973	1/4-28 x 2 G8 HEX C/S	
12-15	880106	FLIP-BACK FRONT RECEIVER W/CLAMP	RIGHT
12-15	880107	FLIP-BACK FRONT RECEIVER W/CLAMP	LEFT
7,17-19	880108	FLIP-BACK ARM REAR RECEIVER W/ BRACKET	
1-11,16-19	880256	FLIP-BACK ARM ASSM DESK Q2	RIGHT
1-11,16-19	880257	FLIP-BACK ARM ASSM DESK Q2	LEFT
1-19	880258	FLIP-BACK ARM ASSM DESK W/RECEIVER Q2	RIGHT
1-19	880259	FLIP-BACK ARM ASSM DESK W/RECEIVER Q2	LEFT
1-19	900695	FLIP-BACK ARM ASSM DESK W/RECEIVER Q2	PAIR

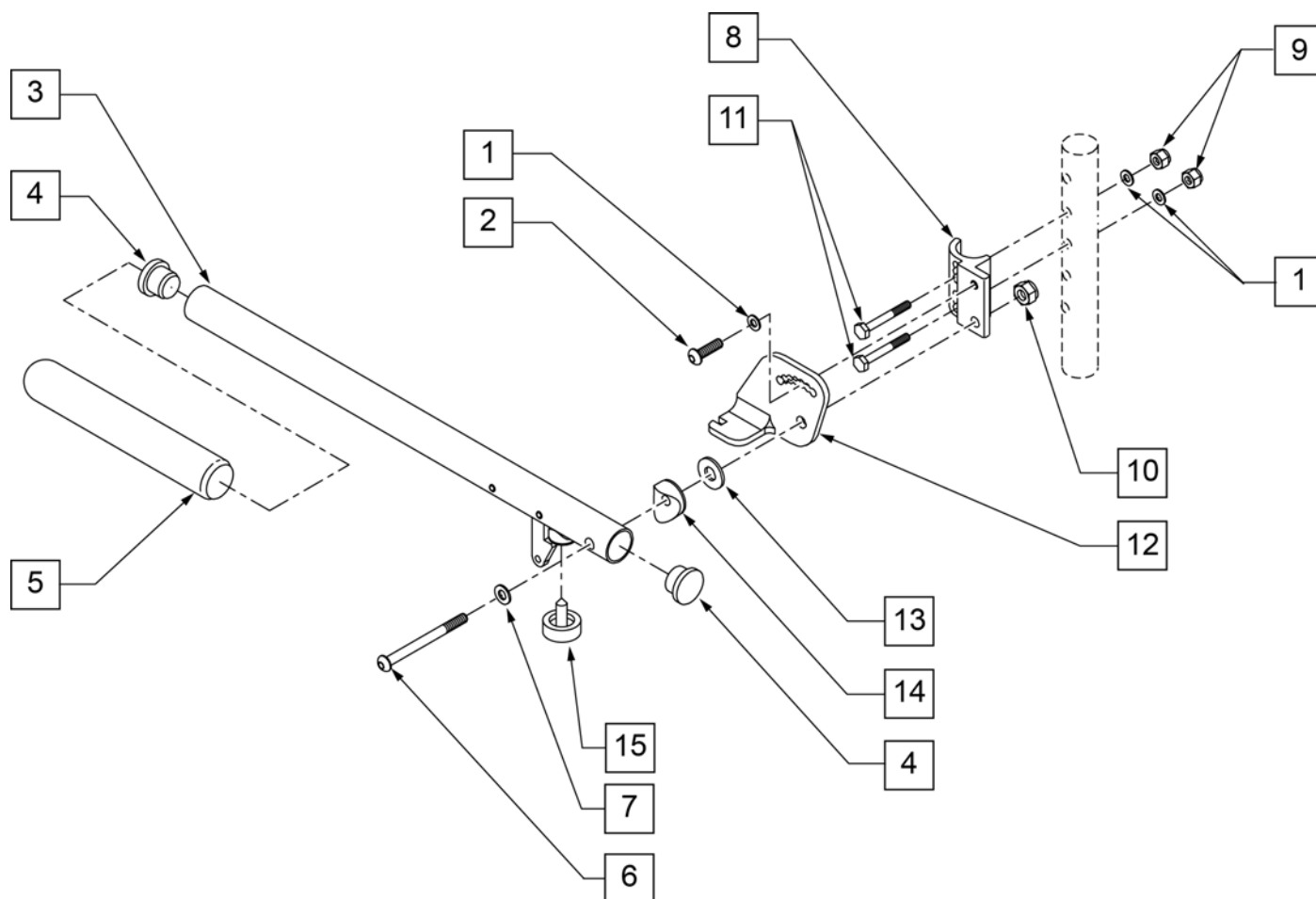

Pos.	Item Number	Description	Remarks
1	106943	1/4 AN WASHER FLAT	
2	107025	1/4-28 x 1/2 BH C/S	
4	561002	1" INSERT PLUG .083	
5	560640	9 x .93 x .155 FOAM GRIP	
6	106983	5/16-18 x 2 BH C/S SHT THREAD	
7	106947	5/16 WASHER FLAT THIN	
8	550168	ADJ LOCK FLIP-UP ARM MNT R	
8	550169	ADJ LOCK FLIP-UP ARM MNT L	
9	106935	1/4-28 NYLOCK NUT THIN	
10	202117	5/16-18 NYLOCK NUT THICK BLK	
11	106972	1/4-28 x 1 1/2 G8 HEX C/S	
12	550474	ADJ FLIP-BACK ARM BRACKET R	
12	550475	ADJ FLIP BACK ARM BRACKET L	
13	561016	1" WASHER 5/16" HOLE	
14	561006	1" SADDLE 5/16" HOLE	
15	560252	1" COVED STEM BUMPER	
3-5	880274	ADJ FLIP-BACK TUBE W/LATCH	
1,2,6,8-10	881272	ADJ LOCK F-UP ARMREST RETRO R	USE FOR ARMREST PRIOR TO 7/22/96
1,2,6,8-10	881273	ADJ LOCK F-UP ARMREST RETRO L	USE FOR ARMREST PRIOR TO 7/22/96
1-15	880272	ADJ FLIP BACK ARMREST R	
1-15	880273	ADJ FLIP BACK ARMREST L	
1-15	900449	ADJ LOCK FLIP-UP ARMREST ASSM	PAIR

NOTE: FOR ARMRESTS PRIOR TO 7/22/96 USE RETRO FIT KIT TO REPLACE ADJUSTABLE FLIP-UP ARM CLAMP.

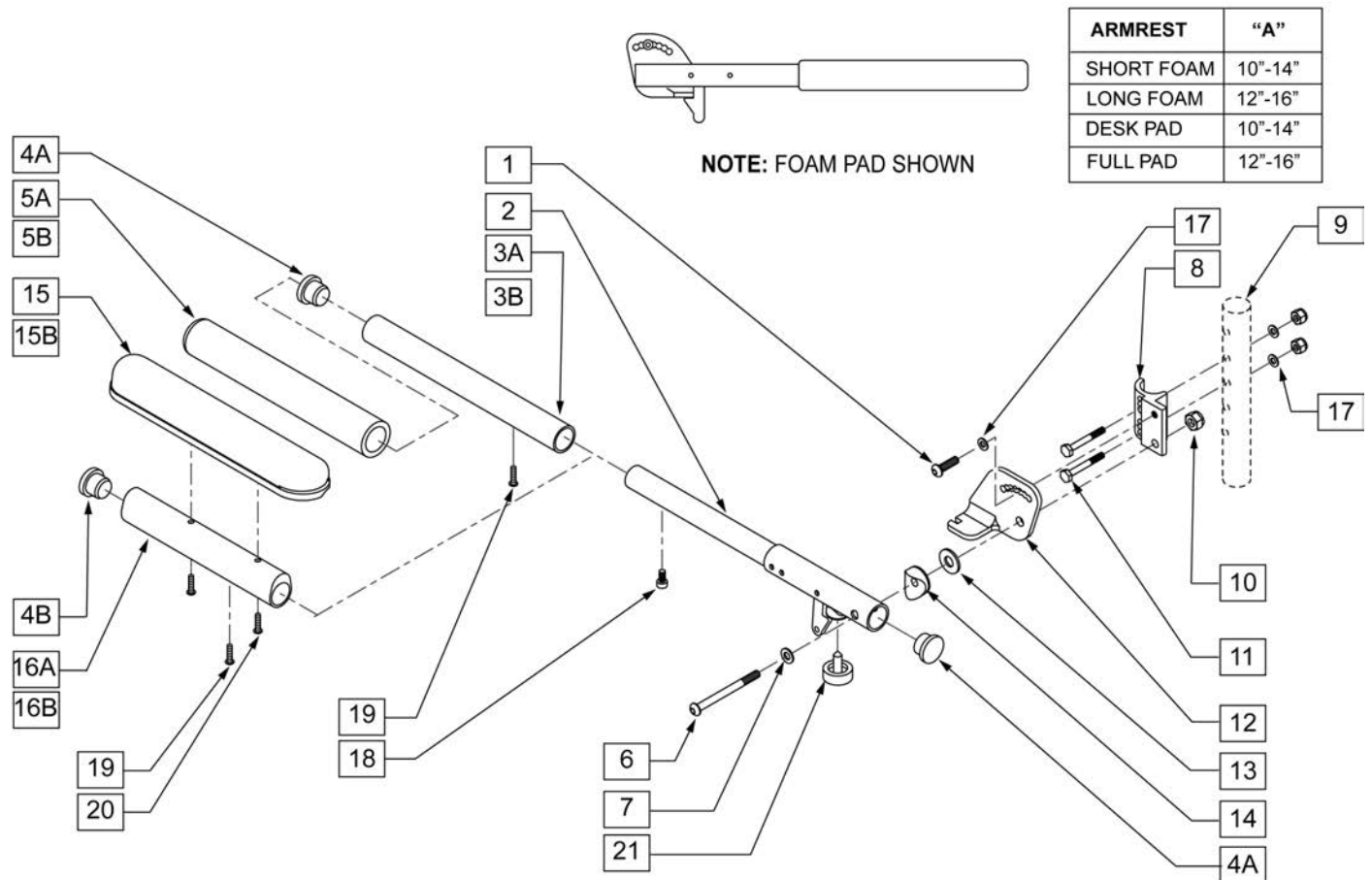


Pos.	Item Number	Description	Remarks
1	107025	BHCS 1/4-28 x 1/2 ROHS	
4A	561002	1" INSERT PLUG .083	
4B	561062	1" INSERT PLUG .065	
5A	560299	7 x .940 x .20 FOAM GRIP	SHORT PAD
5B	560640	9" X .93" X .155" FOAM GRIP	LONG PAD
6	106983	BHCS 5/16-18 x 2 SHLDR ROHS	
7	106947	WASHER FLAT THIN AN 5/16 ROHS	
8	550168	ADJUSTABLE LOCK FLIP-UP ARM MOUNT	RIGHT
8	550169	ADJUSTABLE LOCK FLIP-UP ARM MOUNT	LEFT
9	106935	NYLOCK NUT THIN 1/4 -28 ROHS	
10	202117	5/16-18 NYLOCK NUT THICK BLK	
11	106972	HHCS 1/4-28 x 1 1/2 ROHS	
12	550474	ADJUSTABLE FLIP-BACK ARM BRACKET	RIGHT
12	550475	ADJUSTABLE FLIP-BACK ARM BRACKET	LEFT
13	561016	1" WASHER 5/16" HOLE	
14	561006	1" SADDLE 5/16" HOLE	
15A	561024	ARMREST PAD 10" DESK	

Pos.	Item Number	Description	Remarks
15B	561031	ARMREST PAD 14" FULL	
17	106943	WASHER FLAT AN 1/4 ROHS	
18	107036	SHCS 8 -32 x 3/8 ROHS	
19	225110	10-32 X 1/2" SET SCREW	
20	107022	BHCS 10-32 X 5/8 ROHS	
21	560252	1" COVED STEM BUMPER	
3B, 4A, 5B, 19	882074	FOAM PAD LONG ADJUSTABLE LENGTH	LONG PAD ASSM ONLY
3A, 4A, 5A, 19	882075	FOAM PAD LONG ADJUSTABLE LENGTH	SHORT PAD ASSM ONLY
4B,15B,16B,19,20	882070	FULL PAD ADJUSTABLE LENGTH ASSM	RIGHT FULL PAD ASSM ONLY
4B,15B,16B,19,20	882071	FULL PAD ADJUSTABLE LENGTH ASSM	LEFT FULL PAD ASSM ONLY
4B,15A,16A,19,20	882072	DESK PAD ADJUSTABLE LENGTH ASSM	RIGHT DESK PAD ASSM ONLY
4B,15A,16A,19,20	882073	DESK PAD ADJUSTABLE LENGTH ASSM	LEFT DESK PAD ASSM ONLY
1,2,6-14,17,22	882088	ADJ LENGTH LOCKING FLIP-UP ASSM R	RIGHT ARM ASSM W/O PAD
1,2,6-14,17,22	882089	ADJ LENGTH LOCKING FLIP-UP ASSM R	LEFT ARM ASSM W/O PAD
1,2,3B,4A,5B,6-14,17,19 -21	900454	ADJ LENGTH LOCKING FLIP-UP FOAM LONG	PAIR - LONG FOAM ARM ASSM
1,2,3B,4A,5B,6-14,17,19 -21	881294	ADJ LENGTH LOCKING FLIP-UP FOAM LONG	RIGHT - LONG FOAM ARM ASSM
1,2,3B,4A,5B,6-14,17,19 -21	881295	ADJ LENGTH LOCKING FLIP-UP FOAM LONG	LEFT - LONG FOAM ARM ASSM
1,2,3A,4A,5A,6-14,17,19 -21	900455	ADJ LENGTH LOCKING FLIP-UP FOAM SHORT	PAIR - SHORT FOAM ARM ASSM
1,2,3A,4A,5A,6-14,17,19 -21	881296	ADJ LENGTH LOCKING FLIP-UP FOAM SHORT	RIGHT - SHORT FOAM ARM ASSM
1,2,3A,4A,5A,6-14,17,19 -21	881297	ADJ LENGTH LOCKING FLIP-UP FOAM SHORT	LEFT - SHORT FOAM ARM ASSM
1,2,4B,6-14,15B,16B,17-21	900452	ADJ LENGTH LOCKING FLIP-UP ASSM FULL	PAIR - FULL PAD ARM ASSM
1,2,4B,6-14,15B,16B,17-21	881290	ADJ LENGTH LOCKING FLIP-UP FULL PAD	RIGHT - FULL PAD ARM ASSM
1,2,4B,6-14,15B,16B,17-21	881291	ADJ LENGTH LOCKING FLIP-UP FULL PAD	LEFT - FULL PAD ARM ASSM
1,2,4B,6-14,15A,16A,17-21	900453	ADJ LENGTH LOCKING FLIP-UP ASSM DESK	PAIR - DESK PAD ARM ASSM
1,2,4B,6-14,15A,16A,17-21	881292	ADJ LENGTH LOCKING FLIP-UP DESK PAD	RIGHT - DESK PAD ARM ASSM
1,2,4B,6-14,15A,16A,17-21	881293	ADJ LENGTH LOCKING FLIP-UP DESK PAD	LEFT - DESK PAD ARM ASSM


Note: When adding the Yamaha Xtender to an existing Quickie 2 chair, the rear side frames can not be curved, they must be straight as shown.

Note: The Yamaha Xtender is not transit approved and can not be added to a chair that is transit approved.

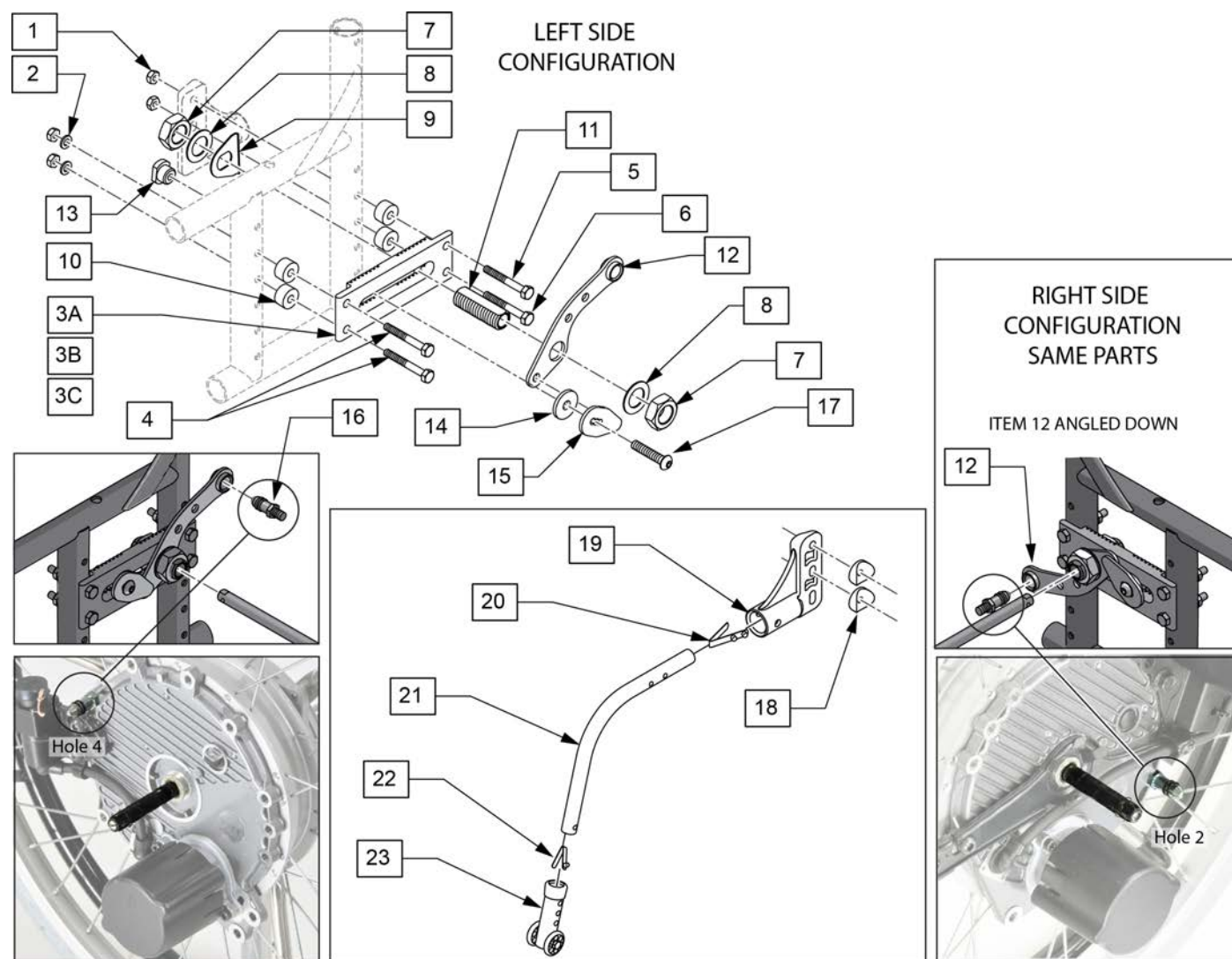

Pos.	Item Number	Description	Remarks
1	106934	1/4-28 NYLOCK NUT	
2	106943	WASHER FLAT AN 1/4 ROHS	
3A	502000	AXLE PLATE Q2/HP/ZP2 INDEX	
3B	502018	CURVED PLATE Q2/HP/ZP2 INDEX	
3C	550645	OFFSET AXLE PLATE	
4	109265	HHCS 1/4-28 X 2-1/2	
5	221201	1/4-28 X 3 G8 HEX C/S	
6	107264	HHCS 1/4-28 X 2-3/4 PL ROHS	
7	116569	NUT HEX 3/4-16 BLACK ZINC	
8	116570	WASHER FLAT 3/4 X 1.25 BLK ZNC	

Pos.	Item Number	Description	Remarks
9	505015	INDEX AXLE WASHER	
10	330031	SUSPENSION SPACER 1/2" P3	
11-17	125331	XTENDER UNIVERSAL BRACKET KIT	
18	561001	1 1/8" SADDLE 1/4" HOLE	
19	920407	ANTI-TIP RECEIVER PR	PAIR
20	301109	SNAP BUTTON DUAL ONE SIDE	
22	301100	1/4" SNAP BTTN SINGLE	
23	881513	ANTI-TIP SLV W/WHLS BLK - 5H (EA)	
19-23	881128	MED ANTI TIP ASSM RT BLK	
19-23	881129	MED ANTI TIP ASSM LT B	
19-22	883008	MED ANTI TIP TUBE R W/ SNAP BUTTON	RIGHT
19-22	883009	MED ANTI TIP TUBE L W/ SNAP BUTTON	LEFT

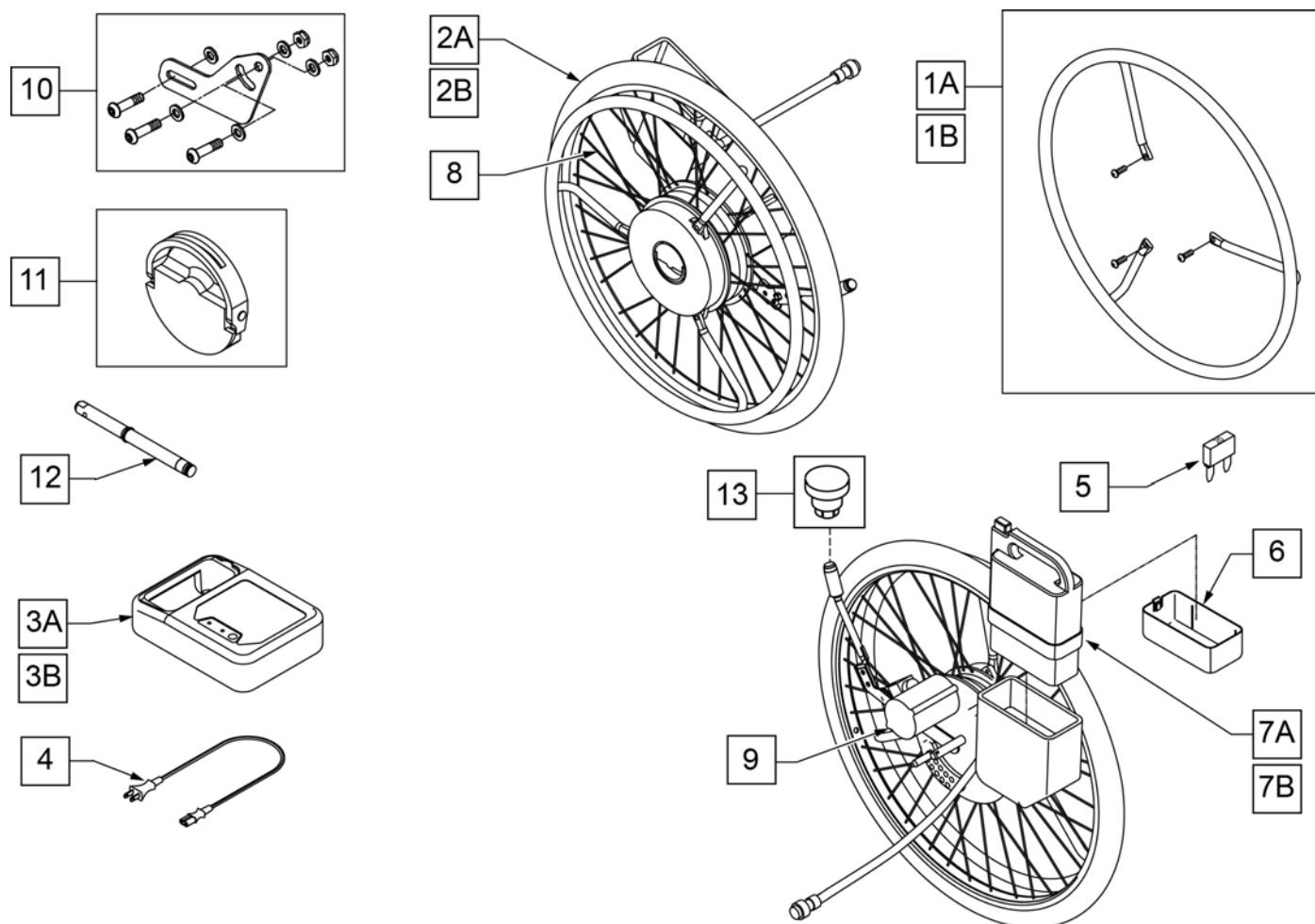


Pos.	Item Number	Description	Remarks
1A	425060	24" XTENDER HANDRIM SS W/HDWR	1.5X Multiplier - Stainless Handrim
1B	425061	24"XTENDER HANDRIM PLST W/HDWR	3X Multiplier - Plastic Coated Handrim
2A	385300	24" PNEUMATIC TIRE	
2B	382325	24" TUBE	
3A,4	114474	CHARGER YAMAHA W/ ADDRESS LBL	Nickel Battery Charger
3B,4	120710	BATTERY CHARGER YAMAHA	Lithium Battery Charger
4	134121	PWR CORD BATT CHARGER YAMAHA	
5	138219	FUSE 20A BLADE	
5-7A	136526	BATTERY YAMAHA	Nickel Battery
5-7B	120649	POWER PACK YAMAHA SPARE LI	Lithium Battery
8	133108	SPOKE SET 24" YAMAHA	INCLUDES 36 SPOKES
9	490013	MOTOR COVER YAMAHA	
10	921763	XTNDR ADJ BRCKT KIT (W/HDWR)	
11	107011	YAMAHA PLATE ASSM KIT	
12	541155	XTNDR QUICK RELEASE AXLE	
13	108273	PWR BUTTON SWITCH	

[08/2013]

Note: If replacing or retrofitting an Xtender, please contact Customer Service to ensure compatibility.

Note: Battery Type :NiMH- Nickel Metal Hydride, Capacity: 24V x 6.7Ah , Weight 2.9 [kg], Fuse: 20A.

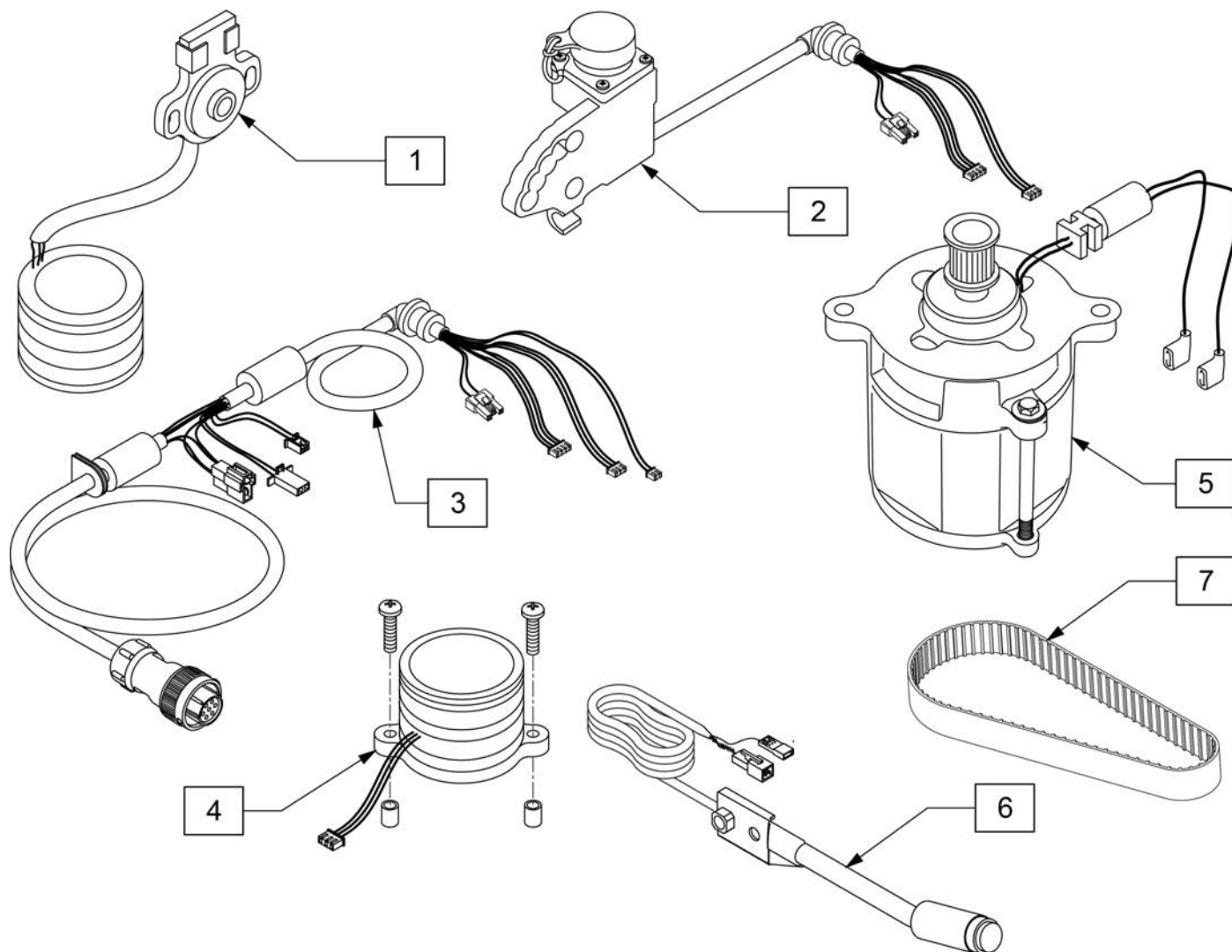


Pos.	Item Number	Description
1	107008	YAMAHA SENSOR KIT
2	107005	YAMAHA RECEPTICAL KIT
3	107010	YAMAHA COUPLER HRNS
4	107007	YAMAHA ROTARY TRANS KIT
5	107009	YAMAHA MOTOR REPLACEMENT KIT
6	107012	YAMAHA SWITCH KIT
7	107006	YAMAHA DRIVE BELT KIT

Note: When adding a Yamaha Xtender to Q2 or Q2HP please contact Customer Service to ensure configuration compatibility.

Note: When adding the Yamaha Xtender to an existing Quickie 2 chair, the rear side frames can not be curved, they must be straight as shown below.



Note: The Yamaha Xtender is not transit approved and can not be added to a chair that is transit approved.

Note: Q2 Axle Plate is not included, please order appropriate axle plate when necessary.

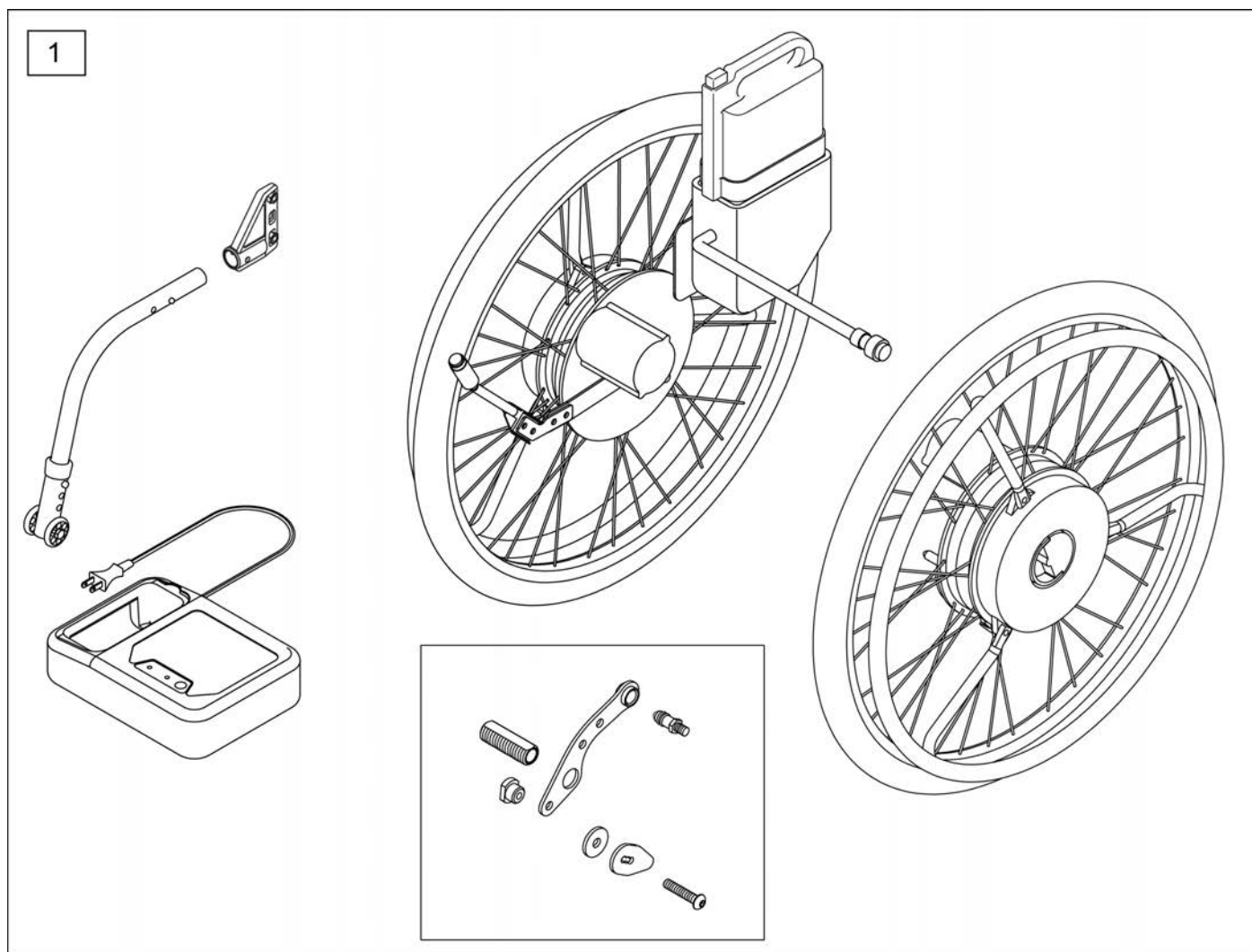

Pos.	Item Number	Description	Remarks
1	125772	24 XTNDR KIT 4X PNEU SS NI	Stainless Steel Handrim, Nickel Batt 4X
1	125773	24 XTNDR KIT 4X PNEU PLST NI	Plastic Coated Handrim, Nickel Batt 4X
1	125774	24 XTNDR KIT 4X PNEU SS LI	Stainless Steel Handrim, Lithium Batt 4X
1	125775	24 XTNDR KIT 4X PNEU PLST LI	Plastic Coated Handrim, Lithium Batt 4X


NOTE: WHEN ADDING A TRAY TO AN EXISTING CHAIR WITH DESK OR FULL LENGTH ARM PADS, ADD 1" TO CURRENT SEAT WIDTH TO ACHIEVE THE BEST TRAY SIZE.

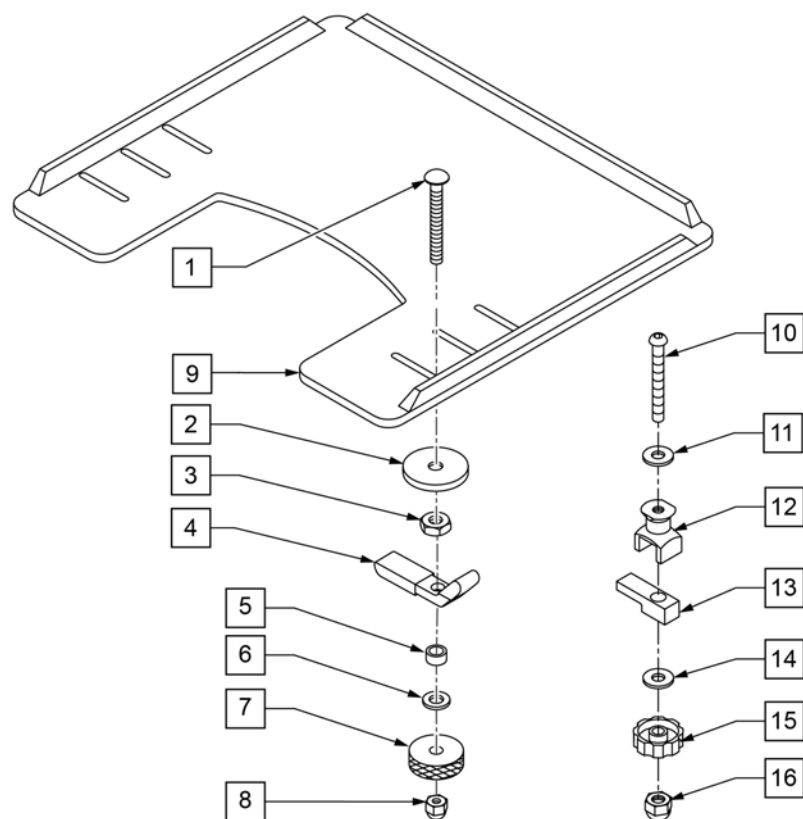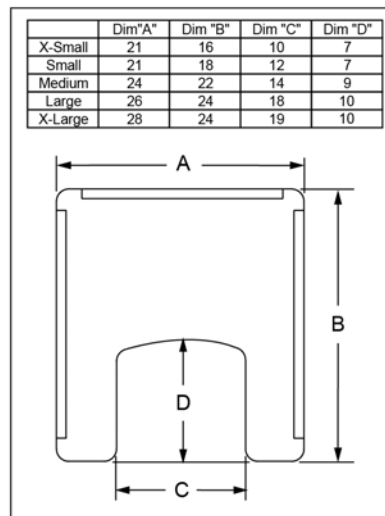**MEASURING GUIDE**

Pos.	Item Number	Description	Remarks
1	226199	5/16-18 x 3 1/2 CARR BOLT	
2	263112	5/16 WASHER FENDER	
3	202110	5/16-18 NUT THIN	
4	550400	TRAY CLAMP	
5	481113	SPACER 5/16 X .380	
6	263100	5/16 AN WASHER FLAT	
7	483026	KNURLED TRAY KNOB	
8	202112	5/16-18 ACORN NUT	
1-8	881051	TRAY HARDWARE KIT TUFFAK NEW STYLE	Pair
9A	562619	TRAY TOP EX-SM W/RAIL	
9B	562620	TRAY TOP SM W/RAIL	
9C	562621	TRAY TOP MED W/RAIL	
9D	562622	TRAY TOP LRG W/RAIL	
9E	562623	TRAY TOP X-LRG W/RAIL	
10-15	881053	TRAY HARDWARE ADJ ARM PWR (14219)	Pair
1-8,9A	902306	TRAY ASSEMBLY EX-SM TUFFAK (10 -12")	Fits 10"-12" wide chairs
1-8,9B	902303	TRAY ASSEMBLY SM TUFFAK (13 -14")	Fits 13"-14" wide chairs
1-8,9C	902304	TRAY ASSEMBLY MED TUFFAK (15 -17")	Fits 15"-17" wide chairs
1-8,9D	902305	TRAY ASSEMBLY LRG TUFFAK (18 -20")	Fits 18"-20" wide chairs
1-8,9E	902309	TRAY ASSEMBLY X-LRG TUFFAK (21 -22")	Fits 21"-22" wide chairs
9A,10-15	902330	TRAY ASSM X-SM TFK 10-12/CAM	Fits 10"-12" wide chairs


Pos.	Item Number	Description	Remarks
9B,10-15	902331	TRAY ASSM SM TUFFAK 13-14/CAM	Fits 13"-14" wide chairs
9C,10-15	902332	TRAY ASSM MED TUFFAK 15-17/CAM	Fits 15"-17" wide chairs
9D,10-15	902333	TRAY ASSM LRG TUFFAK 18-20/CAM	Fits 18"-20" wide chairs

NOTE: ONE 19" RIB AND TWO 15 1/2" RIBS FIT THE EXTRA-SMALL, SMALL, 9-0 AND 9-1 TRAY TABLES. ONE 21" RIB AND TWO 19" RIBS FIT THE MEDIUM AND 9-2 TRAY TABLES. ONE 23 1/2" RIB AND TWO 21" RIBS FIT THE LARGE, 9-2, 9-5, AND 9-6 TRAY TABLES.

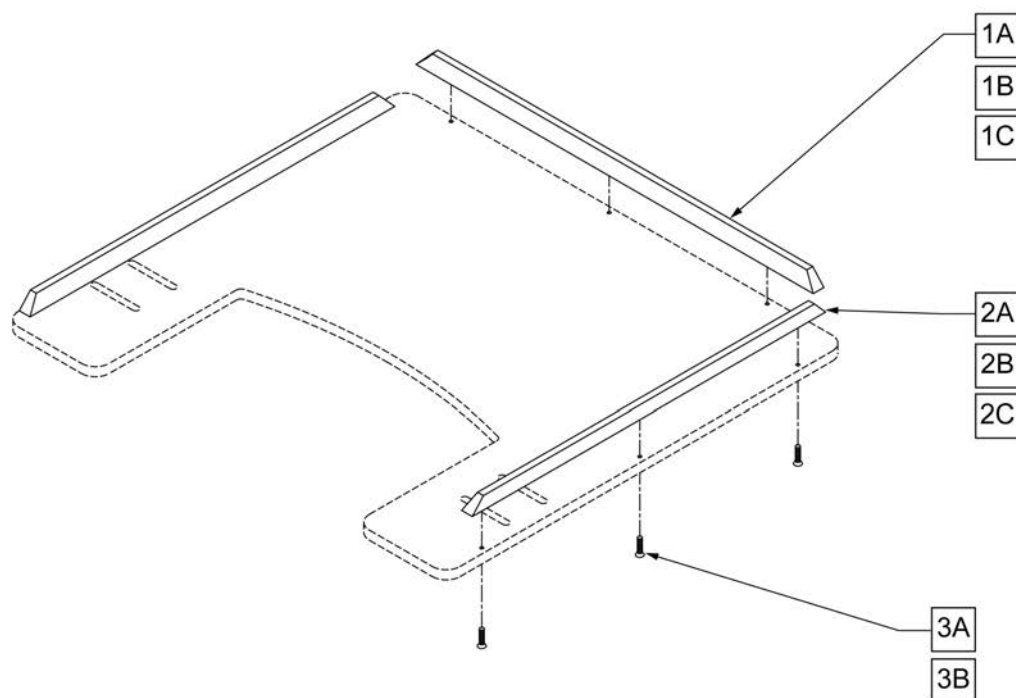


Pos.	Item Number	Description	Pos.	Item Number	Description
1A	9004	19" WOOD TRAY RIBS	2A	9004	19" WOOD TRAY RIBS
1A	9005	21" WOOD TRAY RIBS	2A	9005	21" WOOD TRAY RIBS
1A	9006-QU	23 1/2" WOOD TRAY RIBS	2B	9007-QU	15 1/2" HEAVY DUTY RIB FOR WOOD TRAY
1B	9010	19" HEAVY DUTY RIB FOR WOOD TRAY	2B	9010	19" HEAVY DUTY RIB FOR WOOD TRAY
1B	9013	21" HEAVY DUTY RIB FOR WOOD TRAY	2B	9013	21" HEAVY DUTY RIB FOR WOOD TRAY
1B	9016	23 1/2" HEAVY DUTY RIB FOR WOOD TRAY	2C	9008-QU	15 1/2" HEAVY DUTY RIB FOR 3/8" TUFFAK
1C	9011-QU	19" HEAVY DUTY RIB FOR 3/8" TUFFAK	2C	9011-QU	19" HEAVY DUTY RIB FOR 3/8" TUFFAK
1C	9014	21" HEAVY DUTY RIB FOR 3/8" TUFFAK	2C	9014	21" HEAVY DUTY RIB FOR 3/8" TUFFAK
1C	9017	23 1/2" HEAVY DUTY RIB FOR 3/8" TUFFAK	3A	2268	#6 x 5/8" OVAL PHILLIPS SCREW
2A	9003	15 1/2" WOOD TRAY RIBS	3B	230047	#6 x 3/4" FH PHIL S/M SCREW


Note: Pad cannot be added to an existing tray as snaps must be installed at the factory to function correctly.

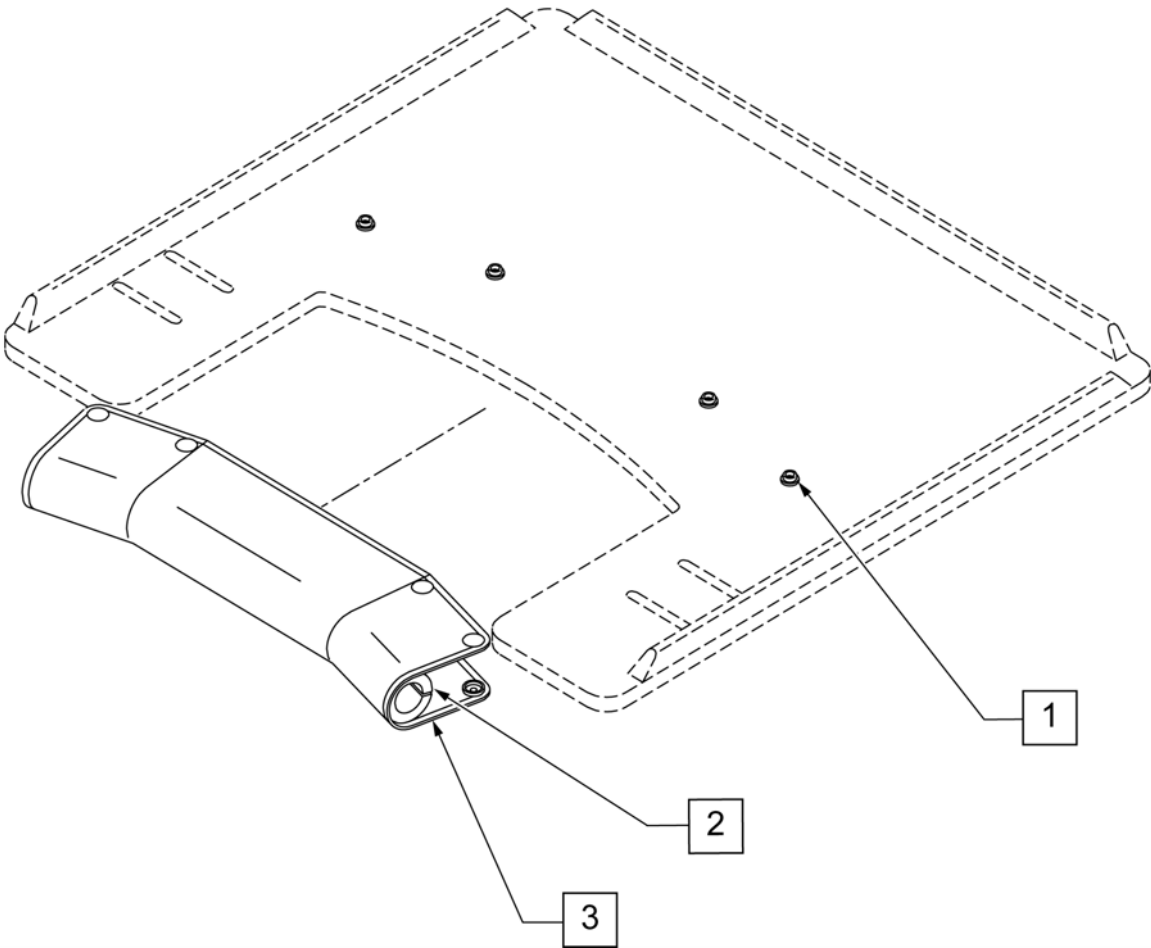


Pos.	Item Number	Description	Remarks
1-3	9PS	PADDED STOMACH CUT OUT	(SPECIFY COLOR)

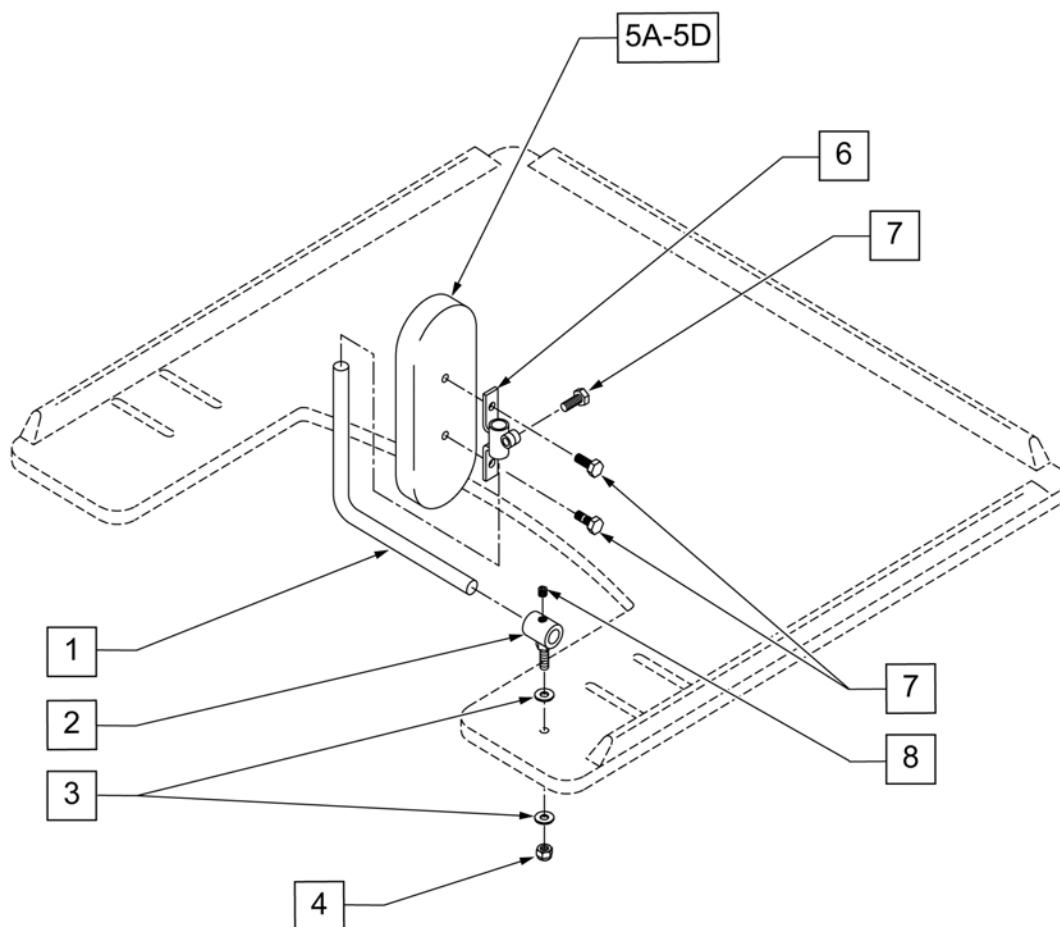


Pos.	Item Number	Description	Remarks
1	450752	HEADREST ADJUSTMENT BAR	
2	450751	HEADREST PAD BUSHING	
3	2702	#10 SAE FLAT WASHER	
4	106932	1/4 -20 NYLOCK NUT HIGH	
5A	15022	4X6 RECTANGULAR PAD	
5B	15013	5 x 7 OVAL PAD	
5C	755020	4 x 5 OVAL PAD	
5D	15018	4 x 6 OVAL PAD	
6	450750	HEADREST PAD BRACKET	
7	221105-QU	1/4-20 x 3/4" HEX C/S	
8	2270	5/16-18 x 1/4" SOC SET SCREW	
1-4,5A,6-8	14980A	TRAY PROTRACTION ASSM-4X6 RECT	PAIR
1-4,5B,6-8	14980B	TRAY PROTRACTION ASSM-5X7 OVAL	PAIR
1-4,5C,6-8	14980D	TRAY PROTRACTION ASSM-4X5 OVAL	PAIR
1-4,5D,6-8	14980E	TRAY PROTRACTION ASSM-4X6 OVAL	PAIR

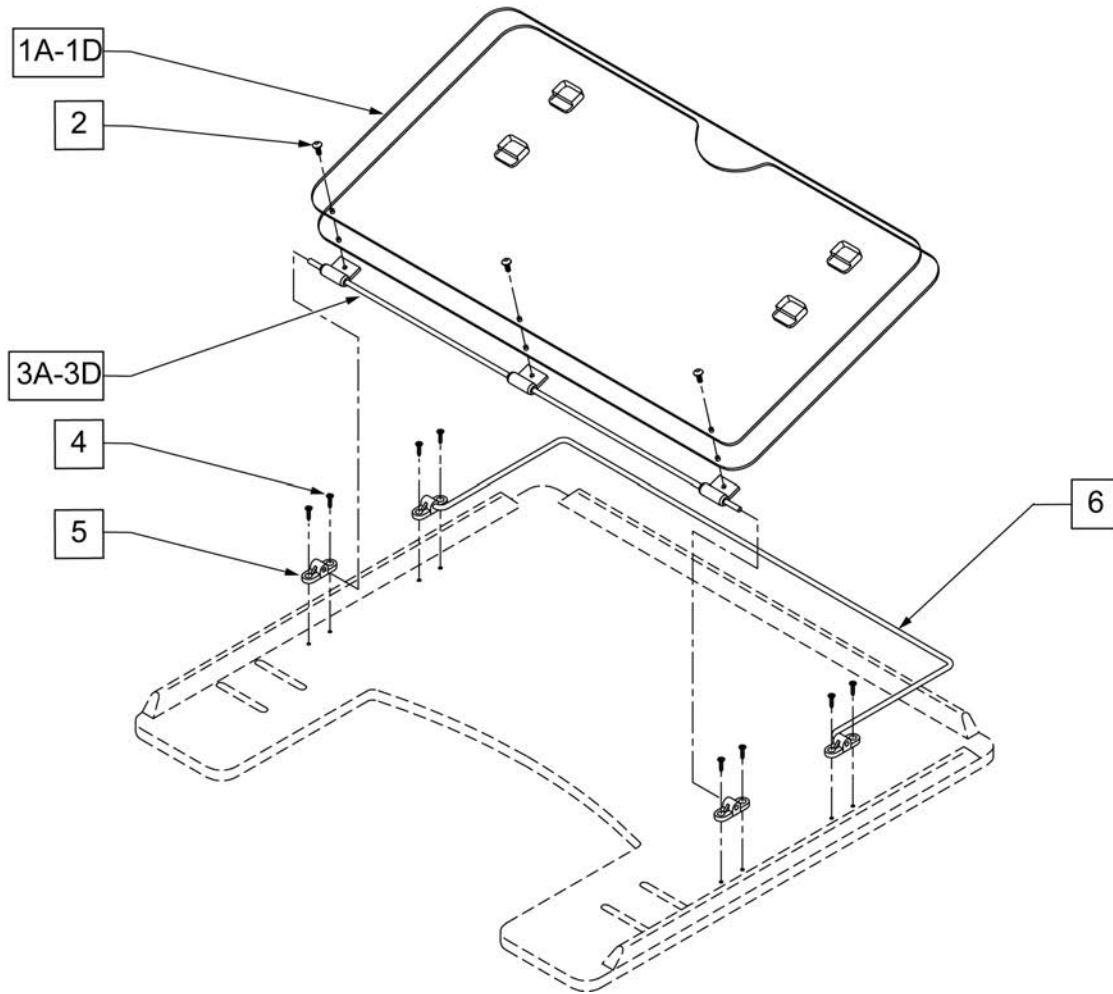


Pos.	Item Number	Description
1A,2	17916	9-1 STYLE 3 OVERLAY ONLY
1B,2	17917	9-2 STYLE 3 OVERLAY ONLY
1C,2	17918	9-5 STYLE 3 OVERLAY ONLY
1D,2	17939	9-6 STYLE 3 OVERLAY ONLY
2	226017	10-32 x 3/8 BH C/S BLK
3A	17922	9-1 HINGED OVERLAY PIVOT BAR
3B	17923	9-2 HINGED OVERLAY PIVOT BAR
3C	17924	9-5 HINGED OVERLAY PIVOT BAR
3D	17929	9-6 HINGED OVERLAY PIVOT BAR
4	2266	#6 x 1/2" FL DH PHL SCREW
5	4061	TRAY OVERLAY MOUNTING BRACKET
6A	17925	TRAY OVERLAY ROD
6B	17926	9-2 OVERLAY ROD
6C	17927	9-5 OVERLAY ROD
6D	17928	9-6 TRAY OVERLAY ROD
1A-6A	17919	9-1 STYLE 3 OVERLAY ASSM
1B-6B	17920	9-2 STYLE 3 OVERLAY ASSM
1C-6C	17921	9-5 STYLE 3 OVERLAY ASSM
1D-6D	17902	9-6 STYLE 3 OVERLAY ASSY

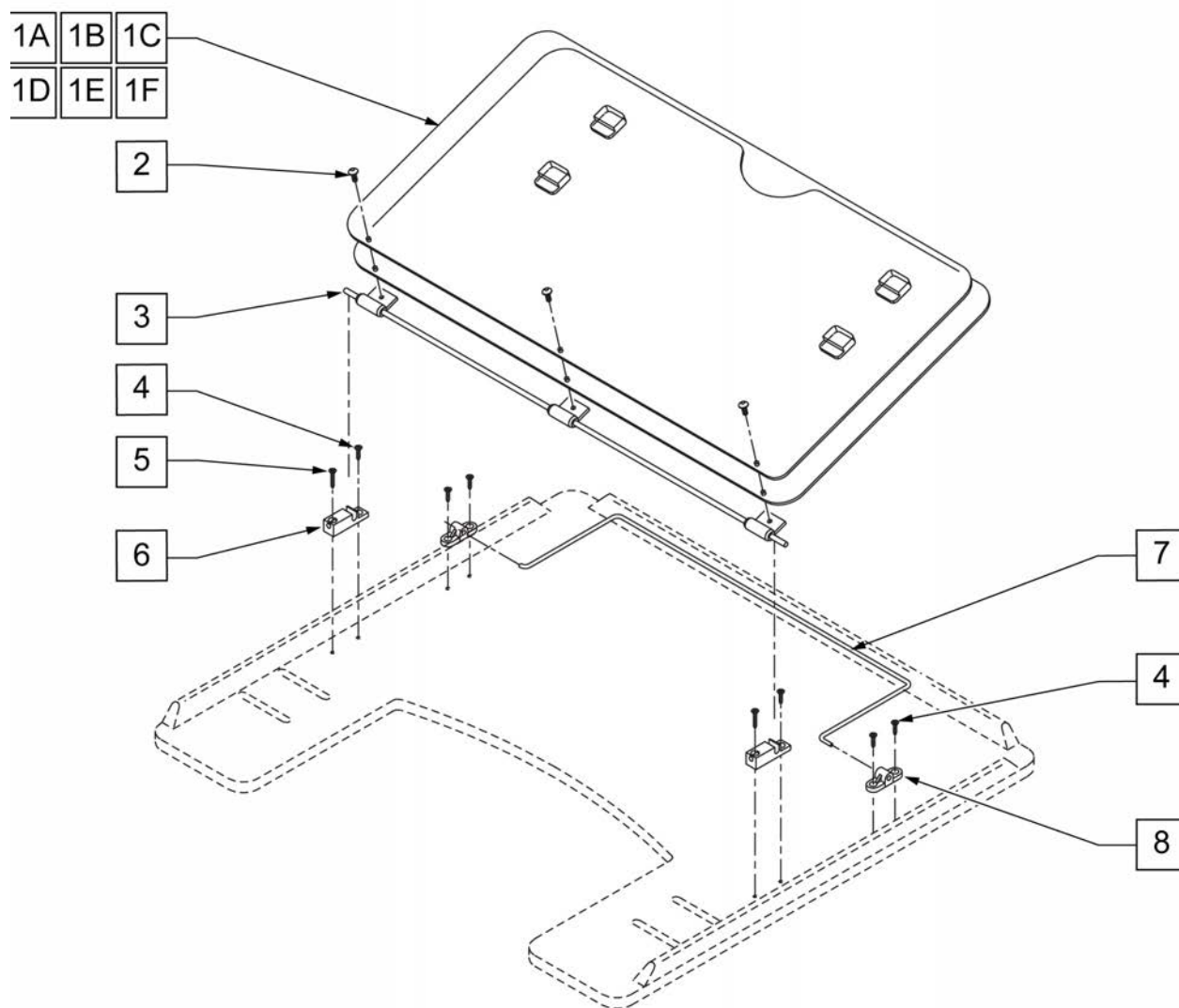



Pos.	Item Number	Description	Pos.	Item Number	Description
1A	17935	9-1 STYLE 2 OVERLAY	3D	17929	9-6 HINGED OVERLAY PIVOT BAR
1B	17936	9-2 STYLE 2 OVERLAY	4	2266	#6 x 1/2" FL DH PHL SCREW
1C	17937	9-5 STYLE 2 OVERLAY	5	230047	#6 x 3/4" FH PHIL S/M SCREW
1D	17938	9-6 STYLE 2 OVERLAY	6	14232	TRAY OVERLAY - QUICK RELEASE
1E	17930	STYLE #1 LEARNING BOARD	7A	17925	TRAY OVERLAY ROD
1F	17931	STYLE #1 FULL LEARNING BOARD	7B	17926	9-2 OVERLAY ROD
1A,2	17916	9-1 STYLE 3 OVERLAY ONLY	7C	17927	9-5 OVERLAY ROD
1B,2	17917	9-2 STYLE 3 OVERLAY ONLY	7D	17928	9-6 TRAY OVERLAY ROD
1C,2	17918	9-5 STYLE 3 OVERLAY ONLY	8	4061	TRAY OVERLAY MOUNTING BRACKET
1D,2	17939	9-6 STYLE 3 OVERLAY ONLY	1-8	17903	9-1 STYLE 3A OVERLAY ASSEMBLY
2	226017	10-32 x 3/8 BH C/S BLK	1-8	17904	9-2 STYLE 3A OVERLAY ASSEMBLY
3A	17922	9-1 HINGED OVERLAY PIVOT BAR	1-8	17905	9-5 STYLE 3A OVERLAY ASSEMBLY
3B	17923	9-2 HINGED OVERLAY PIVOT BAR	1-8	17906	9-6 STYLE 3A OVERLAY ASSEMBLY
3C	17924	9-5 HINGED OVERLAY PIVOT BAR			

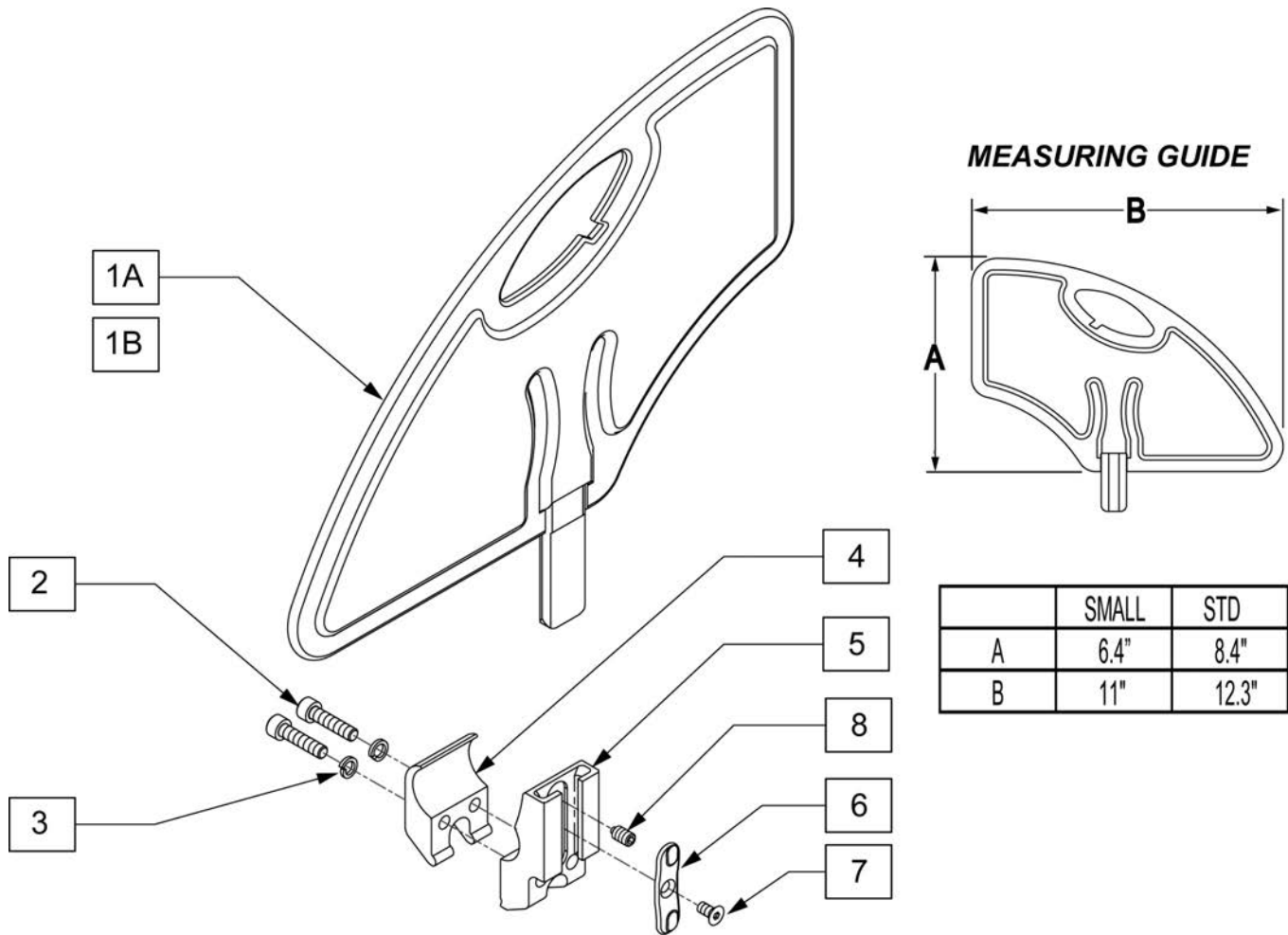


Pos.	Item Number	Description
1A	104164-001	SIDE GUARD ASSY STD
1B	104164-002	SIDE GUARD ASSY SMALL
2	106961	BHCS 1/4-28 X 1-1/4 ROHS
3	107367	WASHER LOCK HI COLLAR 1/4 ROHS
4	106714	REC SIDE GUARD CLAMP THREADED
6	108780	RECEIVER SG PLASTIC INSERT
7	107135	FHCS #6-32 X 1/4 BLK
8	107136	SETSCREW 1/4-28 X 5/16 OVAL PT
5-8	108781	RECIEVER SIDE GUARD ASSY
2-8	107592	RECEIVER SIDE GUARD ASSY
1A-8	107590	SIDE GUARD ASSY W/ CLAMP STD
1B-8	107591	SIDE GUARD ASSY W/ CLAMP SMALL

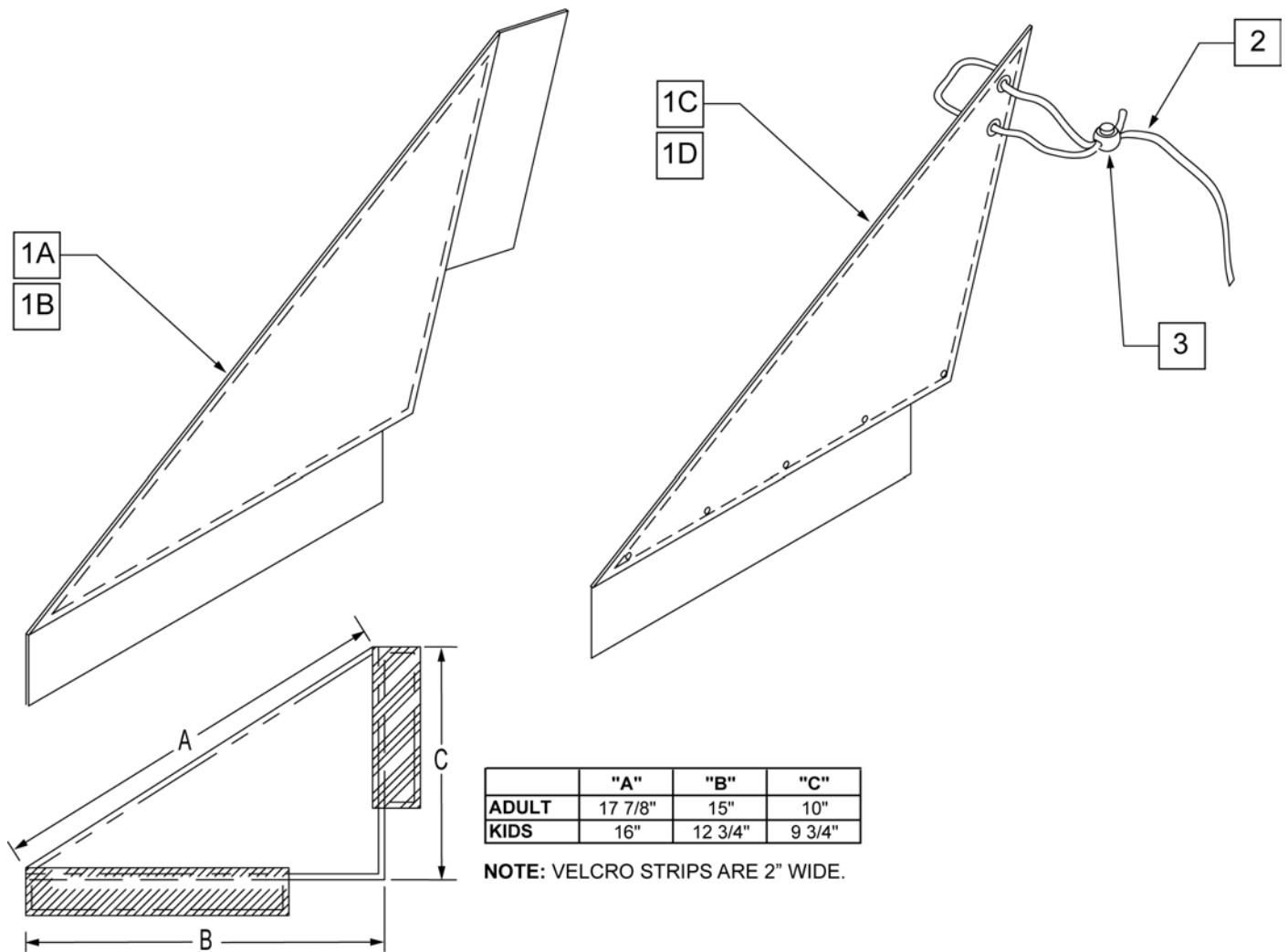

Pos.	Item Number	Description	Remarks
1A	750400	SIDE GUARD REG Q1 R	RIGHT-ADULT-Used w/ Straight Back
1A	750401	SIDE GUARD REG Q1 L	LEFT-ADULT-Used w/ Straight Back
1A	901102	SIDE GUARDS REG NF	PAIR-ADULT-Used w/ Straight Back
1B	901100	SIDE GUARDS KIDS NF	PAIR-KIDS-Used w/ Straight Back
1C	750240	SIDE GUARD FABRIC SHORT Q2 R	RIGHT-KIDS-Used w/ Integral Push Handle
1C	750241	SIDE GUARD FABRIC SHORT Q2 L	LEFT-KIDS-Used w/ Integral Push Handle
1D	750434	SIDE GUARD REG W/VELCRO R	RIGHT-ADULT-Used w/ Integral Push Handle
1D	750435	SIDE GUARD REG W/VELCRO L	LEFT-ADULT-Used w/ Integral Push Handle
2	750050	SIDE GUARD LACE Q2	
3	560220	B-LOCK	
1D-2,3	901109	SIDE GUARDS REG EX/RV	PAIR-ADULT-Used w/ Integral Push Handle
1C,2,3	901121	SIDE GUARDS KIDS FABRIC Q2 NEW	PAIR-KIDS-Used w/ Integral Push Handle

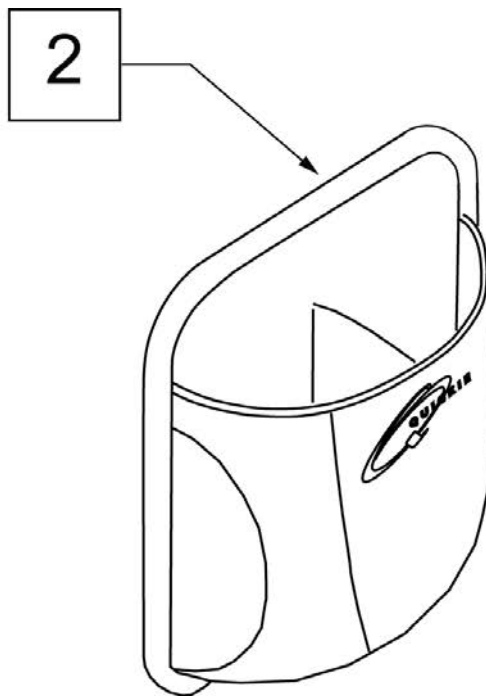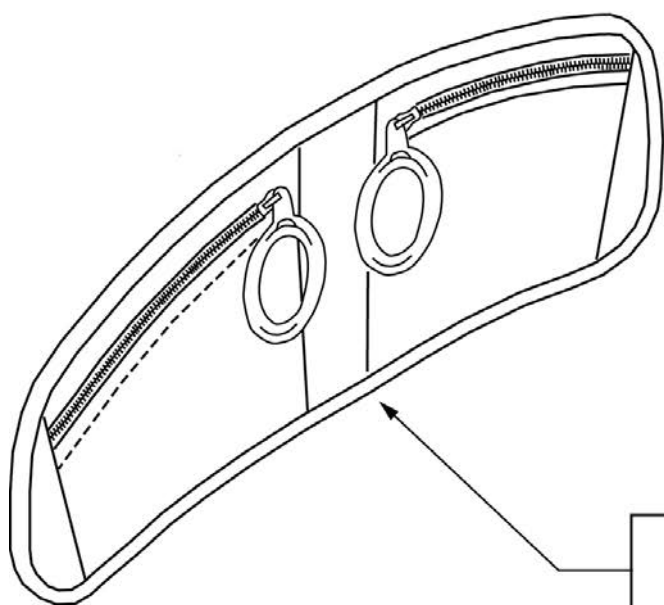


Pos.	Item Number	Description	Remarks
1	013477	LAP ORGANIZER	Size: 18.25"x 7.75"x .75"
2	013475	QUICKIE SIDE POCKET	Size: 6.5"x 6.5"x 2"

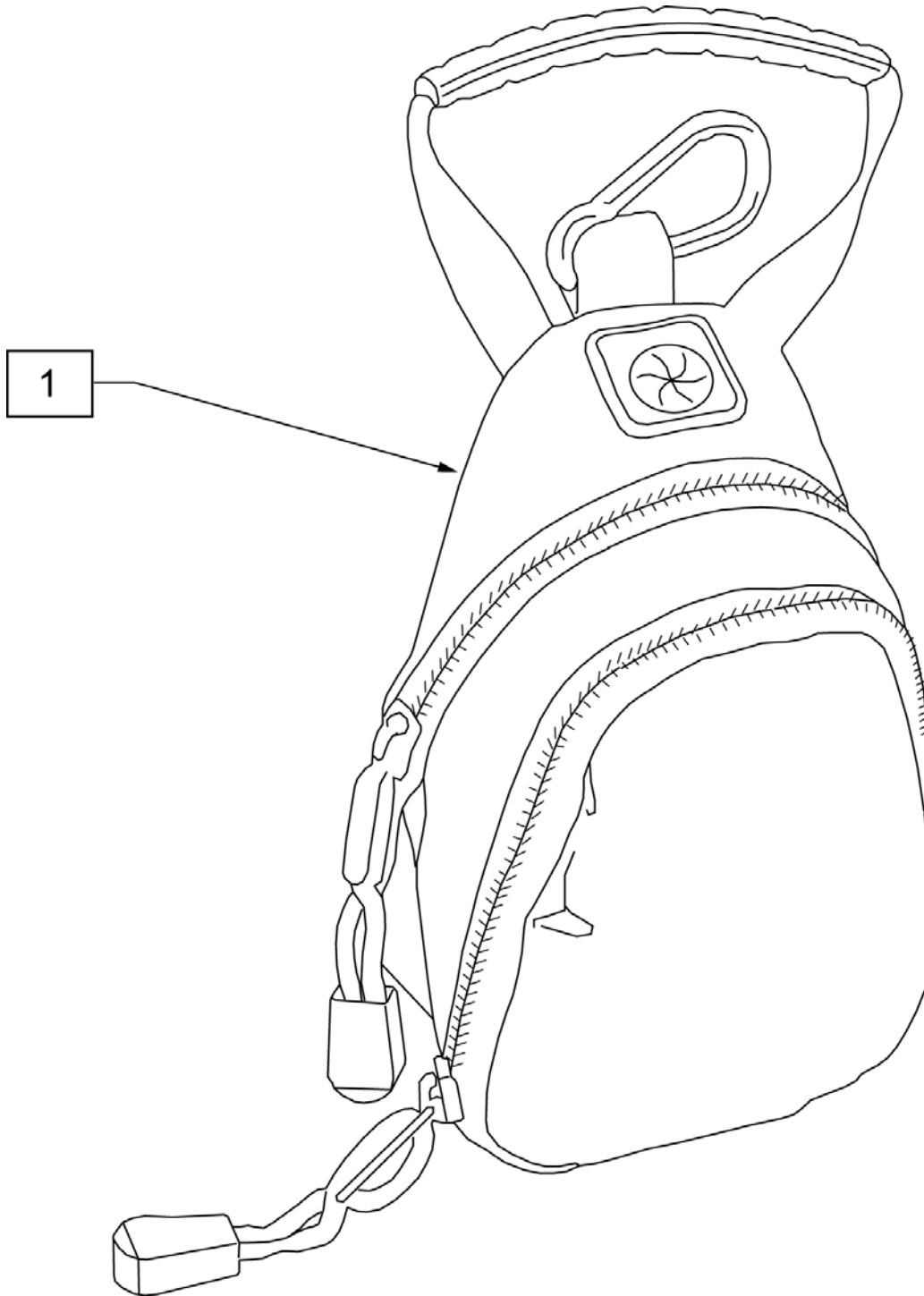


Pos.	Item Number	Description
1	109229	CELL PHONE PACK (ORANGE)

NOTE: KIDS BACKPACKS DO NOT HAVE ADJUSTABLE STRAPS W/ BUCKLES.

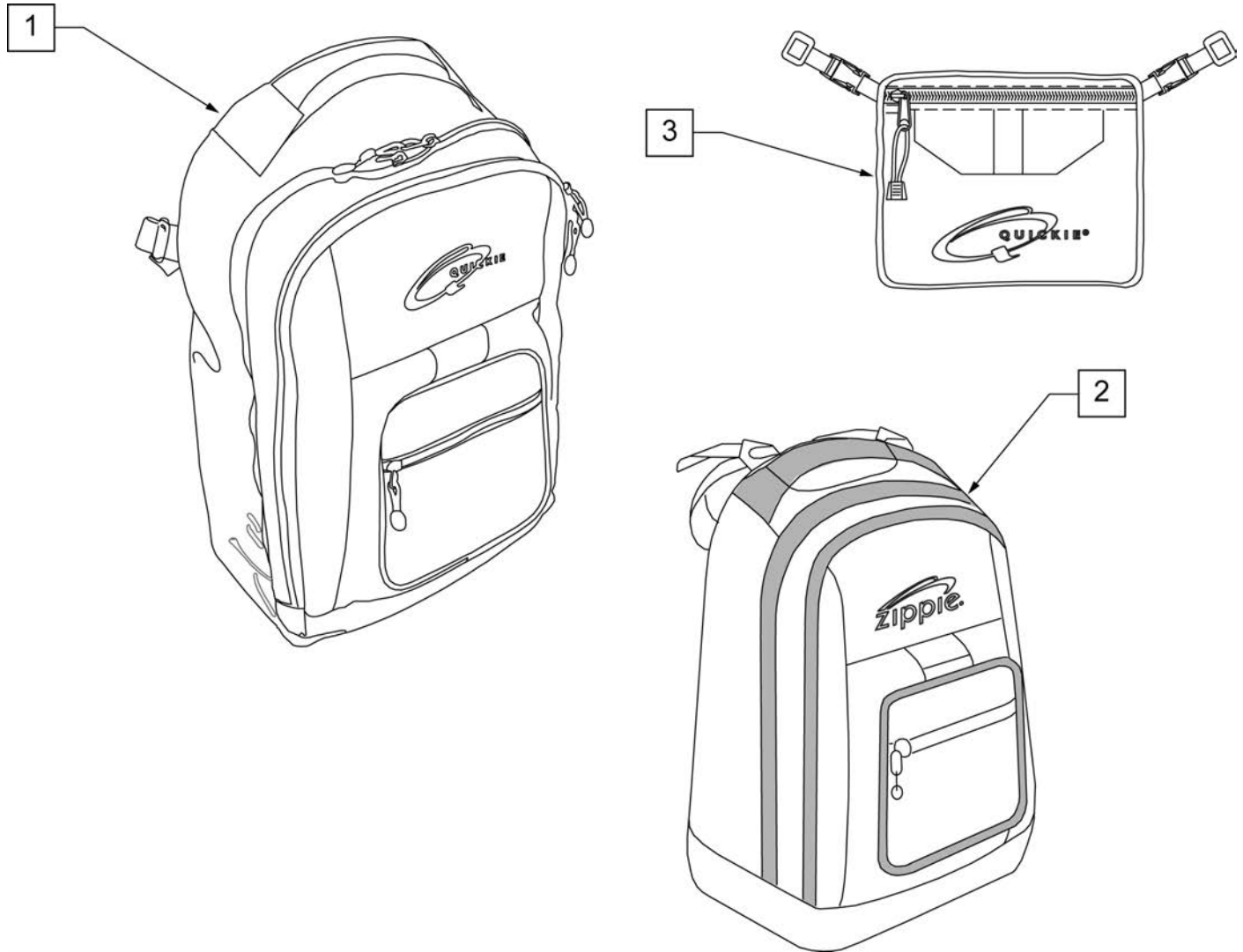

Pos.	Item Number	Description
1	750115	BACK PACK QUICKIE
2	104572	BACKPACK ZIPPIE
3	750307	SEAT POUCH QUICKIE


NOTE: THE Q2 HOLDER WILL INHIBIT THE USE OF THE LOWER TWO MOUNTING HOLES FOR THE AXLE PLATE.

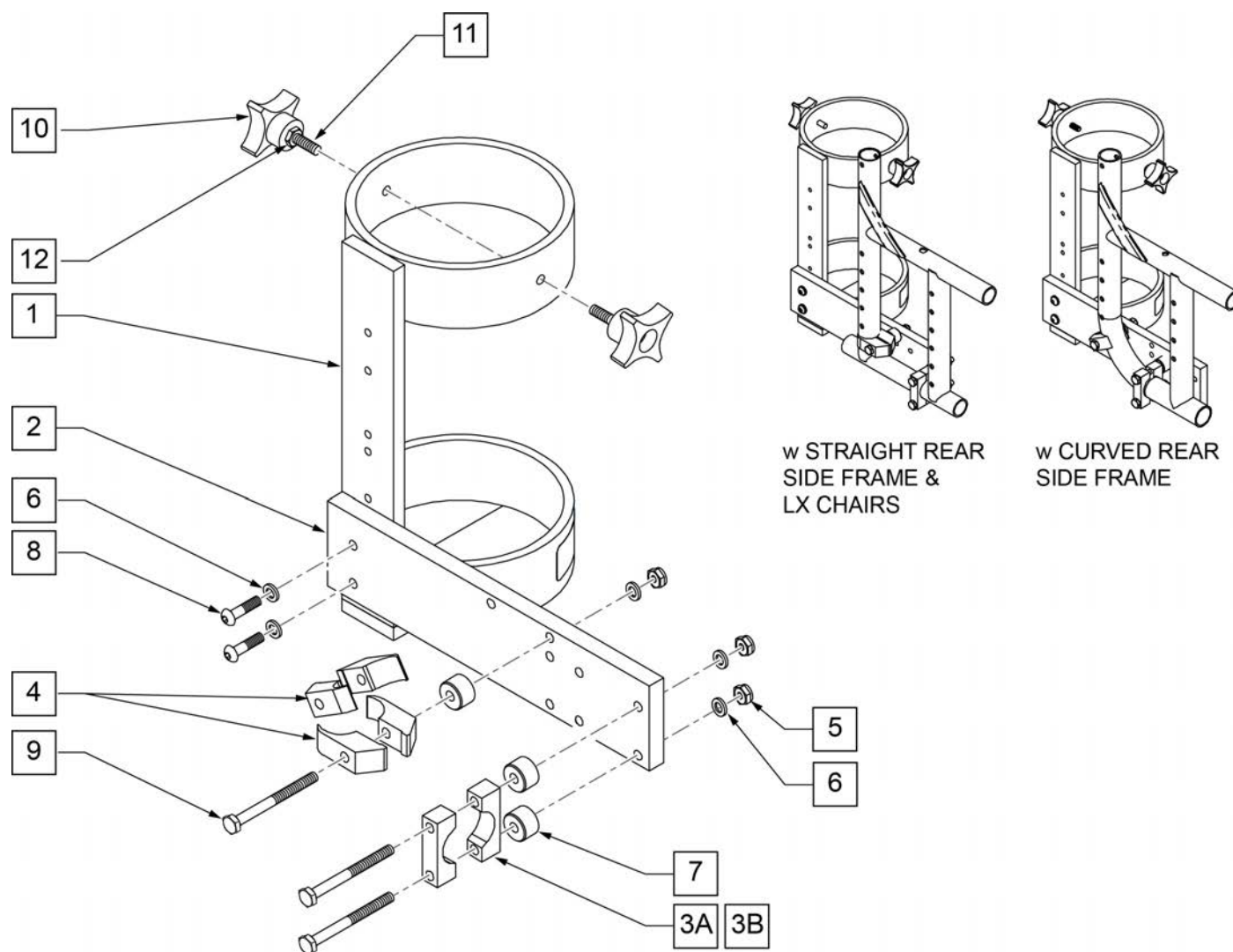

Pos.	Item Number	Description	Remarks
3A	550210	A1 LOW LOCK CLAMP BLK AN	
3B	C550210	A1 CSTM LOW LOCK CLAMP	Only available through "Built 4 Me".
4	550063	A1 PLST SIDE GURD CLAMP	
5	106934	A1 1/4-28 NYLOCK NUT	
6	106943	A1 1/4 AN WASHER FLAT	
7	330031	A1 SUSPENSION SPACER 1/2	
8	226110	A1 1/4-28 X 1 BH C/S	
9	221193	A1 1/4-28 X 2 3/4 G8 HEX	
10	349018	A1 KNOB S/A ADJ STROLLER	
11	226136	A1 1/4-20 X 1 1/2 BH C/S	
12	201121	A1 1/4-20 NUT THIN	
1-2,3A,12	112264	O2 HOLDER ASSM Q2 STRAIGHT REAR SIDE FRAME	STRAIGHT REAR SIDE FRAME
1-2,3B-12	112267	O2 HOLDER ASSM Q2 CURVED REAR SIDE FRAME	CURVED REAR SIDE FRAME


NOTE: Clamps mount to 1 1/8" tubing.

NOTE: IV Pole Assembly will mount to straight & curved rear side frame but will not work when a Depth Adjustable Back option is utilized.


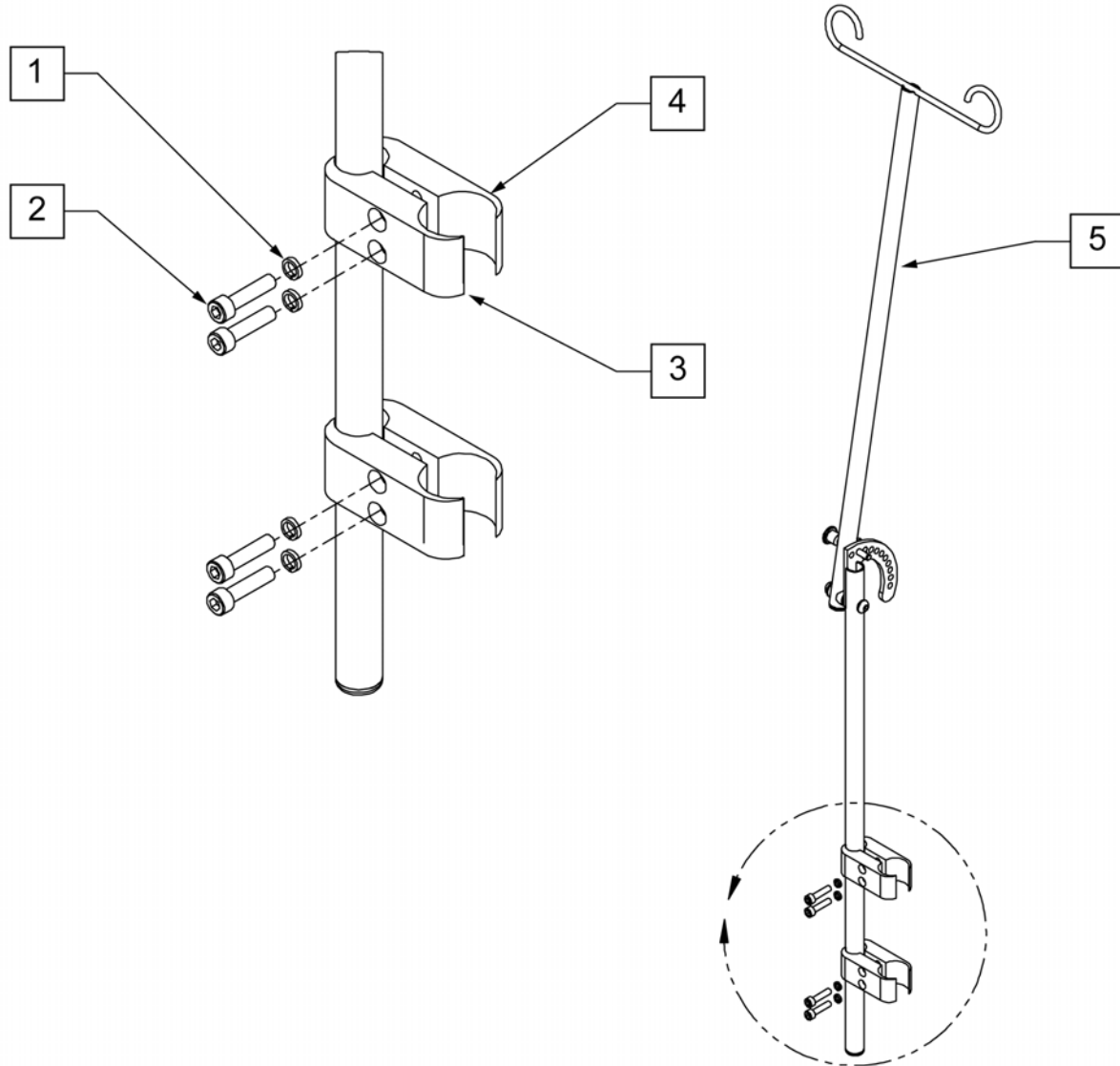


Pos.	Item Number	Description	Remarks
1	107032	SHCS 1/4-28 X 1 ROHS	4.0
2	107367	WASHER LOCK HI COLLAR 1/4 ROHS	4.0
3	550287	A1 CLAMP 1 1/8" - 3/4" W/R EA	2.0
4	550288	A1 CLAMP 1 1/8" - 3/4" THR EA	2.0
5	884141	A1 ADJ IV POLE ASSM EA	1.0
1-5	112261	ADJ IV POLE ASSM W 1 -1/8" CLAMP	

1A

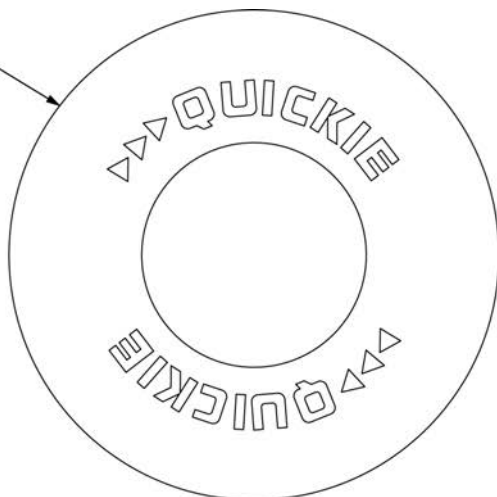


1B

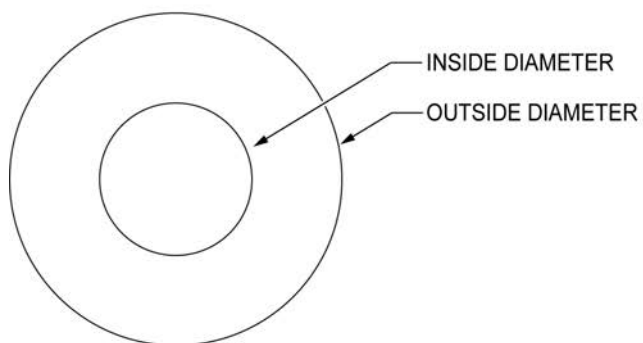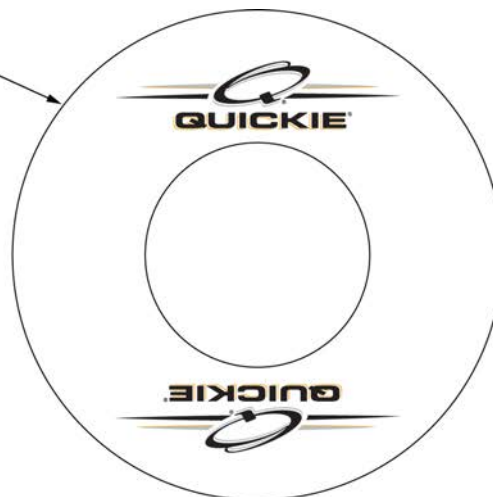
**SPOKE GUARD MEASURING GUIDE**

PART NUMBER	INSIDE DIAMETER	OUTSIDE DIAMETER
902001	8 1/4	15 7/8
902011	8 1/4	18 1/8
902021	10 1/4	19 5/8
902025	10 1/4	20 3/8
902037	10 1/4	21 7/8
902038	10 1/4	23 3/8
103660-001	8 1/4	16
103660-002	8 1/4	18
103663-001	10 1/4	20 1/2
103663-002	10 1/4	23 1/2
103734-001	10 1/4	19 1/4
103734-002	10 1/4	22
119336	10 1/4	19 1/4


Pos.	Item Number	Description	Remarks
>>>>		BLACK SPOKE GUARDS	
1A	902001	SPK GRDS 20" QUICKIE BLK	Pair-BLACK-FITS 20" WHEELS
1A	902011	SPK GRDS 22" QUICKIE BLK	Pair-BLACK-FITS 22" WHEELS or 24" (540) WHEELS W/ PROJECTION HANDRIMS
1A	902021	SPK GRD 24" QUICKIE SCR MT BLK	Pair-BLACK-FITS 24" (540) MAG & SPOKE WHEELS
1A	902025	SPK GRD 24" QUICKIE TAB MT BLK	Pair-BLACK-FITS 25" (559) SPOKE WHEELS
1A	902037	SPK GRD 26" QUICKIE TAB MT BLK	Pair-BLACK-FITS 26" (590) WHEELS
1A	902038	SPK GRD 26" QUICKIE HT BLK	Pair-BLACK-FITS 27" (700) WHEELS
>>>>		CLEAR SPOKE GUARDS	
1B	103660-001	CLEAR SPOKE GUARD 20"	Pair-CLEAR-FITS 20" WHEELS
1B	103660-002	CLEAR SPOKE GUARD 22"	Pair-CLEAR-FITS 22" WHEELS or 24" (540) WHEELS W/ PROJECTION HANDRIMS
1B	103734-001	SPOKE GUARD 24" CLR	Pair-CLEAR-FITS 24" (540) SPOKE WHEELS
1B	119336	SPOKE GUARD 24" CLR	Pair-CLEAR-FITS 24" (540) MAG WHEELS
1B	103663-001	SPOKE GUARD 24" QP CLR	Pair-CLEAR-FITS 25" (559) WHEELS
1B	103734-002	SPOKE GUARD 26" CLR	Pair-CLEAR-FITS 26" (590) WHEELS
1B	103663-002	SPOKE GUARD 26" QP CLR	Pair-CLEAR-FITS 27"(700) WHEELS

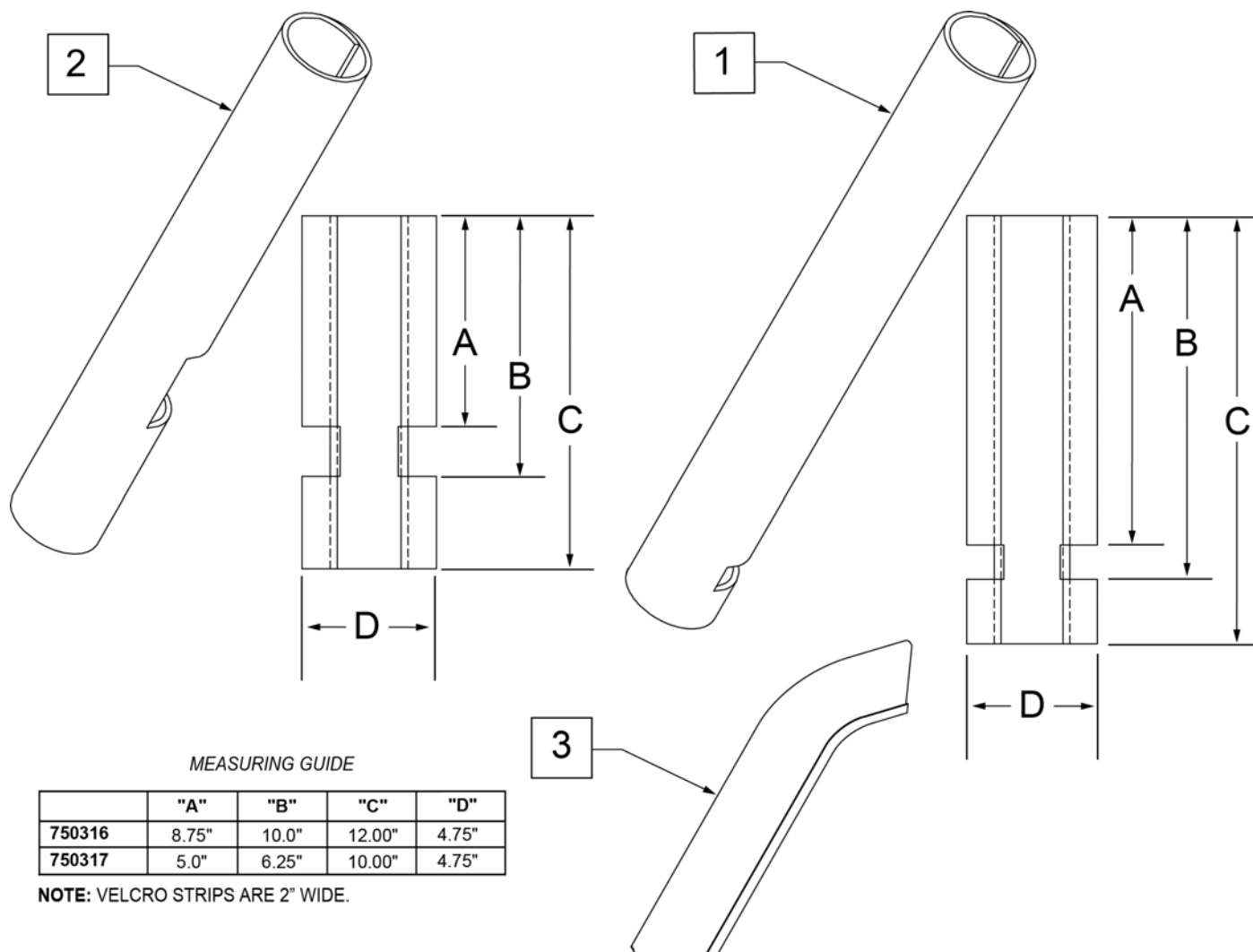


Pos.	Item Number	Description	Remarks
1	750316	IMPACT GUARDS NEOPRENE FIXED (PR)	PAIR-Fixed Front Frame
2	750317	IMPACT GUARDS NEOPRENE S/A	PAIR-S/A Footrest & Q7
3	109231	GEL VELCRO PAD HANGER/FRAME	PAIR- fits S/A & Fixed Front Frame

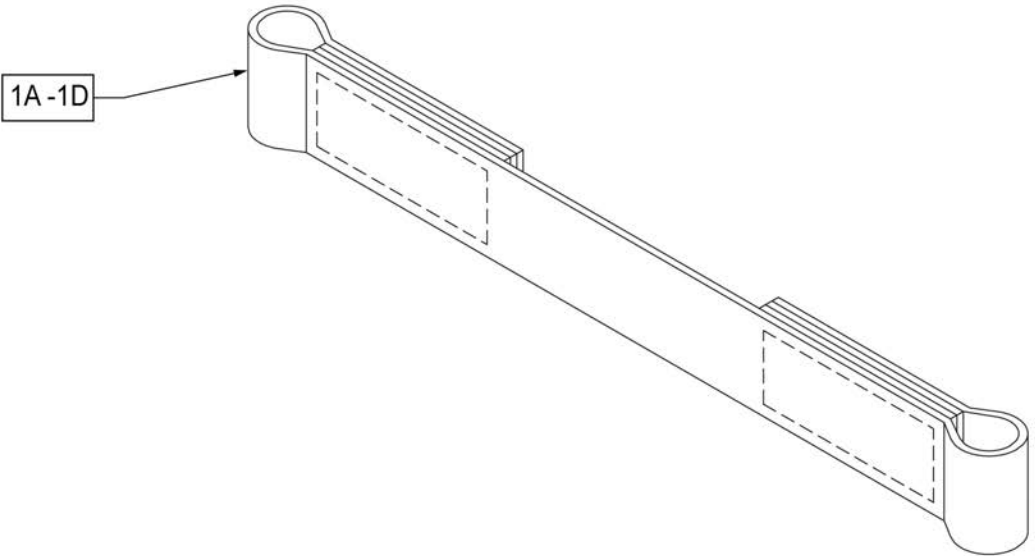

Pos.	Item Number	Description
1A	109356	LEG STRAP SOFT FEEL Q 11 -12"
1B	109357	LEG STRAP SOFT FEEL Q 13 -15"
1C	109366	LEG STRAP SOFT FEEL Q 16 -18"
1D	111487	LEG STRAP SOFT FEEL Q 19 -20"

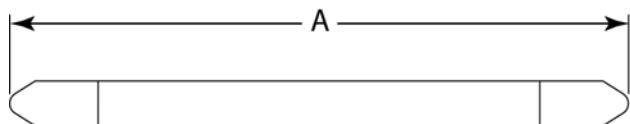
MEASURING GUIDE

DESCRIPTION	DIM "A"
LEG STRAP 11"-12" BLK	27 1/2
LEG STRAP 13"-15" BLK	32 1/2
LEG STRAP 16"-18" BLK	39 1/2
LEG STRAP 19"-20" BLK	45 1/2

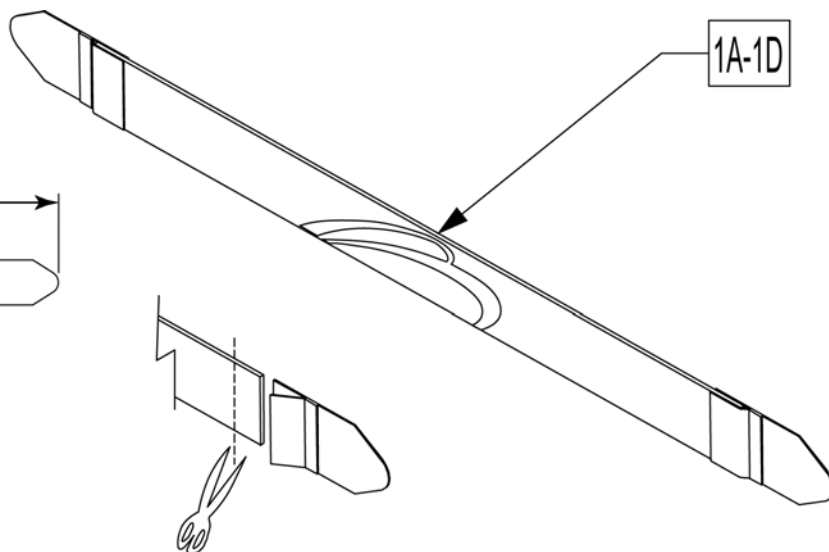


Leg Straps may be custom cut to a smaller size


Pos.	Item Number	Description
1A	113255	LEG STRAP 11"-12" BLK
1B	113256	LEG STRAP 13"-15" BLK
1C	113257	LEG STRAP 16"-18" BLK
1D	113258	LEG STRAP 19"-20" BLK

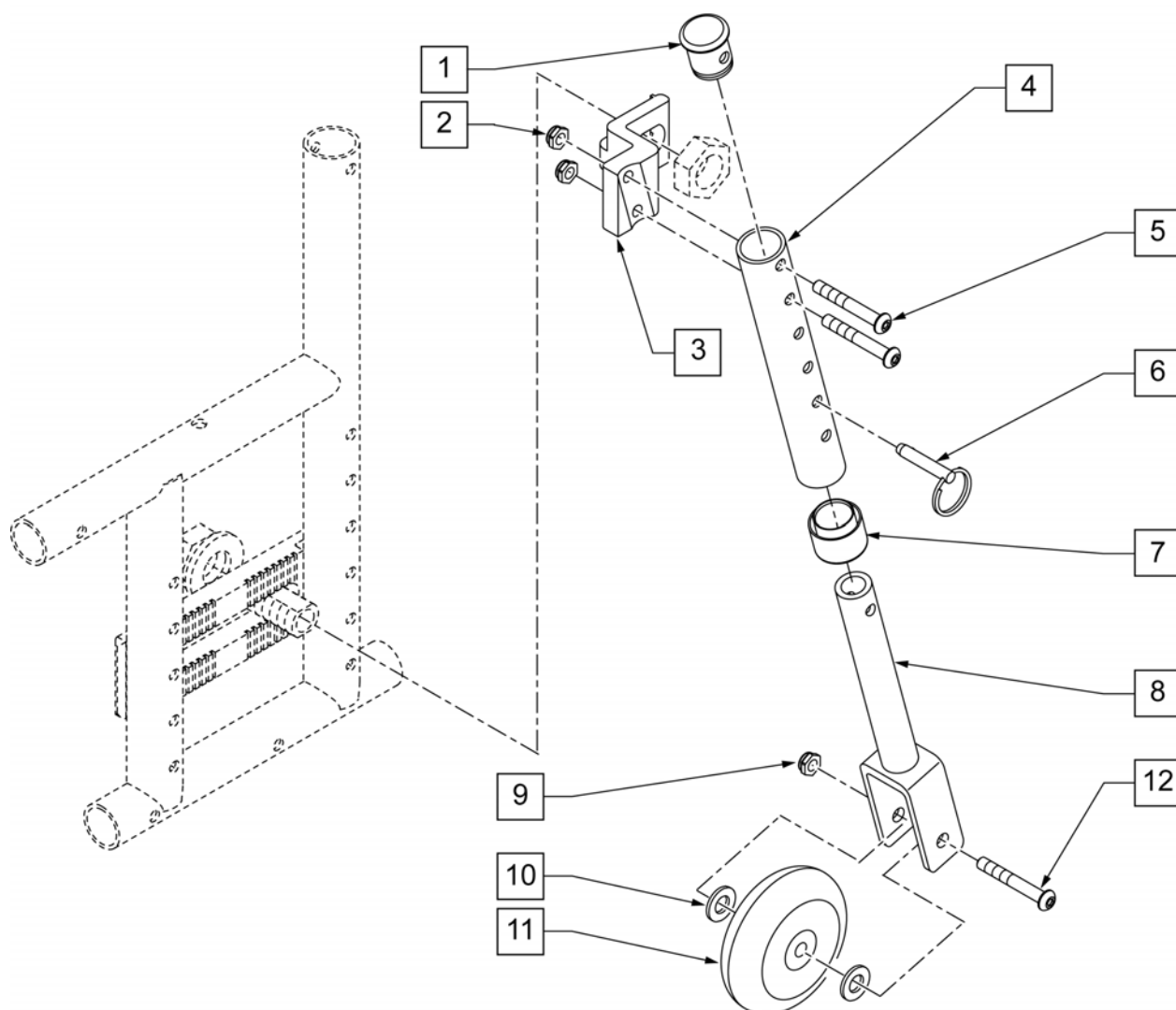

Pos.	Item Number	Description	Remarks
1	560999	1" INSERT PLUG .083 W/1/4 HOLE	
2	106935	NUT NYLOCK THIN 1/4-28 ROHS	
3	550090	BRKT TRAVEL WHL RCVR MNT R Q2	Right
3	550091	BRKT TRAVEL WHL RCVR MNT L Q2	Left
4	528031	TRAVEL WHEEL MOUNT TUBE Q2	
5	106962	1/4-28 x 1 3/4 BH C/S BLK ZN	
6	330027	QUICK RELEASE PIN 1/4"	
7	561018	1" TUBE END BUSHING THICK	
8	542160	FORK EXT WELDMENT TRVL WHL Q2	
9	106937	5/16 NYLOCK NUT THIN	
10	263100	5/16 AN WASHER FLAT	
11	920025	3" MICRO CASTER W/BRNG (PR)	
12	106982	BHCS 5/16-18 X 1-3/4 SHLD ROHS	
1-12	880082	TRAVEL WHEEL ASSY R	Right
1-12	880083	TRAVEL WHEEL ASSY L	Left
1-12	920035	TRAVEL WHL ASSM W/HDWR (PR)	Pair

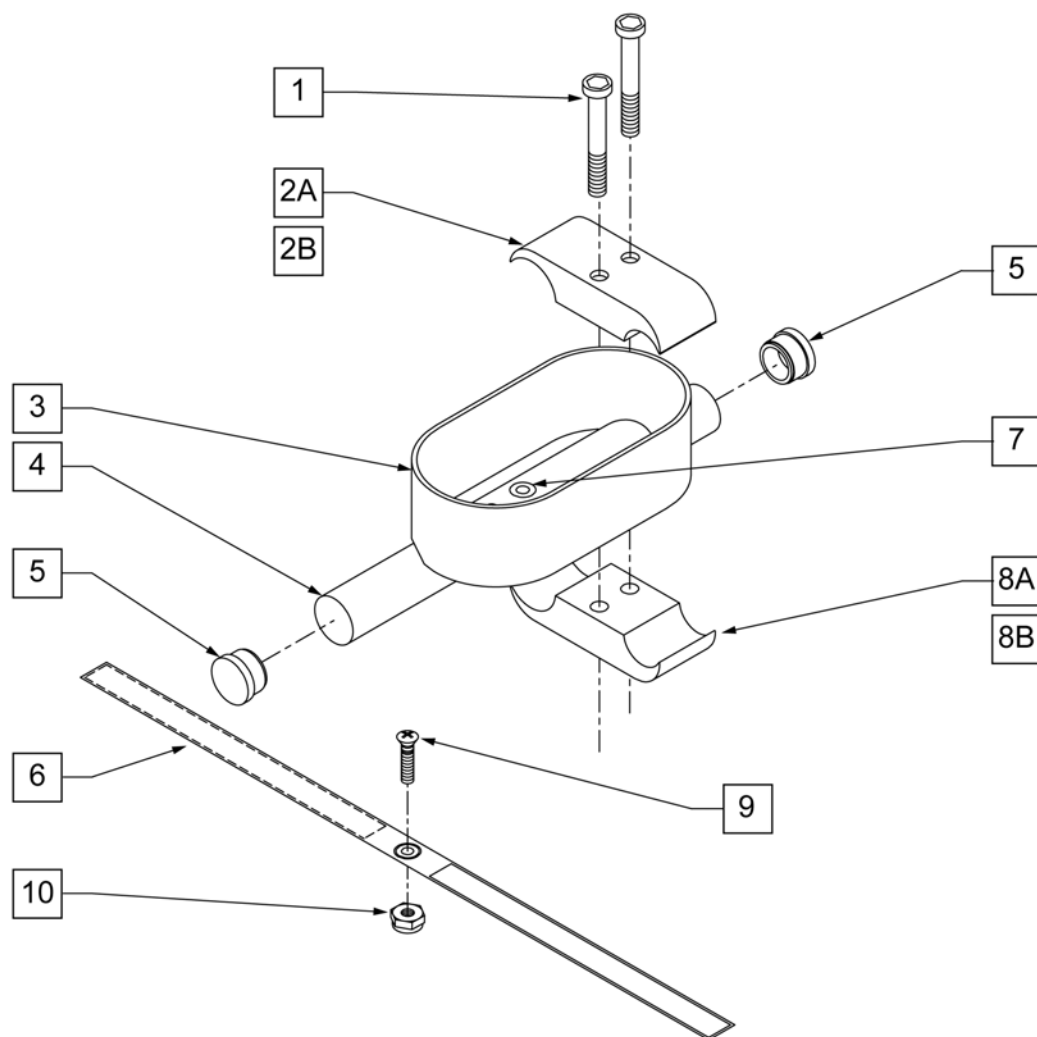


Pos.	Item Number	Description
1	107032	1/4-28 x 1 SOC C/S BLK
2B	550287	CLAMP 1 1/8" - 3/4" W/RECESS
2A	550477	CLAMP 2 1/4" RECESS BLK
6	750495	CRUTCH HOLDER STRAP F
8A	550576	CLAMP 2 1/4' THREADED BLK
8B	550288	CLAMP 1 1/8" - 3/4" THREADED
9	281110	10-32 x 1/2 PAN PHIL
10	200100-QU	10-32 NYLOCK NUT
1,2A,3-5,7,8A	880802	CRUTCH HOLDER F
1,2B,3-5,7,8B	880803	CRUTCH HOLDER CURVED REAR S/F
1,2A,3-7,8A,9,10	901002	CRUTCH HOLDER ASSEMBLY F
1,2B,3-7,8B,9,10	901004	CRUTCH HOLDER ASSEMBLY Q2 CURVED REAR

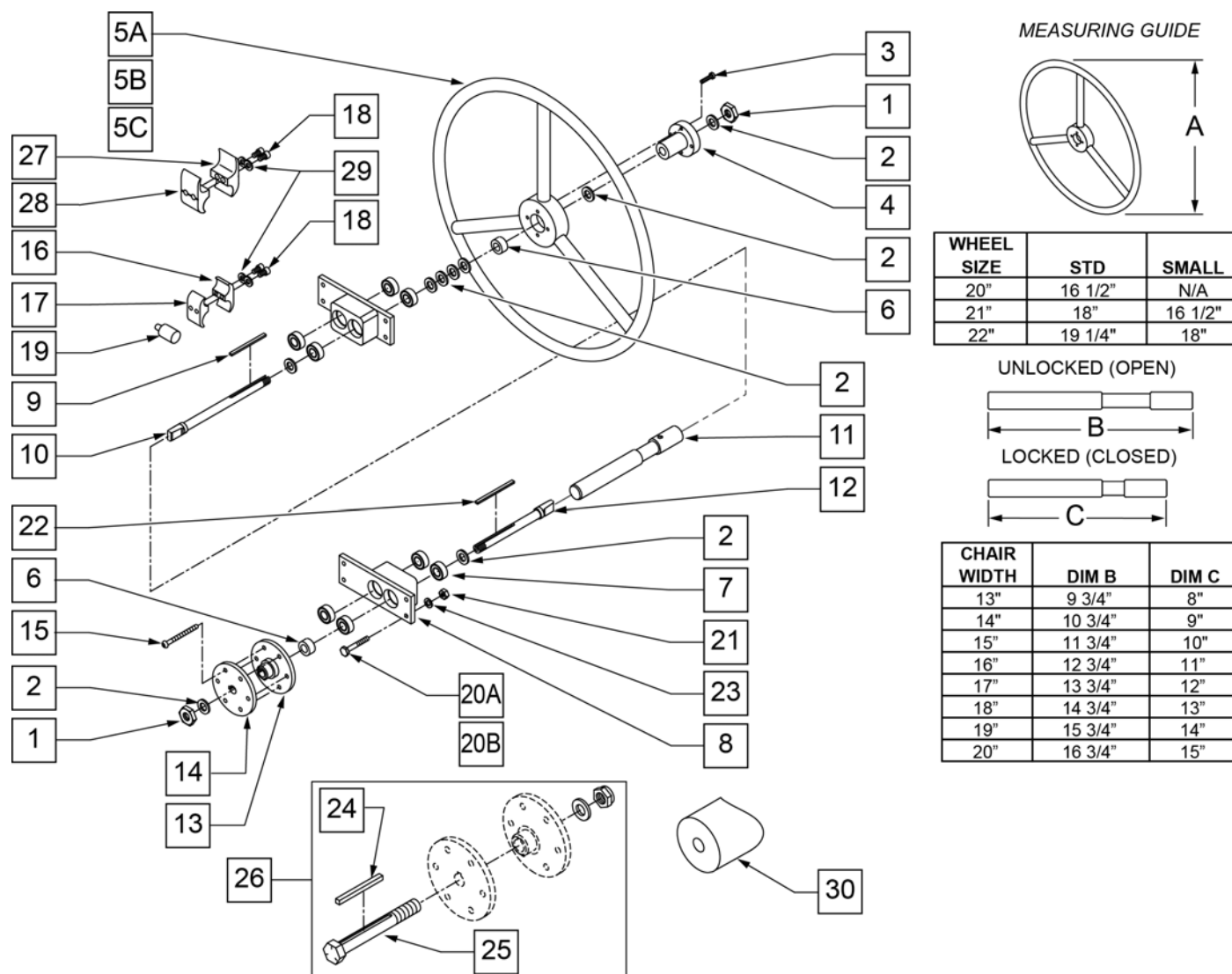


Pos.	Item Number	Description	Remarks
1	106940	NUT NYLOCK THIN 1/2 -20 ROHS	
2	106949	WASHER FLAT AN 1/2 ROHS	
3	106956	SHCS 10 -32 x 3/4 ROHS	
4	106691	HAND RING HUB	
5A	529801	1 AD HR ONLY 20" STD/22" SM	
5B	529800	1 AD HR ONLY 22" STD/24" SM	
5C	529802	1 AD HR ONLY 24" STD	
6	483058	1 AD WHEEL SPACER	
7	349500	1 1/8 x 1/2 BEARING	
7,8	880820	1 AD AXLE PLATE W/BEARINGS BLK	
9	341274	3/16 SQ x 1 1/2 KEY STL	
10	481131	1 AD HANDRIM DRIVE SHAFT	
11A	101097-001	1 AD SHAFT ASSM 12"	
11B	101097-002	1 AD SHAFT ASSM 13"	
11C	101097-003	1 AD SHAFT ASSM 14"	
11D	101097-004	1 AD SHAFT ASSM 15"	
11E	101097-005	1 AD SHAFT ASSM 16"	


Pos.	Item Number	Description	Remarks
11F	101097-006	1 AD SHAFT ASSM 17"	
11G	101097-007	1 AD SHAFT ASSM 18"	
11H	101097-008	1 AD SHAFT ASSM 19"	
11J	101097-009	1 AD SHAFT ASSM 20"	
12	481132	1 AD WHEEL DRIVE SHAFT	
13	500631	WHEEL HUB 1 ARM DRIVE W/TAP	
14	500630	1 AD WHEEL HUB W/O TAP	
15	226166	1/4-28 X 2 1/2 BH C/S BLK	
16	550658	CLAMP 1 1/4" THREADED BLK	
17	550659	CLAMP 1 1/4" RECESS HALF BLK	
18	107032	SHCS 1/4 -28 x 1 ROHS	
19	482030	1 AD DRIVE RECEIVER	
20A	106973	HHCS 1/4 -28 x 2 ROHS	
20B	107264	HHCS 5/16 -24 x 2 3/4 PL ROHS	Used with Anti-tips
21	106934	NUT NYLOCK 1/4 -28 ROHS	
22	341277	3/16 SQ x 2 3/8 KEY STL	
23	106944	WASHER FLAT THIN AN 1/4 ROHS	
24	341278	3/16" SQ x 2 1/4" KEY STL	
25	330014	1 AD ALIGNMENT BOLT	
26	330015	1 AD ALIGNMENT TOOL ASSEMBLY	
27	550245	1 AD CLAMP RECESSED 14"-20"	
28	550244	1 AD CLAMP THREADED 14"-20"	
29	107367	WASHER LOCK HI COLLAR 1/4 ROHS	
30	483071	SADDLE 1 1/8" ALUMINUM 1/4" HOLE	
19-23,29	107639	WHL LCK CLMP ASM	
18,27-29	880729	1 AD CLAMP ASSEMBLY 14"-20"	
1-4,5A,6-11A,12-15,19-23	116619	1AD 12" RETRO 20"STD/22"SM HR	
1-4,5A,6-11B,12-15,19-23	923532	1 AD 13" RETRO (20" STD/22" SM HR)	
1-4,5A,6-11B,12-15,19-23	923532	1AD 13" RETRO 20"STD/22"SM HR	
1-4,5A,6-11C,12-15,19-23	923533	1AD 14" RETRO 20"STD/22"SM HR	
1-4,5A,6-11D,12-15,19-23	923534	1AD 15" RETRO 20"STD/22"SM HR	
1-4,5A,6-11E,12-15,19-23	923535	1AD 16" RETRO 20"STD/22"SM HR	
1-4,5A,6-11F,12-15,19-23	923536	1AD 17" RETRO 20"STD/22"SM HR	
1-4,5A,6-11G,12-15,19-23	923537	1AD 18" RETRO 20"STD/22"SM HR	
1-4,5A,6-11H,12-15,19-23	923538	1AD 19" RETRO 20"STD/22"SM HR	
1-4,5A,6-11J,12-15,19-23	923539	1AD 20" RETRO 20"STD/22"SM HR	
1-4,5B,6-11A,12-15,19-23	116620	1AD 12" RETRO 22"STD/24"SM HR	
1-4,5B,6-11B,12-15,19-23	923512	1 AD 13" RETRO (22" STD/24" SM HR)	
1-4,5B,6-11C,12-15,19-23	923513	1AD 14" RETRO 22"STD/24"SM HR	
1-4,5B,6-11D,12-15,19-23	923514	1AD 15" RETRO 22"STD/24"SM HR	
1-4,5B,6-11E,12-15,19-23	923515	1AD 16" RETRO 22"STD/24"SM HR	
1-4,5B,6-11F,12-15,19-23	923516	1AD 17" RETRO 22"STD/24"SM HR	
1-4,5B,6-11G,12-15,19-23	923517	1AD 18" RETRO 22"STD/24"SM HR	
1-4,5B,6-11H,12-15,19-23	923518	1AD 19" RETRO 22"STD/24"SM HR	
1-4,5B,6-11J,12-15,19-23	923519	1AD 20" RETRO 22"STD/24"SM HR	
1-4,5C,6-11A,12-15,19-23	116621	1AD 12" RETRO 24" STD HR	
1-4,5C,6-11B,12-15,19-23	923522	1 AD 13" (24" STD HR)	
1-4,5C,6-11C,12-15,19-23	923523	1AD 14" RETRO 24" STD HR	
1-4,5C,6-11D,12-15,19-23	923524	1AD 15" RETRO 24" STD HR	
1-4,5C,6-11E,12-15,19-23	923525	1AD 16" RETRO 24" STD HR	
1-4,5C,6-11F,12-15,19-23	923526	1AD 17" RETRO 24" STD HR	
1-4,5C,6-11G,12-15,19-23	923527	1AD 18" RETRO 24" STD HR	
1-4,5C,6-11H,12-15,19-23	923528	1AD 19" RETRO 24" STD HR	
1-4,5C,6-11J,12-15,19-23	923529	1AD 20" RETRO 24" STD HR	

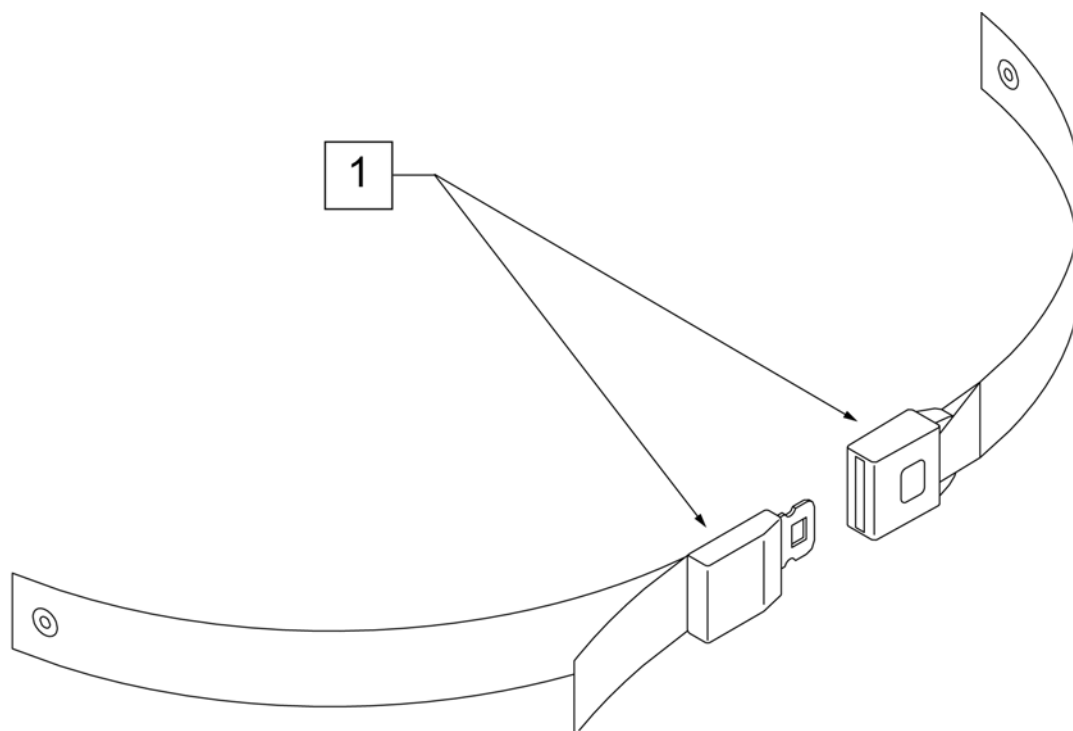

Pos.	Item Number	Description	Remarks
1	750276	1 1/2" POSITION BELT	AUTOSTYLE-64" Max length of belt.

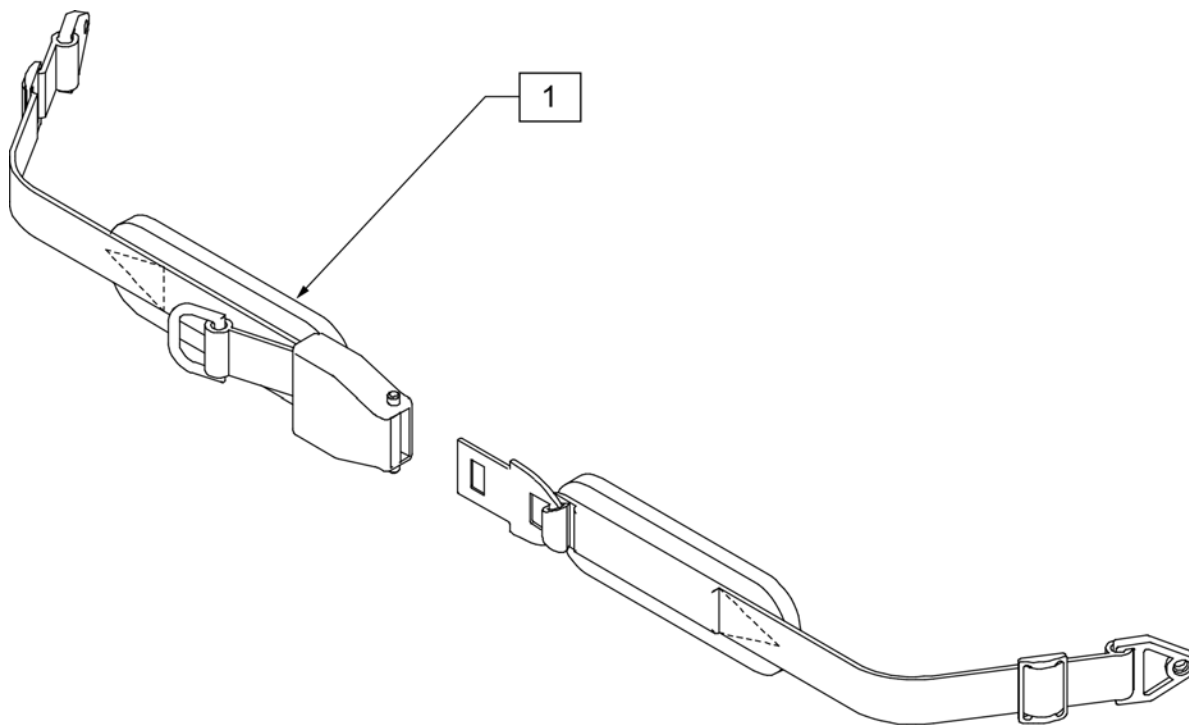


Pos.	Item Number	Description
1	450418-QU	BELT 2" AIRCRAFT BUCKLE
1	450423-QU	BELT 2" AIRCRAFT BUCKLE W/PAD
1	450424	BELT 2" AIRCRAFT BKL LONG W/PAD
1	450425	BELT 2" AIRCRAFT BKL LONG
1	450419-QU	BELT 2" AIRCRAFT BKL W/QUAD LP
1	468419	BELT 2" AIRCRT BKL W/QD LPCZ

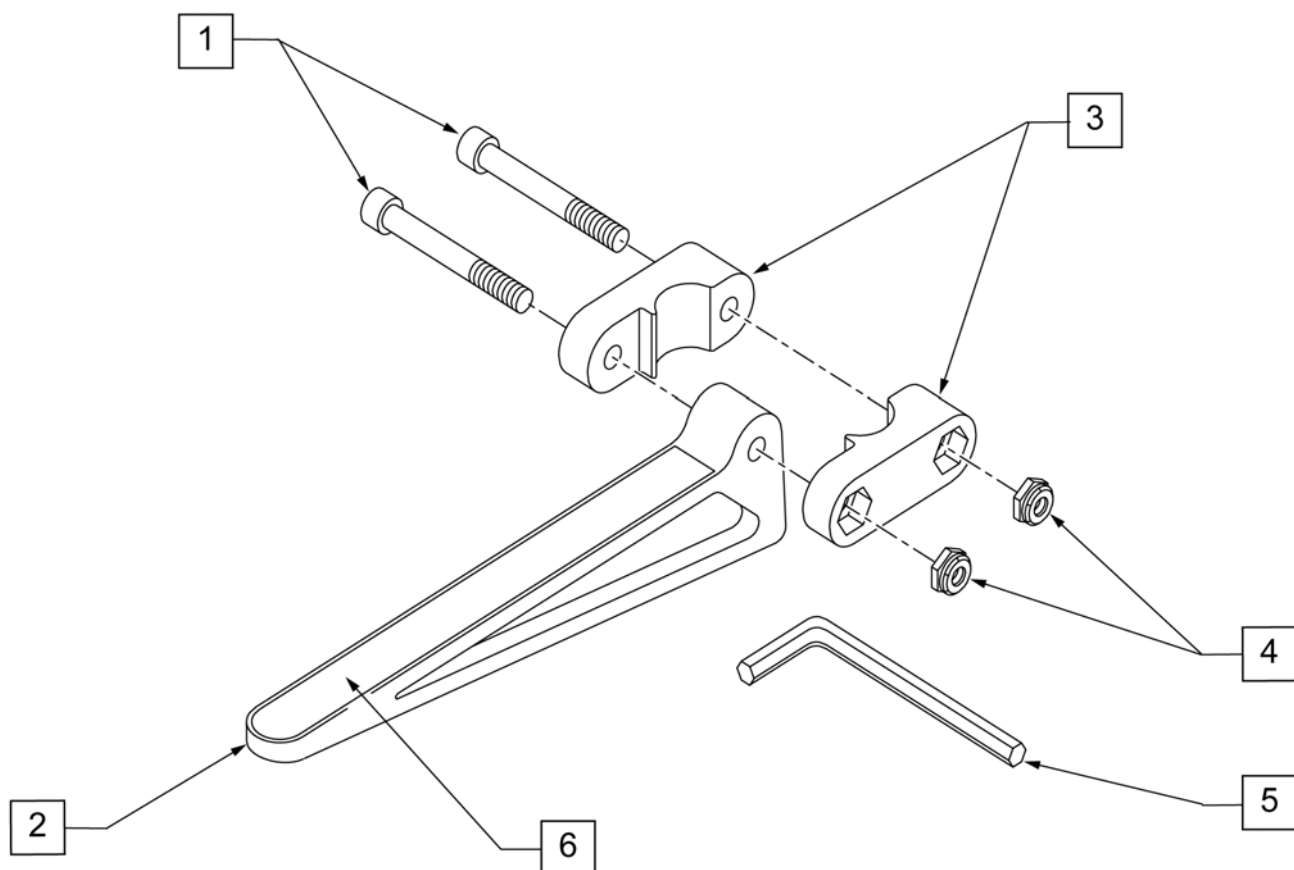


Pos.	Item Number	Description
1	106975	1/4-28 x 1 1/4 SOC C/S
2	561105	CADDY ARM
3	561104	CADDY CLAMP
4	106935	1/4-28 NYLOCK NUT THIN
5	349998	3/16" ALLEN WRENCH
6	580050	CADDY TAPE
1-6	902350	QUICKIE CADDY (PR)


Note: Positioning Belt Attachment Kit updated to Metric Fasteners April 2013. If in doubt order complete kit.

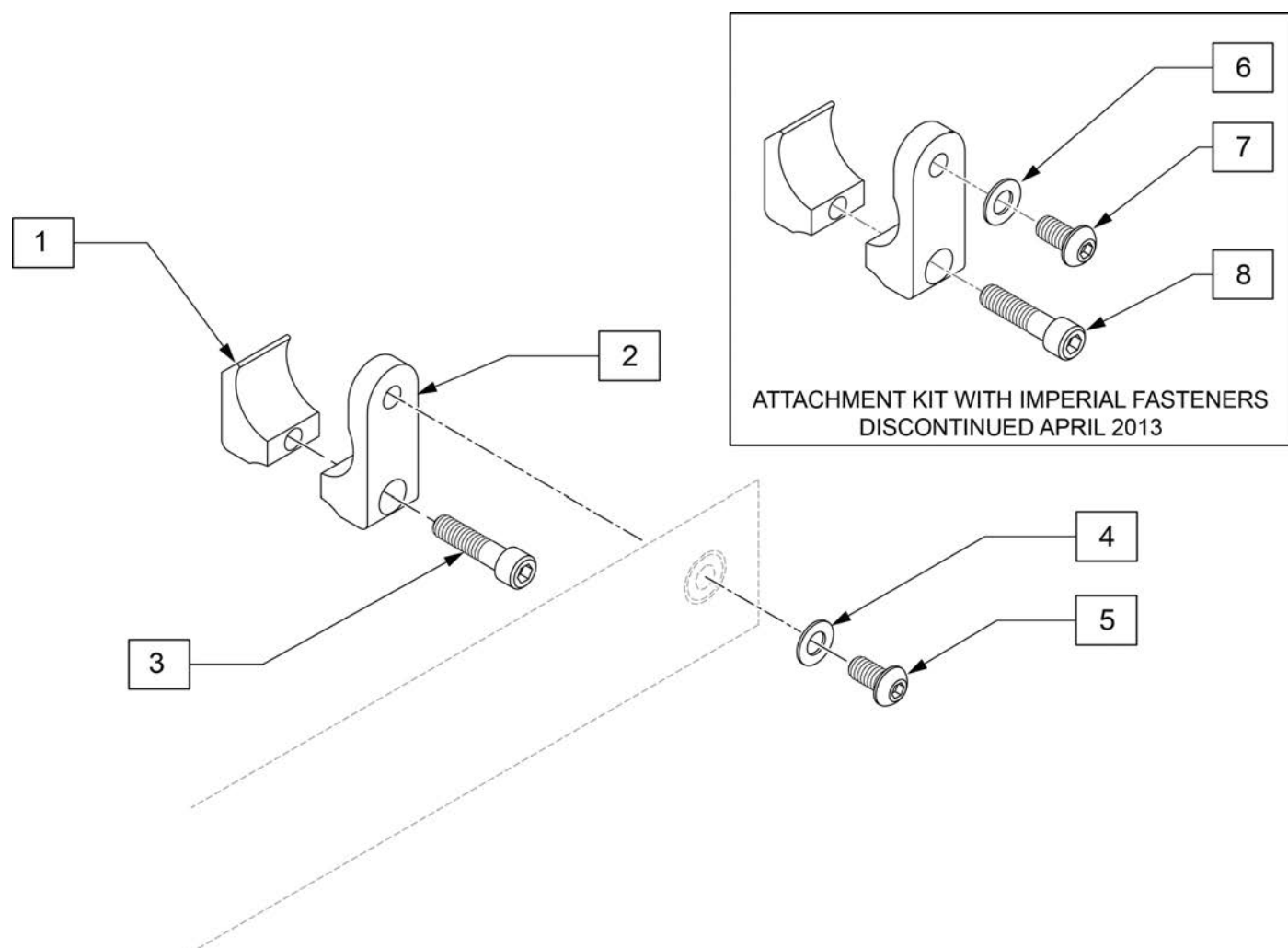

Pos.	Item Number	Description	Remarks
3	102263-003	SHCS M6 X 1 X 25 W/PL INORG	Metric
4	250104	WASHER FLAT M6	Metric
5	103185-005	BHCSM6X1X12FULLPL	Metric
1-5	120750	PSTN BELT CLAMP ASSM PR	Metric, Pair
6	106943	WASHER FLAT AN 1/4 ROHS	Imperial
7	226154	1/4-20 X 1/2 BH C/S W/PATCH	Imperial
8	221116	1/4-20 X 1 SOC HD BLK	Imperial

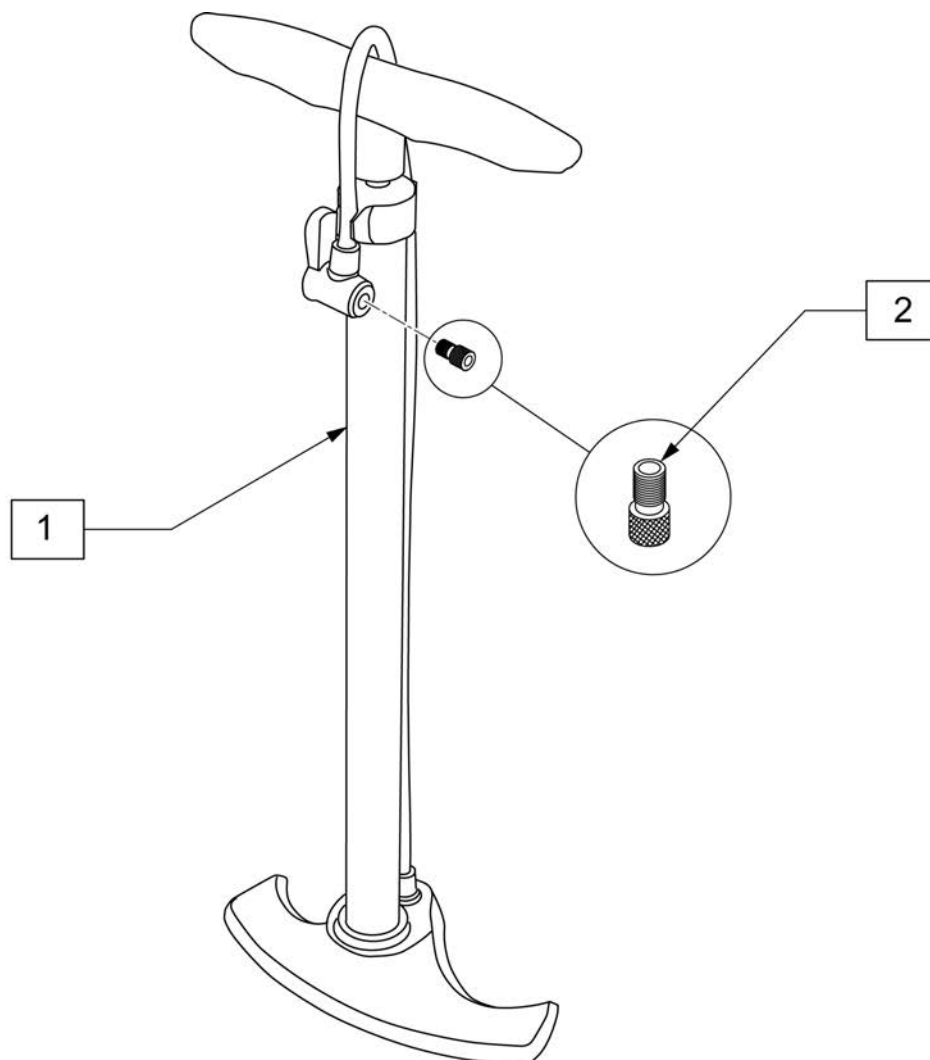

Pos.	Item Number	Description
1	389050	TIRE PUMP
2	381230	VALVE ADAPTER PRESTA

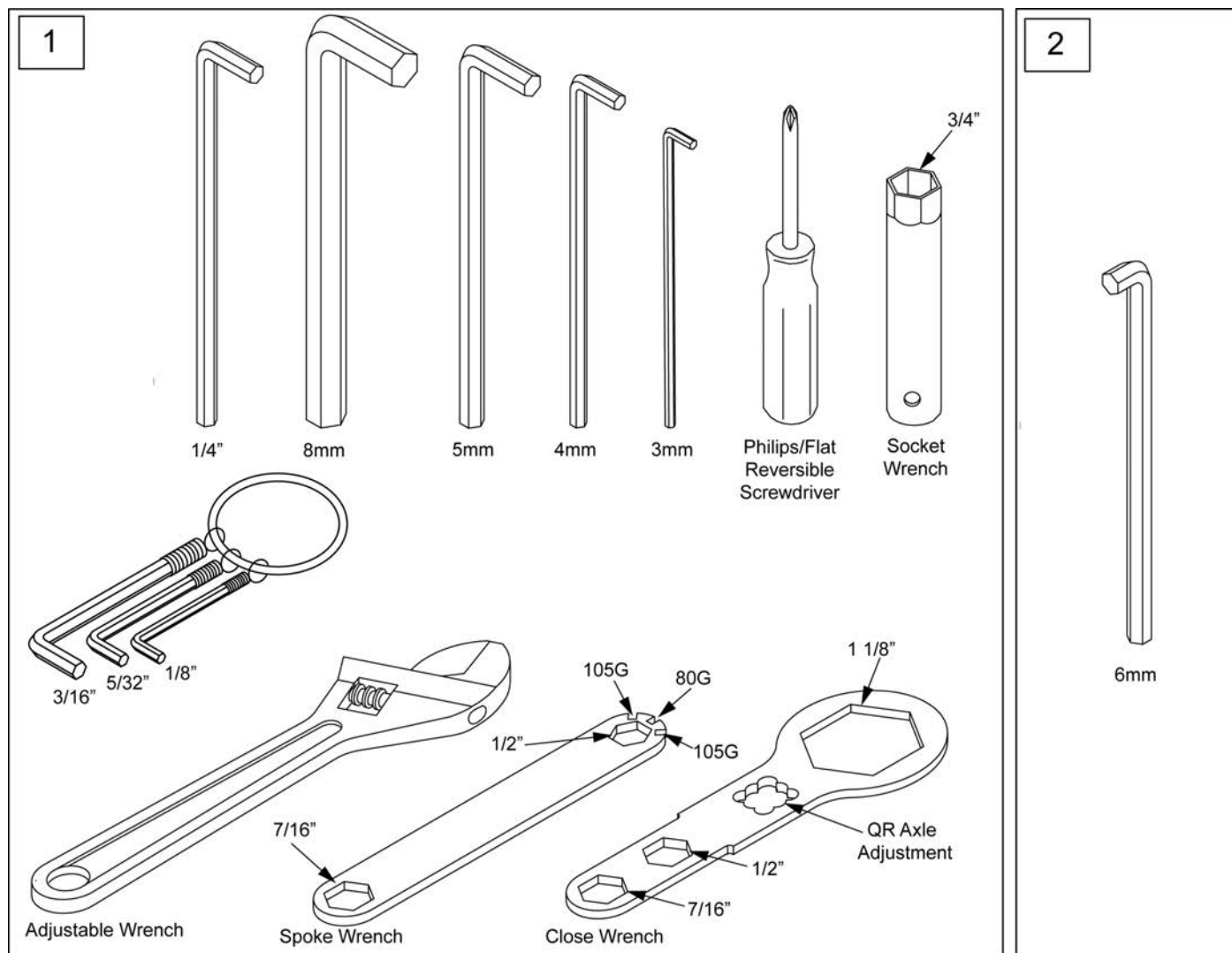

Pos.	Item Number	Description	Remarks
1	108314	QUICKIE COMBO TOOL KIT	Includes pouch
2	113403	6MM ALLEN WRENCH L, 7MM SHORT	caster spline chairs only