




PARTS MANUAL

JANUARY 2014

Front Cover	1
QUICKIE 2 LITE (Chairs w/ S/N Prefix Q2N & Q2L)	4
BASE FRAME	4
FRAMES	4
FIXED FRONT SIDE FRAME	4
SWING-AWAY FRONT SIDE FRAME	6
REAR SIDE FRAME	7
CROSS TUBE ASSEMBLY QUICKIE 2 LITE	8
TRANSIT KIT QUICKIE 2 LITE	11
AXLE PLATES & AXLES	11
QUICK RELEASE AXLES	12
AXLE PLATE ASSEMBLY	13
FRONT CASTERS & FORK	13
FRONT CASTER & FORK	14
FROG LEGS SUSPENSION FORK	18
CASTER PIN LOCKS	20
MICRO CASTERS	21
3 SPOKE CASTERS	22
SOFT ROLL CASTERS	24
ALUMINUM CASTERS	25
ALUMINUM SOFT ROLL CASTERS	26
ANTI-TIPS	26
ANTI-TIP	27
WHEEL LOCKS	27
PUSH TO LOCK WHEEL LOCK	28
PULL TO LOCK WHEEL LOCK	29
ERGO SCISSOR WHEEL LOCK	30
PUSH & PULL WITH GRADE AID ASSM	31
COMPACT WHEEL LOCK	32
WHEEL LOCK EXTENSION HANDLE	33
WHEELS & TIRES	33
WHEELS	33
MAG	34
LITE SPOKE	36
NEW ULTRA LITE SPOKE	37
SPINERGY SPOX (18 SPOKE)	38
SPINERGY LX (12 SPOKE)	39
MOUNTAIN	40
TIRES	40
PNEUMATIC & PNEU. AIRLESS TIRE	41
FULL POLY TIRE	42
HIGH PRESSURE CLINCHER TIRE	43
KNOBBY TIRE	44
SCHWALBE MARATHON PLUS TIRE	45
KIK MAKO TIRE	46
URBAN ASSAULT TIRE	47
MOUNTAIN TIRE	48
TUBES	48
TUBES AND AIRLESS INSERTS	49
SEATING AND POSITIONING	50
SEAT	50
SEAT SLING	51
SEAT CUSHION	53
BACK	53
STANDARD BACK FRAME	54
ANGLE ADJUSTABLE BACK FRAME	56
BACK UPHOLSTERY	57
FOLDING PUSH HANDLE ADJUSTABLE BACKREST UPHOLSTERY	58
ADJUSTABLE BACK UPHOLSTERY	59
INTEGRAL PUSH HANDLE BACKREST UPHOLSTERY	60
HANGERS & FOOTPLATES	60
FIXED FRONT FRAME FOOTPLATES	60
TUBULAR SIDE LATCH FOOTREST ASSM	61
AUTOFOLD FOOTREST ASSM	63
PLATFORM FLIP-UP FOOTPLATE	65
COMPOSITE FOOTPLATE	67
COMPOSITE ANGLE ADJUSTABLE FOOT PLATE	71
ADULT ANGLE-ADJ FOOTPLATE EXT MOUNT	75
S/A FRAME HANGERS	79
HANGERS	79

70 DEG HANGER SWING IN-OUT	79
70 DEG HANGER V SWING IN-OUT	80
80 DEG HANGER SWING IN-OUT	81
FOOTPLATES.....	81
COMPOSITE FOOTPLATE.....	82
COMPOSITE ANGLE ADJUSTABLE FOOT PLATE	86
ADULT ANGLE-ADJ FOOTPLATE EXT MOUNT	90
PLATFORM FLIP-UP FOOTPLATE.....	93
MULTI-POSITION LATCH STUD SUPPORT	95
ARMRESTS & RECEIVERS	95
ARMRESTS.....	95
FLIP BACK HEIGHT ADJUSTABLE LITE ARMREST	96
SINGLE POST HEIGHT ADJUSTABLE ARMREST	97
PADDED SWING-AWAY ARMREST	99
RECEIVERS.....	101
SINGLE POST ARMREST RECEIVER - STD	101
FLIP BACK ARMREST RECEIVER (LITE STYLE ARMREST)	103
PADDED SWING AWAY ARMREST RECEIVER.....	105
ACCESSORIES	105
BACKPACKS AND SEAT POUCHES.....	105
BACKPACKS AND SEAT POUCHES.....	106
CASE LOGIC BAGS.....	107
CELL PHONE PACK	108
SOFT FEEL LEG STRAP	109
ADJUSTABLE LEG STRAP	110
SPOKE GUARDS.....	111
SIDE GUARDS.....	113
CRUTCH HOLDER	114
QUICKIE CADDY	115
AUTO BUCKLE POSITIONING BELTS	116
AIRCRAFT BUCKLE POSITIONING BELT.....	117
TOOL KIT	118
TIRE PUMP	119
IMPACT GUARD NEOPRENE & GEL VELCRO PADS.....	120
Back Cover.....	121

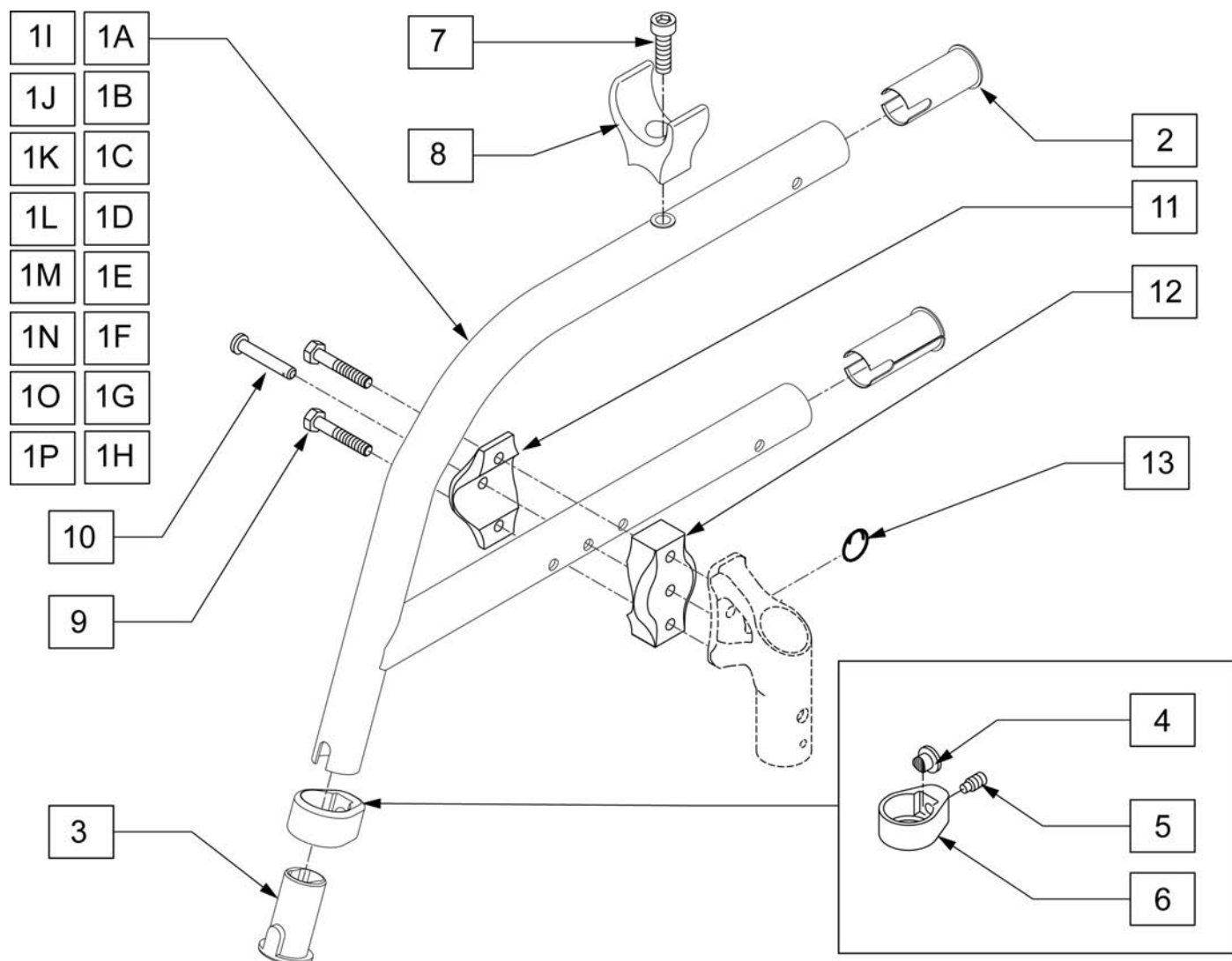


Pos.	Item Number	Description	Remarks
2	107487	FRAME BUSHING KIT (4)	Set of 4
3	104160	BUSHING FOOTREST	
13	349034	SAFETY COTTER	
3-6	107485	FOOTREST CLAMP ASSM	
1A-8	108250S	FX FRM STRT 70 DEG SML	14" Seat Depth,Short
1B-8	107467S	FIX FRAME STRAIGHT 70 DEG REG	15"-16" Seat Depth,REG
1C-8	107468S	FIX FRAME STRAIGHT 70 DEG LONG	17"-18" Seat Depth,LONG
1D-8	107469S	FIX FRAME STRAIGHT 70 DEG X-LONG	19"-20" Seat Depth,X-LONG
1E-8	108251S	FX FRM STRT 80 DEG SML	14" Seat Depth,Short
1F-8	107470S	FIX FRAME STRAIGHT 80 DEG REG	15"-16" Seat Depth,REG
1G-8	107471S	FIX FRAME STRAIGHT 80 DEG LONG	17"-18" Seat Depth,LONG
1H-8	107472S	FIX FRAME STRAIGHT 80 DEG X-LONG	19"-20" Seat Depth,X-LONG
1I-8	108252S	FX FRM NSET 70 DEG SML R	14" Seat Depth,Short,Right
1I-8	108253S	FX FRM NSET 70 DEG SML L	14" Seat Depth,Short,Left
1J-8	107473S	FIX FRAME INSET 70 DEG REG R	15"-16" Seat Depth,REG,RIGHT
1J-8	107474S	FIX FRAME INSET 70 DEG REG L	15"-16" Seat Depth,REG,LEFT
1K-8	107475S	FIX FRAME INSET 70 DEG LONG R	17"-18" Seat Depth,LONG,RIGHT
1K-8	107476S	FIX FRAME INSET 70 DEG LONG L	17"-18" Seat Depth,LONG,LEFT


Pos.	Item Number	Description	Remarks
1L-8	107477S	FIX FRAME INSET 70 DEG X-LONG R	19"-20" Seat Depth,X-LONG,RIGHT
1L-8	107478S	FIX FRAME INSET 70 DEG X-LONG L	19"-20" Seat Depth,X-LONG,LEFT
1M-8	108254S	FX FRM NSET 80 DEG SML R	14" Seat Depth,Short,Right
1M-8	108255S	FX FRM NSET 80 DEG SML L	14" Seat Depth,Short,Left
1N-9	107479S	FIX FRAME INSET 80 DEG REG R	15"-16" Seat Depth,REG,RIGHT
1N-9	107480S	FIX FRAME INSET 80 DEG REG L	15"-16" Seat Depth,REG,LEFT
1O-8	107481S	FIX FRAME INSET 80 DEG LONG R	17"-18" Seat Depth,LONG,RIGHT
1O-8	107482S	FIX FRAME INSET 80 DEG LONG L	17"-18" Seat Depth,LONG,LEFT
1P-8	107483S	FIX FRAME INSET 80 DEG X-LONG R	19"-20" Seat Depth,X-LONG,RIGHT
1P-8	107484S	FIX FRAME INSET 80 DEG X-LONG L	19"-20" Seat Depth,X-LONG,LEFT
7-8	107486	SEAT RAIL SADDLE KIT	
9-13	107589	FIX FRAME CASTER ATTACH KIT	

This diagram shows an exploded view of a mechanical assembly. The components are labeled with numbers in boxes: 1 (a long cylindrical shaft), 2 (a short cylindrical sleeve), 3 (a threaded pin), 4 (a bracket), 5 (a threaded pin with a washer), 6 (a small cylindrical bush), 7 (a screw), 8 (a bracket), 9 (a small cylindrical bush), 10 (a washer), 11 (a washer), and 12 (a cap nut). The assembly is shown in a disassembled state, with dashed lines indicating the alignment and assembly path of the parts.

Pos.	Item Number	Description	Remarks
2	107487	FRAME BUSHING KIT	Set of 4
3,4	107491	LATCH STUD KIT	
5	108623	BHCS 1/4-28X1-1/2 SHLD PL ROHS	
1A-4,7,8	107488S	S/A FRAME REG R	14"-16" Seat Depth,REG-Right
1A-4,7,8	107582S	S/A FRAME REG L	14"-16" Seat Depth,REG-Left
1B-4,7,8	107489S	S/A FRAME LONG R	17"-18" Seat Depth,LONG-Right
1B-4,7,8	107583S	S/A FRAME LONG L	17"-18" Seat Depth,LONG-Left
1C-4,7,8	107490S	S/A FRAME XLONG R	19"-20" Seat Depth,X-LONG-Right
1C-4,7,8	107584S	S/A FRAME XLONG L	19"-20" Seat Depth,X-LONG-Left
7-8	107486	SEAT & RAIL SADDLE KIT	
5,6,9-11	107588	S/A FRM CASTR ATTACH KIT	
12	560735	7/8" ROUND TUBE PLUG	Use in place of SA Hanger

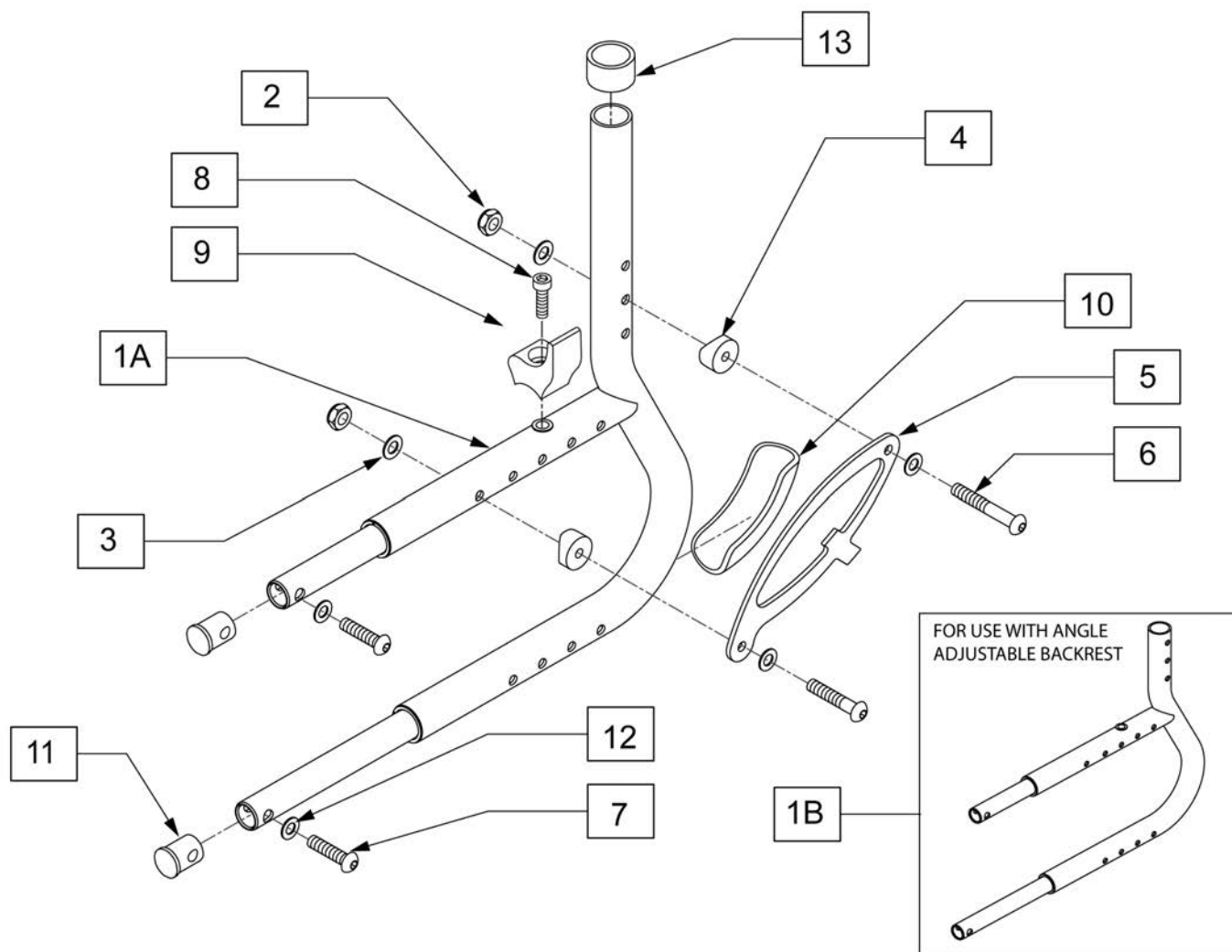

Pos.	Item Number	Description	Remarks
2	106935	NUT NYLOCK THIN 1/4-28 ROHS	
3	106944	WASHER FLAT THIN AN 1/4 ROHS	
4	561087	TUBE SADDLE	
5	104037	BACKREST GUSSET Q	
6	106962	BHCS 1/4-28 X 1-3/4 ROHS	
7	107353	1/4-20 X 1 BH C/S	
8,9	107486	SEAT RAIL SADDLE KIT	
10	104666	WEAR PAD FRAME ASSY	
11	107494	RETAINING PLUG KIT (4)	Set of 4 plugs only
12	104414	WAVE WASHER 6 MM	
13	561018	1" TUBE END BUSHING THICK	
1A-13	107492S	REAR SDE FRM ASSM STD BCK R	STANDARD BACK-Right
1A-13	107585S	REAR SDE FRM ASSM STD BCK L	STANDARD BACK-Left
1B-13	107493S	REAR SDE FRM ASSM ANG BCK R	ANGLE ADJUSTABLE BACK-Right
1B-13	107586S	REAR SDE FRM ASSM ANG BCK L	ANGLE ADJUSTABLE BACK-Left

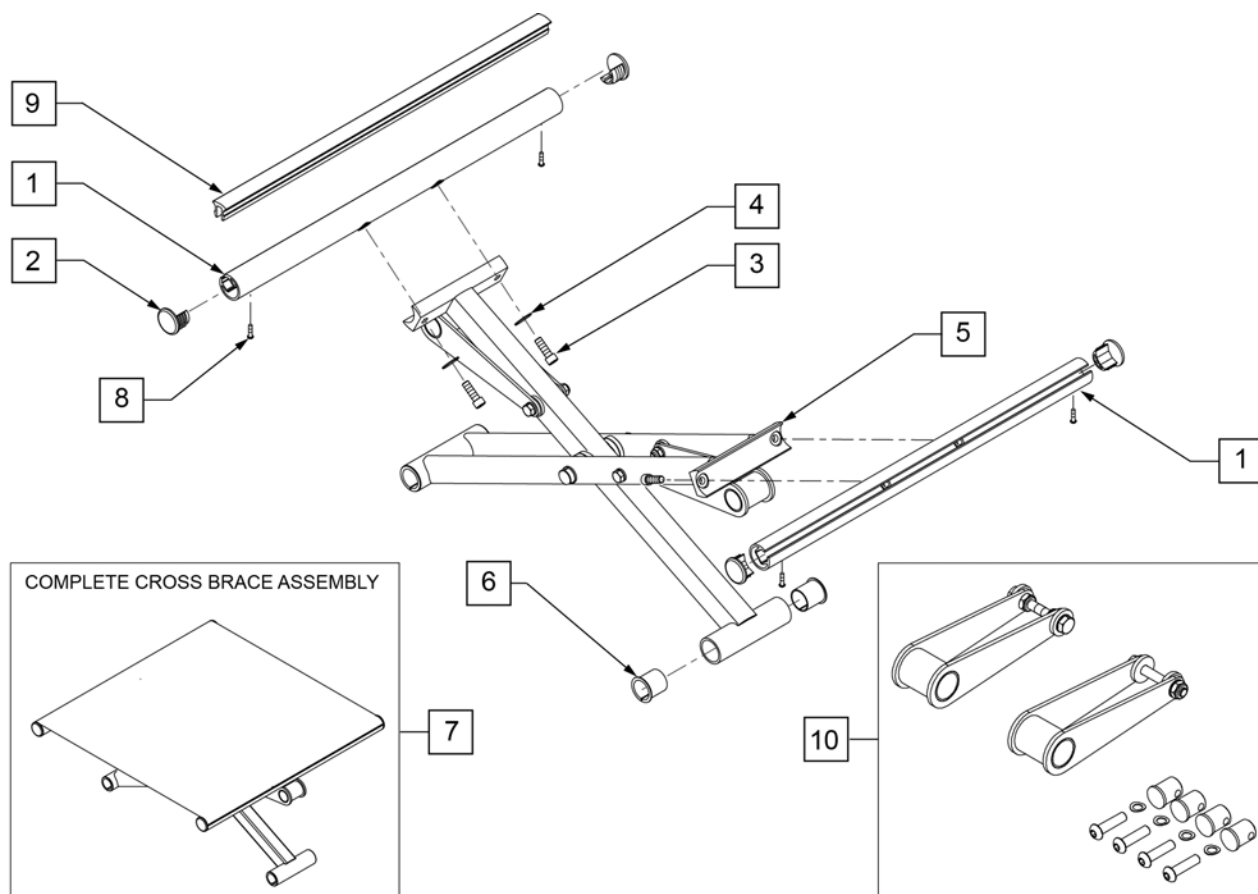


Pos.	Item Number	Description	Remarks
3	106974	SHCS 1/4-28 X 3/4 PL ROHS	
4	106944	WASHER FLAT THIN AN 1/4 ROHS	
6	108116	3/4" NY LINER (4)	SET OF 4
7	107502	X BRACE ASSM 14X14	
7	107503	X BRACE ASSM 14X15	
7	107504	X BRACE ASSM 14X16	
7	107505	X BRACE ASSM 14X17	
7	107506	X BRACE ASSM 14X18	
7	107507	X BRACE ASSM 14X19	
7	107508	X BRACE ASSM 14X20	
7	107509	X BRACE ASSM 15X14	
7	107510	X BRACE ASSM 15X15	
7	107511	X BRACE ASSM 15X16	
7	107512	X BRACE ASSM 15X17	
7	107514	X BRACE ASSM 15X18	
7	107515	X BRACE ASSM 15X19	
7	107516	X BRACE ASSM 15X20	
7	107517	X BRACE ASSM 16X14	
7	107518	X BRACE ASSM 16X15	
7	107519	X BRACE ASSM 16X16	
7	107520	X BRACE ASSM 16X17	
7	107521	X BRACE ASSM 16X18	
7	107522	X BRACE ASSM 16X19	
7	107523	X BRACE ASSM 16X20	

Pos.	Item Number	Description	Remarks
7	107524	X BRACE ASSM 17X14	
7	107525	X BRACE ASSM 17X15	
7	107526	X BRACE ASSM 17X16	
7	107527	X BRACE ASSM 17X17	
7	107528	X BRACE ASSM 17X18	
7	107529	X BRACE ASSM 17X19	
7	107530	X BRACE ASSM 17X20	
7	107531	X BRACE ASSM 18X14	
7	107532	X BRACE ASSM 18X15	
7	107533	X BRACE ASSM 18X16	
7	107534	X BRACE ASSM 18X17	
7	107535	X BRACE ASSM 18X18	
7	107536	X BRACE ASSM 18X19	
7	107537	X BRACE ASSM 18X20	
7	107538	X BRACE ASSM 19X14	
7	107539	X BRACE ASSM 19X15	
7	107540	X BRACE ASSM 19X16	
7	107541	X BRACE ASSM 19X17	
7	107542	X BRACE ASSM 19X18	
7	107543	X BRACE ASSM 19X19	
7	107544	X BRACE ASSM 19X20	
7	107545	X BRACE ASSM 20X14	
7	107546	X BRACE ASSM 20X15	
7	107547	X BRACE ASSM 20X16	
7	107548	X BRACE ASSM 20X17	
7	107549	X BRACE ASSM 20X18	
7	107550	X BRACE ASSM 20X19	
7	107551	X BRACE ASSM 20X20	
8	109819	SCREW #6 x 1/4 SLF TP PHL PN HD	
9	580024	SEAT RAIL INSERT STRIP 14"	
9	580025	SEAT RAIL INSERT STRIP 15"	
9	580026	SEAT RAIL INSERT STRIP 16"	
9	580027	SEAT RAIL INSERT STRIP 17"	
9	580028	SEAT RAIL INSERT STRIP 18"	
9	580029	SEAT RAIL INSERT STRIP 19"	
9	580030	SEAT RAIL INSERT STRIP 20"	
10	115899	Q2 LT X-HINGE REPLACEMENT KIT 14"	
10	115900	Q2 LT X-HINGE REPLACEMENT KIT 15"	
10	115901	Q2 LT X-HINGE REPLACEMENT KIT 16"	
10	115902	Q2 LT X-HINGE REPLACEMENT KIT 17"	
10	115903	Q2 LT X-HINGE REPLACEMENT KIT 18"	
10	115904	Q2 LT X-HINGE REPLACEMENT KIT 19"	
10	115905	Q2 LT X-HINGE REPLACEMENT KIT 20"	
2,8	108117	END PLUG KIT (4)	SET OF 4
5,6	107615	XTUBE W/ SERIAL LBL 14W	14W
5,6	107616	XTUBE W/ SERIAL LBL 15W	15W
5,6	107617	XTUBE W/ SERIAL LBL 16W	16W
5,6	107618	XTUBE W/ SERIAL LBL 17W	17W
5,6	107619	XTUBE W/ SERIAL LBL 18W	18W
5,6	107620	XTUBE W/ SERIAL LBL 19W	19W
5,6	107621	XTUBE W/ SERIAL LBL 20W	20W
1-4,8	107495	SEAT RAIL KIT 14D	14 DEEP-Each
1-4,8	107496	SEAT RAIL KIT 15D	15 DEEP-Each
1-4,8	107498	SEAT RAIL KIT 17D	17 DEEP-Each
1-4,8	107497	SEAT RAIL KIT 16D	16 DEEP-Each
1-4,8	107499	SEAT RAIL KIT 18D	18 DEEP-Each






Pos.	Item Number	Description	Remarks
1-4,8	107500	SEAT RAIL KIT 19D	19 DEEP-Each
1-4,8	107501	SEAT RAIL KIT 20D	20 DEEP-Each

NOTE: In order to properly mount a JAY DUO CUSHION; Qty (2) of the appropriate size SEAT RAIL INSERT (Item 9) must be ordered.


NOTE: Kit Includes all items for adding the Transit Option to an existing chair.

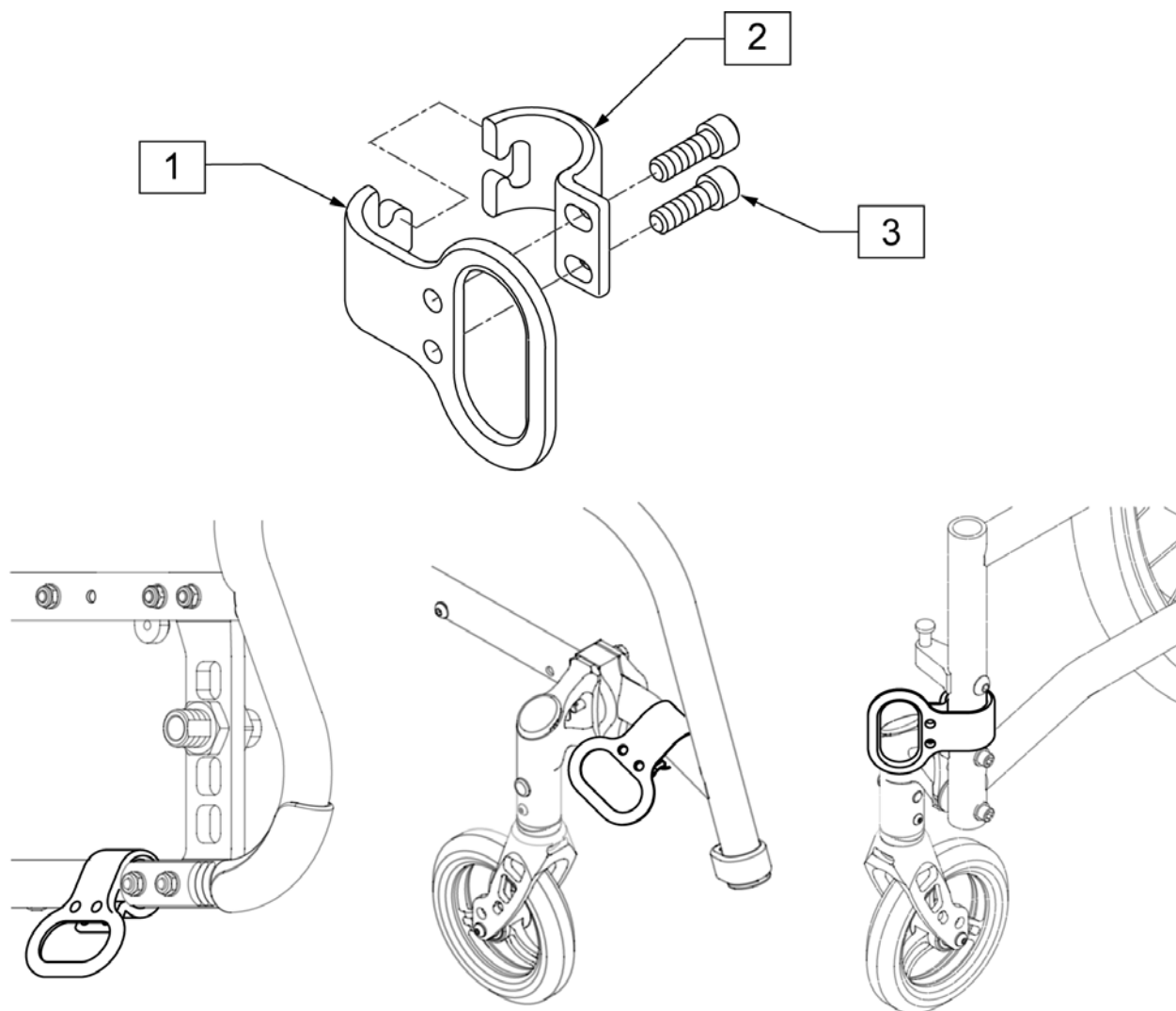


Pos.	Item Number	Description
3	107350	SHCS 1/4-28 X 1/2 PL ROHS
1-3	109277	TRANSIT KIT Q2 LITE

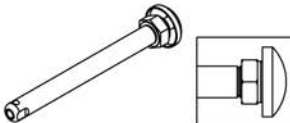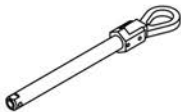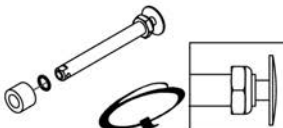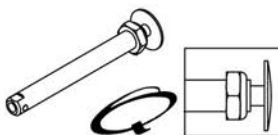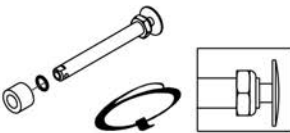Wheel Type	Axle Type		Axle P/N	Spacer P/N
Mag Wheels	Quick Release	1A 	102063 A=4 1/8	N/A
	Quad Quick Release	1B 	920171 A=4 1/8	N/A
Spinergy Wheels & Mountain Wheels	Quick Release	1C 	109385 A=4 1/2	N/A
Lite Spoke Wheels	Quick Release	1D 	107268 A=4 1/4	N/A
Ultra Lite Wheels	Quick Release	1E 	100793 Order Spacer A = 4 1/2"	2C 103093 2D 100768-001

Quick Release Measuring Guide

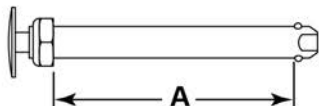

Pos.	Item Number	Description
1A	102063	QR AXLE MAG W/M-12 NUT
1B	920171	QR AXLE MAG W/QUAD NUT
1C	102064	QR AXLE M12 NUT CLR BIG BTTN
1D	107268	QR AXLE AND NUT SHORT BIG BTTN
1E	100793	AXLE QR SHORT ULTRALITE
2C	103093	O-RING 1/2" ID x 11/16" OD BUNA-N
2D	100768-001	SPACER AXLE 22 OD x 12.7 ID x 7mm

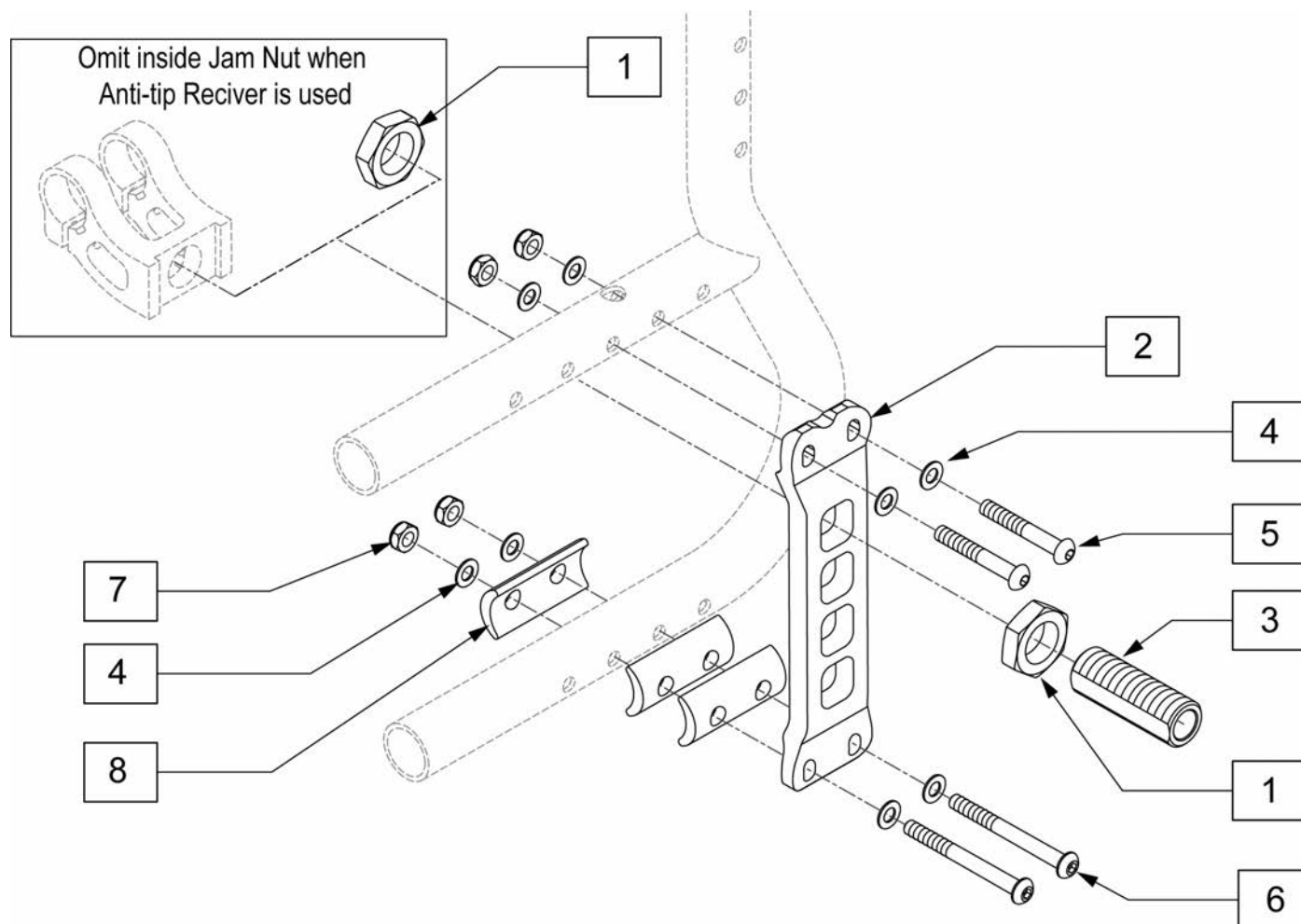


Pos.	Item Number	Description	Remarks
1	104195	NUT JAM ALUMINUM 20MM X 1.5	
2	104265	AXLE PLATE LEFT	Left
2	104266	AXLE PLATE RIGHT	Right
3	104196	AXLE SLEEVE 20MM NONADJ	
4	106943	FLAT AN 1/4 ROHS	
5	107252	BHCS 1/4-28 X 1-5/8 ROHS	
6	106966	BHCS 1/4-28 X 2-1/4 SHLDR ROHS	
7	106935	NYLOCK NUT THIN 1/4-28 ROHS	
8	107554	CAMBER SPACER KIT	(Set of 6) 1 Spacer provides 2 deg of camber.
1-8	107552	AXLE PLATE KIT LEFT	Left
1-8	107553	AXLE PLATE KIT RIGHT	Right

Note: Item 26 is used for additional barrel adjustment. Washers can be added or removed in order to ensure caster barrel is perpendicular to the ground.

Note: As of S/N Q2L-149887 Stem assembly has changed. If replacing stem components on chairs prior to S/N Q2L-149887, replace complete Stem Assembly in pairs.

Part Changes, please see notes/remarks for important information

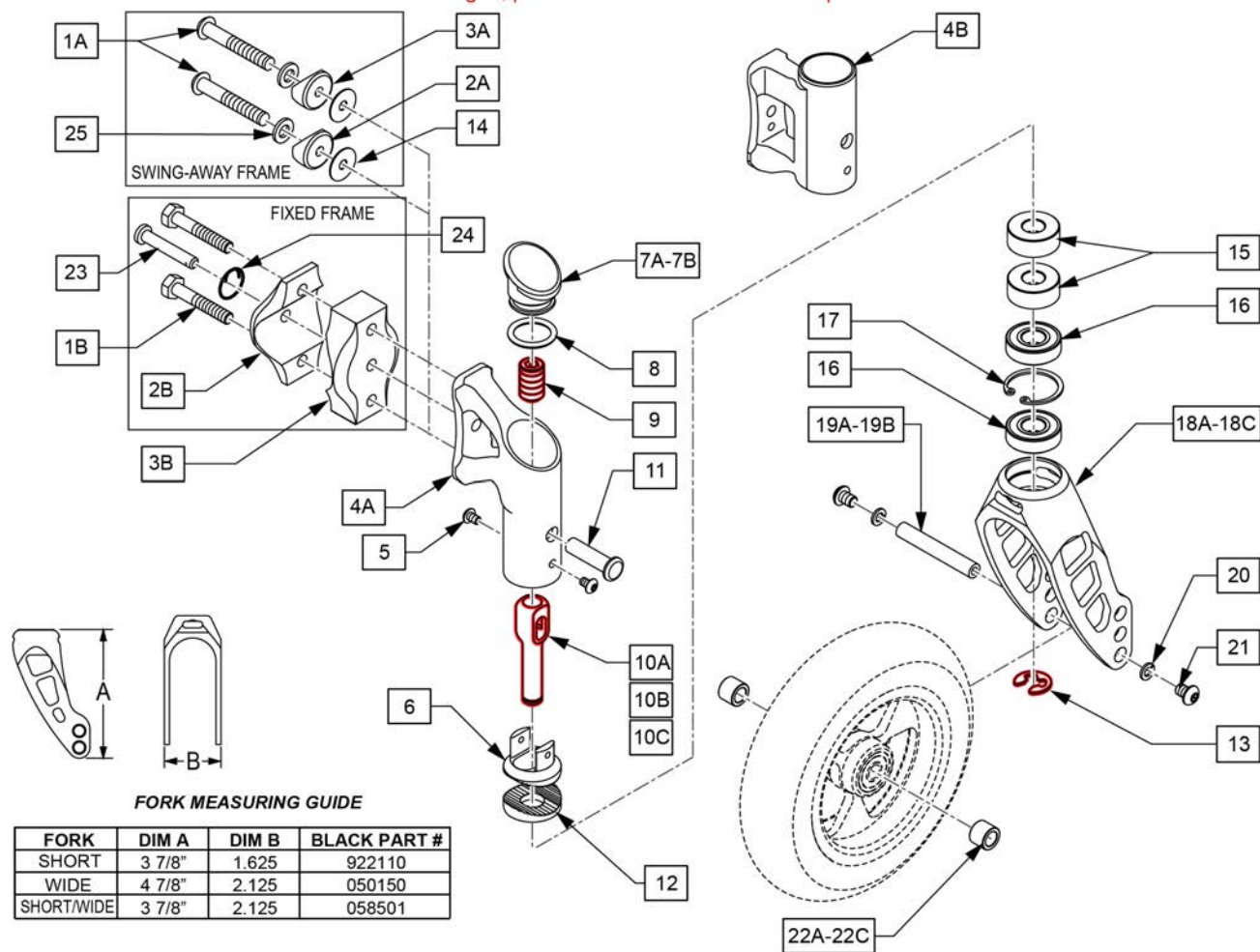


Pos.	Item Number	Description	Remarks
1A	108623	BHCS 1/4-28X1-1/2 SHLD PL ROHS	
1B	106972	HHCS 1/4-28 X 1-1/2 ROHS	
7A,8	104096	CASTER CAP WITH O RING	Standard Caster Cap
7B,8	104120	CASTER CAP HEMI WITH O RING	Hemi Caster Cap
9	106684	SH SET SCREW 1/2-20X3/4 FLT PT	
10A	122989	CASTER STEM ADJUSTABLE SHORT	
10B	122993	CASTER STEM ADJUSTABLE MED	

Pos.	Item Number	Description	Remarks
10C	122994	CASTER STEM ADJUSTABLE TALL	
12	104182	SPLINE CSTR ANGLE ADJ LOWER	
13	114479	E-STYLE BOWED CLIP	
14	113268	WASHER NYLON 1" OD x .312 ID	
15	483275	SPACER, CSTR, ANGLER ADJ .50	Not R'qd with Std stem bolt, Qty 1 R'qd with .5" stem bolt, Qty 2 R'qd with 1" Stem bolt
16	349104	BRNG 28mm OD 12mm ID 8mm THK	
19A	050042	AXLE CASTER 52MM	2" - Used w/ wide fork & short wide fork
19B	481483	AXLE CASTER	1 1/2" - Used w/ short fork
20	262132	WASHER LOCK M5	
21	102259-001	BHCS M5 X .8 X 12 W/PL INORG	
22A	481353	SPACER, BEARING	Used w/ short fork Only
22B	050043-001	SPCR BRNG CSTR 9.65MM	Used w/ wide fork & short wide fork - not used w/ Soft Roll Casters
22C	050043-002	SPCR BRNG CSTR 2.20MM	Used w/ wide fork & short wide fork - Soft Roll Casters
25	106944	WASHER FLAT THIN AN 1/4 ROHS	
16-18A	922111	ASSM INTGRL BRNG FORK SHRT BLK	Short fork
16-18B	050150	ASSM FORK INT BRG WIDE BLK	Wide fork

Pos.	Item Number	Description	Remarks
16-18C	108771	FORK SHORT WIDE ASSM	Short wide fork
4A,5-13,16-18A,19B,20-22A	107555	CSTR ASM SHRT STD	Std caster housing,short fork,std stem
4A,5-13,15-18A,19B,20-22A	107556	CSTR ASM SHRT +.5	Std caster housing,short fork,+.5 stem
4A,5-13,15-18A,19B,20-22A	107557	CSTR ASM SHRT +1	Std caster housing,short fork,+1stem
4A,5-13,16-18C,19B,20-22A	107558	CSTR ASM WD S STD	Std caster housing,short wide fork,std stem
4A,5-13,15-18C,19A,20-22A	107559	CSTR ASM WD S +.5	Std caster housing,short wide fork,+.5 stem
4A,5-13,15-18C,19A,20-22A	107560	CSTR ASM WD S +1	Std caster housing,short wide fork,+1stem
4A,5-13,16-18B,19B,20-22B	107562	CSTR ASM WIDE STD	Std caster housing, wide fork,std stem
4A,5-13,15-18B,19B,20-22B	107563	CSTR ASM WIDE +.5	Std caster housing, wide fork,+.5 stem
4A,5-13,15-18B,19B,20-22B	107564	CSTR ASM WIDE +1	Std caster housing, wide fork,+1stem
4A,5-13,16-18B,19B,20-22C	107565	CSTR ASM WD SR STD	Std caster housing, wide fork for soft roll caster,std stem
4A,5-13,15-18B,19B,20-22C	107566	CSTR ASM WD SR +.5	Std caster housing, wide fork for soft roll caster,+.5 stem
4A,5-13,15-18B,19B,20-22C	107567	CSTR ASM WD SR +1	Std caster housing, wide fork for soft roll caster,+1stem
4B,5-13,16-18A,19B,20-22A	107568	HCSTR ASM SHRT STD	Hemi caster housing,short fork,std stem

Pos.	Item Number	Description	Remarks
4B,5-13,15-18A,19B,20-22A	107569	HCSTR ASM SHRT +.5	Hemi caster housing,short fork,+.5 stem
4B,5-13,15-18A,19B,20-22A	107570	HCSTR ASM SHRT +1	Hemi caster housing,short fork,+1stem
4B,5-13,16-18C,19A,20-22A	107571	HCSTR ASM WD S STD	Hemi caster housing,short wide fork,std stem
4B,5-13,15-18C,19A,20-22A	107572	HCSTR ASM WD S +.5	Hemi caster housing,short wide fork,+.5 stem
4B,5-13,15-18C,19A,20-22A	107573	HCSTR ASM WD S +1	Hemi caster housing,short wide fork,+1stem
4B,5-13,16-18B,19B,20-22B	107574	HCSTR ASM WIDE STD	Hemi caster housing, wide fork,std stem
4B,5-13,15-18B,19B,20-22B	107575	HCSTR ASM WIDE +.5	Hemi caster housing, wide fork,+.5 stem
4B,5-13,15-18B,19B,20-22B	107576	HCSTR ASM WIDE +1	Hemi caster housing, wide fork,+1stem
4B,5-13,16-18B,19B,20-22C	107577	HCSTR ASM WD SR STD	Hemi caster housing, wide fork for soft roll caster,std stem
4B,5-13,15-18B,19B,20-22C	107578	HCSTR ASM WD SR +.5	Hemi caster housing, wide fork for soft roll caster,+.5 stem
4B,5-13,15-18B,19B,20-22C	107579	HCSTR ASM WD SR +1	Hemi caster housing, wide fork for soft roll caster,+1stem
1A,2A,3A,14,25	107588	S/A FRM ATTACH KIT	
1B,2B,3B,23,24	107589	FIX FRM ATTACH KIT	
4A-10A,11,12	126079	STD CASTR HOUSE ASSM STD STEM	
4A-10B,11,12	126080	STD CASTR HOUSE ASSM +.5 STEM	
4A-10C,11,12	126081	STD CASTR HOUSE ASSM +1 STEM	
4B-10A,11,12	126082	HEMI CASTR HOUSE ASSM STD STEM	
4B-10B,11,12	126083	HEMI CAST HOUSE ASSM +.5 STEM	
4B-10C,11,12	126084	HEMI CASTR HOUSE ASSM +1 STEM	

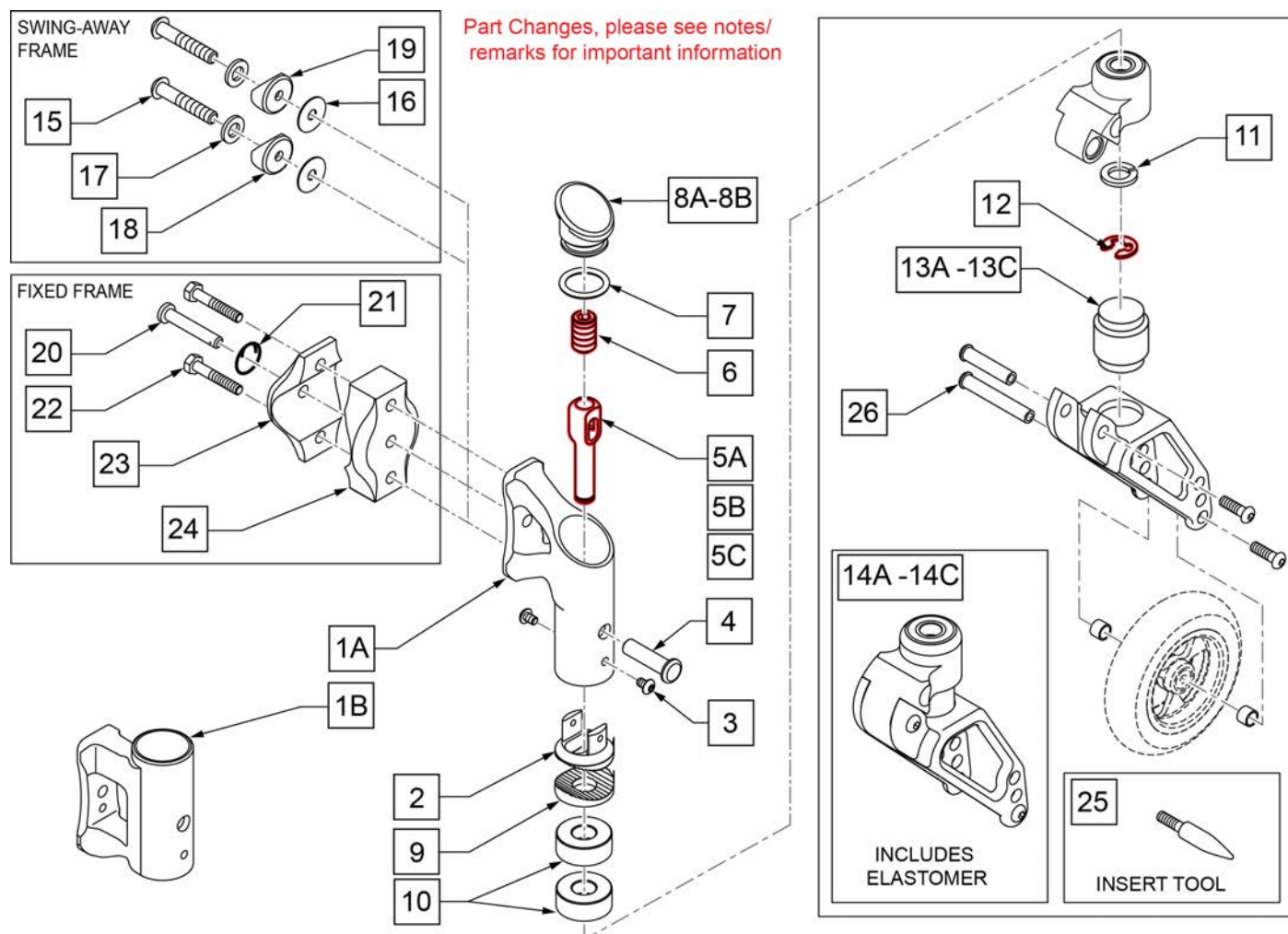

Pos.	Item Number	Description	Remarks
5A	122989	CASTER STEM ADJUSTABLE SHORT	
5B	122993	CASTER STEM ADJUSTABLE MED	
5C	122994	CASTER STEM ADJUSTABLE TALL	
6	106684	SH SET SCREW 1/2-20X3/4 FLT PT	
7,8A	104096	CASTER CAP WITH O RING	
7,8B	104120	CASTER CAP HEMI WITH O RING	
9	502051	SPLINE CSTR ANGLE ADJ LWR	
10	483275	SPACER, CSTR, ANGLER ADJ .50	See Note
11	107359	WASHER LOCK M12	
12	114479	E-STYLE BOWED CLIP	
13A	050156-001	ELASTOMER FROGLEGS SOFT	0-100 lbs
13B	050156-002	ELASTOMER FROGLEGS MEDIUM	101-190 lbs
13C	050156-003	ELASTOMER FROGLEGS FIRM	191-265 lbs
14A	050225-001	FORK FROGLEGS INT BRNG SOFT	0-100 lbs
14B	050225-002	FORK FROGLEGS INT BRNG MED	101-190 lbs
14C	050225-003	FORK FROGLEGS INT BRNG FIRM	191-265 lbs
15	108623	BHCS 1/4-28X1-1/2 SHLD PL ROHS	
16	113268	WASHER NYLON 1IN OD X .312 ID	
17	106944	WASHER FLAT THIN AN 1/4 ROHS	
18	561086	1" HALF SADDLE 1/4" HOLE	

Pos.	Item Number	Description	Remarks
19	561004	1" SADDLE 1/4" HOLE	
20	349027	1/4 X 2 CLEVIS PIN	
21	349034	SAFETY COTTER	
22	106972	HHCS 1/4-28 X 1-1/2 ROHS	
23	500306	INNER CLAMP HALF	
24	500307	OUTER CLAMP HALF	
25	116351	FROG LEG FORK ASSEMBLY TOOL	
15-19	107588	S/A FRM ATTACH KIT	
20-24	107589	FIX FRM ATTACH KIT	
1A-6	104092	CASTER HOUSING STD ASSM	
1B-6	104107	CASTER HOUSING HEMI ASSM	
1A,2-7,8A,9,11,14A	108062	FRG LG CSTR ASM SOFT STD	0-100 lbs, Std Caster Housing, Std Stem
1A,2-7,8A,9-11,14A	108063	FRG LG CSTR ASM SOFT +.5	0-100 lbs, Std Caster Housing, +.5"Stem
1A,2-7,8A,9-11,14A	108064	FRG LG CSTR ASM SOFT +1	0-100 lbs, Std Caster Housing, +1"Stem
1A,2-7,8A,9,11,14B	108072	FRG LG CSTR ASM MED STD	101-190 lbs, Std Caster Housing, Std Stem
1A,2-7,8A,9-11,14B	108073	FRG LG CSTR ASM MED +.5	101-190 lbs, Std Caster Housing, +.5"Stem
1A,2-7,8A,9-11,14B	108074	FRG LG CSTR ASM MED +1	101-190 lbs, Std Caster Housing, +1"Stem
1A,2-7,8A,9,11,14C	108075	FRG LG CSTR ASM FIRM STD	191-265 lbs, Std Caster Housing, Std Stem
1A,2-7,8A,9-11,14C	108076	FRG LG CSTR ASM FIRM +.5	191-265 lbs, Std Caster Housing, +.5"Stem
1A,2-7,8A,9-11,14C	108077	FRG LG CSTR ASM FIRM +1	191-265 lbs, Std Caster Housing, +1"Stem
1B,2-7,8A,9,11,14A	108078	FRG LG HCSTR ASM SOFT STD	0-100 lbs, Hemi Caster Housing, Std Stem
1B,2-7,8A,9-11,14A	108079	FRG LG HCSTR ASM SOFT +.5	0-100 lbs, Hemi Caster Housing, +.5"Stem
1B,2-7,8A,9-11,14A	108080	FRG LG HCSTR ASM SOFT +1	0-100 lbs, Hemi Caster Housing, +1"Stem
1B,2-7,8A,9,11,14B	108081	FRG LG HCSTR ASM MED STD	101-190 lbs, Hemi Caster Housing, Std Stem
1B,2-7,8A,9-11,14B	108082	FRG LG HCSTR ASM MED +.5	101-190 lbs, Hemi Caster Housing, +.5"Stem
1B,2-7,8A,9-11,14B	108083	FRG LG HCSTR ASM MED +1	101-190 lbs, Hemi Caster Housing, +1"Stem
1B,2-7,8A,9,11,14C	108084	FRG LG HCSTR ASM FIRM STD	191-265 lbs, Hemi Caster Housing, Std Stem
1B,2-7,8A,9-11,14C	108085	FRG LG HCSTR ASM FIRM +.5	191-265 lbs, Hemi Caster Housing, +.5"Stem
1B,2-7,8A,9-11,14C	108086	FRG LG HCSTR ASM FIRM +1	191-265 lbs, Hemi Caster Housing, +1"Stem


Note: Item 10 Bearing Spacer not Required with Standard stem Bolt, 1 Required with .5" Stem bolt, 2 Required with 1" Stem bolt.

Note: Item 16 is used for additional barrel adjustment. Washers can be added or removed in order to ensure caster barrel is perpendicular to the ground.


Note: Items 25 and 26 are used to aid in aligning the upper and lower fork assembly during Frog Leg assembly and elastomer replacement. See instruction sheet for details.

Note: Frog Legs will ship unassembled to facilitate replacement of the Caster Stem.

Note: Caster Pin Locks will not adapt to Short Wide Forks.

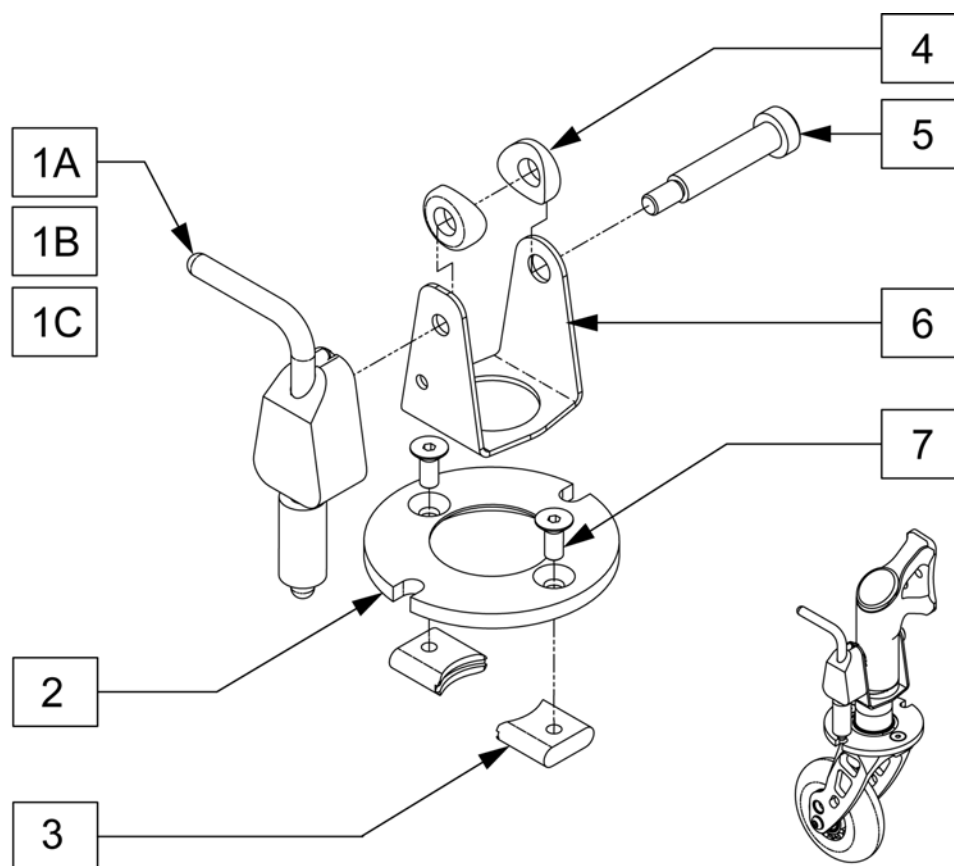

Pos.	Item Number	Description	Remarks
1A	052310-001	BLOCK ASSY CSTR PIN LCK R STD	Right - Used w/ Std Length Caster Stem
1A	052311-001	BLOCK ASSY CSTR PIN LCK L STD	Left - Used w/ Std Length Caster Stem
1B	052310-002	BLOCK ASSY CSTR PIN LCK R+.5	Right - Used w/ +.5" Length Caster Stem
1B	052311-002	BLOCK ASSY CSTR PIN LCK L +.5	Left - Used w/ +.5" Length Caster Stem
1C	052310-003	BLOCK ASSY CSTR PIN LOCK R+1	Right - Used w/ +1" Length Caster Stem
1C	052311-003	BLOCK ASSY CSTR PIN LCK +1	Left - Used w/ +1" Length Caster Stem
2	050047	PLATE CASTER LOCK ANG ADJ	
3	050050	KEY CASTER LOCK ANG ADJ	
4	050053	SADDLE 17.8MM OD 8MM HOLE	
5	050133	SHOULDER SCREW 8MMOD X 35MM LG	
6	107204	YOKE CASTER PIN LOCK	
7	102261	FHCS M5 X 12 INORGANIC	
1A,2-7	107622	CSTR PIN LOCK R STD ASSM	Right - Used w/ Std Length Caster Stem
1B,2-7	107623	CSTR PIN LOCK R +.5 ASSM	Right - Used w/ +.5" Length Caster Stem
1C,2-7	107624	CSTR PIN LOCK R +1 ASSM	Right - Used w/ +1" Length Caster Stem
1A,2-7	107625	CSTR PIN LOCK L STD ASSM	Left - Used w/ Std Length Caster Stem
1B,2-7	107626	CSTR PIN LOCK L +.5 ASSM	Left - Used w/ +.5" Length Caster Stem
1C,2-7	107627	CSTR PIN LOCK L +1 ASSM	Left - Used w/ +1" Length Caster Stem

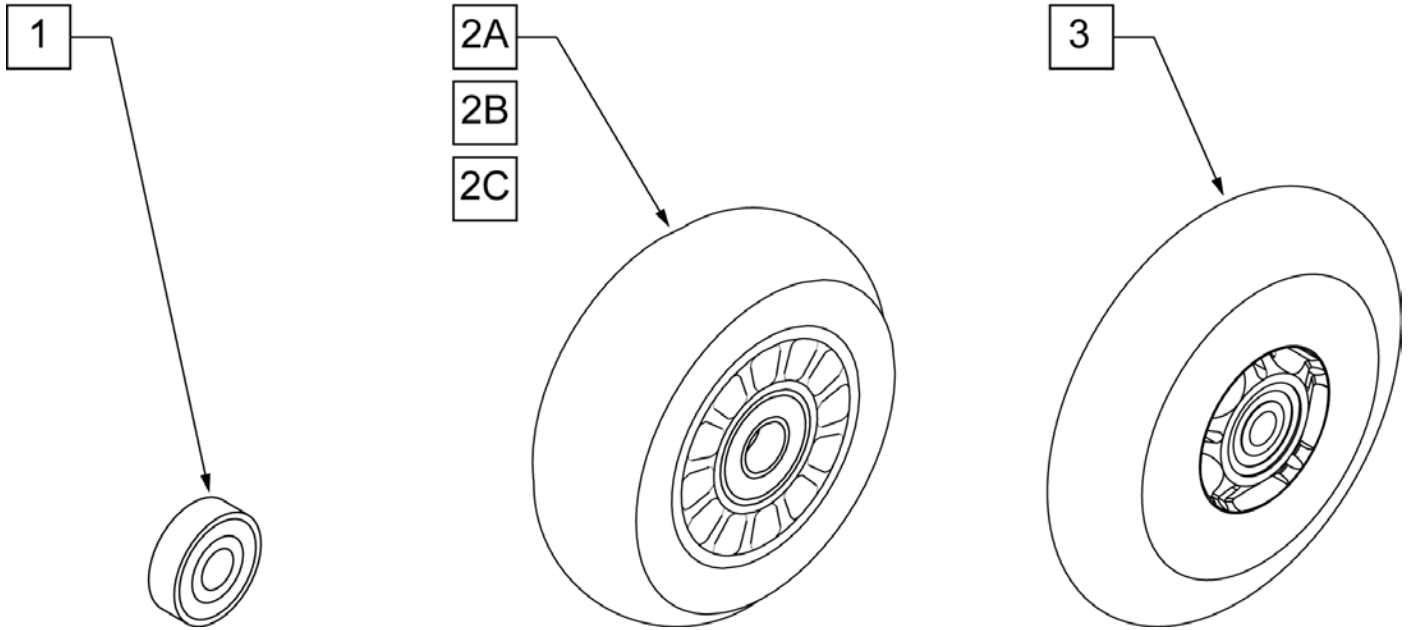


Pos.	Item Number	Description	HCPC	Remarks
1	349502	7/8"x 5/16" BEARING		
2A	400108	4" MICRO CASTER LIGHTED	K0077	4" MICRO LIGHTED
2B	920025	3" MICRO CASTER W/ BEARING -PAIR	K0077	PAIR 3" MICRO
2C	400107	3" MICRO CASTER LIGHTED	K0077	3" MICRO LIGHTED
3	052129	5" MICRO CASTER LIGHTED	K0077	5" MICRO LIGHTED


NOTE: 7" and 8" Casters are not available on the Q2 or the GT.

Note: If using 6" casters on Q7, Anti Flutter should be installed. See Standard Multi Position Fork page.


Note: Casters wider than 1" will not fit on Quickie Ti Titanium forks.

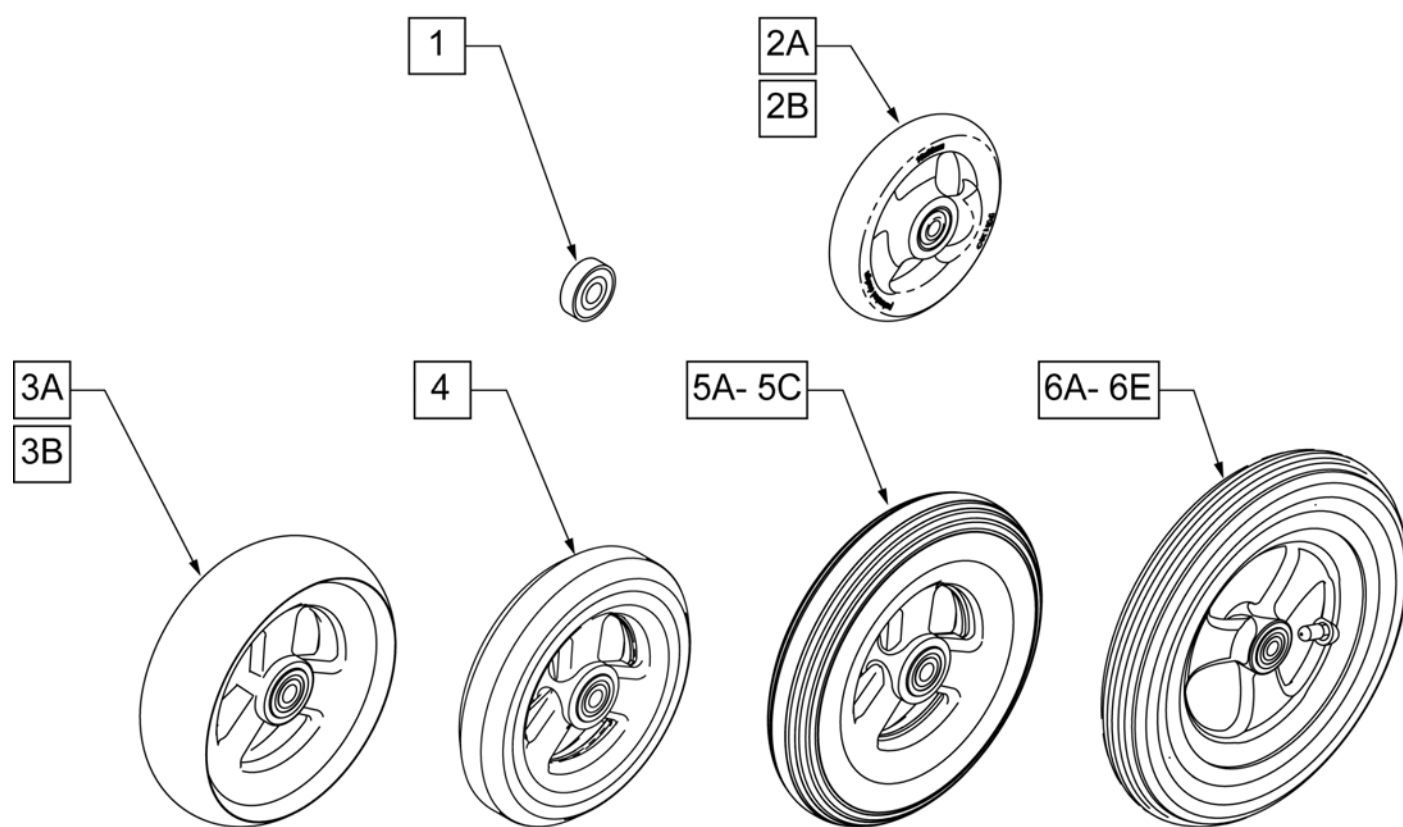

Pos.	Item Number	Description	Remarks
1	349502	7/8"x 5/16" BEARING	
2A	101957	4"x1-1/4 POLY 3 SPOKE CSTR	4" X 1.25 SEMI-PNEUMATIC
2B	101098	CASTER 4" PLASTIC/POLY W/BRG	4" LOW PROFILE POLY
3A	101259	5"x1-1/2" POLY 3 SPOKE CSTR	5" x 1.5" SEMI-PNEUMATIC
3B	101255	6"x1-1/2" POLY 3 SPOKE CSTR	6" x 1.5" SEMI-PNEUMATIC
4	101254	5" LOW POLY 3 SPOKE CASTER	5" LOW PROFILE POLY
5A	101256	6" SEMI PNEU 3 SPOKE CASTER	6" POLY
5B	101257	6" PNEUMATIC 3 SPOKE CASTER	6" PNEUMATIC
5C	383245	6" PNEUMATIC TIRE & TUBE	
6A	400212	CASTER 8" PNEU 3 SPK ASSM	8" PNEUMATIC
6B	400213	CASTER 8" POLY 3 SPK ASSM	8" POLY
6C	400214	CASTER 8" LOW POLY 3 SPK ASSM	7" x LOW PROFILE POLY
6D	383255	8" PNEUMATIC TIRE & TUBE	
6E	400215	CASTER 8" WIDE POLY 3 SPK ASSM	8" x 1.5" LOW POLY

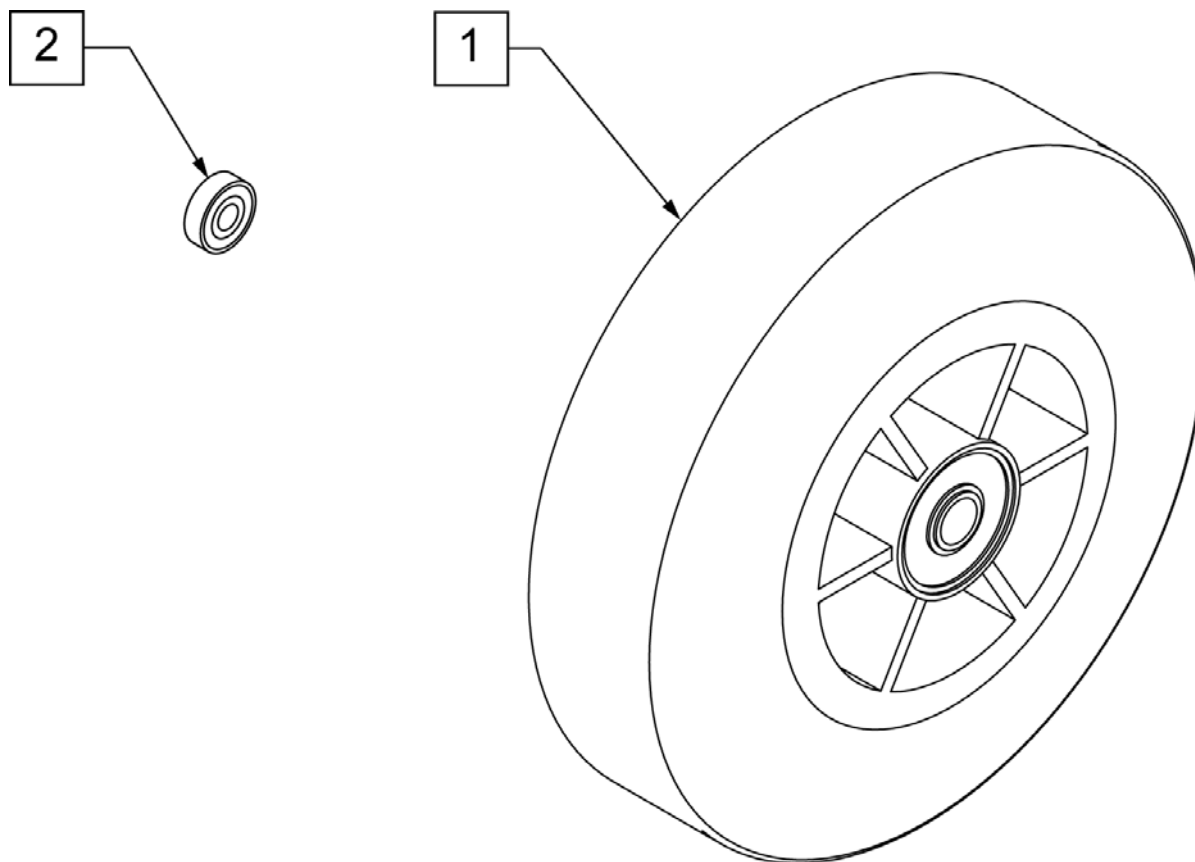


Pos.	Item Number	Description	HCPC	Remarks
1	400103	CASTER 5" SOFT ROLL W/BRNGS	K0077	5" SOFT ROLL SEMI PNEU (GREY)
2	349502	7/8"x 5/16" BEARING		

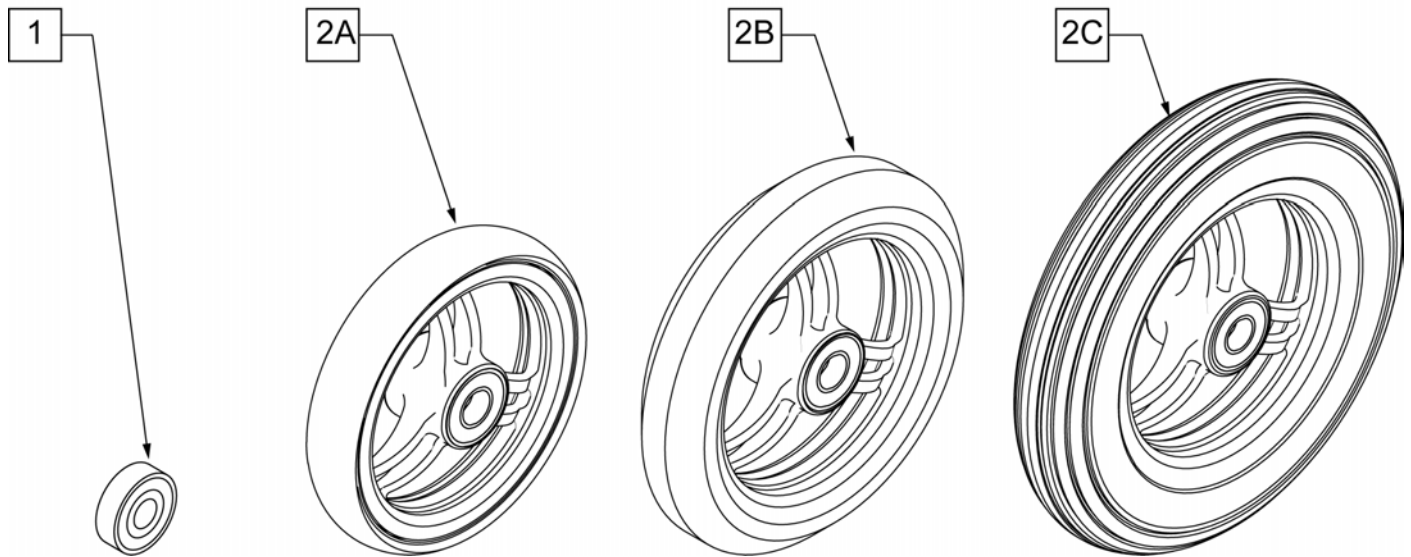

Pos.	Item Number	Description	HCPC	Remarks
1	349502	7/8"x 5/16" BEARING		
2A	101946-001	CASTR 4X1 ALUM/POLY W/BRG CLR	K0077	4" x 1" ALUMINUM POLY
2B	101946-002	CASTR 5X1 ALUM/POLY W/BRG CLR	K0077	5" x 1" ALUMINUM POLY
2C	101946-003	CASTR 6X1 ALUM/POLY W/BRG CLR	K0077	6" x 1" ALUMINUM POLY




SIDE VIEW

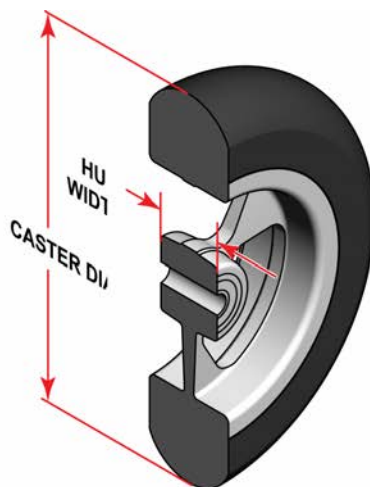

CASTER MEASURING GUIDE


TIRE DIAMETER	HUB WIDTH
3	1
4 1/8	1
5	1
6	1

Pos.	Item Number	Description	HCPC	Remarks
>>>>		3 INCH DIAMETER CASTERS WITH SILVER HUBS		
1	113394	WHEEL CASTER 3" ULTRALITE SFTRLL	K0077	3" x 1.5" Aluminum Soft Roll - Black tire - Silver hub
>>>>		4 INCH DIAMETER CASTERS WITH SILVER HUBS		
1	113395	WHEEL CASTER 4" ULTRALITE SFTRLL	K0077	4" x 1.5" Aluminum Soft Roll - Black tire - Silver hub
>>>>		5 INCH DIAMETER CASTERS WITH SILVER HUBS		
1	113396	WHEEL CASTER 5" ULTRALITE SFTRLL	K0077	5" x 1.5" Aluminum Soft Roll - Black tire - Silver hub
>>>>		6 INCH DIAMETER CASTERS WITH SILVER HUBS		
1	113397	WHEEL CASTER 6" ULTRALITE SFTRLL	K0077	6" x 1.5" Aluminum Soft Roll - Black tire - Silver hub

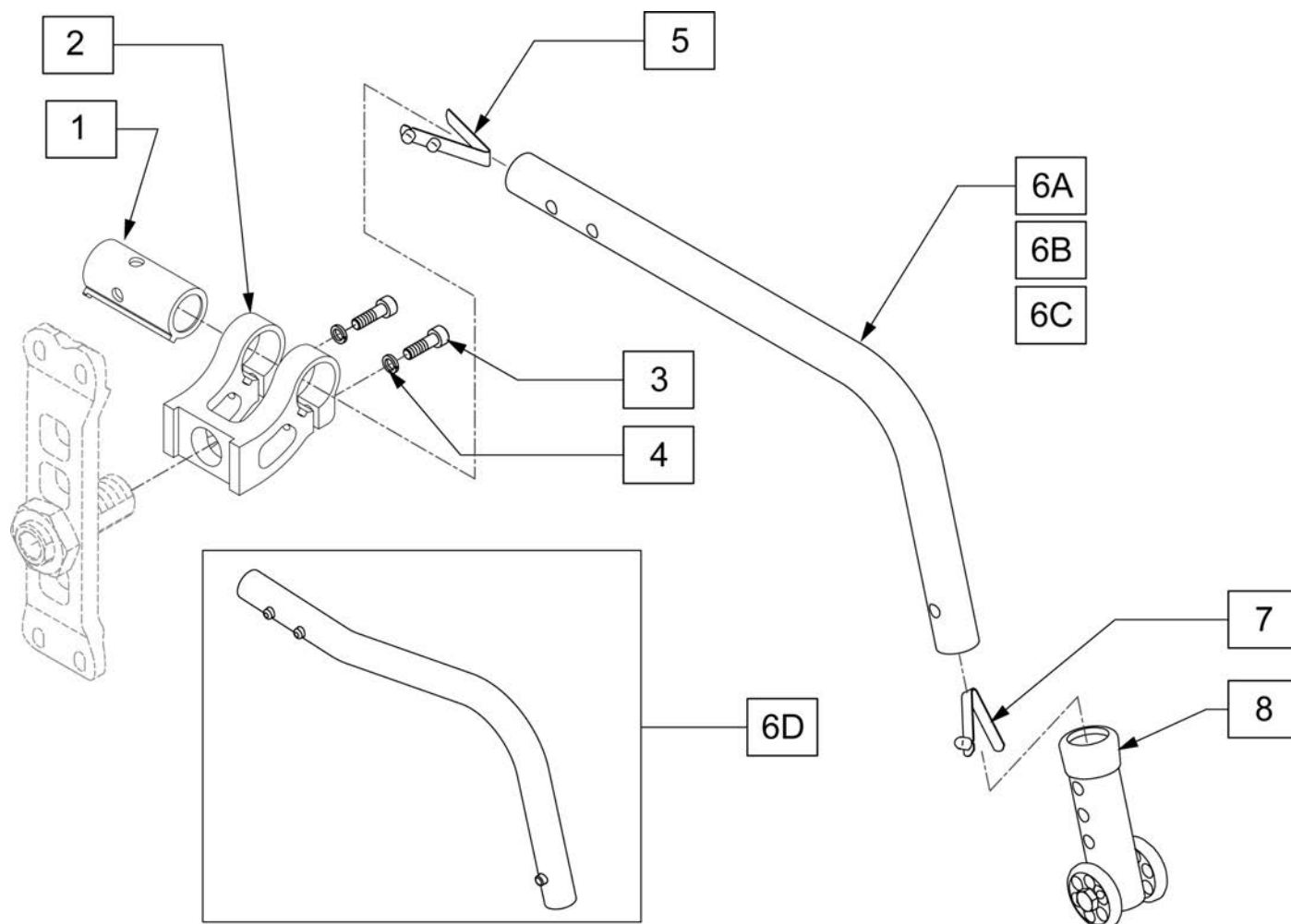


Pos.	Item Number	Description	Remarks
1	104349-001	SLEEVE RECEIVER ANTI-TIP STD	
2	104197	RECEIVER NON ADJ ANTI TIP	
3	106974	SHCS 1/4-28 X 3/4 PL ROHS	
4	107367	1/4 WASHER ROHS	
8	104640	ANTI TIP ARM ADJ. LONG	
1-4	107636	ANTI TIP RECEIVER ASSM	
5,6A,7,8	107628	ANTI TIP ARM SML L ASM	Left - 22" wheels
5,6A,7,8	107629	ANTI TIP ARM SML R ASM	Right - 22" wheels
5,6B,7,8	107630	ANTI TIP ARM MED L ASM	Left - 24" wheels
5,6B,7,8	107631	ANTI TIP ARM MED R ASM	Right - 24" wheels
5,6C,7,8	107632	ANTI TIP ARM LRG L ASM	Left - 25" & 26" wheels
5,6C,7,8	107633	ANTI TIP ARM LRG R ASM	Right - 25" & 26" wheels
5,6D,7,8	107634	ANTI TIP ARM BND L ASM	Left - 20" wheels
5,6D,7,8	107635	ANTI TIP ARM BND R ASM	Right - 20" wheels

NOTE: ATTACHMENT CLAMP mounts to 1" diameter tubing.

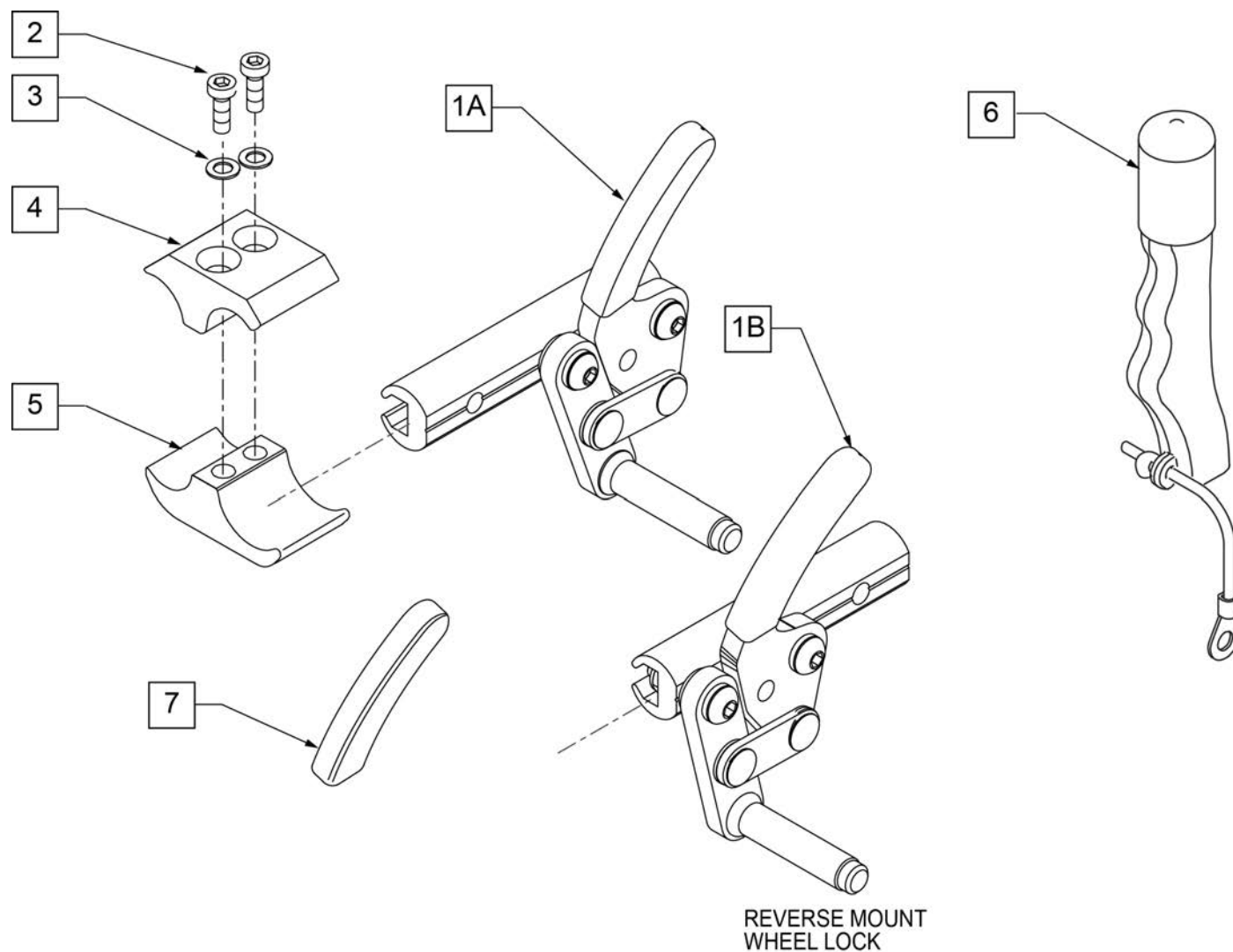


Pos.	Item Number	Description	Remarks
1A	104216-001	WHEEL LOCK ASSEMBLY PUSH RIGHT	RIGHT
1A	104216-002	WHEEL LOCK ASSEMBLY PUSH LEFT	LEFT
1B	109643	WHEEL LOCK HI PUSH REV R	RIGHT
1B	109644	WHEEL LOCK HI PUSH REV LEFT	LEFT
2	107032	SHCS 1/4-28 X 1 ROHS	
3	107367	1/4 WASHER HI COLLAR LOCK BLK	
6	118068	WHEEL LOCK EXTENSION ASSY	each
7	104141	COVER CURVED WHEEL LOCK VINYL	
1A,6	107637	WHEEL LOCK ASSM (PUSH) R W/ EXT	RIGHT
1A,6	107638	WHEEL LOCK ASSM (PUSH) L W/ EXT	LEFT
2-5	107639	WHL LCK CLMP ASM	
1B,6	112623	WHEEL LOCK ASSM (PUSH REV) R W/ EXT	RIGHT
1B,6	112624	WHEEL LOCK ASSM (PUSH REV) L W/ EXT	LEFT

NOTE: ATTACHMENT CLAMP mounts to 1" diameter tubing.

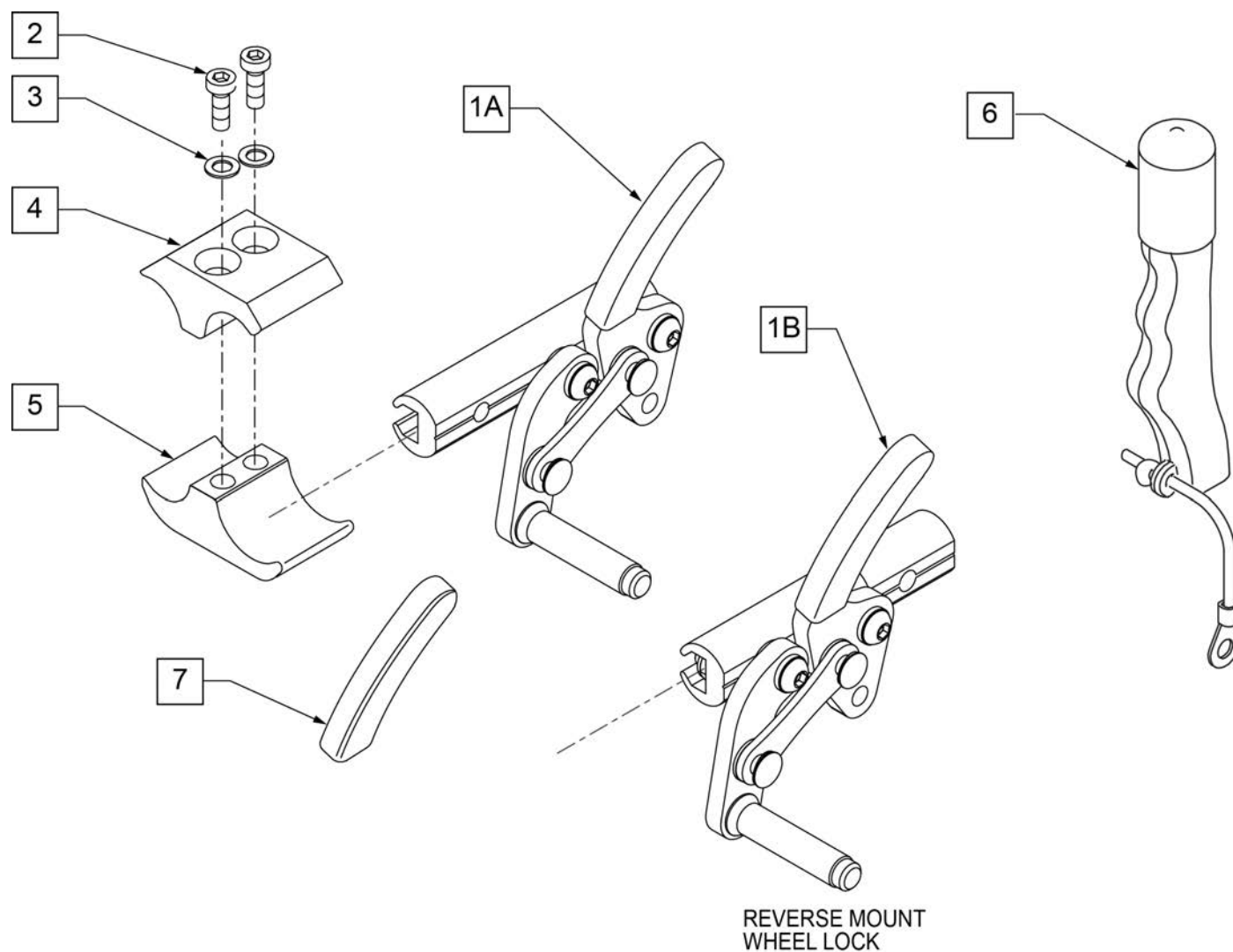


Pos.	Item Number	Description	Remarks
1A	104227-001	WHEEL LOCK ASSEMBLY (PULL) RIGHT	RIGHT
1A	104227-002	WHEEL LOCK ASSM (PULL) LEFT	LEFT
1B	109647	WHEEL LOCK HI PULL REV R	RIGHT
1B	109648	WHEEL LOCK HI PULL REV L	LEFT
2	107032	SHCS 1/4-28 X 1 ROHS	
3	107367	1/4 WASHER HI COLLAR LOCK BLK	
6	118068	WHEEL LOCK EXTENSION ASSY	each
7	104141	COVER CURVED WHEEL LOCK VINYL	
2-5	107639	WHL LCK CLMP ASM	
1A,6	107640	WHEEL LOCK ASSM (PULL) R W/ EXT	RIGHT
1A,6	107641	WHEEL LOCK ASSM (PULL) L W/ EXT	LEFT
1B,6	112625	WHEEL LOCK HI PULL REV R W/ EXT	RIGHT
1B,6	112626	WHEEL LOCK HI PULL REV L W/ EXT	LEFT

NOTE: ATTACHMENT CLAMP mounts to 1" diameter tubing.

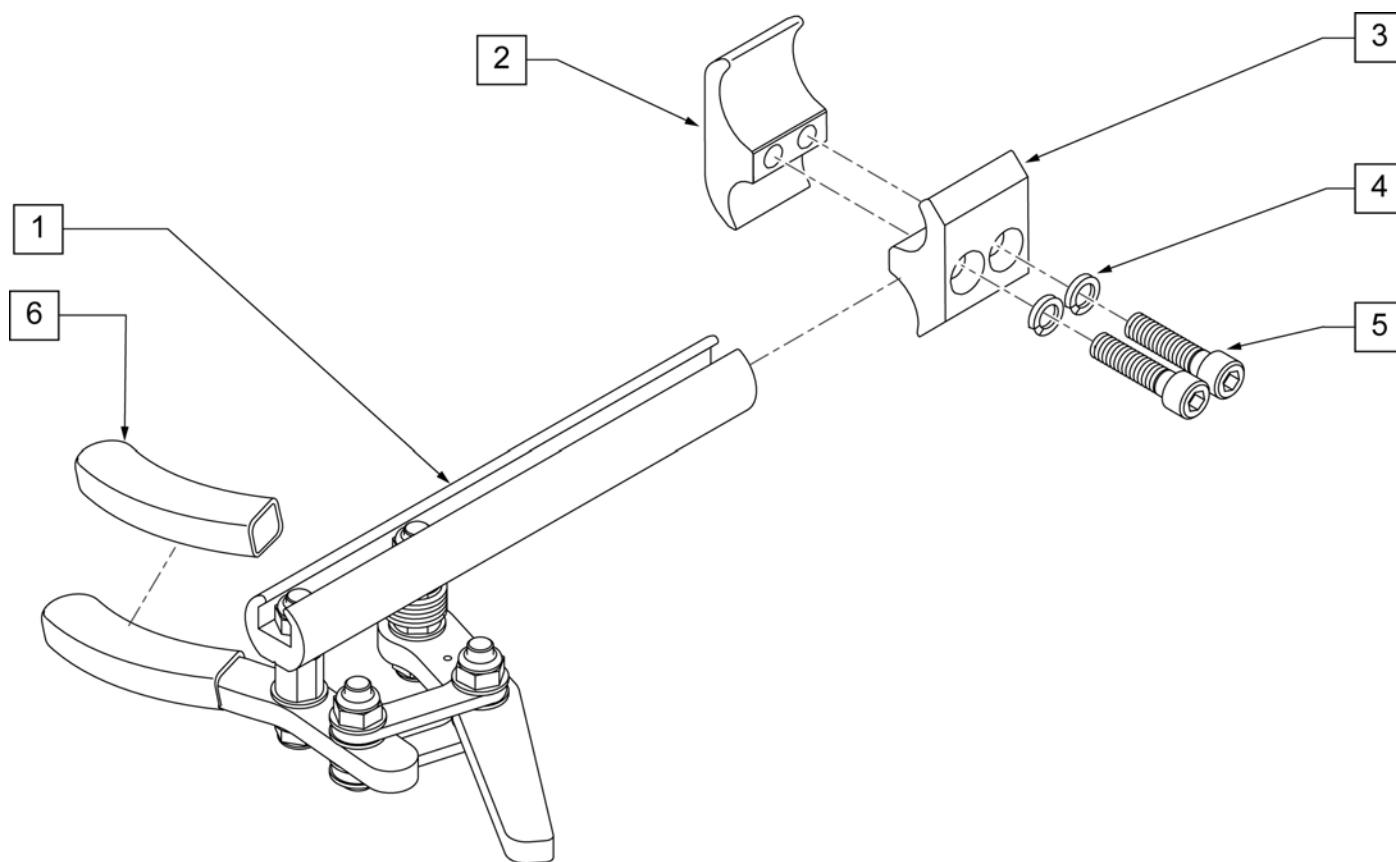

Pos.	Item Number	Description	Remarks
1	104726-001	ASSY WHL LOCK SCISSOR ERGO	RIGHT
1	104726-002	ASSY WHL LOCK SCISSOR ERGO	LEFT
4	107367	1/4 WASHER HI COLLAR LOCK BLK	
5	107032	SHCS 1/4-28 X 1 ROHS	
6	104141	COVER CURVED WHEELLOCK VIN	
2-6	107639	WHL LCK CLMP ASM	


NOTE: ATTACHMENT CLAMP mounts to 1" diameter tubing.

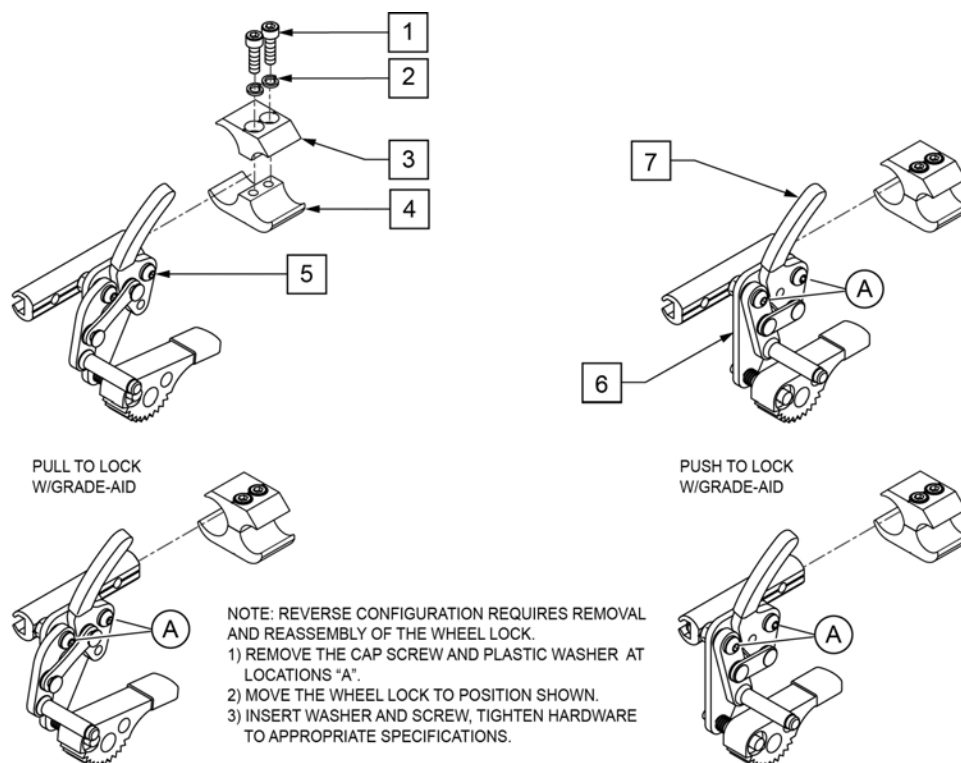

Pos.	Item Number	Description	Remarks
1	107032	SHCS 1/4-28 X 1 ROHS	
2	107367	1/4 WASHER HI COLLAR LOCK BLK	
1-4	107639	WHL LCK CLMP ASM	
5	107667	WHEEL LOCK PULL GRADE AID L	LEFT (Pull)
5	107668	WHEEL LOCK PULL GRADE AID R	RIGHT (Pull)
6	107665	WHEEL LOCK PUSH GRADE AID L	LEFT (Push)
6	107666	WHEEL LOCK PUSH GRADE AID R	RIGHT (Push)
7	104141	COVER CURVED WHEEL LOCK VIN	

NOTE: Refer to "WHEEL LOCK AVAILABILITY" Page for possible mount locations.


NOTE: ATTACHMENT CLAMP mounts to 1" diameter tubing.

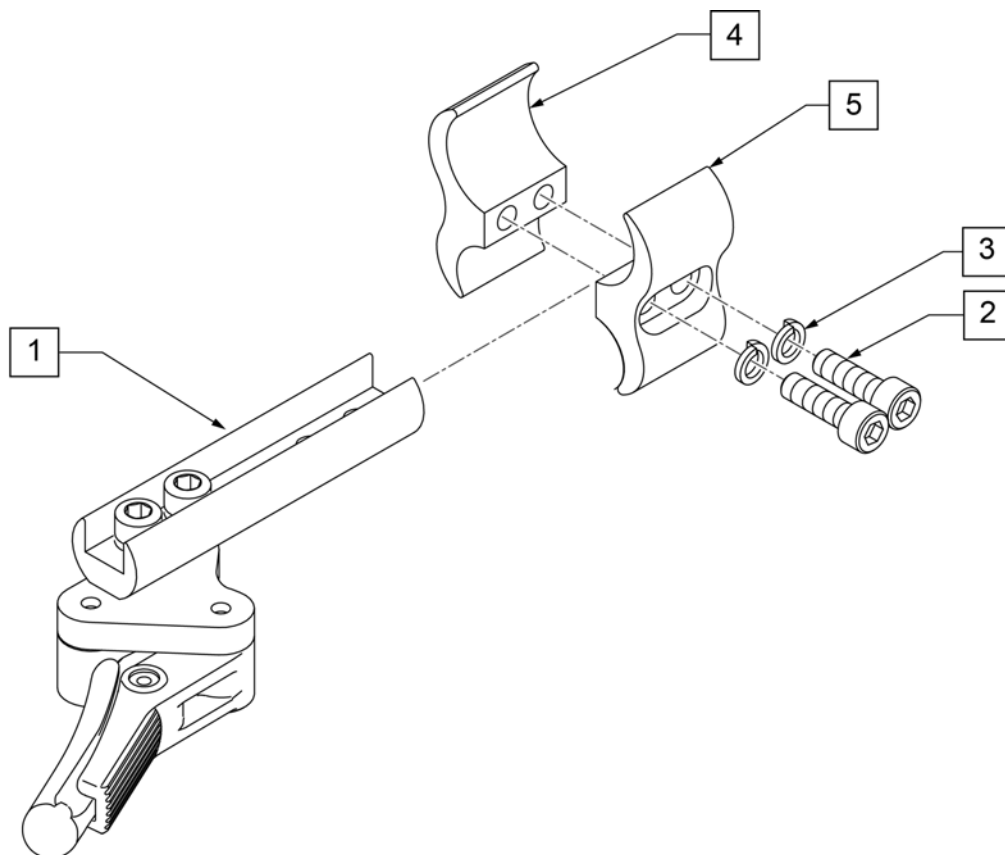

Pos.	Item Number	Description
1	066608-001	CMPCT WHLLCK STD ASSY LEFT
1	066608-002	CMPCT WHLLCK STD ASSY RIGHT
2	107032	SHCS 1/4-28 X 1 ROHS
3	107367	1/4 WASHER HI COLLAR LOCK BLK
2-5	107639	WHL LCK CLMP ASM

WHEEL LOCK EXTENSION HANDLE

[089/2011]

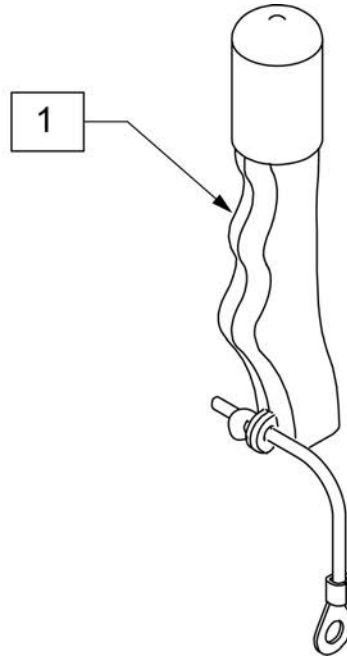

Pos.	Item Number	Description	Remarks
1	119563	WHEEL LOCK EXT ASSY (PAIR)	Pair



Pos.	Item Number	Description	Remarks
1-3	401094	20" MAG W/BRNG 6 SPOKE	20"
1-3	401093	22" MAG W/BRNG 6 SPOKE	22"(501mm)
1-3	401092	24" MAG W/BRNG 6 SPOKE	24"(540mm)
2	349500	1 1/8 X 1/2 BEARING	
3	521564	MAG BEARING SPACER	
4	101932	SPCR .194ID X .375 OD X .750	Used when previously ordered with Omit handrims or Natural Fit Handrims

(10/2010)

NOTE: THIS WHEEL UTILIZES SCREW MOUNT HANDRIMS - THE HANDRIM HAS A THREADED HOLE THAT MOUNTS THE HANDRIM TO THE WHEEL THROUGH A SPACER AND FASTNER.

NOTE: IF MAG WHEEL ASSEMBLY WAS ORDER WITH OUT HANDRIM AND A HANDRIM IS ADDED TO THE WHEEL QTY 6 OF 561037 AND 281175 MUST BE ORDERED TO PROPERLY MOUNT THE HANDRIM.

NOTE: THIS WHEEL UTILIZES SCREW MOUNT HANDRIMS - THE HANDRIM HAS A THREADED HOLE THAT MOUNTS THE HANDRIM TO THE WHEEL THROUGH A SPACER AND FASTNER.



Pos.	Item Number	Description	Remarks
1A	116178	WHEEL ASSY MBL 20"	20" (451)
1B	116179	WHEEL ASSY MBL 22"	WHL RIM/SPOKE MBL 22 (501)
1C	116180	WHEEL ASSY MBL 24"	WHL RIM/SPOKE MBL 24 (540)
1D	116181	WHEEL ASSY MBL 25"	WHL RIM/SPOKE MBL 25 (559)
1E	116182	WHEEL ASSY MBL 26"	WHL RIM/SPOKE MBL 26 (590)
2	114097	BEARING, REPLACEMENT, MBL	12.7mm i.d. - 28mm o.d. - 8mm wide
3A	118100	MBL LT 20IN SPOKE RPLMNT KIT	Kit contains 6 inner and 6 outer spokes
3B	118101	MBL LT 22IN SPOKE RPLMNT KIT	Kit contains 6 inner and 6 outer spokes
3C	118102	MBL LT 24IN SPOKE RPLMNT KIT	Kit contains 6 inner and 6 outer spokes
3D	118103	MBL LT 25IN SPOKE RPLMNT KIT	Kit contains 6 inner and 6 outer spokes
3E	118104	MBL LT 26IN SPOKE RPLMNT KIT	Kit contains 6 inner and 6 outer spokes

NOTE: THIS WHEEL UTILIZES TAB MOUNT HANDRIMS - THE HANDRIM HAS A STRAIGHT TAB THAT MOUNTS TO THE WHEEL THROUGH THE RIM.




Pos.	Item Number	Description	Remarks
1A	112857	ULTRA LIGHTWEIGHT WHEEL 20"	20"
1B	112858	ULTRA LIGHT WHEEL 22"	22"(501mm)
1C	112859	ULTRA LIGHTWEIGHT WHEEL 24"	24"(540mm)
1D	112860	ULTRA LIGHTWEIGHT WHEEL 25"	25"(559mm)
2	349500	1 1/8 X 1/2 BEARING	
3A	118105	ULTRA LT 20IN SPOKE RPLMNT KIT	Kit contains 12 spokes
3B	118106	ULTRA LT 22IN SPOKE RPLMNT KIT	Kit contains 12 spokes
3C	118107	ULTRA LT 24IN SPOKE RPLMNT KIT	Kit contains 12 spokes
3D	118108	ULTRA LT 25IN SPOKE RPLMNT KIT	Kit contains 12 spokes

NOTE: THIS WHEEL UTILIZES TAB MOUNT HANDRIMS - THE HANDRIM HAS A STRAIGHT TAB THAT MOUNTS TO THE WHEEL THROUGH THE RIM.




Pos.	Item Number	Description	Remarks
1	762438	SPINERGY STD WHEEL 540 BLK SPK	24"(540mm)
1	762441	SPINERGY STD WHL 559 BLK SPK	25"(559mm)
1	762437	SPINERGY STD WHEEL 590 BLK SPK	26"(590mm)
1	762442	SPINERGY STD WHEEL 540 BLUE	24"(540mm)
1	762444	SPINERGY STD WHEEL 559 BLU	25"(559mm)
1	762446	SPINERGY STD WHEEL 590 BLUE	26"(590mm)
1	762443	SPINERGY STD WHEEL 540 RED	24"(540mm)
1	762445	SPINERGY STD WHEEL 559 RED	25"(559mm)
1	762447	SPINERGY STD WHEEL 590 RED	26"(590mm)
1	762456	SPINERGY STD WHEEL 540 WHT	24"(540mm)
1	762463	SPINERGY STD WHL 559 WHT SPK	25"(559mm)
1	762467	SPINERGY STD WHEEL 590 WHT	26"(590mm)
1	762433	SPINERGY STD WHEEL 540 YELLOW	24"(540mm)
1	762462	SPINERGY STD WHL 559 YLW SPK	25"(559mm)
1	762431	SPINERGY STD WHEEL 590 YELLOW	26"(590mm)
2	349500	1 1/8 X 1/2 BEARING	

NOTE: THIS WHEEL UTILIZES TAB MOUNT HANDRIMS - THE HANDRIM HAS A STRAIGHT TAB THAT MOUNTS TO THE WHEEL THROUGH THE RIM.

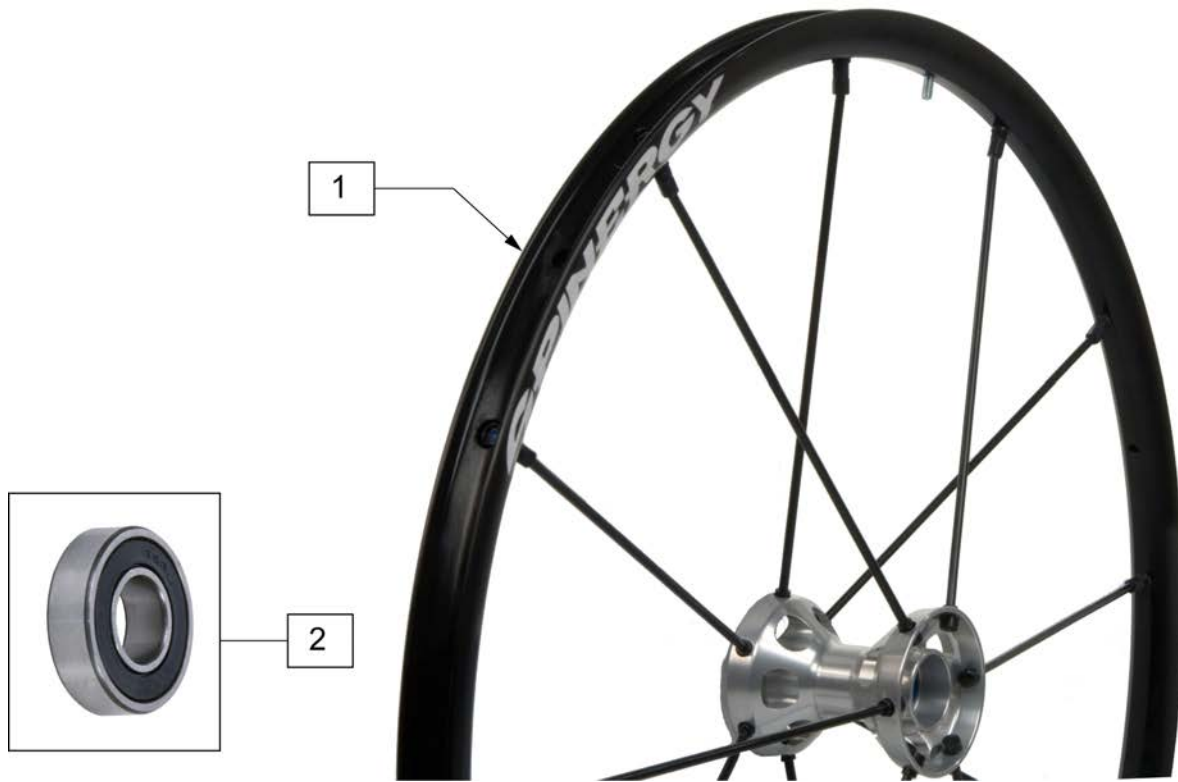

Pos.	Item Number	Description	Remarks	Pos.	Item Number	Description	Remarks
1	101988-001	SPINRGY SPT LT EX WHL 540 BLK	24"(540mm)	1	101992-003	SPINRGY SPT LT EX WHL 590 RED	26"(590mm)
1	101989-001	SPINRGY LT EX WHEEL 540 BLK	24"(540mm)	1	101993-003	SPINRGY LT EX WHEEL 590 RED	26"(590mm)
1	101990-001	SPINRGY SPT LT EX WHL 559 BLK	25"(559mm)	1	101988-005	SPINRGY SPT LT EX WHL 540 WHT	24"(540mm)
1	101991-001	SPINRGY LT EX WHEEL 559 BLK	25"(559mm)	1	101989-005	SPINRGY LT EX WHEEL 540 WHT	24"(540mm)
1	101992-001	SPINRGY SPT LT EX WHL 590 BLK	26"(590mm)	1	101990-005	SPINRGY SPT LT EX WHL 559 WHT	25"(559mm)
1	101993-001	SPINRGY LT EX WHEEL 590 BLK	26"(590mm)	1	101991-005	SPINRGY LT EX WHEEL 559 WHT	25"(559mm)
1	101988-002	SPINRGY SPT LT EX WHL 540 BLU	24"(540mm)	1	101992-005	SPINRGY SPT LT EX WHL 590 WHT	26"(590mm)
1	101989-002	SPINRGY LT EX WHEEL 540 BLU	24"(540mm)	1	101993-005	SPINRGY LT EX WHEEL 590 WHT	26"(590mm)
1	101990-002	SPINRGY SPT LT EX WHL 559 BLU	25"(559mm)	1	101988-004	SPINRGY SPT LT EX WHL 540 YLW	24"(540mm)
1	101991-002	SPINRGY LT EX WHEEL 559 BLU	25"(559mm)	1	101989-004	SPINRGY LT EX WHEEL 540 YLW	24"(540mm)
1	101992-002	SPINRGY SPT LT EX WHL 590 BLU	26"(590mm)	1	101990-004	SPINRGY SPT LT EX WHL 559 YLW	25"(559mm)
1	101993-002	SPINRGY LT EX WHEEL 590 BLU	26"(590mm)	1	101991-004	SPINRGY LT EX WHEEL 559 YLW	25"(559mm)
1	101988-003	SPINRGY SPT LT EX WHL 540 RED	24"(540mm)	1	101992-004	SPINRGY SPT LT EX WHL 590 YLW	26"(590mm)
1	101989-003	SPINRGY LT EX WHEEL 540 RED	24"(540mm)	1	101993-004	SPINRGY LT EX WHEEL 590 YLW	26"(590mm)
1	101990-003	SPINRGY SPT LT EX WHL 559 RED	25"(559mm)	2	349500	1 1/8 X 1/2 BEARING	
1	101991-003	SPINRGY LT EX WHEEL 559 RED	25"(559mm)				

NOTE: THIS WHEEL UTILIZES SCREW MOUNT HANDRIMS - THE HANDRIM HAS A THREADED HOLE THAT MOUNTS THE HANDRIM TO THE WHEEL THROUGH A SPACER AND FASTNER.

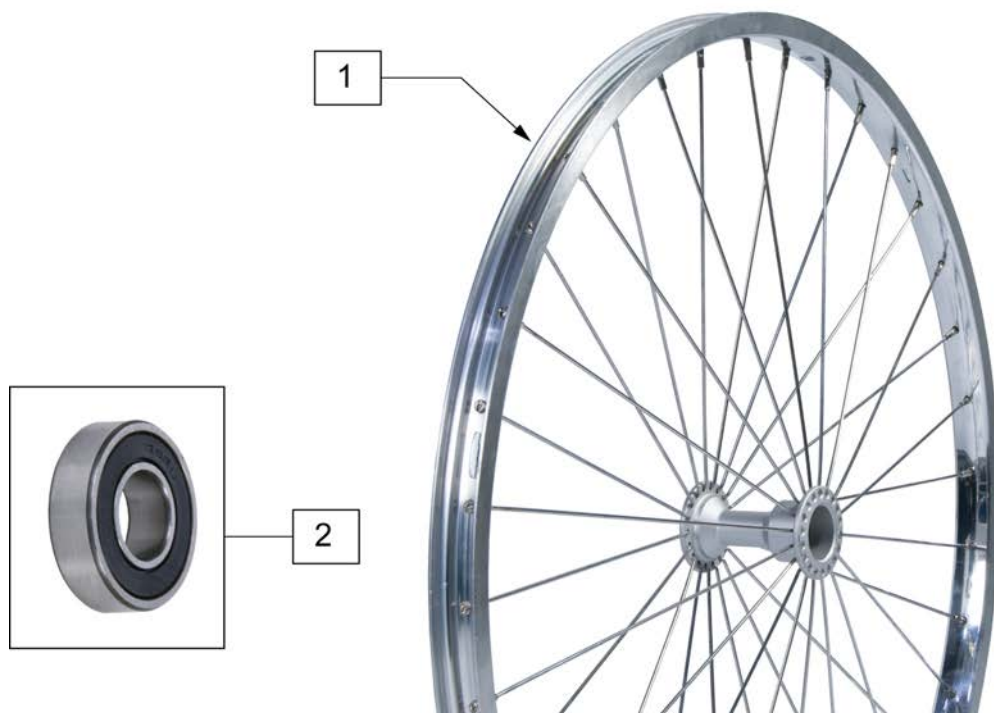

Pos.	Item Number	Description	Remarks
1	401075	24 RIM ASSM X 1.5 SUN PSH BTTN	22" (507MM)
1	401076	24" HT RIM ASSM SUN PUSH BTTN	25" (559MM)
2	349500	1 1/8 X 1/2 BEARING	



Pos.	Item Number	Description	Remarks
1	384706	TIRE PNEU 12-1/2" X 2-1/4"	12-1/2" X 2-1/4"
1	385237	16" X 1.75" PNEU TIRE	
1	385225	TIRE PNEUMATIC 18"	18" x 1 3/8
1	385250	20" PNEUMATIC TIRE	20" x 1 3/8
1	385275	22" PNEUMATIC TIRE	22" x 1 3/8 (501mm)
1	385300	24" PNEUMATIC TIRE	24" x 1 3/8 (540mm)
1	385325	26" PNEUMATIC TIRE	26" x 1 3/8 (590mm)
1	384614	24" x 2.125 PNEUMATIC TIRE	24"x2 1/8 (540mm) used w/ M6



Pos.	Item Number	Description	Remarks
1	386232	20" POLY TIRE STD	20"
1	386242	22" POLY TIRE STD	22" (501mm)
1	386252	24" POLY TIRE STD	24" (540mm)




Pos.	Item Number	Description	Remarks
1	385280	22" x 1" HI PRES CLINCHER	22" (501mm)
1	389020	24" x 1" HI PRES CLNCHR TIRE	24" (540mm)
1	389012	QP 24" x 1" HT TURB BLK WR/BD 559	25" (559mm)
1	389030	26" x 1" HI PRES CLNCHR TIRE	26" (590mm)
1	105919	22" x 1" HI PRESS CLNCHR ULTRALIGHT	22" (489mm)



Pos.	Item Number	Description	Remarks
1	385354	24" X 1-3/8" KNOBBIE TIRE	24"(540mm)



Pos.	Item Number	Description	Remarks
1	102066-004	TIRE SCHWALBE MP 489 BLK	22" (489mm)
1	102066-001	TIRE SCHWBL MPE 540 BLK	24" (540mm)
1	102066-002	TIRE SCHWBL MPE 559 BLK	25" (559mm)
1	102066-003	TIRE SCHWBL MPE 590 BLK	26" (590mm)

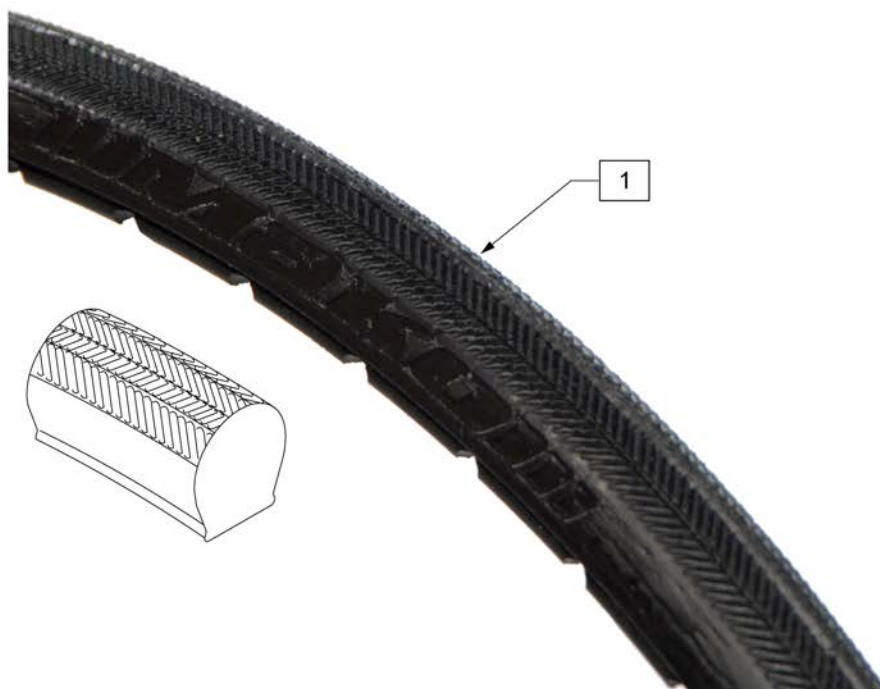


Pos.	Item Number	Description	Remarks
1	102060-001	TIRE KIK MAKO MAG SUN 540 BLK	24"(540mm) Black
1	102060-002	TIRE KIK MAKO MAG SUN 540 YLW	24"(540mm) Yellow
1	102060-003	TIRE KIK MAKO MAG SUN 540 RED	24"(540mm) Red
1	102060-004	TIRE KIK MAKO MAG SUN 540 BLU	24"(540mm) Blue
1	102060-005	TIRE KIK MAKO MAG SUN 540 DKG	24"(540mm) Dark Grey
1	102061-001	TIRE KIK MAKO SPIN 559 BLK	25"(559mm) Black
1	102061-002	TIRE KIK MAKO SPIN 559 YLW	25"(559mm) Yellow
1	102061-003	TIRE KIK MAKO SPIN 559 RED	25"(559mm) Red
1	102061-004	TIRE KIK MAKO SPIN 559 BLU	25"(559mm) Blue
1	102061-005	TIRE KIK MAKO SPIN 559 DKGRPH	25"(559mm) Dark Grey



Pos.	Item Number	Description	Remarks
1	109213	TIRE 24" URBAN ASSAULT	22"(507mm)
1	109214	TIRE 26" URBAN ASSAULT	26"(590mm)



Pos.	Item Number	Description	Remarks
1	384604	TIRE CS 24 X 2.1 SMOKER 507MM	22"(501mm)
1	384606	TIRE CS 26 X 2.1 SMOKER 559MM	25"(559mm)

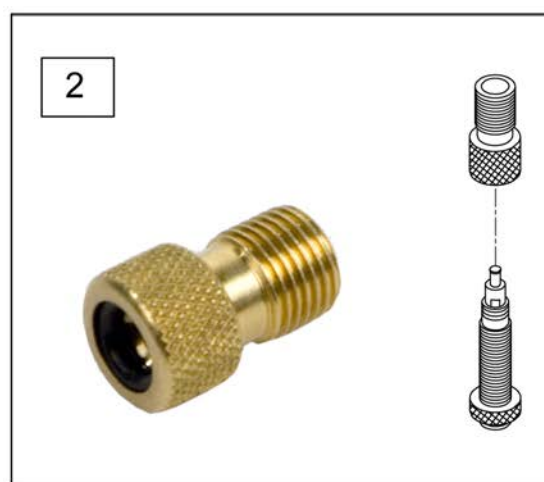


Pos.	Item Number	Description	Remarks
1A	382305	22" X 1" TUBE PRESTA VALVE	
1A	382330	24" TUBE PRESTA VALVE (540MM)	
1A	384545	QP 24" HT INNER TUBE (559MM)	
1A	382355	26" TUBE PRESTA VALVE (590MM)	
1A	384550	QP 26" HT INNER TUBE (700C)	
1B	384736	INNER TUBE 12-1/2" X 2-1/4"	
1B	384738	16" X 1.75" INNER TUBE	
1B	382255	TUBE 18" TIRE	18" X 1-3/4" TUBE
1B	382275	20" TUBE	20" 1-3/8" TUBE
1B	382300	22" TUBE	22" X 1-3/8" TUBE
1B	382336	TUBE PRIMO 24" 25-540 SV	24" X 1" TUBE
1B	382325	24 X 1.3/8IN TUBE REG	
1B	382334	TUBE INLINE 24 X 1.75	
1B	384551	26" X 1" TUBE SCHRADER	
1B	382350	26" TUBE	26" X 1-3/8" TUBE
1B	382361	TUBE 26 X 1.5/1.95/ S/V	
1B	382365	26 X 1.25/1.5 TUBE SHRADER	
1B	382375	24" x 2.125 TUBE	
1C	102972-001	12 1/2 IN AIRLESS INSERT	12-1/2" X 2-1/4"
1C	102973-001	16IN AIRLESS INSERT	16" X 1-3/4"
1C	102973-002	18IN AIRLESS INSERT	18" X 1-3/4"

Pos.	Item Number	Description	Remarks
1C	102973-003	20IN AIRLESS INSERT	20" X 1-3/8"
1C	102972-002	AIRLESS INSERT 20 X 2-1/8 P3	
1C	102973-004	22IN AIRLESS INSERT	22" X 1-3/8"
1C	102973-005	24IN AIRLESS INSERT	24" X 1-3/8"
1C	102972-003	AIRLESS INSERT 24 X 2-1/8	
1C	108259	26" AIRLESS INSERT	26" X 1-3/8"
2	381230	VALVE ADAPTER PRESTA	


Note: When replacing the Seat Sling it is recommended to order the Seat Rail End Plug Kit.

Note: On Quickie 2 chairs with S/N prefix (Q2V) & Quickie 2 Lite chairs the diameter of the Seat Rail Rod is 8mm (.31") which is larger than the Q2 chairs with S/N prefix (Q2 & Q2H) which is .25" in diameter.

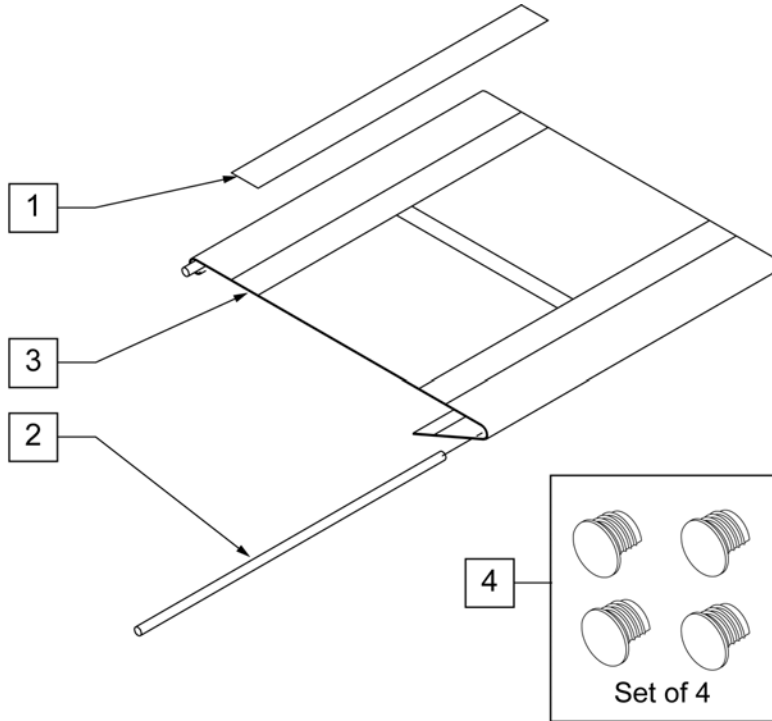


MEASURING GUIDE

CHAIR WIDTH	DIM A	DIM B
12"	16.5"	13.5"
13"	17.5"	14.5"
14"	18.5"	15.5"
15"	19.5"	16.5"
16"	20.5"	17.5"
17"	21.5"	18.5"
18"	22.5"	19.5"
19"	23.5"	20.5"
20"	24.5"	21.5"

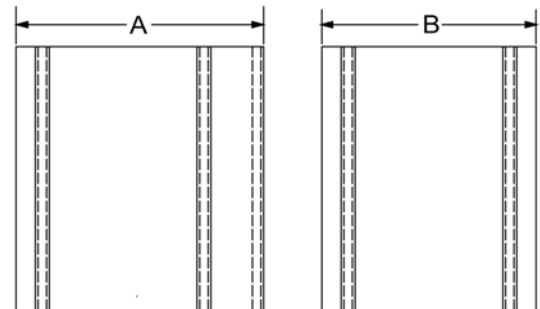


Pos.	Item Number	Description	Remarks	Pos.	Item Number	Description	Remarks
1	901169	SEAT SLING VELCRO STRIP 2"		3	722603	SEAT SLING 13" X 13" CHNL	
2	106584	SEAT RAIL ROD 12"		3	722604	SEAT SLING 13" X 14" CHNL	
2	106585	SEAT RAIL ROD 13"		3	722605	SEAT SLING 13" X 15" CHNL	
2	106586	SEAT RAIL ROD 14"		3	722606	SEAT SLING 13" X 16" CHNL	
2	106587	SEAT RAIL ROD 15"		3	722607	SEAT SLING 13" X 17" CHNL	
2	106588	SEAT RAIL ROD 16"		3	722608	SEAT SLING 13" X 18" CHNL	
2	106589	SEAT RAIL ROD 17"				14" Seat Slings	
2	106590	SEAT RAIL ROD 18"		3	722616	SEAT SLING 14" X 10" CHNL	
2	106591	SEAT RAIL ROD 19"		3	722617	SEAT SLING 14" X 11" CHNL	
2	106592	SEAT RAIL ROD 20"		3	722618	SEAT SLING 14" X 12" CHNL	
		12" Wide Seat Slings		3	722619	SEAT SLING 14" X 13" CHNL	
3	722590	SEAT SLING 12" X 10" CHNL		3	722620	SEAT SLING 14" X 14" CHNL	
3	722591	SEAT SLING 12" X 11" CHNL		3	722621	SEAT SLING 14" X 15" CHNL	
3	722592	SEAT SLING 12" X 12" CHNL		3	722622	SEAT SLING 14" X 16" CHNL	
3	722593	SEAT SLING 12" X 13" CHNL		3	722623	SEAT SLING 14" X 17" CHNL	
3	722594	SEAT SLING 12" X 14" CHNL		3	722624	SEAT SLING 14" X 18" CHNL	
3	722595	SEAT SLING 12" X 15" CHNL		3	722625	SEAT SLING 14" X 19" CHNL	
3	722596	SEAT SLING 12" X 16" CHNL		3	722626	SEAT SLING 14" X 20" CHNL	
3	722597	SEAT SLING 12" X 17" CHNL				15" Seat Slings	
		13" Wide Seat Slings		3	722630	SEAT SLING 15" X 14" CHNL	
3	722600	SEAT SLING 13" X 10" CHNL		3	722631	SEAT SLING 15" X 15" CHNL	
3	722601	SEAT SLING 13" X 11" CHNL		3	722632	SEAT SLING 15" X 16" CHNL	
3	722602	SEAT SLING 13" X 12" CHNL		3	722633	SEAT SLING 15" X 17" CHNL	

Pos.	Item Number	Description	Remarks	Pos.	Item Number	Description	Remarks
3	722634	SEAT SLING 15" X 18" CHNL		3	722662	SEAT SLING 18" X 16" CHNL	
3	722635	SEAT SLING 15" X 19" CHNL		3	722663	SEAT SLING 18" X 17" CHNL	
3	722636	SEAT SLING 15" X 20" CHNL		3	722664	SEAT SLING 18" X 18" CHNL	
		16" Seat Slings		3	722665	SEAT SLING 18" X 19" CHNL	
3	722640	SEAT SLING 16" X 14" CHNL		3	722666	SEAT SLING 18" X 20" CHNL	
3	722641	SEAT SLING 16" X 15" CHNL				19" Seat Slings	
3	722642	SEAT SLING 16" X 16" CHNL		3	722670	SEAT SLING 19" X 14" CHNL	
3	722643	SEAT SLING 16" X 17" CHNL		3	722671	SEAT SLING 19" X 15" CHNL	
3	722644	SEAT SLING 16" X 18" CHNL		3	722672	SEAT SLING 19" X 16" CHNL	
3	722645	SEAT SLING 16" X 19" CHNL		3	722673	SEAT SLING 19" X 17" CHNL	
3	722646	SEAT SLING 16" X 20" CHNL		3	722674	SEAT SLING 19" X 18" CHNL	
		17" Seat Slings		3	722675	SEAT SLING 19" X 19" CHNL	
3	722650	SEAT SLING 17" X 14" CHNL		3	722676	SEAT SLING 19" X 20" CHNL	
3	722651	SEAT SLING 17" X 15" CHNL				20" Seat Slings	
3	722652	SEAT SLING 17" X 16" CHNL		3	722680	SEAT SLING 20" X 14" CHNL	
3	722653	SEAT SLING 17" X 17" CHNL		3	722681	SEAT SLING 20" X 15" CHNL	
3	722654	SEAT SLING 17" X 18" CHNL		3	722682	SEAT SLING 20" X 16" CHNL	
3	722655	SEAT SLING 17" X 19" CHNL		3	722683	SEAT SLING 20" X 17" CHNL	
3	722656	SEAT SLING 17" X 20" CHNL		3	722684	SEAT SLING 20" X 18" CHNL	
		18" Seat Slings		3	722685	SEAT SLING 20" X 19" CHNL	
3	722660	SEAT SLING 18" X 14" CHNL		3	722686	SEAT SLING 20" X 20" CHNL	
3	722661	SEAT SLING 18" X 15" CHNL		4	108117	SEAT RAIL END PLUG KIT	Set of 4

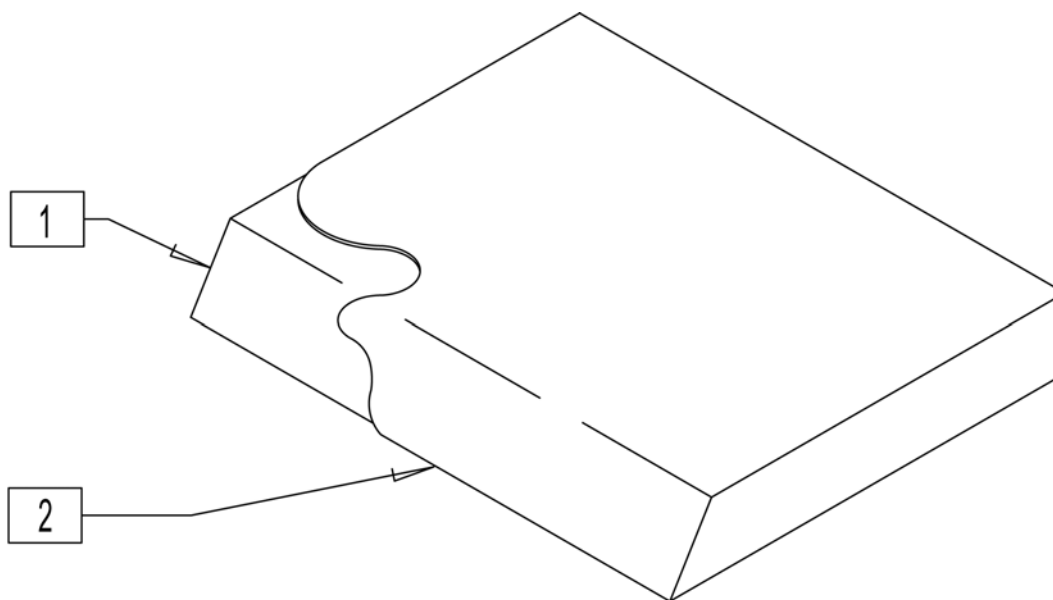


Pos.	Item Number	Description	Pos.	Item Number	Description
1-2	700322	SEAT CUSHION 14" X 14" X 2"	1-2	700642	SEAT CUSHION 17" X 18" X 2"
1-2	700332	SEAT CUSHION 14" X 15" X 2"	1-2	700652	SEAT CUSHION 17" X 19" X 2"
1-2	700342	SEAT CUSHION 14" X 16" X 2"	1-2	700662	SEAT CUSHION 17" X 20" X 2"
1-2	700352	SEAT CUSHION 14" X 17" X 2"	1-2	700702	SEAT CUSHION 18" X 14" X 2"
1-2	700362	SEAT CUSHION 14" X 18" X 2"	1-2	700712	SEAT CUSHION 18" X 15" X 2"
1-2	700372	SEAT CUSHION 14" X 19" X 2"	1-2	700722	SEAT CUSHION 18" X 16" X 2"
1-2	700382	SEAT CUSHION 14" X 20" X 2"	1-2	700732	SEAT CUSHION 18" X 17" X 2"
1-2	700422	SEAT CUSHION 15" X 14" X 2"	1-2	700742	SEAT CUSHION 18" X 18" X 2"
1-2	700432	SEAT CUSHION 15" X 15" X 2"	1-2	700752	SEAT CUSHION 18" X 19" X 2"
1-2	700442	SEAT CUSHION 15" X 16" X 2"	1-2	700762	SEAT CUSHION 18" X 20" X 2"
1-2	700452	SEAT CUSHION 15" X 17" X 2"	1-2	700802	SEAT CUSHION 19" X 14" X 2"
1-2	700462	SEAT CUSHION 15" X 18" X 2"	1-2	700812	SEAT CUSHION 19" X 15" X 2"
1-2	700472	SEAT CUSHION 15" X 19" X 2"	1-2	700822	SEAT CUSHION 19" X 16" X 2"
1-2	700482	SEAT CUSHION 15" X 20" X 2"	1-2	700832	SEAT CUSHION 19" X 17" X 2"
1-2	700522	SEAT CUSHION 16" X 14" X 2"	1-2	700842	SEAT CUSHION 19" X 18" X 2"
1-2	700532	SEAT CUSHION 16" X 15" X 2"	1-2	700852	SEAT CUSHION 19" X 19" X 2"
1-2	700542	SEAT CUSHION 16" X 16" X 2"	1-2	700862	SEAT CUSHION 19" X 20" X 2"
1-2	700552	SEAT CUSHION 16" X 17" X 2"	1-2	700902	SEAT CUSHION 20" X 14" X 2"
1-2	700562	SEAT CUSHION 16" X 18" X 2"	1-2	700912	SEAT CUSHION 20" X 15" X 2"
1-2	700572	SEAT CUSHION 16" X 19" X 2"	1-2	700922	SEAT CUSHION 20" X 16" X 2"
1-2	700582	SEAT CUSHION 16" X 20" X 2"	1-2	700932	SEAT CUSHION 20" X 17" X 2"
1-2	700602	SEAT CUSHION 17" X 14" X 2"	1-2	700942	SEAT CUSHION 20" X 18" X 2"
1-2	700612	SEAT CUSHION 17" X 15" X 2"	1-2	700952	SEAT CUSHION 20" X 19" X 2"
1-2	700622	SEAT CUSHION 17" X 16" X 2"	1-2	700962	SEAT CUSHION 20" X 20" X 2"
1-2	700632	SEAT CUSHION 17" X 17" X 2"			

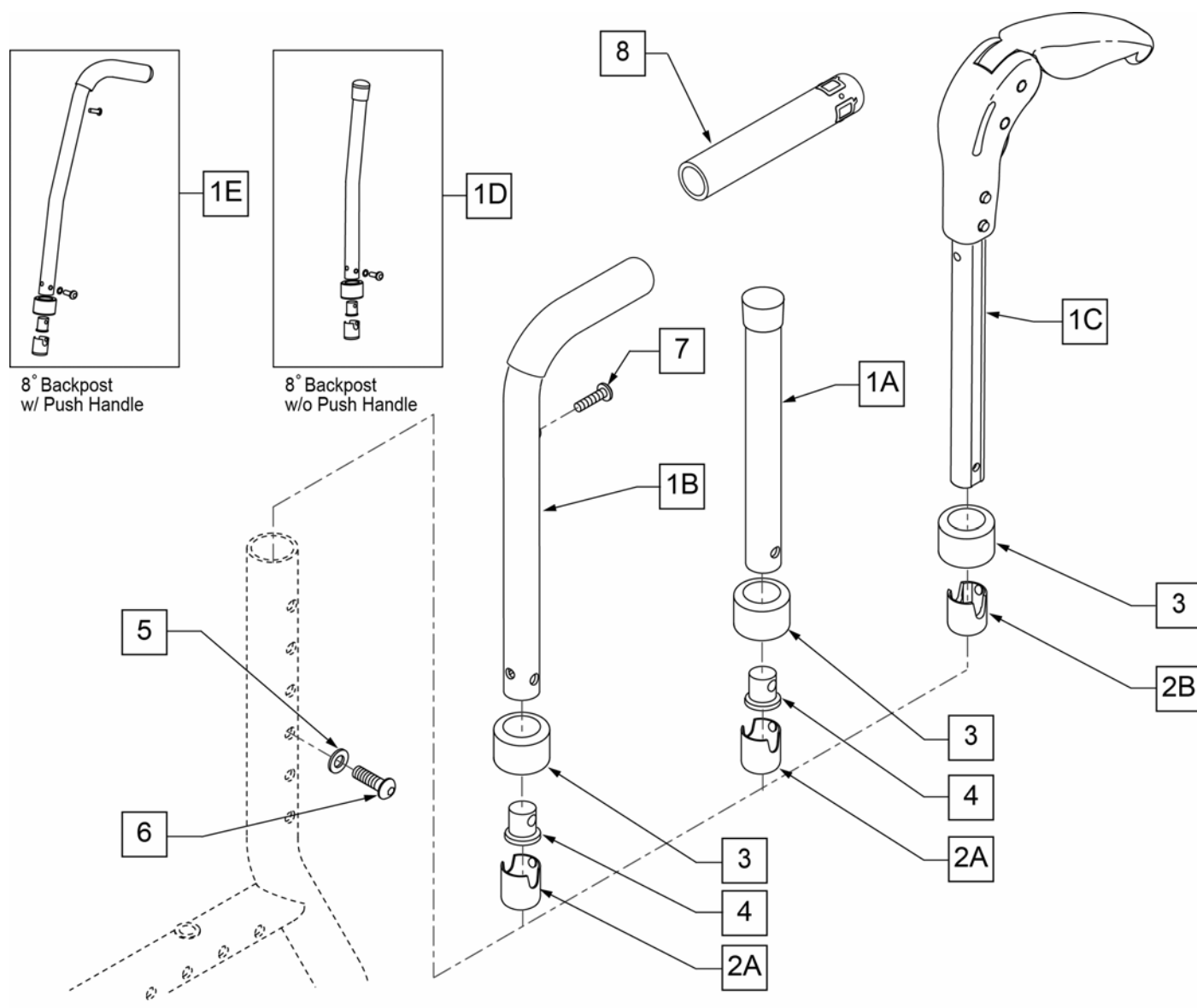


Pos.	Item Number	Description	Remarks
2A	104187	BUSHING BCKRST NONFLDNG	Straight & Integral Push Handle
2B	104369	BUSHING BCKRST FOLDING HANDLE	Folding Push Handle Only
3	561018	1" TUBE END BUSHING THICK	
4	483001	Q2 FRAME/FOOT RETAINING PLUG	
5	104414	WAVE WASHER 6 MM	
6	108107	BHCS 1/4-20 X 3/4 ROHS	
7	107384	10/32 X 5/8 PAN HD PHIL	
8	920026	QUICKIE GRIP 3/4" W/INSTR SH.	Replacement grips include Q-logo
1A,2A,3-6	107642	STRT BACKPOST LOW ASM	Low, 9-12" Back Ht
1A,2A,3-6	107643	STRT BACKPOST MED ASM	Med, 13-16" Back Height
1A,2A,3-6	107644	STRT BACKPOST TAL ASM	Tall, 17-20" Back Height
1B,2A-8	107645	INTGRL P/H LOW ASM	Low, 9-12" Back Ht
1B,2A-8	107646	INTGRL P/H MED ASM	Med, 13-16" Back Height
1B,2A-8	107647	INTGRL P/H TAL ASM	Tall, 17-20" Back Height
1C,2B,3,5,6	107648	FLD P/H FXD MED ASSM	Med, 12-15" Back Height

Pos.	Item Number	Description	Remarks
1C,2B,3,5,6	107649	FLD P/H FXD TAL ASSM	Tall, 16-19" Back Height
1D,2A,3-6	112392	8 DEG BACKPOST MED ASM W/O PUSH HANDLE	
1D,2A,3-6	112393	8 DEG BACKPOST TALL ASSM W/O PUSH HANDLE	
1E,2A-8	112394	8 DEG INTGRL PUSH HANDLE MED ASSM	
1E,2A-8	112395	8 DEG INTGRL PUSH HANDLE TALL ASSM	

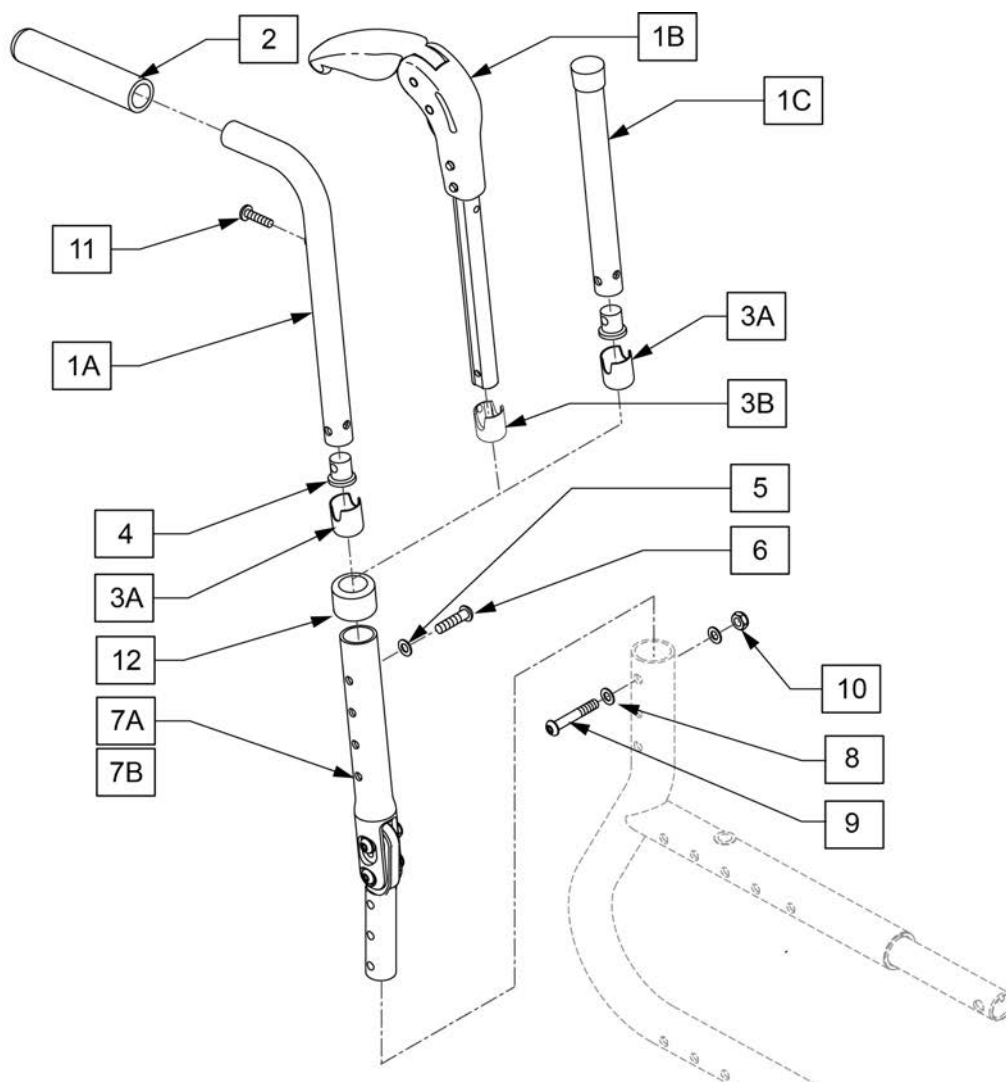


Pos.	Item Number	Description	Remarks
1A,2	104093-001	NON-FOLDING PH ASSY SHORT	Low-See Assm for Back Ht
1A,2	104093-002	NON-FOLD PH ASSY MEDIUM	Med-See Assm for Back Ht
1B	882606	FLDG PSH HNDL W/INSRT MED	Med-See Assm for Back Ht
1C	104236-001	TUBE BACKREST STRAIGHT SHT.	Low-See Assm for Back Ht
1C	104236-002	TUBE BACKREST STRAIGHT MED.	Med-See Assm for Back Ht
3A	104187	BUSHING BCKRST NONFLDNG	Straight & Integral Push Handle
3B	104369	BUSHING BCKRST FOLDING HANDLE	Folding Push Handle Only
4	483001	Q2 FRAME/FOOT RETAINING PLUG	
5	104414	WAVE WASHER 6 MM	
6	108107	BHCS 1/4-20 X 3/4 ROHS	
7A	104228-001	ANGLE ADJUST BACKREST SHT	3" PIVOT
7B	104228-003	ANGLE ADJUST BACKREST TLL	4" PIVOT
8	106943	WASHER FLAT AN 1/4 ROHS	
9	106972	HHCS 1/4-28 X 1-1/2 ROHS	
10	106935	NUT NYLOCK THIN 1/4-28 ROHS	
11	107384	PAN HD PHIL 10/32 X 5/8 ROHS	
12	561018	1" TUBE END BUSHING THICK	
1C,3A-6,7A	107650	STRT 3" ANG ADJ LOW ASM	3" PIVOT - Low, 13-16" Back Height

Pos.	Item Number	Description	Remarks
1C,3A-6,7A	107651	STRT 3" ANG ADJ MED ASM	3" PIVOT - Med, 17-20" Back Height
1A-6,7A,11	107652	INTGRL P/H 3" ANG ADJ LOW ASM	3" PIVOT - Low, 13-16" Back Height
1A-6,7A,11	107653	INTGRL P/H 3" ANG ADJ MED ASM	3" PIVOT - Med, 17-20" Back Height
1B,3B-6,7A	107654	FLD P/H 3" ANG ADJ MED ASSM	3" PIVOT - Med, 16-19" Back Height
1C,3A-6,7B	107655	STRT 4" ANG ADJ LOW ASM	4" PIVOT - Low, 14-17" Back Ht
1C,3A-6,7B	107656	STRT 4" ANG ADJ MED ASM	4" PIVOT - Med, 18-21" Back Ht
1A-6,7B,11	107657	INTGRL P/H 4" ANG ADJ LOW ASM	4" PIVOT - Low, 14-17" Back Ht
1A-6,7B,11	107658	INTGRL P/H 4 " ANG ADJ MED ASM	4" PIVOT - Med, 18-21" Back Ht
1B,3B-6,7B	107659	FLD P/H 4" ANG ADJ MED ASSM	4" PIVOT - Med, 17-20" Back Ht

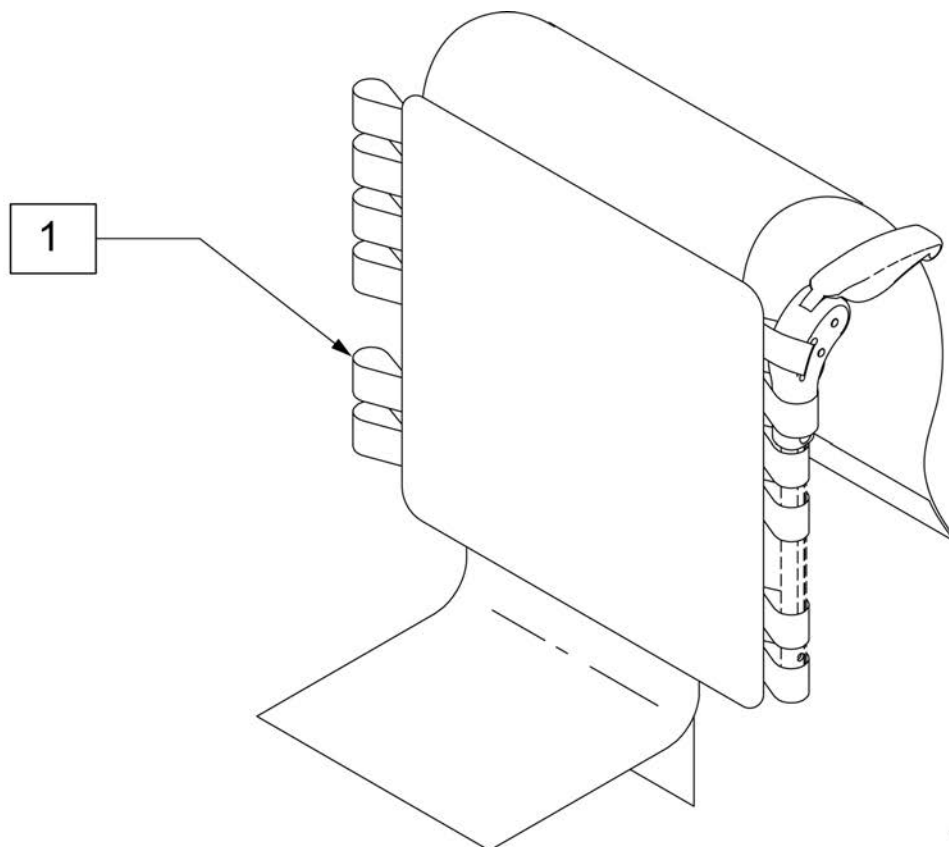

Pos.	Item Number	Description	Remarks
1	687468	Upholstery, FPH, Adj, CE, 14W X 12H	Med Std Back Frame
1	687471	Upholstery, FPH, Adj, CE, 14W X 16H	Tall Std Back Frame/Med Ang Adj Back Frame
1	687474	Upholstery, FPH, Adj, CE, 15W X 12H	Med Std Back Frame
1	687477	Upholstery, FPH, Adj, CE, 15W X 16H	Tall Std Back Frame/Med Ang Adj Back Frame
1	687480	Upholstery, FPH, Adj, CE, 16W X 12H	Med Std Back Frame
1	687483	Upholstery, FPH, Adj, CE, 16W X 16H	Tall Std Back Frame/Med Ang Adj Back Frame
1	687486	Upholstery, FPH, Adj, CE, 17W X 12H	Med Std Back Frame
1	687489	Upholstery, FPH, Adj, CE, 17W X 16H	Tall Std Back Frame/Med Ang Adj Back Frame
1	687492	Upholstery, FPH, Adj, CE, 18W X 12H	Med Std Back Frame
1	687495	Upholstery, FPH, Adj, CE, 18W X 16H	Tall Std Back Frame/Med Ang Adj Back Frame
1	687541	Upholstery, FPH, Adj, CE, 19W X 12H	Med Std Back Frame
1	687543	Upholstery, FPH, Adj, CE, 19W X 16H	Tall Std Back Frame/Med Ang Adj Back Frame
1	687545	Upholstery, FPH, Adj, CE, 20W X 12H	Med Std Back Frame
1	687547	Upholstery, FPH, Adj, CE, 20W X 16H	Tall Std Back Frame/Med Ang Adj Back Frame

Note: For complete back upholstery order both Item 1 Back Upholstery & Item 2 Back Strap Cover.

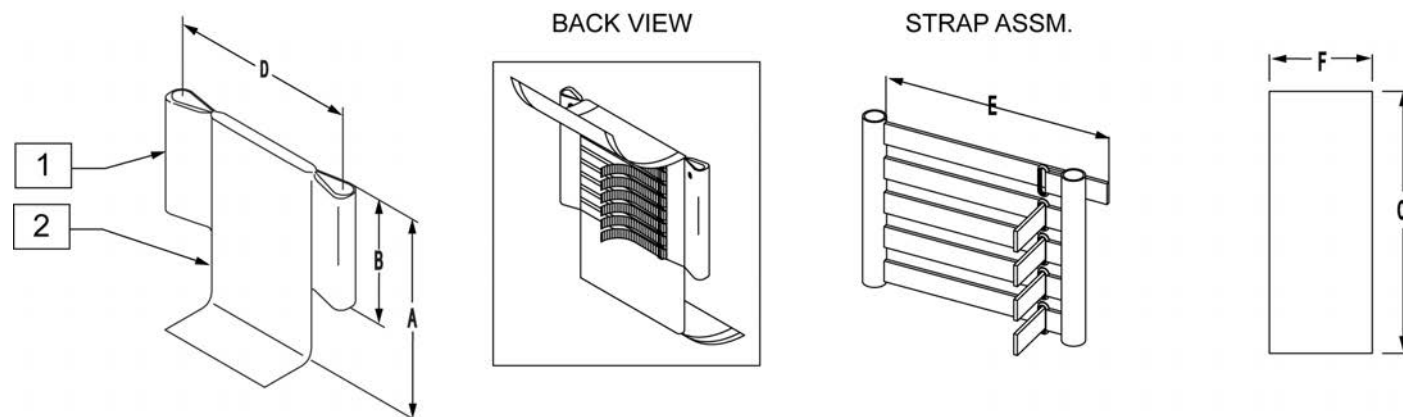


ADJUSTABLE BACK UPHOLSTERY MEASURING GUIDE


HEIGHT TABLE		STRAP ASSM.		PAD ASSM.
	BACK HEIGHT "A"	NO. OF STRAPS	HEIGHT "B"	LENGTH "C"
SHORT/ LOW	11-14"	5	6 1/2"	25"
MED	12-16"	6	8 1/2"	29"
TALL	14-18"	8	10 1/2"	33"
X-TALL	16-20"	9	12 1/2"	37"

HEIGHT TABLE		WIDTH "D"					
		SMALL 10-12"	MED 12-14"	LARGE 14-16"	X-LARGE 16-18"	X-LARGE 18-20"	X-LARGE 20-22"
STRAP ASSM	STRAP LENGTH "E"	9 1/2"	13 1/2"	17 1/2"	17 1/2"	21 1/2"	21 1/2"
PAD ASSM	WIDTH "F"	8"	10"	12"	14"	16"	18"

Pos.	Item Number	Description	Remarks
1	680236	GROWING ADJ BACK UPHOLSTERY 14-16 SHORT ZGS	14-15 wide, Low Std Back
1	680238	GROWING ADJ BACK UPHOLSTERY 14-16 TALL ZGS	14-15 wide, Med Std Back/Low Ang Adj Back
1	680222	GROWING ADJ BACK UPHOLSTERY 14-16 X-TALL ZGS	14-15 wide, Tall Std Back/Med Ang Adj Back
1	680239	GROWING ADJ BACK UPHOLSTERY 16-18 SHORT ZGS	16-17 wide, Low Std Back
1	680241	GROWING ADJ BACK UPHOLSTERY 16-18 TALL ZGS	16-17 wide, Med Std Back/Low Ang Adj Back
1	680242	GROWING ADJ BACK UPHOLSTERY 16-18 X-TALL ZGS	16-17 wide, Tall Std Back/Med Ang Adj Back
1	680243	GROWING ADJ BACK UPHOLSTERY 18-22 SHORT ZGS	18-20 wide, Low Std Back
1	680245	GROWING ADJ BACK UPHOLSTERY 18-22 TALL ZGS	18-20 wide, Med Std Back/Low Ang Adj Back
1	680246	GROWING ADJ BACK UPHOLSTERY 18-22 X-TALL ZGS	18-20 wide, Tall Std Back/Med Ang Adj Back
2	684486	ADJ BCK STRP CVR 14-16 SHT ZGS	14-15 wide, Low Std Back
2	684488	ADJUSTABLE BACK STRAP COVER 14-16 TALL ZGS	14-15 wide, Med Std Back/Low Ang Adj Back
2	684499	ADJ BCK STRP CVR 14-16 XTL ZGS	14-15 wide, Tall Std Back/Med Ang Adj Back
2	684489	ADJUSTABLE BACK STRAP COVER 16-18 SHORT ZGS	16-17 wide, Low Std Back
2	684491	ADJUSTABLE BACK STRAP COVER 16-18 TALL ZGS	16-17 wide, Med Std Back/Low Ang Adj Back
2	684492	ADJUSTABLE BACK STRAP COVER 16-18 X-TALL ZGS	16-17 wide, Tall Std Back/Med Ang Adj Back
2	684493	ADJUSTABLE BACK STRAP COVER 18-20 SHORT ZGS	18-20 wide, Low Std Back
2	684495	ADJUSTABLE BACK STRAP COVER 18-20 TALL ZGS	18-20 wide, Med Std Back/Low Ang Adj Back
2	684496	ADJUSTABLE BACK STRAP COVER 18-20 X-TALL ZGS	18-20 wide, Tall Std Back/Med Ang Adj Back

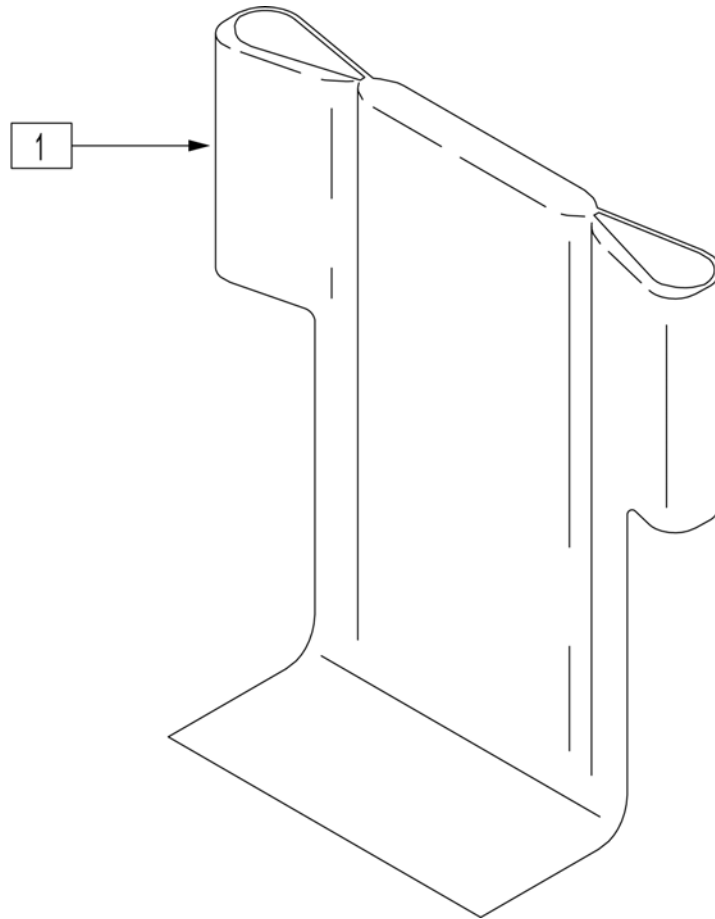


Pos.	Item Number	Description	Remarks
1	680454	BACK UPH 14" 11-14/12-16 RDG	14" Wide - Low Back Height
1	680455	BACK UPH 15" 11-14/12-16 RDG	15" Wide - Low Back Height
1	680456	BACK UPH 16" 11-14/12-16 RDG	16" Wide - Low Back Height
1	680457	BACK UPH 17" 11-14/12-16 RDG	17" Wide - Low Back Height
1	680458	BACK UPH 18" 11-14/12-16 RDG	18" Wide - Low Back Height
1	680459	BACK UPH 19" 11-14/12-16 RDG	19" Wide - Low Back Height
1	680460	BACK UPH 20" 11-14/12-16 RDG	20" Wide - Low Back Height
1	680464	BACK UPH 14" 14-18 RDG	14" Wide - Med STD Back Ht/ Low Angle Adg Back
1	680465	BACK UPH 15" 14-18 RDG	15" Wide - Med STD Back Ht/ Low Angle Adg Back
1	680466	BACK UPH 16" 14-18 RDG	16" Wide - Med STD Back Ht/ Low Angle Adg Back
1	680467	BACK UPH 17" 14-18 RDG	17" Wide - Med STD Back Ht/ Low Angle Adg Back
1	680468	BACK UPH 18" 14-18 RDG	18" Wide - Med STD Back Ht/ Low Angle Adg Back
1	680469	BACK UPH 19" 14-18 RDG	19" Wide - Med STD Back Ht/ Low Angle Adg Back
1	680470	BACK UPH 20" 14-18 RDG	20" Wide - Med STD Back Ht/ Low Angle Adg Back
1	680474	BACK UPH 14" 16-20 RDG	14" Wide - Tall STD Back Ht/ Med Angle Adg Back
1	680475	BACK UPH 15" 16-20 RDG	15" Wide - Tall STD Back Ht/ Med Angle Adg Back
1	680476	BACK UPH 16" 16-20 RDG	16" Wide - Tall STD Back Ht/ Med Angle Adg Back
1	680477	BACK UPH 17" 16-20 RDG	17" Wide - Tall STD Back Ht/ Med Angle Adg Back
1	680478	BACK UPH 18" 16-20 RDG	18" Wide - Tall STD Back Ht/ Med Angle Adg Back
1	680479	BACK UPH 19" 16-20 RDG	19" Wide - Tall STD Back Ht/ Med Angle Adg Back
1	680480	BACK UPH 20" 16-20 RDG	20" Wide - Tall STD Back Ht/ Med Angle Adg Back

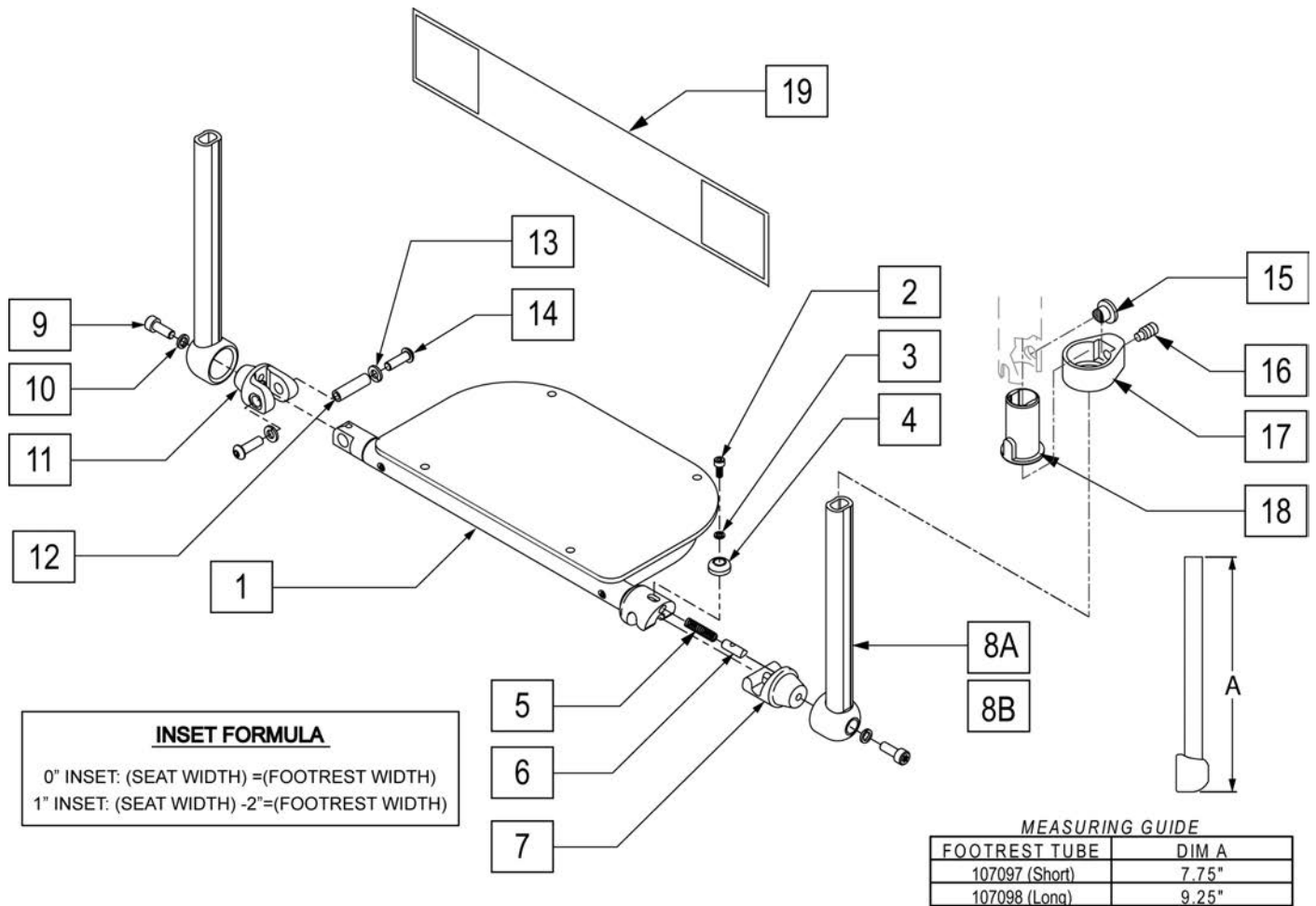


Pos.	Item Number	Description	Remarks
2	230132	SHCS M4 X .7 X 10	
3	050129-002	LOCKWASHER HIGH COLLAR M4	
4	052289	BUTTON LEVER LATCH	
5	100837	SPRING COMP .25OD X 1.0LG	
6	052103	LATCH PLUNGER FOOTREST	
7	100842	ASSY LATCH PIVOT FOOTREST	
8A	107097	WELDMENT TUBE FOOTREST STD	Short
8B	107098	WELDMENT TUBE FOOTREST LNG	Long
9	102263-001	SHCS M6X1X16 FULL NP	
10	107362	LOCKWASHER HIGH COLLAR M6	
11	052022	CLEVIS PIVOT TUBE	
12	052282	PIN PIVOT FOOTREST	
13	262133	WASHER LOCK M6	
14	013147	BHCS M6X1X12MM BLK	
18	104160	BUSHING FOOTREST	
19	740130	LEG STRAP 10"-14" BLK	
19	740131	LEG STRAP 15"-17" BLK	
19	740132	LEG STRAP 18"-20" BLK	
1-6	100830-010	ASSY TUBE FRST TBLR W-CVR 12W	12 wide
1-6	100830-011	ASSY TUBE FRST TBLR W-CVR 13W	13 wide
1-6	100830-012	ASSY TUBE FRST TBLR W-CVR 14W	14 wide

Pos.	Item Number	Description	Remarks
1-6	100830-013	ASSY TUBE FRST TBLR W-CVR 15W	15 wide
1-6	100830-014	ASSY TUBE FRST TBLR W-CVR 16W	16 wide
1-6	100830-015	ASSY TUBE FRST TBLR W-CVR 17W	17 wide
1-6	100830-016	ASSY TUBE FRST TBLR W-CVR 18W	18 wide
1-6	100830-017	ASSY TUBE FRST TBLR W-CVR 19W	19 wide
1-6	100830-018	ASSY TUBE FRST TBLR W-CVR 20W	20 wide
1-8A,9-14,19	107686	ASSM 12W SHORT	12 wide (Short)
1-8A,9-14,19	107687	ASSM 13W SHORT	13 wide (Short)
1-8A,9-14,19	107688	ASSM 14W SHORT	14 wide (Short)
1-8A,9-14,19	107689	ASSM 15W SHORT	15 wide (Short)
1-8A,9-14,19	107690	ASSM 16W SHORT	16 wide (Short)
1-8A,9-14,19	107691	ASSM 17W SHORT	17 wide (Short)
1-8A,9-14,19	107692	ASSM 18W SHORT	18 wide (Short)
1-8A,9-14,19	107693	ASSM 19W SHORT	19 wide (Short)
1-8A,9-14,19	107694	ASSM 20W SHORT	20 wide (Short)
1-8B,9-14,19	107695	ASSM 12W LONG	12 wide (Long)
1-8B,9-14,19	107696	ASSM 13W LONG	13 wide (Long)
1-8B,9-14,19	107697	ASSM 14W LONG	14 wide (Long)
1-8B,9-14,19	107698	ASSM 15W LONG	15 wide (Long)
1-8B,9-14,19	107699	ASSM 16W LONG	16 wide (Long)
1-8B,9-14,19	107700	ASSM 17W LONG	17 wide (Long)
1-8B,9-14,19	107701	ASSM 18W LONG	18 wide (Long)
1-8B,9-14,19	107702	ASSM 19W LONG	19 wide (Long)
1-8B,9-14,19	107703	ASSM 20W SHORT	20 wide (Long)
15-18	107485	FOOTREST CLAMP ASSM	

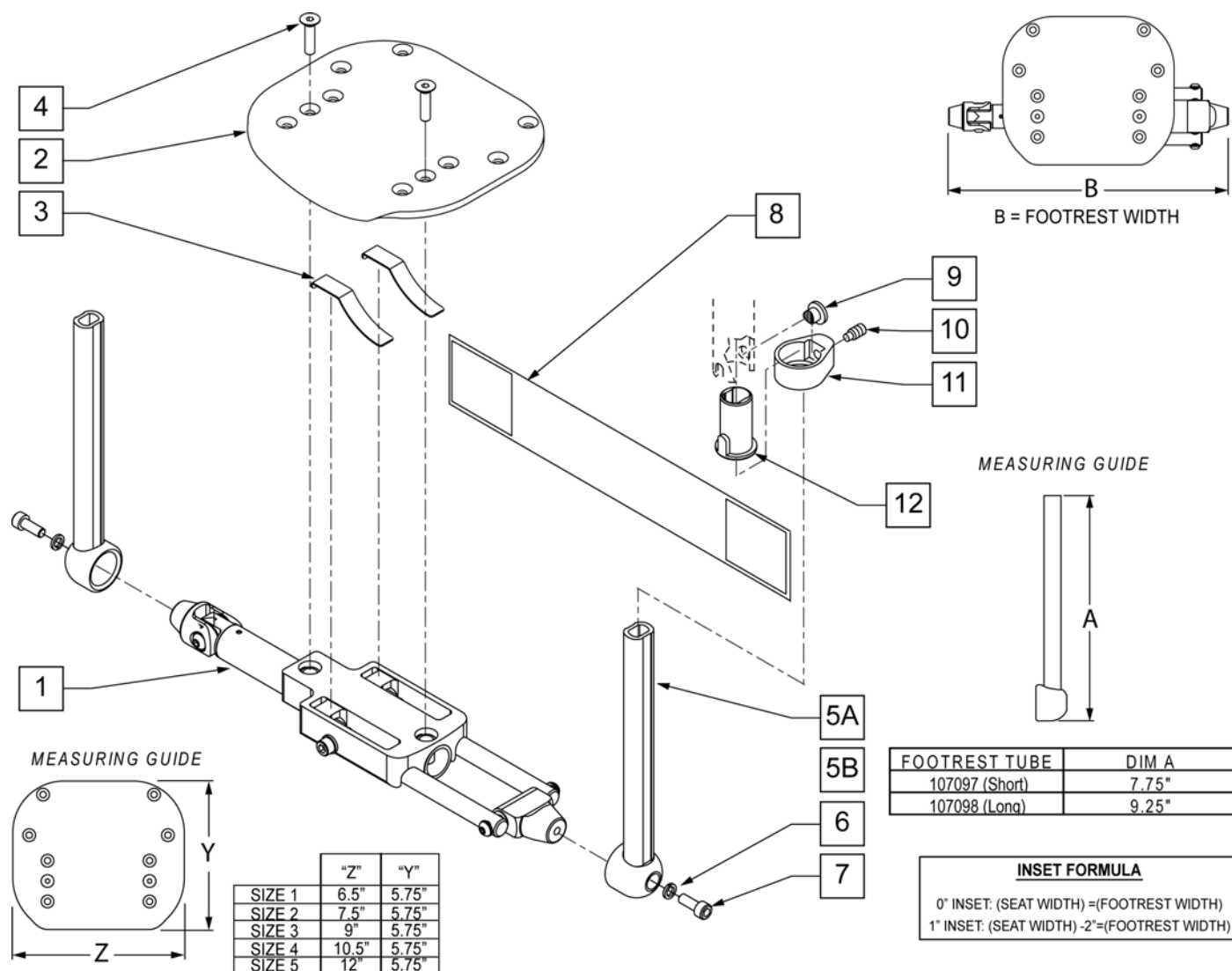

Pos.	Item Number	Description	Remarks
2	054853-001	FOOTPLATE SIZE 1	12" wide
2	054853-002	FOOTPLATE SIZE 2	13-14" wide
2	054853-003	FOOTPLATE SIZE 3	15-16" wide
2	054853-004	FOOTPLATE SIZE 4	17-18" wide
2	054853-005	FOOTPLATE SIZE 5	19-20" wide
3	054852	SPRING LEAF FOOTREST	
4	102265-003	FHCS M6X1X25 FULL NP	
5A	107097	WELDMENT TUBE FOOTREST STD	Short
5B	107098	WELDMENT TUBE FOOTREST LNG	Long
6	107362	LOCKWASHER HIGH COLLAR M6	
7	102263-001	SHCS M6X1X16 FULL NP	
8	740130	LEG STRAP 10"-14" BLK	12-14" wide
8	740131	LEG STRAP 15"-17" BLK	15-17" wide
8	740132	LEG STRAP 18"-20" BLK	18-20" wide
12	104160	BUSHING FOOTREST	
1-4	102830-001	ASSY FOOTPLATE AUTOFOLD 12W	12 wide
1-4	102830-002	ASSY FOOTPLATE AUTOFOLD 13W	13 wide

Pos.	Item Number	Description	Remarks
1-4	102830-003	ASSY FOOTPLATE AUTOFOLD 14W	14 wide
1-4	102830-004	ASSY FOOTPLATE AUTOFOLD 15W	15 wide
1-4	102830-005	ASSY FOOTPLATE AUTOFOLD 16W	16 wide
1-4	102830-006	ASSY FOOTPLATE AUTOFOLD 17W	17 wide
1-4	102830-007	ASSY FOOTPLATE AUTOFOLD 18W	18 wide
1-4	102830-008	ASSY FOOTPLATE AUTOFOLD 19W	19 wide
1-4	102830-009	ASSY FOOTPLATE AUTOFOLD 20W	20 wide
1-5A,6-8	107597	ASSM 12W SHORT	12 wide (Short)
1-5A,6-8	107598	ASSM 13W SHORT	13 wide (Short)
1-5A,6-8	107599	ASSM 14W SHORT	14 wide (Short)
1-5A,6-8	107600	ASSM 15W SHORT	15 wide (Short)
1-5A,6-8	107601	ASSM 16W SHORT	16 wide (Short)
1-5A,6-8	107602	ASSM 17W SHORT	17 wide (Short)
1-5A,6-8	107603	ASSM 18W SHORT	18 wide (Short)
1-5A,6-8	107604	ASSM 19W SHORT	19 wide (Short)
1-5A,6-8	107605	ASSM 20W SHORT	20 wide (Short)
1-5B,6-8	107606	ASSM 12W LONG	12 wide (Long)
1-5B,6-8	107607	ASSM 13W LONG	13 wide (Long)
1-5B,6-8	107608	ASSM 14W LONG	14 wide (Long)
1-5B,6-8	107609	ASSM 15W LONG	15 wide (Long)
1-5B,6-8	107610	ASSM 16W LONG	16 wide (Long)
1-5B,6-8	107611	ASSM 17W LONG	17 wide (Long)
1-5B,6-8	107612	ASSM 18W LONG	18 wide (Long)
1-5B,6-8	107613	ASSM 19W LONG	19 wide (Long)
1-5B,6-8	107614	ASSM 20W SHORT	20 wide (Long)
9-12	107485	FOOTREST CLAMP ASSM	

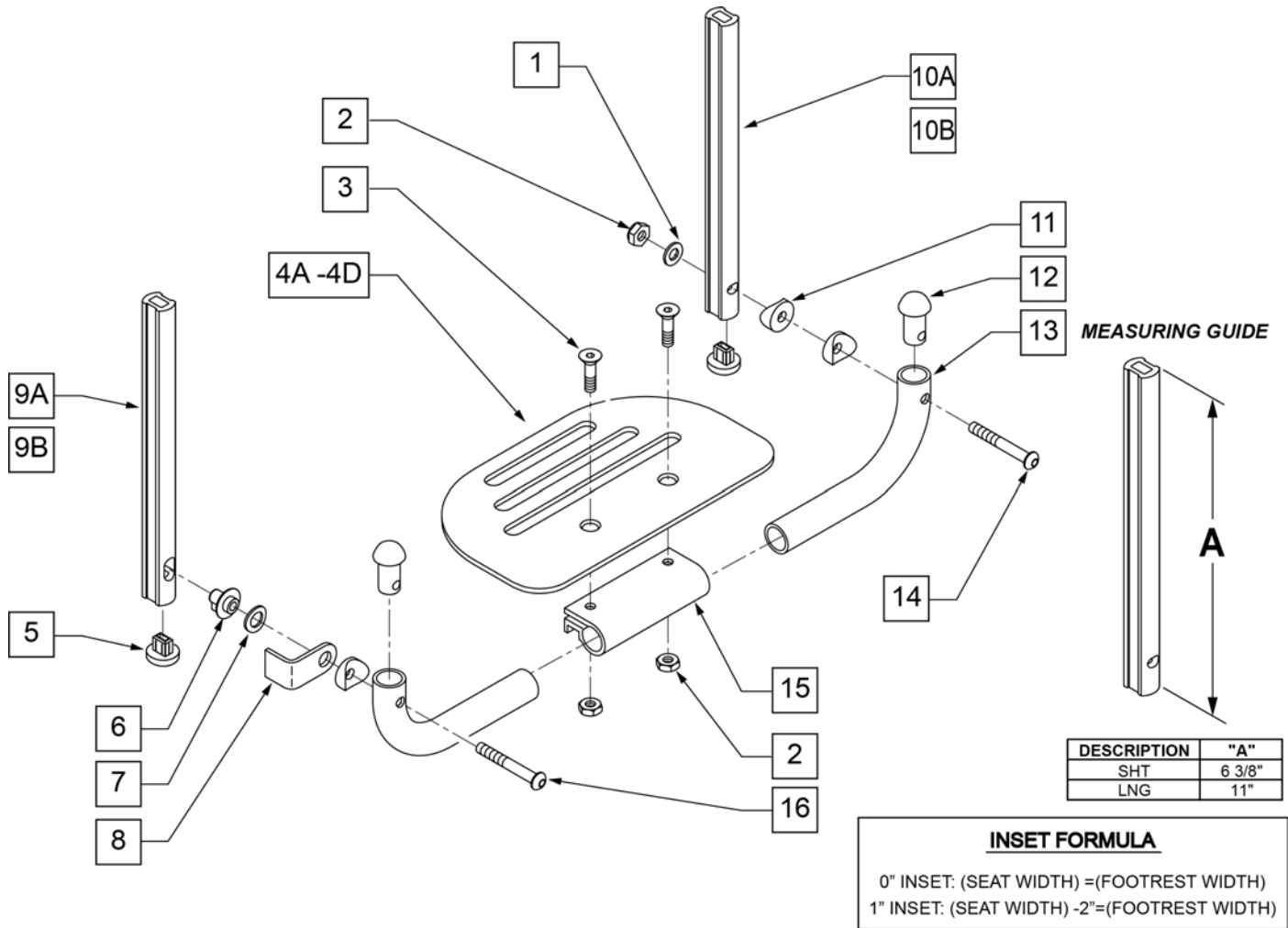

Pos.	Item Number	Description	Remarks
1	106944	WASHER FLAT THIN AN 1/4 ROHS	
2	106941	NUT AUTOMATION 1/4-28 ROHS	
3	106978	FHCS 1/4-28 X 7/8 ROHS	
4A	500651	PLATFORM F/PLT 8 x 5 1/2"	12 -14" Footrest Width
4B	500652	PLATFORM F/PLT 10" x 5 1/2"	15 -17" Footrest Width
4C	500653	PLATFORM F/PLT 12" x 5 1/2"	18 -20" Footrest Width
4D	500654	PLATFORM F/PLT 14" x 5 1/2"	21 -22" Footrest Width
5	113221	PLUG DOUBLE D EXT TUBE	
6	540410	F/UP FOOTREST KEY STUD D2	
7	107385	WASHER CURVED 3/8 ROHS	
8	505078	FOOTREST RETAINING BRACKET	
9A	106875	EXT TBE F/UP KEY SHT	Short -Latch Side
9B	106876	EXT TBE F/UP KEY LNG	Long -Latch Side
10A	106873	EXT TBE F/UP SLT SHT	Short -Pivot Side
10B	106874	EXT TBE F/UP SLT LNG	Long -Pivot Side
11	561003	3/4" SADDLE THIN	
12	561034	SHORT DOME PLUG	
14	106964	BHCS 1/4-28 X 2 ROHS	
15	550351	PLATFORM FOOTPLATE CLAMP	

Pos.	Item Number	Description	Remarks
16	107027	BHCS 1/4-28 X 1-1/4 SHLDR ROHS	
17A	740130	LEG STRAP 10"-14" BLK	12-14" Footrest Width
17B	740131	LEG STRAP 15"-17" BLK	15-17" Footrest Width
17C	740132	LEG STRAP 18"-20" BLK	18-20" Footrest Width
1-4A,6-8,11-16	900306	PLTFM F/P PLT BLK 12" D2 W/ INSTR	12" Footrest Width
1-4A,6-8,11-16	900307	PLTFM F/P PLT BLK 13" D2 W/ INSTR	13" Footrest Width
1-4A,6-8,11-16	900308	PLTFM F/P PLT BLK 14" D2 W /INSTR	14" Footrest Width
1-4B,6-8,11-16	900309	PLTFM F/P PLT BLK 15" D2 W /INSTR	15" Footrest Width
1-4B,6-8,11-16	900310	PLTFM F/P PLT BLK 16" D2 W /INSTR	16" Footrest Width
1-4B,6-8,11-16	900311	PLTFM F/P PLT BLK 17" D2 W /INSTR	17" Footrest Width
1-4C,6-8,11-16	900312	PLTFM F/P PLT BLK 18" D2 W /INSTR	18" Footrest Width
1-4C,6-8,11-16	900313	PLTFM F/P PLT BLK 19" D2 W /INSTR	19" Footrest Width
1-4C,6-8,11-16	900314	PLTFM F/P PLT BLK 20" D2 W/ INSTR	20" Footrest Width
1-4D,6-8,11-16	900315	PLTFM F/P PLT BLK 21" D2 W/ INSTR	21" Footrest Width
1-4D,6-8,11-16	900316	PLTFM F/P PLT BLK 22" D2 W/ INSTR	22" Footrest Width
		SHORT EXT ASSEMBLIES	
1-3,4A,5-8,9A,10B,11-16	108043	PLTFRM FLIP FTRST SHT 12W	12" wide (Short)
1-3,4A,5-8,9A,10B,11-16	108044	PLTFRM FLIP FTRST SHT 13W	13" wide (Short)
1-3,4A,5-8,9A,10B,11-16	108045	PLTFRM FLIP FTRST SHT 14W	14" wide (Short)
1-3,4B,5-8,9A,10B,11-16	108046	PLTFRM FLIP FTRST SHT 15W	15" wide (Short)
1-3,4B,5-8,9A,10B,11-16	108047	PLTFRM FLIP FTRST SHT 16W	16" wide (Short)
1-3,4B,5-8,9A,10B,11-16	108048	PLTFRM FLIP FTRST SHT 17W	17" wide (Short)
1-3,4C,5-8,9A,10B,11-16	108049	PLTFRM FLIP FTRST SHT 18W	18" wide (Short)
1-3,4C,5-8,9A,10B,11-16	108051	PLTFRM FLIP FTRST SHT 19W	19" wide (Short)
1-3,4C,5-8,9A,10B,11-16	108052	PLTFRM FLIP FTRST SHT 20W	20" wide (Short)
1-3,4D,5-8,9A,10B,11-16	111554	PLTFRM FLIP FTRST SHT 21W	21" wide (Short)
1-3,4D,5-8,9A,10B,11-16	111555	PLTFRM FLIP FTRST SHT 22W	22" wide (Short)
		LONG EXT ASSEMBLIES	
1-3,4A,5-8,9B,10A,11-16	108053	PLTFRM FLIP FTRST LONG 12W	12" wide (Long)
1-3,4A,5-8,9B,10A,11-16	108054	PLTFRM FLIP FTRST LONG 13W	13" wide (Long)
1-3,4A,5-8,9B,10A,11-16	108055	PLTFRM FLIP FTRST LONG 14W	14" wide (Long)
1-3,4B,5-8,9B,10A,11-16	108056	PLTFRM FLIP FTRST LONG 15W	15" wide (Long)
1-3,4B,5-8,9B,10A,11-16	108057	PLTFRM FLIP FTRST LONG 16W	16" wide (Long)
1-3,4B,5-8,9B,10A,11-16	108058	PLTFRM FLIP FTRST LONG 17W	17" wide (Long)
1-3,4C,5-8,9B,10A,11-16	108059	PLTFRM FLIP FTRST LONG 18W	18" wide (Long)
1-3,4C,5-8,9B,10A,11-16	108060	PLTFRM FLIP FTRST LONG 19W	19" wide (Long)
1-3,4C,5-8,9B,10A,11-16	108061	PLTFRM FLIP FTRST LONG 20W	20" wide (Long)
1-3,4D,5-8,9B,10A,11-16	111556	PLTFRM FLIP FTRST LONG 21W	21" wide (Long)
1-3,4D,5-8,9B,10A,11-16	111557	PLTFRM FLIP FTRST LONG 22W	22" wide (Long)

COMPOSITE FOOTPLATE

[09/2011]

NOTE: FOOTPLATE ASSEMBLIES ARE SET TO FOOTREST WIDTH. PLEASE SEE "FOOTPLATE FORMULA" TABLE FOR SEAT WIDTH CONVERSION.

INSET FORMULA
0" Inset (Seat Width) = Footrest Width
1" Inset (Seat Width) -2 = Footrest Width
"V" Hanger (Seat Width) -4 = Footrest Width

Composite Footplates w/ Ext & HL for Swing I/O Hangers
(Order 1 each)

Width	60°, 70° & 70° Hemi Style Hangers (Bent Extension)			80° Style hanger ELR (Straight Extension)			70° V Style Hangers	
	Short	Medium	Long	Short	Medium	Long	Medium	Long
13" Right	107746	107760	107774	107704	107718	107732	107788	107802
13" Left	107753	107767	107781	107711	107725	107739	107795	107809
14" Right	107747	107761	107775	107705	107719	107733	107789	107803
14" Left	107754	107768	107782	107712	107726	107740	107796	107810
15" Right	107748	107762	107776	107706	107720	107734	107790	107804
15" Left	107755	107769	107783	107713	107727	107741	107797	107811
16" Right	107749	107763	107777	107707	107721	107735	107791	107805
16" Left	107756	107770	107784	107714	107728	107742	107798	107812
17" Right	107750	107764	107778	107708	107722	107736	107792	107806
17" Left	107757	107771	107785	107715	107729	107743	107799	107813
18" Right	107751	107765	107779	107709	107723	107737	107793	107807
18" Left	107758	107772	107786	107716	107730	107744	107800	107814
19"-20" Rt	107752	107766	107780	107710	107724	107738	V STYLE IS NOT AVAILABLE IN THESE SEAT WIDTHS	
19"-20" Lt	107759	107773	107787	107717	107731	107745		
21"-22" Rt	111528	111530	111532	111522	111524	111526		
21"-22" Lt	111529	111531	111533	111523	111525	111527		

FOOTPLATE MEASURING GUIDE

PART NO	DESCRIPTION	DIM -A-	DIM -B-
063216-006	FOOTPLATE COMPOSITE 157 R	6 1/8"	6"
063216-012	FOOTPLATE COMPOSITE 157 L	6 1/8"	6"
063216-015	FOOTPLATE COMPOSITE 170 R	6 5/8"	6"
063216-018	FOOTPLATE COMPOSITE 170 L	6 5/8"	6"
063216-005	FOOTPLATE COMPOSITE 182 R	7 1/8"	6"
063216-011	FOOTPLATE COMPOSITE 182 L	7 1/8"	6"
063216-014	FOOTPLATE COMPOSITE 195 R	7 5/8"	6"
063216-017	FOOTPLATE COMPOSITE 195 L	7 5/8"	6"
063216-004	FOOTPLATE COMPOSITE 207 R	8 1/8"	6"
063216-010	FOOTPLATE COMPOSITE 207 L	8 1/8"	6"
063216-013	FOOTPLATE COMPOSITE 220 R	8 5/8"	6 1/4"
063216-016	FOOTPLATE COMPOSITE 220 L	8 5/8"	6 1/4"
063216-003	FOOTPLATE COMPOSITE 232 R	9 1/8"	6 1/4"
063216-009	FOOTPLATE COMPOSITE 232 L	9 1/8"	6 1/4"
063216-002	FOOTPLATE COMPOSITE 257 R	10 1/8"	6 1/4"
063216-008	FOOTPLATE COMPOSITE 257 L	10 1/8"	6 1/4"
063216-001	FOOTPLATE COMPOSITE 282 R	11 1/8"	6 1/4"
063216-007	FOOTPLATE COMPOSITE 282 L	11 1/8"	6 1/4"

Pos.	Item Number	Description	Remarks
1A-5	120660	SHT STRT EXT COMP HDWR	80 deg hangers + ELR only
1B-5	120661	MED STRT EXT COMP HDWR	80 deg hangers + ELR only
1C-5	120662	LNG STRT EXT COMP HDWR	80 deg hangers + ELR only
1D-5	120663	SHT BNT EXT COMP HDWR	60,70,70 Hemi Hangers
1E-5	120664	MED BNT EXT COMP HDWR	60,70,70 Hemi Hangers
1F-5	120665	LNG BNT EXT COMP HDWR	60,70,70 Hemi Hangers
1G-5	120666	SHT V EXT R COMP HDWR	70 deg V hangers only-LEFT
1G-5	120667	SHT V EXT L COMP HDWR	70 deg V hangers only-RIGHT
1H-5	120668	LNG V EXT R COMP HDWR	70 deg V hangers only-LEFT
1H-5	120669	LNG V EXT L COMP HDWR	70 deg V hangers only-RIGHT
2A	063216-006	COMPOSITE FOOTPLATE RIGHT	RIGHT 13" Footrest Width
2A	063216-012	COMPOSITE FOOTPLATE LEFT	LEFT 13" Footrest Width
2B	063216-015	FOOTPLATE COMPOSITE RIGHT	RIGHT 14" Footrest Width

Pos.	Item Number	Description	Remarks
2B	063216-018	FOOTPLATE COMPOSITE LEFT	LEFT 14" Footrest Width
2C	063216-005	COMPOSITE FOOTPLATE RIGHT	RIGHT 15" Footrest Width
2C	063216-011	COMPOSITE FOOTPLATE LEFT	LEFT 15" Footrest Width
2D	063216-014	FOOTPLATE COMPOSITE RIGHT	RIGHT 16" Footrest Width
2D	063216-017	FOOTPLATE COMPOSITE LEFT	LEFT 16" Footrest Width
2E	063216-004	COMPOSITE FOOTPLATE RIGHT	RIGHT 17" Footrest Width
2E	063216-010	COMPOSITE FOOTPLATE LEFT	LEFT 17" Footrest Width
2F	063216-013	FOOTPLATE COMPOSITE RIGHT	RIGHT 18" Footrest Width
2F	063216-016	FOOTPLATE COMPOSITE LEFT	LEFT 18" Footrest Width
2G	063216-003	COMPOSITE FOOTPLATE RIGHT	RIGHT 19" & 20" Footrest Width
2G	063216-009	COMPOSITE FOOTPLATE LEFT	LEFT 19" & 20" Footrest Width
2H	063216-002	FOOTPLATE COMPOSITE RIGHT	RIGHT 21" & 22" Footrest Width
2H	063216-008	FOOTPLATE COMPOSITE LEFT	LEFT 21" & 22" Footrest Width
3	104377	3/4" SADDLE THIN	
4	103175	SHCS M6 X 1 X 40 INORG	
5	104381	NUT HEX NYLOCK M6X1.0	Inserted in bottom of footplate.
6	104328	HEEL LOOP PEG	
7A	104380-001	ADJUSTABLE FOOT REST STRAP	14" wide
7B	104380-002	ADJUSTABLE FOOT REST STRAP	15-16" wide
7C	104380-003	ADJUSTABLE FOOT REST STRAP	17-18" wide
7D	104380-004	ADJUSTABLE FOOT REST STRAP	19-22" wide
8	102268-001	HHCS M6 X 1 X 25	
12	104160	BUSHING FOOTREST	
9-12	107485	FOOTREST CLAMP ASSM	
>>>>		60°,70°,70° HEMI HANGERS - SHORT EXT - RIGHT	
1D,2A,3-6,7A,8	107746	COMP BNT SHT EXT ASM 13" W R	Right- 70D- 13" W- SHORT
1D,2B,3-6,7A,8	107747	COMP BNT SHT EXT ASM 14" W R	Right- 70D- 14" W- SHORT
1D,2C,3-6,7B,8	107748	COMP BNT SHT EXT ASM 15" W R	Right- 70D- 15" W- SHORT
1D,2D,3-6,7B,8	107749	COMP BNT SHT EXT ASM 16" W R	Right- 70D- 16" W- SHORT
1D,2E,3-6,7C,8	107750	COMP BNT SHT EXT ASM 17" W R	Right- 70D- 17" W- SHORT
1D,2F,3-6,7C,8	107751	COMP BNT SHT EXT ASM 18" W R	Right- 70D- 18" W- SHORT
1D,2G,3-6,7D,8	107752	COMP BNT SHT EXT ASM 19 & 20"W R	Right- 70D- 19 & 20" W- SHORT
1D,2H,3-6,7D,8	111528	COMP BNT SHT EXT ASM 21 & 22"W R	Right- 70D- 21 & 22" W- SHORT
>>>>		60°,70°,70° HEMI HANGERS - SHORT EXT - LEFT	
1D,2A,3-6,7A,8	107753	COMP BNT SHT EXT ASM 13" W L	Left- 70D- 13 W- SHORT
1D,2B,3-6,7A,8	107754	COMP BNT SHT EXT ASM 14" W L	Left- 70D- 14 W- SHORT
1D,2C,3-6,7B,8	107755	COMP BNT SHT EXT ASM 15" W L	Left- 70D- 15 W- SHORT
1D,2D,3-6,7B,8	107756	COMP BNT SHT EXT ASM 16" W L	Left- 70D- 16 W- SHORT
1D,2E,3-6,7C,8	107757	COMP BNT SHT EXT ASM 17" W L	Left- 70D- 17 W- SHORT
1D,2F,3-6,7C,8	107758	COMP BNT SHT EXT ASM 18" W L	Left- 70D- 18 W- SHORT
1D,2G,3-6,7D,8	107759	COMP BNT SHT EXT ASM 19 & 20"W L	Left- 70D- 19 & 20" W- SHORT
1D,2H,3-6,7D,8	111529	COMP BNT SHT EXT ASM 21 & 22"W L	Left- 70D- 21 & 22" W- SHORT
>>>>		60°,70°,70° HEMI HANGERS - MED EXT - RIGHT	
1E,2A,3-6,7A,8	107760	COMP BNT MED EXT ASM 13" W R	Right- 70D- 13" W- MED
1E,2B,3-6,7A,8	107761	COMP BNT MED EXT ASM 14" W R	Right- 70D- 14" W- MED
1E,2C,3-6,7B,8	107762	COMP BNT MED EXT ASM 15" W R	Right- 70D- 15" W- MED
1E,2D,3-6,7B,8	107763	COMP BNT MED EXT ASM 16" W R	Right- 70D- 16" W- MED
1E,2E,3-6,7C,8	107764	COMP BNT MED EXT ASM 17" W R	Right- 70D- 17" W- MED
1E,2F,3-6,7C,8	107765	COMP BNT MED EXT ASM 18" W R	Right- 70D- 18" W- MED
1E,2G,3-6,7D,8	107766	COMP BNT MED EXT ASM 19 & 20"W R	Right- 70D- 19 & 20" W- MED
1E,2H,3-6,7D,8	111530	COMP BNT MED EXT ASM 21 & 22"W R	Right- 70D- 21 & 22" W- MED
>>>>		60°,70°,70° HEMI HANGERS - MED EXT - LEFT	
1E,2A,3-6,7A,8	107767	COMP BNT MED EXT ASM 13" W L	Left- 70D- 13 W- MED
1E,2B,3-6,7A,8	107768	COMP BNT MED EXT ASM 14" W L	Left- 70D- 14 W- MED
1E,2C,3-6,7B,8	107769	COMP BNT MED EXT ASM 15" W L	Left- 70D- 15 W- MED
1E,2D,3-6,7B,8	107770	COMP BNT MED EXT ASM 16" W L	Left- 70D- 16 W- MED

Pos.	Item Number	Description	Remarks
1E,2E,3-6,7C,8	107771	COMP BNT MED EXT ASM 17" W L	Left- 70D- 17 W- MED
1E,2F,3-6,7C,8	107772	COMP BNT MED EXT ASM 18" W L	Left- 70D- 18 W- MED
1E,2G,3-6,7D,8	107773	COMP BNT MED EXT ASM 19 & 20"W L	Left- 70D- 19 & 20" W- MED
1E,2H,3-6,7D,8	111531	COMP BNT MED EXT ASM 21 & 22"W L	Left- 70D- 21 & 22" W- MED
>>>>		60°,70°,70° HEMI HANGERS - LONG EXT - RIGHT	
1F,2A,3-6,7A,8	107774	COMP BNT LONG EXT ASM 13" W R	Right- 70D- 13" W- LONG
1F,2B,3-6,7A,8	107775	COMP BNT LONG EXT ASM 14" W R	Right- 70D- 14" W- LONG
1F,2C,3-6,7B,8	107776	COMP BNT LONG EXT ASM 15" W R	Right- 70D- 15" W- LONG
1F,2D,3-6,7B,8	107777	COMP BNT LONG EXT ASM 16" W R	Right- 70D- 16" W- LONG
1F,2E,3-6,7C,8	107778	COMP BNT LONG EXT ASM 17" W R	Right- 70D- 17" W- LONG
1F,2F,3-6,7C,8	107779	COMP BNT LONG EXT ASM 18" W R	Right- 70D- 18" W- LONG
1F,2G,3-6,7D,8	107780	COMP BNT LONG EXT ASM 19 & 20"W R	Right- 70D- 19 & 20" W- LONG
1F,2H,3-6,7D,8	111532	COMP BNT LONG EXT ASM 21 & 22"W R	Right- 70D- 21 & 22" W- LONG
>>>>		60°,70°,70° HEMI HANGERS - LONG EXT - LEFT	
1F,2A,3-6,7A,8	107781	COMP BNT LONG EXT ASM 13" W L	Left- 70D- 13 W- LONG
1F,2B,3-6,7A,8	107782	COMP BNT LONG EXT ASM 14" W L	Left- 70D- 14 W- LONG
1F,2C,3-6,7B,8	107783	COMP BNT LONG EXT ASM 15" W L	Left- 70D- 15 W- LONG
1F,2D,3-6,7B,8	107784	COMP BNT LONG EXT ASM 16" W L	Left- 70D- 16 W- LONG
1F,2E,3-6,7C,8	107785	COMP BNT LONG EXT ASM 17" W L	Left- 70D- 17 W- LONG
1F,2F,3-6,7C,8	107786	COMP BNT LONG EXT ASM 18" W L	Left- 70D- 18 W- LONG
1F,2G,3-6,7D,8	107787	COMP BNT LONG EXT ASM 19 & 20"W L	Left- 70D- 19 & 20" W- LONG
1F,2H,3-6,7D,8	111533	COMP BNT LONG EXT ASM 21 & 22"W L	Left- 70D- 21 & 22" W- LONG
>>>>		70 DEGREE "V" HANGERS - MED EXT - RIGHT	
1G,2A,3-8	107788	COMP V MED EXT ASM 13W R	Right- 70D V hangers only 13 W- MED
1G,2B,3-8	107789	COMP V MED EXT ASM 14W R	Right- 70D V hangers only 14 W- MED
1G,2C,3-8	107790	COMP V MED EXT ASM 15W R	Right- 70D V hangers only 15 W- MED
1G,2D,3-8	107791	COMP V MED EXT ASM 16W R	Right- 70D V hangers only 16 W- MED
1G,2E,3-8	107792	COMP V MED EXT ASM 17W R	Right- 70D V hangers only 17 W- MED
1G,2F,3-8	107793	COMP V MED EXT ASM 18W R	Right- 70D V hangers only 18 W- MED
>>>>		70 DEGREE "V" HANGERS - MED EXT - LEFT	
1G,2A,3-8	107795	COMP V MED EXT ASM 13W L	Left- 70D V hangers only 13 W- MED
1G,2B,3-8	107796	COMP V MED EXT ASM 14W L	Left- 70D V hangers only 14 W- MED
1G,2C,3-8	107797	COMP V MED EXT ASM 15W L	Left- 70D V hangers only 15 W- MED
1G,2D,3-8	107798	COMP V MED EXT ASM 16W L	Left- 70D V hangers only 16 W- MED
1G,2E,3-8	107799	COMP V MED EXT ASM 17W L	Left- 70D V hangers only 17 W- MED
1G,2F,3-8	107800	COMP V MED EXT ASM 18W L	Left- 70D V hangers only 18 W- MED
>>>>		70 DEGREE "V" HANGERS - LONG EXT - RIGHT	
1H,2A,3-8	107802	COMP V LNG EXT ASM 13W R	Right- 70D V hangers only 13 W- LONG
1H,2B,3-8	107803	COMP V LNG EXT ASM 14W R	Right- 70D V hangers only 14 W- LONG
1H,2C,3-8	107804	COMP V LNG EXT ASM 15W R	Right- 70D V hangers only 15 W- LONG
1H,2D,3-8	107805	COMP V LNG EXT ASM 16W R	Right- 70D V hangers only 16 W- LONG
1H,2E,3-8	107806	COMP V LNG EXT ASM 17W R	Right- 70D V hangers only 17 W- LONG
1H,2F,3-8	107807	COMP V LNG EXT ASM 18W R	Right- 70D V hangers only 18 W- LONG
>>>>		70 DEGREE "V" HANGERS - LONG EXT - LEFT	
1H,2A,3-8	107809	COMP V LNG EXT ASM 13W L	Left- 70D V hangers only 13 W- LONG
1H,2B,3-8	107810	COMP V LNG EXT ASM 14W L	Left- 70D V hangers only 14 W- LONG
1H,2C,3-8	107811	COMP V LNG EXT ASM 15W L	Left- 70D V hangers only 15 W- LONG
1H,2D,3-8	107812	COMP V LNG EXT ASM 16W L	Left- 70D V hangers only 16 W- LONG
1H,2E,3-8	107813	COMP V LNG EXT ASM 17W L	Left- 70D V hangers only 17 W- LONG
1H,2F,3-8	107814	COMP V LNG EXT ASM 18W L	Left- 70D V hangers only 18 W- LONG
>>>>		80 DEGREE HANGERS - SHORT EXT - RIGHT	
1A,2A,3-6,7A,8	107704	COMP STRT SHT EXT ASM 13"W R	Right- 80D- 13" W- SHORT
1A,2B,3-6,7A,8	107705	COMP STRT SHT EXT ASM 14"W R	Right- 80D- 14" W- SHORT
1A,2C,3-6,7B,8	107706	COMP STRT SHT EXT ASM 15"W R	Right- 80D- 15" W- SHORT
1A,2D,3-6,7B,8	107707	COMP STRT SHT EXT ASM 16"W R	Right- 80D- 16" W- SHORT
1A,2E,3-6,7C,8	107708	COMP STRT SHT EXT ASM 17"W R	Right- 80D- 17" W- SHORT

Pos.	Item Number	Description	Remarks
1A,2F,3-6,7C,8	107709	COMP STRT SHT EXT ASM 18"W R	Right- 80D- 18" W- SHORT
1A,2G,3-6,7D,8	107710	COMP STRT SHT EXT ASM 19 & 20"W R	Right- 80D- 19 & 20" W- SHORT
1A,2H,3-6,7D,8	111522	COMP STRT SHT EXT ASM 21 & 22"W R	Right- 80D- 21 & 22" W- SHORT
>>>>		80 DEGREE HANGERS - SHORT EXT - LEFT	
1A,2A,3-6,7A,8	107711	COMP STRT SHT EXT ASM 13"W L	LEFT- 80D- 13" W- SHORT
1A,2B,3-6,7A,8	107712	COMP STRT SHT EXT ASM 14"W L	LEFT- 80D- 14" W- SHORT
1A,2C,3-6,7B,8	107713	COMP STRT SHT EXT ASM 15"W L	LEFT- 80D- 15" W- SHORT
1A,2D,3-6,7B,8	107714	COMP STRT SHT EXT ASM 16"W L	LEFT- 80D- 16" W- SHORT
1A,2E,3-6,7C,8	107715	COMP STRT SHT EXT ASM 17"W L	LEFT- 80D- 17" W- SHORT
1A,2F,3-6,7C,8	107716	COMP STRT SHT EXT ASM 18"W L	LEFT- 80D- 18" W- SHORT
1A,2G,3-6,7D,8	107717	COMP STRT SHT EXT ASM 19 & 20"W L	LEFT- 80D- 19 & 20" W- SHORT
1A,2H,3-6,7D,8	111523	COMP STRT SHT EXT ASM 21 & 22"W L	LEFT- 80D- 21 & 22" W- SHORT
>>>>		80 DEGREE HANGERS - MED EXT - RIGHT	
1B,2A,3-6,7A,8	107718	COMP STRT MED EXT ASM 13"W R	Right- 80D- 13" W- MED
1B,2B,3-6,7A,8	107719	COMP STRT MED EXT ASM 14"W R	Right- 80D- 14" W- MED
1B,2C,3-6,7B,8	107720	COMP STRT MED EXT ASM 15"W R	Right- 80D- 15" W- MED
1B,2D,3-6,7B,8	107721	COMP STRT MED EXT ASM 16"W R	Right- 80D- 16" W- MED
1B,2E,3-6,7C,8	107722	COMP STRT MED EXT ASM 17"W R	Right- 80D- 17" W- MED
1B,2F,3-6,7C,8	107723	COMP STRT MED EXT ASM 18"W R	Right- 80D- 18" W- MED
1B,2G,3-6,7D,8	107724	COMP STRT MED EXT ASM 19 & 20"W R	Right- 80D- 19 & 20" W- MED
1B,2H,3-6,7D,8	111524	COMP STRT MED EXT ASM 21 & 22"W R	Right- 80D- 21 & 22" W- MED
>>>>		80 DEGREE HANGERS - MED EXT - LEFT	
1B,2A,3-6,7A,8	107725	COMP STRT MED EXT ASM 13"W L	LEFT- 80D- 13" W- MED
1B,2B,3-6,7A,8	107726	COMP STRT MED EXT ASM 14"W L	LEFT- 80D- 14" W- MED
1B,2C,3-6,7B,8	107727	COMP STRT MED EXT ASM 15"W L	LEFT- 80D- 15" W- MED
1B,2D,3-6,7B,8	107728	COMP STRT MED EXT ASM 16"W L	LEFT- 80D- 16" W- MED
1B,2E,3-6,7C,8	107729	COMP STRT MED EXT ASM 17"W L	LEFT- 80D- 17" W- MED
1B,2F,3-6,7C,8	107730	COMP STRT MED EXT ASM 18"W L	LEFT- 80D- 18" W- MED
1B,2G,3-6,7D,8	107731	COMP STRT MED EXT ASM 19 & 20"W L	LEFT- 80D- 19 & 20" W- MED
1B,2H,3-6,7D,8	111525	COMP STRT MED EXT ASM 21 & 22"W L	LEFT- 80D- 21 & 22" W- MED
>>>>		80 DEGREE HANGERS - LONG EXT - RIGHT	
1C,2A,3-6,7A,8	107732	COMP STRT LONG EXT ASM 13"W R	Right- 80D- 13" W- LONG
1C,2B,3-6,7A,8	107733	COMP STRT LONG EXT ASM 14"W R	Right- 80D- 14" W- LONG
1C,2C,3-6,7B,8	107734	COMP STRT LONG EXT ASM 15"W R	Right- 80D- 15" W- LONG
1C,2D,3-6,7B,8	107735	COMP STRT LONG EXT ASM 16"W R	Right- 80D- 16" W- LONG
1C,2E,3-6,7C,8	107736	COMP STRT LONG EXT ASM 17"W R	Right- 80D- 17" W- LONG
1C,2F,3-6,7C,8	107737	COMP STRT LONG EXT ASM 18"W R	Right- 80D- 18" W- LONG
1C,2G,3-6,7D,8	107738	COMP STRT LONG EXT ASM 19 & 20"W R	Right- 80D- 19 & 20" W- LONG
1C,2H,3-6,7D,8	111526	COMP STRT LONG EXT ASM 21 & 22"W R	Right- 80D- 21 & 22" W- LONG
>>>>		80 DEGREE HANGERS - LONG EXT - LEFT	
1C,2A,3-6,7A,8	107739	COMP STRT LONG EXT ASM 13"W L	LEFT- 80D- 13" W- LONG
1C,2B,3-6,7A,8	107740	COMP STRT LONG EXT ASM 14"W L	LEFT- 80D- 14" W- LONG
1C,2C,3-6,7B,8	107741	COMP STRT LONG EXT ASM 15"W L	LEFT- 80D- 15" W- LONG
1C,2D,3-6,7B,8	107742	COMP STRT LONG EXT ASM 16"W L	LEFT- 80D- 16" W- LONG
1C,2E,3-6,7C,8	107743	COMP STRT LONG EXT ASM 17"W L	LEFT- 80D- 17" W- LONG
1C,2F,3-6,7C,8	107744	COMP STRT LONG EXT ASM 18"W L	LEFT- 80D- 18" W- LONG
1C,2G,3-6,7D,8	107745	COMP STRT LONG EXT ASM 19 & 20"W L	LEFT- 80D- 19 & 20" W- LONG
1C,2H,3-6,7D,8	111527	COMP STRT LONG EXT ASM 21 & 22"W L	LEFT- 80D- 21 & 22" W- LONG

NOTE: FOOTPLATE ASSEMBLIES ARE SET TO FOOTREST WIDTH. PLEASE SEE "FOOTPLATE FORMULA" TABLE FOR SEAT WIDTH CONVERSION.

INSET FORMULA


0" Inset (Seat Width) = Footrest Width
 1" Inset (Seat Width) - 2 = Footrest Width
 "V" Hanger (Seat Width) - 4 = Footrest Width

Composite Angle Adjustable "CAA" Footplate w/ Ext & HL for Swing I/O Hangers


(Order 1 each)

Ext. Tube Width	60°, 70° & 70° Hemi Style Hangers (Bent Extension)			80° Style hanger (Straight Extension)			70° V Style Hangers	
	Short	Medium	Long	Short	Medium	Long	Medium	Long
15" Right	107858	107872	107886	107816	107830	107844	107900	107914
15" Left	107865	107879	107893	107823	107837	107851	107907	107921
16" Right	107859	107873	107887	107817	107831	107845	107901	107915
16" Left	107866	107880	107894	107824	107838	107852	107908	107922
17" Right	107860	107874	107888	107818	107832	107846	107902	107916
17" Left	107867	107881	107895	107825	107839	107853	107909	107923
18" Right	107861	107875	107889	107819	107833	107847	107903	107917
18" Left	107868	107882	107896	107826	107840	107854	107910	107924
19" Right	107862	107876	107890	107820	107834	107848	V STYLE IS NOT	
19" Left	107869	107883	107897	107827	107841	107855		
20" Right	107863	107877	107891	107821	107835	107849		
20" Left	107870	107884	107898	107828	107842	107856		
21"-22" Rt	107864	107878	107892	107822	107836	107850		
21"-22" Lt	107871	107885	107899	107829	107843	107857		

FOOTPLATE MEASURING GUIDE

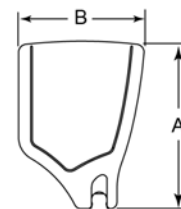

PART NO	DESCRIPTION	DIM -A-	DIM -B-
063216-006	FOOTPLATE COMPOSITE 157 R	6 1/8"	6"
063216-012	FOOTPLATE COMPOSITE 157 L	6 1/8"	6"
063216-015	FOOTPLATE COMPOSITE 170 R	6 5/8"	6"
063216-018	FOOTPLATE COMPOSITE 170 L	6 5/8"	6"
063216-005	FOOTPLATE COMPOSITE 182 R	7 1/8"	6"
063216-011	FOOTPLATE COMPOSITE 182 L	7 1/8"	6"
063216-014	FOOTPLATE COMPOSITE 195 R	7 5/8"	6"
063216-017	FOOTPLATE COMPOSITE 195 L	7 5/8"	6"
063216-004	FOOTPLATE COMPOSITE 207 R	8 1/8"	6"
063216-010	FOOTPLATE COMPOSITE 207 L	8 1/8"	6"
063216-013	FOOTPLATE COMPOSITE 220 R	8 5/8"	6 1/4"
063216-016	FOOTPLATE COMPOSITE 220 L	8 5/8"	6 1/4"
063216-003	FOOTPLATE COMPOSITE 232 R	9 1/8"	6 1/4"
063216-009	FOOTPLATE COMPOSITE 232 L	9 1/8"	6 1/4"
063216-002	FOOTPLATE COMPOSITE 257 R	10 1/8"	6 1/4"
063216-008	FOOTPLATE COMPOSITE 257 L	10 1/8"	6 1/4"
063216-001	FOOTPLATE COMPOSITE 282 R	11 1/8"	6 1/4"
063216-007	FOOTPLATE COMPOSITE 282 L	11 1/8"	6 1/4"

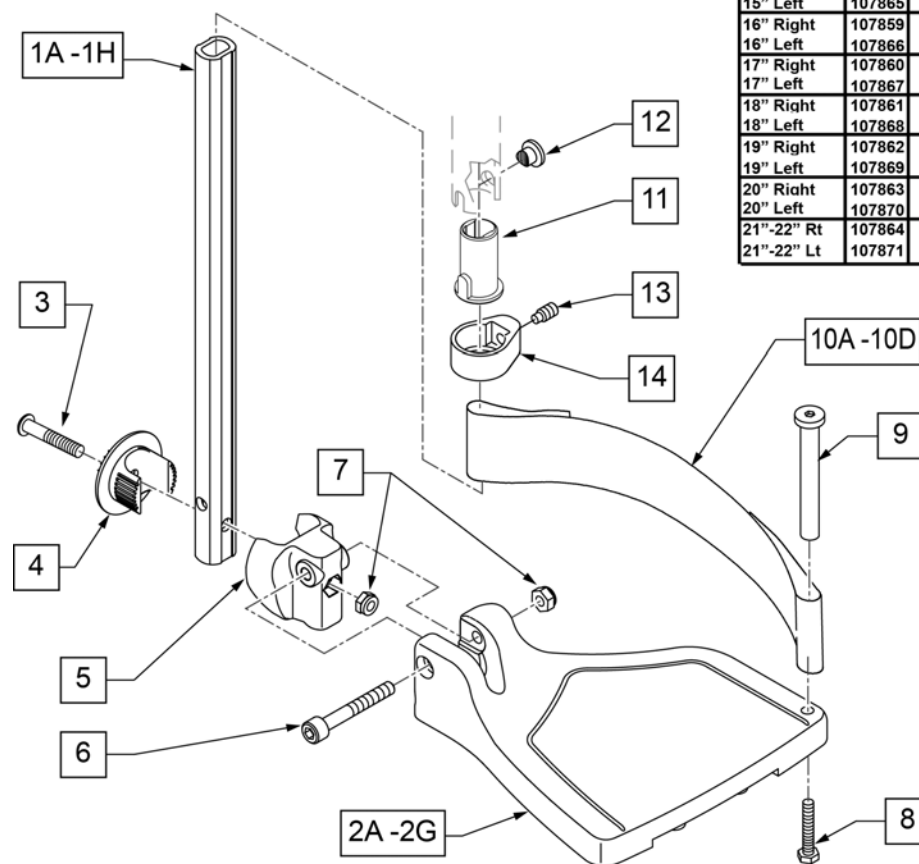

Pos.	Item Number	Description	Remarks
1A,6,7	120660	SHT STRT EXT COMP HDWR	80 deg hangers only
1B,6,7	120661	MED STRT EXT COMP HDWR	80 deg hangers only
1C,6,7	120662	LNG STRT EXT COMP HDWR	80 deg hangers only
1D,6,7	120663	SHT BNT EXT COMP HDWR	60,70,70 Hemi Hangers
1E,6,7	120664	MED BNT EXT COMP HDWR	60,70,70 Hemi Hangers
1F,6,7	120665	LNG BNT EXT COMP HDWR	60,70,70 Hemi Hangers
1G,6,7	120666	SHT V EXT R COMP HDWR	70 deg V hangers only-LEFT
1G,6,7	120667	SHT V EXT L COMP HDWR	70 deg V hangers only-RIGHT
1H,6,7	120668	LNG V EXT R COMP HDWR	70 deg V hangers only-LEFT
1H,6,7	120669	LNG V EXT L COMP HDWR	70 deg V hangers only-RIGHT
2A	063216-006	COMPOSITE FOOTPLATE RIGHT	RIGHT 15" Footrest Width
2A	063216-012	COMPOSITE FOOTPLATE LEFT	LEFT 15" Footrest Width
2B	063216-015	COMPOSITE FOOTPLATE RIGHT	RIGHT 16" Footrest Width

Pos.	Item Number	Description	Remarks
2B	063216-018	COMPOSITE FOOTPLATE LEFT	LEFT 16" Footrest Width
2C	063216-005	COMPOSITE FOOTPLATE RIGHT	RIGHT 17" Footrest Width
2C	063216-011	COMPOSITE FOOTPLATE LEFT	LEFT 17" Footrest Width
2D	063216-014	COMPOSITE FOOTPLATE RIGHT	RIGHT 18" Footrest Width
2D	063216-017	COMPOSITE FOOTPLATE LEFT	LEFT 18" Footrest Width
2E	063216-004	COMPOSITE FOOTPLATE RIGHT	RIGHT 19" Footrest Width
2E	063216-010	COMPOSITE FOOTPLATE LEFT	LEFT 19" Footrest Width
2F	063216-013	COMPOSITE FOOTPLATE RIGHT	RIGHT 20" Footrest Width
2F	063216-016	COMPOSITE FOOTPLATE LEFT	LEFT 20" Footrest Width
2G	063216-003	COMPOSITE FOOTPLATE RIGHT	RIGHT 21" & 22" Footrest Width
2G	063216-009	COMPOSITE FOOTPLATE LEFT	LEFT 21" & 22" Footrest Width
3	108233	BHCS M6 X 1 X 40	
4	054242	ANGLE COVER	
5	054241	ANGLE SEGMENT	
6	103175	SHCS M6 X 1 X 40 INORG	
7	104381	NUT HEX NYLOCK M6X1.0	
8	102268-001	HHCS M6X1X25	Inserted in bottom of footplate.
9	104328	HEEL LOOP PEG	
10A	104380-001	ADJUSTABLE FOOT REST STRAP	15" Footrest Width
10B	104380-002	ADJUSTABLE FOOT REST STRAP	16" Footrest Width
10C	104380-003	ADJUSTABLE FOOT REST STRAP	17-18" Footrest Width
10D	104380-004	ADJUSTABLE FOOT REST STRAP	19-22" Footrest Width
11	104160	BUSHING FOOTREST	
11-14	107485	FOOTREST CLAMP ASSM	
>>>>		60°,70°,70° HEMI HANGERS - SHORT EXT - RIGHT	
1D,2A,3-10A	107858	CAA BNT SHT EXT ASM 15W R	Right- 60° & 60° & 70° 15 W- SHORT
1D,2B,3-10B	107859	CAA BNT SHT EXT ASM 16W R	Right- 60° & 70° 16 W- SHORT
1D,2C,3-10B	107860	CAA BNT SHT EXT ASM 17W R	Right- 60° & 70° 17 W- SHORT
1D,2D,3-10C	107861	CAA BNT SHT EXT ASM 18W R	Right- 60° & 70° 18 W- SHORT
1D,2E,3-10C	107862	CAA BNT SHT EXT ASM 19W R	Right- 60° & 70° 19 W- SHORT
1D,2F,3-10D	107863	CAA BNT SHT EXT ASM 20W R	Right- 60° & 70° 20 W- SHORT
1D,2G,3-10D	107864	CAA BNT SHT EXT ASM 21&22W R	Right- 60° & 70° 21&22 W- SHORT
>>>>		60°,70°,70° HEMI HANGERS - SHORT EXT - LEFT	
1D,2A,3-10A	107865	CAA BNT SHT EXT ASM 15W L	Left- 60° & 70° 15 W- SHORT
1D,2B,3-10B	107866	CAA BNT SHT EXT ASM 16W L	Left- 60° & 70° 16 W- SHORT
1D,2C,3-10B	107867	CAA BNT SHT EXT ASM 17W L	Left- 60° & 70° 17 W- SHORT
1D,2D,3-10C	107868	CAA BNT SHT EXT ASM 18W L	Left- 60° & 70° 18 W- SHORT
1D,2E,3-10C	107869	CAA BNT SHT EXT ASM 19W L	Left- 60° & 70° 19 W- SHORT
1D,2F,3-10D	107870	CAA BNT SHT EXT ASM 20W L	Left- 60° & 70° 20 W- SHORT
1D,2G,3-10D	107871	CAA BNT SHT EXT ASM 21&22W L	Left- 60° & 70° 21&22 W- SHORT
>>>>		60°,70°,70° HEMI HANGERS - MED EXT - RIGHT	
1E,2A,3-10A	107872	CAA BNT MED EXT ASM 15W R	Right- 60° & 70° 15 W- MED
1E,2B,3-10B	107873	CAA BNT MED EXT ASM 16W R	Right- 60° & 70° 16 W- MED
1E,2C,3-10B	107874	CAA BNT MED EXT ASM 17W R	Right- 60° & 70° 17 W- MED
1E,2D,3-10C	107875	CAA BNT MED EXT ASM 18W R	Right- 60° & 70° 18 W- MED
1E,2E,3-10C	107876	CAA BNT MED EXT ASM 19W R	Right- 60° & 70° 19 W- MED
1E,2F,3-10D	107877	CAA BNT MED EXT ASM 20W R	Right- 60° & 70° 20 W- MED
1E,2G,3-10D	107878	CAA BNT MED EXT ASM 21&22W R	Right- 60° & 70° 21&22 W- MED
>>>>		60°,70°,70° HEMI HANGERS - MED EXT - LEFT	
1E,2A,3-10A	107879	CAA BNT MED EXT ASM 15W L	Left- 60° & 70° 15 W- MED
1E,2B,3-10B	107880	CAA BNT MED EXT ASM 16W L	Left- 60° & 70° 16 W- MED
1E,2C,3-10B	107881	CAA BNT MED EXT ASM 17W L	Left- 60° & 70° 17 W- MED
1E,2D,3-10C	107882	CAA BNT MED EXT ASM 18W L	Left- 60° & 70° 18 W- MED
1E,2E,3-10C	107883	CAA BNT MED EXT ASM 19W L	Left- 60° & 70° 19 W- MED
1E,2F,3-10D	107884	CAA BNT MED EXT ASM 20W L	Left- 60° & 70° 20 W- MED
1E,2G,3-10D	107885	CAA BNT MED EXT ASM 21&22W L	Left- 60° & 70° 21&22 W- MED

Pos.	Item Number	Description	Remarks
>>>>		60°,70°,70° HEMI HANGERS - LONG EXT - RIGHT	
1F,2A,3-10A	107886	CAA BNT LNG EXT ASM 15W R	Right- 60° & 70° 15 W- LONG
1F,2B,3-10B	107887	CAA BNT LNG EXT ASM 16W R	Right- 60° & 70° 16 W- LONG
1F,2C,3-10B	107888	CAA BNT LNG EXT ASM 17W R	Right- 60° & 70° 17 W- LONG
1F,2D,3-10C	107889	CAA BNT LNG EXT ASM 18W R	Right- 60° & 70° 18 W- LONG
1F,2E,3-10C	107890	CAA BNT LNG EXT ASM 19W R	Right- 60° & 70° 19 W- LONG
1F,2F,3-10D	107891	CAA BNT LNG EXT ASM 20W R	Right- 60° & 70° 20 W- LONG
1F,2G,3-10D	107892	CAA BNT LNG EXT ASM 21&22W R	Right- 60° & 70° 21&22 W- LONG
>>>>		60°,70°,70° HEMI HANGERS - LONG EXT - LEFT	
1F,2A,3-10A	107893	CAA BNT LNG EXT ASM 15W L	Left- 60° & 70° 15 W- LONG
1F,2B,3-10B	107894	CAA BNT LNG EXT ASM 16W L	Left- 60° & 70° 16 W- LONG
1F,2C,3-10B	107895	CAA BNT LNG EXT ASM 17W L	Left- 60° & 70° 17 W- LONG
1F,2D,3-10C	107896	CAA BNT LNG EXT ASM 18W L	Left- 60° & 70° 18 W- LONG
1F,2E,3-10C	107897	CAA BNT LNG EXT ASM 19W L	Left- 60° & 70° 19 W- LONG
1F,2F,3-10D	107898	CAA BNT LNG EXT ASM 20W L	Left- 60° & 70° 20 W- LONG
1F,2G,3-10D	107899	CAA BNT LNG EXT ASM 21&22W L	Left- 60° & 70° 21&22 W- LONG
>>>>		70° "V" HANGERS - MED EXT - RIGHT	
1G,2A,3-10A	107900	CAA V MED EXT ASM 15W R	Right- 70° V hangers only 15 W- MED
1G,2B,3-10B	107901	CAA V MED EXT ASM 16W R	Right- 70° V hangers only 16 W- MED
1G,2C,3-10B	107902	CAA V MED EXT ASM 17W R	Right- 70° V hangers only 17 W- MED
1G,2D,3-10C	107903	CAA V MED EXT ASM 18W R	Right- 70° V hangers only 18 W- MED
>>>>		70° "V" HANGERS - MED EXT - LEFT	
1G,2A,3-10A	107907	CAA V MED EXT ASM 15W L	Left- 70° V hangers only 15 W- MED
1G,2B,3-10B	107908	CAA V MED EXT ASM 16W L	Left- 70° V hangers only 16 W- MED
1G,2C,3-10B	107909	CAA V MED EXT ASM 17W L	Left- 70° V hangers only 17 W- MED
1G,2D,3-10C	107910	CAA V MED EXT ASM 18W L	Left- 70° V hangers only 18 W- MED
>>>>		70° "V" HANGERS - LONG EXT - RIGHT	
1H,2A,3-10A	107914	CAA V LNG EXT ASM 15W R	Right- 70° V hangers only 15 W- LONG
1H,2B,3-10B	107915	CAA V LNG EXT ASM 16W R	Right- 70° V hangers only 16 W- LONG
1H,2C,3-10B	107916	CAA V LNG EXT ASM 17W R	Right- 70° V hangers only 17 W- LONG
1H,2D,3-10C	107917	CAA V LNG EXT ASM 18W R	Right- 70° V hangers only 18 W- LONG
>>>>		70° "V" HANGERS - LONG EXT - LEFT	
1H,2A,3-10A	107921	CAA V LNG EXT ASM 15W L	Left- 70° V hangers only 15 W- LONG
1H,2B,3-10B	107922	CAA V LNG EXT ASM 16W L	Left- 70° V hangers only 16 W- LONG
1H,2C,3-10B	107923	CAA V LNG EXT ASM 17W L	Left- 70° V hangers only 17 W- LONG
1H,2D,3-10C	107924	CAA V LNG EXT ASM 18W L	Left- 70° V hangers only 18 W- LONG
>>>>		80° HANGERS - SHORT EXT - RIGHT	
1A,2A,3-10A	107816	CAA STRT SHT EXT ASM 15W R	Right- 80° 15 W- SHORT
1A,2B,3-10B	107817	CAA STRT SHT EXT ASM 16W R	Right- 80° 16 W- SHORT
1A,2C,3-10B	107818	CAA STRT SHT EXT ASM 17W R	Right- 80° 17 W- SHORT
1A,2D,3-10C	107819	CAA STRT SHT EXT ASM 18W R	Right- 80° 18 W- SHORT
1A,2E,3-10C	107820	CAA STRT SHT EXT ASM 19W R	Right- 80° 19 W- SHORT
1A,2F,3-10D	107821	CAA STRT SHT EXT ASM 20W R	Right- 80° 20 W- SHORT
1A,2G,3-10D	107822	CAA STRT SHT EXT ASM 21&22W R	Right- 80° 21&22 W- SHORT
>>>>		80° HANGERS - SHORT EXT - LEFT	
1A,2A,3-10A	107823	CAA STRT SHT EXT ASM 15W L	Left- 80° 15 W- SHORT
1A,2B,3-10B	107824	CAA STRT SHT EXT ASM 16W L	Left- 80° 16 W- SHORT
1A,2C,3-10B	107825	CAA STRT SHT EXT ASM 17W L	Left- 80° 17 W- SHORT
1A,2D,3-10C	107826	CAA STRT SHT EXT ASM 18W L	Left- 80° 18 W- SHORT
1A,2E,3-10C	107827	CAA STRT SHT EXT ASM 19W L	Left- 80° 19 W- SHORT
1A,2F,3-10D	107828	CAA STRT SHT EXT ASM 20W L	Left- 80° 20 W- SHORT
1A,2G,3-10D	107829	CAA STRT SHT EXT ASM 21&22W L	Left- 80° 21&22 W- SHORT
>>>>		80° HANGERS - MED EXT - RIGHT	
1B,2A,3-10A	107830	CAA STRT MED EXT ASM 15W R	Right- 80° 15 W- MED
1B,2B,3-10B	107831	CAA STRT MED EXT ASM 16W R	Right- 80° 16 W- MED
1B,2C,3-10B	107832	CAA STRT MED EXT ASM 17W R	Right- 80° 17 W- MED


Pos.	Item Number	Description	Remarks
1B,2D,3-10C	107833	CAA STRT MED EXT ASM 18W R	Right- 80° 18 W- MED
1B,2E,3-10C	107834	CAA STRT MED EXT ASM 19W R	Right- 80° 19 W- MED
1B,2F,3-10D	107835	CAA STRT MED EXT ASM 20W R	Right- 80° 20 W- MED
1B,2G,3-10D	107836	CAA STRT MED EXT ASM 21&22W R	Right- 80° 21&22 W- MED
>>>>		80° HANGERS - MED EXT - LEFT	
1B,2A,3-10A	107837	CAA STRT MED EXT ASM 15W L	Left- 80° 15 W- MED
1B,2B,3-10B	107838	CAA STRT MED EXT ASM 16W L	Left- 80° 16 W- MED
1B,2C,3-10B	107839	CAA STRT MED EXT ASM 17W L	Left- 80° 17 W- MED
1B,2D,3-10C	107840	CAA STRT MED EXT ASM 18W L	Left- 80° 18 W- MED
1B,2E,3-10C	107841	CAA STRT MED EXT ASM 19W L	Left- 80° 19 W- MED
1B,2F,3-10D	107842	CAA STRT MED EXT ASM 20W L	Left- 80° 20 W- MED
1B,2G,3-10D	107843	CAA STRT MED EXT ASM 21&22W L	Left- 80° 21&22 W- MED
>>>>		80° HANGERS - LONG EXT - RIGHT	
1C,2A,3-10A	107844	CAA STRT LNG EXT ASM 15W R	Right- 80° 15 W- LONG
1C,2B,3-10B	107845	CAA STRT LNG EXT ASM 16W R	Right- 80° 16 W- LONG
1C,2C,3-10B	107846	CAA STRT LNG EXT ASM 17W R	Right- 80° 17 W- LONG
1C,2D,3-10C	107847	CAA STRT LNG EXT ASM 18W R	Right- 80° 18 W- LONG
1C,2E,3-10C	107848	CAA STRT LNG EXT ASM 19W R	Right- 80° 19 W- LONG
1C,2F,3-10D	107849	CAA STRT LNG EXT ASM 20W R	Right- 80° 20 W- LONG
1C,2G,3-10D	107850	CAA STRT LNG EXT ASM 21&22W R	Right- 80° 21&22 W- LONG
>>>>		80° HANGERS - LONG EXT - LEFT	
1C,2A,3-10A	107851	CAA STRT LNG EXT ASM 15W L	Left- 80° 15 W- LONG
1C,2B,3-10B	107852	CAA STRT LNG EXT ASM 16W L	Left- 80° 16 W- LONG
1C,2C,3-10B	107853	CAA STRT LNG EXT ASM 17W L	Left- 80° 17 W- LONG
1C,2D,3-10C	107854	CAA STRT LNG EXT ASM 18W L	Left- 80° 18 W- LONG
1C,2E,3-10C	107855	CAA STRT LNG EXT ASM 19W L	Left- 80° 19 W- LONG
1C,2F,3-10D	107856	CAA STRT LNG EXT ASM 20W L	Left- 80° 20 W- LONG
1C,2G,3-10D	107857	CAA STRT LNG EXT ASM 21&22W L	Left- 80° 21&22 W- LONG

NOTE: FOOTPLATE ASSEMBLIES ARE SET TO FOOTREST WIDTH. PLEASE SEE "FOOTPLATE FORMULA" TABLE FOR SEAT WIDTH CONVERSION.


Adult Angle Adjustable "AAA" Footplates w/ Ext & HL for Swing I/O Hangers								
(Order 1 each)								
Ext. Tube	60°, 70° & 70° Hemi Style Hangers (Bent Extension)			80° Style hanger (Straight Extension)			70° V Style Hangers	
Width	Short	Med	Long	Short	Med	Long	Med	Long
10"-11" Rt	NOT AVAILABLE IN THIS WIDTH			NOT AVAILABLE IN THIS WIDTH			111546	111550
10"-11" Lt	NOT AVAILABLE IN THIS WIDTH			NOT AVAILABLE IN THIS WIDTH			111547	111551
12-13" Rt	111540	111542	111544	111534	111536	111538	111548	111552
12-13" Lt	111541	111543	111545	111535	111537	111539	111549	111553
14"-15" Rt	107973	107987	108001	107928	107942	107956	108015	108029
14"-15" Lt	107980	107994	108008	107935	107949	107963	108022	108036
16"-17" Rt	107975	107989	108003	107930	107944	107958	108017	108031
16"-17" Lt	107982	107996	108010	107937	107951	107965	108024	108038
18"-22" Rt	107977	107991	108005	107932	107946	107960	108019	108033
18"-22" Lt	107984	107998	108012	107939	107953	107968	108026	108040

INSET FORMULA
 0" Inset (Seat Width) = Footrest Width
 1" Inset (Seat Width) - 2 = Footrest Width
 "V" Hanger (Seat Width) - 4 = Footrest Width

MEASURING GUIDE

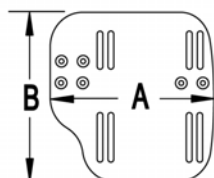


CHAIR WIDTH	"A"	"B"
10" - 11"	3 1/2"	7 1/4"
12" - 13"	4 1/2"	7 3/4"
14" - 15"	5 1/2"	7 3/4"
16" - 17"	6 1/2"	7 3/4"
18" - 20"	7 1/2"	7 3/4"

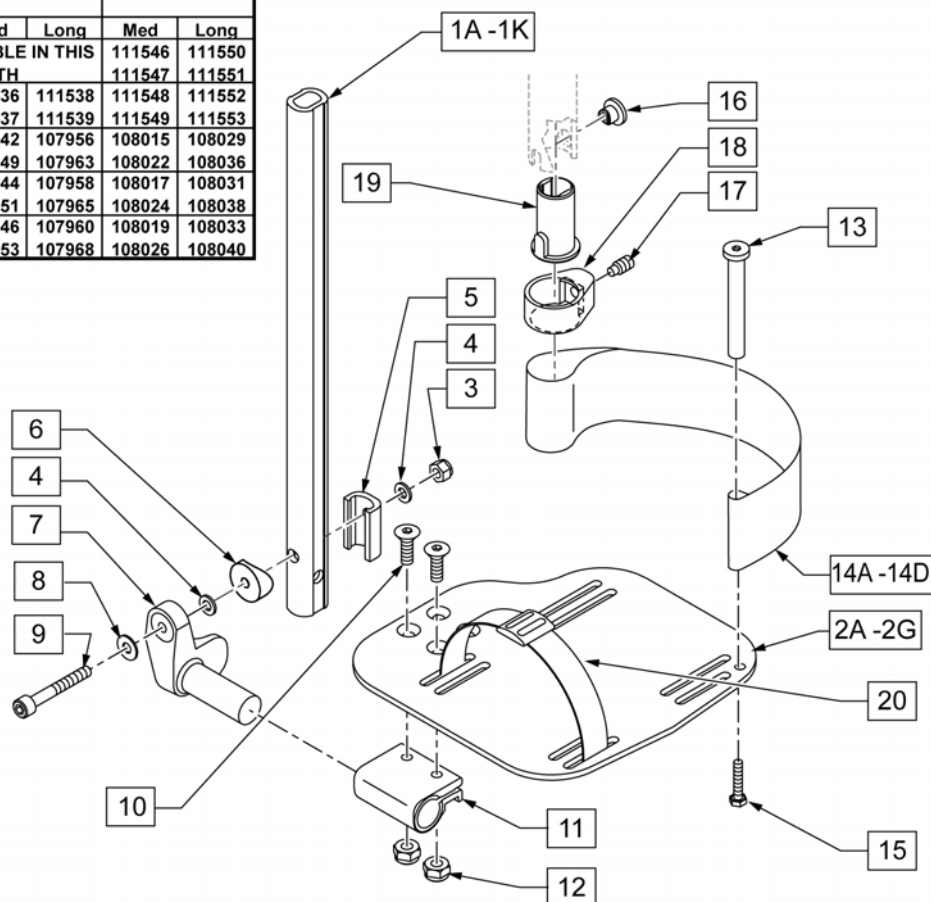


Pos.	Item Number	Description	Remarks
1A,3-5,8,9	120670	SHT STRT EXT AAA HDWR	80 deg Hangers Only
1B,3-5,8,9	120671	MED STRT EXT AAA HDWR	80 deg Hangers Only
1C,3-5,8,9	120672	LNG STRT EXT AAA HDWR	80 deg Hangers Only
1D,3-5,8,9	120673	SHT BNT EXT AAA HDWR	60,70,70 Hemi Hangers
1E,3-5,8,9	120674	MED BNT EXT AAA HDWR	60,70,70 Hemi Hangers
1F,3-5,8,9	120675	LNG BNT EXT AAA HDWR	60,70,70 Hemi Hangers
1G,3-5,8,9	120676	SHT V EXT R AAA HDWR	Left - 70 deg V Hangers Only
1H,3-5,8,9	120677	SHT V EXT L AAA HDWR	Right - 70 deg V Hangers Only
1I,3-5,8,9	120678	LNG V EXT R AAA HDWR	Left - 70 deg V Hangers Only
1J,3-5,8,9	120679	LNG V EXT L AAA HDWR	Right - 70 deg V Hangers Only
2A	500656	ADULT ANG ADJ F/PLT 10/11" R	RIGHT 10-11" Footrest Width
2A	500657	ADULT ANG ADJ F/PLT 10/11" L	LEFT 10-11" Footrest Width
2B	500658	ADULT ANG ADJ F/PLT 12/13" R	RIGHT 12-13" Footrest Width

Pos.	Item Number	Description	Remarks
2B	500659	ADULT ANG ADJ F/PLT 12/13" L	LEFT 12-13" Footrest Width
2C	500660	ADLT ANGL ADJ F/PLT 14/15	RIGHT 14-15" Footrest Width
2C	500661	ADLT ANGL ADJ F/PLT 14/15	LEFT 14-15" Footrest Width
2D	500662	ADLT ANGL ADJ F/PLT 16/17	RIGHT 16-17" Footrest Width
2D	500663	ADLT ANGL ADJ F/PLT 16/17	LEFT 16-17" Footrest Width
2E	500664	ADLT ANGL ADJ F/PLT 18-20	RIGHT 18-22" Footrest Width
2E	500665	ADLT ANGL ADJ F/PLT 18-20	LEFT 18-22" Footrest Width
3	104381	NUT HEX NYLOCK M6X1.0	
4	106944	WASHER FLAT THIN AN 1/4 ROHS	
5	107199	HANGER EXTENSION SADDLE	
6	104377	3/4" SADDLE THIN	
7	541002	FTRST MOUNTING BRKT-3/8 RAD R	RIGHT
7	541003	FTRST MOUNTING BRKT-3/8 RAD L	LEFT
8	106945	WASHER FLAT SAE 1/4 ROHS	
9	104399	SHCS M6X1X45 12 MM PART THD	
10	106978	FHCS 1/4-28 X 7/8 ROHS	
11	550350	3/4" CLMP ADJ FT PLT	
12	106935	NUT NYLOCK THIN 1/4-28 ROHS	
13	104328	POST HEEL LOOP	
14A	104380-001	ADJUSTABLE FOOT REST STRAP	12 - 14" WIDE
14B	104380-002	ADJUSTABLE FOOT REST STRAP	15 - 16" WIDE
14C	104380-003	ADJUSTABLE FOOT REST STRAP	17 - 18" WIDE
14D	104380-004	ADJUSTABLE FOOT REST STRAP	19 - 20" WIDE
15	102268-001	HHCS M6 X 1 X 25	
3-6,8,9	120659	EXT MNT HARDWARE 1 SIDE	
16-19	107485	FOOTREST CLAMP ASSM	
20	740260	TOE-ANKLE STRAP ADJ ADULT FOOTPLATE	
>>>>		80° HANGERS - SHORT EXT - RIGHT	
1A,2B,3-14A,15	111534	AAA STRT SHT EXT ASM 12-13" W R	Right- 80D- 12-13" W- SHORT
1A,2C,3-14B,15	107928	AAA STRT SHT EXT ASM 14-15" W R	Right- 80D- 14-15" W- SHORT
1A,2D,3-14C,15	107930	AAA STRT SHT EXT ASM 16-17" W R	Right- 80D- 16-17" W- SHORT
1A,2E,3-14D,15	107932	AAA STRT SHT EXT ASM 18-22" W R	Right- 80D- 18-22" W- SHORT
>>>>		80° HANGERS - SHORT EXT - LEFT	
1A,2B,3-14A,15	111535	AAA STRT SHT EXT ASM 12-13" W L	Left- 80D- 12-13" W- SHORT
1A,2C,3-14B,15	107935	AAA STRT SHT EXT ASM 14" W L	Left- 80D- 14-15" W- SHORT
1A,2D,3-14C,15	107937	AAA STRT SHT EXT ASM 16" W L	Left- 80D- 16-17" W- SHORT
1A,2E,3-14D,15	107939	AAA STRT SHT EXT ASM 18" W L	Left- 80D- 18-22" W- SHORT
>>>>		80° HANGERS - MED EXT - RIGHT	
1B,2B,3-14A,15	111536	AAA STRT MED EXT ASM 12-13" W R	Right- 80D- 12-13" W- MED
1B,2C,3-14B,15	107942	AAA STRT MED EXT ASM 14-15" W R	Right- 80D- 14-15" W- MED
1B,2D,3-14C,15	107944	AAA STRT MED EXT ASM 16-17" W R	Right- 80D- 16-17" W- MED
1B,2E,3-14D,15	107946	AAA STRT MED EXT ASM 18-22" W R	Right- 80D- 18-22" W- MED
>>>>		80° HANGERS - MED EXT - LEFT	
1B,2B,3-14A,15	111537	AAA STRT MED EXT ASM 12-13" W L	Left- 80D- 12-13" W- MED
1B,2C,3-14B,15	107949	AAA STRT MED EXT ASM 14" W L	Left- 80D- 14-15" W- MED
1B,2D,3-14C,15	107951	AAA STRT MED EXT ASM 16" W L	Left- 80D- 16-17" W- MED
1B,2E,3-14D,15	107953	AAA STRT MED EXT ASM 18" W L	Left- 80D- 18-22" W- MED
>>>>		80° HANGERS - LONG EXT - RIGHT	
1C,2B,3-14A,15	111538	AAA STRT LNG EXT ASM 12-13" W R	Right- 80D- 12-13" W- LONG
1C,2C,3-14B,15	107956	AAA STRT LNG EXT ASM 14 -15" W R	Right- 80D- 14-15" W- LONG
1C,2D,3-14C,15	107958	AAA STRT LNG EXT ASM 16 -17" W R	Right- 80D- 16-17" W- LONG
1C,2E,3-14D,15	107960	AAA STRT LNG EXT ASM 18 -22" W R	Right- 80D- 18-22" W- LONG
>>>>		80° HANGERS - LONG EXT - LEFT	
1C,2B,3-14A,15	111539	AAA STRT LNG EXT ASM 12-13" W L	Left- 80D- 12-13" W- LONG
1C,2C,3-14B,15	107963	AAA STRT LNG EXT ASM 14W L	Left- 80D- 14-15" W- LONG
1C,2D,3-14C,15	107965	AAA STRT LNG EXT ASM 16W L	Left- 80D- 16-17" W- LONG


Pos.	Item Number	Description	Remarks
1C,2E,3-14D,15	107968	AAA STRT LNG EXT ASM 18W L	Left- 80D- 18-22" W- LONG
>>>>		70° HANGERS - SHORT EXT - RIGHT	
1D,2B,3-14A,15	111540	AAA BNT SHT EXT ASM 12-13" W R	Right- 60D,70D,70D HEMI - 12-13" W- SHORT
1D,2C,3-14B,15	107973	AAA BNT SHT EXT ASM 14 -15" W R	Right- 60D,70D,70D HEMI - 14-15" W- SHORT
1D,2D,3-14C,15	107975	AAA BNT SHT EXT ASM 16 -17" W R	Right- 60D,70D,70D HEMI - 16-17" W- SHORT
1D,2E,3-14D,15	107977	AAA BNT SHT EXT ASM 18 -22" W R	Right- 60D,70D,70D HEMI - 18-22" W- SHORT
>>>>		70° HANGERS - SHORT EXT - LEFT	
1D,2B,3-14A,15	111541	AAA BNT SHT EXT ASM 12-13" W L	Left- 60D,70D,70D HEMI - 12-13" W- SHORT
1D,2C,3-14B,15	107980	AAA BNT SHT EXT ASM 14 -15" W L	Left- 60D,70D,70D HEMI - 14-15" W- SHORT
1D,2D,3-14C,15	107982	AAA BNT SHT EXT ASM 16 -17" W L	Left- 60D,70D,70D HEMI - 16-17" W- SHORT
1D,2E,3-14D,15	107984	AAA BNT SHT EXT ASM 18 -22" W L	Left- 60D,70D,70D HEMI - 18-22" W- SHORT
>>>>		70° HANGERS - MED EXT - RIGHT	
1E,2B,3-14A,15	111542	AAA BNT MED EXT ASM 12-13" W R	Right- 60D,70D,70D HEMI - 12-13" W- MED
1E,2C,3-14B,15	107987	AAA BNT MED EXT ASM 14 -15" W R	Right- 60D,70D,70D HEMI - 14-15" W- MED
1E,2D,3-14C,15	107989	AAA BNT MED EXT ASM 16 -17" W R	Right- 60D,70D,70D HEMI - 16-17" W- MED
1E,2E,3-14D,15	107991	AAA BNT MED EXT ASM 18 -22" W R	Right- 60D,70D,70D HEMI - 18-22" W- MED
>>>>		70° HANGERS - MED EXT - LEFT	
1E,2B,3-14A,15	111543	AAA BNT MED EXT ASM 12-13" W L	Left- 60D,70D,70D HEMI - 12-13" W- MED
1E,2C,3-14B,15	107994	AAA BNT MED EXT ASM 14 -15" W L	Left- 60D,70D,70D HEMI - 14-15" W- MED
1E,2D,3-14C,15	107996	AAA BNT MED EXT ASM 16 -17" W L	Left- 60D,70D,70D HEMI - 16-17" W- MED
1E,2E,3-14D,15	107998	AAA BNT MED EXT ASM 18 -22" W L	Left- 60D,70D,70D HEMI - 18-22" W- MED
>>>>		70° HANGERS - LONG EXT - RIGHT	
1F,2B,3-14A,15	111544	AAA BNT LONG EXT ASM 12-13" W R	Right- 60D,70D,70D HEMI - 12-13" W- LONG
1F,2C,3-14B,15	108001	AAA BNT LONG EXT ASM 14 -15" W R	Right- 60D,70D,70D HEMI - 14-15" W- LONG
1F,2D,3-14C,15	108003	AAA BNT LONG EXT ASM 16 -17" W R	Right- 60D,70D,70D HEMI - 16-17" W- LONG
1F,2E,3-14D,15	108005	AAA BNT LONG EXT ASM 18 -22" W R	Right- 60D,70D,70D HEMI - 18-22" W- LONG
>>>>		70° HANGERS - LONG EXT - LEFT	
1F,2B,3-14A,15	111545	AAA BNT LONG EXT ASM 12-13" W L	Left- 60D,70D,70D HEMI - 12-13" W- LONG
1F,2C,3-14B,15	108008	AAA BNT LONG EXT ASM 14 -15" W L	Left- 60D,70D,70D HEMI - 14-15" W- LONG
1F,2D,3-14C,15	108010	AAA BNT LONG EXT ASM 16 -17" W L	Left- 60D,70D,70D HEMI - 16-17" W- LONG
1F,2E,3-14D,15	108012	AAA BNT LONG EXT ASM 18 -22" W L	Left- 60D,70D,70D HEMI - 18-22" W- LONG
>>>>		70° "V" HANGERS - MED EXT - RIGHT	
1G,2A,3-15	111546	AAA V MED EXT ASM 10-11" W R	Right- 70D V hangers only 10-11" W- MED
1G,2B,3-15	111548	AAA V MED EXT ASM 12-13" W R	Right- 70D V hangers only 12-13" W- MED
1G,2C,3-15	108015	AAA V MED EXT ASM 14 -15" W R	Right- 70D V hangers only 14-15" W- MED
1G,2D,3-15	108017	AAA V MED EXT ASM 16 -17" W R	Right- 70D V hangers only 16-18" W- MED
1G,2E,3-15	108019	AAA V MED EXT ASM 18 -18" W R	Right- 70D V hangers only 18-22" W- MED
>>>>		70° "V" HANGERS - MED EXT - LEFT	
1G,2A,3-15	111547	AAA V MED EXT ASM 10-11" W L	Left- 70D V hangers only 10-11" W- MED
1G,2B,3-15	111549	AAA V MED EXT ASM 12-13" W L	Left- 70D V hangers only 12-13" W- MED
1G,2C,3-15	108022	AAA V MED EXT ASM 14 -15" W L	Left- 70D V hangers only 14-15" W- MED
1G,2D,3-15	108024	AAA V MED EXT ASM 16 -17" W L	Left- 70D V hangers only 16-18" W- MED
1G,2E,3-15	108026	AAA V MED EXT ASM 18 -22" W L	Left- 70D V hangers only 18-22" W- MED
>>>>		70° "V" HANGERS - LONG EXT - RIGHT	
1H,2A,3-15	111550	AAA V LONG EXT ASM 10-11" W R	Right- 70D V hangers only 10-11" W- LONG
1H,2B,3-15	111552	AAA V LONG EXT ASM 12-13" W R	Right- 70D V hangers only 12-13" W- LONG
1H,2C,3-15	108029	AAA V LONG EXT ASM 14 -15" W R	Right- 70D V hangers only 14-15" W- LONG
1H,2D,3-15	108031	AAA V LONG EXT ASM 16 -17" W R	Right- 70D V hangers only 16-18" W- LONG
1H,2E,3-15	108033	AAA V LONG EXT ASM 18 -22" W R	Right- 70D V hangers only 18-22" W- LONG
>>>>		70° "V" HANGERS - LONG EXT - LEFT	
1H,2A,3-15	111551	AAA V LONG EXT ASM 10-11" W L	Left- 70D V hangers only 10-11" W- LONG
1H,2B,3-15	111553	AAA V LONG EXT ASM 12-13" W L	Left- 70D V hangers only 12-13" W- LONG
1H,2C,3-15	108036	AAA V LONG EXT ASM 14 -15" W L	Left- 70D V hangers only 14-15" W- LONG
1H,2D,3-15	108038	AAA V LONG EXT ASM 16 -17" W L	Left- 70D V hangers only 16-18" W- LONG
1H,2E,3-15	108040	AAA V LONG EXT ASM 18 -22" W L	Left- 70D V hangers only 18-22" W- LONG

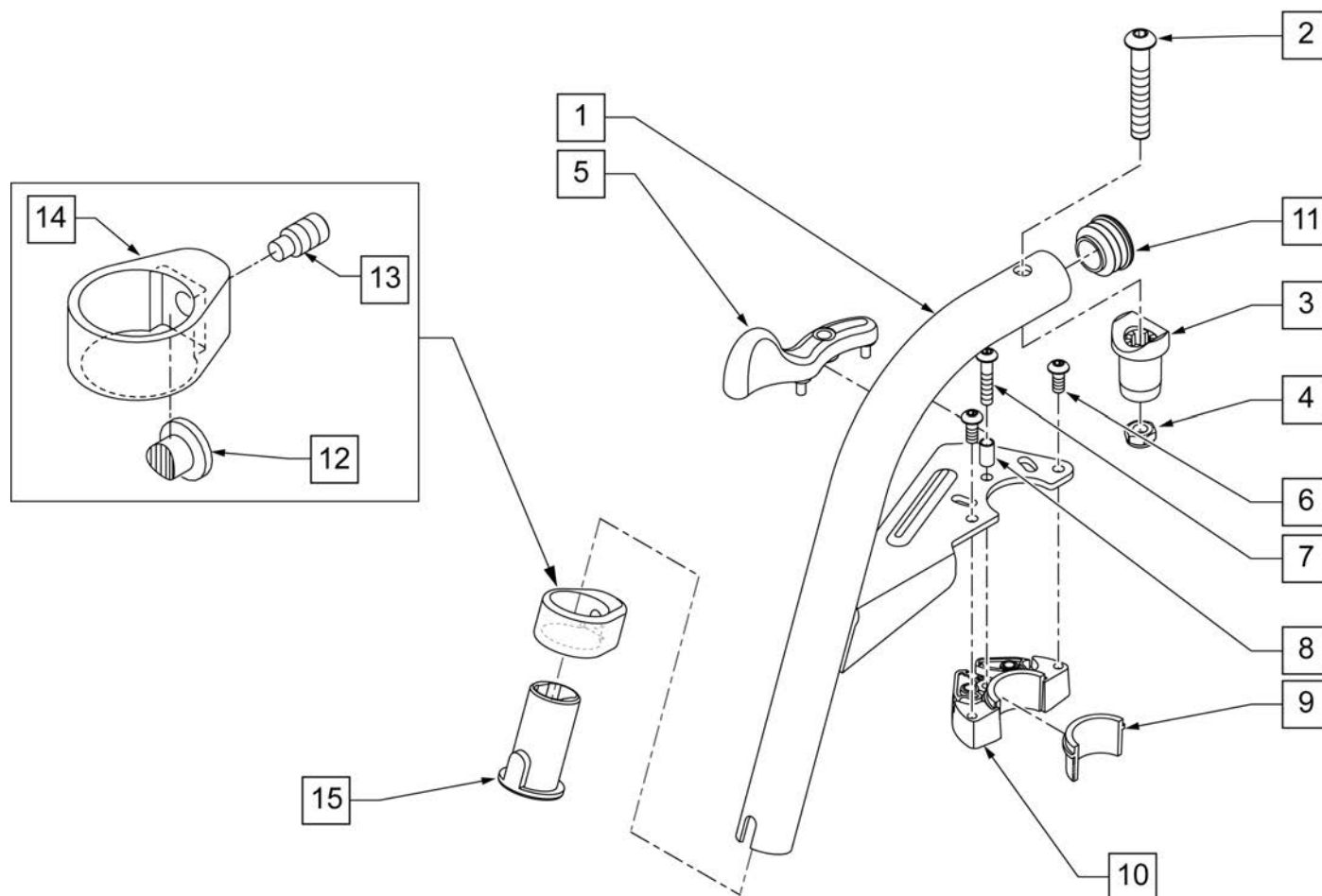

Pos.	Item Number	Description	Remarks
2	106983	5/16-18 X 2 BH C/S SHT THREAD	
3,4	561197	PIVOT SADDLE PLASTIC	
5	052296	SWING I/O LATCH LEVER RIGHT	Right
5	052297	SWING I/O LATCH LEVER LEFT	Left
6	102259-001	BHCS M5 X .8 X 12 W/PL INORG	
7	230494	M5-30 BTTN HD SOC C/S BLK	
8	052054	BUSHING PIVOT HANGER SWING I/O	
11	561062	1" INSERT PLUG .065 RIBS	
15	104160	BUSHING FOOTREST	
1-15	107671S	70 DEG SWING IN/OUT HANGER ASSEMBLY LEFT	SPECIFY COLOR,LEFT
1-15	107670S	70 DEG SWING IN/OUT HANGER ASSEMBLY RIGHT	SPECIFY COLOR,RIGHT
12-15	107485	FOOTREST CLAMP ASSM	
6,9,10	120976	ASSY LATCH BLOCK ANTI LIFT	

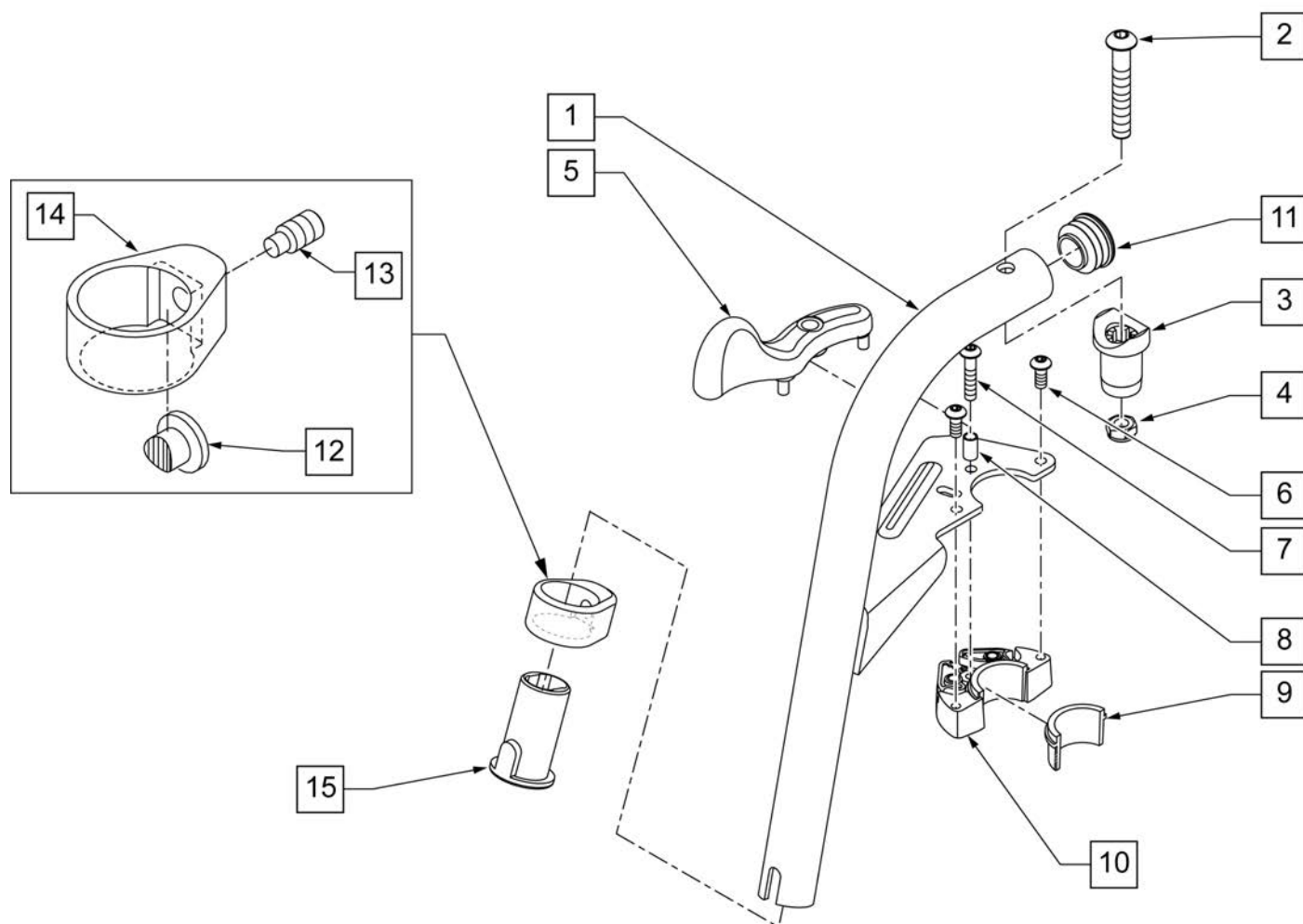

Pos.	Item Number	Description	Remarks
2	106983	5/16-18 X 2 BH C/S SHT THREAD	
3	561197	PIVOT SADDLE PLASTIC	Includes nylock nut
5	052296	SWING I/O LATCH LEVER RIGHT	Right
5	052297	SWING I/O LATCH LEVER LEFT	Left
7	230494	M5-30 BTTN HD SOC C/S BLK	
8	052054	BUSHING PIVOT HANGER SWING I/O	
11	561062	1" INSERT PLUG .065 RIBS	
15	104160	BUSHING FOOTREST	
1-15	107675S	ASSEMBLY LEFT	SPECIFY COLOR,LEFT
1-15	107674S	ASSEMBLY RIGHT	SPECIFY COLOR,RIGHT
12-15	107485	FOOTREST CLAMP ASSM	
6,9,10	120976	ASSY LATCH BLOCK ANTI LIFT	

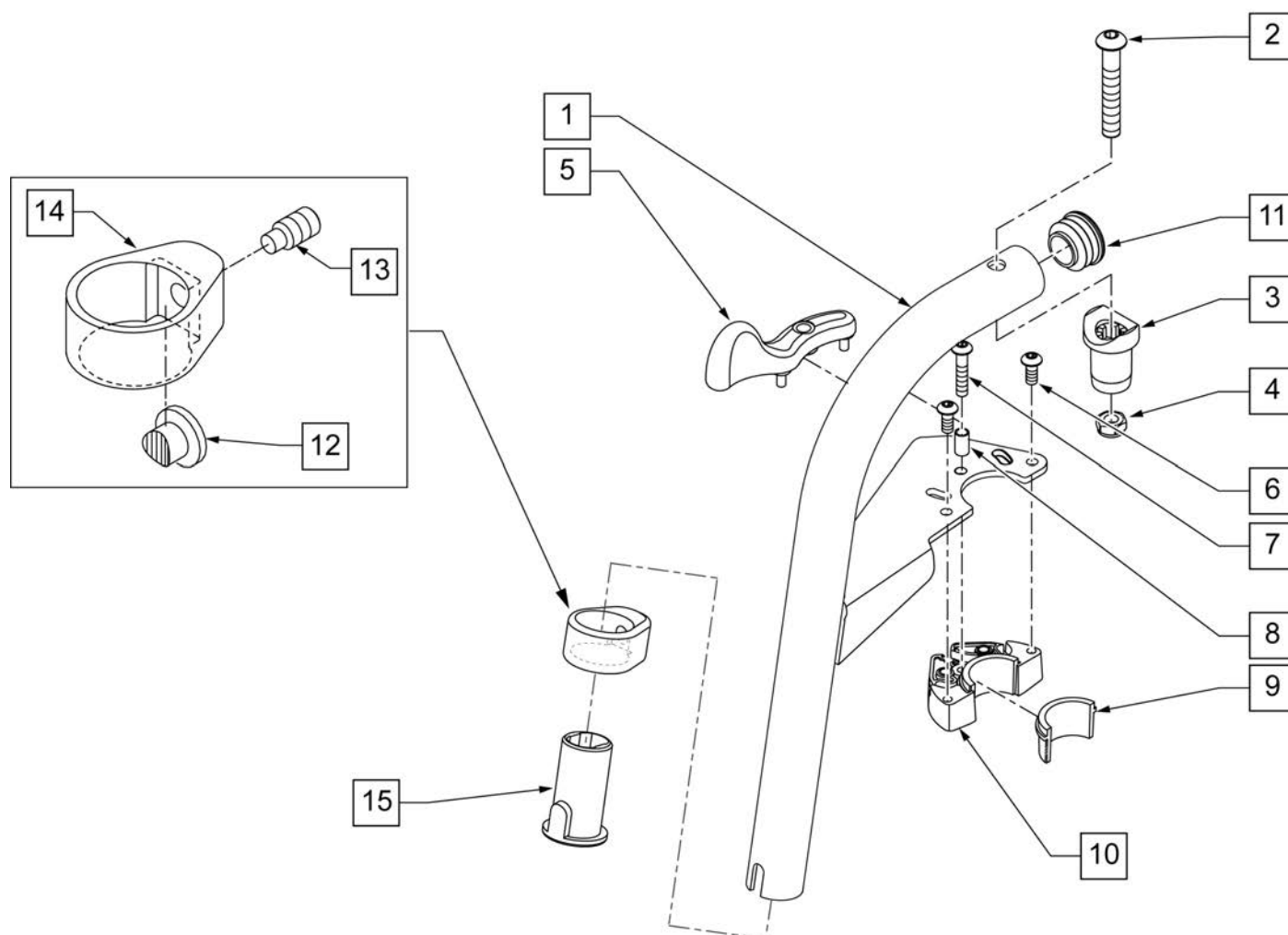

Pos.	Item Number	Description	Remarks
2	106983	5/16-18 X 2 BH C/S SHT THREAD	
3,4	561197	PIVOT SADDLE PLASTIC	
5	052296	SWING I/O LATCH LEVER RIGHT	Right
5	052297	SWING I/O LATCH LEVER LEFT	Left
6	102259-001	BHCS M5 X .8 X 12 W/PL INORG	
7	230494	M5-30 BTTN HD SOC C/S BLK	
8	052054	BUSHING PIVOT HANGER SWING I/O	
11	561062	1" INSERT PLUG .065 RIBS	
15	104160	BUSHING FOOTREST	
1-15	107673S	ASSEMBLY LEFT	SPECIFY COLOR,LEFT
1-15	107672S	ASSEMBLY RIGHT	SPECIFY COLOR,RIGHT
12-15	107485	FOOTREST CLAMP ASSM	
6,9,10	120976	ASSY LATCH BLOCK ANTI LIFT	

NOTE: FOOTPLATE ASSEMBLIES ARE SET TO FOOTREST WIDTH. PLEASE SEE "FOOTPLATE FORMULA" TABLE FOR SEAT WIDTH CONVERSION.

INSET FORMULA
 0" Inset (Seat Width) = Footrest Width
 1" Inset (Seat Width) -2 = Footrest Width
 "V" Hanger (Seat Width) -4 = Footrest Width

Composite Footplates w/ Ext & HL for Swing I/O Hangers
 (Order 1 each)

Ext. Tube Width	60°, 70° & 70° Hemi Style Hangers (Bent Extension)			80° Style hanger ELR (Straight Extension)			70° V Style Hangers	
	Short	Medium	Long	Short	Medium	Long	Medium	Long
13" Right	107746	107760	107774	107704	107718	107732	107788	107802
13" Left	107753	107767	107781	107711	107725	107739	107795	107809
14" Right	107747	107761	107775	107705	107719	107733	107789	107803
14" Left	107754	107768	107782	107712	107726	107740	107796	107810
15" Right	107748	107762	107776	107706	107720	107734	107790	107804
15" Left	107755	107769	107783	107713	107727	107741	107797	107811
16" Right	107749	107763	107777	107707	107721	107735	107791	107805
16" Left	107756	107770	107784	107714	107728	107742	107798	107812
17" Right	107750	107764	107778	107708	107722	107736	107792	107806
17" Left	107757	107771	107785	107715	107729	107743	107799	107813
18" Right	107751	107765	107779	107709	107723	107737	107793	107807
18" Left	107758	107772	107786	107716	107730	107744	107800	107814
19"-20" Rt	107752	107766	107780	107710	107724	107738	V STYLE IS NOT AVAILABLE IN THESE SEAT WIDTHS	
19"-20" Lt	107759	107773	107787	107717	107731	107745		
21"-22" Rt	111528	111530	111532	111522	111524	111526		
21"-22" Lt	111529	111531	111533	111523	111525	111527		

FOOTPLATE MEASURING GUIDE

PART NO	DESCRIPTION	DIM -A-	DIM -B-
063216-006	FOOTPLATE COMPOSITE 157 R	6 1/8"	6"
063216-012	FOOTPLATE COMPOSITE 157 L	6 1/8"	6"
063216-015	FOOTPLATE COMPOSITE 170 R	6 5/8"	6"
063216-018	FOOTPLATE COMPOSITE 170 L	6 5/8"	6"
063216-005	FOOTPLATE COMPOSITE 182 R	7 1/8"	6"
063216-011	FOOTPLATE COMPOSITE 182 L	7 1/8"	6"
063216-014	FOOTPLATE COMPOSITE 195 R	7 5/8"	6"
063216-017	FOOTPLATE COMPOSITE 195 L	7 5/8"	6"
063216-004	FOOTPLATE COMPOSITE 207 R	8 1/8"	6"
063216-010	FOOTPLATE COMPOSITE 207 L	8 1/8"	6"
063216-013	FOOTPLATE COMPOSITE 220 R	8 5/8"	6 1/4"
063216-016	FOOTPLATE COMPOSITE 220 L	8 5/8"	6 1/4"
063216-003	FOOTPLATE COMPOSITE 232 R	9 1/8"	6 1/4"
063216-009	FOOTPLATE COMPOSITE 232 L	9 1/8"	6 1/4"
063216-002	FOOTPLATE COMPOSITE 257 R	10 1/8"	6 1/4"
063216-008	FOOTPLATE COMPOSITE 257 L	10 1/8"	6 1/4"
063216-001	FOOTPLATE COMPOSITE 282 R	11 1/8"	6 1/4"
063216-007	FOOTPLATE COMPOSITE 282 L	11 1/8"	6 1/4"

Pos.	Item Number	Description	Remarks
1A-5	120660	SHT STRT EXT COMP HDWR	80 deg hangers + ELR only
1B-5	120661	MED STRT EXT COMP HDWR	80 deg hangers + ELR only
1C-5	120662	LNG STRT EXT COMP HDWR	80 deg hangers + ELR only
1D-5	120663	SHT BNT EXT COMP HDWR	60,70,70 Hemi Hangers
1E-5	120664	MED BNT EXT COMP HDWR	60,70,70 Hemi Hangers
1F-5	120665	LNG BNT EXT COMP HDWR	60,70,70 Hemi Hangers
1G-5	120666	SHT V EXT R COMP HDWR	70 deg V hangers only-LEFT
1G-5	120667	SHT V EXT L COMP HDWR	70 deg V hangers only-RIGHT
1H-5	120668	LNG V EXT R COMP HDWR	70 deg V hangers only-LEFT
1H-5	120669	LNG V EXT L COMP HDWR	70 deg V hangers only-RIGHT
2A	063216-006	COMPOSITE FOOTPLATE RIGHT	RIGHT 13" Footrest Width
2A	063216-012	COMPOSITE FOOTPLATE LEFT	LEFT 13" Footrest Width
2B	063216-015	FOOTPLATE COMPOSITE RIGHT	RIGHT 14" Footrest Width

Pos.	Item Number	Description	Remarks
2B	063216-018	FOOTPLATE COMPOSITE LEFT	LEFT 14" Footrest Width
2C	063216-005	COMPOSITE FOOTPLATE RIGHT	RIGHT 15" Footrest Width
2C	063216-011	COMPOSITE FOOTPLATE LEFT	LEFT 15" Footrest Width
2D	063216-014	FOOTPLATE COMPOSITE RIGHT	RIGHT 16" Footrest Width
2D	063216-017	FOOTPLATE COMPOSITE LEFT	LEFT 16" Footrest Width
2E	063216-004	COMPOSITE FOOTPLATE RIGHT	RIGHT 17" Footrest Width
2E	063216-010	COMPOSITE FOOTPLATE LEFT	LEFT 17" Footrest Width
2F	063216-013	FOOTPLATE COMPOSITE RIGHT	RIGHT 18" Footrest Width
2F	063216-016	FOOTPLATE COMPOSITE LEFT	LEFT 18" Footrest Width
2G	063216-003	COMPOSITE FOOTPLATE RIGHT	RIGHT 19" & 20" Footrest Width
2G	063216-009	COMPOSITE FOOTPLATE LEFT	LEFT 19" & 20" Footrest Width
2H	063216-002	FOOTPLATE COMPOSITE RIGHT	RIGHT 21" & 22" Footrest Width
2H	063216-008	FOOTPLATE COMPOSITE LEFT	LEFT 21" & 22" Footrest Width
3	104377	3/4" SADDLE THIN	
4	103175	SHCS M6 X 1 X 40 INORG	
5	104381	NUT HEX NYLOCK M6X1.0	Inserted in bottom of footplate.
6	104328	HEEL LOOP PEG	
7A	104380-001	ADJUSTABLE FOOT REST STRAP	14" wide
7B	104380-002	ADJUSTABLE FOOT REST STRAP	15-16" wide
7C	104380-003	ADJUSTABLE FOOT REST STRAP	17-18" wide
7D	104380-004	ADJUSTABLE FOOT REST STRAP	19-22" wide
8	102268-001	HHCS M6 X 1 X 25	
12	104160	BUSHING FOOTREST	
9-12	107485	FOOTREST CLAMP ASSM	
>>>>		60°,70°,70° HEMI HANGERS - SHORT EXT - RIGHT	
1D,2A,3-6,7A,8	107746	COMP BNT SHT EXT ASM 13" W R	Right- 70D- 13" W- SHORT
1D,2B,3-6,7A,8	107747	COMP BNT SHT EXT ASM 14" W R	Right- 70D- 14" W- SHORT
1D,2C,3-6,7B,8	107748	COMP BNT SHT EXT ASM 15" W R	Right- 70D- 15" W- SHORT
1D,2D,3-6,7B,8	107749	COMP BNT SHT EXT ASM 16" W R	Right- 70D- 16" W- SHORT
1D,2E,3-6,7C,8	107750	COMP BNT SHT EXT ASM 17" W R	Right- 70D- 17" W- SHORT
1D,2F,3-6,7C,8	107751	COMP BNT SHT EXT ASM 18" W R	Right- 70D- 18" W- SHORT
1D,2G,3-6,7D,8	107752	COMP BNT SHT EXT ASM 19 & 20"W R	Right- 70D- 19 & 20" W- SHORT
1D,2H,3-6,7D,8	111528	COMP BNT SHT EXT ASM 21 & 22"W R	Right- 70D- 21 & 22" W- SHORT
>>>>		60°,70°,70° HEMI HANGERS - SHORT EXT - LEFT	
1D,2A,3-6,7A,8	107753	COMP BNT SHT EXT ASM 13" W L	Left- 70D- 13 W- SHORT
1D,2B,3-6,7A,8	107754	COMP BNT SHT EXT ASM 14" W L	Left- 70D- 14 W- SHORT
1D,2C,3-6,7B,8	107755	COMP BNT SHT EXT ASM 15" W L	Left- 70D- 15 W- SHORT
1D,2D,3-6,7B,8	107756	COMP BNT SHT EXT ASM 16" W L	Left- 70D- 16 W- SHORT
1D,2E,3-6,7C,8	107757	COMP BNT SHT EXT ASM 17" W L	Left- 70D- 17 W- SHORT
1D,2F,3-6,7C,8	107758	COMP BNT SHT EXT ASM 18" W L	Left- 70D- 18 W- SHORT
1D,2G,3-6,7D,8	107759	COMP BNT SHT EXT ASM 19 & 20"W L	Left- 70D- 19 & 20" W- SHORT
1D,2H,3-6,7D,8	111529	COMP BNT SHT EXT ASM 21 & 22"W L	Left- 70D- 21 & 22" W- SHORT
>>>>		60°,70°,70° HEMI HANGERS - MED EXT - RIGHT	
1E,2A,3-6,7A,8	107760	COMP BNT MED EXT ASM 13" W R	Right- 70D- 13" W- MED
1E,2B,3-6,7A,8	107761	COMP BNT MED EXT ASM 14" W R	Right- 70D- 14" W- MED
1E,2C,3-6,7B,8	107762	COMP BNT MED EXT ASM 15" W R	Right- 70D- 15" W- MED
1E,2D,3-6,7B,8	107763	COMP BNT MED EXT ASM 16" W R	Right- 70D- 16" W- MED
1E,2E,3-6,7C,8	107764	COMP BNT MED EXT ASM 17" W R	Right- 70D- 17" W- MED
1E,2F,3-6,7C,8	107765	COMP BNT MED EXT ASM 18" W R	Right- 70D- 18" W- MED
1E,2G,3-6,7D,8	107766	COMP BNT MED EXT ASM 19 & 20"W R	Right- 70D- 19 & 20" W- MED
1E,2H,3-6,7D,8	111530	COMP BNT MED EXT ASM 21 & 22"W R	Right- 70D- 21 & 22" W- MED
>>>>		60°,70°,70° HEMI HANGERS - MED EXT - LEFT	
1E,2A,3-6,7A,8	107767	COMP BNT MED EXT ASM 13" W L	Left- 70D- 13 W- MED
1E,2B,3-6,7A,8	107768	COMP BNT MED EXT ASM 14" W L	Left- 70D- 14 W- MED
1E,2C,3-6,7B,8	107769	COMP BNT MED EXT ASM 15" W L	Left- 70D- 15 W- MED
1E,2D,3-6,7B,8	107770	COMP BNT MED EXT ASM 16" W L	Left- 70D- 16 W- MED

Pos.	Item Number	Description	Remarks
1E,2E,3-6,7C,8	107771	COMP BNT MED EXT ASM 17" W L	Left- 70D- 17 W- MED
1E,2F,3-6,7C,8	107772	COMP BNT MED EXT ASM 18" W L	Left- 70D- 18 W- MED
1E,2G,3-6,7D,8	107773	COMP BNT MED EXT ASM 19 & 20"W L	Left- 70D- 19 & 20" W- MED
1E,2H,3-6,7D,8	111531	COMP BNT MED EXT ASM 21 & 22"W L	Left- 70D- 21 & 22" W- MED
>>>>		60°,70°,70° HEMI HANGERS - LONG EXT - RIGHT	
1F,2A,3-6,7A,8	107774	COMP BNT LONG EXT ASM 13" W R	Right- 70D- 13" W- LONG
1F,2B,3-6,7A,8	107775	COMP BNT LONG EXT ASM 14" W R	Right- 70D- 14" W- LONG
1F,2C,3-6,7B,8	107776	COMP BNT LONG EXT ASM 15" W R	Right- 70D- 15" W- LONG
1F,2D,3-6,7B,8	107777	COMP BNT LONG EXT ASM 16" W R	Right- 70D- 16" W- LONG
1F,2E,3-6,7C,8	107778	COMP BNT LONG EXT ASM 17" W R	Right- 70D- 17" W- LONG
1F,2F,3-6,7C,8	107779	COMP BNT LONG EXT ASM 18" W R	Right- 70D- 18" W- LONG
1F,2G,3-6,7D,8	107780	COMP BNT LONG EXT ASM 19 & 20"W R	Right- 70D- 19 & 20" W- LONG
1F,2H,3-6,7D,8	111532	COMP BNT LONG EXT ASM 21 & 22"W R	Right- 70D- 21 & 22" W- LONG
>>>>		60°,70°,70° HEMI HANGERS - LONG EXT - LEFT	
1F,2A,3-6,7A,8	107781	COMP BNT LONG EXT ASM 13" W L	Left- 70D- 13 W- LONG
1F,2B,3-6,7A,8	107782	COMP BNT LONG EXT ASM 14" W L	Left- 70D- 14 W- LONG
1F,2C,3-6,7B,8	107783	COMP BNT LONG EXT ASM 15" W L	Left- 70D- 15 W- LONG
1F,2D,3-6,7B,8	107784	COMP BNT LONG EXT ASM 16" W L	Left- 70D- 16 W- LONG
1F,2E,3-6,7C,8	107785	COMP BNT LONG EXT ASM 17" W L	Left- 70D- 17 W- LONG
1F,2F,3-6,7C,8	107786	COMP BNT LONG EXT ASM 18" W L	Left- 70D- 18 W- LONG
1F,2G,3-6,7D,8	107787	COMP BNT LONG EXT ASM 19 & 20"W L	Left- 70D- 19 & 20" W- LONG
1F,2H,3-6,7D,8	111533	COMP BNT LONG EXT ASM 21 & 22"W L	Left- 70D- 21 & 22" W- LONG
>>>>		70 DEGREE "V" HANGERS - MED EXT - RIGHT	
1G,2A,3-8	107788	COMP V MED EXT ASM 13W R	Right- 70D V hangers only 13 W- MED
1G,2B,3-8	107789	COMP V MED EXT ASM 14W R	Right- 70D V hangers only 14 W- MED
1G,2C,3-8	107790	COMP V MED EXT ASM 15W R	Right- 70D V hangers only 15 W- MED
1G,2D,3-8	107791	COMP V MED EXT ASM 16W R	Right- 70D V hangers only 16 W- MED
1G,2E,3-8	107792	COMP V MED EXT ASM 17W R	Right- 70D V hangers only 17 W- MED
1G,2F,3-8	107793	COMP V MED EXT ASM 18W R	Right- 70D V hangers only 18 W- MED
>>>>		70 DEGREE "V" HANGERS - MED EXT - LEFT	
1G,2A,3-8	107795	COMP V MED EXT ASM 13W L	Left- 70D V hangers only 13 W- MED
1G,2B,3-8	107796	COMP V MED EXT ASM 14W L	Left- 70D V hangers only 14 W- MED
1G,2C,3-8	107797	COMP V MED EXT ASM 15W L	Left- 70D V hangers only 15 W- MED
1G,2D,3-8	107798	COMP V MED EXT ASM 16W L	Left- 70D V hangers only 16 W- MED
1G,2E,3-8	107799	COMP V MED EXT ASM 17W L	Left- 70D V hangers only 17 W- MED
1G,2F,3-8	107800	COMP V MED EXT ASM 18W L	Left- 70D V hangers only 18 W- MED
>>>>		70 DEGREE "V" HANGERS - LONG EXT - RIGHT	
1H,2A,3-8	107802	COMP V LNG EXT ASM 13W R	Right- 70D V hangers only 13 W- LONG
1H,2B,3-8	107803	COMP V LNG EXT ASM 14W R	Right- 70D V hangers only 14 W- LONG
1H,2C,3-8	107804	COMP V LNG EXT ASM 15W R	Right- 70D V hangers only 15 W- LONG
1H,2D,3-8	107805	COMP V LNG EXT ASM 16W R	Right- 70D V hangers only 16 W- LONG
1H,2E,3-8	107806	COMP V LNG EXT ASM 17W R	Right- 70D V hangers only 17 W- LONG
1H,2F,3-8	107807	COMP V LNG EXT ASM 18W R	Right- 70D V hangers only 18 W- LONG
>>>>		70 DEGREE "V" HANGERS - LONG EXT - LEFT	
1H,2A,3-8	107809	COMP V LNG EXT ASM 13W L	Left- 70D V hangers only 13 W- LONG
1H,2B,3-8	107810	COMP V LNG EXT ASM 14W L	Left- 70D V hangers only 14 W- LONG
1H,2C,3-8	107811	COMP V LNG EXT ASM 15W L	Left- 70D V hangers only 15 W- LONG
1H,2D,3-8	107812	COMP V LNG EXT ASM 16W L	Left- 70D V hangers only 16 W- LONG
1H,2E,3-8	107813	COMP V LNG EXT ASM 17W L	Left- 70D V hangers only 17 W- LONG
1H,2F,3-8	107814	COMP V LNG EXT ASM 18W L	Left- 70D V hangers only 18 W- LONG
>>>>		80 DEGREE HANGERS - SHORT EXT - RIGHT	
1A,2A,3-6,7A,8	107704	COMP STRT SHT EXT ASM 13"W R	Right- 80D- 13" W- SHORT
1A,2B,3-6,7A,8	107705	COMP STRT SHT EXT ASM 14"W R	Right- 80D- 14" W- SHORT
1A,2C,3-6,7B,8	107706	COMP STRT SHT EXT ASM 15"W R	Right- 80D- 15" W- SHORT
1A,2D,3-6,7B,8	107707	COMP STRT SHT EXT ASM 16"W R	Right- 80D- 16" W- SHORT
1A,2E,3-6,7C,8	107708	COMP STRT SHT EXT ASM 17"W R	Right- 80D- 17" W- SHORT

Pos.	Item Number	Description	Remarks
1A,2F,3-6,7C,8	107709	COMP STRT SHT EXT ASM 18"W R	Right- 80D- 18" W- SHORT
1A,2G,3-6,7D,8	107710	COMP STRT SHT EXT ASM 19 & 20"W R	Right- 80D- 19 & 20" W- SHORT
1A,2H,3-6,7D,8	111522	COMP STRT SHT EXT ASM 21 & 22"W R	Right- 80D- 21 & 22" W- SHORT
>>>>		80 DEGREE HANGERS - SHORT EXT - LEFT	
1A,2A,3-6,7A,8	107711	COMP STRT SHT EXT ASM 13"W L	LEFT- 80D- 13" W- SHORT
1A,2B,3-6,7A,8	107712	COMP STRT SHT EXT ASM 14"W L	LEFT- 80D- 14" W- SHORT
1A,2C,3-6,7B,8	107713	COMP STRT SHT EXT ASM 15"W L	LEFT- 80D- 15" W- SHORT
1A,2D,3-6,7B,8	107714	COMP STRT SHT EXT ASM 16"W L	LEFT- 80D- 16" W- SHORT
1A,2E,3-6,7C,8	107715	COMP STRT SHT EXT ASM 17"W L	LEFT- 80D- 17" W- SHORT
1A,2F,3-6,7C,8	107716	COMP STRT SHT EXT ASM 18"W L	LEFT- 80D- 18" W- SHORT
1A,2G,3-6,7D,8	107717	COMP STRT SHT EXT ASM 19 & 20"W L	LEFT- 80D- 19 & 20" W- SHORT
1A,2H,3-6,7D,8	111523	COMP STRT SHT EXT ASM 21 & 22"W L	LEFT- 80D- 21 & 22" W- SHORT
>>>>		80 DEGREE HANGERS - MED EXT - RIGHT	
1B,2A,3-6,7A,8	107718	COMP STRT MED EXT ASM 13"W R	Right- 80D- 13" W- MED
1B,2B,3-6,7A,8	107719	COMP STRT MED EXT ASM 14"W R	Right- 80D- 14" W- MED
1B,2C,3-6,7B,8	107720	COMP STRT MED EXT ASM 15"W R	Right- 80D- 15" W- MED
1B,2D,3-6,7B,8	107721	COMP STRT MED EXT ASM 16"W R	Right- 80D- 16" W- MED
1B,2E,3-6,7C,8	107722	COMP STRT MED EXT ASM 17"W R	Right- 80D- 17" W- MED
1B,2F,3-6,7C,8	107723	COMP STRT MED EXT ASM 18"W R	Right- 80D- 18" W- MED
1B,2G,3-6,7D,8	107724	COMP STRT MED EXT ASM 19 & 20"W R	Right- 80D- 19 & 20" W- MED
1B,2H,3-6,7D,8	111524	COMP STRT MED EXT ASM 21 & 22"W R	Right- 80D- 21 & 22" W- MED
>>>>		80 DEGREE HANGERS - MED EXT - LEFT	
1B,2A,3-6,7A,8	107725	COMP STRT MED EXT ASM 13"W L	LEFT- 80D- 13" W- MED
1B,2B,3-6,7A,8	107726	COMP STRT MED EXT ASM 14"W L	LEFT- 80D- 14" W- MED
1B,2C,3-6,7B,8	107727	COMP STRT MED EXT ASM 15"W L	LEFT- 80D- 15" W- MED
1B,2D,3-6,7B,8	107728	COMP STRT MED EXT ASM 16"W L	LEFT- 80D- 16" W- MED
1B,2E,3-6,7C,8	107729	COMP STRT MED EXT ASM 17"W L	LEFT- 80D- 17" W- MED
1B,2F,3-6,7C,8	107730	COMP STRT MED EXT ASM 18"W L	LEFT- 80D- 18" W- MED
1B,2G,3-6,7D,8	107731	COMP STRT MED EXT ASM 19 & 20"W L	LEFT- 80D- 19 & 20" W- MED
1B,2H,3-6,7D,8	111525	COMP STRT MED EXT ASM 21 & 22"W L	LEFT- 80D- 21 & 22" W- MED
>>>>		80 DEGREE HANGERS - LONG EXT - RIGHT	
1C,2A,3-6,7A,8	107732	COMP STRT LONG EXT ASM 13"W R	Right- 80D- 13" W- LONG
1C,2B,3-6,7A,8	107733	COMP STRT LONG EXT ASM 14"W R	Right- 80D- 14" W- LONG
1C,2C,3-6,7B,8	107734	COMP STRT LONG EXT ASM 15"W R	Right- 80D- 15" W- LONG
1C,2D,3-6,7B,8	107735	COMP STRT LONG EXT ASM 16"W R	Right- 80D- 16" W- LONG
1C,2E,3-6,7C,8	107736	COMP STRT LONG EXT ASM 17"W R	Right- 80D- 17" W- LONG
1C,2F,3-6,7C,8	107737	COMP STRT LONG EXT ASM 18"W R	Right- 80D- 18" W- LONG
1C,2G,3-6,7D,8	107738	COMP STRT LONG EXT ASM 19 & 20"W R	Right- 80D- 19 & 20" W- LONG
1C,2H,3-6,7D,8	111526	COMP STRT LONG EXT ASM 21 & 22"W R	Right- 80D- 21 & 22" W- LONG
>>>>		80 DEGREE HANGERS - LONG EXT - LEFT	
1C,2A,3-6,7A,8	107739	COMP STRT LONG EXT ASM 13"W L	LEFT- 80D- 13" W- LONG
1C,2B,3-6,7A,8	107740	COMP STRT LONG EXT ASM 14"W L	LEFT- 80D- 14" W- LONG
1C,2C,3-6,7B,8	107741	COMP STRT LONG EXT ASM 15"W L	LEFT- 80D- 15" W- LONG
1C,2D,3-6,7B,8	107742	COMP STRT LONG EXT ASM 16"W L	LEFT- 80D- 16" W- LONG
1C,2E,3-6,7C,8	107743	COMP STRT LONG EXT ASM 17"W L	LEFT- 80D- 17" W- LONG
1C,2F,3-6,7C,8	107744	COMP STRT LONG EXT ASM 18"W L	LEFT- 80D- 18" W- LONG
1C,2G,3-6,7D,8	107745	COMP STRT LONG EXT ASM 19 & 20"W L	LEFT- 80D- 19 & 20" W- LONG
1C,2H,3-6,7D,8	111527	COMP STRT LONG EXT ASM 21 & 22"W L	LEFT- 80D- 21 & 22" W- LONG

NOTE: FOOTPLATE ASSEMBLIES ARE SET TO FOOTREST WIDTH. PLEASE SEE "FOOTPLATE FORMULA" TABLE FOR SEAT WIDTH CONVERSION.

INSET FORMULA


0" Inset (Seat Width) = Footrest Width
 1" Inset (Seat Width) - 2 = Footrest Width
 "V" Hanger (Seat Width) - 4 = Footrest Width

Composite Angle Adjustable "CAA" Footplate w/ Ext & HL for Swing I/O Hangers


(Order 1 each)

Ext. Tube Width	60°, 70° & 70° Hemi Style Hangers (Bent Extension)			80° Style hanger (Straight Extension)			70° V Style Hangers	
	Short	Medium	Long	Short	Medium	Long	Medium	Long
15" Right	107858	107872	107886	107816	107830	107844	107900	107914
15" Left	107865	107879	107893	107823	107837	107851	107907	107921
16" Right	107859	107873	107887	107817	107831	107845	107901	107915
16" Left	107866	107880	107894	107824	107838	107852	107908	107922
17" Right	107860	107874	107888	107818	107832	107846	107902	107916
17" Left	107867	107881	107895	107825	107839	107853	107909	107923
18" Right	107861	107875	107889	107819	107833	107847	107903	107917
18" Left	107868	107882	107896	107826	107840	107854	107910	107924
19" Right	107862	107876	107890	107820	107834	107848	V STYLE IS NOT	
19" Left	107869	107883	107897	107827	107841	107855		
20" Right	107863	107877	107891	107821	107835	107849		
20" Left	107870	107884	107898	107828	107842	107856		
21"-22" Rt	107864	107878	107892	107822	107836	107850		
21"-22" Lt	107871	107885	107899	107829	107843	107857		

FOOTPLATE MEASURING GUIDE

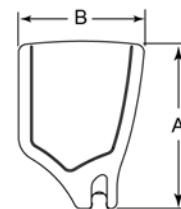

PART NO	DESCRIPTION	DIM -A-	DIM -B-
063216-006	FOOTPLATE COMPOSITE 157 R	6 1/8"	6"
063216-012	FOOTPLATE COMPOSITE 157 L	6 1/8"	6"
063216-015	FOOTPLATE COMPOSITE 170 R	6 5/8"	6"
063216-018	FOOTPLATE COMPOSITE 170 L	6 5/8"	6"
063216-005	FOOTPLATE COMPOSITE 182 R	7 1/8"	6"
063216-011	FOOTPLATE COMPOSITE 182 L	7 1/8"	6"
063216-014	FOOTPLATE COMPOSITE 195 R	7 5/8"	6"
063216-017	FOOTPLATE COMPOSITE 195 L	7 5/8"	6"
063216-004	FOOTPLATE COMPOSITE 207 R	8 1/8"	6"
063216-010	FOOTPLATE COMPOSITE 207 L	8 1/8"	6"
063216-013	FOOTPLATE COMPOSITE 220 R	8 5/8"	6 1/4"
063216-016	FOOTPLATE COMPOSITE 220 L	8 5/8"	6 1/4"
063216-003	FOOTPLATE COMPOSITE 232 R	9 1/8"	6 1/4"
063216-009	FOOTPLATE COMPOSITE 232 L	9 1/8"	6 1/4"
063216-002	FOOTPLATE COMPOSITE 257 R	10 1/8"	6 1/4"
063216-008	FOOTPLATE COMPOSITE 257 L	10 1/8"	6 1/4"
063216-001	FOOTPLATE COMPOSITE 282 R	11 1/8"	6 1/4"
063216-007	FOOTPLATE COMPOSITE 282 L	11 1/8"	6 1/4"

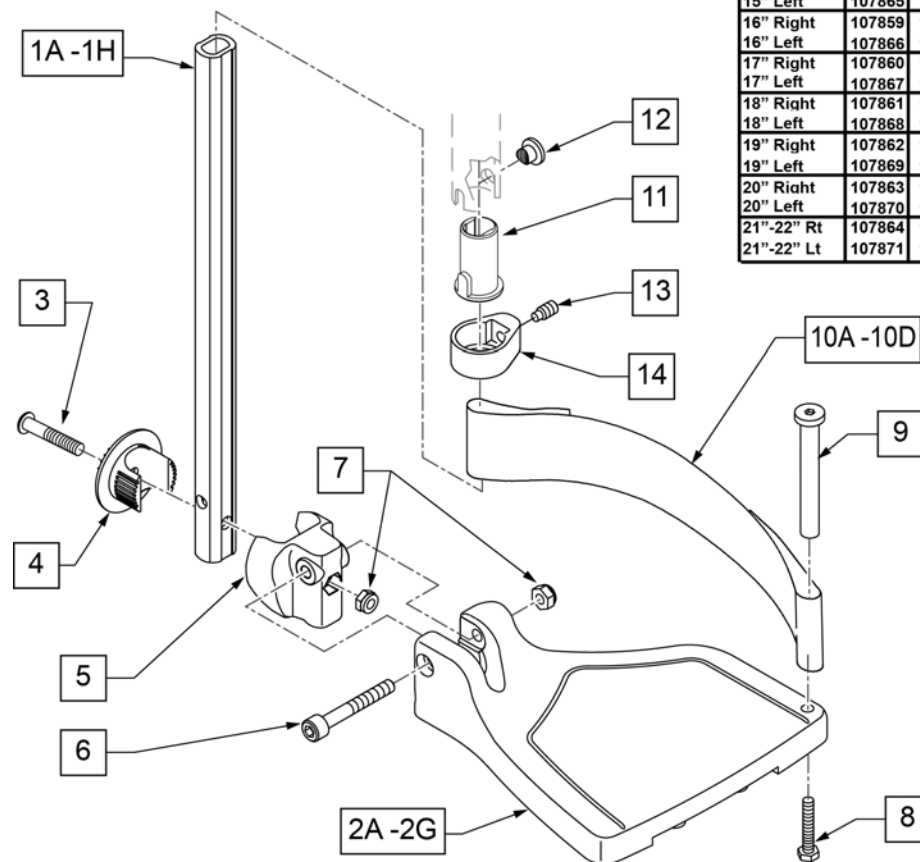

Pos.	Item Number	Description	Remarks
1A,6,7	120660	SHT STRT EXT COMP HDWR	80 deg hangers only
1B,6,7	120661	MED STRT EXT COMP HDWR	80 deg hangers only
1C,6,7	120662	LNG STRT EXT COMP HDWR	80 deg hangers only
1D,6,7	120663	SHT BNT EXT COMP HDWR	60,70,70 Hemi Hangers
1E,6,7	120664	MED BNT EXT COMP HDWR	60,70,70 Hemi Hangers
1F,6,7	120665	LNG BNT EXT COMP HDWR	60,70,70 Hemi Hangers
1G,6,7	120666	SHT V EXT R COMP HDWR	70 deg V hangers only-LEFT
1G,6,7	120667	SHT V EXT L COMP HDWR	70 deg V hangers only-RIGHT
1H,6,7	120668	LNG V EXT R COMP HDWR	70 deg V hangers only-LEFT
1H,6,7	120669	LNG V EXT L COMP HDWR	70 deg V hangers only-RIGHT
2A	063216-006	COMPOSITE FOOTPLATE RIGHT	RIGHT 15" Footrest Width
2A	063216-012	COMPOSITE FOOTPLATE LEFT	LEFT 15" Footrest Width
2B	063216-015	COMPOSITE FOOTPLATE RIGHT	RIGHT 16" Footrest Width

Pos.	Item Number	Description	Remarks
2B	063216-018	COMPOSITE FOOTPLATE LEFT	LEFT 16" Footrest Width
2C	063216-005	COMPOSITE FOOTPLATE RIGHT	RIGHT 17" Footrest Width
2C	063216-011	COMPOSITE FOOTPLATE LEFT	LEFT 17" Footrest Width
2D	063216-014	COMPOSITE FOOTPLATE RIGHT	RIGHT 18" Footrest Width
2D	063216-017	COMPOSITE FOOTPLATE LEFT	LEFT 18" Footrest Width
2E	063216-004	COMPOSITE FOOTPLATE RIGHT	RIGHT 19" Footrest Width
2E	063216-010	COMPOSITE FOOTPLATE LEFT	LEFT 19" Footrest Width
2F	063216-013	COMPOSITE FOOTPLATE RIGHT	RIGHT 20" Footrest Width
2F	063216-016	COMPOSITE FOOTPLATE LEFT	LEFT 20" Footrest Width
2G	063216-003	COMPOSITE FOOTPLATE RIGHT	RIGHT 21" & 22" Footrest Width
2G	063216-009	COMPOSITE FOOTPLATE LEFT	LEFT 21" & 22" Footrest Width
3	108233	BHCS M6 X 1 X 40	
4	054242	ANGLE COVER	
5	054241	ANGLE SEGMENT	
6	103175	SHCS M6 X 1 X 40 INORG	
7	104381	NUT HEX NYLOCK M6X1.0	
8	102268-001	HHCS M6X1X25	Inserted in bottom of footplate.
9	104328	HEEL LOOP PEG	
10A	104380-001	ADJUSTABLE FOOT REST STRAP	15" Footrest Width
10B	104380-002	ADJUSTABLE FOOT REST STRAP	16" Footrest Width
10C	104380-003	ADJUSTABLE FOOT REST STRAP	17-18" Footrest Width
10D	104380-004	ADJUSTABLE FOOT REST STRAP	19-22" Footrest Width
11	104160	BUSHING FOOTREST	
11-14	107485	FOOTREST CLAMP ASSM	
>>>>		60°,70°,70° HEMI HANGERS - SHORT EXT - RIGHT	
1D,2A,3-10A	107858	CAA BNT SHT EXT ASM 15W R	Right- 60° & 60° & 70° 15 W- SHORT
1D,2B,3-10B	107859	CAA BNT SHT EXT ASM 16W R	Right- 60° & 70° 16 W- SHORT
1D,2C,3-10B	107860	CAA BNT SHT EXT ASM 17W R	Right- 60° & 70° 17 W- SHORT
1D,2D,3-10C	107861	CAA BNT SHT EXT ASM 18W R	Right- 60° & 70° 18 W- SHORT
1D,2E,3-10C	107862	CAA BNT SHT EXT ASM 19W R	Right- 60° & 70° 19 W- SHORT
1D,2F,3-10D	107863	CAA BNT SHT EXT ASM 20W R	Right- 60° & 70° 20 W- SHORT
1D,2G,3-10D	107864	CAA BNT SHT EXT ASM 21&22W R	Right- 60° & 70° 21&22 W- SHORT
>>>>		60°,70°,70° HEMI HANGERS - SHORT EXT - LEFT	
1D,2A,3-10A	107865	CAA BNT SHT EXT ASM 15W L	Left- 60° & 70° 15 W- SHORT
1D,2B,3-10B	107866	CAA BNT SHT EXT ASM 16W L	Left- 60° & 70° 16 W- SHORT
1D,2C,3-10B	107867	CAA BNT SHT EXT ASM 17W L	Left- 60° & 70° 17 W- SHORT
1D,2D,3-10C	107868	CAA BNT SHT EXT ASM 18W L	Left- 60° & 70° 18 W- SHORT
1D,2E,3-10C	107869	CAA BNT SHT EXT ASM 19W L	Left- 60° & 70° 19 W- SHORT
1D,2F,3-10D	107870	CAA BNT SHT EXT ASM 20W L	Left- 60° & 70° 20 W- SHORT
1D,2G,3-10D	107871	CAA BNT SHT EXT ASM 21&22W L	Left- 60° & 70° 21&22 W- SHORT
>>>>		60°,70°,70° HEMI HANGERS - MED EXT - RIGHT	
1E,2A,3-10A	107872	CAA BNT MED EXT ASM 15W R	Right- 60° & 70° 15 W- MED
1E,2B,3-10B	107873	CAA BNT MED EXT ASM 16W R	Right- 60° & 70° 16 W- MED
1E,2C,3-10B	107874	CAA BNT MED EXT ASM 17W R	Right- 60° & 70° 17 W- MED
1E,2D,3-10C	107875	CAA BNT MED EXT ASM 18W R	Right- 60° & 70° 18 W- MED
1E,2E,3-10C	107876	CAA BNT MED EXT ASM 19W R	Right- 60° & 70° 19 W- MED
1E,2F,3-10D	107877	CAA BNT MED EXT ASM 20W R	Right- 60° & 70° 20 W- MED
1E,2G,3-10D	107878	CAA BNT MED EXT ASM 21&22W R	Right- 60° & 70° 21&22 W- MED
>>>>		60°,70°,70° HEMI HANGERS - MED EXT - LEFT	
1E,2A,3-10A	107879	CAA BNT MED EXT ASM 15W L	Left- 60° & 70° 15 W- MED
1E,2B,3-10B	107880	CAA BNT MED EXT ASM 16W L	Left- 60° & 70° 16 W- MED
1E,2C,3-10B	107881	CAA BNT MED EXT ASM 17W L	Left- 60° & 70° 17 W- MED
1E,2D,3-10C	107882	CAA BNT MED EXT ASM 18W L	Left- 60° & 70° 18 W- MED
1E,2E,3-10C	107883	CAA BNT MED EXT ASM 19W L	Left- 60° & 70° 19 W- MED
1E,2F,3-10D	107884	CAA BNT MED EXT ASM 20W L	Left- 60° & 70° 20 W- MED
1E,2G,3-10D	107885	CAA BNT MED EXT ASM 21&22W L	Left- 60° & 70° 21&22 W- MED

Pos.	Item Number	Description	Remarks
>>>>		60°,70°,70° HEMI HANGERS - LONG EXT - RIGHT	
1F,2A,3-10A	107886	CAA BNT LNG EXT ASM 15W R	Right- 60° & 70° 15 W- LONG
1F,2B,3-10B	107887	CAA BNT LNG EXT ASM 16W R	Right- 60° & 70° 16 W- LONG
1F,2C,3-10B	107888	CAA BNT LNG EXT ASM 17W R	Right- 60° & 70° 17 W- LONG
1F,2D,3-10C	107889	CAA BNT LNG EXT ASM 18W R	Right- 60° & 70° 18 W- LONG
1F,2E,3-10C	107890	CAA BNT LNG EXT ASM 19W R	Right- 60° & 70° 19 W- LONG
1F,2F,3-10D	107891	CAA BNT LNG EXT ASM 20W R	Right- 60° & 70° 20 W- LONG
1F,2G,3-10D	107892	CAA BNT LNG EXT ASM 21&22W R	Right- 60° & 70° 21&22 W- LONG
>>>>		60°,70°,70° HEMI HANGERS - LONG EXT - LEFT	
1F,2A,3-10A	107893	CAA BNT LNG EXT ASM 15W L	Left- 60° & 70° 15 W- LONG
1F,2B,3-10B	107894	CAA BNT LNG EXT ASM 16W L	Left- 60° & 70° 16 W- LONG
1F,2C,3-10B	107895	CAA BNT LNG EXT ASM 17W L	Left- 60° & 70° 17 W- LONG
1F,2D,3-10C	107896	CAA BNT LNG EXT ASM 18W L	Left- 60° & 70° 18 W- LONG
1F,2E,3-10C	107897	CAA BNT LNG EXT ASM 19W L	Left- 60° & 70° 19 W- LONG
1F,2F,3-10D	107898	CAA BNT LNG EXT ASM 20W L	Left- 60° & 70° 20 W- LONG
1F,2G,3-10D	107899	CAA BNT LNG EXT ASM 21&22W L	Left- 60° & 70° 21&22 W- LONG
>>>>		70° "V" HANGERS - MED EXT - RIGHT	
1G,2A,3-10A	107900	CAA V MED EXT ASM 15W R	Right- 70° V hangers only 15 W- MED
1G,2B,3-10B	107901	CAA V MED EXT ASM 16W R	Right- 70° V hangers only 16 W- MED
1G,2C,3-10B	107902	CAA V MED EXT ASM 17W R	Right- 70° V hangers only 17 W- MED
1G,2D,3-10C	107903	CAA V MED EXT ASM 18W R	Right- 70° V hangers only 18 W- MED
>>>>		70° "V" HANGERS - MED EXT - LEFT	
1G,2A,3-10A	107907	CAA V MED EXT ASM 15W L	Left- 70° V hangers only 15 W- MED
1G,2B,3-10B	107908	CAA V MED EXT ASM 16W L	Left- 70° V hangers only 16 W- MED
1G,2C,3-10B	107909	CAA V MED EXT ASM 17W L	Left- 70° V hangers only 17 W- MED
1G,2D,3-10C	107910	CAA V MED EXT ASM 18W L	Left- 70° V hangers only 18 W- MED
>>>>		70° "V" HANGERS - LONG EXT - RIGHT	
1H,2A,3-10A	107914	CAA V LNG EXT ASM 15W R	Right- 70° V hangers only 15 W- LONG
1H,2B,3-10B	107915	CAA V LNG EXT ASM 16W R	Right- 70° V hangers only 16 W- LONG
1H,2C,3-10B	107916	CAA V LNG EXT ASM 17W R	Right- 70° V hangers only 17 W- LONG
1H,2D,3-10C	107917	CAA V LNG EXT ASM 18W R	Right- 70° V hangers only 18 W- LONG
>>>>		70° "V" HANGERS - LONG EXT - LEFT	
1H,2A,3-10A	107921	CAA V LNG EXT ASM 15W L	Left- 70° V hangers only 15 W- LONG
1H,2B,3-10B	107922	CAA V LNG EXT ASM 16W L	Left- 70° V hangers only 16 W- LONG
1H,2C,3-10B	107923	CAA V LNG EXT ASM 17W L	Left- 70° V hangers only 17 W- LONG
1H,2D,3-10C	107924	CAA V LNG EXT ASM 18W L	Left- 70° V hangers only 18 W- LONG
>>>>		80° HANGERS - SHORT EXT - RIGHT	
1A,2A,3-10A	107816	CAA STRT SHT EXT ASM 15W R	Right- 80° 15 W- SHORT
1A,2B,3-10B	107817	CAA STRT SHT EXT ASM 16W R	Right- 80° 16 W- SHORT
1A,2C,3-10B	107818	CAA STRT SHT EXT ASM 17W R	Right- 80° 17 W- SHORT
1A,2D,3-10C	107819	CAA STRT SHT EXT ASM 18W R	Right- 80° 18 W- SHORT
1A,2E,3-10C	107820	CAA STRT SHT EXT ASM 19W R	Right- 80° 19 W- SHORT
1A,2F,3-10D	107821	CAA STRT SHT EXT ASM 20W R	Right- 80° 20 W- SHORT
1A,2G,3-10D	107822	CAA STRT SHT EXT ASM 21&22W R	Right- 80° 21&22 W- SHORT
>>>>		80° HANGERS - SHORT EXT - LEFT	
1A,2A,3-10A	107823	CAA STRT SHT EXT ASM 15W L	Left- 80° 15 W- SHORT
1A,2B,3-10B	107824	CAA STRT SHT EXT ASM 16W L	Left- 80° 16 W- SHORT
1A,2C,3-10B	107825	CAA STRT SHT EXT ASM 17W L	Left- 80° 17 W- SHORT
1A,2D,3-10C	107826	CAA STRT SHT EXT ASM 18W L	Left- 80° 18 W- SHORT
1A,2E,3-10C	107827	CAA STRT SHT EXT ASM 19W L	Left- 80° 19 W- SHORT
1A,2F,3-10D	107828	CAA STRT SHT EXT ASM 20W L	Left- 80° 20 W- SHORT
1A,2G,3-10D	107829	CAA STRT SHT EXT ASM 21&22W L	Left- 80° 21&22 W- SHORT
>>>>		80° HANGERS - MED EXT - RIGHT	
1B,2A,3-10A	107830	CAA STRT MED EXT ASM 15W R	Right- 80° 15 W- MED
1B,2B,3-10B	107831	CAA STRT MED EXT ASM 16W R	Right- 80° 16 W- MED
1B,2C,3-10B	107832	CAA STRT MED EXT ASM 17W R	Right- 80° 17 W- MED


Pos.	Item Number	Description	Remarks
1B,2D,3-10C	107833	CAA STRT MED EXT ASM 18W R	Right- 80° 18 W- MED
1B,2E,3-10C	107834	CAA STRT MED EXT ASM 19W R	Right- 80° 19 W- MED
1B,2F,3-10D	107835	CAA STRT MED EXT ASM 20W R	Right- 80° 20 W- MED
1B,2G,3-10D	107836	CAA STRT MED EXT ASM 21&22W R	Right- 80° 21&22 W- MED
>>>>		80° HANGERS - MED EXT - LEFT	
1B,2A,3-10A	107837	CAA STRT MED EXT ASM 15W L	Left- 80° 15 W- MED
1B,2B,3-10B	107838	CAA STRT MED EXT ASM 16W L	Left- 80° 16 W- MED
1B,2C,3-10B	107839	CAA STRT MED EXT ASM 17W L	Left- 80° 17 W- MED
1B,2D,3-10C	107840	CAA STRT MED EXT ASM 18W L	Left- 80° 18 W- MED
1B,2E,3-10C	107841	CAA STRT MED EXT ASM 19W L	Left- 80° 19 W- MED
1B,2F,3-10D	107842	CAA STRT MED EXT ASM 20W L	Left- 80° 20 W- MED
1B,2G,3-10D	107843	CAA STRT MED EXT ASM 21&22W L	Left- 80° 21&22 W- MED
>>>>		80° HANGERS - LONG EXT - RIGHT	
1C,2A,3-10A	107844	CAA STRT LNG EXT ASM 15W R	Right- 80° 15 W- LONG
1C,2B,3-10B	107845	CAA STRT LNG EXT ASM 16W R	Right- 80° 16 W- LONG
1C,2C,3-10B	107846	CAA STRT LNG EXT ASM 17W R	Right- 80° 17 W- LONG
1C,2D,3-10C	107847	CAA STRT LNG EXT ASM 18W R	Right- 80° 18 W- LONG
1C,2E,3-10C	107848	CAA STRT LNG EXT ASM 19W R	Right- 80° 19 W- LONG
1C,2F,3-10D	107849	CAA STRT LNG EXT ASM 20W R	Right- 80° 20 W- LONG
1C,2G,3-10D	107850	CAA STRT LNG EXT ASM 21&22W R	Right- 80° 21&22 W- LONG
>>>>		80° HANGERS - LONG EXT - LEFT	
1C,2A,3-10A	107851	CAA STRT LNG EXT ASM 15W L	Left- 80° 15 W- LONG
1C,2B,3-10B	107852	CAA STRT LNG EXT ASM 16W L	Left- 80° 16 W- LONG
1C,2C,3-10B	107853	CAA STRT LNG EXT ASM 17W L	Left- 80° 17 W- LONG
1C,2D,3-10C	107854	CAA STRT LNG EXT ASM 18W L	Left- 80° 18 W- LONG
1C,2E,3-10C	107855	CAA STRT LNG EXT ASM 19W L	Left- 80° 19 W- LONG
1C,2F,3-10D	107856	CAA STRT LNG EXT ASM 20W L	Left- 80° 20 W- LONG
1C,2G,3-10D	107857	CAA STRT LNG EXT ASM 21&22W L	Left- 80° 21&22 W- LONG

NOTE: FOOTPLATE ASSEMBLIES ARE SET TO FOOTREST WIDTH. PLEASE SEE "FOOTPLATE FORMULA" TABLE FOR SEAT WIDTH CONVERSION.


Adult Angle Adjustable "AAA" Footplates w/ Ext & HL for Swing I/O Hangers								
(Order 1 each)								
Ext. Tube	60°, 70° & 70° Hemi Style Hangers (Bent Extension)			80° Style hanger (Straight Extension)			70° V Style Hangers	
Width	Short	Med	Long	Short	Med	Long	Med	Long
10"-11" Rt	NOT AVAILABLE IN THIS WIDTH			NOT AVAILABLE IN THIS WIDTH			111546	111550
10"-11" Lt	NOT AVAILABLE IN THIS WIDTH			NOT AVAILABLE IN THIS WIDTH			111547	111551
12-13" Rt	111540	111542	111544	111534	111536	111538	111548	111552
12-13" Lt	111541	111543	111545	111535	111537	111539	111549	111553
14"-15" Rt	107973	107987	108001	107928	107942	107956	108015	108029
14"-15" Lt	107980	107994	108008	107935	107949	107963	108022	108036
16"-17" Rt	107975	107989	108003	107930	107944	107958	108017	108031
16"-17" Lt	107982	107996	108010	107937	107951	107965	108024	108038
18"-22" Rt	107977	107991	108005	107932	107946	107960	108019	108033
18"-22" Lt	107984	107998	108012	107939	107953	107968	108026	108040

INSET FORMULA
 0" Inset (Seat Width) = Footrest Width
 1" Inset (Seat Width) - 2 = Footrest Width
 "V" Hanger (Seat Width) - 4 = Footrest Width

MEASURING GUIDE

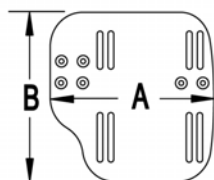


CHAIR WIDTH	"A"	"B"
10" - 11"	3 1/2"	7 1/4"
12" - 13"	4 1/2"	7 3/4"
14" - 15"	5 1/2"	7 3/4"
16" - 17"	6 1/2"	7 3/4"
18" - 20"	7 1/2"	7 3/4"

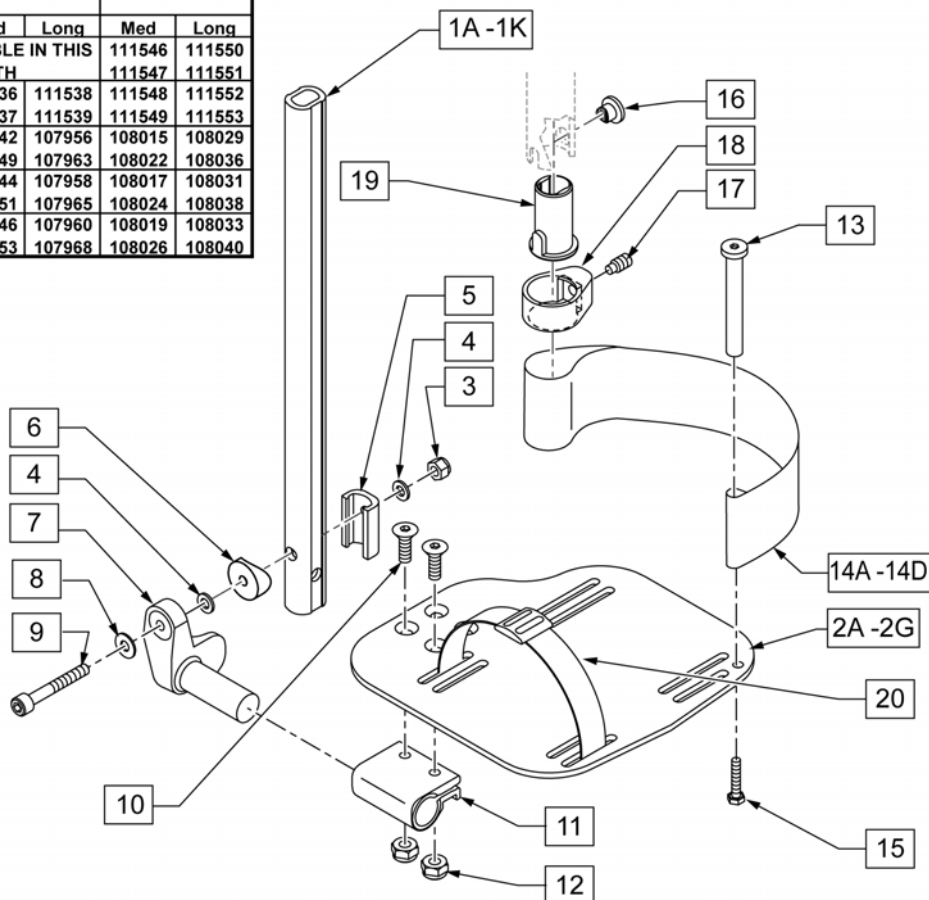

Pos.	Item Number	Description	Remarks
1A,3-5,8,9	120670	SHT STRT EXT AAA HDWR	80 deg Hangers Only
1B,3-5,8,9	120671	MED STRT EXT AAA HDWR	80 deg Hangers Only
1C,3-5,8,9	120672	LNG STRT EXT AAA HDWR	80 deg Hangers Only
1D,3-5,8,9	120673	SHT BNT EXT AAA HDWR	60,70,70 Hemi Hangers
1E,3-5,8,9	120674	MED BNT EXT AAA HDWR	60,70,70 Hemi Hangers
1F,3-5,8,9	120675	LNG BNT EXT AAA HDWR	60,70,70 Hemi Hangers
1G,3-5,8,9	120676	SHT V EXT R AAA HDWR	Left - 70 deg V Hangers Only
1H,3-5,8,9	120677	SHT V EXT L AAA HDWR	Right - 70 deg V Hangers Only
1I,3-5,8,9	120678	LNG V EXT R AAA HDWR	Left - 70 deg V Hangers Only
1J,3-5,8,9	120679	LNG V EXT L AAA HDWR	Right - 70 deg V Hangers Only
2A	500656	ADULT ANG ADJ F/PLT 10/11" R	RIGHT 10-11" Footrest Width
2A	500657	ADULT ANG ADJ F/PLT 10/11" L	LEFT 10-11" Footrest Width
2B	500658	ADULT ANG ADJ F/PLT 12/13" R	RIGHT 12-13" Footrest Width



Pos.	Item Number	Description	Remarks
2B	500659	ADULT ANG ADJ F/PLT 12/13" L	LEFT 12-13" Footrest Width
2C	500660	ADLT ANGL ADJ F/PLT 14/15	RIGHT 14-15" Footrest Width
2C	500661	ADLT ANGL ADJ F/PLT 14/15	LEFT 14-15" Footrest Width
2D	500662	ADLT ANGL ADJ F/PLT 16/17	RIGHT 16-17" Footrest Width
2D	500663	ADLT ANGL ADJ F/PLT 16/17	LEFT 16-17" Footrest Width
2E	500664	ADLT ANGL ADJ F/PLT 18-20	RIGHT 18-22" Footrest Width
2E	500665	ADLT ANGL ADJ F/PLT 18-20	LEFT 18-22" Footrest Width
3	104381	NUT HEX NYLOCK M6X1.0	
4	106944	WASHER FLAT THIN AN 1/4 ROHS	
5	107199	HANGER EXTENSION SADDLE	
6	104377	3/4" SADDLE THIN	
7	541002	FTRST MOUNTING BRKT-3/8 RAD R	RIGHT
7	541003	FTRST MOUNTING BRKT-3/8 RAD L	LEFT
8	106945	WASHER FLAT SAE 1/4 ROHS	
9	104399	SHCS M6X1X45 12 MM PART THD	
10	106978	FHCS 1/4-28 X 7/8 ROHS	
11	550350	3/4" CLMP ADJ FT PLT	
12	106935	NUT NYLOCK THIN 1/4-28 ROHS	
13	104328	POST HEEL LOOP	
14A	104380-001	ADJUSTABLE FOOT REST STRAP	12 - 14" WIDE
14B	104380-002	ADJUSTABLE FOOT REST STRAP	15 - 16" WIDE
14C	104380-003	ADJUSTABLE FOOT REST STRAP	17 - 18" WIDE
14D	104380-004	ADJUSTABLE FOOT REST STRAP	19 - 20" WIDE
15	102268-001	HHCS M6 X 1 X 25	
3-6,8,9	120659	EXT MNT HARDWARE 1 SIDE	
16-19	107485	FOOTREST CLAMP ASSM	
20	740260	TOE-ANKLE STRAP ADJ ADULT FOOTPLATE	
>>>>		80° HANGERS - SHORT EXT - RIGHT	
1A,2B,3-14A,15	111534	AAA STRT SHT EXT ASM 12-13" W R	Right- 80D- 12-13" W- SHORT
1A,2C,3-14B,15	107928	AAA STRT SHT EXT ASM 14-15" W R	Right- 80D- 14-15" W- SHORT
1A,2D,3-14C,15	107930	AAA STRT SHT EXT ASM 16-17" W R	Right- 80D- 16-17" W- SHORT
1A,2E,3-14D,15	107932	AAA STRT SHT EXT ASM 18-22" W R	Right- 80D- 18-22" W- SHORT
>>>>		80° HANGERS - SHORT EXT - LEFT	
1A,2B,3-14A,15	111535	AAA STRT SHT EXT ASM 12-13" W L	Left- 80D- 12-13" W- SHORT
1A,2C,3-14B,15	107935	AAA STRT SHT EXT ASM 14" W L	Left- 80D- 14-15" W- SHORT
1A,2D,3-14C,15	107937	AAA STRT SHT EXT ASM 16" W L	Left- 80D- 16-17" W- SHORT
1A,2E,3-14D,15	107939	AAA STRT SHT EXT ASM 18" W L	Left- 80D- 18-22" W- SHORT
>>>>		80° HANGERS - MED EXT - RIGHT	
1B,2B,3-14A,15	111536	AAA STRT MED EXT ASM 12-13" W R	Right- 80D- 12-13" W- MED
1B,2C,3-14B,15	107942	AAA STRT MED EXT ASM 14-15" W R	Right- 80D- 14-15" W- MED
1B,2D,3-14C,15	107944	AAA STRT MED EXT ASM 16-17" W R	Right- 80D- 16-17" W- MED
1B,2E,3-14D,15	107946	AAA STRT MED EXT ASM 18-22" W R	Right- 80D- 18-22" W- MED
>>>>		80° HANGERS - MED EXT - LEFT	
1B,2B,3-14A,15	111537	AAA STRT MED EXT ASM 12-13" W L	Left- 80D- 12-13" W- MED
1B,2C,3-14B,15	107949	AAA STRT MED EXT ASM 14" W L	Left- 80D- 14-15" W- MED
1B,2D,3-14C,15	107951	AAA STRT MED EXT ASM 16" W L	Left- 80D- 16-17" W- MED
1B,2E,3-14D,15	107953	AAA STRT MED EXT ASM 18" W L	Left- 80D- 18-22" W- MED
>>>>		80° HANGERS - LONG EXT - RIGHT	
1C,2B,3-14A,15	111538	AAA STRT LNG EXT ASM 12-13" W R	Right- 80D- 12-13" W- LONG
1C,2C,3-14B,15	107956	AAA STRT LNG EXT ASM 14 -15" W R	Right- 80D- 14-15" W- LONG
1C,2D,3-14C,15	107958	AAA STRT LNG EXT ASM 16 -17" W R	Right- 80D- 16-17" W- LONG
1C,2E,3-14D,15	107960	AAA STRT LNG EXT ASM 18 -22" W R	Right- 80D- 18-22" W- LONG
>>>>		80° HANGERS - LONG EXT - LEFT	
1C,2B,3-14A,15	111539	AAA STRT LNG EXT ASM 12-13" W L	Left- 80D- 12-13" W- LONG
1C,2C,3-14B,15	107963	AAA STRT LNG EXT ASM 14W L	Left- 80D- 14-15" W- LONG
1C,2D,3-14C,15	107965	AAA STRT LNG EXT ASM 16W L	Left- 80D- 16-17" W- LONG

Pos.	Item Number	Description	Remarks
1C,2E,3-14D,15	107968	AAA STRT LNG EXT ASM 18W L	Left- 80D- 18-22" W- LONG
>>>>		70° HANGERS - SHORT EXT - RIGHT	
1D,2B,3-14A,15	111540	AAA BNT SHT EXT ASM 12-13" W R	Right- 60D,70D,70D HEMI - 12-13" W- SHORT
1D,2C,3-14B,15	107973	AAA BNT SHT EXT ASM 14 -15" W R	Right- 60D,70D,70D HEMI - 14-15" W- SHORT
1D,2D,3-14C,15	107975	AAA BNT SHT EXT ASM 16 -17" W R	Right- 60D,70D,70D HEMI - 16-17" W- SHORT
1D,2E,3-14D,15	107977	AAA BNT SHT EXT ASM 18 -22" W R	Right- 60D,70D,70D HEMI - 18-22" W- SHORT
>>>>		70° HANGERS - SHORT EXT - LEFT	
1D,2B,3-14A,15	111541	AAA BNT SHT EXT ASM 12-13" W L	Left- 60D,70D,70D HEMI - 12-13" W- SHORT
1D,2C,3-14B,15	107980	AAA BNT SHT EXT ASM 14 -15" W L	Left- 60D,70D,70D HEMI - 14-15" W- SHORT
1D,2D,3-14C,15	107982	AAA BNT SHT EXT ASM 16 -17" W L	Left- 60D,70D,70D HEMI - 16-17" W- SHORT
1D,2E,3-14D,15	107984	AAA BNT SHT EXT ASM 18 -22" W L	Left- 60D,70D,70D HEMI - 18-22" W- SHORT
>>>>		70° HANGERS - MED EXT - RIGHT	
1E,2B,3-14A,15	111542	AAA BNT MED EXT ASM 12-13" W R	Right- 60D,70D,70D HEMI - 12-13" W- MED
1E,2C,3-14B,15	107987	AAA BNT MED EXT ASM 14 -15" W R	Right- 60D,70D,70D HEMI - 14-15" W- MED
1E,2D,3-14C,15	107989	AAA BNT MED EXT ASM 16 -17" W R	Right- 60D,70D,70D HEMI - 16-17" W- MED
1E,2E,3-14D,15	107991	AAA BNT MED EXT ASM 18 -22" W R	Right- 60D,70D,70D HEMI - 18-22" W- MED
>>>>		70° HANGERS - MED EXT - LEFT	
1E,2B,3-14A,15	111543	AAA BNT MED EXT ASM 12-13" W L	Left- 60D,70D,70D HEMI - 12-13" W- MED
1E,2C,3-14B,15	107994	AAA BNT MED EXT ASM 14 -15" W L	Left- 60D,70D,70D HEMI - 14-15" W- MED
1E,2D,3-14C,15	107996	AAA BNT MED EXT ASM 16 -17" W L	Left- 60D,70D,70D HEMI - 16-17" W- MED
1E,2E,3-14D,15	107998	AAA BNT MED EXT ASM 18 -22" W L	Left- 60D,70D,70D HEMI - 18-22" W- MED
>>>>		70° HANGERS - LONG EXT - RIGHT	
1F,2B,3-14A,15	111544	AAA BNT LONG EXT ASM 12-13" W R	Right- 60D,70D,70D HEMI - 12-13" W- LONG
1F,2C,3-14B,15	108001	AAA BNT LONG EXT ASM 14 -15" W R	Right- 60D,70D,70D HEMI - 14-15" W- LONG
1F,2D,3-14C,15	108003	AAA BNT LONG EXT ASM 16 -17" W R	Right- 60D,70D,70D HEMI - 16-17" W- LONG
1F,2E,3-14D,15	108005	AAA BNT LONG EXT ASM 18 -22" W R	Right- 60D,70D,70D HEMI - 18-22" W- LONG
>>>>		70° HANGERS - LONG EXT - LEFT	
1F,2B,3-14A,15	111545	AAA BNT LONG EXT ASM 12-13" W L	Left- 60D,70D,70D HEMI - 12-13" W- LONG
1F,2C,3-14B,15	108008	AAA BNT LONG EXT ASM 14 -15" W L	Left- 60D,70D,70D HEMI - 14-15" W- LONG
1F,2D,3-14C,15	108010	AAA BNT LONG EXT ASM 16 -17" W L	Left- 60D,70D,70D HEMI - 16-17" W- LONG
1F,2E,3-14D,15	108012	AAA BNT LONG EXT ASM 18 -22" W L	Left- 60D,70D,70D HEMI - 18-22" W- LONG
>>>>		70° "V" HANGERS - MED EXT - RIGHT	
1G,2A,3-15	111546	AAA V MED EXT ASM 10-11" W R	Right- 70D V hangers only 10-11" W- MED
1G,2B,3-15	111548	AAA V MED EXT ASM 12-13" W R	Right- 70D V hangers only 12-13" W- MED
1G,2C,3-15	108015	AAA V MED EXT ASM 14 -15" W R	Right- 70D V hangers only 14-15" W- MED
1G,2D,3-15	108017	AAA V MED EXT ASM 16 -17" W R	Right- 70D V hangers only 16-18" W- MED
1G,2E,3-15	108019	AAA V MED EXT ASM 18 -18" W R	Right- 70D V hangers only 18-22" W- MED
>>>>		70° "V" HANGERS - MED EXT - LEFT	
1G,2A,3-15	111547	AAA V MED EXT ASM 10-11" W L	Left- 70D V hangers only 10-11" W- MED
1G,2B,3-15	111549	AAA V MED EXT ASM 12-13" W L	Left- 70D V hangers only 12-13" W- MED
1G,2C,3-15	108022	AAA V MED EXT ASM 14 -15" W L	Left- 70D V hangers only 14-15" W- MED
1G,2D,3-15	108024	AAA V MED EXT ASM 16 -17" W L	Left- 70D V hangers only 16-18" W- MED
1G,2E,3-15	108026	AAA V MED EXT ASM 18 -22" W L	Left- 70D V hangers only 18-22" W- MED
>>>>		70° "V" HANGERS - LONG EXT - RIGHT	
1H,2A,3-15	111550	AAA V LONG EXT ASM 10-11" W R	Right- 70D V hangers only 10-11" W- LONG
1H,2B,3-15	111552	AAA V LONG EXT ASM 12-13" W R	Right- 70D V hangers only 12-13" W- LONG
1H,2C,3-15	108029	AAA V LONG EXT ASM 14 -15" W R	Right- 70D V hangers only 14-15" W- LONG
1H,2D,3-15	108031	AAA V LONG EXT ASM 16 -17" W R	Right- 70D V hangers only 16-18" W- LONG
1H,2E,3-15	108033	AAA V LONG EXT ASM 18 -22" W R	Right- 70D V hangers only 18-22" W- LONG
>>>>		70° "V" HANGERS - LONG EXT - LEFT	
1H,2A,3-15	111551	AAA V LONG EXT ASM 10-11" W L	Left- 70D V hangers only 10-11" W- LONG
1H,2B,3-15	111553	AAA V LONG EXT ASM 12-13" W L	Left- 70D V hangers only 12-13" W- LONG
1H,2C,3-15	108036	AAA V LONG EXT ASM 14 -15" W L	Left- 70D V hangers only 14-15" W- LONG
1H,2D,3-15	108038	AAA V LONG EXT ASM 16 -17" W L	Left- 70D V hangers only 16-18" W- LONG
1H,2E,3-15	108040	AAA V LONG EXT ASM 18 -22" W L	Left- 70D V hangers only 18-22" W- LONG


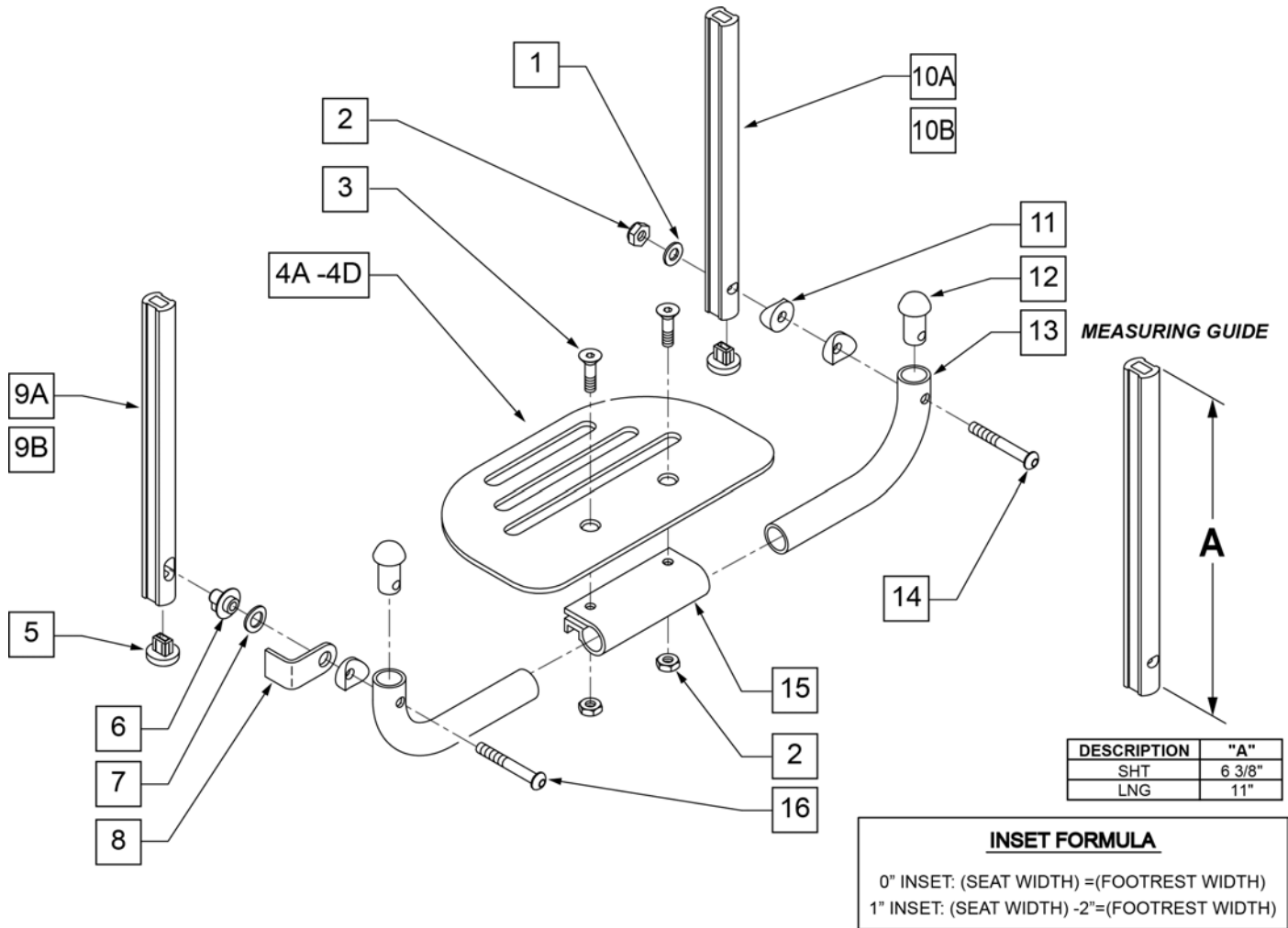


Pos.	Item Number	Description	Remarks
1	106944	WASHER FLAT THIN AN 1/4 ROHS	
2	106941	NUT AUTOMATION 1/4-28 ROHS	
3	106978	FHCS 1/4-28 X 7/8 ROHS	
4A	500651	PLATFORM F/PLT 8 x 5 1/2"	12 -14" Footrest Width
4B	500652	PLATFORM F/PLT 10" x 5 1/2"	15 -17" Footrest Width
4C	500653	PLATFORM F/PLT 12" x 5 1/2"	18 -20" Footrest Width
4D	500654	PLATFORM F/PLT 14" x 5 1/2"	21 -22" Footrest Width
5	113221	PLUG DOUBLE D EXT TUBE	
6	540410	F/UP FOOTREST KEY STUD D2	
7	107385	WASHER CURVED 3/8 ROHS	
8	505078	FOOTREST RETAINING BRACKET	
9A	106875	EXT TBE F/UP KEY SHT	Short -Latch Side
9B	106876	EXT TBE F/UP KEY LNG	Long -Latch Side
10A	106873	EXT TBE F/UP SLT SHT	Short -Pivot Side
10B	106874	EXT TBE F/UP SLT LNG	Long -Pivot Side
11	561003	3/4" SADDLE THIN	
12	561034	SHORT DOME PLUG	
14	106964	BHCS 1/4-28 X 2 ROHS	
15	550351	PLATFORM FOOTPLATE CLAMP	


Pos.	Item Number	Description	Remarks
16	107027	BHCS 1/4-28 X 1-1/4 SHLDR ROHS	
17A	740130	LEG STRAP 10"-14" BLK	12-14" Footrest Width
17B	740131	LEG STRAP 15"-17" BLK	15-17" Footrest Width
17C	740132	LEG STRAP 18"-20" BLK	18-20" Footrest Width
1-4A,6-8,11-16	900306	PLTFM F/P PLT BLK 12" D2 W/ INSTR	12" Footrest Width
1-4A,6-8,11-16	900307	PLTFM F/P PLT BLK 13" D2 W/ INSTR	13" Footrest Width
1-4A,6-8,11-16	900308	PLTFM F/P PLT BLK 14" D2 W /INSTR	14" Footrest Width
1-4B,6-8,11-16	900309	PLTFM F/P PLT BLK 15" D2 W /INSTR	15" Footrest Width
1-4B,6-8,11-16	900310	PLTFM F/P PLT BLK 16" D2 W /INSTR	16" Footrest Width
1-4B,6-8,11-16	900311	PLTFM F/P PLT BLK 17" D2 W /INSTR	17" Footrest Width
1-4C,6-8,11-16	900312	PLTFM F/P PLT BLK 18" D2 W /INSTR	18" Footrest Width
1-4C,6-8,11-16	900313	PLTFM F/P PLT BLK 19" D2 W /INSTR	19" Footrest Width
1-4C,6-8,11-16	900314	PLTFM F/P PLT BLK 20" D2 W/ INSTR	20" Footrest Width
1-4D,6-8,11-16	900315	PLTFM F/P PLT BLK 21" D2 W/ INSTR	21" Footrest Width
1-4D,6-8,11-16	900316	PLTFM F/P PLT BLK 22" D2 W/ INSTR	22" Footrest Width
		SHORT EXT ASSEMBLIES	
1-3,4A,5-8,9A,10B,11-16	108043	PLTFRM FLIP FTRST SHT 12W	12" wide (Short)
1-3,4A,5-8,9A,10B,11-16	108044	PLTFRM FLIP FTRST SHT 13W	13" wide (Short)
1-3,4A,5-8,9A,10B,11-16	108045	PLTFRM FLIP FTRST SHT 14W	14" wide (Short)
1-3,4B,5-8,9A,10B,11-16	108046	PLTFRM FLIP FTRST SHT 15W	15" wide (Short)
1-3,4B,5-8,9A,10B,11-16	108047	PLTFRM FLIP FTRST SHT 16W	16" wide (Short)
1-3,4B,5-8,9A,10B,11-16	108048	PLTFRM FLIP FTRST SHT 17W	17" wide (Short)
1-3,4C,5-8,9A,10B,11-16	108049	PLTFRM FLIP FTRST SHT 18W	18" wide (Short)
1-3,4C,5-8,9A,10B,11-16	108051	PLTFRM FLIP FTRST SHT 19W	19" wide (Short)
1-3,4C,5-8,9A,10B,11-16	108052	PLTFRM FLIP FTRST SHT 20W	20" wide (Short)
1-3,4D,5-8,9A,10B,11-16	111554	PLTFRM FLIP FTRST SHT 21W	21" wide (Short)
1-3,4D,5-8,9A,10B,11-16	111555	PLTFRM FLIP FTRST SHT 22W	22" wide (Short)
		LONG EXT ASSEMBLIES	
1-3,4A,5-8,9B,10A,11-16	108053	PLTFRM FLIP FTRST LONG 12W	12" wide (Long)
1-3,4A,5-8,9B,10A,11-16	108054	PLTFRM FLIP FTRST LONG 13W	13" wide (Long)
1-3,4A,5-8,9B,10A,11-16	108055	PLTFRM FLIP FTRST LONG 14W	14" wide (Long)
1-3,4B,5-8,9B,10A,11-16	108056	PLTFRM FLIP FTRST LONG 15W	15" wide (Long)
1-3,4B,5-8,9B,10A,11-16	108057	PLTFRM FLIP FTRST LONG 16W	16" wide (Long)
1-3,4B,5-8,9B,10A,11-16	108058	PLTFRM FLIP FTRST LONG 17W	17" wide (Long)
1-3,4C,5-8,9B,10A,11-16	108059	PLTFRM FLIP FTRST LONG 18W	18" wide (Long)
1-3,4C,5-8,9B,10A,11-16	108060	PLTFRM FLIP FTRST LONG 19W	19" wide (Long)
1-3,4C,5-8,9B,10A,11-16	108061	PLTFRM FLIP FTRST LONG 20W	20" wide (Long)
1-3,4D,5-8,9B,10A,11-16	111556	PLTFRM FLIP FTRST LONG 21W	21" wide (Long)
1-3,4D,5-8,9B,10A,11-16	111557	PLTFRM FLIP FTRST LONG 22W	22" wide (Long)

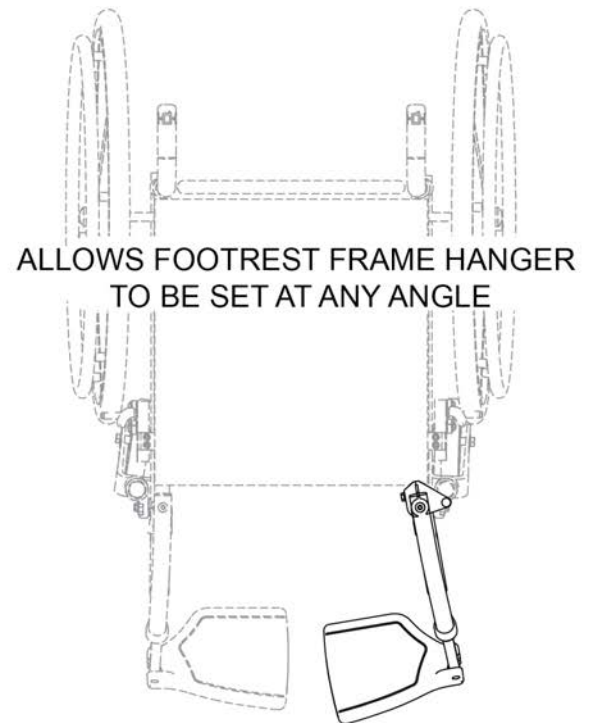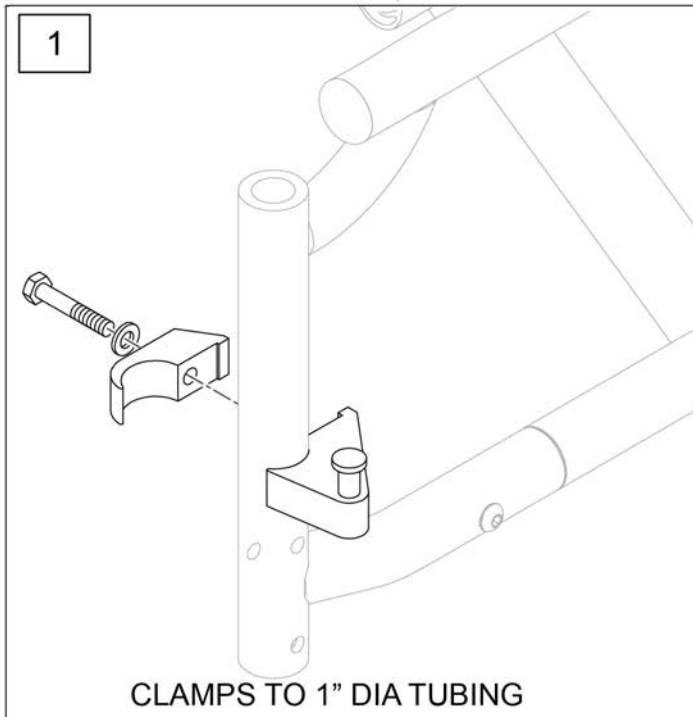

Pos.	Item Number	Description	Remarks
1	112807	LATCH STUD SUPPORT R MULTIPOS	Single
1	112808	LATCH STUD SUPPORT L MULTIPOS	Single
1	112809	LATCH STUD SUPPORT PR MULTIPOS	Pair

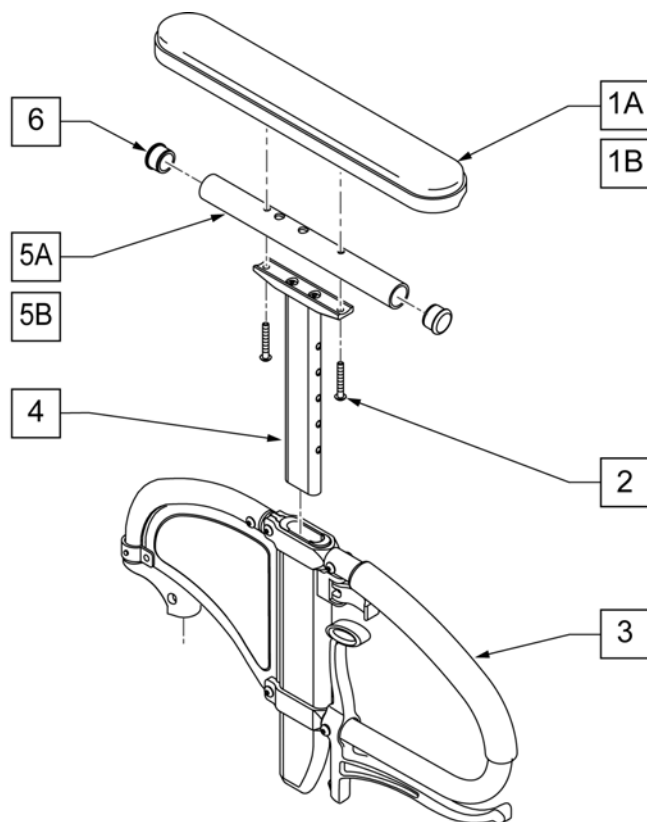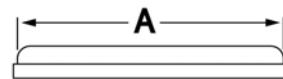
**MEASURING GUIDE**

	DESK	FULL
A	10"	14"

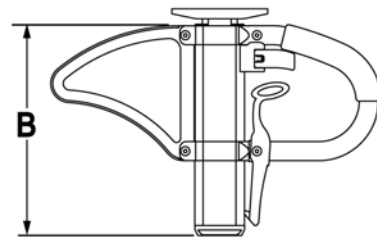

	STD
B	9.75"

Pos.	Item Number	Description	Remarks
1A	561024	ARMREST PAD 10" DESK	CLASSIC DESK
1B	561031	ARMREST PAD 14" FULL	CLASSIC FULL
2	107024	10-32 X 1 1/2 BH C/S BLK	
3,4	109405	SPHA ARMREST ASSM RIGHT	RIGHT
3,4	109400	SPHA ARMREST ASSM LEFT	LEFT
4	116313	SPHA ARM PAD MOUNT	
5A	521812	HT ADJ PD TUBE 10IN DESK POWE	
5B	521813	HT ADJ PD TUBE 14IN FULL POWE	
6	560735	7/8" ROUND TUBE PLUG	
1A,2,4,5A,6	115221	SPHA UPPER CLASSIC DESK PAD	Classic-Desk
1B,2,4,5B,6	115222	SPHA UPPER CLASSIC FULL PAD	Classic-Full
1A-5A,6	117834	SPHA FLP BCK LITE ARM DESK R	RIGHT
1A-5A,6	117835	SPHA FLP BCK LITE ARM DESK L	LEFT
1B-5B,6	117836	SPHA FLP BCK LITE ARM FULL R	RIGHT
1B-5B,6	117837	SPHA FLP BCK LITE ARM FULL L	LEFT

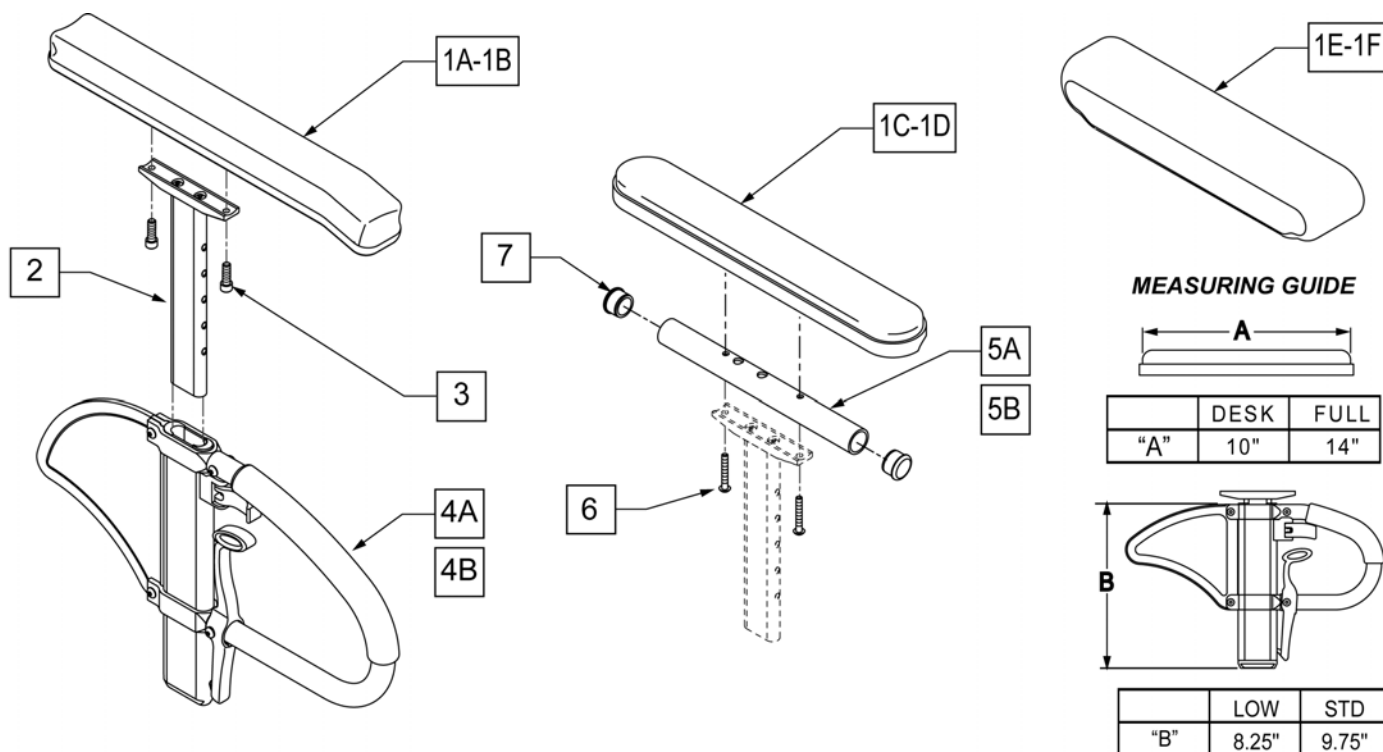

Pos.	Item Number	Description	Remarks
1A	104568-001	SPHA PAD DESK	CONTOUR DESK
1B	104568-002	SPHA PAD FULL	CONTOUR FULL
1C	561024	ARMREST PAD 10" DESK	CLASSIC DESK
1D	561031	ARMREST PAD 14" FULL	CLASSIC FULL
1E	561280	ARMREST PAD 10" LEFT W/F	Leftt-Waterfall Desk
1E	561281	ARMREST PAD 10" RIGHT W/F	Rightt-Waterfall Desk
1F	561284	ARMREST PAD 14" LEFT W/F	Leftt-Waterfall Full
1F	561285	ARMREST PAD 14" RIGHT W/F	Rightt-Waterfall Full
2	116312	SNGL POST UPPR ASSM LOW	LOW/SMALL
2	116313	SNGL POST UPPR ASSM STD	STANDARD
3	106956	SHCS 10-32 X 3/4 ROHS	
2,4A	104509-001	SPHA ARMREST ASSY STD	STANDARD
2,4B	104509-002	SPHA ARMREST ASSY LOW	LOW/SMALL
5A	521812	HT ADJ PD TUBE 10IN DESK POWE	Desk
5B	521813	HT ADJ PD TUBE 14IN FULL POWE	Full
6	107024	BHCS 10-32 X 1 1/2 ROHS	
7	560735	7/8" ROUND TUBE PLUG	
1C-4A-5A,6,7	108599	SPHA STD CLASIC DESK PAD R	Right
1C-4A-5A,6,7	108603	SPHA STD CLASIC DESK PAD L	Left
1D-4A-5B-7	108601	SPHA STD CLASIC FULL PAD R	Right
1D-4A-5B-7	108605	SPHA STD CLASIC FULL PAD L	Left
1A-4A	108595	SPHA STD CNTUR DESK PAD	
1B-4A	108597	SPHA STD CNTUR FULL PAD	
1C-4B-5A,6,7	108600	SPHA LOW CLASIC DESK PAD R	Right
1C-4B-5A,6,7	108604	SPHA LOW CLASIC DESK PAD L	Left
1D-4B-5B-7	108602	SPHA LOW CLASIC FULL PAD R	Right
1D-4B-5B-7	108606	SPHA LOW CLASIC FULL PAD L	Left
1A-4B	108596	SPHA LOW CNTUR DESK PAD	

Pos.	Item Number	Description	Remarks
1B-4B	108598	SPHA LOW CNTUR FULL PAD	


NOTE: Effective on GTi chairs after S/N T4-505800 & T4A-505900.

NOTE: Effective on GT chairs after S/N R3-012700 & R3A-012762.

NOTE: Armrest heights that can be achieved are; 7, 8, & 9 with the RECEIVER IN THE DOWN position, and 10, 11, & 12 with RECEIVER IN THE UP position.

NOTE: GT,GTi & Q2 Lite chair uses Low Armrest only.

Part Changes, please see notes/remarks for important information

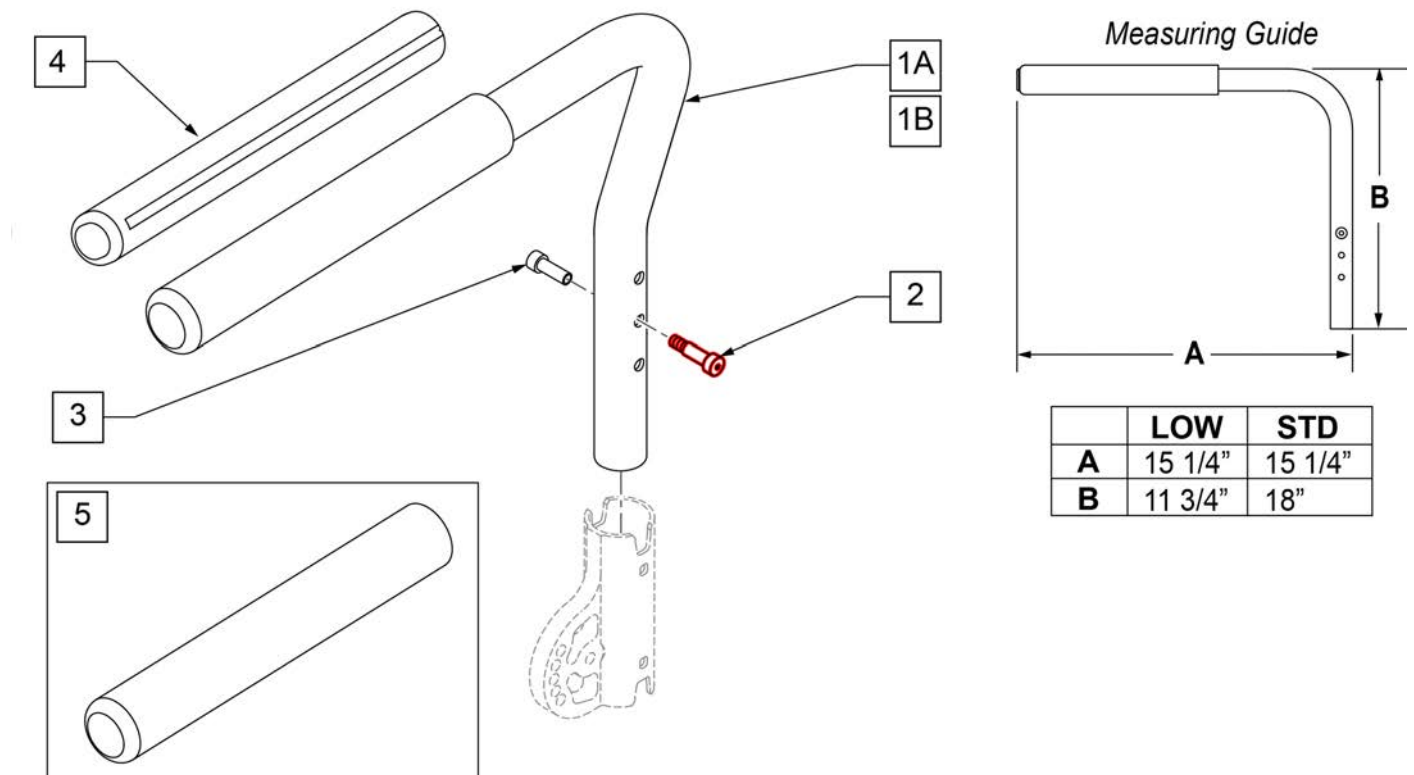

Pos.	Item Number	Description	Remarks
1A-3	111153	SWING-AWAY ARMREST LOW W/ BOLT R	RIGHT-LOW
1A-3	111154	SWING-AWAY ARMREST LOW W/ BOLT L	LEFT -LOW
1B-3	111155	SWING-AWAY ARMREST STD W/ BOLT R	RIGHT-STD
1B-3	111156	SWING-AWAY ARMREST STD W/ BOLT L	LEFT -STD
2,3	108118	SEXBOL TSA ARMREST KIT	
4	108315	GEL PAD S/ ARM	PAIR
5	560640	9X.93X.155 FOAM GRIP	Single replacement grip

NOTE: USED ON QUICKIE 2 CHAIRS W/ S/N PREFIX Q2V.


NOTE: USED ON QUICKIE 2 LITE CHAIRS.

NOTE: USED ON ZIPPIE TS CHAIRS W/ S/N PREFIX Z1A & Z1SE.


NOTE: USED ON ZIPPIE GS CHAIRS W/ S/N PREFIX ZGS & ZGSE.

NOTE: USED ON ZIPPIE 2 CHAIRS W/ S/N PREFIX Z2A.

NOTE: CLAMP MOUNTS TO 1" TUBING.

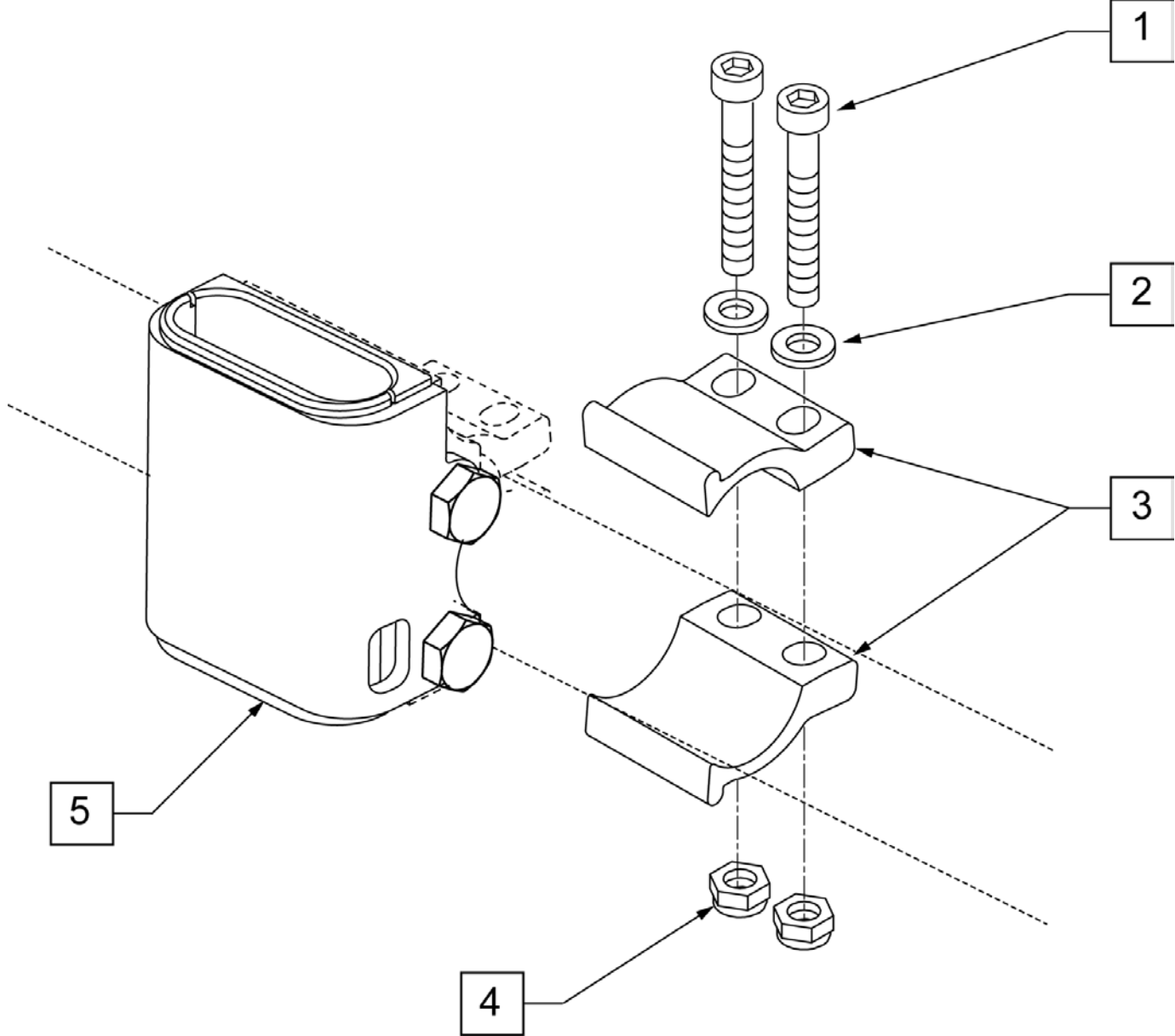

Pos.	Item Number	Description
1	106975	SHCS 1/4 -28 x 1 1/4 ROHS
2	106943	WASHER FLAT AN 1/4 ROHS
3	107123	CLAMP, RECEIVER, 1" OD
4	106935	NUT NYLOCK THIN 1/4 -28 ROHS
5	107124	RECEIVER, SPHA 1" OD
1-5	108101	SPHA RECEIVER W/ CLAMP KIT

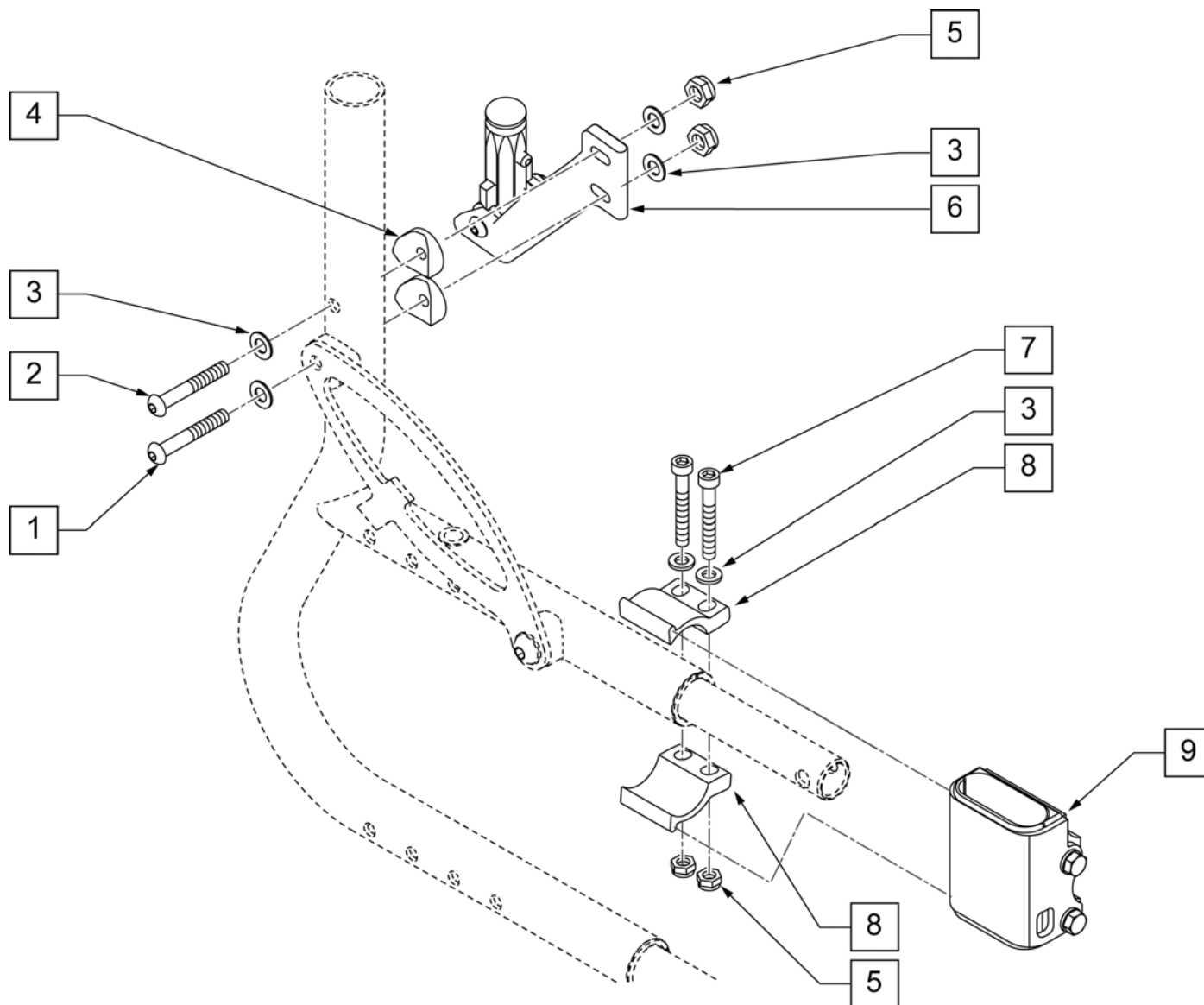


Pos.	Item Number	Description	Remarks
1	106964	BHCS 1/4-28 X 2 ROHS	
2	106962	BHCS 1/4 -28 x 1 3/4 ROHS	
3	106943	WASHER FLAT AN 1/4 ROHS	
4	561004	1" SADDLE 1/4" HOLE	
5	106935	NUT NYLOCK THIN 1/4 -28 ROHS	
6	109413	SPHA FB RECEIVER ASSM RH	RIGHT
6	109414	SPHA FB RECEIVER ASSM LH	LEFT
7	106975	SHCS 1/4 -28 x 1 1/4 ROHS	
8	107123	CLAMP, RECEIVER, 1" OD	
9	107124	SPHA RECEIVER KIT	
3,5,7-9	108101	SPHA RECIEVER W/ CLAMP KIT	
1-6	120657	REAR RECIVER ASSM - RIGHT	RIGHT
1-6	120658	REAR RECIVER ASSM - LEFT	LEFT

(04/2010)


NOTE: USED ON QUICKIE 2 LITE CHAIRS.

NOTE: Item 6 can be flipped for additional adjustment, when flipped Left becomes Right and Right becomes Left.


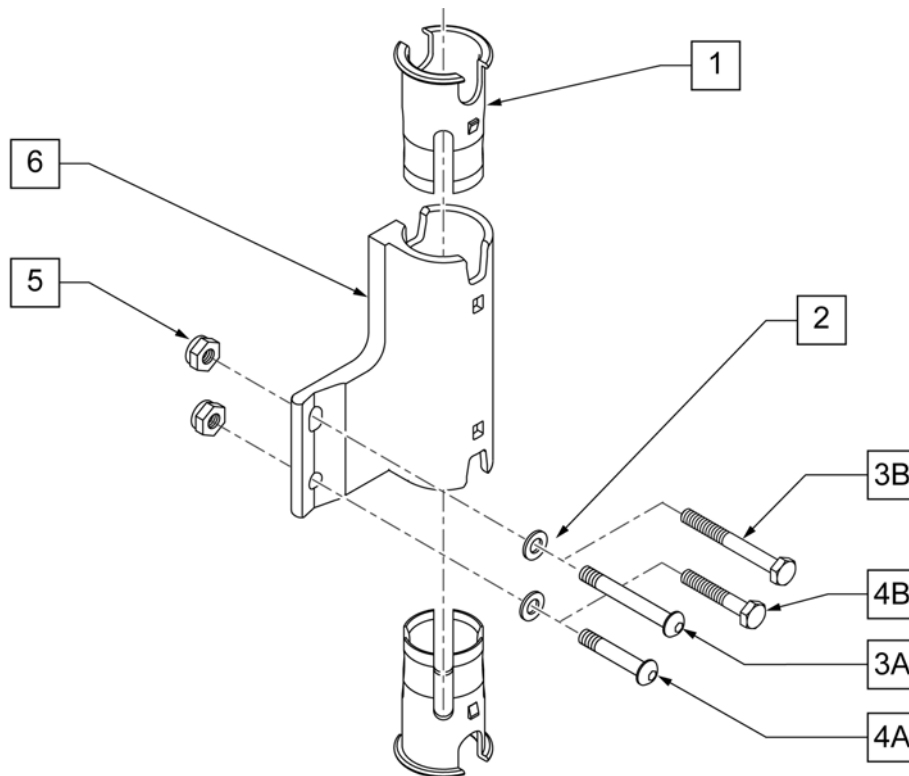


Pos.	Item Number	Description	Remarks
1	106505	SLEEVE SWING AWAY ARMREST	
2	106943	WASHER FLAT AN 1/4 ROHS	
3A	106964	BHCS 1/4-28 X 2 ROHS	Q2 LITE ONLY
3B	107351	HHCS 1/4-28 X 2 1/4 ROHS	Q2 ONLY
4A	106962	BHCS 1/4-28 X 1-3/4 ROHS	Q2 LITE ONLY
4B	109265	HHCS 1/4-28 X 2-1/2 ROHS	Q2 ONLY
5	106935	NUT NYLOCK THIN 1/4-28 ROHS	
1,6	106720	RECEIVER, SA ASSY, RH	RIGHT - SHOWN
1,6	106721	RECEIVER, SA ASSY, LH	LEFT


NOTE: KIDS BACKPACKS DO NOT HAVE ADJUSTABLE STRAPS W/ BUCKLES.

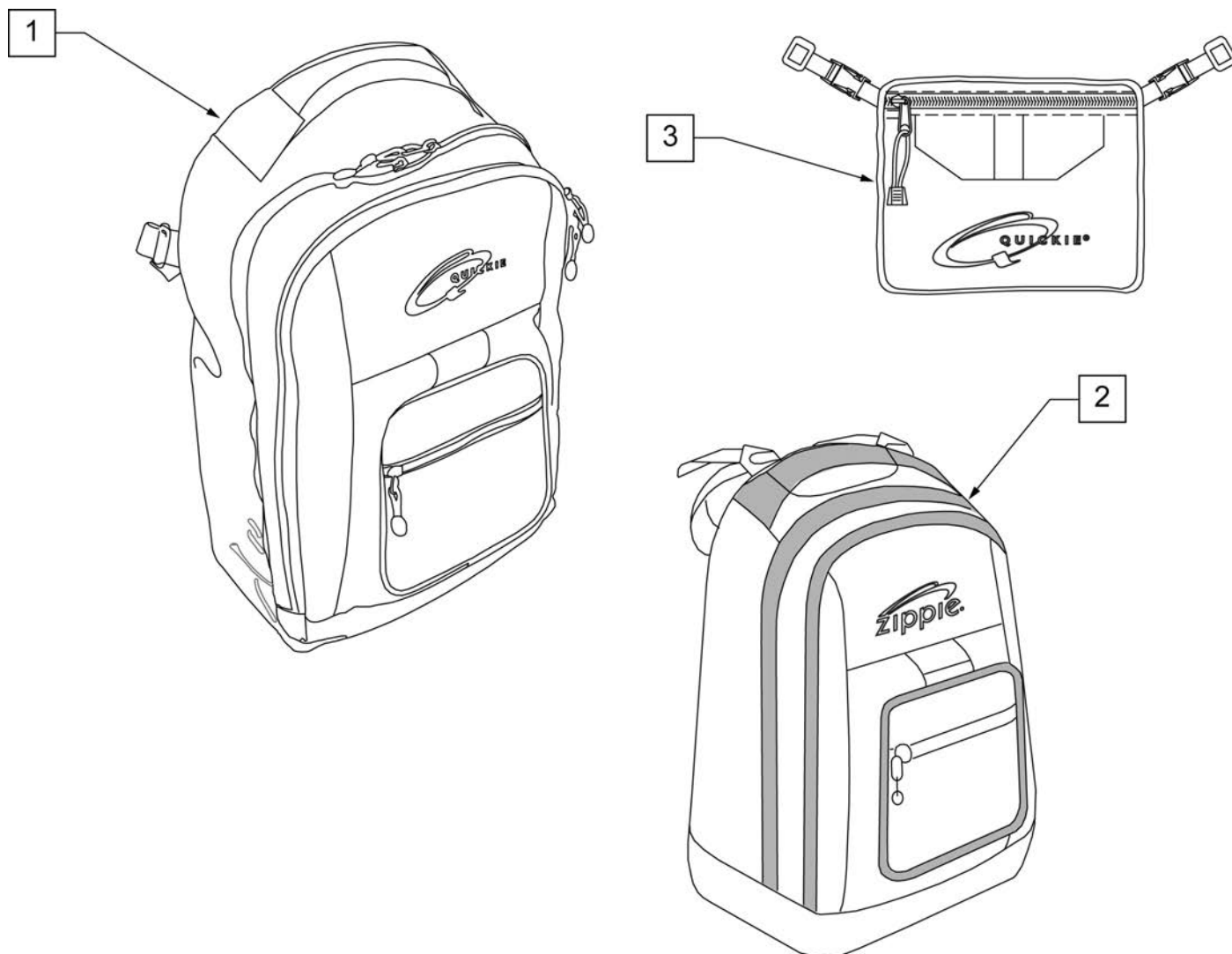


Pos.	Item Number	Description
1	750115	BACK PACK QUICKIE
2	104572	BACKPACK ZIPPIE
3	750307	SEAT POUCH QUICKIE

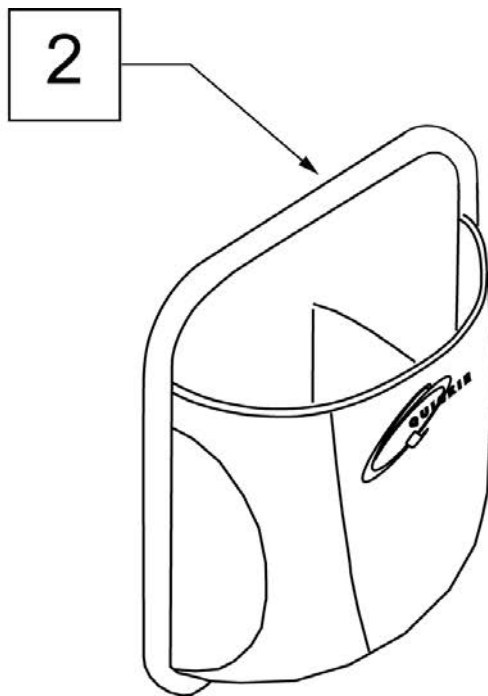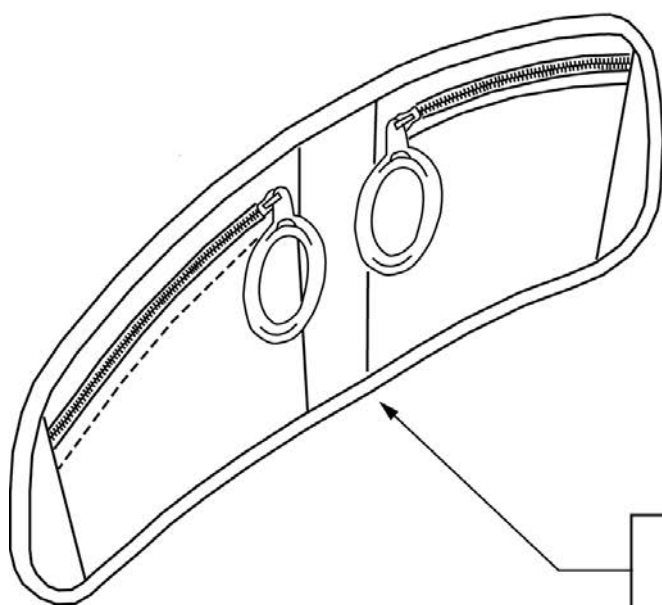


Pos.	Item Number	Description	Remarks
1	013477	LAP ORGANIZER	Size: 18.25"x 7.75"x .75"
2	013475	QUICKIE SIDE POCKET	Size: 6.5"x 6.5"x 2"

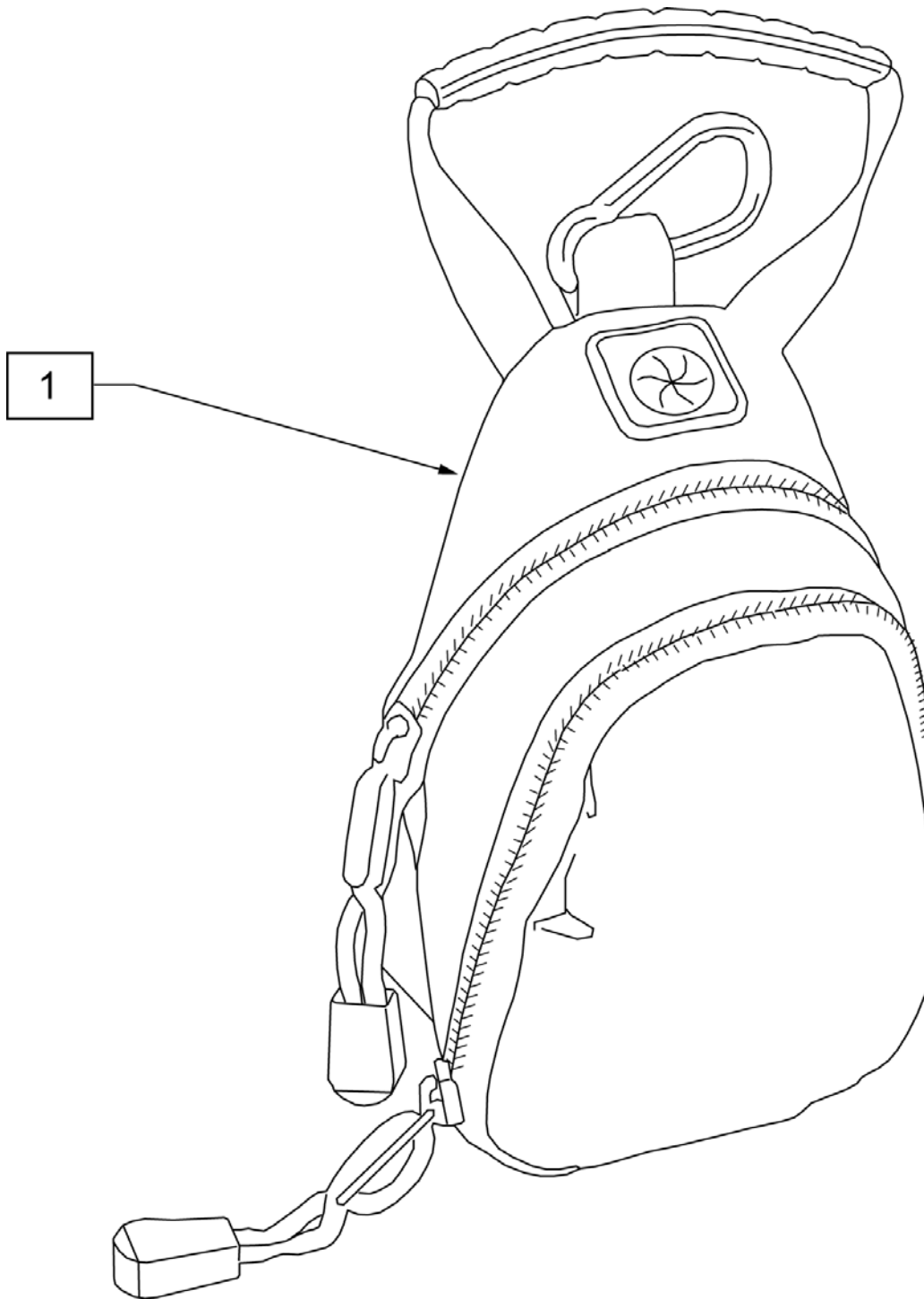


Pos.	Item Number	Description
1	109229	CELL PHONE PACK (ORANGE)

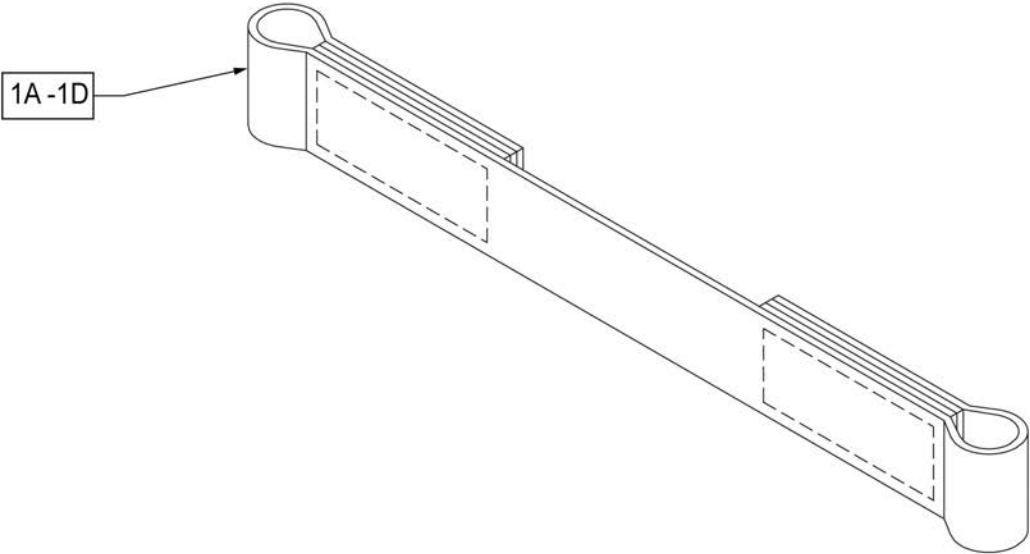

Pos.	Item Number	Description
1A	109356	LEG STRAP SOFT FEEL Q 11 -12"
1B	109357	LEG STRAP SOFT FEEL Q 13 -15"
1C	109366	LEG STRAP SOFT FEEL Q 16 -18"
1D	111487	LEG STRAP SOFT FEEL Q 19 -20"

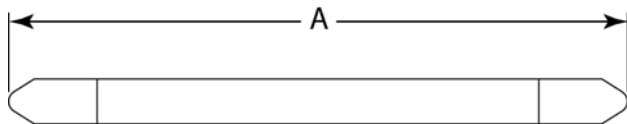MEASURING GUIDE


DESCRIPTION	DIM "A"
LEG STRAP 11"-12" BLK	27 1/2
LEG STRAP 13"-15" BLK	32 1/2
LEG STRAP 16"-18" BLK	39 1/2
LEG STRAP 19"-20" BLK	45 1/2

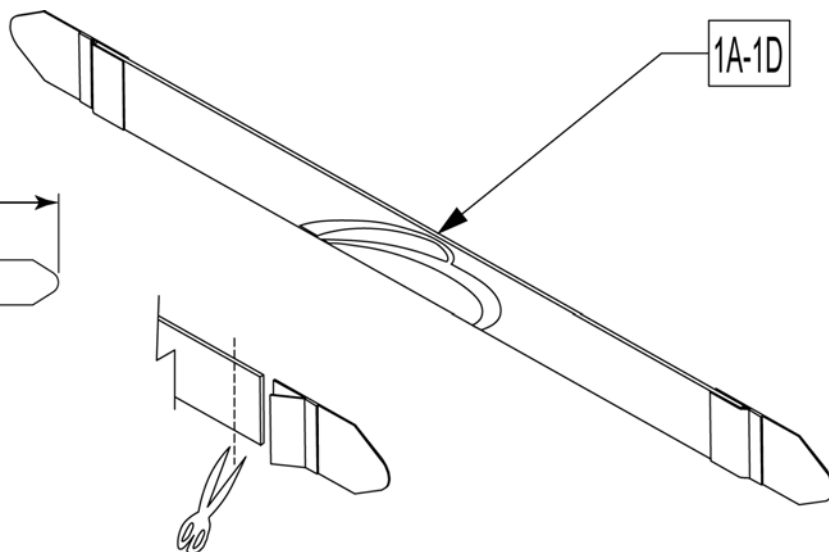



Leg Straps may be custom cut to a smaller size

Pos.	Item Number	Description
1A	113255	LEG STRAP 11"-12" BLK
1B	113256	LEG STRAP 13"-15" BLK
1C	113257	LEG STRAP 16"-18" BLK
1D	113258	LEG STRAP 19"-20" BLK

1A

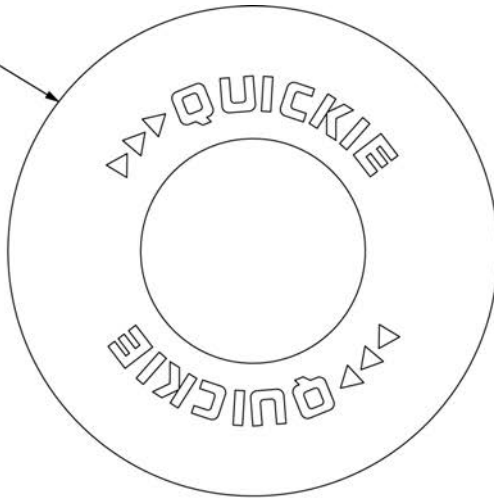


1B

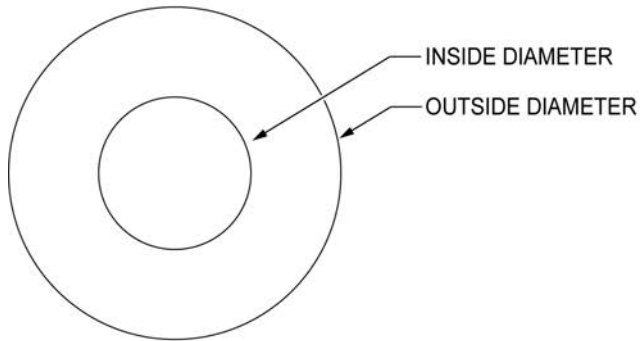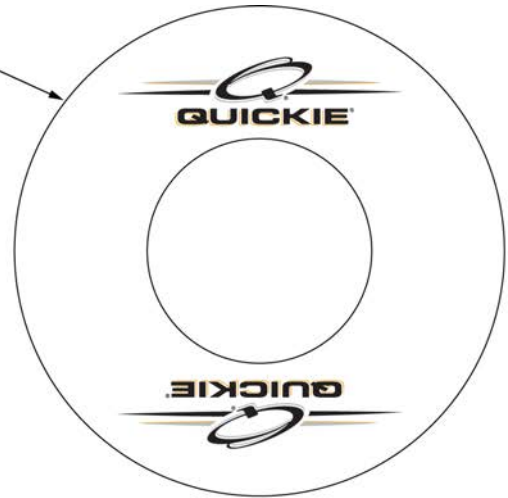
**SPOKE GUARD MEASURING GUIDE**

PART NUMBER	INSIDE DIAMETER	OUTSIDE DIAMETER
902001	8 1/4	15 7/8
902011	8 1/4	18 1/8
902021	10 1/4	19 5/8
902025	10 1/4	20 3/8
902037	10 1/4	21 7/8
902038	10 1/4	23 3/8
103660-001	8 1/4	16
103660-002	8 1/4	18
103663-001	10 1/4	20 1/2
103663-002	10 1/4	23 1/2
103734-001	10 1/4	19 1/4
103734-002	10 1/4	22
119336	10 1/4	19 1/4


Pos.	Item Number	Description	Remarks
>>>>		BLACK SPOKE GUARDS	
1A	902001	SPK GRDS 20" QUICKIE BLK	Pair-BLACK-FITS 20" WHEELS
1A	902011	SPK GRDS 22" QUICKIE BLK	Pair-BLACK-FITS 22" WHEELS or 24" (540) WHEELS W/ PROJECTION HANDRIMS
1A	902021	SPK GRD 24" QUICKIE SCR MT BLK	Pair-BLACK-FITS 24" (540) MAG & SPOKE WHEELS
1A	902025	SPK GRD 24" QUICKIE TAB MT BLK	Pair-BLACK-FITS 25" (559) SPOKE WHEELS
1A	902037	SPK GRD 26" QUICKIE TAB MT BLK	Pair-BLACK-FITS 26" (590) WHEELS
1A	902038	SPK GRD 26" QUICKIE HT BLK	Pair-BLACK-FITS 27" (700) WHEELS
>>>>		CLEAR SPOKE GUARDS	
1B	103660-001	CLEAR SPOKE GUARD 20"	Pair-CLEAR-FITS 20" WHEELS
1B	103660-002	CLEAR SPOKE GUARD 22"	Pair-CLEAR-FITS 22" WHEELS or 24" (540) WHEELS W/ PROJECTION HANDRIMS
1B	103734-001	SPOKE GUARD 24" CLR	Pair-CLEAR-FITS 24" (540) SPOKE WHEELS
1B	119336	SPOKE GUARD 24" CLR	Pair-CLEAR-FITS 24" (540) MAG WHEELS
1B	103663-001	SPOKE GUARD 24" QP CLR	Pair-CLEAR-FITS 25" (559) WHEELS
1B	103734-002	SPOKE GUARD 26" CLR	Pair-CLEAR-FITS 26" (590) WHEELS
1B	103663-002	SPOKE GUARD 26" QP CLR	Pair-CLEAR-FITS 27"(700) WHEELS

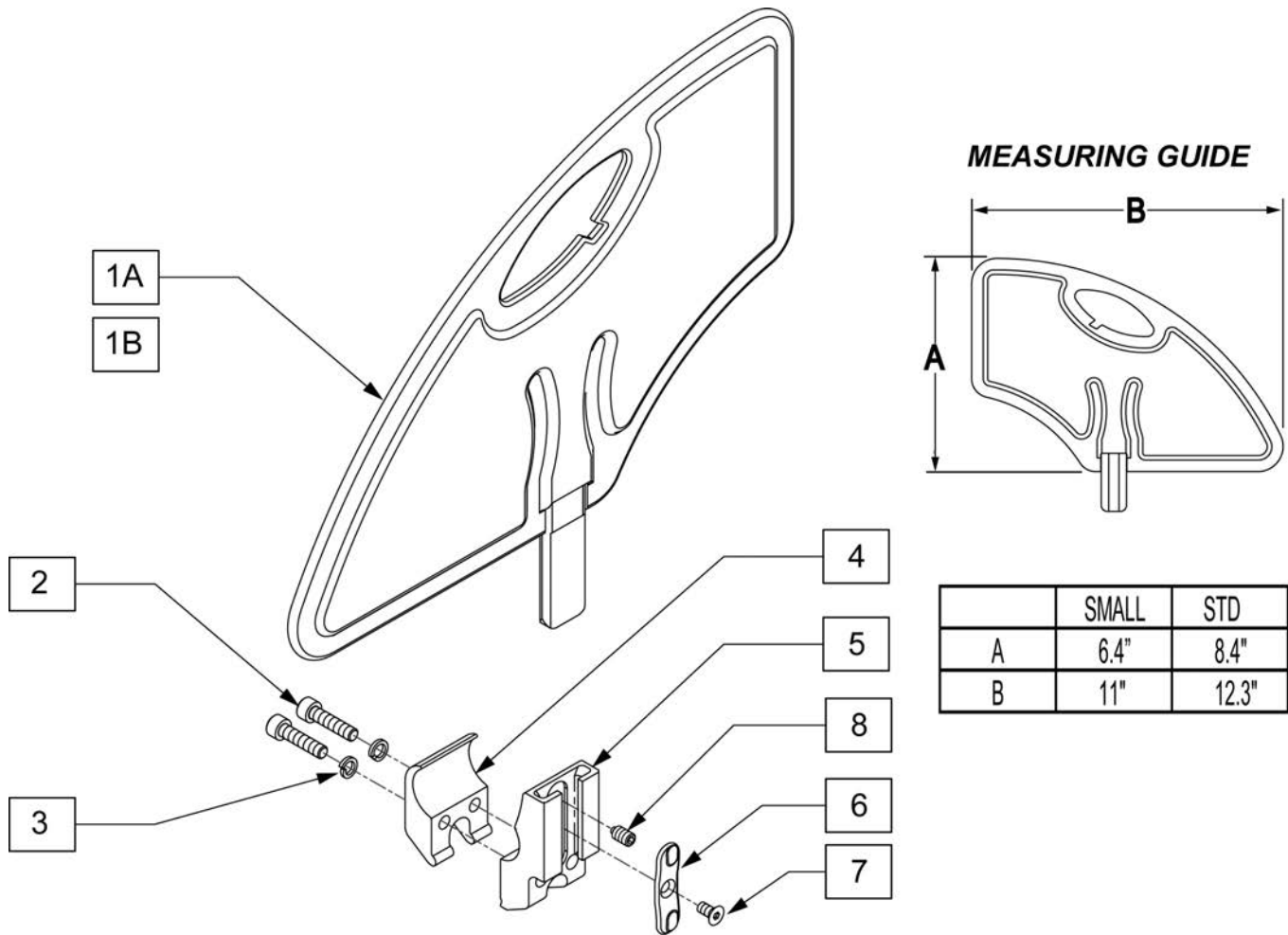


Pos.	Item Number	Description
1A	104164-001	SIDE GUARD ASSY STD
1B	104164-002	SIDE GUARD ASSY SMALL
2	106961	BHCS 1/4-28 X 1-1/4 ROHS
3	107367	WASHER LOCK HI COLLAR 1/4 ROHS
4	106714	REC SIDE GUARD CLAMP THREADED
6	108780	RECEIVER SG PLASTIC INSERT
7	107135	FHCS #6-32 X 1/4 BLK
8	107136	SETSCREW 1/4-28 X 5/16 OVAL PT
5-8	108781	RECIEVER SIDE GUARD ASSY
2-8	107592	RECEIVER SIDE GUARD ASSY
1A-8	107590	SIDE GUARD ASSY W/ CLAMP STD
1B-8	107591	SIDE GUARD ASSY W/ CLAMP SMALL

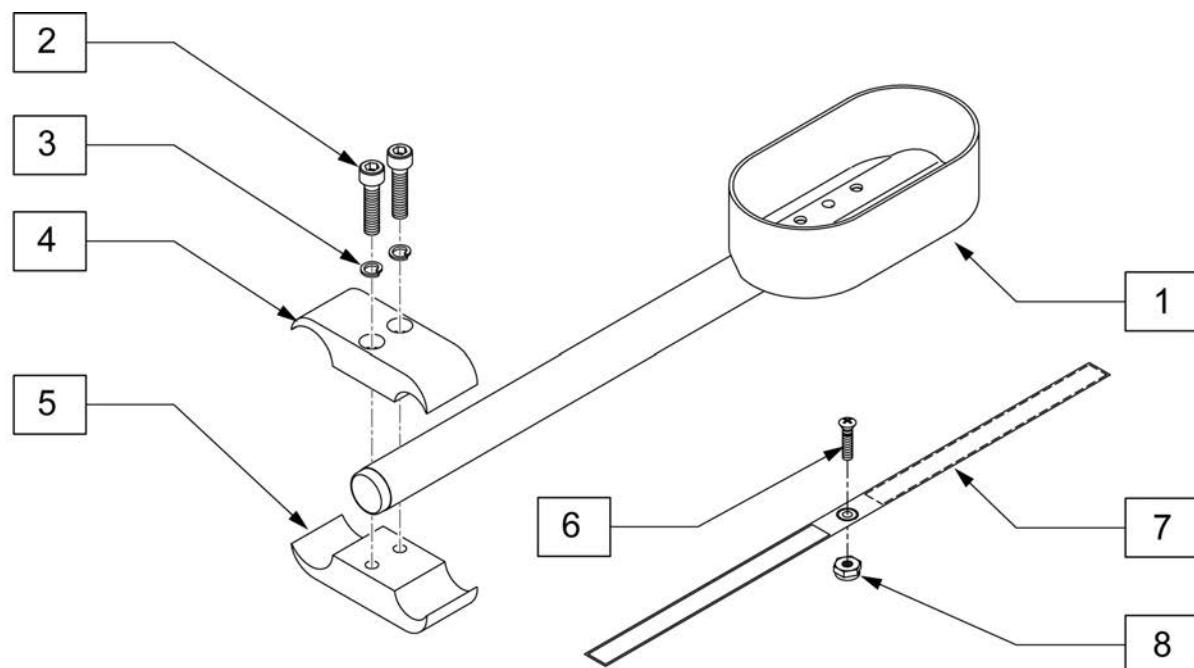


Pos.	Item Number	Description	Remarks
1	119353	12" CRUTCH HOLDER ASSY RH	
1	119354	12" CRUTCH HOLDER ASSY LH	
2	107032	SHCS 1/4-28 X 1	
3	455037-QU	1/4 X 1 FLAT WASHER	
2-5	884101	VENT TRAY CLAMP ASSM S626	
6	281110	10-32 x 1/2 PAN PHIL	
7	750495	CRUTCH HOLDER STRAP F	
8	200100-QU	10-32 NYLOCK NUT	
1-8	119242	CSTM Q2 LITE CRUTCH HOLDER R	Right
1-8	119270	CSTM Q2 LITE CRUTCH HOLDER L	Left

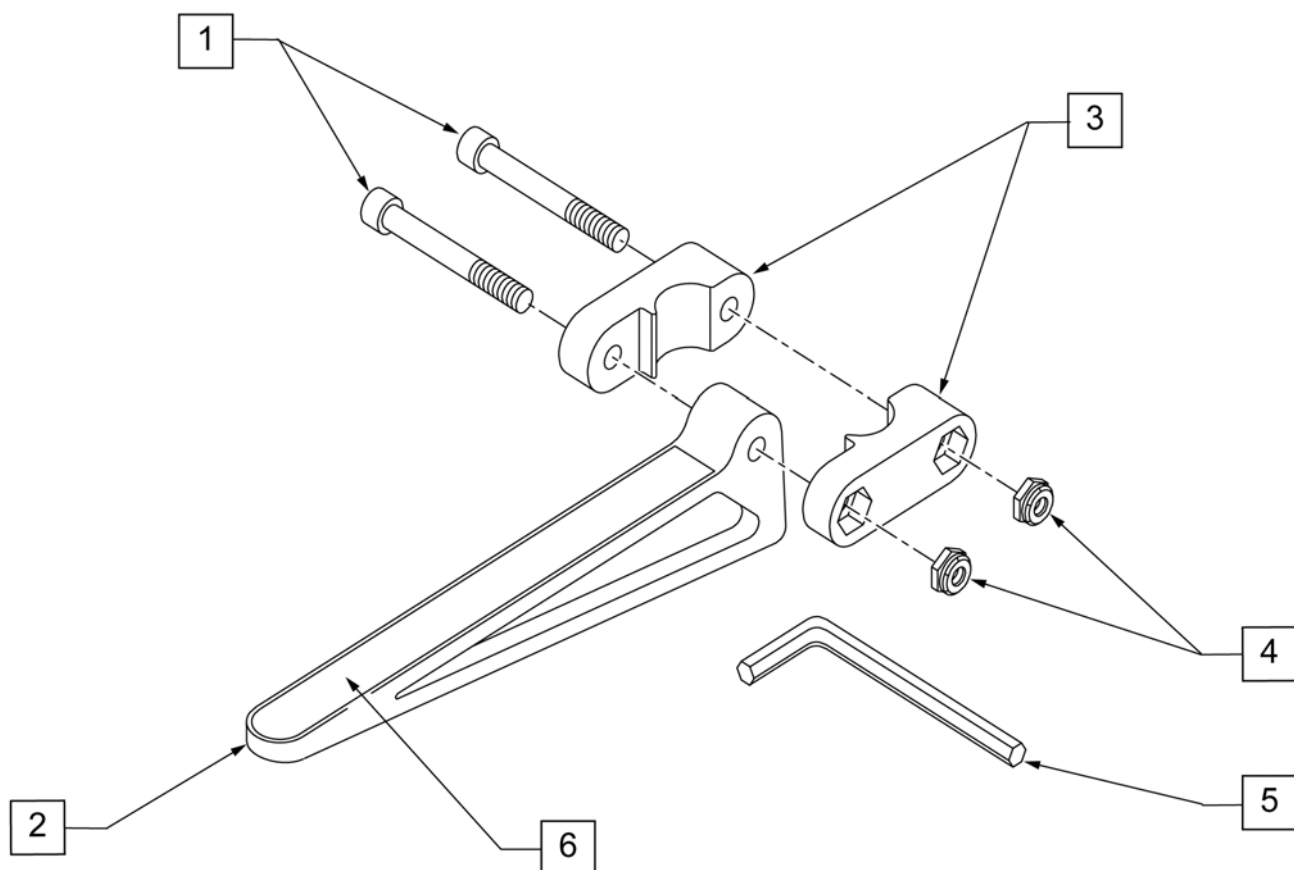


Pos.	Item Number	Description	Remarks
1	226128	1/4-28 x 1 1/4 SOC C/S	
2	561105	CADDY ARM	
3	561104	CADDY CLAMP	
4	106935	NUT NYLOCK THIN 1/4-28 ROHS	
5	349998	3/16" ALLEN WRENCH	
6	580050	CADDY TAPE	
1-6	902350	QUICKIE CADDY (PR)	PAIR

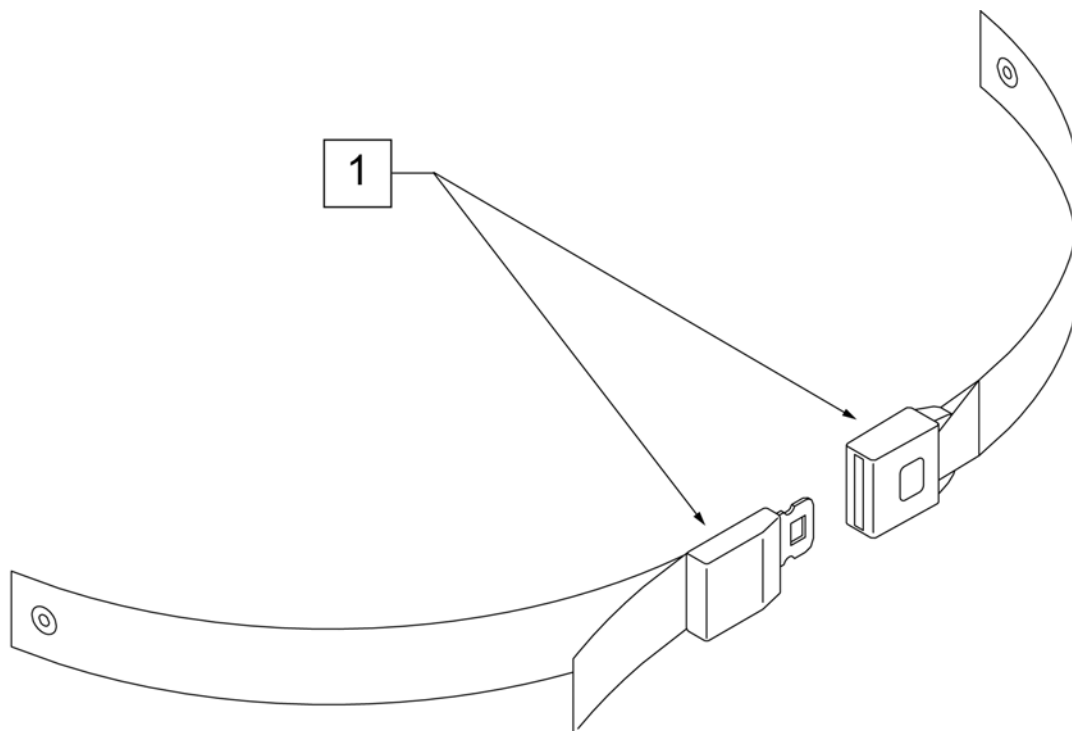


Pos.	Item Number	Description	Remarks
1	750276	1 1/2" POSITION BELT	AUTOSTYLE-64" Max length of belt.

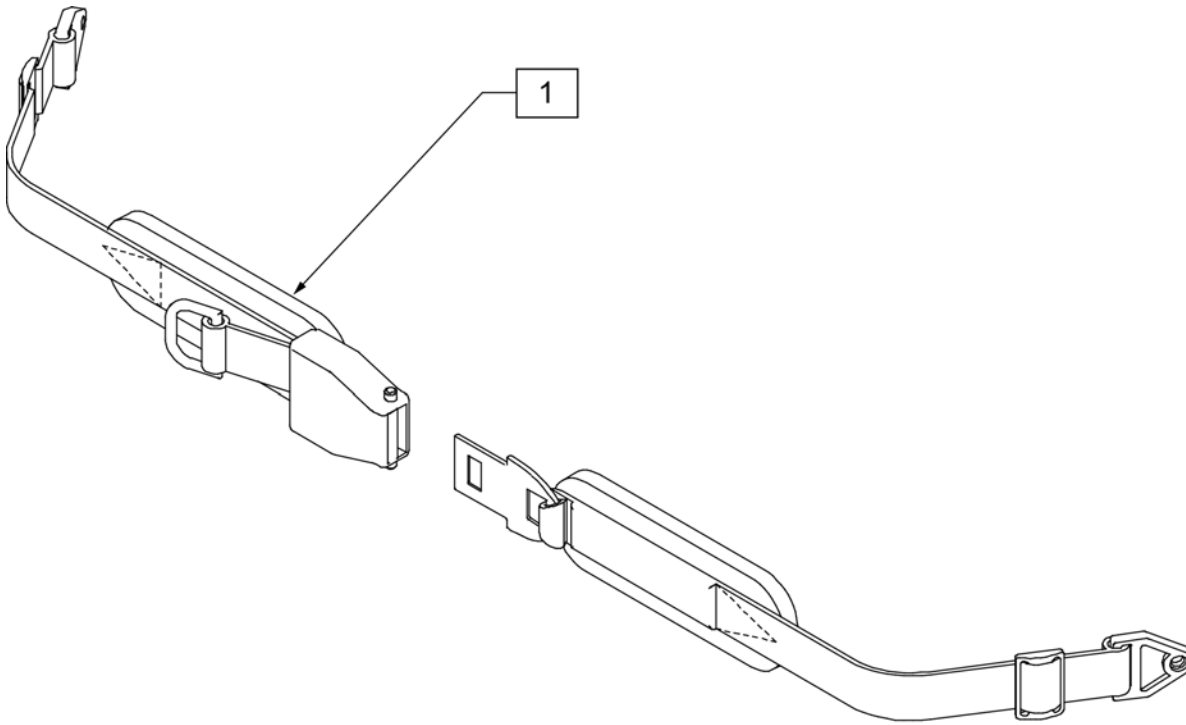


Pos.	Item Number	Description
1	450418-QU	BELT 2" AIRCRAFT BUCKLE
1	450423-QU	BELT 2" AIRCRAFT BUCKLE W/PAD
1	450424	BELT 2" AIRCRAFT BKL LONG W/PAD
1	450425	BELT 2" AIRCRAFT BKL LONG
1	450419-QU	BELT 2" AIRCRAFT BKL W/QUAD LP
1	468419	BELT 2" AIRCRT BKL W/QD LPCZ

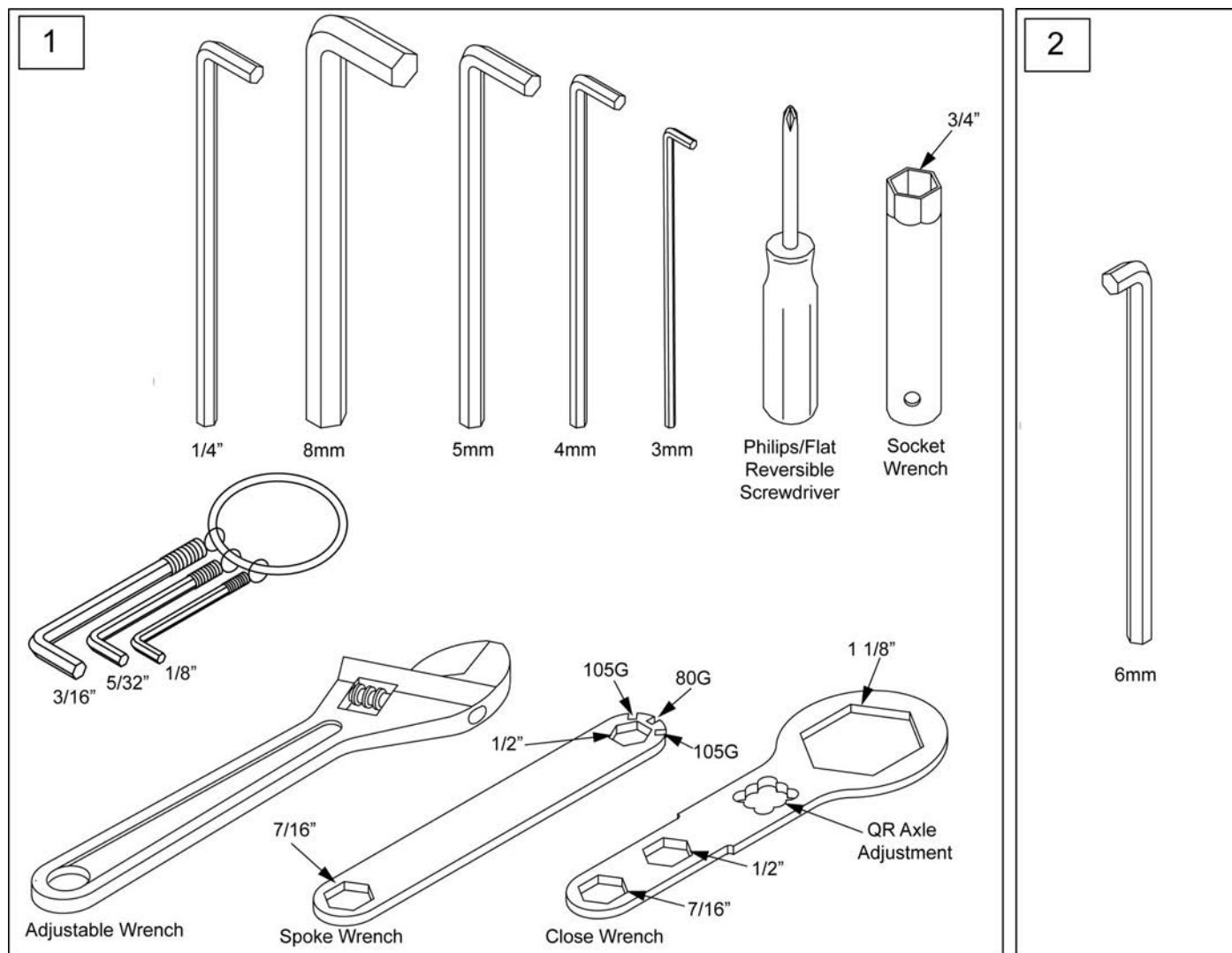

Pos.	Item Number	Description	Remarks
1	108314	QUICKIE COMBO TOOL KIT	Includes pouch
2	113403	6MM ALLEN WRENCH L, 7MM SHORT	caster spline chairs only

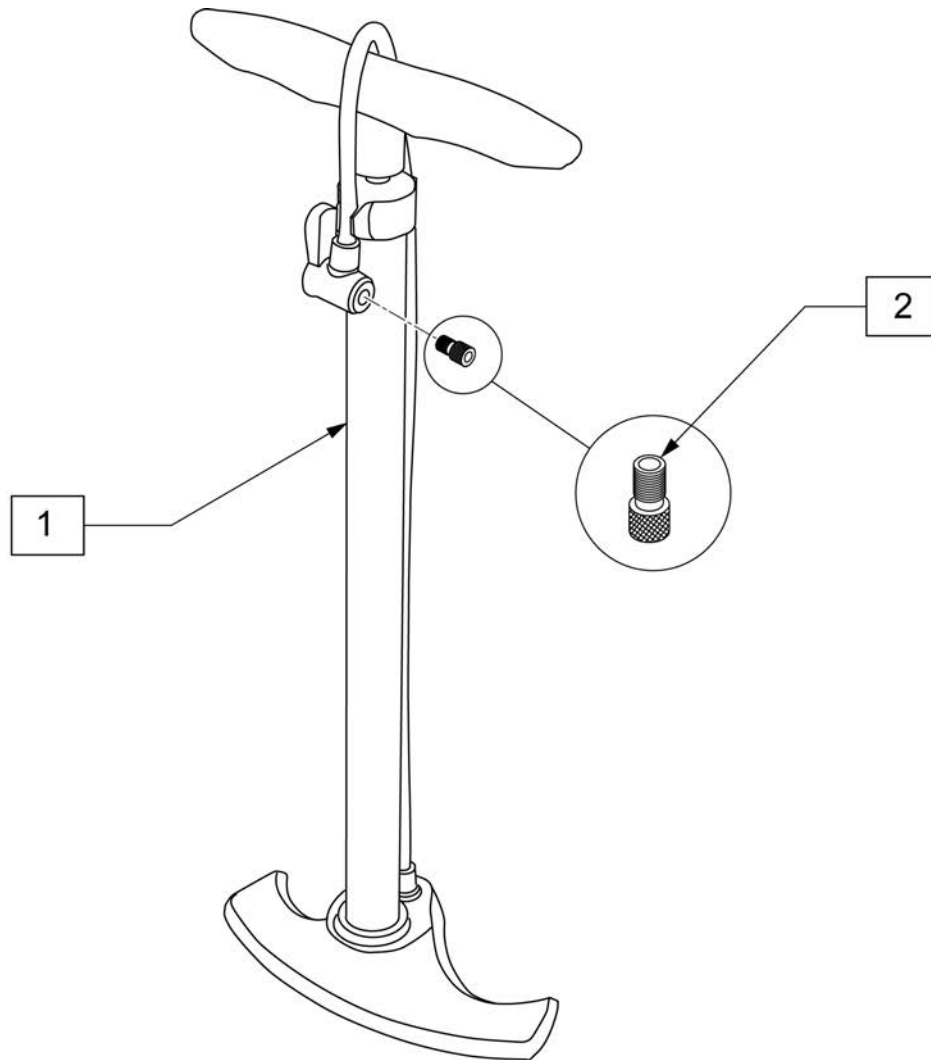

Pos.	Item Number	Description
1	389050	TIRE PUMP
2	381230	VALVE ADAPTER PRESTA

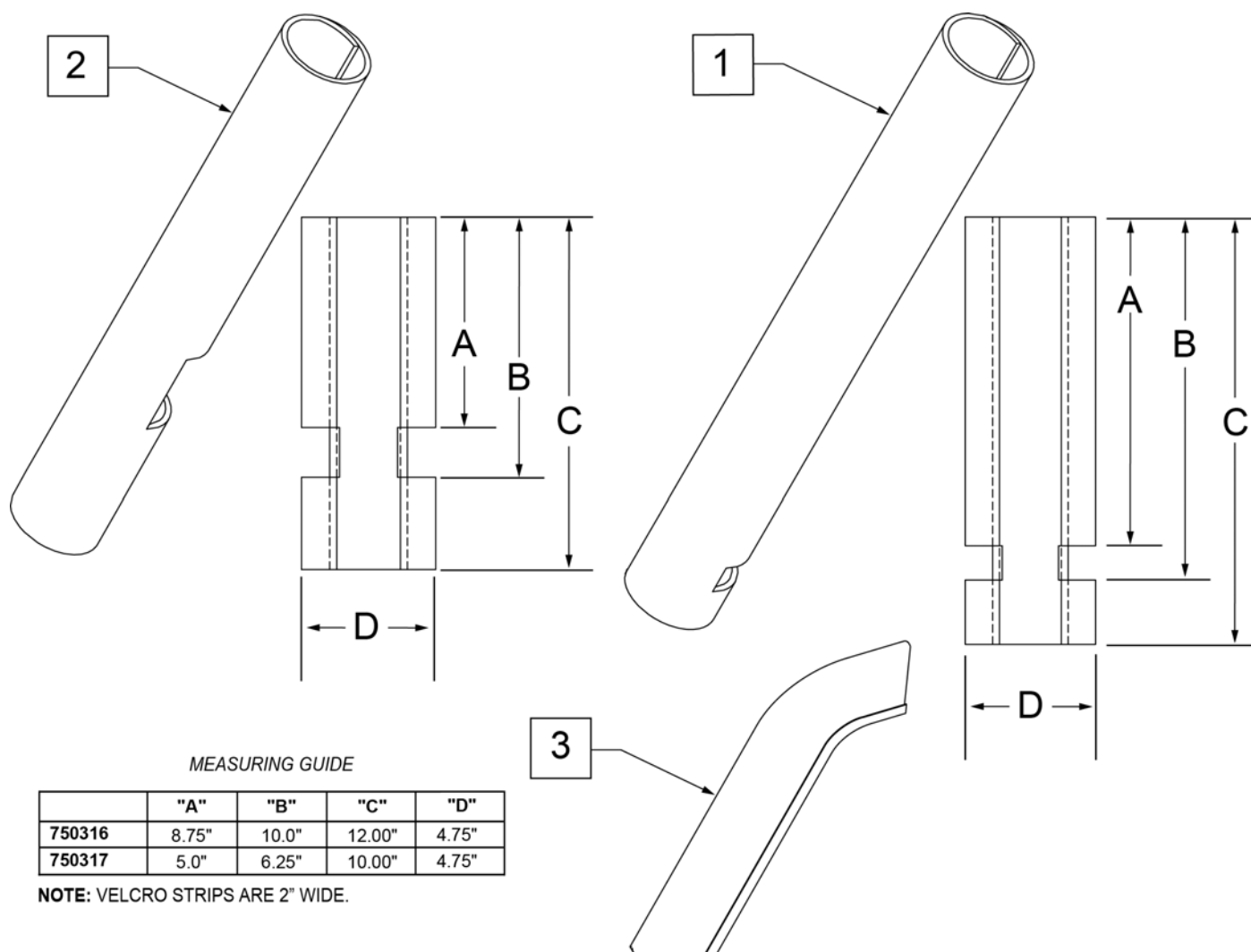

Pos.	Item Number	Description	Remarks
1	750316	IMPACT GUARDS NEOPRENE FIXED (PR)	PAIR-Fixed Front Frame
2	750317	IMPACT GUARDS NEOPRENE S/A	PAIR-S/A Footrest & Q7
3	109231	GEL VELCRO PAD HANGER/FRAME	PAIR- fits S/A & Fixed Front Frame



PARTS MANUAL

MAY 2013